



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000.

Kit parts:

- | | |
|-----------------------|----------------------|
| 1. 2 ea Carriage Bolt | 5. 2 ea Spacer |
| 2. 2 ea Large Washer | 6. 2 ea Washer |
| 3. 2 ea Thin Nut | 7. 2 ea Knurled Knob |
| 4. 2 ea Tray Clamp | 8. 2 ea Acorn Nut |

1. PREPPING FOR ASSEMBLY:

- Ensure tray is facing up.
- Center the tray across the chair (from armrest to armrest).
- Determine which tray slot to use and approximate bolt position.
- Install hardware as shown in Figure 1.

2. INSTALLING CLAMPING HARDWARE:

- Insert carriage bolt (1) through tray slot as determined from step 1c.
- Install washer (2) to carriage bolt (1) on the underside of the tray.
- Install nut (3) after washer (2). Tighten nut (3) ensuring that the bolt is in the desired position in the slot.
- Slide spacer (5) into hole in clamp (4) then slide both onto carriage bolt (1), follow with washer (6).
- Screw knurled knob (7) onto the carriage bolt (1).
- Screw acorn nut (8) onto carriage bolt (1), tighten securely.
- Repeat all steps on opposite side of the tray.

3. INSTALLING TRAY ONTO THE CHAIR:

- Install the tray onto the chair by sliding the armrest between the tray bottom and the tray clamp (4). Figure 2.
- Tighten both knobs (7) enough to prevent the tray from slipping.

! WARNING

DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Sunrise Technical Service Department before operating and/or servicing the device. Failure to do so may result in damage and/or injury.

For more information and important warnings, see your chair's owner's manual at www.Sunrisemedical.com/Help-support.

