

INSTRUCTION SHEET

Sling 2" Extension Hardware Installation

(Please read and understand instructions before beginning installation)

Tools Required:

4mm hex key

Parts Required:

BZU2099 Sling 2" Extension
Hardware
New Seat Sling 2" deeper than
current sling

Removal Procedure:

- 1) Using a 4mm hex wrench (A) remove all the screws (B) attaching the existing seat sling.(Fig. 1)

Note: Chair shown unfolded for clarity. It may be easier to remove and install screws with Chair in the partially folded position to reduce tension on the screws.

- 2) Remove both plugs (C) at the front of the seat cross tube assemblies (Fig. 2).

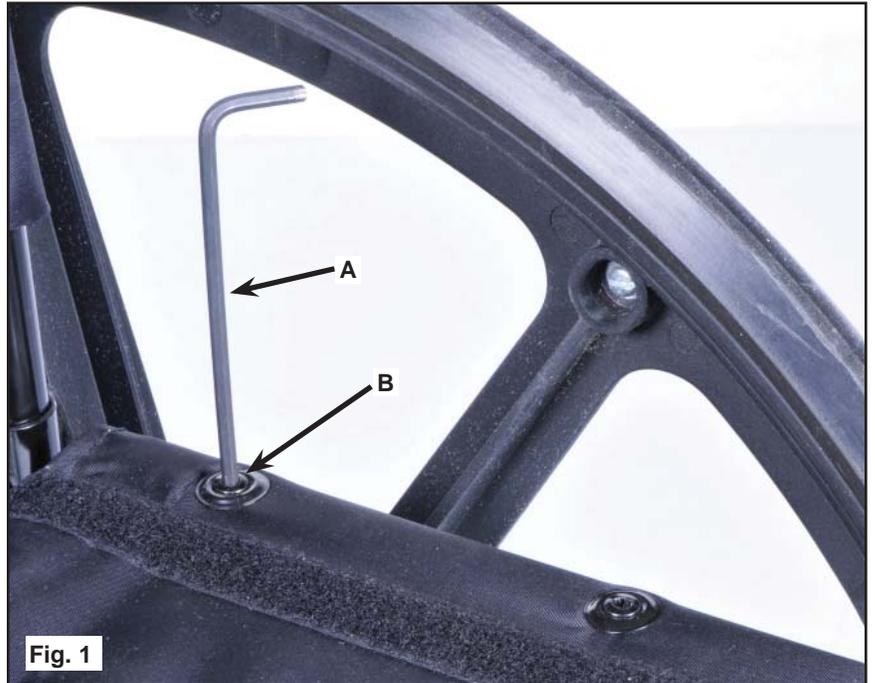


Fig. 1

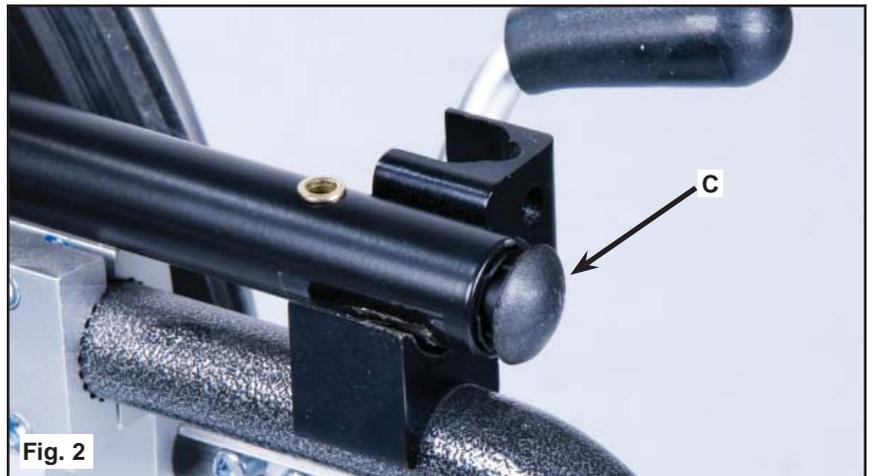


Fig. 2

Installation Procedure:

- 1) Insert both 2" extension Plugs (D) into the open cross tube assemblies at the front of the chair (Fig. 3).
- 2) Install the new 2" longer seat sling using a 4 mm hex wrench and the screws provided.



Fig. 3