

What's included...



- **Size B Fluid Bladder Standard Fill Volume - Qty 1**
Part Number = FBJFUSION
- **Size B Fluid Bladder 20% Under filled - Qty 1**
Part Number = MFBFUSION
(Must specify 20% under fill in notes when ordering replacement)



- **Extra Large (XL) Fluid Supplement Pad – Qty 2**
Part Number = J3CAFXL (2ea)
2 XL Pads are equivalent to 20% over fill



- **Large (L) Fluid Supplement Pad – Qty 2**
Part Number = J3CAFLG (2ea)
2 L Pads are equivalent to 15% over fill



- **Medium (M) Fluid Supplement Pad – Qty 2**
Part Number = J3CAFMD (2ea)
2 M Pads are equivalent to 10% over fill




- **Small (S) Fluid Supplement Pad – Qty 2**
Part Number = J3CAFSD (2ea)
2 S Pads are equivalent to 5% over fill

How it works...

How to achieve an Overfilled Bladder

Note: Start with fluid bladder that does not have a modified fluid volume

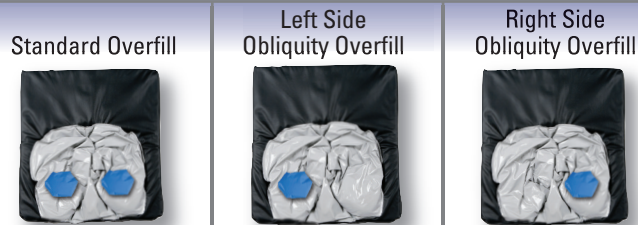

	Standard Overfill	Left Side Obliquity Overfill	Right Side Obliquity Overfill
5% Overfill	2 Small Pads	1 Small Pad	1 Small Pad
10% Overfill	2 Medium Pads	1 Medium Pad	1 Medium Pad
15% Overfill	2 Large Pads	1 Large Pad	1 Large Pad
20% Overfill	2 Extra Large Pads	1 Extra Large Pad	1 Extra Large Pad

How to achieve an Underfilled Bladder

Note: Start with fluid bladder labeled 20% underfilled



	Standard Underfill	Left Side Obliquity Underfill	Right Side Obliquity Underfill
5% Underfill	2 Large Pads	1 Large Pad	1 Large Pad
10% Underfill	2 Medium Pads	1 Medium Pad	1 Medium Pad
15% Underfill	2 Small Pads	1 Small Pad	1 Small Pad
20% Underfill	No Pads	No Pads	No Pads