



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000



Your kit contains 2 complete Anti-tip tube assemblies.

Tools needed: 5mm Hex key, 10mm wrench

## A. REMOVE RUBBER CAPS 1 2

Wet the area under the rubber cap.



- a. Rotate and remove both caps(A) from the tubes.
- b. Dry off the area before inserting the anti-tips.

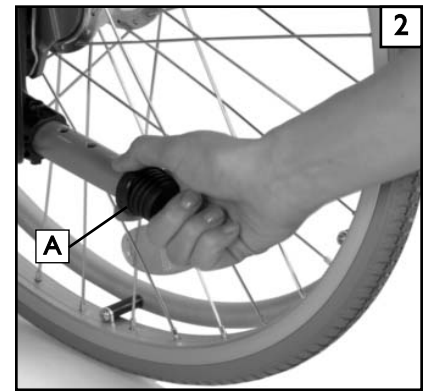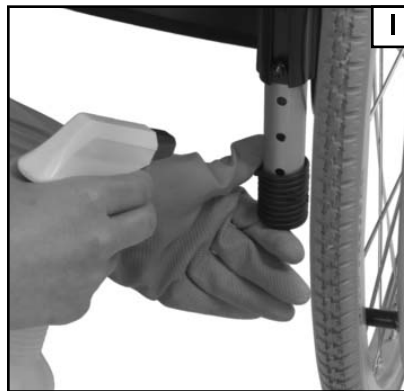

## B. INSTALL THE ANTI-TIPS 3

Note: Before installation, adjust to shortest length.

- a. Push down on the Snap button(B) and insert the anti-tip into the lower rear frame tube, until the button pops out.
- b. Install the other anti-tip.

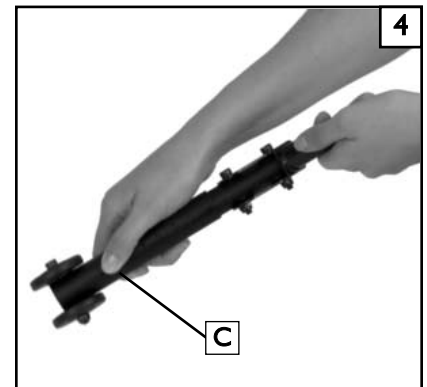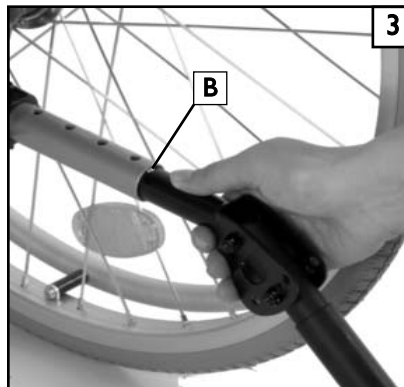

## C. ADJUST THE ANTI-TIPS 4 5 6


Note: To prevent unintended tipping, the anti-tip wheel height should be adjusted 1.5 and 2" from the riding surface, both anti-tips must be the same height.

### 1. Adjust the anti-tip wheel length

- a. Push in the Snap button(C) and move the anti-tip wheels up or down. (see note)
- b. If the wheels are still too high, adjust the angle.

### 2. Adjust the anti-tip angle

- a. Using a 10mm wrench and 5mm hex key Loosen screw (D) and remove bolt (E)
- b. Adjust the angle (see note). Re-insert the hardware and tighten for both sides

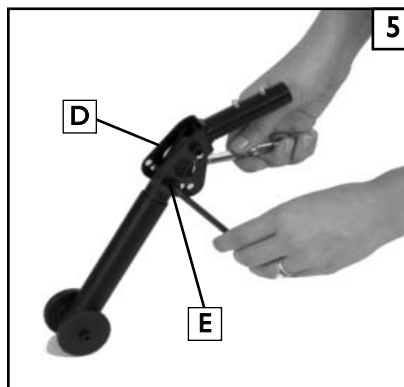

## WARNING

Do not operate or service the device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie wheelchair. Failure to do so may cause damage or injury