



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000

Your kit contains 2 complete Elevating Legrest assemblies.




Tools needed: 5mm hex key, 6mm hex key, and a 13mm wrench

## A. FOOTPLATES

1 2 3

Use the existing footplates and extension tube.

- A 4mm hex key will loosen the set screw(A) do not remove it.
- Remove the heel loop (B)
- Remove the footplate/tube (C) from the hanger
- Loosen ELR set screw (D) with a 4mm hex key. **Do Not Remove it.**
- Insert footplate and extension tube into ELR.

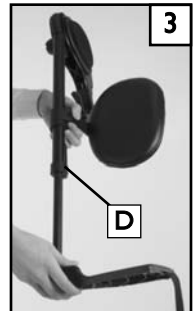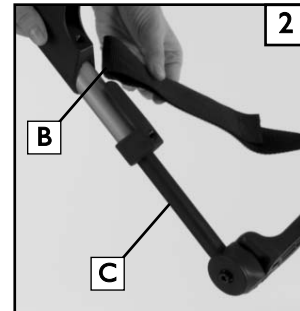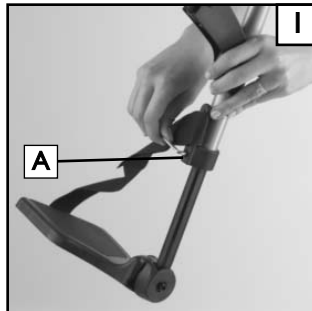


## B. ADJUSTMENT

4 5 6

Insert the Footplate and extension tube into ELR.

- Adjust the footplate to the correct height (2" above riding surface), and Tighten the screw
- Use the 4mm hex key (F) to adjust the calf pad up or down
- Use a 13mm wrench and 6mm hex key to adjust the calf pad angle, or depth. After adjustment is complete tighten the hardware.

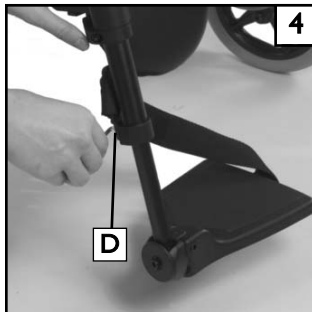

## C. ELEVATE YOUR FOOTREST

7 8

Using the ergonomic knob:

- Turn the knob to elevate your legrest. You can adjust it when red is visible under the knob.
- Check after adjustment to make sure the ELR is locked in place. The legrest should click when locked, with no red visible under the knob.



### ⚠ WARNING

Attention Premium dealers and qualified technicians. If you have any questions, or do not understand the instructions and warnings in this Instruction sheet, please contact the Sunrise Medical Technical Service Dept. before operating and/or servicing the Breezy wheelchair. Failure to do so may result in damage and/or injury.