SUNRISE MEDICAL

I. Location and Attachment

VOYAGE POSTURAL COMPONENTS COMFORT FIT ATS ANTERIOR TRUNK SUPPORT LI'L KIDDOS

INSTALLING THE COMFORT FIT ATS (LI'L KIDDOS)

The ATS can be mounted to the backshell using a $2 \times M6 \times 1 \times 12$ bolts, 4×12 washers, 4×12 nylock nuts, $2 \times M6 \times 1 \times 35$, and re-using the 2 removed screws.

I 2 3 4 5	
-----------	--

- a. The upper belt brackets can mount to any available area in the open slots (A).
- b. Using the 4mm hex key and a 10mm wrench, remove the bolts circled in red (B). The lower belt brackets will mount in this location.
- c. Mount the lower belt brackets in location (B) by either reattaching the hardware (C) that was removed previously, or by using new hardware that may have been provided with the belt.
- d. Use a nylock nut, washer, and M6 bolt for each upper belt bracket (D)



 a. Check to see if the backshell is difficult to recline after tightening the bolts. If the recline mechanism is stiff, you may loosen the screw(C) slightly until the recline functions freely.









