



ACCESS

PARTS MANUAL


AUGUST 2022



improving people's lives

Cover	1
Product	1
QUICKIE ACCESS	5
BASE FRAME	5
FRAME	5
BASE FRAME AND TILT FRAME ASSEMBLY	5
GAS STRUT	7
FOOT RELEASE	8
TRANSIT KIT ISO	9
TILT CABLES	9
TILT HANDLE & CABLE FOR STROLLER HANDLE AND RECLINING BACKREST	10
TILT HANDLE & CABLE FOR HEIGHT ADJUSTABLE BACKREST	11
AXLE PLATES & AXLES	12
QUICK RELEASE AXLES MATRIX	13
STANDARD AXLE PLATE	14
THREADED AXLE	15
CASTERS & FORKS	15
CASTERS	15
3 SPOKE CASTERS	16
ALUMINUM CASTERS	18
MICRO CASTERS	19
8" x 2" CASTERS	20
CASTER TIRES & TUBES	21
CASTER PLATES & CASTER FORKS	21
MULTI-POSITION FORK ASSEMBLY	22
ANTI-TIPS	22
ANTI-TIP	23
WHEEL LOCKS	24
PUSH/PULL TO LOCK WHEEL LOCK	25
WHEEL LOCK EXTENSION HANDLE	26
ATTENDANT WHEEL LOCK	27
DRUM BRAKE	28
HUB LOCK	29
WHEELS & TIRES	30
WHEELS	30
12" MAG	31
16" MAG	32
5 SPOKE MAG	33
QUICKIE SPOKE	35
TIRES	35
PNEUMATIC & PNEU. AIRLESS TIRE	36
FULL POLY TIRE	37
LOW POLY TIRE	38
HANDRIMS	38
ALUMINUM ANODIZED (SCREW MOUNT)	39
ALUMINUM ANODIZED (TAB MOUNT)	40
ALUMINUM ANODIZED 5 SPOKE MAG (SCREW MOUNT)	41
PLASTIC COATED (SCREW MOUNT)	42
PLASTIC COATED (REG TAB MOUNT)	43
PLASTIC COATED 5 SPOKE MAG (SCREW MOUNT)	44
NATURAL FIT (SCREW MOUNT)	45
NATURAL FIT (TAB MOUNT)	47
NATURAL FIT 5 SPOKE MAG (SCREW MOUNT)	49
TUBES	49
TUBES AND AIRLESS INSERTS	50
SEATING AND POSITIONING	50
SEAT	50
SEAT FRAME	51
SEAT PAN	53
BACK	54
BACKREST UPHOLSTERY	54
TENSION ADJ BALLISTIC NYLON UPHOLSTERY	55
TENSION ADJ EXO UPHOLSTERY	57
BACKRESTS	61
RIGID FRAME HEIGHT ADJ ANGLE ADJ PUSH HANDLE BACKPOST/STROLLER	62
FIXED HEIGHT STROLLER HANDLE BACKPOST	63
RECLINING BACKREST FIXED ASSEMBLY	65
RECLINING BACKREST HT ADJUSTABLE W/ADJ STROLLER HANDLE	67
STROLLER HANDLE EXTENSION-REMOVABLE	69
TOWEL BAR	71


DYNAMIC BACK	73
TILT CABLES	74
TILT HANDLE & CABLE FOR STROLLER HANDLE AND RECLINING BACKREST	75
TILT HANDLE & CABLE FOR HEIGHT ADJUSTABLE BACKREST	76
HANGERS & FOOTPLATES	77
EXTENSION MOUNT HANGERS	77
60 DEG SWING IN/OUT HANGER	78
70 DEG SWING IN/OUT HANGER	79
80 DEG SWING IN/OUT HANGER	80
EXTENSION MOUNT ELR & ALR (PLUNGER STYLE)	80
SWING IN-OUT EXT MOUNT ELR & ALR RETROFIT (PLUNGER STYLE) (EFFECTIVE 2/6/17)	81
SWING IN-OUT EXT MOUNT ELR REPLACEMENT PARTS (PLUNGER STYLE) (EFFECTIVE 2/6/17)	82
SWING IN-OUT ALR EXT MNT REPLACEMENT PARTS (PLUNGER STYLE)(EFFECTIVE 2/6/17)	84
ELR & ALR CALF PAD 14 -15"	86
ELR & ALR CALF PAD 16-20"	87
ELR ALR KNUCKLE COVERS	88
EXTENSION MOUNT FOOTPLATES & EXTENSION TUBES	88
COMPOSITE FOOTPLATE	89
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE	93
ADULT ALUMINUM ANGLE-ADJ FOOTPLATE EXT MOUNT	96
ELR/ALR ALUMINUM FOOTPLATE	99
HIGH MOUNT EXTENSION TUBE	101
JAY LOWER EXTREMITY S/A SUPPORT	103
ARMRESTS & RECEIVERS	103
FLIP-UP ARMRESTS	103
LENGTH ADJUSTABLE LOCKING FLIP-UP ARMREST (USE ON DAB FRAMES ONLY)	104
ADJUSTABLE LOCKING FLIP-UP ARMREST (USE ON DAB FRAMES ONLY)	106
SINGLE POST HEIGHT ADJUSTABLE ARMREST	107
DUAL POST FLIP BACK HT ADJ ARMREST	109
ACCESSORIES	110
BACKPACKS & SEAT POUCHES	110
BACKPACKS AND SEAT POUCHES	111
LEG STRAPS	111
SOFT FEEL LEG STRAP	112
POSITIONING BELTS	112
AIRCRAFT BUCKLE POSITIONING BELTS & ATTACHMENT KITS	113
AUTO BUCKLE POSITIONING BELT & ATTACHMENT KITS	115
JAY 2 POINT POSITIONING BELT	117
TRAYS	119
TUFFAK TRAY (MANUAL & PEDIATRIC CHAIRS)	119
PADDED STOMACH CUTOUT	121
PROTRACTOR PAD	122
HINGED OVERLAY NON-REMOVABLE	123
HINGED OVERLAY REMOVABLE	124
TRAY RIBS TUFFAK	126
IMPACT GUARD NEOPRENE & GEL VELCRO PADS	127
JAY LOWER EXTREMITY S/A SUPPORT	128
ANGLE ADJUSTABLE IV POLE	129
OXYGEN TANK HOLDER	130
WATER BOTTLE AND CAGE	131
TIRE PUMP	132
TOOL KIT	133

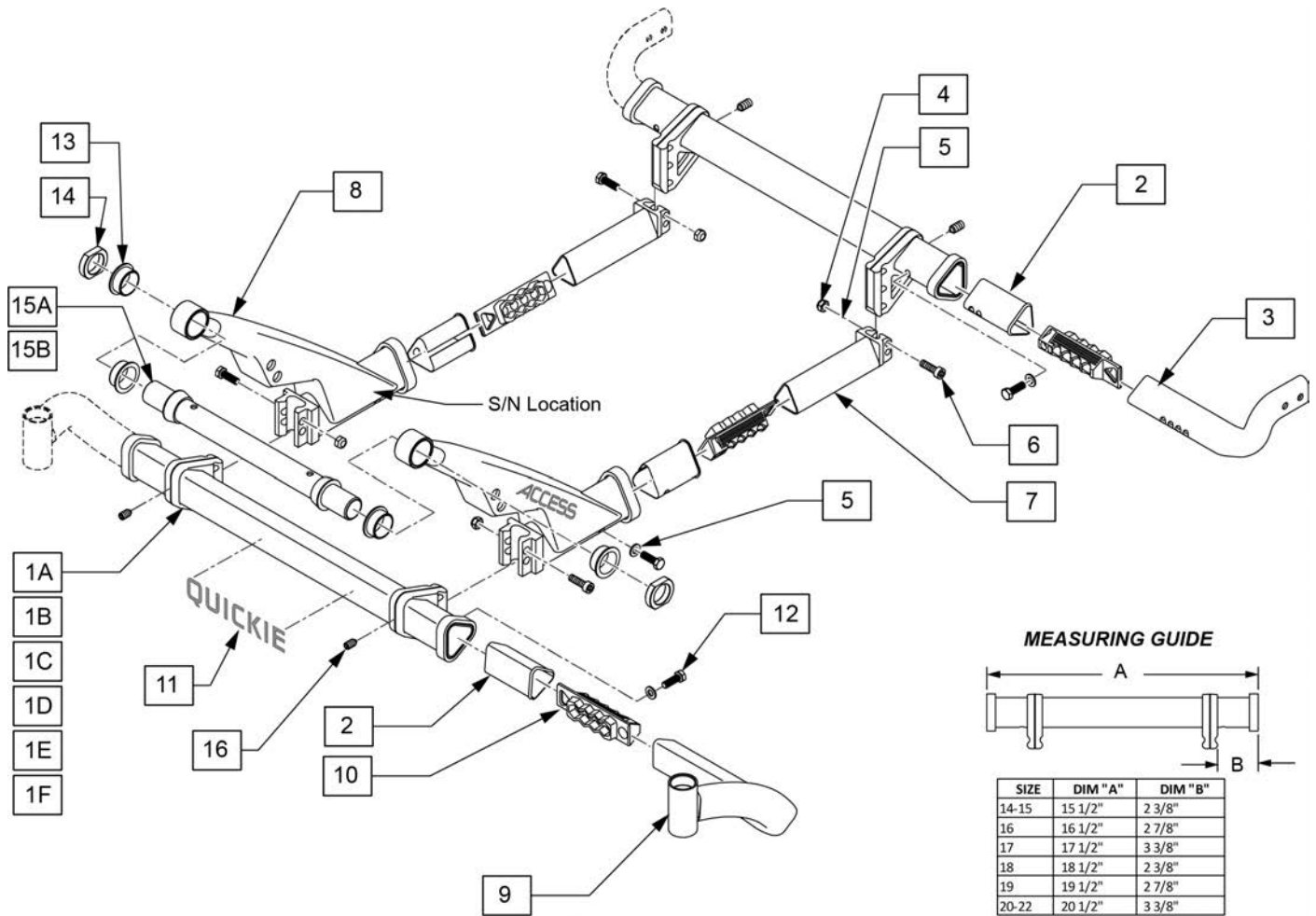



Pos.	Item Number	Description	Remarks
1A	251361	CROSS TUBE WELD 14-15	front or rear
1B	251362	CROSS TUBE WELD 16	front or rear
1C	251363	CROSS TUBE WELD 17	front or rear
1D	251364	CROSS TUBE WELD 18	front or rear
1E	251365	CROSS TUBE WELD 19	front or rear
1F	251366	CROSS TUBE WELD 20-22	front or rear
2	251017	BUSHING, TRI	
3	168933S	REAR LEG PAIR	Pair
9	168934S	CASTER LEG PAIR	Pair
10	250702	HOLDER ASSY	
11	MK-129804	Quickie Decal, Access Rear Axle	
13	251348	BEARING IGUS GFI-1618-08	
14	251504	PIVOT NUT	
15A	251493	PIVOT TUBE WELDMENT A	
15B	251530	PIVOT TUBE WELDMENT B	
1A-15A	168774S	BASE & TILT FRAME ASSEMBLY 14-15	
1B-15A	168775S	BASE & TILT FRAME ASSEMBLY 16	
1C-15A	168776S	BASE & TILT FRAME ASSEMBLY 17	
1D-15B	168777S	BASE & TILT FRAME ASSEMBLY 18	
1E-15B	168778S	BASE & TILT FRAME ASSEMBLY 19	
1F-15B	168779S	BASE & TILT FRAME ASSEMBLY 20-22	
16	243522	SS M8X1.25X10 DOG END	

Pos.	Item Number	Description	Remarks
4,6	168772	FRAME FASTENER KIT	
5,10,12	168773	LEG FASTENER KIT	


Tilt Configuration	User's Weight	Part Number
5° - 20°	up to 300 lbs (136 kg)	253448
0° - 25°	up to 300 lbs (136 kg)	252535

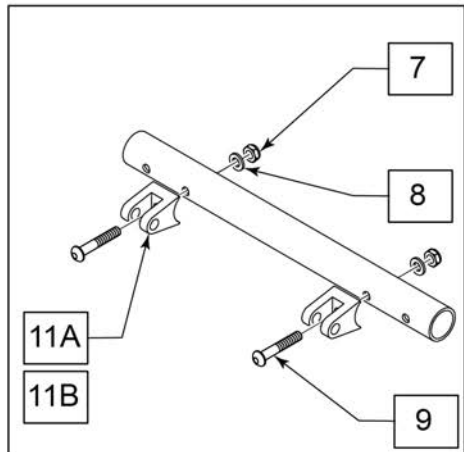


Configuration	User's Weight	Gas Shock		Position	
		1A	1B	1	2
1	up to 150 lbs (up to 68 kg)	☒	☐	☒	☐
2	151-180 lbs (69-82 kg)	☒	☐	☐	☒
3	181-210 lbs (83-95 kg)	☐	☒	☒	☐
4	211-240 lbs (96-109 kg)	☐	☒	☐	☒
5	241-270 lbs (110-122 kg)	☐	☒	☐	☒
6	271-300 lbs (123-136 kg)	☐	☒	☐	☒

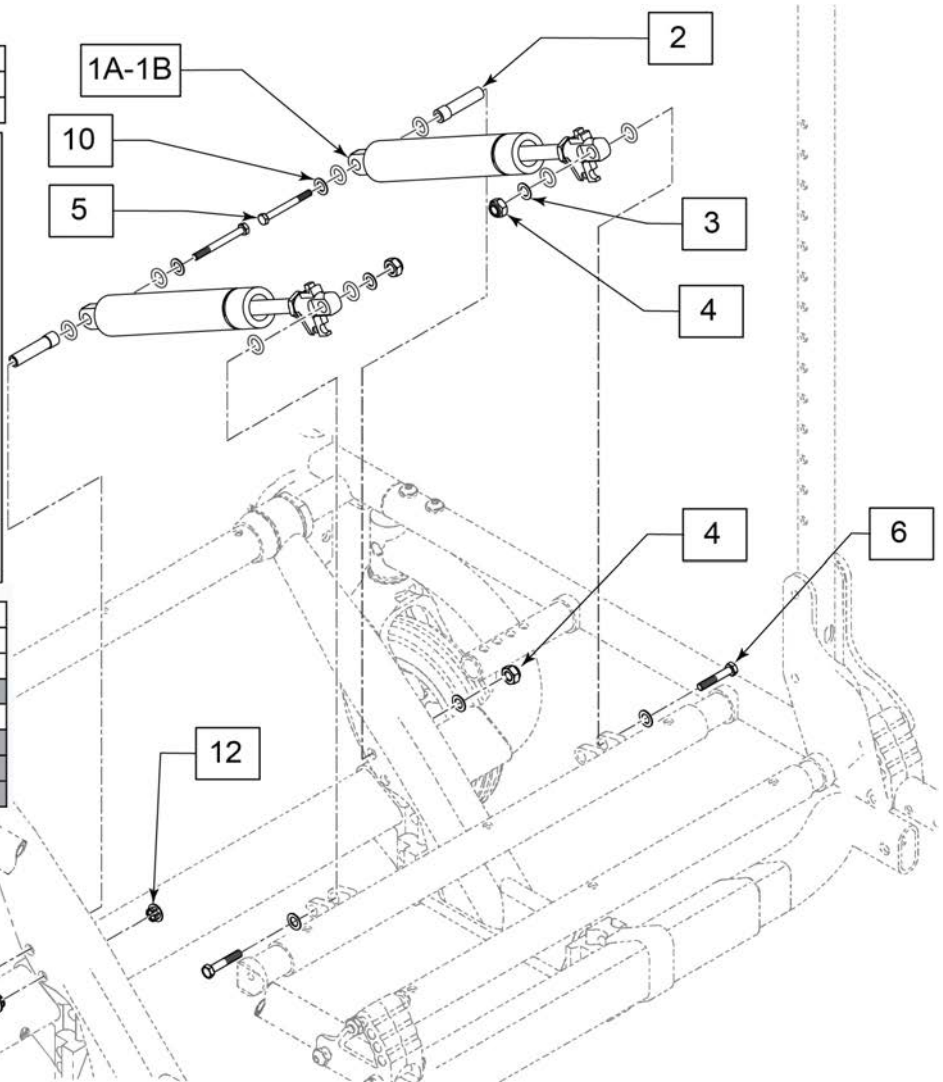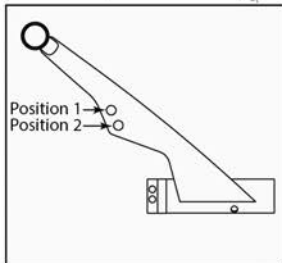


Pos.	Item Number	Description	Remarks
2-10	168782	GAS STRUT FASTENER KIT	
1A,3-6,10	168780	GAS STRUT MED W/FASTENERS	
1B,3-6,10	168781	GAS STRUT HIGH W/FASTENERS	
11A	252535	BOLT ON STRUT MOUNT	0-25 Deg
11B	253448	STRUT MOUNT -5 TO 20	-5-20 Deg
12	255697	NYLON PLSTIC SNAP-IN PNEL PLUG	4 RQ'D

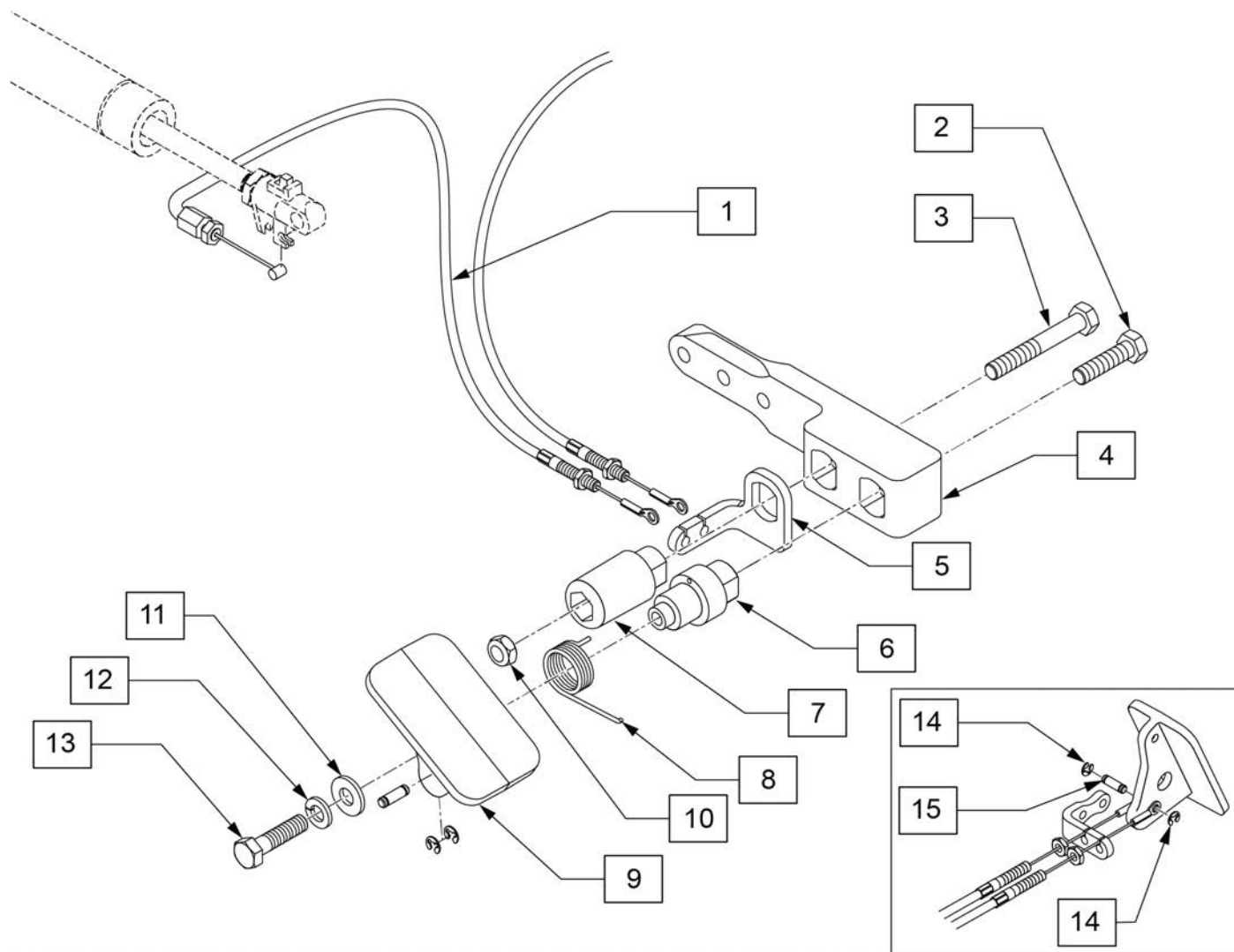


Pos.	Item Number	Description	Remarks
1	254732	TILT FOOT RELEASE CABLE 15"	Use on pedal side Gas Strut
1	254718	TILT FOOT RELEASE CABLE 25"	Use on non-pedal side Gas Strut
5	132258	FOOT RELEASE ADAPTOR BRACKET A	Right Mount
5	132644	FOOT RELEASE ADAPTOR BRACKET B	Left Mount
6	106377	SHAFT PIVOT PEDAL	
7	107167	POST STOP FOOT RELEASE	
8	106668	TORSION SPRING RH	
9	107165	PEDAL FOOT RELEASE	
1-15	168783	FOOT RELEASE ASSM LH	Right Mount
1-15	168784	FOOT RELEASE ASSM RH	Left Mount
2,3,10-15	168785	FOOT RELEASE FASTENERS	

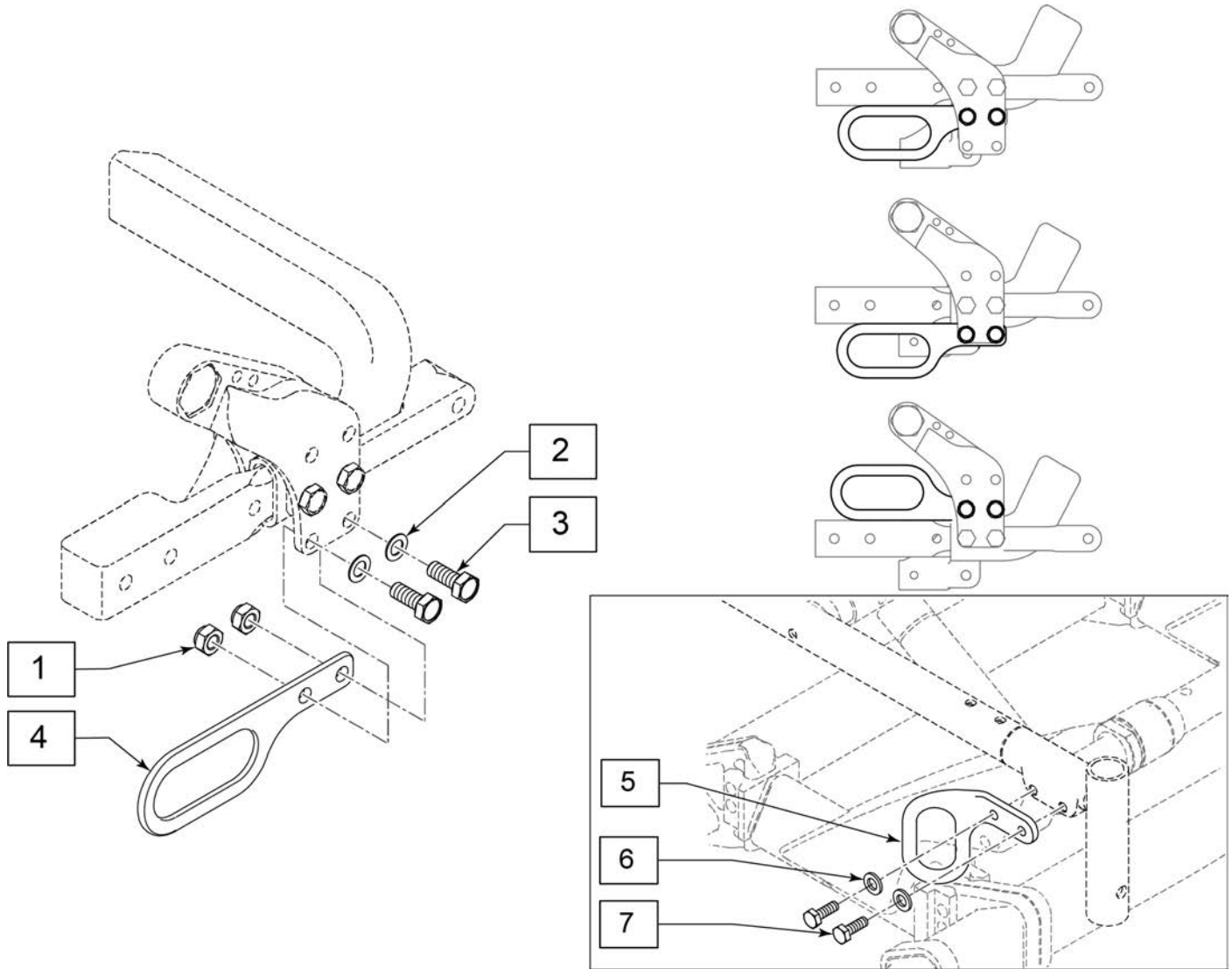

Pos.	Item Number	Description	Remarks
1-7	168786	COMPLETE TRANSIT KIT ACCESS	w/transit decals
1,2,3,6,7	168787	TRANSIT FASTENERS ACCESS	




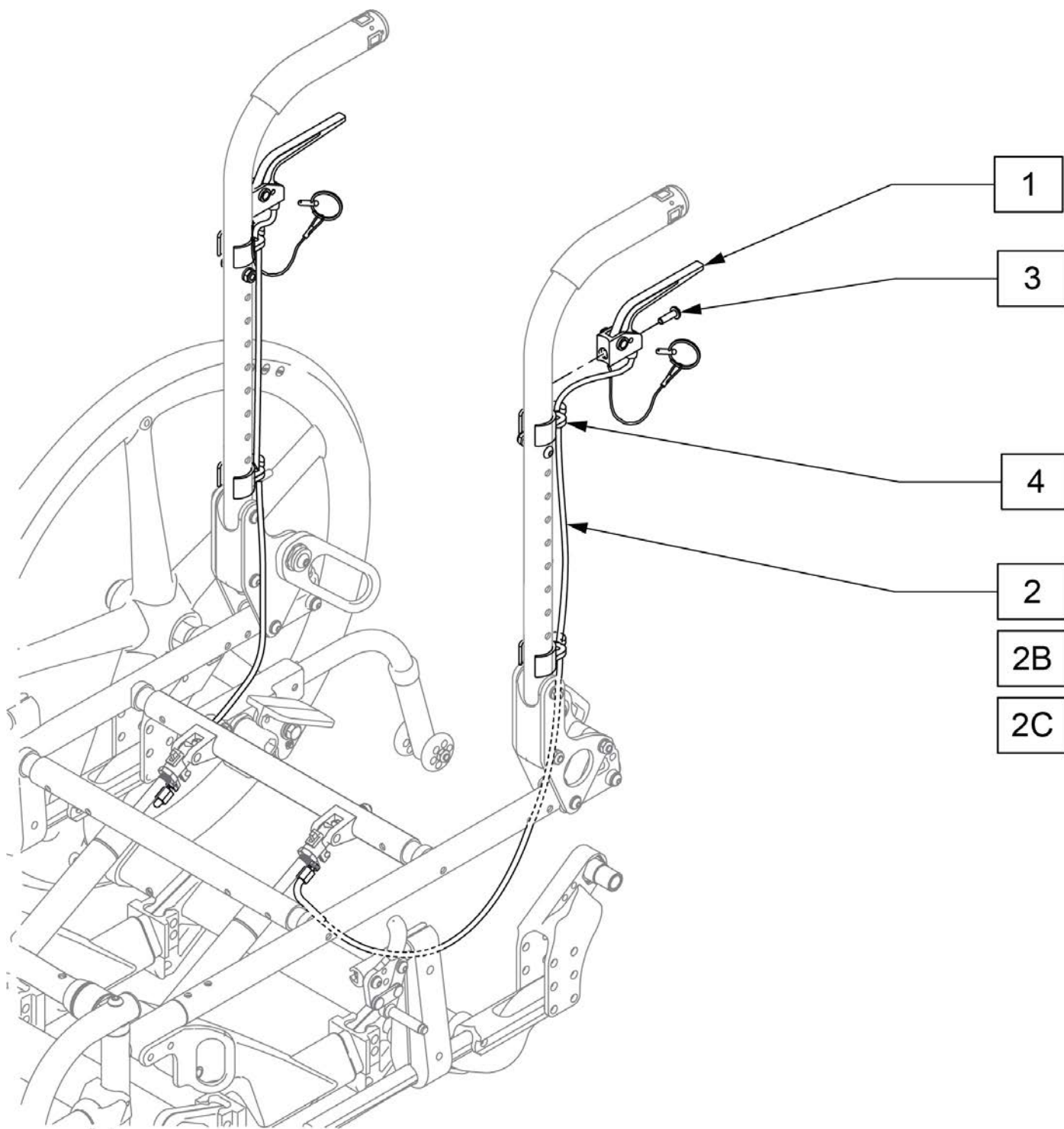



Pos.	Item Number	Description	Remarks
1	580110	CYL LOCK LEVER ASSM	
2	253844	CABLE ASSY TILT KNEE PIVOT LONG	LONG 44"
3	281120	10 -32 X 5/8 PAN HD PHIL	
4	108134	CLIP CABLE 1" PACK (10)	Pack of 10

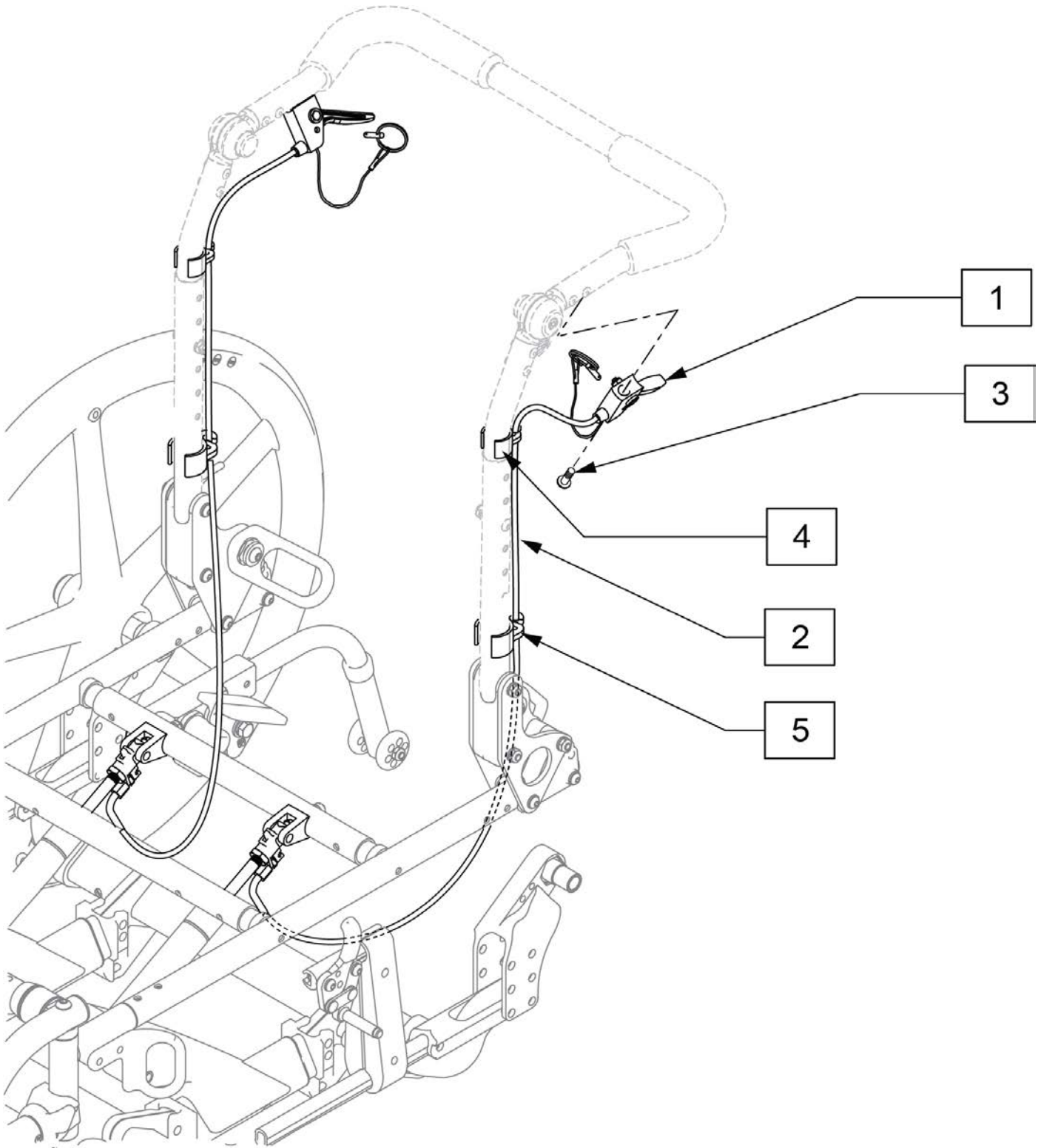

Pos.	Item Number	Description	Remarks
1	580109	PLUNGER PIN LEVER ASSM	
2	253845	CABLE ASSY TILT MED KNEE PIVOT	X-LONG 54"
3	281120	10 -32 X 5/8 PAN HD PHIL	
4	108135	CLIP CABLE 7/8" PACK (10)	Pack of 10
5	108134	CLIP CABLE 1" PACK (10)	Pack of 10


Note: If chair has a Hub Lock please reference that parts page for replacement Axle Sleeve and Axles.

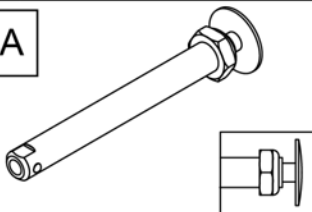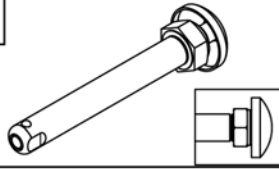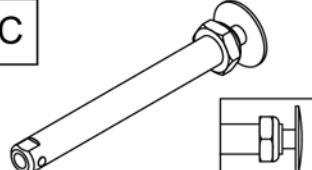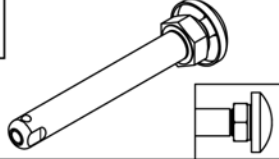Wheel Type	Axle Type		Axle P/N	Spacer P/N
5 Spoke Mag	Quick Release	1A 	251661 A = 3 3/16"	N/A
12" & 16" 3-Spoke Mag	Quick Release	1B 	252149 A = 2 3/4"	N/A
Standard Spoke	Quick Release	1C 	252151 A = 3 1/4"	N/A
Drum Brake	Quick Release	1D 	251438 A = 3 1/4"	N/A

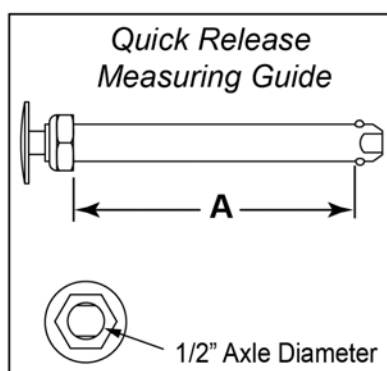


Pos.	Item Number	Description
1A	251661	QR AXLE ASSEMBLY
1B	252149	QR AXLE MAG SHRT M12 NUT
1C	252151	QR AXLE M12 NUT MED ASSM
1D	251438	QR AXLE

Note: If chair has a Hub Lock please reference that parts page for replacement Axle Sleeve and Axles.

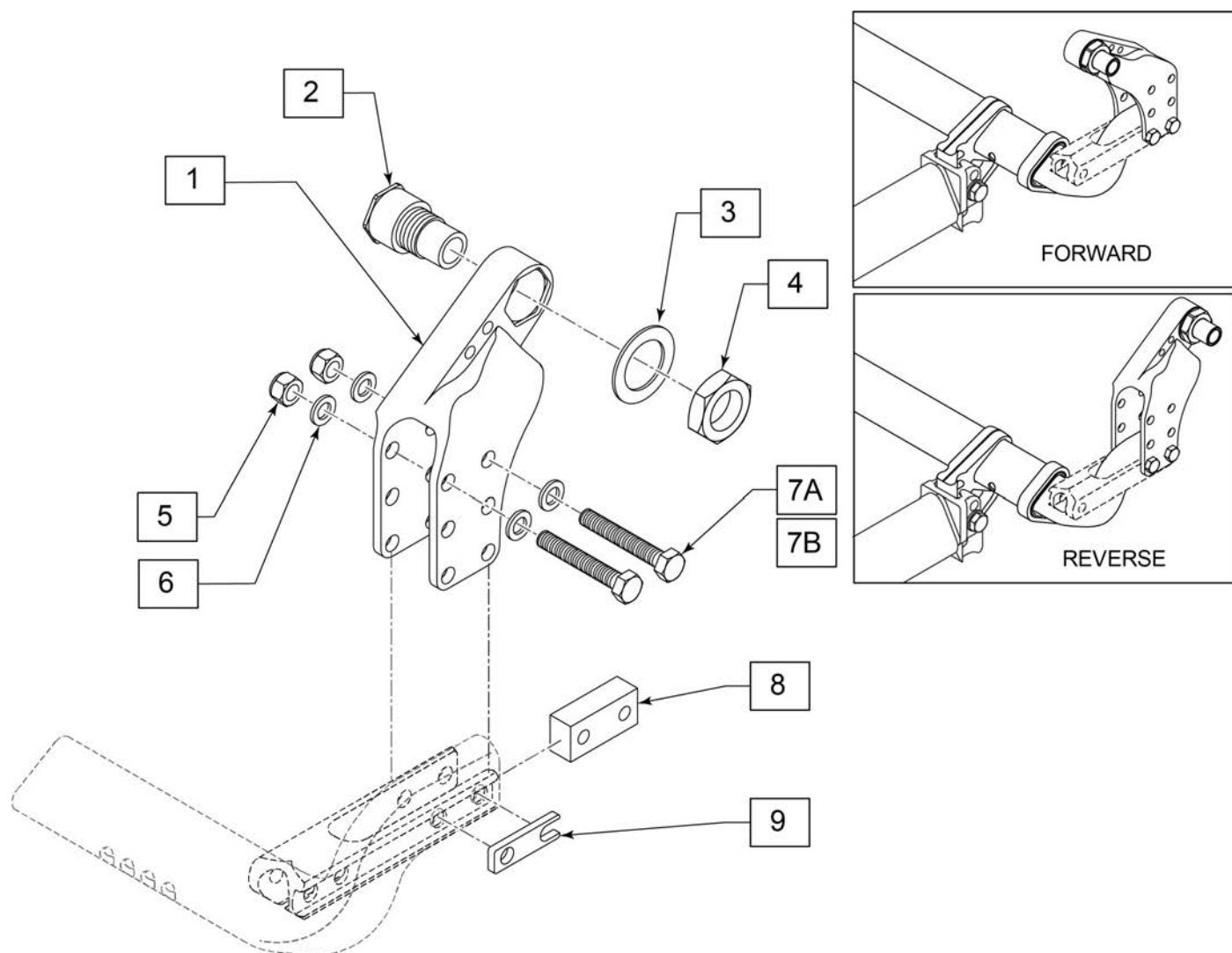


Pos.	Item Number	Description	Remarks
3	104195	NUT JAM ALUMINUM 20MM X 1.5	
4	134384	WASHER FLAT M20	
1-7A,7B,9	168788S	AXLE PLATE ASSEMBLY STD A	
1-7A,7B,9	168789S	AXLE PLATE ASSEMBLY STD B	
5-7A,7B	168792	AXLE PLATE FASTENER KIT	One side
8	256190	M8 2 Hole Spacer	
9	256200	M8 Nut Plate	

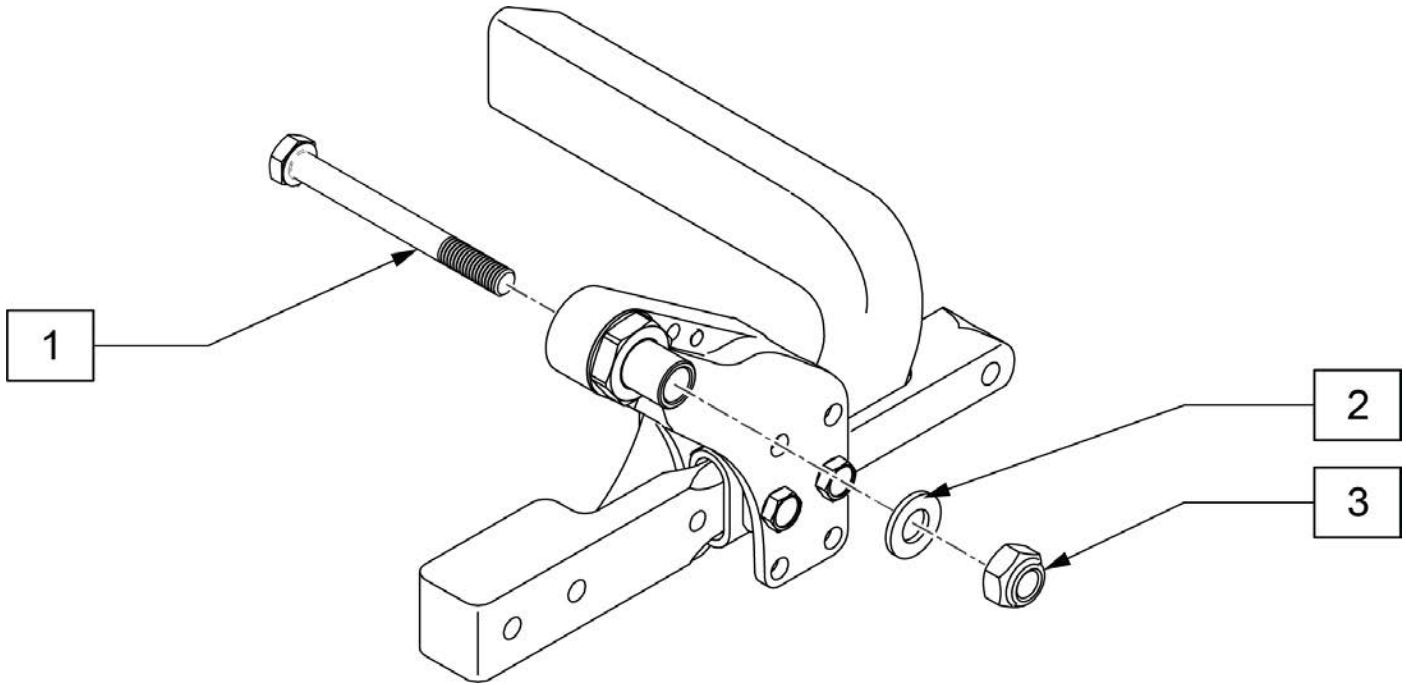

Pos.	Item Number	Description	Remarks
1-3	168793	THREADED AXLE ACCESS	For one side. Fits Mag 12 & 16" wheels only


NOTE: 7" and 8" Casters are not available with Integral Bearing Forks.

Note: When 6" casters or larger casters are used, Anti Flutter Kit must be installed. See Standard Multi Position Fork page.

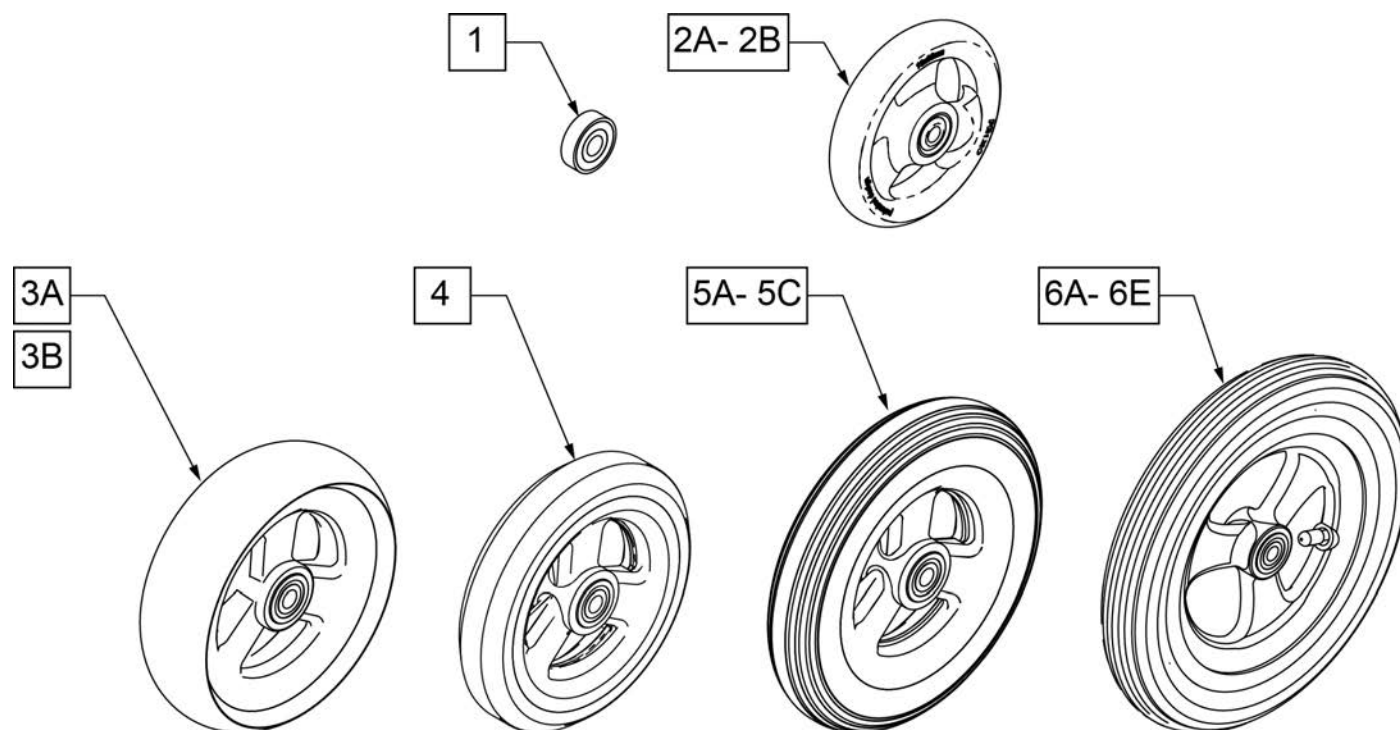


**CASTER
MEASURING GUIDE**

PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH	PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH
101957	4.3	1.0	101256	5.75	1.0
101098	4.0	1.0	101257	5.9	1.0
101259	4.9	1.0	400212	8.0	1.0
101255	5.7	1.0	400213	7.8	1.0
101254	4.9	1.0	400214	6.9	1.0
			400215	7.25	1.0


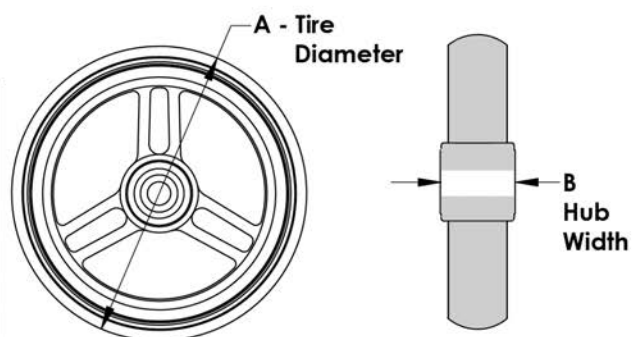


Pos.	Item Number	Description	Remarks
1	349502	7/8"x 5/16" BEARING	
2A	101957	4"x1-1/4 POLY 3 SPOKE CSTR	4" X 1.25 SEMI-PNEUMATIC
2B	101098	CASTER 4" PLASTIC/POLY W/BRG	4" LOW PROFILE POLY
3A	101259	5"x1-1/2" POLY 3 SPOKE CSTR	5" x 1.5" SEMI-PNEUMATIC
3B	101255	6"x1-1/2" POLY 3 SPOKE CSTR	6" x 1.5" SEMI-PNEUMATIC
4	101254	5" LOW POLY 3 SPOKE CASTER	5" LOW PROFILE POLY
5A	101256	6" SEMI PNEU 3 SPOKE CASTER	6" POLY
5B	101257	6" PNEUMATIC 3 SPOKE CASTER	6" PNEUMATIC
5C	383245	6" PNEUMATIC TIRE & TUBE	
6A	400212	CASTER 8" PNEU 3 SPK ASSM	8" PNEUMATIC
6B	400213	CASTER 8" POLY 3 SPK ASSM	8" POLY
6C	400214	CASTER 7" LOW POLY 3 SPK ASSM	7" x LOW PROFILE POLY
6D	383255	8" PNEUMATIC TIRE & TUBE	
6E	400215	CASTER 8" WIDE POLY 3 SPK ASSM	8" x 1.5" LOW POLY

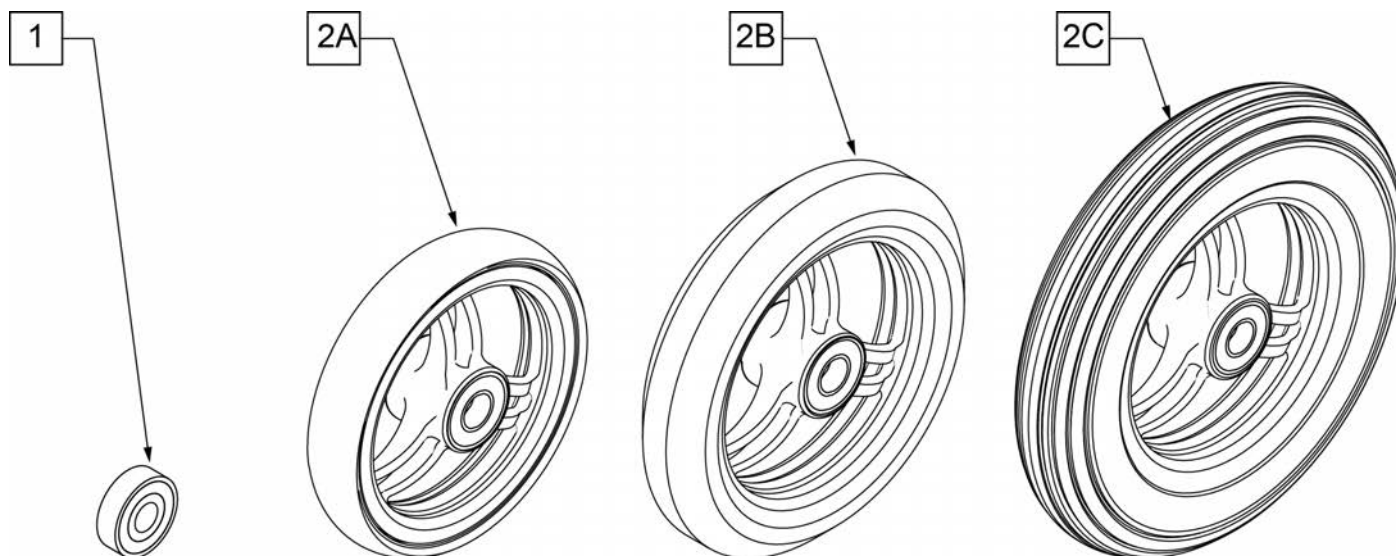


**CASTER
MEASURING GUIDE**

"A" TIRE DIAMETER	"B" HUB WIDTH
4"	1"
5"	1"
6"	1"

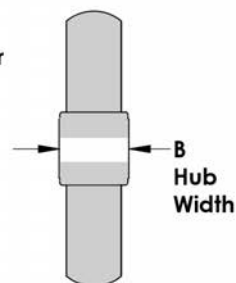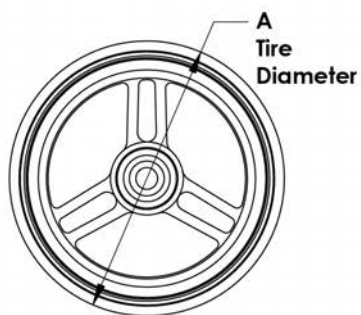


Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	101946-001	CASTR 4X1 ALUM/POLY W/BRG CLR	K0077	4" x 1" ALUMINUM POLY
2B	101946-002	CASTR 5X1 ALUM/POLY W/BRG CLR	K0077	5" x 1" ALUMINUM POLY
2C	101946-003	CASTR 6X1 ALUM/POLY W/BRG CLR	K0077	6" x 1" ALUMINUM POLY

Note: to ensure that the lights will function in the Micro Light Up Caster, qty 2 each of p/n 106947 will be required per caster when retrofitting Micro Light Up Casters to the chair.

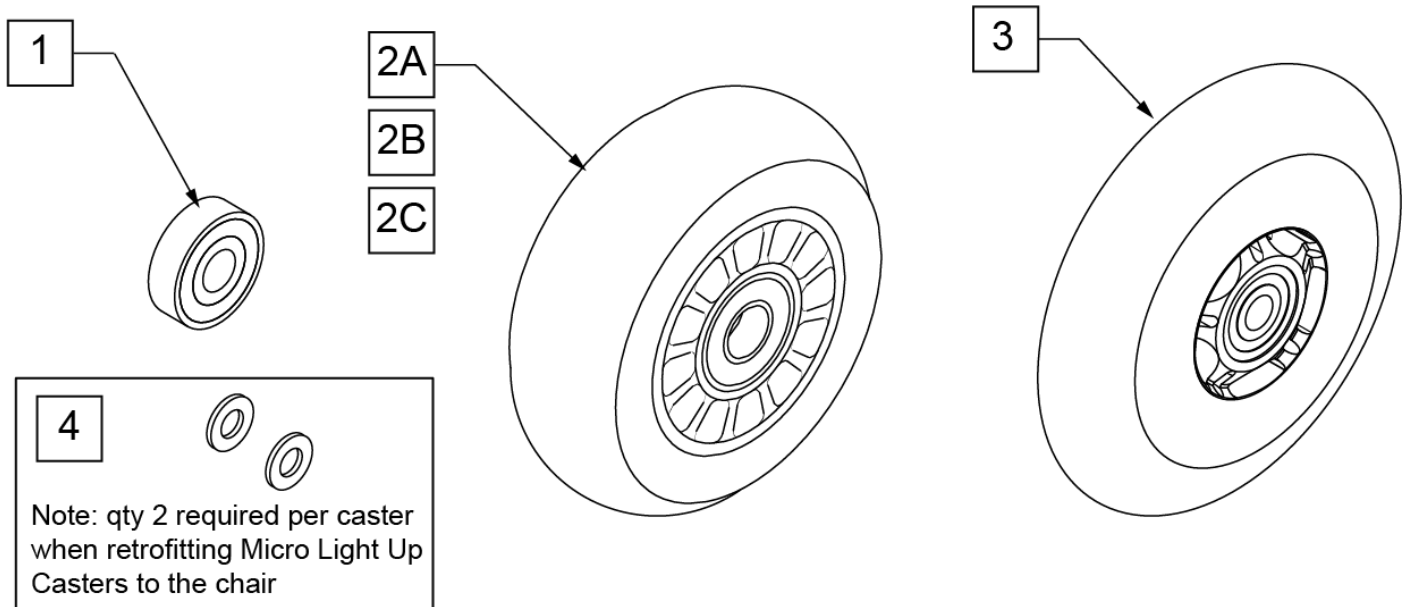


CASTER MEASURING GUIDE


PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH
400108	4.0	1.0
400107	3.0	1.0
052129	4.7	1.0

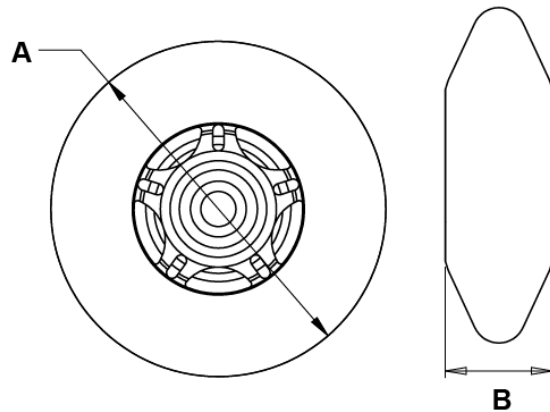

Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	400108	4" MICRO CASTER LIGHTED	K0077	4" MICRO LIGHTED
2B	920025	3" MICRO CASTER W/ BEARING -PAIR	K0077	PAIR 3" MICRO
2C	400107	3" MICRO CASTER LIGHTED	K0077	3" MICRO LIGHTED
3	052129	5" MICRO CASTER LIGHTED	K0077	5" MICRO LIGHTED
4	106947	WASHER FLAT THIN AN 5/16 ROH		

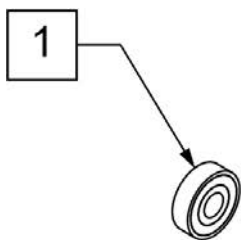


2A -2D

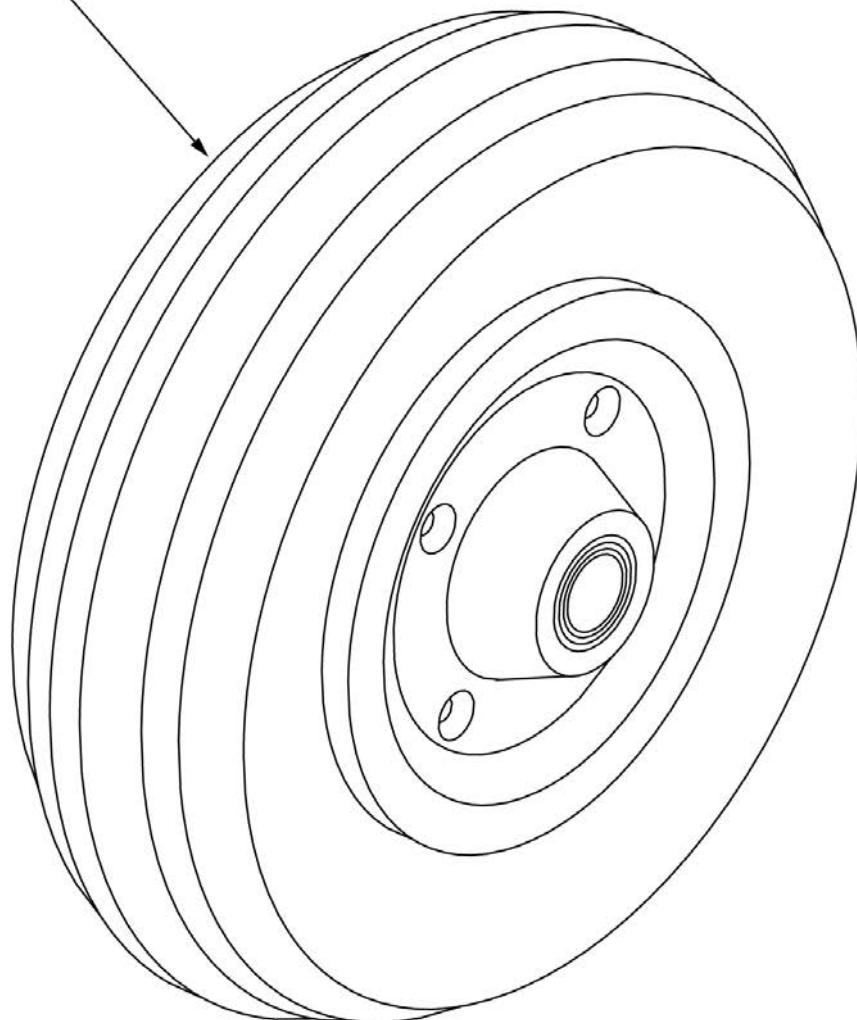


**CASTER
MEASURING GUIDE**

TIRE DIAMETER	HUB WIDTH
8"	2.6"


Pos.	Item Number	Description	Remarks
1	349502	7/8"x 5/16" BEARING	
2A	400700	8" X 2" CMPST W/PNEU & BEARING	8" x 2" PNEUMATIC
2B	400702	8" X 2" CMPST W/PNEU INSRT&BRG	8" x 2" PNEUMATIC W/ INSERT
2C	384700	TIRE PNEUMATIC 8" x 2"	
2D	384730	INNER TUBE CASTER 8" x 2"	

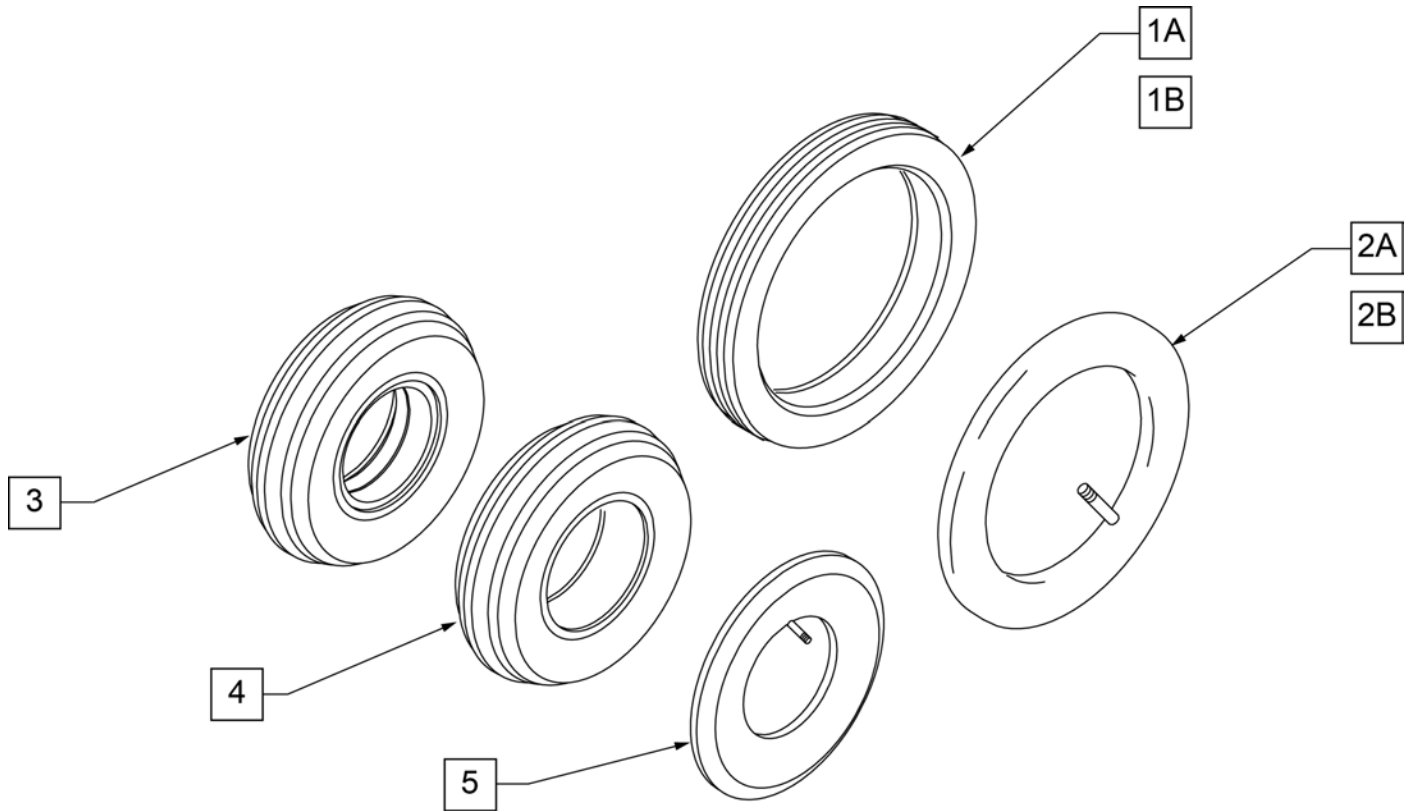



Pos.	Item Number	Description
1A	383250	8" PNEUMATIC TIRE
1B	383240	6" PNEUMATIC TIRE
2A	382250	8" TUBE
2B	382245	6" TUBE
3	384701	TIRE FOAMED INSERT 8" x 2"
4	384700	TIRE PNEUMATIC 8" x 2"
5	384730	INNER TUBE CASTER 8" x 2"
1A,2A	383255	8" PNEUMATIC TIRE AND TUBE

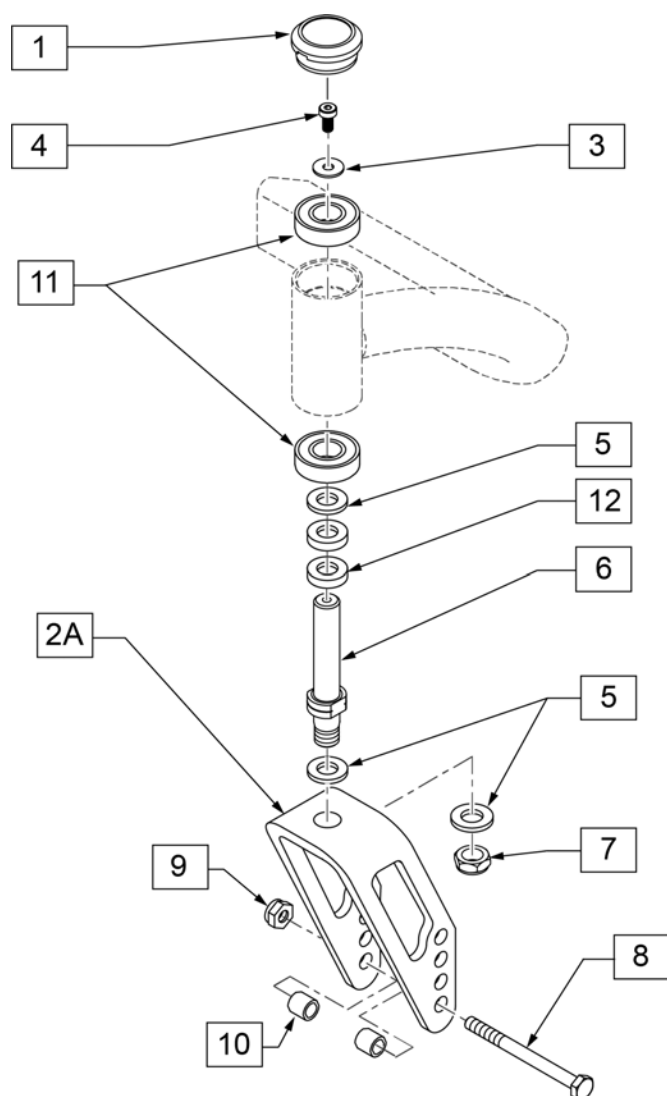


MEASURING GUIDE

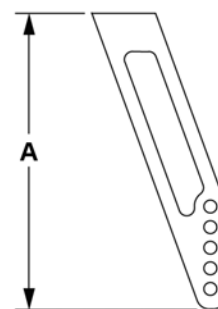

POS	DESCRIPTION	DIM "A"
2A	ANO FORK	5"
2B	LONG	7"
2C	8" X 2"	6 1/8"

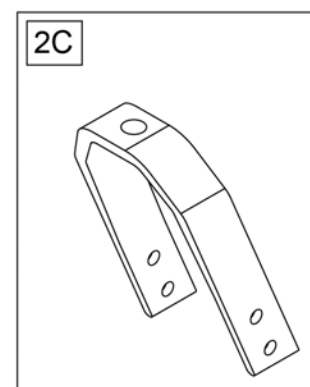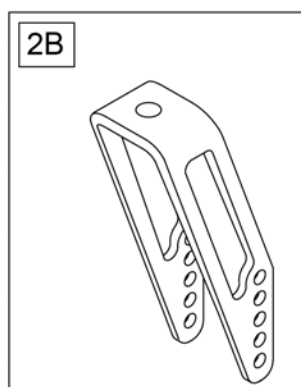


Pos.	Item Number	Description	Remarks
3-7,12	168794	REPLACEMENT STEM KIT	
2A,3-12	168795S	ANO FORK REPLACEMENT KIT	Short
2B,3-12	169009S	LONG FORK REPLACEMENT KIT ANO	Tall
2C,3-12	169008S	8" X 2" FORK REPLACEMENT KIT ANO	
3,4,5,12	168796	STEM FASTENER KIT	
8-10	168797	CASTER AXLE KIT	
11	349500	1 1/8 X 1/2 BEARING	

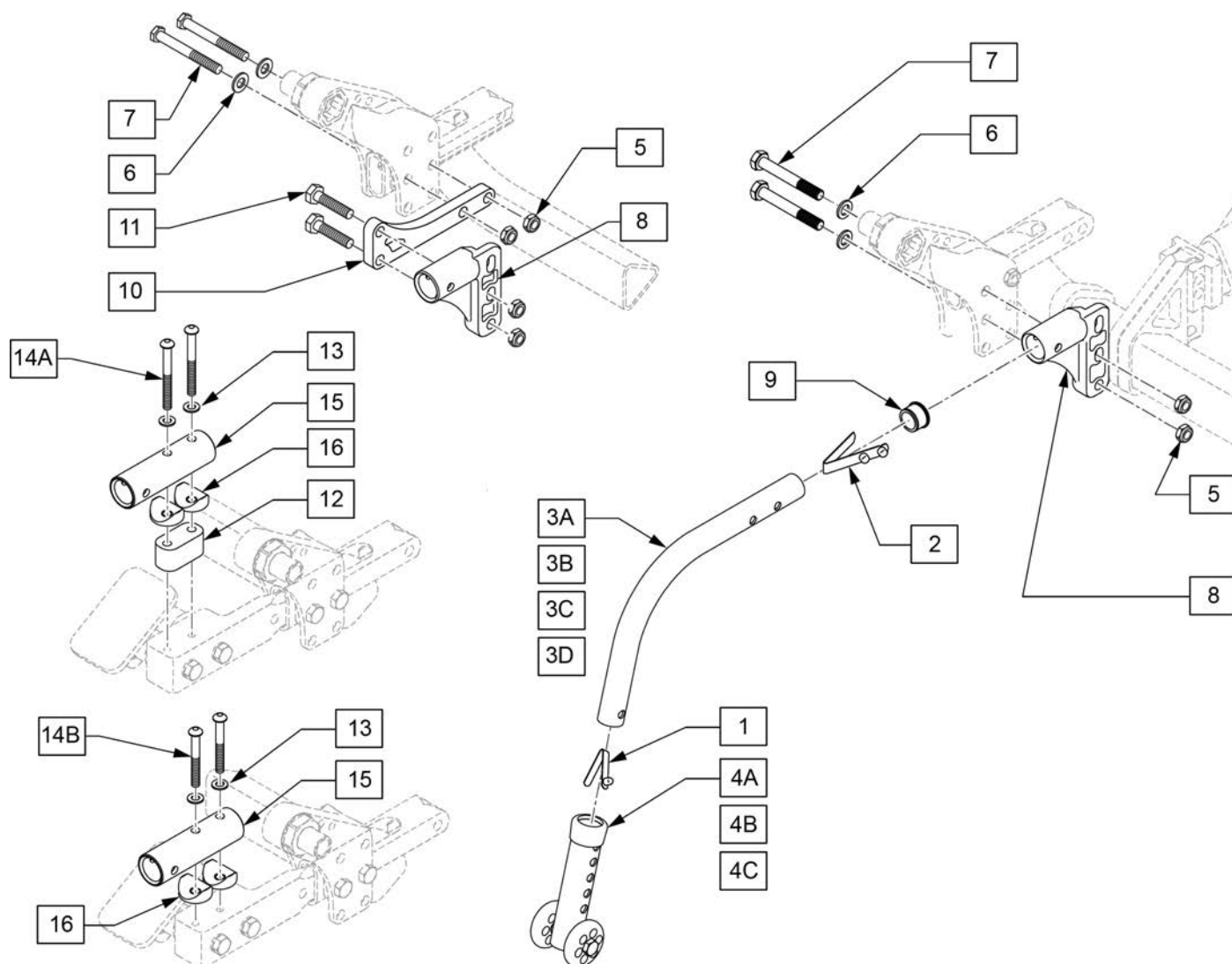


Pos.	Item Number	Description	Remarks
1-3A	882964	ANTI-TIP TUBE SMALL BLK	Right
1-3A	882965	ANTI-TIP TUBE SMALL BLK	Left
1-3B	882898	ANTI-TIP TB KDZ R BL W/SNP BT	Right
1-3B	882899	ANTI-TIP TB KDZ L BL W/SNP BT	Left
1-3C	104219-002	ANTI TIP TUBE ASSY CRVD RH	Right
1-3C	104219-001	ANTI TIP TUBE ASSY CRVD LH	Left
1-3D	882896	ANTI-TIP TB REG R B W/SNAP BT	Right
1-3D	882897	ANTI-TIP TB REG L B W/SNAP BT	Left
2	301100	1/4 SNAP BUTTON SINGLE	
1	301109	SNAP BUTTON DUAL	
4A	881512	ANTI-TIP SLV W/WHLS BLK 4H	
4B	104157	ANTI TIP ARM ADJUSTER	
4C	104639	ANTI TIP ARM ADJ. MED	
4D	104640	ANTI TIP ARM ADJ. LONG	
5	210085	NUT HEX NYLOCK M8 BLACK	
6	106946	WASHER FLAT AN 5/16 ROHS	
7	014161	HHCS M8 X 1.25 X 60 FULL THRD	
9	560735	7/8" ROUND TUBE PLUG	

Pos.	Item Number	Description	Remarks
8	251050	RECEIVER, ANTI-TIP, M8 - A	Right
8	251051	RECEIVER, ANTI-TIP, M8 - B	Left
10	256125	ANIT-TIP TRANSITION BRACKET	
11	013923	HHCS M8 X 1.25 X 25 W PTCH	
12	256128	SPACER PLATE M6 X 9.16 THICK	
13	250104	WASHER FLAT M6	
14A	113320	BHCS,M6X1X40,PART,PL	
14B	058901-004	BHCS M6X1X55 PART PL	
15	256127	ANIT-TIP TUBULAR RECEIVER	
16	561004	Q2 SADDLE	
5-8	168812	ANTI TIP RECEIVER KIT 1 A	Right
5-8	169730	ANTITIP RECEIVER ASSM 1 B	Left
5-8,10,11	168813	ANTI TIP RECEIVER 2 KIT A	Right
5-8,10,11	169731	ANTITIP RECEIVER ASSM 2 B	Left
12-14A,15,16	168814	ANTITIP RECEIVER ASSM 3	Right or Left
13,14B,15,16	168815	ANTITIP RECEIVER ASSM 4	Right or Left
1-3B,4A	168816	REAR ANTI TIP ASSM 1	Right
1-3B,4A	169732	ANTITIP TUBE ASSEMBLY 1 LH	Left
1-3B,4B	168817	REAR ANTI TIP ASSM 2	Right
1-3B,4B	169733	ANTITIP TUBE ASSEMBLY 2 LH	Left
1-3C,4B	168818	REAR ANTI TIP ASSM 3	Right
1-3C,4B	169734	ANTITIP TUBE ASSEMBLY 3 LH	Left
1-3C,4C	168819	REAR ANTI TIP ASSM 4	Right
1-3C,4C	169735	ANTITIP TUBE ASSEMBLY 4 LH	Left
1-3A,4B	168820	REAR ANTI TIP ASSM 5	Right
1-3A,4B	169736	ANTITIP TUBE ASSEMBLY 5 LH	Left
1-3A,4C	168821	REAR ANTI TIP ASSM 6	Right
1-3A,4C	169737	ANTITIP TUBE ASSEMBLY 6 LH	Left
1-3D,4C	168822	REAR ANTI TIP ASSM 7	Right
1-3D,4C	169738	ANTITIP TUBE ASSEMBLY 7 LH	Left
1-3A,4D	168823	REAR ANTI TIP ASSM 8	Right
1-3A,4D	169739	ANTITIP TUBE ASSEMBLY 8 LH	Left
1-3D,4D	168824	REAR ANTI TIP ASSM 9	Right
1-3D,4D	169740	ANTITIP TUBE ASSEMBLY 9 LH	Left
	168799	ACCESS ANTI-TIP RH	Right
	168800	ACCESS ANTI-TIP LH	Left
?	168798	ANTI-TIP FASTENERS 1 ACCESS	ONE SIDE
	TBD	ANTI-TIP FASTENERS 2 ACCESS	ONE SIDE
	TBD	ANTI-TIP FASTENERS 3 ACCESS	ONE SIDE

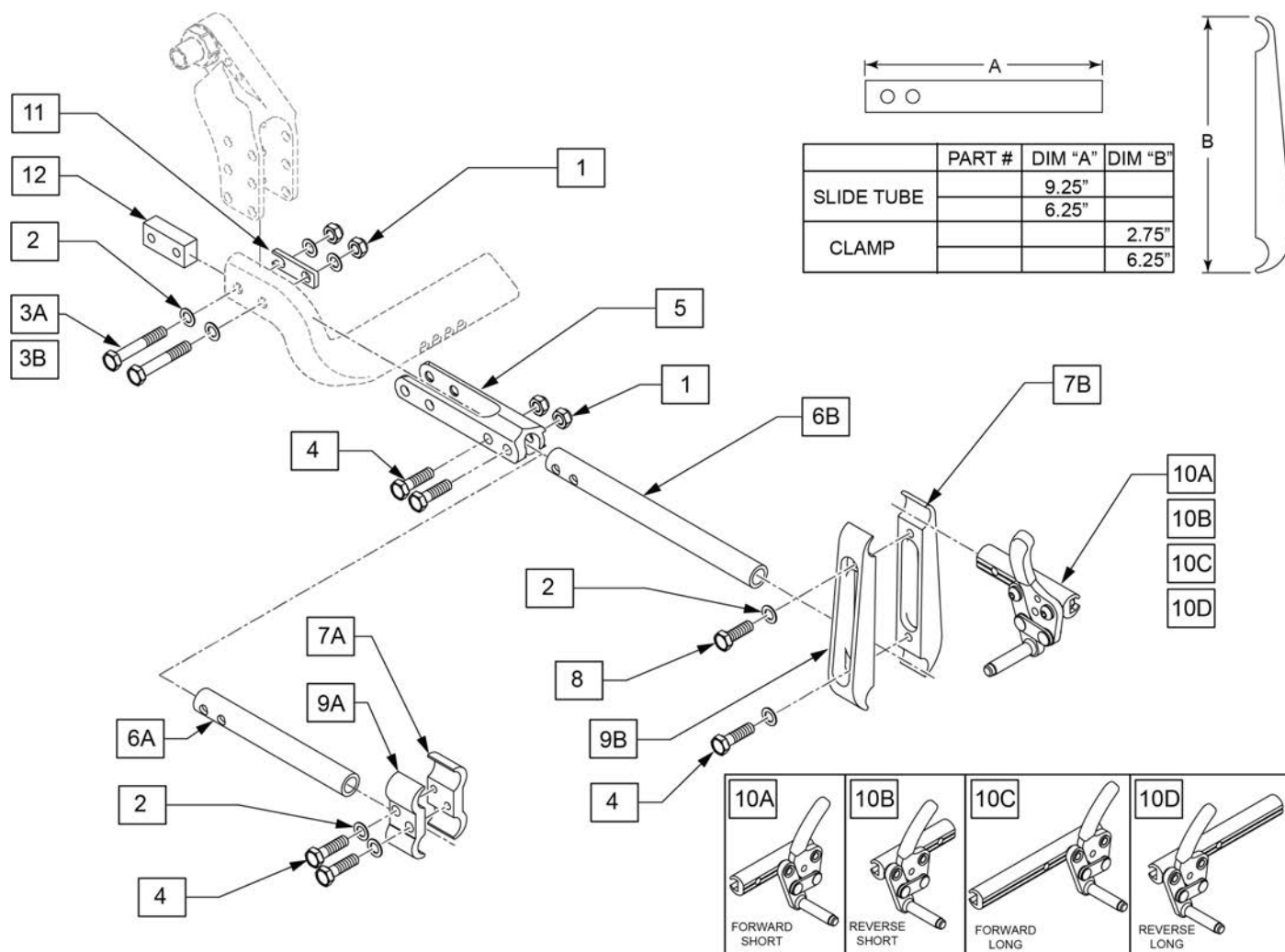


Pos.	Item Number	Description
5	251419	WHEEL LOCK MNT, DBL D
6A	256201	WHEEL LOCK SLIDE TUBE SHORT
6B	256202	WHEEL LOCK SLIDE TUNE LONG
10A	104216-001	WHEEL LOCK PUSH R FWD SHT
10A	104216-002	WHEEL LOCK PUSH L FWD SHT
10B	104227-001	WHEEL LOCK PULL R FWD SHT
10B	104227-002	WHEEL LOCK PULL L FWD SHT
10C	104216-003	WHEEL LOCK PUSH R REV SHT
10C	104216-004	WHEEL LOCK PUSH L REV SHT
10D	104227-003	WHEEL LOCK PULL R REV SHT
10D	104227-004	WHEEL LOCK PULL L REV SHT
1-3A,3B,4,8	168801	WL FASTENER KIT
2,7A,8,9A	168811	WL CLAMP KIT SHT
2,3B,4,7AB,8,9B	168802	WL CLAMP KIT
1-3A,3B-6A,7A,9A,11,12	168803	WL MNT SHT POST SHT CLAMP KIT
1-3A,3B-6A,7B,8,9B,11,12	168804	WL MNT SHT POST LNG CLAMP KIT
1-3A,3B-6B,7A,9A,11,12	168805	WL MNT LNG POST SHT CLAMP KIT
1-3A,3B-6B,7B,8,9B,11,12	168806	WL MNT LNG POST LNG CLAMP KIT


Note: used on High Mount Push and Pull Wheel Locks only.

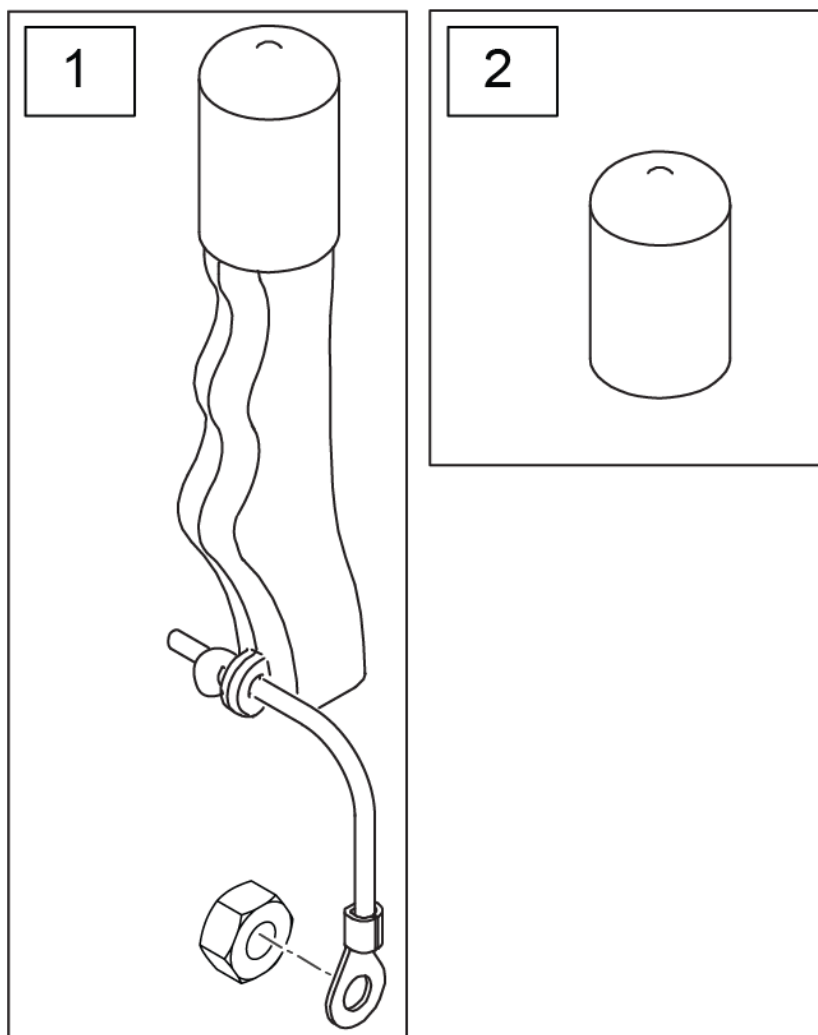


Pos.	Item Number	Description	Remarks
1	161930	WHEEL LOCK EXT ASSY (PAIR) KIT	Pair
2	560600	FOAM GRIP 1 1/2 X 0.705 X 0.1	

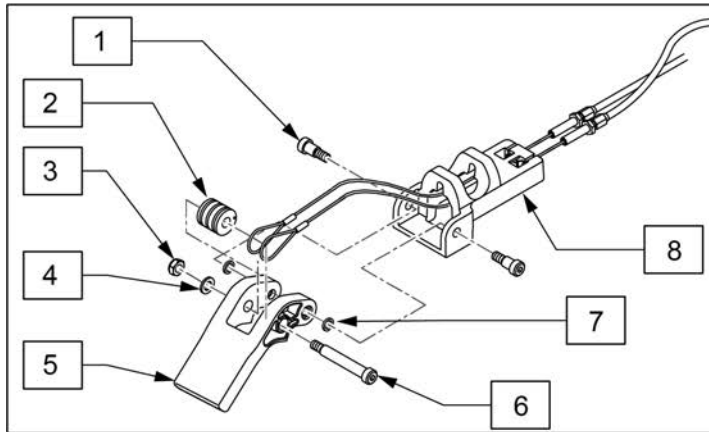

30 ATTENDANT WL FASTENER KIT			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
10		HHCS M8 X 1.25 X 45 NP	2
11		WASHER FLAT AN 5/16 ROHS	10
14		HHCS M8 X 1.25 X 35 NP	1
15		NUT HEX NYLOCK (M8)	3
28		HHCS M8 X 1.25 X 25 NP	1
29A		HHCS M8 X 1.25 X 50 BLK	1
29B		HHCS M8 X 1.25 X 60 FULL THRD	1

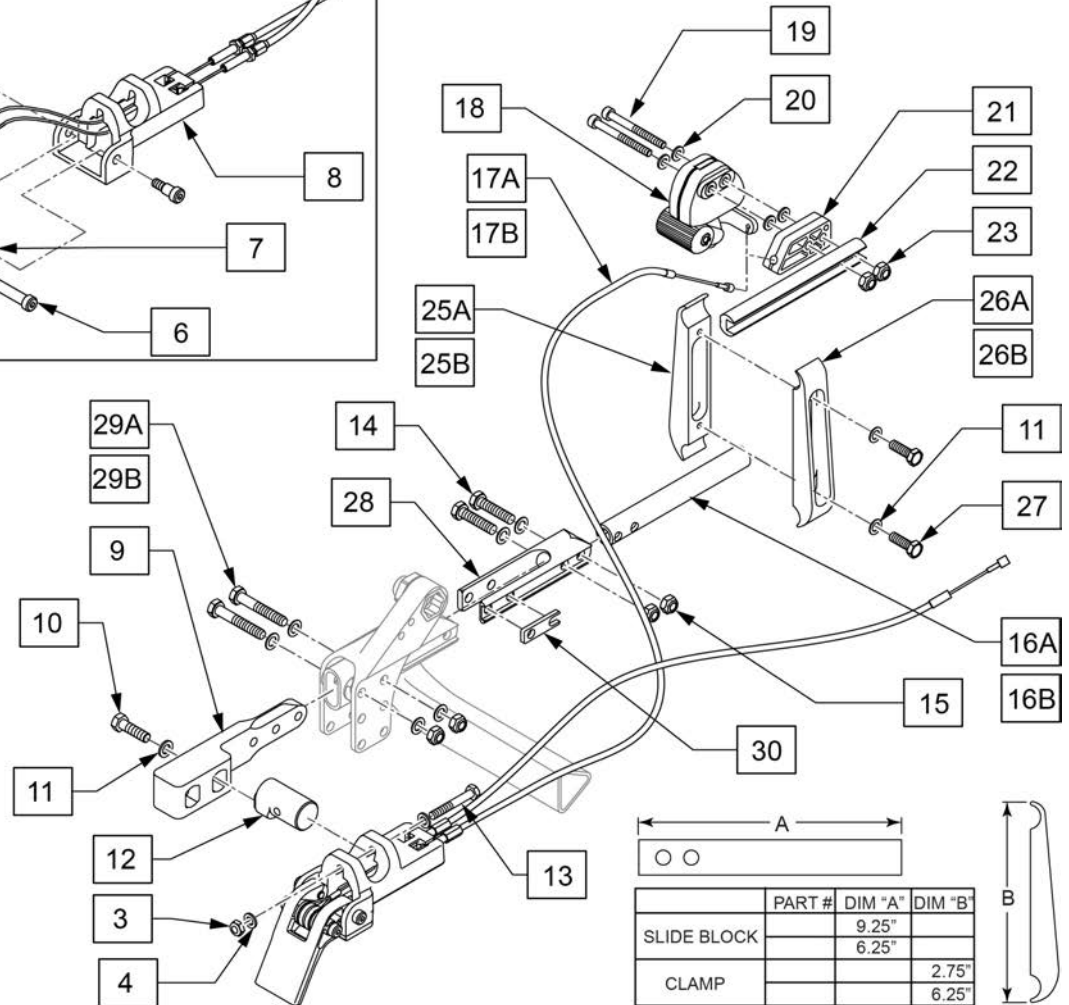


Pos.	Item Number	Description	Remarks
12	250923	MOUNT, ATTENDANT W.L.	
16A	256201	WHEEL LOCK SLIDE TUBE SHORT	
16B	256202	WHEEL LOCK SLIDE TUNE LONG	
17A	254718	TILT FOOT RELEASE CABLE 25"	
17B	254732	TILT FOOT RELEASE CABLE 15"	
18	119952	REMOTE LOCK RIGHT SIDE US	
18	119953	REMOTE LOCK LEFT SIDE US	
19	103190-013	SHCS M6 x 1 x 60 PART NP	
20	262133	WASHER LOCK M6	
21	506184	SPACER ATTENDANT WHEEL LOCK	
22	126230	SLIDE ADJ SHRT FOOT LOCK	
23	104381	NUT HEX NYLOCK M6 X 1.0	
1-8,13,17A,17B	168836	FOOTPEDAL W/CABLES	
1-16A,17-30	168838	ATTENDANT COMPLETE RETRO STD	
1-16B,17-30	168839	ATTENDANT COMPLETE RETRO SHT	
30	168837	ATTENDANT WL FASTENER KIT	kit for 1 side
1-8,13,17A,17B	168840	ATTNT PEDAL REPLACEMENT ASSM	
11,25A,26A,27	168811	WL CLAMP KIT SHT	
11,25B,26B,27	168802	WL CLAMP KIT	

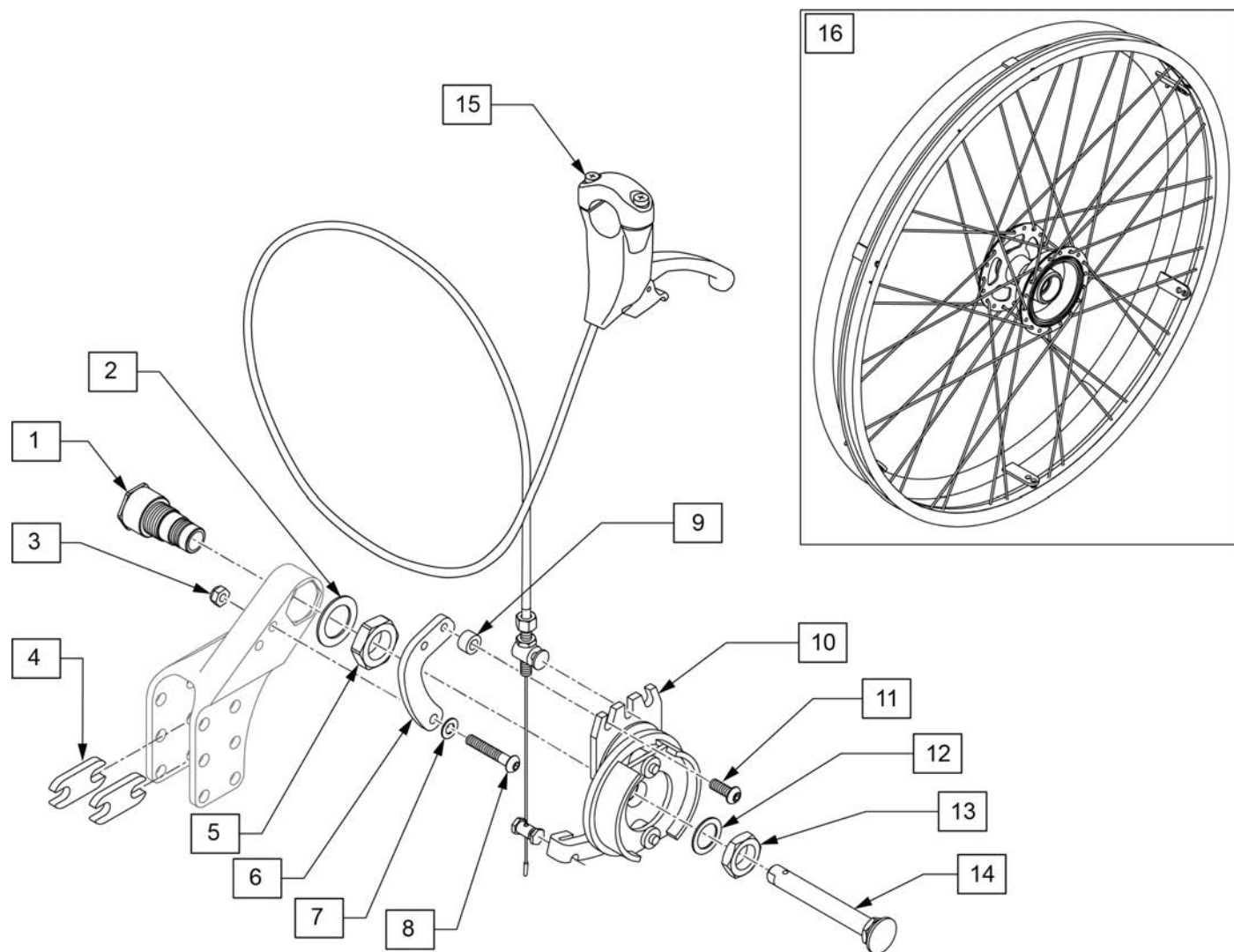

Pos.	Item Number	Description	Remarks
14	251438	QR AXLE - DRUM - 110MM	
15	000228007	PULL CABLE 750/600 INS COGALV	for Std PH w/ 17"-19" Ht range
15	000228032	PULL CABLE 750/600 INS COGALV	for Std PH w/ 12"-17" Ht range
10-13	168841	DRUM MECH RH	
10-13	168842	DRUM MECH LH	
3,4,7-9,11	168843	DRUM BRAKE FASTENER KIT	
1,2,5,12,13	168844	DRUM BRAKE SLEEVE KIT	
1-16	168845	DRUM COMPLETE RETRO KIT	
16	000851369	A-WHEEL -TRBR 24" GALV	Drum Brake Wheel

Note: Must order wheel & axle separately.

Note: If retrofitting the wheel assembly (2 each), axle plate hardware (168792 x 2 each) and axles (249231 x 2 each) MUST be ordered separately.

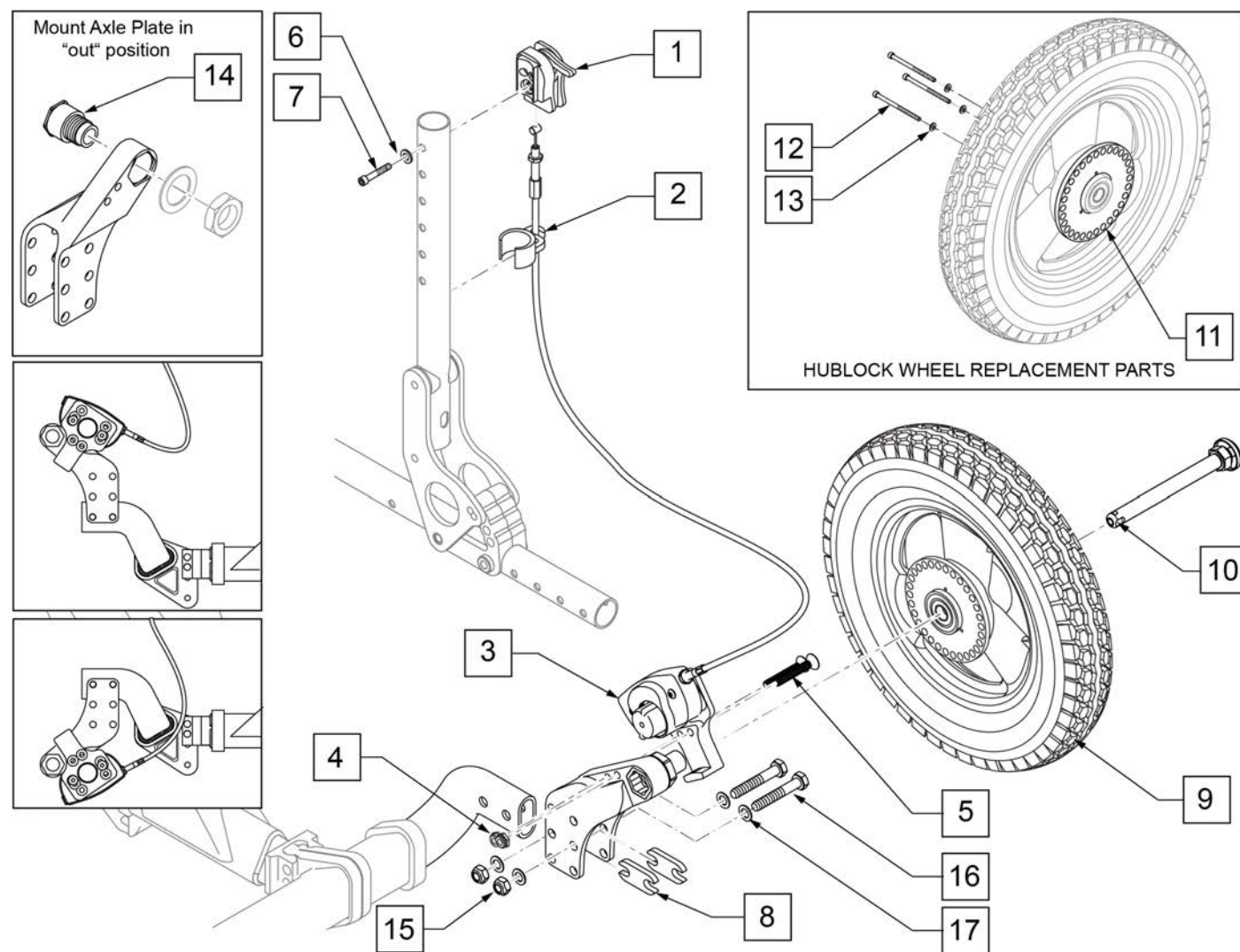


Pos.	Item Number	Description	Remarks
1	112426	ASSY LEVER HUB LOCK R	
1	112427	ASSY LEVER HUB LOCK L	
2	108134	CABLE CLIP 1" PACK (10)	PACK OF 10
6	106943	WASHER FLAT AN 1/4 ROHS	
7	106976	SHCS 1/4 -28 x 1 1/2 ROHS	
9	112226-001	WHEEL HUB LOCK 12" PNEU	
9	112226-002	WHEEL HUB LOCK 12" AIRLESS	
9	112226-003	WHEEL HUB LOCK 12" LOW POLY	
9	112227-001	WHEEL HUB LOCK 16" PNEU	
9	112227-002	WHEEL HUB LOCK 16" AIRLESS	
9	112226-004	WHEEL HUB LOCK 12IN PNEU BLK	Black Tire

Pos.	Item Number	Description	Remarks
9	112226-005	WHEEL HUB LOCK 12IN INSRT BLK	Black Tire
10	251438	QR AXLE	
11	111845	INDEX WHEEL 12IN	
12	112231	SHCS6-32 X 2-1/4 ROHS	
13	112230	WASHER FLAT #6	
1-7	168846	HUBLOCK RETRO DN PD3XL KIT RH	All back types w/ axle plate down
1-7	168847	HUBLOCK RETRO DN PD3XL KIT LH	All back types w/ axle plate down
1-7	168807	HUBLOCK RETRO UP PD3XL KIT RH	fixed, Ht Adj Tall, Dynamic w/ axle plate up
1-7	168808	HUBLOCK RETRO UP PD3XLKIT LH	fixed, Ht Adj Tall, Dynamic w/ axle plate up
1-7	168809	HUBLOCK RETRO PD2XL KIT RH	Recline Back,Ht Adj Std w/ axle plate up
1-7	168810	HUBLOCK RETRO PD2XLKIT LH	Recline Back,Ht Adj Std w/ axle plate up
4,5,8	168848	ACCESS HUBLOCK FASTENER KIT	
14	249231	AXLE SLEEVE - HUB LOCK	
15-17	168792	AXLE PLATE FASTENER KIT	one side


Note: Black Tires are not sold separately; please order the appropriate assembly part number.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

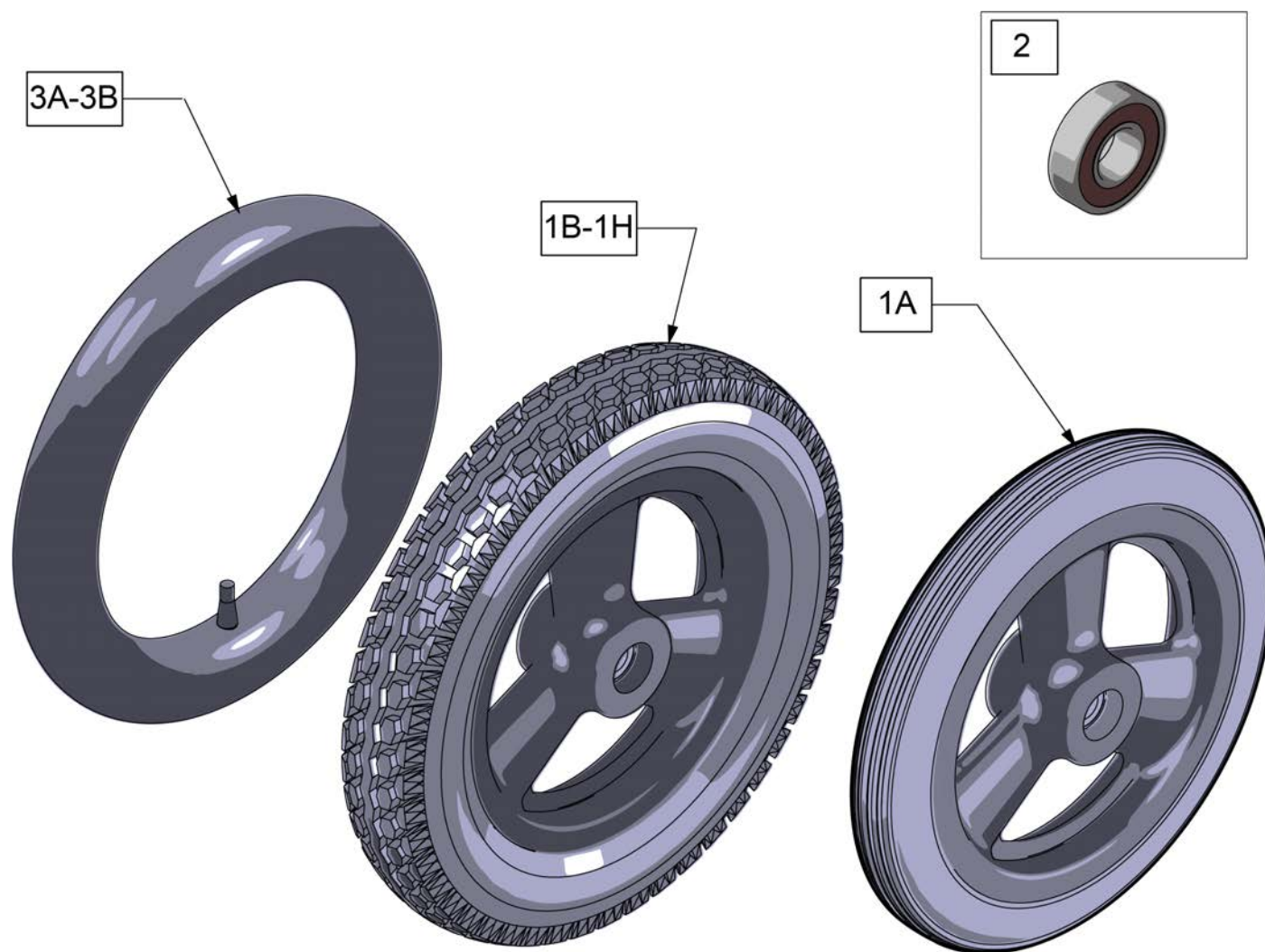

Pos.	Item Number	Description	Remarks
1A	101466	12 IN MAG POLY TIRE 3 SPOKE	Low Poly
1B	101661	12" MAG PNEU 3 SPOKE	Grey Tire; for Quickie & Zippie Iris see Black Wheel Assm p/n's below
1C	101662	12" MAG PNEU INSRT 3 SPK	Grey Tire; for Quickie & Zippie Iris see Black Wheel Assm p/n's below
1D	384706	TIRE PNEU 12-1/2" X 2-1/4"	Grey Tire only
1E	243593	12IN MAG PNEU 3 SPOKE BLK	Black tire, only available on Quickie and Zippie IRIS at this time
1F	243594	12IN MAG PNEU INSRT 3 SPK BLK	Black tire, only available on Quickie and Zippie IRIS at this time
1G	112226-004	WHEEL HUB LOCK 12IN PNEU BLK	Black tire for Hub lock, only available on Quickie and Zippie IRIS at this time
1H	112226-005	WHEEL HUB LOCK 12IN INSRT BLK	Black tire for Hub lock, only available on Quickie and Zippie IRIS at this time
2	349500	1 1/8 X 1/2 BEARING	
3A	384736	INNER TUBE 12-1/2" X 2-1/4"	
3B	102972-001	12 1/2 IN AIRLESS INSERT	

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

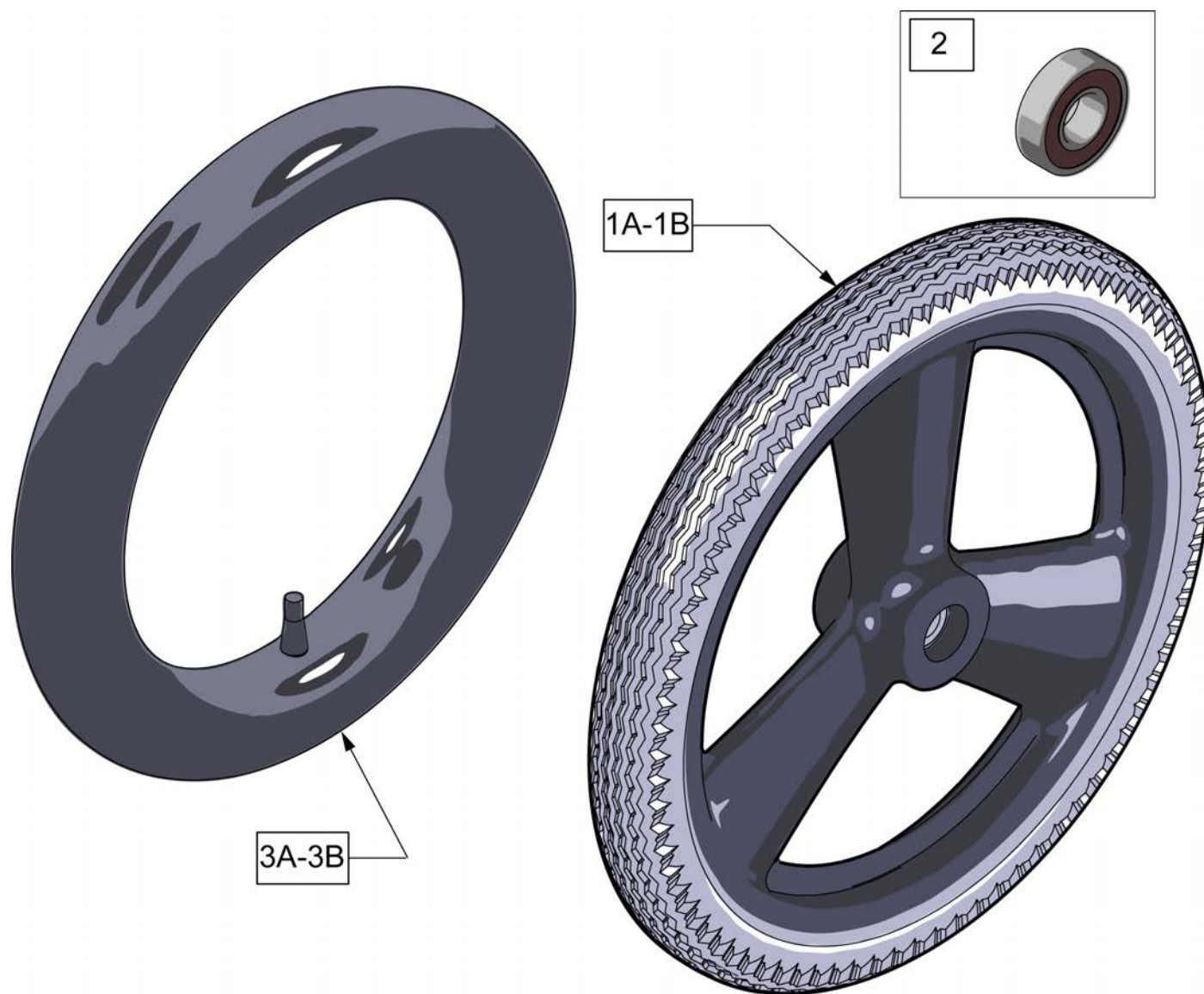


Pos.	Item Number	Description	Remarks
1A	101667	16 IN MAG PNEU 3 SPOKE	
1B	101668	16 IN MAG PNEU W/INSERT	
1C	385237	16" X 1.75" PNEU TIRE	Tire Only
2	349500	1 1/8 X 1/2 BEARING	
3A	384738	16" X 1.75" INNER TUBE	
3B	102973-001	16IN AIRLESS INSERT	

Note: 5 Spoke Mag is not compatible with One Arm Drive or Projection Handrims.

Note: 5 Spoke Mag utilizes exclusive screw mounted handrims.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

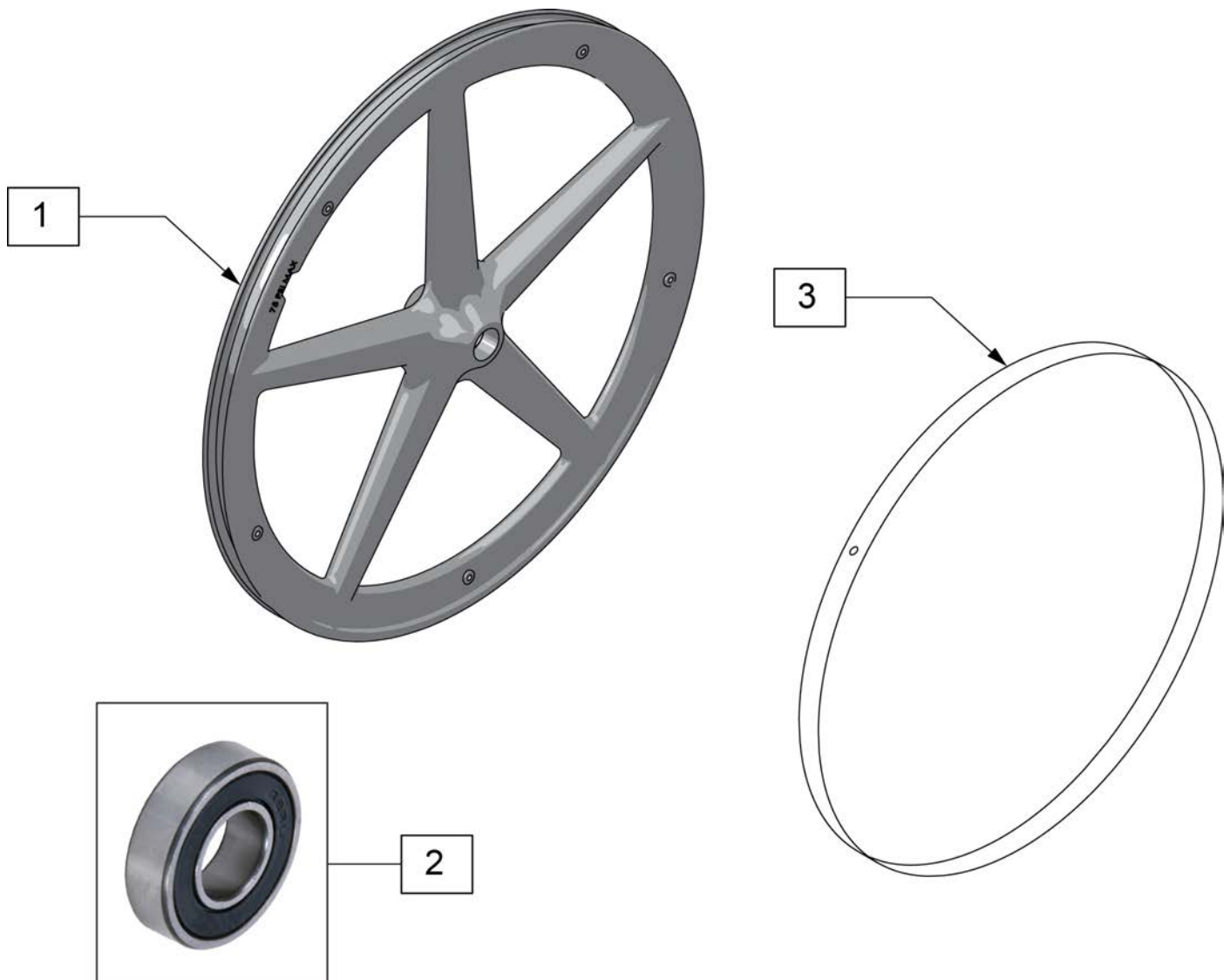

Pos.	Item Number	Description	Remarks
1,2	128985	20" 5 SPOKE MAG WHEEL	20" (451mm)
1,2	128986	22" 5 SPOKE MAG WHEEL	22" (501mm)
1,2	126405	24" MAG W/BRNG 5 SPOKE	24" (540mm)
2	349500	1 1/8 X 1/2 BEARING	
3	129325	20" MAG RIM STRIP	
3	129324	22" MAG RIM STRIP	
3	127912	24" MAG RIM STRIP	

Note: this wheel utilizes Screw Mount Handrims - the handrim has a threaded hole that mounts to the wheel through a spacer and fastener.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.




Pos.	Item Number	Description	Remarks
1A	401001	WHEEL ASSM PUSH BUTTON 18"	18"
1B	401070	20" RIM ASSM PUSH BUTTON	20"
1C	401072	22" RIM ASSM PUSH BUTTON	22"(501mm)
1D	401074	24" RIM ASSM PUSH BUTTON	24"(540mm)
1E	401077	26IN RIM ASSM PUSH BUTTON	26"(590mm)
2	349500	1 1/8 X 1/2 BEARING	
3A	362247	SPOKE 154MM X 2.0MM	
3B	362250	QP SPOKE 201MM X 2.0MM	
3C	362275	QP SPOKE 226MM X 2.0MM	
3D	362300	QP SPOKE 244MM X 2.0MM	
3E	362325	QP SPOKE 270MM X 2.0MM	




Pos.	Item Number	Description	Remarks
1	385237	16" X 1.75" PNEU TIRE	
1	385250	20" PNEUMATIC TIRE	
1	385275	22" PNEUMATIC TIRE	22" x 1 3/8 (501mm)
1	385300	24" PNEUMATIC TIRE	24" x 1 3/8 (540mm)
1	385325	26" PNEUMATIC TIRE	26" x 1 3/8 (590mm)
1	384614	24" x 2.125 PNEUMATIC TIRE	24"x2 1/8 (540mm) used w/ M6






Pos.	Item Number	Description	Remarks
1	386232	20" POLY TIRE STD	20"
1	386242	22" POLY TIRE STD	22" (501mm)
1	386252	24" POLY TIRE STD	24" (540mm)

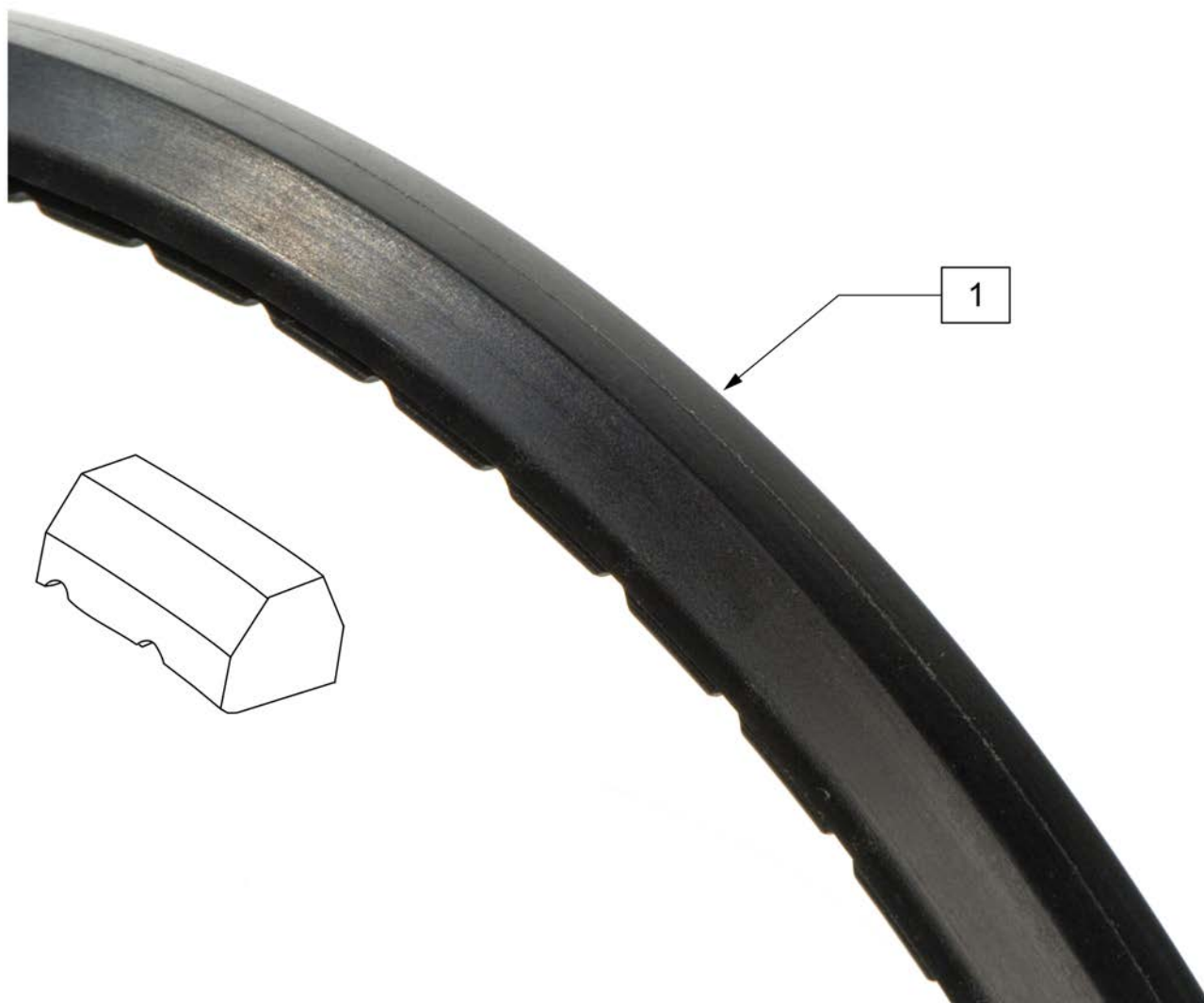


Pos.	Item Number	Description	Remarks
1	386262	20" LP POLY TIRE BR	20" =16" dia
1	386263	22" LP POLY TIRE BR	22"(501mm) = 18"dia
1	386254	24" POLY LOW PROFILE	24"(540mm)= 23" dia

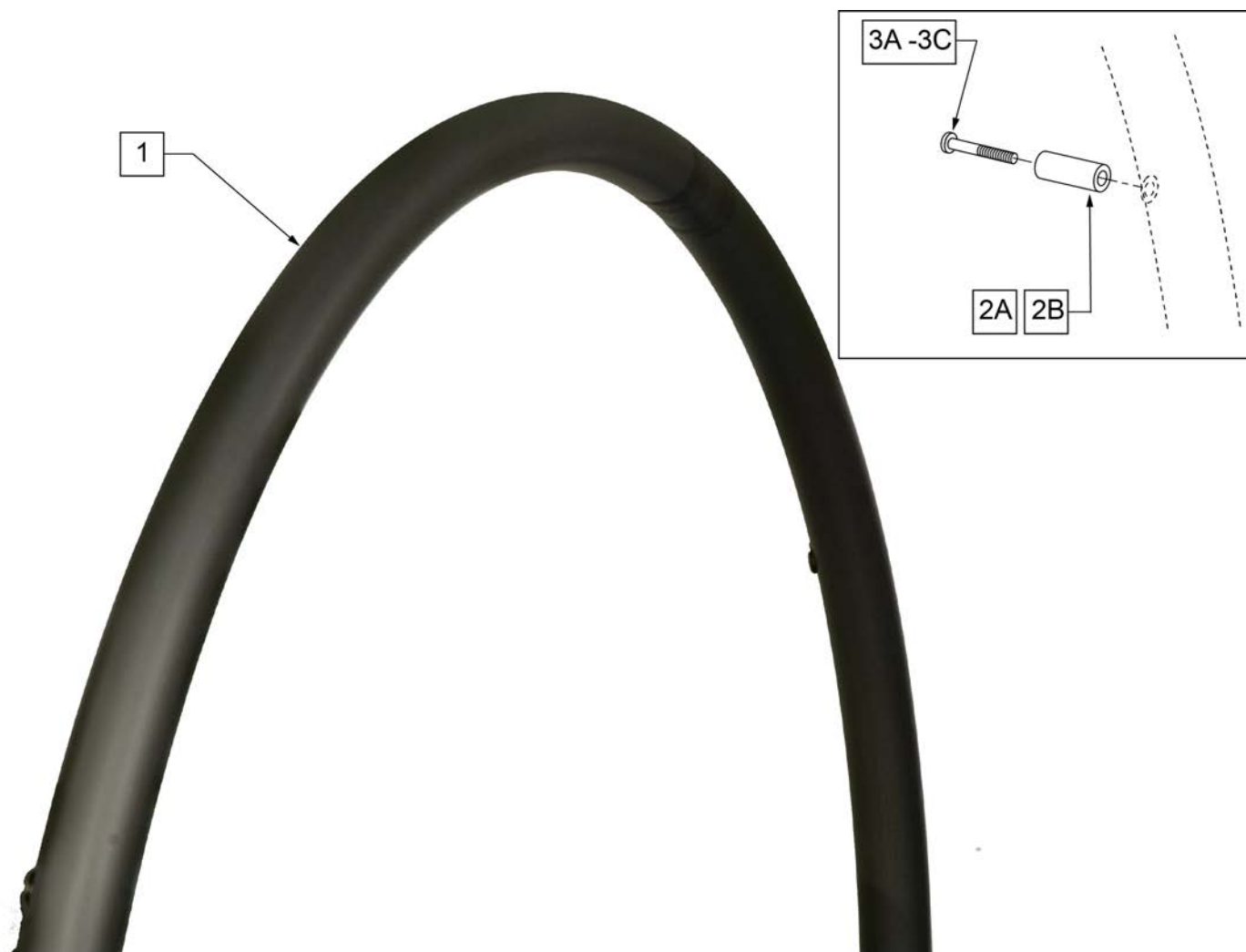


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>	LITE SPOKE	>>>>>
1,2A,3A	159508	20IN LT SPK AA HNDRLM W/HDWR	20" LITE SPOKE
1,2A,3A	159509	22IN LT SPK AA HNDRLM W/HDWR	22" (501) LITE SPOKE
1,2A,3A	159510	24IN LT SPK AA HNDRLM W/HDWR	24" (540) LITE SPOKE
1,2A,3A	159511	25IN LT SPK AA HNDRLM W/HDWR	25" (559) LITE SPOKE
1,2A,3A	159512	26IN LT SPK AA HNDRLM W/HDWR	26" (590) LITE SPOKE
2A,3A	159513	LT SPK AL AN HNDRLM HDWR	LITE SPOKE, SET OF 6
>>>>>	>>>>>>>>	SPOKE	>>>>>
1,2B,3B	425931	20" HR 6SMT W/HDWR	20" SPOKE
1,2B,3B	425941	22" HR 6SMT W/HDWR	22" (501) SPOKE
1,2B,3B	426000	24" HR 6SMT W/ HDWR	24" (540) SPOKE
2B,3B	159514	SPK AL AN HNDRLM HDWR	SPOKE, SET OF 6
>>>>>	>>>>>>>>	MAG	>>>>>
1,3C	425934	20" HR 6SMT 6MAG W/HDWR	20" MAG
1,3C	425951	22" HR 6SMT 6MAG W/HDWR	22" (501) MAG
1,3C	426009	24" HR 6SMT 6MAG W/HDWR	24" (540) MAG
3C	281175	10-32 X 2 1/8 PAN PHIL SS	MAG
>>>>>	>>>>>>>>	M6 SPOKE	>>>>>
1,2B,3B	425941	22" HR 6SMT W/HDWR	24" (540) SPOKE

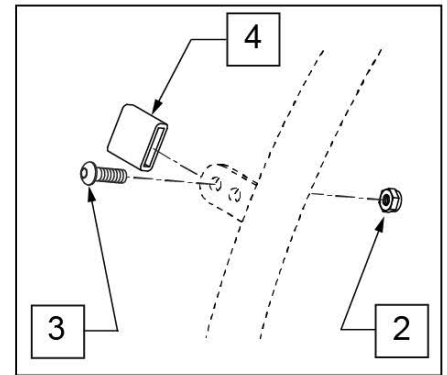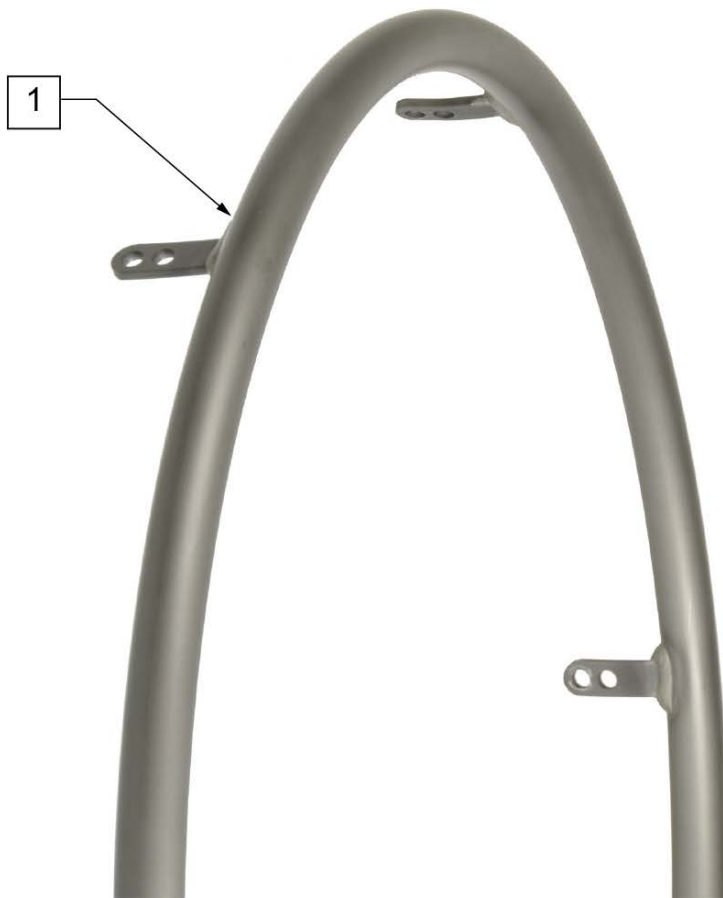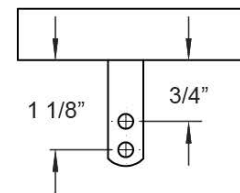**MEASURING GUIDE**

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	CROSS SPOKE & SPOKE	>>>>>
1 -4	425095	HANDRIM 18" 4T W/HDWR ANO	18" SPOKE
1	529529	QP 24" HR .065 (6) TAB ANO	24" (540) CROSS SPOKE -SHORT & REG TAB
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (590) CROSS SPOKE
>>>>>	>>>>>>>>>>	CROSS SPOKE QP (QUICKIE PERFORMANCE)	>>>>>
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) CROSS SPOKE TENNIS & BBALL
1 -4	425045	24" HR SHT(6T)ANO W/X/S HDWR	25" (590) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	425048	26" HR (6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL -REG TAB
1 -4	425055	26" HR SHT(6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	426015	QP 24" HR .049 W/HDWR SMT	24" (540) TENNIS
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (559) TENNIS & BASKETBALL
1 -4	426050	QP 26" HR .065 W/HDWR (6T)	26" (700) TENNIS & BASKETBALL
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) TENNIS & BASKETBALL
>>>>>	>>>>>>>>>>	SPINERGY & PROTON	>>>>>
1	529529	QP 24" HR .065 (6) TAB ANO	24" (540) SPINERGY
1 -4	425038	QP 24" HR .049 W/HDWR (6T)	25" (559) SPINERGY
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) SPINERGY
>>>>>	>>>>>>>>>>	MOUNTAIN	>>>>>
1 -4	425020	22" HR 4T W/HDWR	24" (540) MOUNTAIN
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	26" (590) MOUNTAIN
2-4	159515	AA TAB MOUNT HANDRIM HARDWARE	SET OF 6

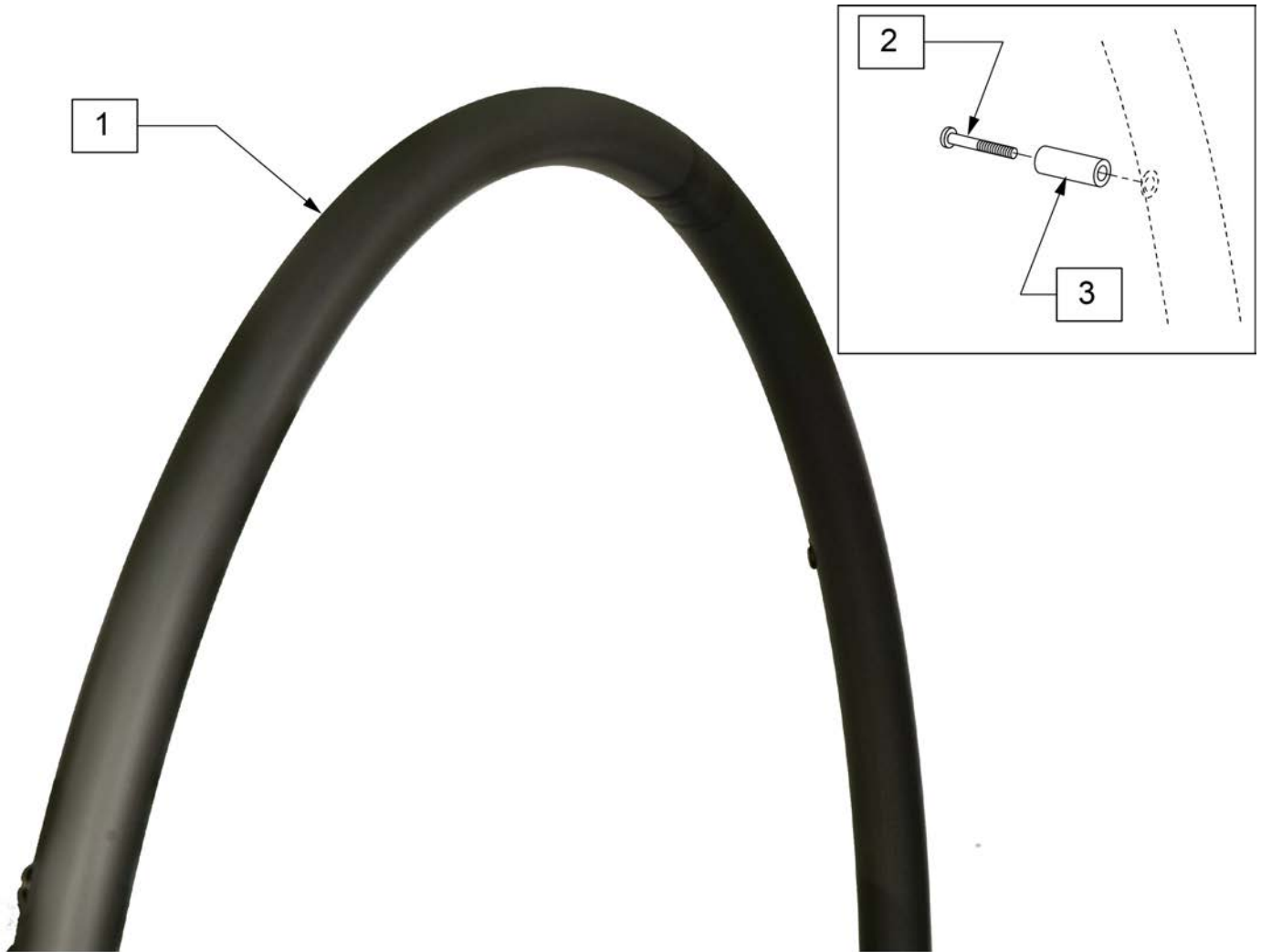


Pos.	Item Number	Description	Remarks
1-3	136158	20" ALU ANO HND RIM KIT	
1-3	136157	22" ALU ANO HND RIM KIT	
1-3	136156	24" ALU ANO HND RIM KIT	
2,3	159507	5 SPK HNDRM HDWR	set of 5

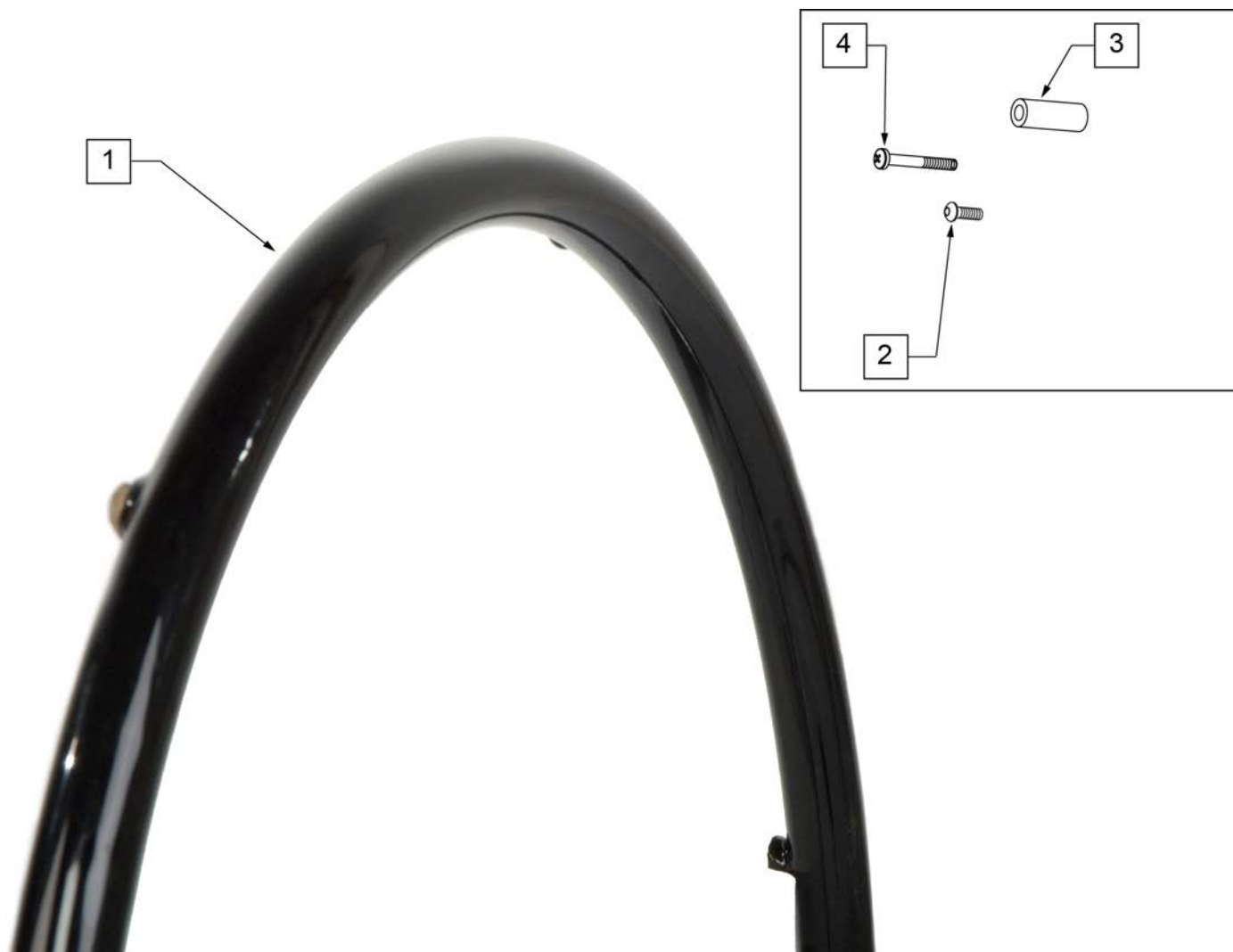

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	SPOKE & LITE SPOKE	>>>>>
1-3	425933	20" HR 6SMT PLST W/HDWR	20" SPOKE
1-3	425943	22" HR 6SMT PLST W/HDWR	22" (501) SPOKE
1-3	426004	24" HR 6SMT PLST W/HDWR	24" (540) SPOKE
>>>>>	>>>>>>>>>>	MAG	>>>>>
1,4	425935	20" HR 6SMT 6MAG PLST W/HDWR	20" MAG
1,4	425953	22" HR 6SMT 6MAG PLST W/HDWR	22" (501) MAG
1,4	426013	24" HR 6SMT 6MAG PLST W/HDWR	24" (540) MAG
>>>>>	>>>>>>>>>>	M6 SPOKE	>>>>>
1-3	425943	22" HR 6SMT 6 MAG PLST W/ HDWR	24" (540) M6 SPOKE
2,3	159534	PLST COATED HNDRM HARDWARE	SPOKE; SET OF 6
4	281175	10 -32 x 2 1/8 PAN PHIL SS	6 SPOKE MAG




Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	MOUNTAIN	>>>>>
1,2	425022	22" HR 4T PLST W/ HDWR	24" (540)
1,2	425032	24"HR 6T PLST BB/NTBB/TN W/HDWR	26" (590)
>>>>>	>>>>>>>>>	SPINERGY & PROTON	>>>>>
1,2	425039	24"HR 6T BLK PLST W/HDWR	24" (540)
1,2	425032	24"HR 6T PLST BB/NTBB/TN W/HDWR	25" (559)
1,2	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
>>>>>	>>>>>>>>>	QP (QUICKIE PERFORMANCE)	>>>>>
1,2	425032	24"HR 6T PLST BB/NTBB/TN W/HDWR	25" (559)
1,2	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
1,2	426060	QP 26" HR W/ HDWR (6T) PLASTIC	26" (700)
2	159535	PLST COATED TAB HNDRM HDWR	

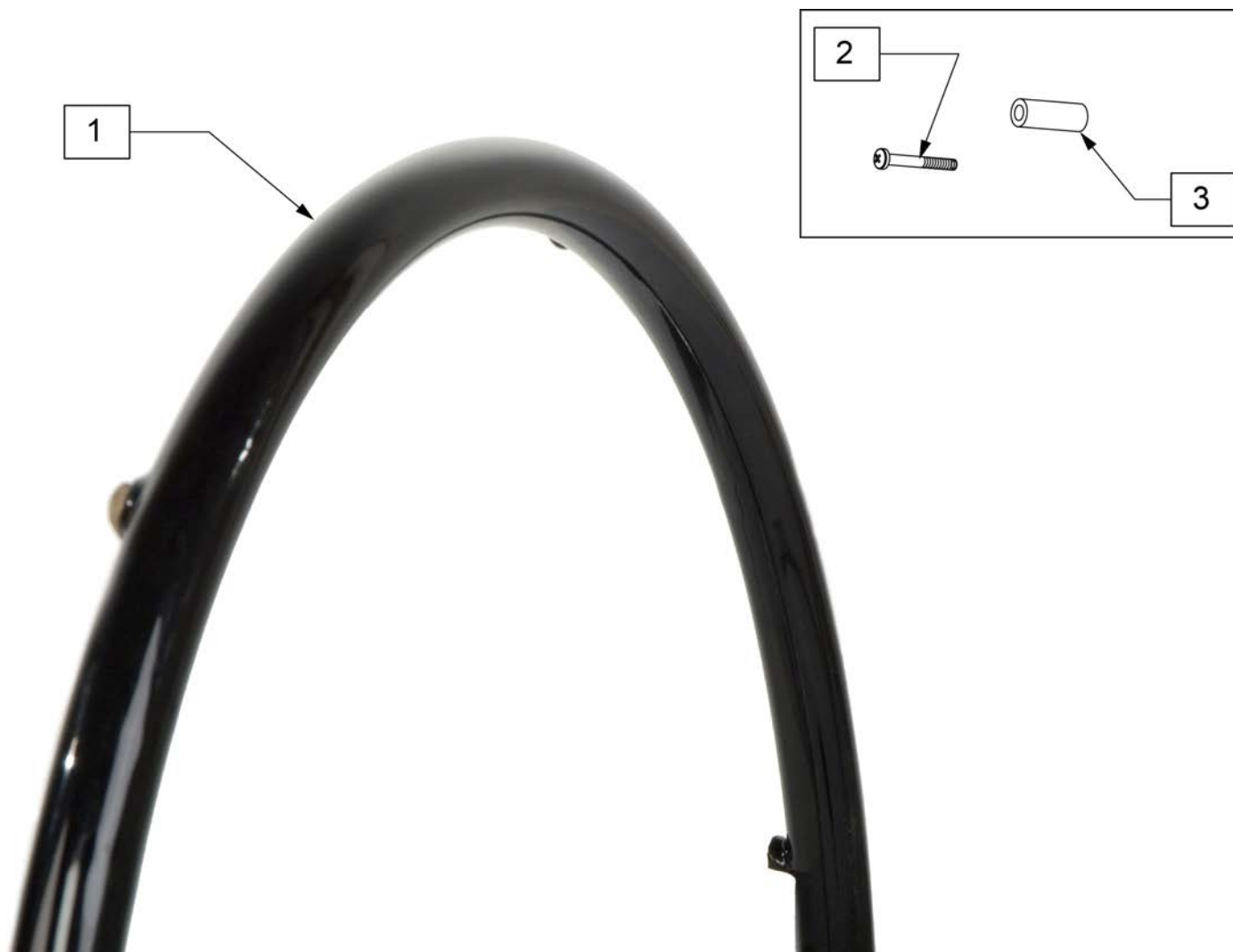

Pos.	Item Number	Description
2,3	159507	5 SPK HNDRM HDWR
1-3	136155	20" PLSTC COATED HND RIM KIT
1-3	136154	22" PLSTC COATED HND RIM KIT
1-3	136153	24" PLSTC COATED HND RIM KIT

Note: Screw Mount Handrims are utilized with Lite Spoke, Spoke and 6 Spoke Mag Wheels.

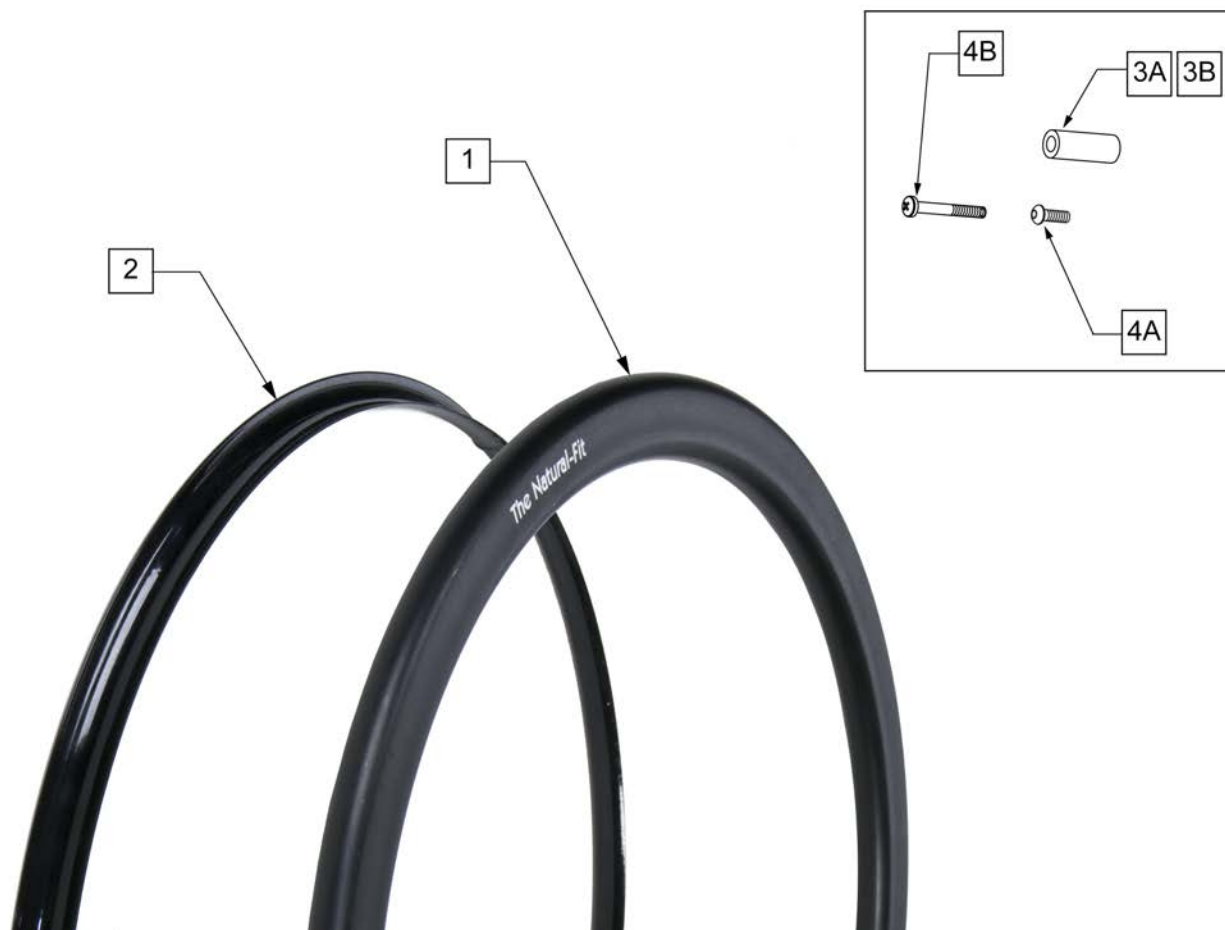

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	STD GRIP THUMB GUARD	>>>>>
2	102168-001	GUARD THUMB NAT FIT 22 PC BLK	22" (501)- Std Grip Thumb Guard
2	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)- Std Grip Thumb Guard
2	102168-005	GUARD THUMB NAT FIT 25 PC BLK	25" (559)- Std Grip Thumb Guard
2	102168-007	GUARD THUMB NAT FIT 26 PC BLK	26" (590)- Std Grip Thumb Guard
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
2	102168-002	GUARD THUMB NAT FIT 22 SG BLK	22" (501)- Super Grip Thumb Guard
2	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)- Super Grip Thumb Guard
2	102168-006	GUARD THUMB NAT FIT 25 SG BLK	25" (559)- Super Grip Thumb Guard
2	102168-008	THUMB GUARD NAT FIT 26 SG BLK	26" (590)- Super Grip Thumb Guard
>>>>>	>>>>>>>>>	NATURAL FIT LITE HANDRIM	>>>>>
1,3A,4A	159516	22 NAT-FIT LT HNDRIM W/HDWR	22" (501)
1,3A,4A	159517	24 NAT-FIT LT HNDRIM W/HDWR	24" (540)
1,3A,4A	159518	25 NAT-FIT LT HNDRIM W/HDWR	25" (559)
1,3A,4A	159519	26 NAT-FIT LT HNDRIM W/HDWR	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT HANDRIM	>>>>>
1,3A,4A	159520	22 NAT-FIT HNDRIM W/HDWR	22" (501)
1,3A,4A	159521	24 NAT-FIT HNDRIM W/HDWR	24" (540)

Pos.	Item Number	Description	Remarks
1,3A,4A	159522	25 NAT-FIT HNDRIM W/HDWR	25" (559)
1,3A,4A	159523	26 NAT-FIT HNDRIM W/HDWR	26" (590)
3A,4A	159524	NAT-FIT HNDRIM LT HDWR SET 6	USED W/ LITE SPOKE & SPOKE; SET OF 6
3B	101932	SPCR .194ID X .375 OD X .750	USED W/ MAG
4B	281175	10-32 X 2 1/8 PAN PHIL SS	USED W/ MAG

Note: Tab Mount Hand Rims are utilized with Spinergy & Toplino Wheels.



Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	STD GRIP THUMB GRIP	>>>>>
1	102168-001	GUARD THUMB NAT FIT 22 PC BLK	22" (501)
1	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)
1	102168-005	GUARD THUMB NAT FIT 25 PC BLK	25" (559)
1	102168-007	GUARD THUMB NAT FIT 26 PC BLK	26" (590)
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
1	102168-002	GUARD THUMB NAT FIT 22 SG BLK	22" (501)
1	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)
1	102168-006	GUARD THUMB NAT FIT 25 SG BLK	25" (559)
1	102168-008	THUMB GUARD NAT FIT 26 SG BLK	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT LITE HANDRIM	>>>>>
2-4	159525	22 NAT-FIT LT TB HNDRIM W/HDWR	22" (501)
2-4	159526	24 NAT-FIT LT TB HNDRIM W/HDWR	24" (540)
2-4	159527	25 NAT-FIT LT TB HNDRIM W/HDWR	25" (559)

Pos.	Item Number	Description	Remarks
2-4	159528	26 NAT-FIT LT TB HNDRLM W/HDWR	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT -W/O THUMB GRIP	>>>>>
2-4	159529	22 NAT-FIT TB HNDRLM W/HDWR	22" (501)
2-4	159530	24 NAT-FIT TB HNDRLM W/HDWR	24" (540)
2-4	159531	25 NAT-FIT TB HNDRLM W/HDWR	25" (559)
2-4	159532	26 NAT-FIT TB HNDRLM W/HDWR	26" (590)
3,4	159533	NAT-FIT TAB MNT HARDWARE	SET OF 6


NOTE: fits 24" 5 Spoke Wheel only.



Pos.	Item Number	Description	Remarks
1A	126558	HNDRM NAT FIT LT OVAL 5 RVT	24" Light
1B	126559	HNDRM NATURAL FIT OVAL 5 RVT	24" Standard
2A	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)- Super Grip Thumb Guard
2B	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)- Standard Grip Thumb Guard
3	101932	SPCR .194ID X .375 OD X .750	
4	281175	10-32 X 2 1/8 PAN PHIL SS	




Pos.	Item Number	Description	Remarks
1A	Discontinued	Presta Valves Discontinued	
1A	384736	INNER TUBE 12-1/2" X 2-1/4"	
1A	384738	16" X 1.75" INNER TUBE	
1A	382255	TUBE 18" TIRE	18" X 1-3/4" TUBE
1A	382275	20" TUBE	20" 1-3/8" TUBE
1A	382300	22" TUBE	22" X 1-3/8" TUBE
1A	382336	TUBE PRIMO 24" 25-540 SV	24" X 1" TUBE
1A	382325	24 X 1.3/8IN TUBE REG	
1A	382334	TUBE INLINE 24 X 1.75	
1A	384551	26" X 1" TUBE SCHRADER	
1A	382350	26" TUBE	26" X 1-3/8" TUBE
1A	382361	TUBE 26 X 1.5/1.95/ S/V	
1A	382365	26 X 1.25/1.5 TUBE SHRADER	
1A	382375	24" x 2.125 TUBE	
1B	102972-001	12 1/2 IN AIRLESS INSERT	12-1/2" X 2-1/4"
1B	102973-001	16IN AIRLESS INSERT	16" X 1-3/4"
1B	102973-002	18IN AIRLESS INSERT	18" X 1-3/4"
1B	102973-003	20IN AIRLESS INSERT	20" X 1-3/8"
1B	102972-002	AIRLESS INSERT 20 X 2-1/8 P3	
1B	102973-004	22IN AIRLESS INSERT	22" X 1-3/8"
1B	102973-005	24IN AIRLESS INSERT	24" X 1-3/8"
1B	102972-003	AIRLESS INSERT 24 X 2-1/8	
1B	108259	26" AIRLESS INSERT	26" X 1-3/8"

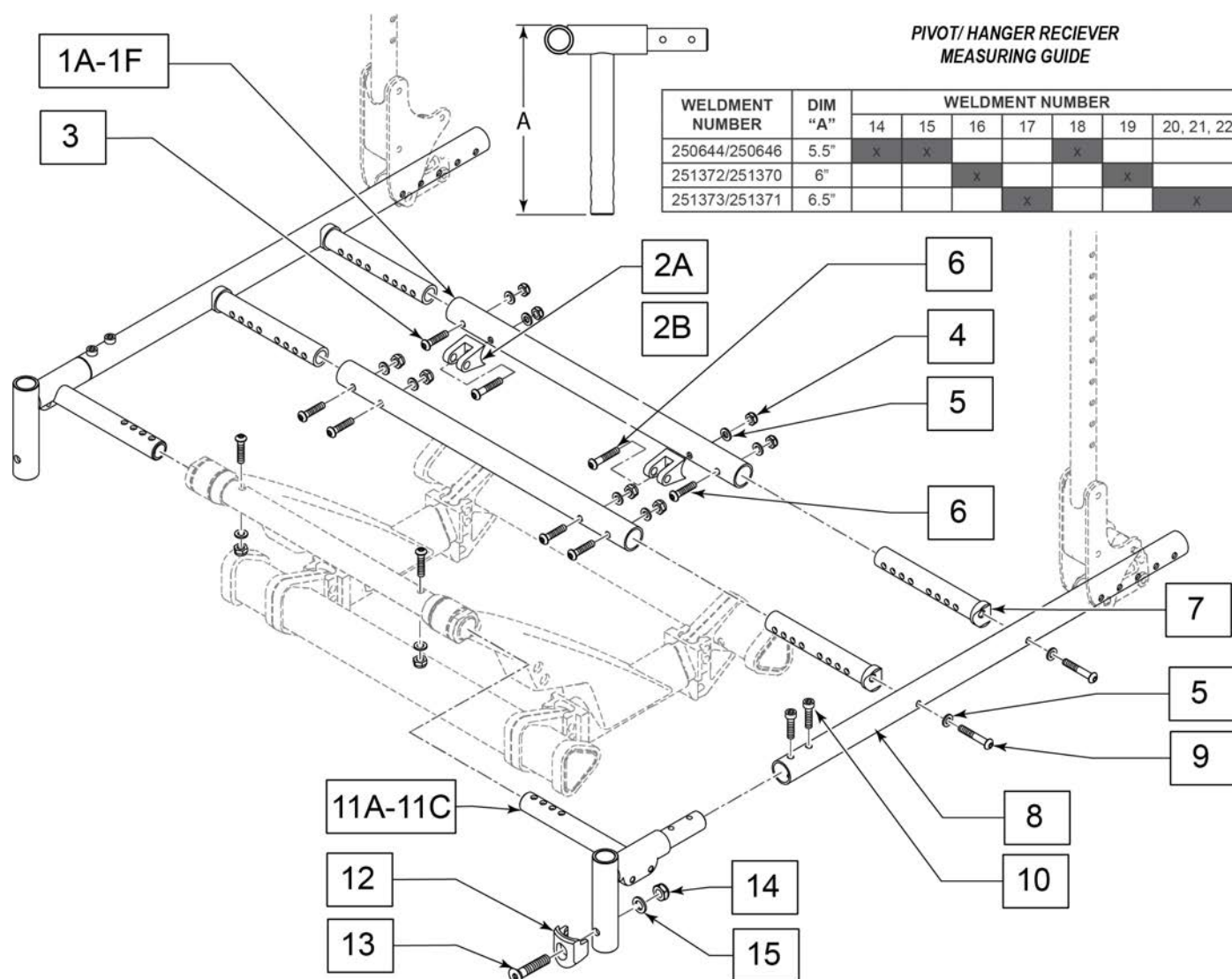

Pos.	Item Number	Description	Remarks
2A	253448	STRUT MOUNT -5 TO 20	-5-20 Deg
2B	252535	BOLT ON STRUT MOUNT	0-25 Deg
8	251374S	SEAT FRAME TUBE SHORT	
8	251375S	SEAT FRAME RAIL MEDIUM	
8	253498S	SEAT FRAME RAIL LONG, STEEL	
8	253499S	SEAT FRAME RAIL XLONG, STEEL	
13-16	119514	LATCH RECEIVER KIT	
1A,3-5	168849	CROSS TUBE KIT 14	
1B,3-5	168850	CROSS TUBE KIT 15	
1C,3-5	168851	CROSS TUBE KIT 16	
1D,3-5	168852	CROSS TUBE KIT 17	
1E,3-5	168853	CROSS TUBE KIT 18	
1F,3-5	168854	CROSS TUBE KIT 19	
1G,3-5	168855	CROSS TUBE KIT 20-22	
5,7,9	168856	SEAT TUBE SADDLE KIT	1 of Each
10,11,12A,13-15	168857S	PIVOT HANGER KIT RIGHT A	Right
10,11,12A,13-15	168858S	PIVOT HANGER KIT LEFT A	Left
10,11,12B,13-15	168859S	PIVOT HANGER KIT RIGHT B	Right

Pos.	Item Number	Description	Remarks
10,11,12B,13-15	168860S	PIVOT HANGER KIT LEFT B	Left
10,11,12C,13-15	168861S	PIVOT HANGER KIT RIGHT C	Right
10,11,12C,13-15	168862S	PIVOT HANGER KIT LEFT C	Left
3,-5,9,10	168863	SEAT FRAME FASTENER KIT	


13 SEAT PAN CLAMP KIT			
POS #	ILLUSTRATION	DESCRIPTION	QTY
4		BHCS 5/16 -18 x 3/4	4
7		BHCS 5/16-18x1 3/4	2
5		WASHER FLAT AN 5/16 ROHS	6
6		NUT NYLOCK THIN 5/16-18	6
8		SPACER SEAT PAN	4
9		CLAMP SEAT PAN	4
10		1/4-28 X 1.85 BH C/S BLK ZN	2
11		WASHER FLAT AN 1/4	2
12		NUT NYLOCK THIN 1/4-28	2

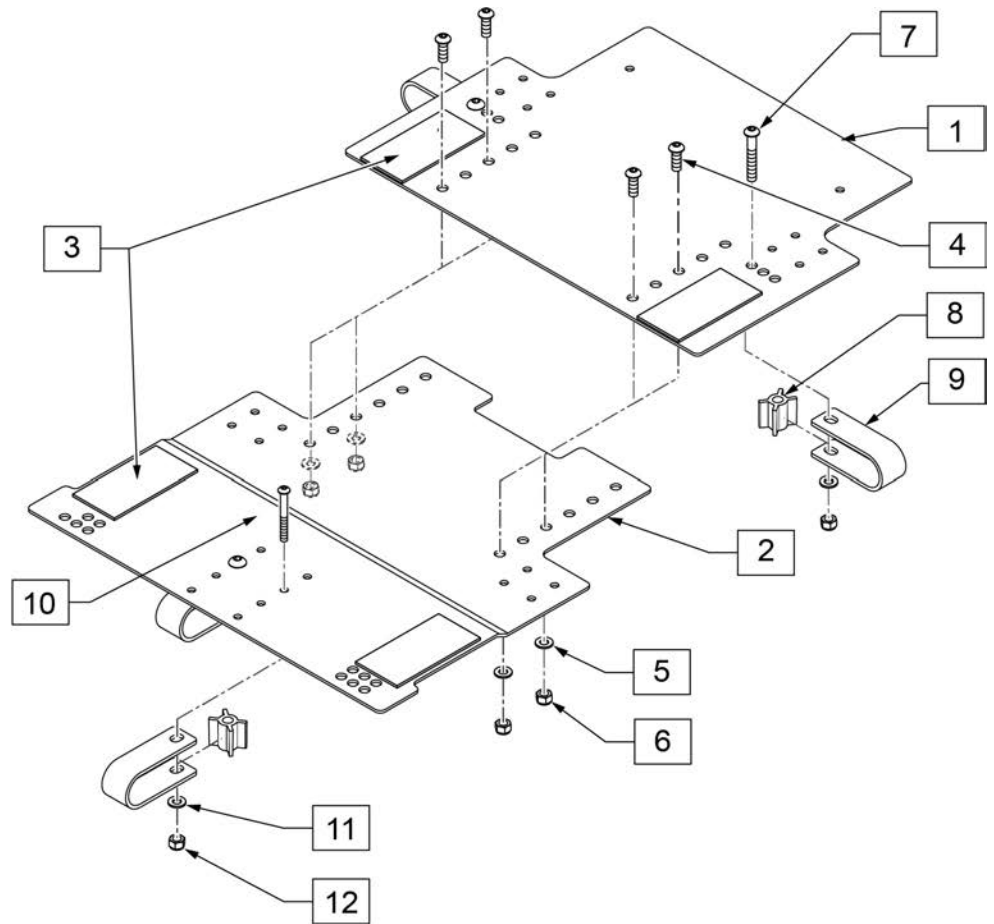

Pos.	Item Number	Description	Remarks
<<FRONT SEAT PANS WITH VELCRO>>			
2,3	168865	SEAT PAN FRONT 14 -16 VELCRO	Fits 14W STD
2,3	168866	SEAT PAN FRONT 15-17 VELCRO	Fits 15W STD
2,3	168867	SEAT PAN FRONT 16 -18 VELCRO	Fits 16W STD
2,3	168868	SEAT PAN FRONT 17-19 VELCRO	Fits 17W STD
2,3	168869	SEAT PAN FRONT 18 -20 VELCRO	Fits 18W STD
2,3	168870	SEAT PAN FRONT 19-21 VELCRO	Fits 19W STD
2,3	168871	SEAT PAN FRONT 20 -22 VELCRO	Fits 20W STD
<<REAR SEAT PANS WITH VELCRO>>			
1,3	168872	SEAT PAN REAR 14 -16 VELCRO	Fits 14W STD
1,3	168873	SEAT PAN REAR 15-17 VELCRO	Fits 15W STD
1,3	168874	SEAT PAN REAR 16 -18 VELCRO	Fits 16W STD
1,3	168875	SEAT PAN REAR 17-19 VELCRO	Fits 17W STD
1,3	168876	SEAT PAN REAR 18 -20 VELCRO	Fits 18W STD
1,3	168877	SEAT PAN REAR 19-21 VELCRO	Fits 19W STD
1,3	168878	SEAT PAN REAR 20 -22 VELCRO	Fits 20W STD
<<SEAT PAN ASSEMBLIES>>			
1-6	168879	SEAT PAN SET ONLY 14 -16	Fits 14W STD
1-6	168880	SEAT PAN SET ONLY 15-17	Fits 15W STD
1-6	168881	SEAT PAN SET ONLY 16 -18	Fits 16W STD
1-6	168882	SEAT PAN SET ONLY 17-19	Fits 17W STD
1-6	168883	SEAT PAN SET ONLY 18 -20	Fits 18W STD
1-6	168884	SEAT PAN SET ONLY 19-21	Fits 19W STD

Pos.	Item Number	Description	Remarks
1-6	168885	SEAT PAN SET ONLY 20 -22	Fits 20W STD
		<<KITS>>	
13	168864	SEAT PAN HDWR KIT	1 SIDE

Note: Tension Adjustable Ballistic Nylon Upholstery not available with the Tall (18"-24") Height Adjustable Backrest, Reclining Backrest, Dynamic Backrest, O2 Holder, IV Pole, JAY Anterior Thoracic supports or JAY Padded Trunk Strap accessories on ACCESS. If retrofitting Tension Adjustable Ballistic Nylon Upholstery, a Towel Bar is required. See the Towel Bar parts page for item numbers

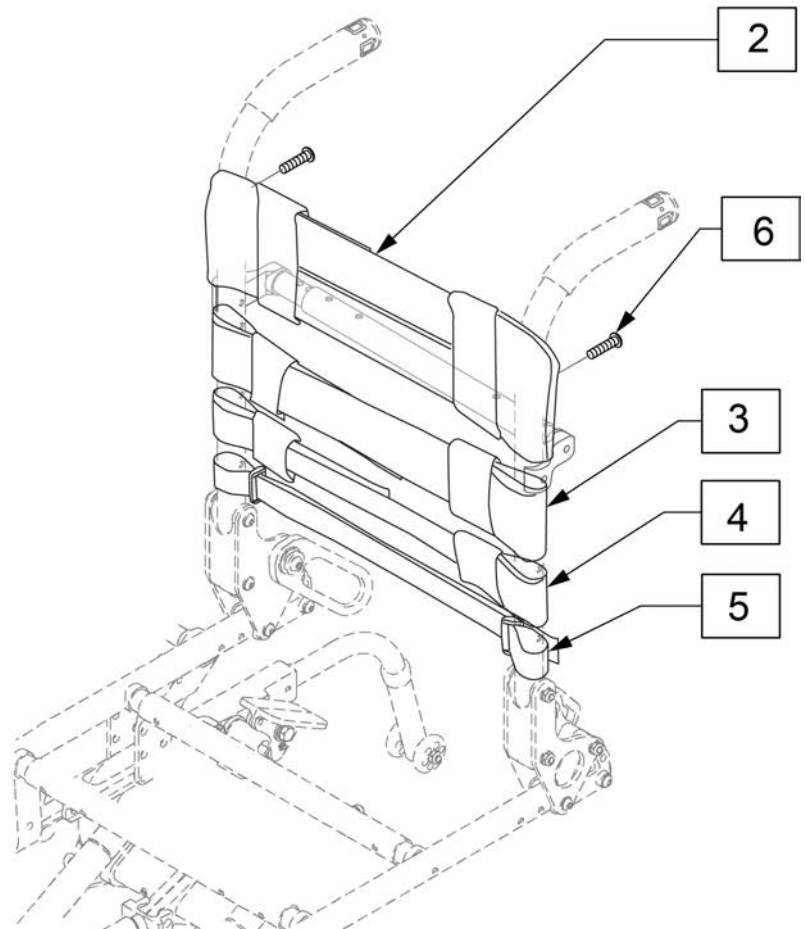

	Fixed HT	Adj HT
Med Fits	13", 14", 15"	12"-16"
Tall Fits	16", 17"	14"-18"
X-Tall Fits	18", 19"	16"-20"


Pos.	Item Number	Description	Remarks
2	113712	TOP CAP, STD, 11 -12"	Used for Omit Push Handle & Integral Push Handle only
2	113713	TOP CAP, STD, 13 -14"	Used for Omit Push Handle & Integral Push Handle only
2	113714	TOP CAP, STD, 15 -16"	Used for Omit Push Handle & Integral Push Handle only
2	113715	TOP CAP, STD, 17 -18"	Used for Omit Push Handle & Integral Push Handle only
2	113716	TOP CAP, STD, 19 -20"	Used for Omit Push Handle & Integral Push Handle only
3	113728	STRAP PADDED 2", 13 -14"	
3	113729	STRAP PADDED 2", 15 -16"	
3	113730	STRAP PADDED 2", 17 -18"	
3	113731	STRAP PADDED 2", 19 -20"	
4	113733	STRAP PADDED 1", 13 -14"	
4	113734	STRAP PADDED 1", 15 -16"	
4	113735	STRAP PADDED 1", 16 -17"	
4	113736	STRAP PADDED 1", 18 -19"	
5	113717	POSITION STRAP 1", 11 -12	
5	113718	POSITION STRAP 1", 13 -14	
5	113719	POSITION STRAP 1", 15 -16	

Pos.	Item Number	Description	Remarks
5	113720	POSITION STRAP 1", 17 -18	
5	113721	POSITION STRAP 1", 19 -20	
6	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
2-6	114880	ADJ UPH 13-14W STRAP KIT	
2-6	114881	ADJ UPH 15-16W STRAP KIT	
2-6	114882	ADJ UPH 17-18W STRAP KIT	
2-6	114883	ADJ UPH 19-20W STRAP KIT	
1A,2-6	113321-011	ADJ UP 14W X-SHT ASM	
1B,2-6	113321-012	ADJ UP 14W SHT ASM	
1C,2-6	113321-013	ADJ UP 14W MED ASM	
1D,2-6	113321-014	ADJ UP 14W TAL ASM	
1E,2-6	113321-015	ADJ UP 14W X-TAL ASM	
1A,2-6	113321-016	ADJ UP 15W X-SHT ASM	
1B,2-6	113321-017	ADJ UP 15W SHT ASM	
1C,2-6	113321-018	ADJ UP 15W MED ASM	
1D,2-6	113321-019	ADJ UP 15W TAL ASM	
1E,2-6	113321-020	ADJ UP 15W X-TAL ASM	
1A,2-6	113321-021	ADJ UP 16W X-SHT ASM	
1B,2-6	113321-022	ADJ UP 16W SHT ASM	
1C,2-6	113321-023	ADJ UP 16W MED ASM	
1D,2-6	113321-024	ADJ UP 16W TAL ASM	
1E,2-6	113321-025	ADJ UP 16W X-TAL ASM	
1A,2-6	113321-026	ADJ UP 17W X-SHT ASM	
1B,2-6	113321-027	ADJ UP 17W SHT ASM	
1C,2-6	113321-028	ADJ UP 17W MED ASM	
1D,2-6	113321-029	ADJ UP 17W TAL ASM	
1E,2-6	113321-030	ADJ UP 17W X-TAL ASM	
1A,2-6	113321-031	ADJ UP 18W X-SHT ASM	
1B,2-6	113321-032	ADJ UP 18W SHT ASM	
1C,2-6	113321-033	ADJ UP 18W MED ASM	
1D,2-6	113321-034	ADJ UP 18W TAL ASM	
1E,2-6	113321-035	ADJ UP 18W X-TAL ASM	
1A,2-6	113321-036	ADJ UP 19W X-SHT ASM	
1B,2-6	113321-037	ADJ UP 19W SHT ASM	
1C,2-6	113321-038	ADJ UP 19W MED ASM	
1D,2-6	113321-039	ADJ UP 19W TAL ASM	
1E,2-6	113321-040	ADJ UP 19W X-TAL ASM	
1A,2-6	113321-041	ADJ UP 20W X-SHT ASM	
1B,2-6	113321-042	ADJ UP 20W SHT ASM	
1C,2-6	113321-043	ADJ UP 20W MED ASM	
1D,2-6	113321-044	ADJ UP 20W TAL ASM	
1E,2-6	113321-045	ADJ UP 20W X-TAL ASM	

Note: Each back height uses a different variation of back straps; replacement strap kits contain more straps than needed to support all back heights.

Note: Item 6 is required when ordering a complete back upholstery; order 2 each of item 6.

Note: Tension Adjustable EXO Upholstery not available with the Tall (18"-24") Height Adjustable Backrest, Reclining Backrest, Dynamic Backrest, O2 Holder, IV Pole, JAY Anterior Thoracic supports or JAY Padded Trunk Strap accessories on ACCESS. If retrofitting Tension Adjustable EXO Upholstery, a Towel Bar is required. See the Towel Bar parts page for item numbers

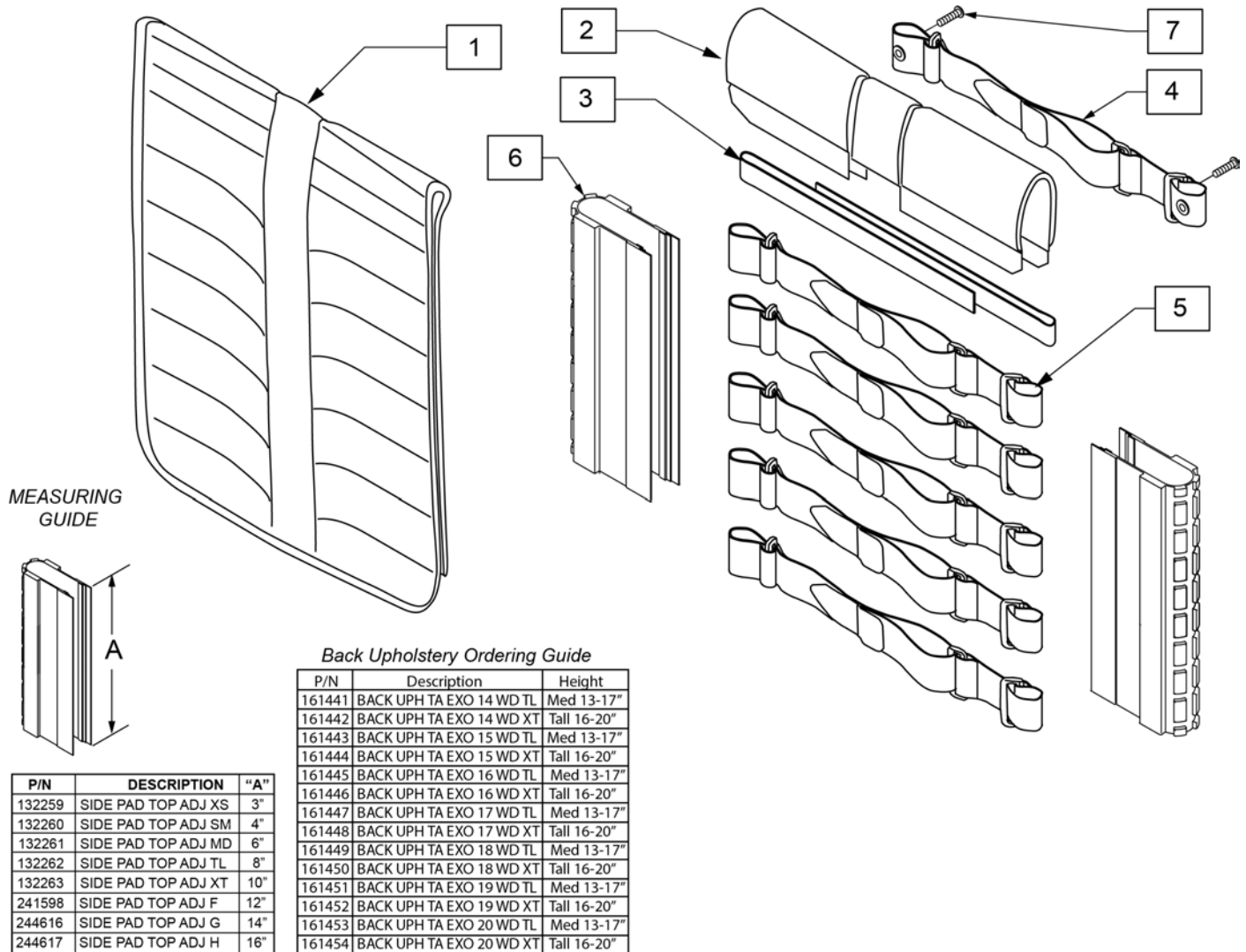


Pos.	Item Number	Description	Remarks
2	244215	BACKREST PAD SMALL 14"	Seat width 14"
2	244216	BACKREST PAD SMALL 15"	Seat width 15"
2	131876	BACKREST PAD SMALL 16"	Seat width 16"
2	244217	BACKREST PAD LARGE 17"	Seat width 17"
2	244218	BACKREST PAD LARGE 18"	Seat width 18"

Pos.	Item Number	Description	Remarks
2	244219	BACKREST PAD LARGE 19"	Seat width 19"
2	131877	BACKREST PAD LARGE 20"	Seat width 20"
3	052435-002	UPH,VNTD ADJ.,FPH STRP,14-15"W	Fold down push handle only
3	052435-003	UPH,VNTD ADJ.,FPH STRP,16-17"W	Fold down push handle only
3	052435-004	UPH,VNTD ADJ.,FPH STRP,18-20"W	Fold down push handle only
4	132337	BACK BELT ASSY W/ GROMMET 10-16	Integral, Omit & Folding w/Height Adj push handles
4	132338	BACK BELT ASSY W/ GROMMET 17-18	Integral, Omit & Folding w/Height Adj push handles
4	132339	BACK BELT ASSY W/ GROMMET 19-22	Integral, Omit & Folding w/Height Adj push handles
5	132297	BACK BELT ASSY 10-16	
5	132298	BACK BELT ASSY 17-18	
5	132299	BACK BELT ASSY 19-20	
6	132259	SIDE PAD TOP ADJ XS	Ht:11"; for Integral PH OR Folding PH w/ Ht Adj; see ordering guide
6	132260	SIDE PAD TOP ADJ SM	Ht:12",13"; Adj.Ht:10-14"; for Integral PH OR Folding PH w/ Ht Adj; see ordering guide
6	132261	SIDE PAD TOP ADJ MD	Ht:14",15"; Adj.Ht:12-16"; for Integral PH OR Folding PH w/ Ht Adj; see ordering guide
6	132262	SIDE PAD TOP ADJ TL	Ht:16"; Adj.Ht:14-18"; for Integral PH OR Folding PH w/ Ht Adj; see ordering guide
6	132263	SIDE PAD TOP ADJ XT	Adj.Ht:16-20"; for Integral PH OR Folding PH w/ Ht Adj; see ordering guide
6	241598	SIDE PAD TOP ADJ F	for Integral PH OR Folding PH w/ Ht Adj; see ordering guide
6	244616	SIDE PAD TOP ADJ G	for Integral PH OR Folding PH w/ Ht Adj; see ordering guide
6	244617	SIDE PAD TOP ADJ H	for Integral PH OR Folding PH w/ Ht Adj; see ordering guide
7	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
1-5	161675	BACK UPH TA EXO 14 WD XS	
1-5	161676	BACK UPH TA EXO 14 WD SM	
1-5	161677	BACK UPH TA EXO 14 WD MD	
1-5	161441	BACK UPH TA EXO 14 WD TL	
1-5	161442	BACK UPH TA EXO 14 WD XT	
1-5	161678	BACK UPH TA EXO 15 WD XS	
1-5	161679	BACK UPH TA EXO 15 WD SM	
1-5	161680	BACK UPH TA EXO 15 WD MD	
1-5	161443	BACK UPH TA EXO 15 WD TL	
1-5	161444	BACK UPH TA EXO 15 WD XT	
1-5	161681	BACK UPH TA EXO 16 WD XS	
1-5	161682	BACK UPH TA EXO 16 WD SM	
1-5	161683	BACK UPH TA EXO 16 WD MD	
1-5	161445	BACK UPH TA EXO 16 WD TL	
1-5	161446	BACK UPH TA EXO 16 WD XT	
1-5	161684	BACK UPH TA EXO 17 WD XS	
1-5	161685	BACK UPH TA EXO 17 WD SM	
1-5	161686	BACK UPH TA EXO 17 WD MD	
1-5	161447	BACK UPH TA EXO 17 WD TL	
1-5	161448	BACK UPH TA EXO 17 WD XT	
1-5	161687	BACK UPH TA EXO 18 WD XS	
1-5	161688	BACK UPH TA EXO 18 WD SM	
1-5	161689	BACK UPH TA EXO 18 WD MD	
1-5	161449	BACK UPH TA EXO 18 WD TL	
1-5	161450	BACK UPH TA EXO 18 WD XT	
1-5	161690	BACK UPH TA EXO 19 WD XS	
1-5	161691	BACK UPH TA EXO 19 WD SM	
1-5	161692	BACK UPH TA EXO 19 WD MD	
1-5	161451	BACK UPH TA EXO 19 WD TL	
1-5	161452	BACK UPH TA EXO 19 WD XT	
1-5	161693	BACK UPH TA EXO 20 WD XS	
1-5	161694	BACK UPH TA EXO 20 WD SM	
1-5	161695	BACK UPH TA EXO 20 WD MD	
1-5	161453	BACK UPH TA EXO 20 WD TL	


Pos.	Item Number	Description	Remarks
1-5	161454	BACK UPH TA EXO 20 WD XT	
1-5	161716	BACK UPH TA EXO 14 WD XS ORNG	
1-5	161717	BACK UPH TA EXO 14 WD SM ORNG	
1-5	161718	BACK UPH TA EXO 14 WD MD ORNG	
1-5	161719	BACK UPH TA EXO 14 WD TL ORNG	
1-5	161720	BACK UPH TA EXO 14 WD XT ORNG	
1-5	161721	BACK UPH TA EXO 15 WD XS ORNG	
1-5	161722	BACK UPH TA EXO 15 WD SM ORNG	
1-5	161723	BACK UPH TA EXO 15 WD MD ORNG	
1-5	161724	BACK UPH TA EXO 15 WD TL ORNG	
1-5	161725	BACK UPH TA EXO 15 WD XT ORNG	
1-5	161726	BACK UPH TA EXO 16 WD XS ORNG	
1-5	161727	BACK UPH TA EXO 16 WD SM ORNG	
1-5	161728	BACK UPH TA EXO 16 WD MD ORNG	
1-5	161729	BACK UPH TA EXO 16 WD TL ORNG	
1-5	161730	BACK UPH TA EXO 16 WD XT ORNG	
1-5	161731	BACK UPH TA EXO 17 WD XS ORNG	
1-5	161732	BACK UPH TA EXO 17 WD SM ORNG	
1-5	161733	BACK UPH TA EXO 17 WD MD ORNG	
1-5	161734	BACK UPH TA EXO 17 WD TL ORNG	
1-5	161735	BACK UPH TA EXO 17 WD XT ORNG	
1-5	161736	BACK UPH TA EXO 18 WD XS ORNG	
1-5	161737	BACK UPH TA EXO 18 WD SM ORNG	
1-5	161738	BACK UPH TA EXO 18 WD MD ORNG	
1-5	161739	BACK UPH TA EXO 18 WD TL ORNG	
1-5	161740	BACK UPH TA EXO 18 WD XT ORNG	
1-5	161741	BACK UPH TA EXO 19 WD XS ORNG	
1-5	161742	BACK UPH TA EXO 19 WD SM ORNG	
1-5	161743	BACK UPH TA EXO 19 WD MD ORNG	
1-5	161744	BACK UPH TA EXO 19 WD TL ORNG	
1-5	161745	BACK UPH TA EXO 19 WD XT ORNG	
1-5	161746	BACK UPH TA EXO 20 WD XS ORNG	
1-5	161747	BACK UPH TA EXO 20 WD SM ORNG	
1-5	161748	BACK UPH TA EXO 20 WD MD ORNG	
1-5	161749	BACK UPH TA EXO 20 WD TL ORNG	
1-5	161750	BACK UPH TA EXO 20 WD XT ORNG	
1-5	161771	BACK UPH TA EXO 14 WD XS SLVR	
1-5	161772	BACK UPH TA EXO 14 WD SM SLVR	
1-5	161773	BACK UPH TA EXO 14 WD MD SLVR	
1-5	161774	BACK UPH TA EXO 14 WD TL SLVR	
1-5	161775	BACK UPH TA EXO 14 WD XT SLVR	
1-5	161776	BACK UPH TA EXO 15 WD XS SLVR	
1-5	161777	BACK UPH TA EXO 15 WD SM SLVR	
1-5	161778	BACK UPH TA EXO 15 WD MD SLVR	
1-5	161779	BACK UPH TA EXO 15 WD TL SLVR	
1-5	161780	BACK UPH TA EXO 15 WD XT SLVR	
1-5	161781	BACK UPH TA EXO 16 WD XS SLVR	
1-5	161782	BACK UPH TA EXO 16 WD SM SLVR	
1-5	161783	BACK UPH TA EXO 16 WD MD SLVR	
1-5	161784	BACK UPH TA EXO 16 WD TL SLVR	
1-5	161785	BACK UPH TA EXO 16 WD XT SLVR	
1-5	161786	BACK UPH TA EXO 17 WD XS SLVR	
1-5	161787	BACK UPH TA EXO 17 WD SM SLVR	
1-5	161788	BACK UPH TA EXO 17 WD MD SLVR	
1-5	161789	BACK UPH TA EXO 17 WD TL SLVR	
1-5	161790	BACK UPH TA EXO 17 WD XT SLVR	
1-5	161791	BACK UPH TA EXO 18 WD XS SLVR	

Pos.	Item Number	Description	Remarks
1-5	161792	BACK UPH TA EXO 18 WD SM SLVR	
1-5	161793	BACK UPH TA EXO 18 WD MD SLVR	
1-5	161794	BACK UPH TA EXO 18 WD TL SLVR	
1-5	161795	BACK UPH TA EXO 18 WD XT SLVR	
1-5	161796	BACK UPH TA EXO 19 WD XS SLVR	
1-5	161797	BACK UPH TA EXO 19 WD SM SLVR	
1-5	161798	BACK UPH TA EXO 19 WD MD SLVR	
1-5	161799	BACK UPH TA EXO 19 WD TL SLVR	
1-5	161800	BACK UPH TA EXO 19 WD XT SLVR	
1-5	161801	BACK UPH TA EXO 20 WD XS SLVR	
1-5	161802	BACK UPH TA EXO 20 WD SM SLVR	
1-5	161803	BACK UPH TA EXO 20 WD MD SLVR	
1-5	161859	BACK UPH TA EXO 20 WD TL SLVR	
1-5	161805	BACK UPH TA EXO 20 WD XT SLVR	
1-5	161826	BACK UPH TA EXO 14 WD XS BLUE	
1-5	161827	BACK UPH TA EXO 14 WD SM BLUE	
1-5	161828	BACK UPH TA EXO 14 WD MD BLUE	
1-5	161829	BACK UPH TA EXO 14 WD TL BLUE	
1-5	161830	BACK UPH TA EXO 14 WD XT BLUE	
1-5	161831	BACK UPH TA EXO 15 WD XS BLUE	
1-5	161832	BACK UPH TA EXO 15 WD SM BLUE	
1-5	161833	BACK UPH TA EXO 15 WD MD BLUE	
1-5	161834	BACK UPH TA EXO 15 WD TL BLUE	
1-5	161835	BACK UPH TA EXO 15 WD XT BLUE	
1-5	161836	BACK UPH TA EXO 16 WD XS BLUE	
1-5	161837	BACK UPH TA EXO 16 WD SM BLUE	
1-5	161838	BACK UPH TA EXO 16 WD MD BLUE	
1-5	161839	BACK UPH TA EXO 16 WD TL BLUE	
1-5	161840	BACK UPH TA EXO 16 WD XT BLUE	
1-5	161841	BACK UPH TA EXO 17 WD XS BLUE	
1-5	161842	BACK UPH TA EXO 17 WD SM BLUE	
1-5	161843	BACK UPH TA EXO 17 WD MD BLUE	
1-5	161844	BACK UPH TA EXO 17 WD TL BLUE	
1-5	161845	BACK UPH TA EXO 17 WD XT BLUE	
1-5	161846	BACK UPH TA EXO 18 WD XS BLUE	
1-5	161847	BACK UPH TA EXO 18 WD SM BLUE	
1-5	161848	BACK UPH TA EXO 18 WD MD BLUE	
1-5	161849	BACK UPH TA EXO 18 WD TL BLUE	
1-5	161850	BACK UPH TA EXO 18 WD XT BLUE	
1-5	161851	BACK UPH TA EXO 19 WD XS BLUE	
1-5	161852	BACK UPH TA EXO 19 WD SM BLUE	
1-5	161853	BACK UPH TA EXO 19 WD MD BLUE	
1-5	161854	BACK UPH TA EXO 19 WD TL BLUE	
1-5	161855	BACK UPH TA EXO 19 WD XT BLUE	
1-5	161856	BACK UPH TA EXO 20 WD XS BLUE	
1-5	161857	BACK UPH TA EXO 20 WD SM BLUE	
1-5	161858	BACK UPH TA EXO 20 WD MD BLUE	
1-5	161804	BACK UPH TA EXO 20 WD TL BLUE	
1-5	161860	BACK UPH TA EXO 20 WD XT BLUE	
1-5	161881	BACK UPH TA EXO 14 WD XS RED	
1-5	161882	BACK UPH TA EXO 14 WD SM RED	
1-5	161883	BACK UPH TA EXO 14 WD MD RED	
1-5	161884	BACK UPH TA EXO 14 WD TL RED	
1-5	161885	BACK UPH TA EXO 14 WD XT RED	
1-5	161886	BACK UPH TA EXO 15 WD XS RED	
1-5	161887	BACK UPH TA EXO 15 WD SM RED	
1-5	161888	BACK UPH TA EXO 15 WD MD RED	


Pos.	Item Number	Description	Remarks
1-5	161889	BACK UPH TA EXO 15 WD TL RED	
1-5	161890	BACK UPH TA EXO 15 WD XT RED	
1-5	161891	BACK UPH TA EXO 16 WD XS RED	
1-5	161892	BACK UPH TA EXO 16 WD SM RED	
1-5	161893	BACK UPH TA EXO 16 WD MD RED	
1-5	161894	BACK UPH TA EXO 16 WD TL RED	
1-5	161895	BACK UPH TA EXO 16 WD XT RED	
1-5	161896	BACK UPH TA EXO 17 WD XS RED	
1-5	161897	BACK UPH TA EXO 17 WD SM RED	
1-5	161898	BACK UPH TA EXO 17 WD MD RED	
1-5	161899	BACK UPH TA EXO 17 WD TL RED	
1-5	161900	BACK UPH TA EXO 17 WD XT RED	
1-5	161901	BACK UPH TA EXO 18 WD XS RED	
1-5	161902	BACK UPH TA EXO 18 WD SM RED	
1-5	161903	BACK UPH TA EXO 18 WD MD RED	
1-5	161904	BACK UPH TA EXO 18 WD TL RED	
1-5	161905	BACK UPH TA EXO 18 WD XT RED	
1-5	161906	BACK UPH TA EXO 19 WD XS RED	
1-5	161907	BACK UPH TA EXO 19 WD SM RED	
1-5	161908	BACK UPH TA EXO 19 WD MD RED	
1-5	161909	BACK UPH TA EXO 19 WD TL RED	
1-5	161910	BACK UPH TA EXO 19 WD XT RED	
1-5	161911	BACK UPH TA EXO 20 WD XS RED	
1-5	161912	BACK UPH TA EXO 20 WD SM RED	
1-5	161913	BACK UPH TA EXO 20 WD MD RED	
1-5	161914	BACK UPH TA EXO 20 WD TL RED	
1-5	161915	BACK UPH TA EXO 20 WD XT RED	

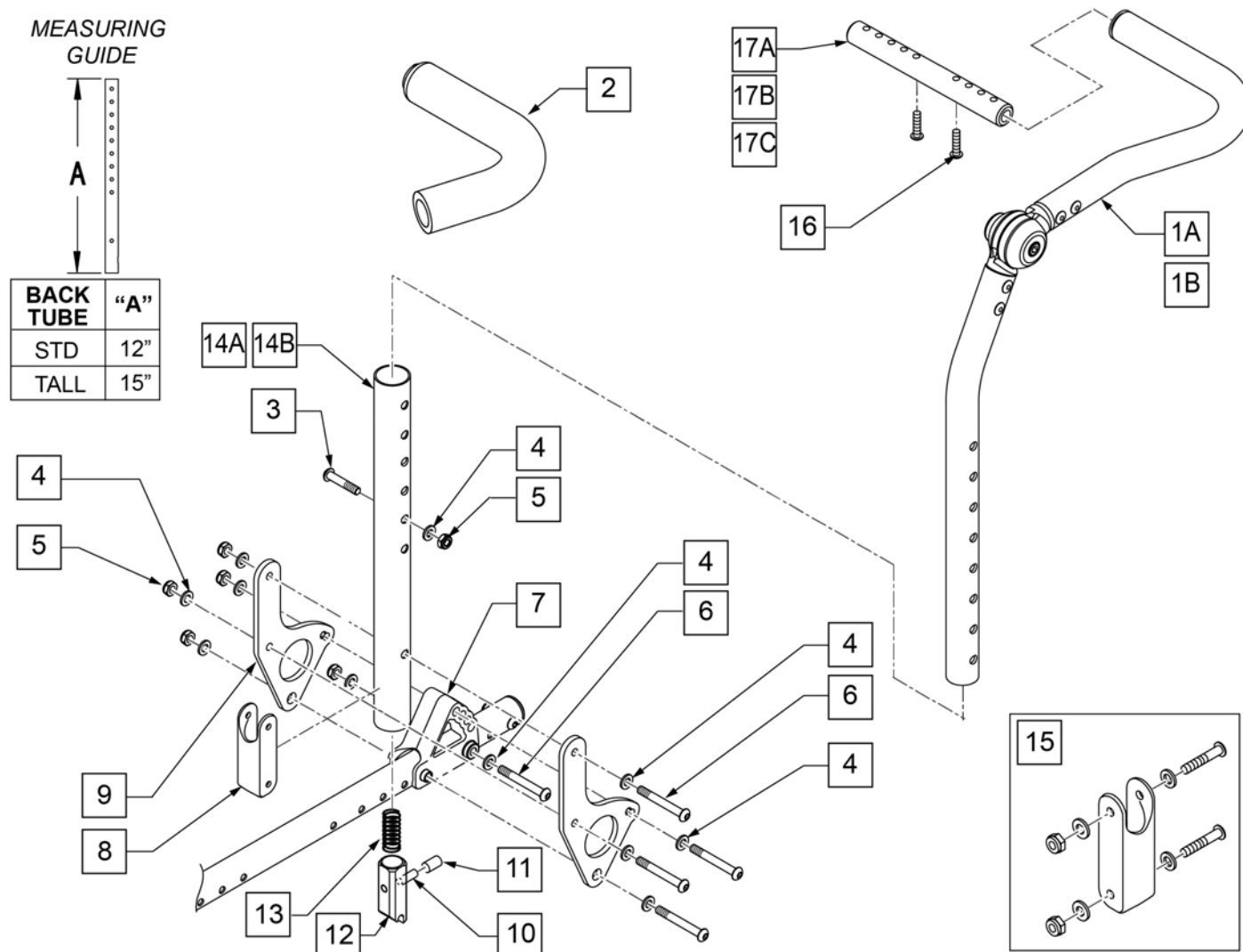

Pos.	Item Number	Description	Remarks
1A,2	110962-001	BACKREST UPPER HEIGHT ADJ PED R STD	RIGHT - Std - includes foam
1A,2	110963-001	BACKREST UPPER HEIGHT ADJ PED L STD	LEFT - Std - includes foam
1B,2	110962-002	BACKREST UPPER HEIGHT ADJ PED R LONG	RIGHT - Long Extended - includes foam
1B,2	110963-002	BACKREST UPPER HEIGHT ADJ PED L LONG	LEFT - Long Extended - includes foam
2	560638	FOAM GRIP .840 ID SHORT	
3-6	168890	HT ADJ BACK FASTENERS	
3-14A	168886	HT ADJ BACK LOWER ASSM RH STD	RIGHT - Std Height (17 -23" Back Heights)
3-14A	168887	HT ADJ BACK LOWER ASSM LH STD	LEFT - Std Height (17 -23" Back Heights)
3-14B	168888	HT ADJ BACK LOWER ASSM RH TLL	RIGHT - Tall Height (20 -26" Back Heights)
3-14B	168889	HT ADJ BACK LOWER ASSM LH TLL	LEFT - Tall Height (20 -26" Back Heights)
15	126099	BACK REST SADDLE W/ HDWR PAIR	Pair-Replacement Latch Stop Kit
15	126100	BACK REST SADDLE W/ HDWR EACH	Each-Replacement Latch Stop Kit
16	106960	BHCS 1/4 -28 x 7/8 ROHS	
17A	050541-001	SHAFT GROWTH PED SHT	Used on 10"-14" wide chairs
17B	050541-002	SHAFT GROWTH PED MED	Used on 12"-16" wide chairs
17C	521392	SHAFT GROWTH HANDLE LONG	Used on 14"-18" wide chairs

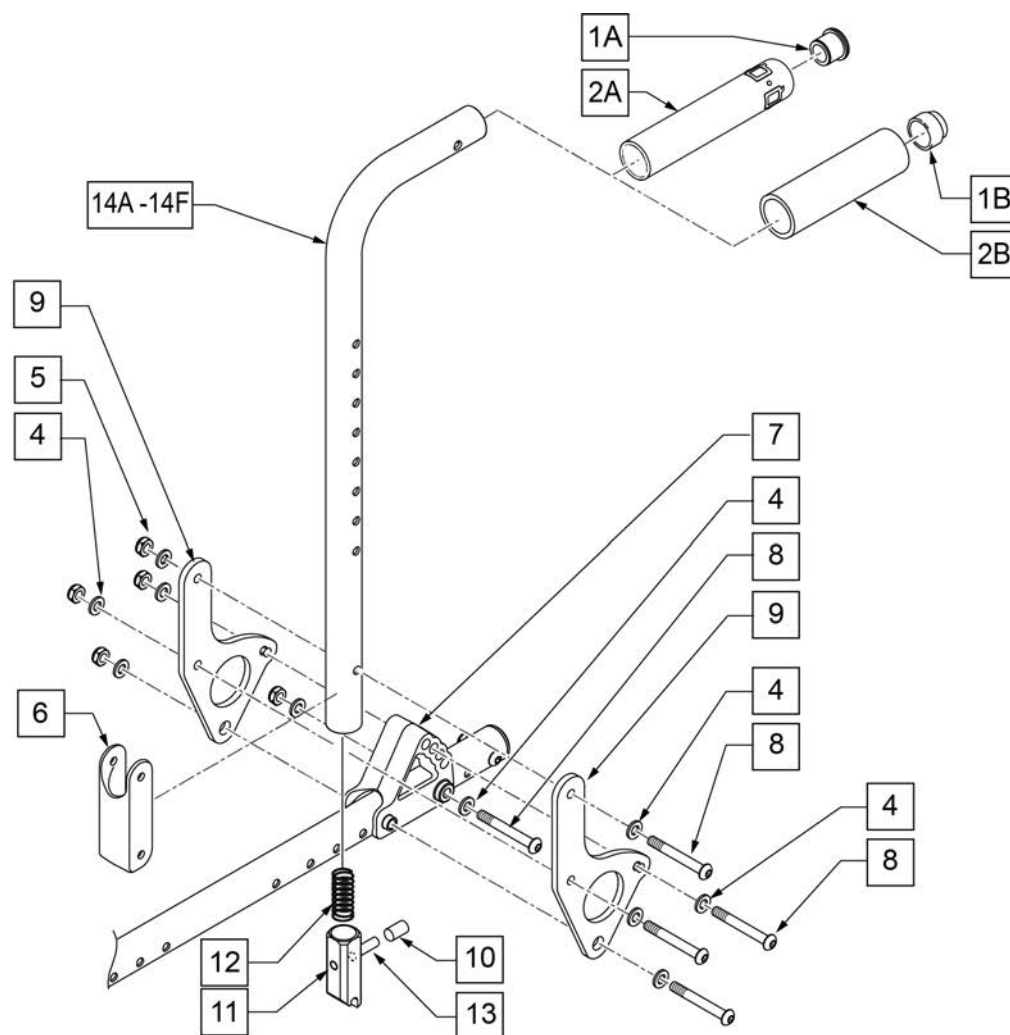

MEASURING GUIDE

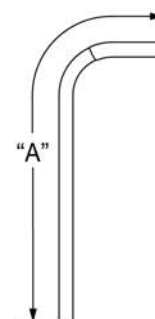


BACK TUBE	"A"
STD	25 3/4"
MED	28 3/4"
TALL	31 3/4"

NOTE: ABOVE DIMENSIONS ARE FOR BACK TUBE ONLY AND DO NOT INCLUDE FOAM GRIP.

Pos.	Item Number	Description	Remarks
1A	561002	1" INSERT PLUG .083	
1B	561082	1" TUBE END BUSHING THIN	USED W/ REMOVABLE STROLLER HANDLE
2A	920027	QUICKIE GRIP 1" W/INSTR SHEET	
2B	920023	QUICKIE GRIP 1" W/HOLE W/INSTR	USED W/ REMOVABLE STROLLER HANDLE
1A,2A,4-14A	168891	BACKREST FIXED HT R STD KIT	RIGHT LOW- Includes Plates & Latch
1A,2A,4-14A	168892	BACKREST FIXED HT L STD KIT	LEFT LOW- Includes Plates & Latch
1A,2A,4-14B	168893	BACKREST FIXED HT R MED KIT	RIGHT MEDIUM- Includes Plates & Latch
1A,2A,4-14B	168894	BACKREST FIXED HT L MED KIT	LEFT MEDIUM- Includes Plates & Latch
1A,2A,4-14C	168895	BACKREST FIXED HT R TA KIT	RIGHT TALL- Includes Plates & Latch
1A,2A,4-15C	168896	BACKREST FIXED HT L TALL KIT	LEFT TALL- Includes Plates & Latch
1B,2B,4-14D	168897	BACKREST FXD HT STRLR R STD KIT	RIGHT LOW- Includes Plates & Latch - Used w/ removable stroller handle
1B,2B,4-14D	168898	BACKREST FXD HT STRLR L STD KIT	LEFT LOW- Includes Plates & Latch - Used w/ removable stroller handle


Pos.	Item Number	Description	Remarks
1B,2B,4-14E	168899	BACKREST FXD HT STRLR R MED KIT	RIGHT MEDIUM- Includes Plates & Latch - Used w/ removable stroller handle
1B,2B,4-14E	168900	BACKREST FXD HT STRLR L MED KIT	LEFT MEDIUM- Includes Plates & Latch - Used w/ removable stroller handle
1B,2B,4-14F	168901	BACKREST FXD HT STRLR R LG KIT	RIGHT TALL- Includes Plates & Latch - Used w/ removable stroller handle
1B,2B,4-15F	168902	BACKREST FXD HT STRLR L TLL KIT	LEFT TALL- Includes Plates & Latch - Used w/ removable stroller handle
1A,2A,14A	842080	STROLLER HANDLE W/ FOAM STD ZPS/P5/QT	LOW
1A,2A,14B	842079	STROLLER HANDLE W/ FOAM MED ZP'S/QT	MED
1A,2A,14C	842081	STROLLER HANDLE W/ FOAM TALL ZP'S/QT	TALL
1B,2B,14D	842067	EXT STROLLER HANDLE STD	LOW -Used w/ Removable Stroller Handle
1B,2B,14E	842066	EXT STROLLER HANDLE MED	MED -Used w/ Removable Stroller Handle
1B,2B,14F	842068	EXT STROLLER HANDLE TALL	TALL -Used w/ Removable Stroller Handle
4,5,8	168890	HT ADJ BACK FASTENERS	

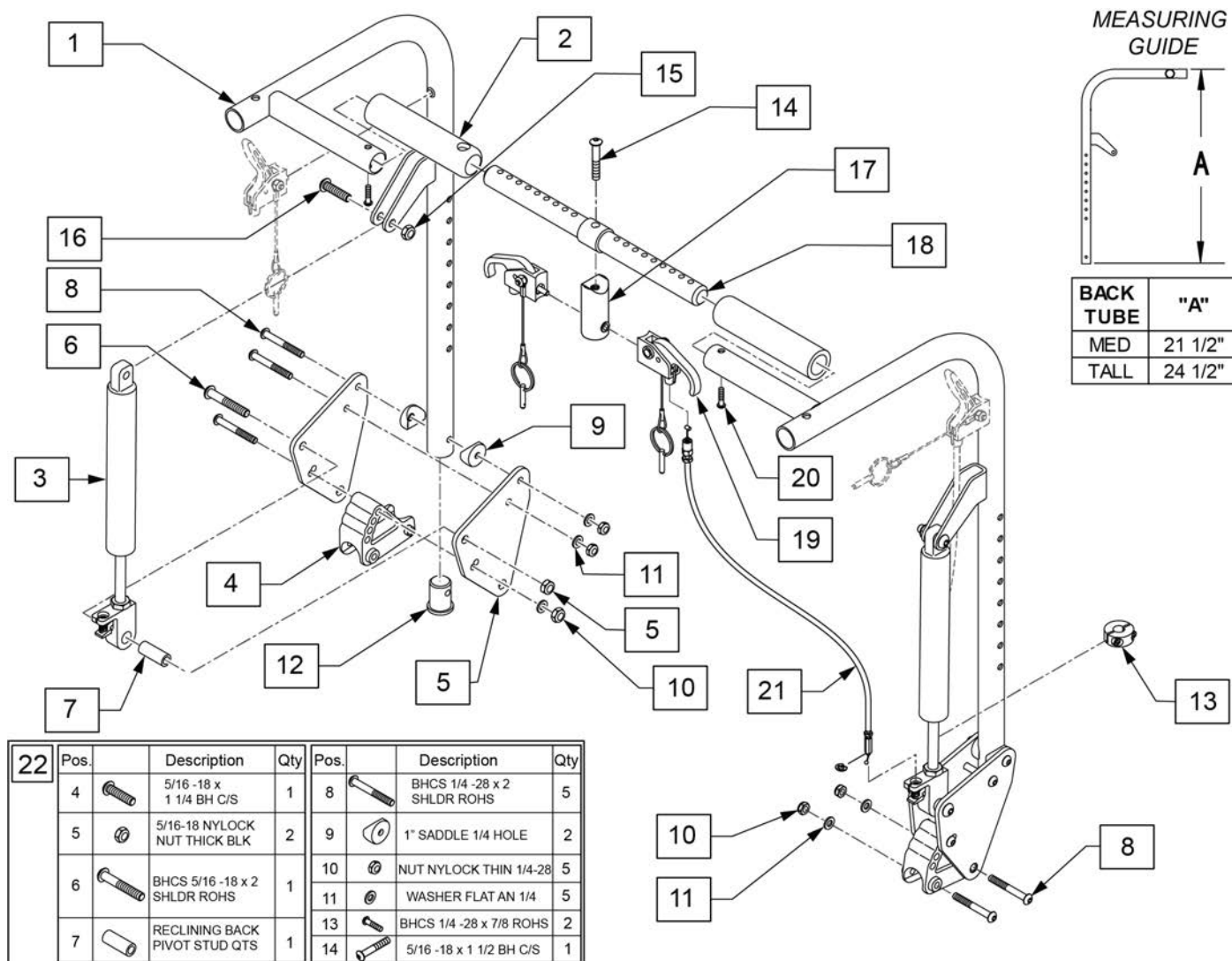


Pos.	Item Number	Description	Remarks
1	105869-001	WELDMOUNT WIDTH ADJ RECLINING BACK -MED	Right
1	105870-001	WELDMOUNT WIDTH ADJ RECLINING BACK -MED	Left
1	105869-002	WELDMOUNT WIDTH ADJ RECLINING BACK -TALL	Right
1	105870-002	WELDMOUNT WIDTH ADJ RECLINING BACK -TALL	Left
2	560664	FOAM GRIP RECLINING BCK QTS	
3	342065	BLOC-O-LIFT 200N x 11.58	
4	560517	BLOCK ANGLE BACKREST MOUNT	
1-22	168830	RECLINE BACKREST MED RETRO	
1-22	168831	RECLINE BACKREST TALL RETRO	
22	168832	RECLINE BACKREST HDWR KIT	
13	342049	10 MM I.D. SPLIT COLLAR	
1-13	256142	BK RST REC ASSY R MED	
1-13	256143	BK RST REC ASSY L MED	
1-13	256144	BK RST REC ASSY R TALL	
1-13	256145	BK RST REC ASSY L TALL	
14	106981	5/16 -18 x 1 1/2 BH C/S	
15	202117	5/16-18 NYLOCK NUT THICK BLK	
16	106980	5/16 -18 x 1 1/4 BH C/S	

Pos.	Item Number	Description	Remarks
17	483139	T45 RELEASE LEVER MNTG BLOCK	
18	500775	ADJ BAR QTS RECLINING BACK	
19	580111	CYL LOCK LEVER ASSM RCLNR	
20	106960	BHCS 1/4 -28 x 7/8 ROHS	
21	342057	CYL LOCK CABLE ASSM	MEDIUM
21	342056	CYL LOCK CABLE ASSM T45	TALL

MEASURING GUIDE

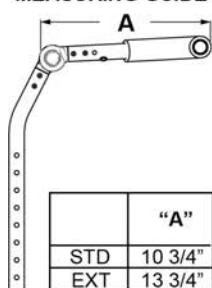

30 HARDWARE KIT HT/ADJ RECLN BACK			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
1		BHCs 1/4-28 X 1-1/4 SHLDR	1
3		1/4-28 NYLOCK NUT THIN BLK ZN	1
4		WASHER FLAT AN 1/4	5
5		NUT NYLOCK THIN 1/4-28	5
7		Q2 SADDLE	2
8		RECLINING BACK PIVOT STUD	1
9		NUT NYLOCK THIN 5/16-18	2
11		BHCs 1/4-28 X 2 SHLDR	5
13		BHCs 5/16-28 X 2 SHLDR	1
16		BHCs 5/16-18 X 1-1/4	1

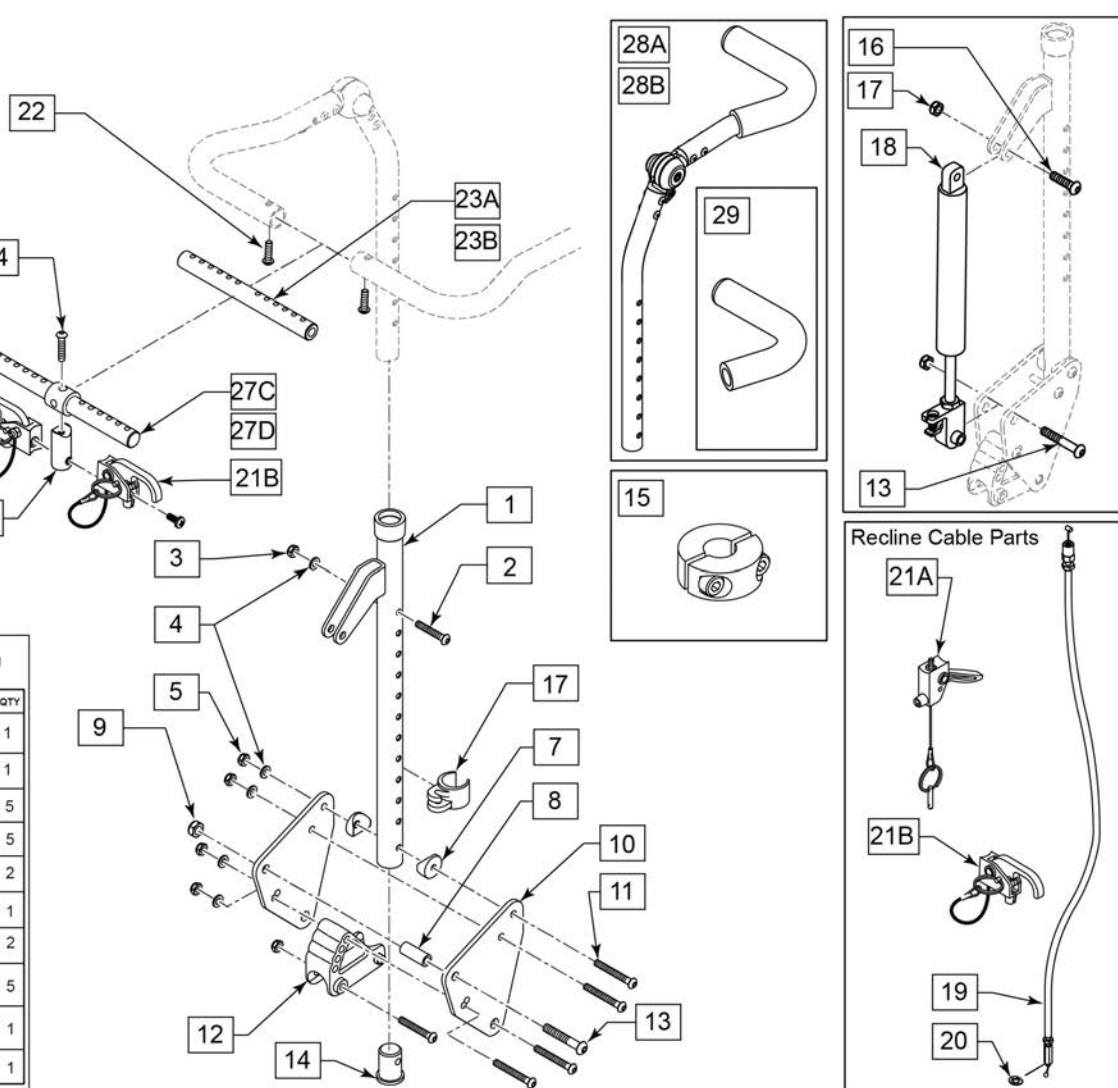

Pos.	Item Number	Description	Remarks
		Std Stroller Handle extension w/Foot Tilt Release	
1-18,22,21A,23A,28A	169710	HT ADJ STL HL STD FTLK 14	std- 14 wide
1-18,22,21A,23B,28A	169711	HT ADJ STL HL STD FTLK 15-22	std-15-22 wide
		Long Stroller Handle extension w/Foot Tilt Release	
1-18,22,21A,23A,28B	169712	HT ADJ STL HL LNG FTLK 14	long- 14 wide
1-18,22,21A,23B,28B	169713	HT ADJ STL HL LNG FTLK 15-22	long-15-22 wide
		Std Stroller Handle extension Dual Trigger Tilt Release	
1-18,22,21A,21B,24-26,27A,28A	169706	HT ADJ STL HL STD DTRG 14	std- 14 wide
1-18,22,21A,21B,24-26,27B,28A	169707	HT ADJ STL HL STD DTRG 15-22	std-15-22 wide
		Long Stroller Handle extension Dual Trigger Tilt Release	
1-18,22,21A,21B,24-26,27A,28A	169708	HT ADJ STL HL LNG DTRG 14	long- 14 wide
1-18,22,21A,21B,24-26,27B,28A	169709	HT ADJ STL HL LNG DTRG 15-22	long-15-22 wide
12	560517	BLOCK ANGLE BACKREST MOUNT	
14	244139	HT ADJ RCLN TUBE PLUG	
15	342049	10 MM I.D. SPLIT COLLAR	
17	108134	CLIP CABLE 1" (10)	PACK OF 10
19	342057	CYLINDER LOCK CABLE ASSM	RECLINE CABLE
26	281120	10-32 x 5/8 PAN HD PHIL	


Pos.	Item Number	Description	Remarks
20	342000	1/4 E-CLIP	
21A	580111	CYL LOCK LEVER ASSM RCLNR	For Dual Trigger Tilt release
22,23A	161542	SHAFT GROWTH HANDLE KIT SHRT	14W - for Foot Release Tilt
22,23B	161543	SHAFT GROWTH HANDLE KIT LNG	15-22W - forFoot Release Tilt
23A	243026	RCLN CNTR BAR ADJ STRLLR 10-18	
23B	243027	RCLN CNTR BAR ADJ STRLLR 15-23	
24	106981	BHCS 5/16-18 X 1 1/2	
25	483139	T45 RELEASE LEVER MNTG BLOCK	
28A	110968-001	BKRST UPPER HT ADJ R STD	std-15-22 wide
28A	110969-001	BKRST UPPER HT ADJ L STD	std-15-22 wide
28B	110968-002	BKRST UPPER HT ADJ R EXT	long-15-22 wide
28B	110969-002	BKRST UPPER HT ADJ L EXT	long-15-22 wide
29	560664	FOAM GRIP RECLINING BACK QTS	
30	161546	HARDWARE KIT HT/ADJ RECLN BK	
15-18	161544	GAS SPRING REPLACEMENT ASSM	
19,20,21A	161545	RECLINE LEVER W/CABLE	for Foot Release Tilt
21B,23A,24,25,26	161547	DUAL TRIGGER ASSM 10-16 W	14-16 for Dual Trigger Tilt release
21B,23B,24,25,26	163660	DUAL TRIGGER ASSM 17-22 W	17-22 for Dual Trigger Tilt release

NOTE: FOR TILT CABLES AND TRIGGERS, PLEASE REFER TO "TILT HANDLE & CABLE" PARTS PAGES

NOTE: Stroller handle extension works with 1" tubing backrest only. Example: DAB on Q2 or Standard backrest on positioning chairs.

NOTE: Not compatible with Swing-Away Adjustable Stroller Handle.


Note: For Q2X S/N chairs this is an "Off Chair" option only.

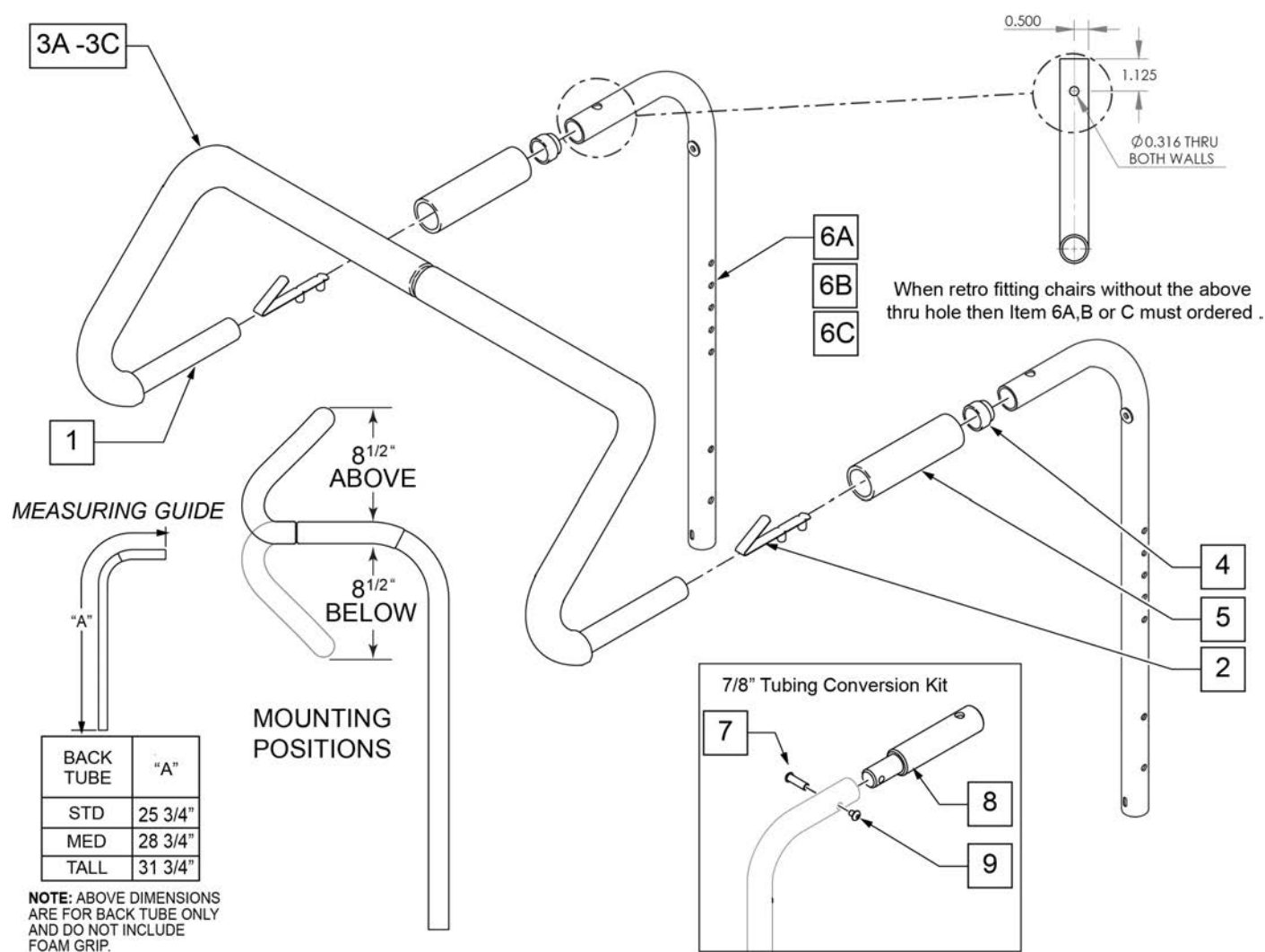


Pos.	Item Number	Description	Remarks
2	301108	5/16 SNAP BUTTON DUAL ONE SIDE	
3A	560615	STROLLER EXTENSION GRIP 10 -13"	10 -13" WIDE
3B	560616	STROLLER EXTENSION GRIP 14 -17"	14 -17" WIDE
3C	560617	STROLLER EXTENSION GRIP 18 -20"	18 -20" WIDE
4	561082	1" TUBE END BUSHING THIN	
5	920023	QUICKIE GRIP 1 W/ HOLE	
1, 2, 3A	842050	STROLLER EXTENSION ASSM 10"	

Pos.	Item Number	Description	Remarks
1, 2, 3A	842051	STROLLER EXTENSION ASSM 11"	
1, 2, 3A	842052	STROLLER EXTENSION ASSM 12"	
1, 2, 3A	842053	STROLLER EXTENSION ASSM 13"	
1, 2, 3B	842054	STROLLER EXTENSION ASSM 14"	
1, 2, 3B	842055	STROLLER EXTENSION ASSM 15"	
1, 2, 3B	842056	STROLLER EXTENSION ASSM 16"	
1, 2, 3B	842057	STROLLER EXTENSION ASSM 17"	
1, 2, 3C	842058	STROLLER EXTENSION ASSM 18"	
1, 2, 3C	842059	STROLLER EXTENSION ASSM 19	
1, 2, 3C	842063	STROLLER EXTENSION ASSM 20	
6A	842067	EXT STROLLER HANDLE STD	STANDARD/LOW
6B	842066	EXT STROLLER HANDLE MED	MEDIUM
6C	842068	EXT STROLLER HANDLE TALL	TALL
7-9	125324	STRLLR HNDL CNVERSN KIT 7/8 PH	To fit to 7/8" Push Handle; includes hardware for both sides

Note: The Towel Bar is REQUIRED when Tension Adjustable Upholstery is ordered on the Quickie ACCESS. The Towel Bar is not compatible when back upholstery is not ordered on the Quickie ACCESS.

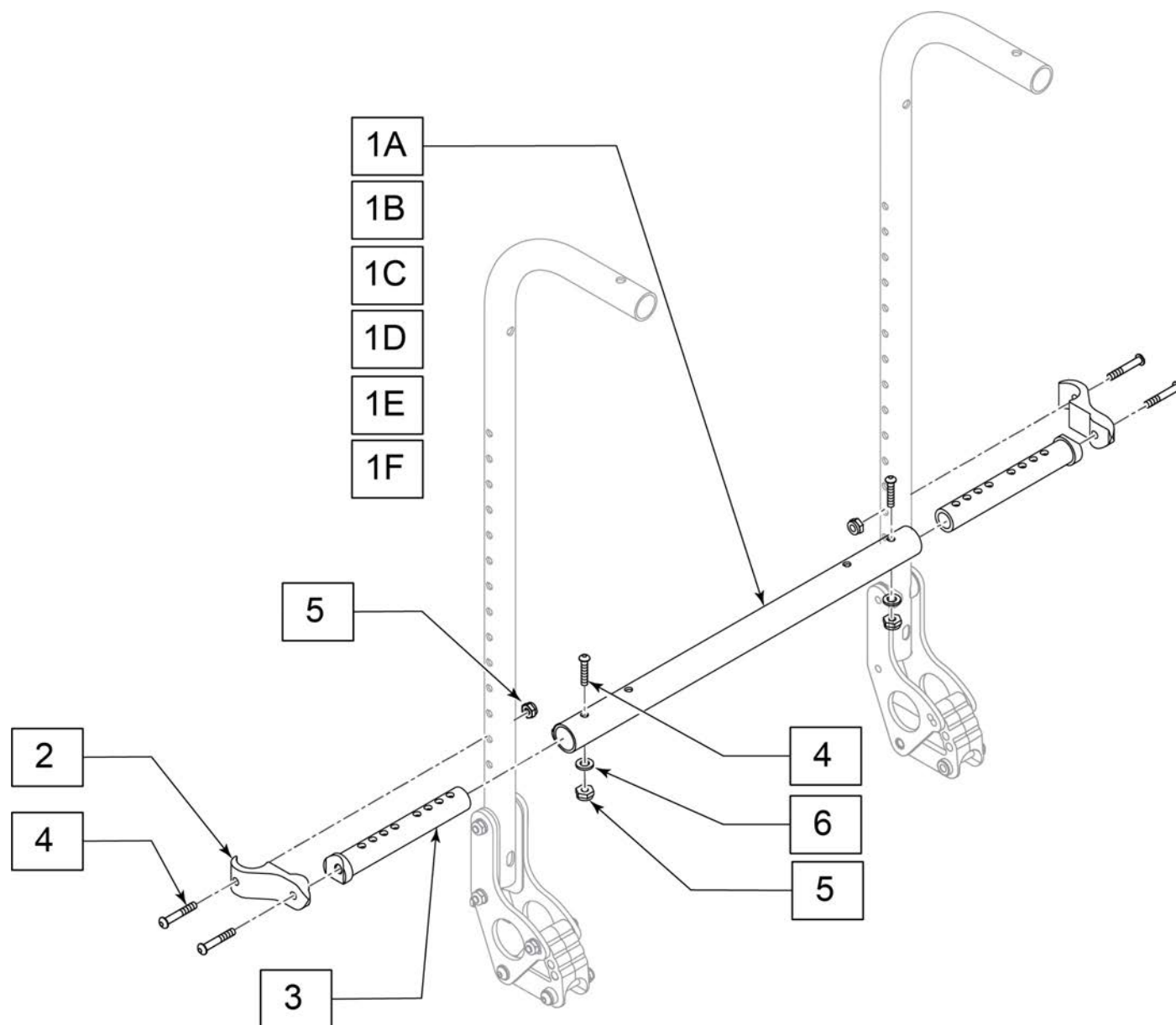


Pos.	Item Number	Description
2	550119	STRUT TUBE MOUNT BRACKET
4	107028	BHCS 1/4-28 X 1-1/2
5	106935	NUT NYLOCK THIN 1/4-28
6	106943	WASHER FLAT AN 1/4 ROHS
1A,5-7	168903	TOWEL BAR 14-15
1B,5-7	168904	TOWEL BAR 16
1C,5-7	168905	TOWEL BAR 17
1D,5-7	168906	TOWEL BAR 18
1E,5-7	168907	TOWEL BAR 19
1F,5-7	168908	TOWEL BAR 20-22

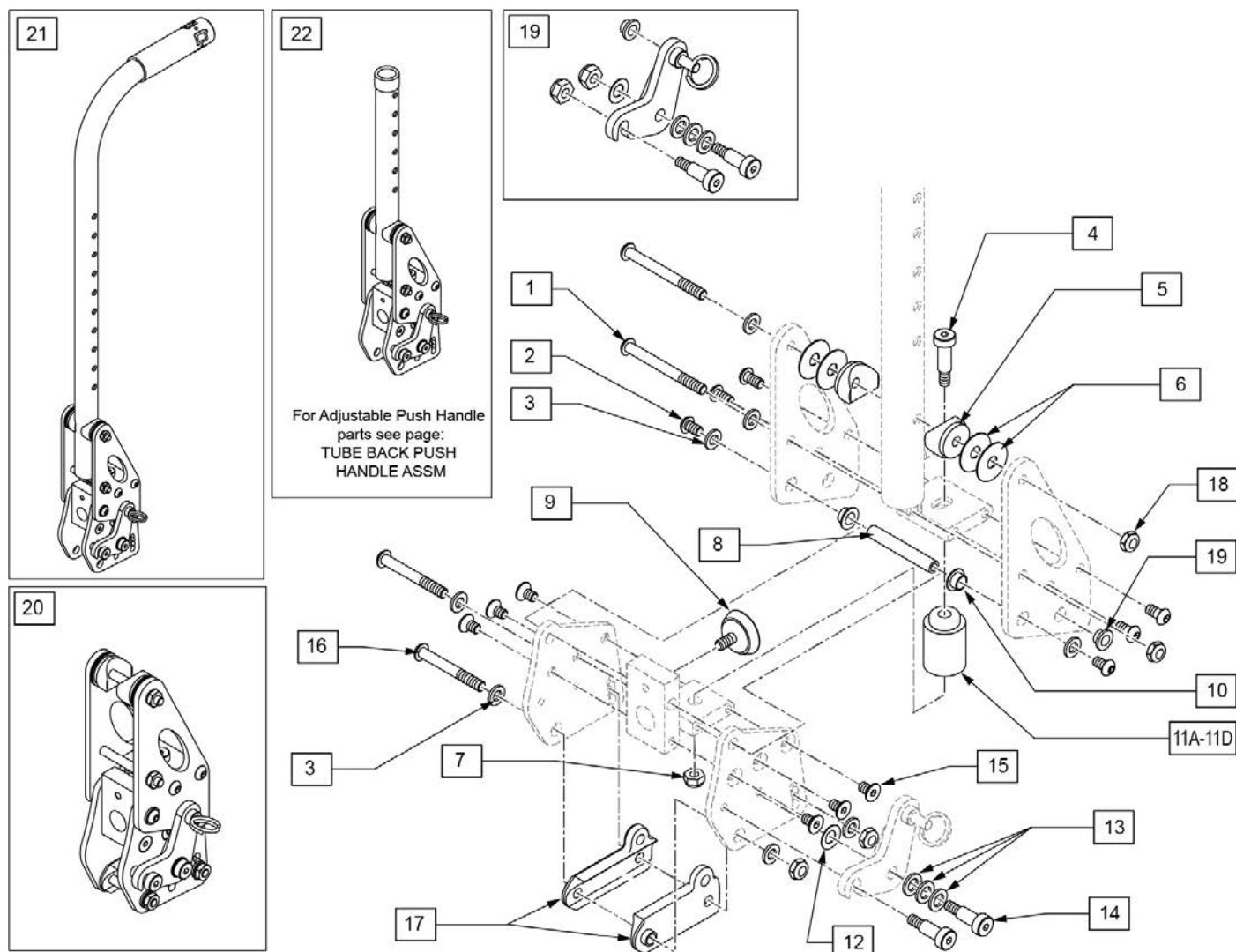


Pos.	Item Number	Description	Remarks
1	106967	BHCS 1/4-28 X 2-1/2 SHLDR ROHS	
2	103185-005	BHCS M6 X 1 X 12 FULL PL	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	011649-003	SHLDR SCREW M6 X 8MM OD X 45MM	
5	561087	BACK REL SADDLE RIGID	
6	113268	WASHER NYLON 1IN OD X .312 ID	
7	106935	NUT NYLOCK THIN 1/4-28 ROHS	
8	119410	PIVOT DYNAMIC BACKREST	
9	349560	ZP RUBBER STOP	
10	119418	IGLIDE BUSHING .22 DEEP	
11A	124044	SOFT RED SUSPENSION KIT	
11B	124060	MED BLUE SUSPENSION KIT	
11C	124064	FIRM GREEN SUSPENSION KIT	
11D	124082	X-FIRM BLACK SUSPENSION KIT	
12	262134	5/16 WASHER SPRING DISC	
13	106947	WASHER FLAT THIN AN 5/16 ROHS	
14	011649-001	SHLDR SCREW 8MM OD X 16MM LG	
15	013450	FHCS M6 X 1 X 10MM W/PCH	
16	106965	BHCS 1/4-28 X 2 SHLDR ROHS	


Pos.	Item Number	Description	Remarks
17	560518	SADDLE BACKREST Q2 DAB RIGHT	Right
17	560519	SADDLE BACKREST Q2 DAB LEFT	Left
18	104381	NUT HEX NYLOCK M6X1.0 INORG	
19	120943	DB LATCH REPLACEMENT KIT	Left & Right
20	121894	DAB DYNAMIC BACKREST ASSM RH	Right; Blue elastomer installed; Red, Green & Black available separately
20	121895	DAB DYNAMIC BACKREST ASSM LH	Left; Blue elastomer installed; Red, Green & Black available separately
21	123932	DYBCK ZPQ2 STR EXT ASSM STD RH	Right- Standard Stroller Handle
21	123933	DYBCK ZPQ2 STR EXT ASSM STD LH	Left- Standard Stroller Handle
21	123935	DYBCK ZPQ2 STR EXT ASSM MED RH	Right- Med Stroller Handle
21	123941	DYBCK ZPQ2 STR EXT ASSM MED LH	Left- Med Stroller Handle
21	123942	DYBCK ZPQ2 STR EXT ASSM TLL RH	Right- Tall Stroller Handle
21	123943	DYBCK ZPQ2 STR EXT ASSM TLL LH	Left- Tall Stroller Handle
22	123928	DYBCK ASSM ZPQ2 STD HGT ADJ RH	Right-Standard Ht Adjustable
22	123929	DYBCK ASSM ZPQ2 STD HGT ADJ L	Left-Standard Ht Adjustable
22	123930	DYBCK ASSM ZPQ2 TLL HGT ADJ RH	Right-Tall Ht Adjustable
22	123931	DYBCK ASSM ZPQ2 TLL HGT ADJ LH	Left-Tall Ht Adjustable
		Complete retro kit	
3,16-18,22	123944	ZP-Q2 DY BK PR ASM HGT ADJ STD	Pair; Blue elastomer installed; Red, Green & Black available separately
3,16-18,22	123945	ZP-Q2 DY BK PR ASM HGT ADJ TLL	Pair; Blue elastomer installed; Red, Green & Black available separately
3,16-18,21	123946	ZP-Q2 DY BK PR ASM STR EXT STD	Pair; Blue elastomer installed; Red, Green & Black available separately
3,16-18,21	123948	ZP-Q2 DY BK PR ASM STR EXT MED	Pair; Blue elastomer installed; Red, Green & Black available separately
3,16-18,21	123949	ZP-Q2 DY BK PR ASM STR EXT TLL	Pair; Blue elastomer installed; Red, Green & Black available separately

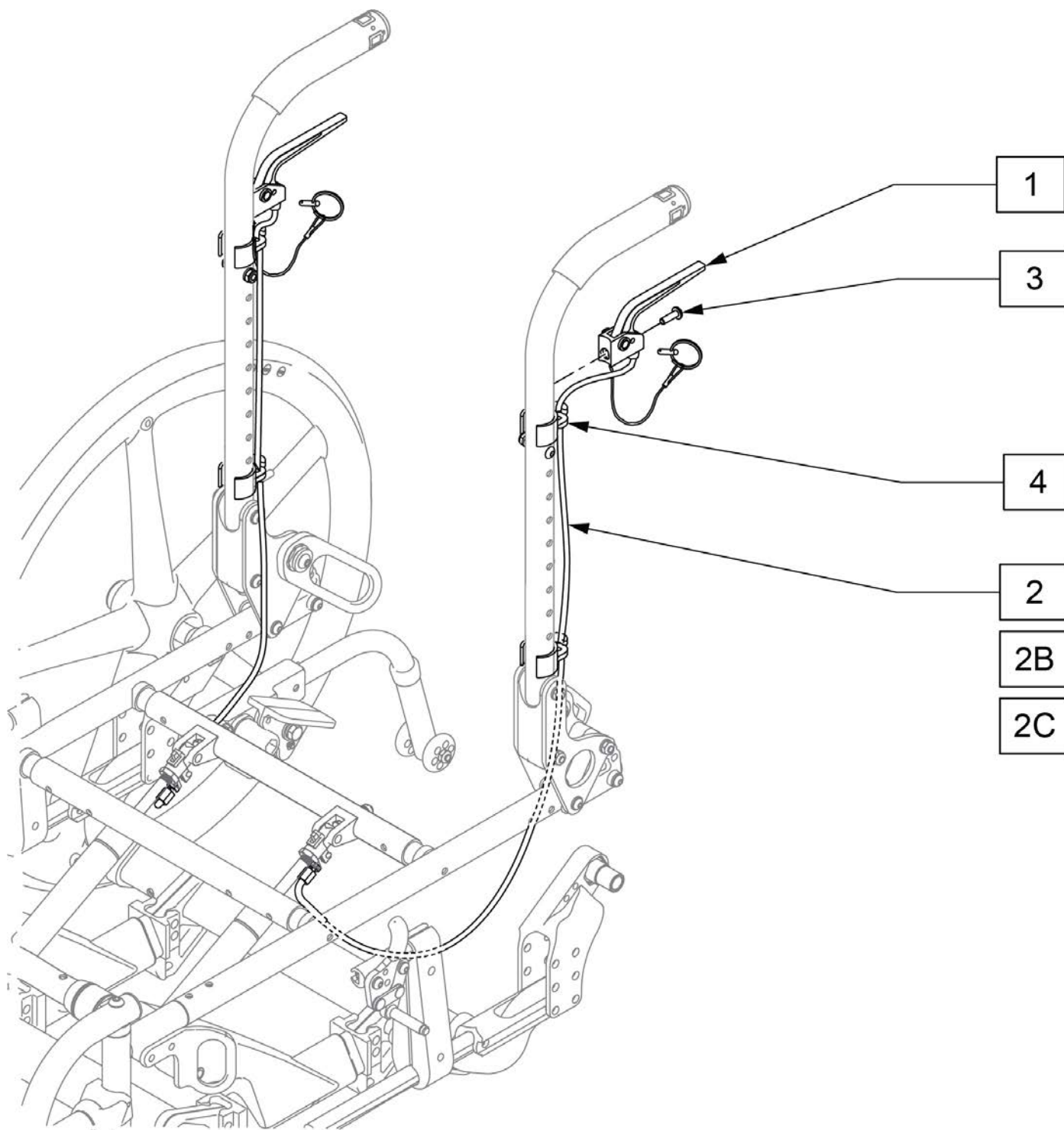


Pos.	Item Number	Description	Remarks
1	580110	CYL LOCK LEVER ASSM	
2	253844	CABLE ASSY TILT KNEE PIVOT LONG	LONG 44"
3	281120	10 -32 X 5/8 PAN HD PHIL	
4	108134	CLIP CABLE 1" PACK (10)	Pack of 10

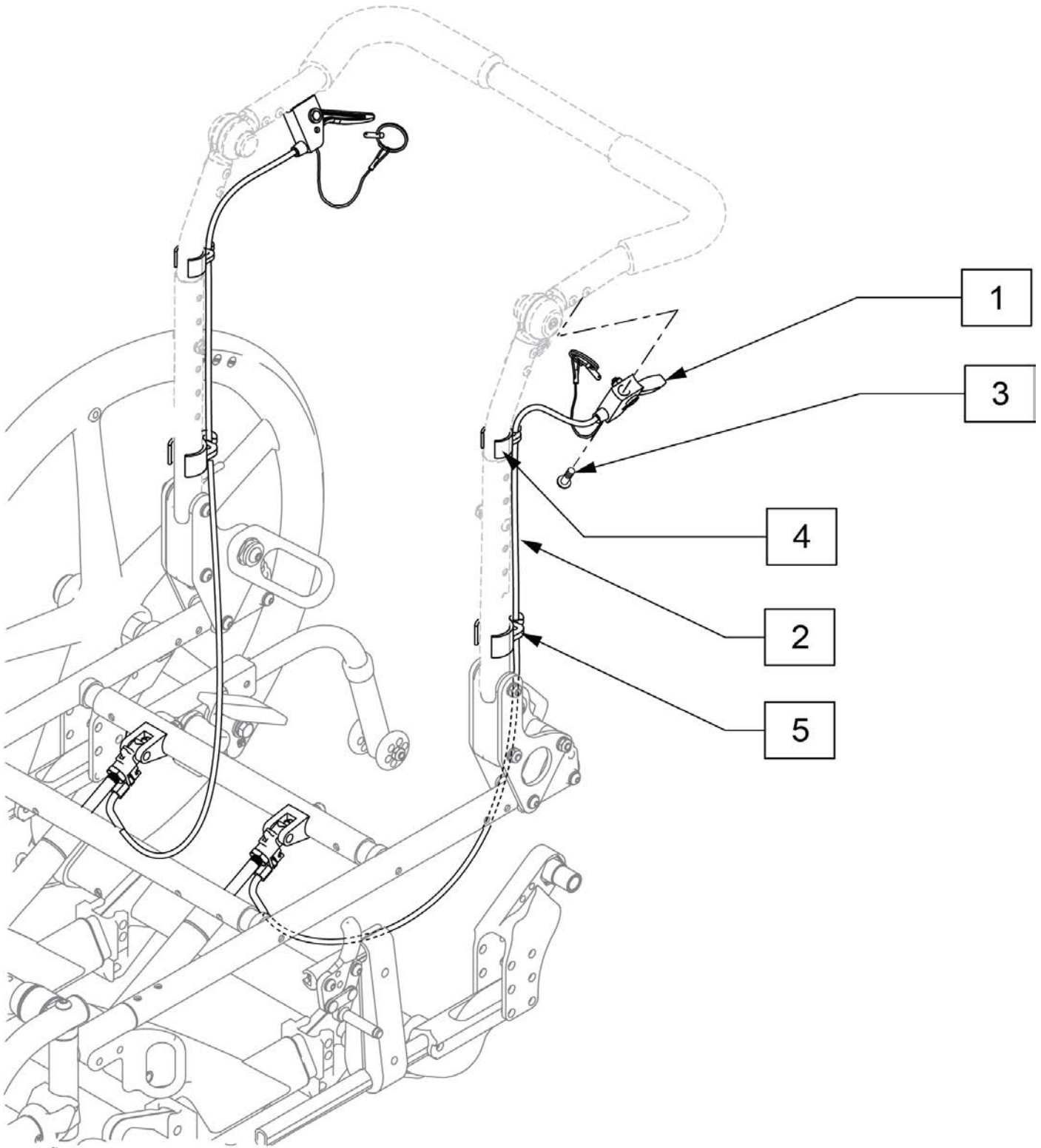


Pos.	Item Number	Description	Remarks
1	580109	PLUNGER PIN LEVER ASSM	
2	253845	CABLE ASSY TILT MED KNEE PIVOT	X-LONG 54"
3	281120	10 -32 X 5/8 PAN HD PHIL	
4	108135	CLIP CABLE 7/8" PACK (10)	Pack of 10
5	108134	CLIP CABLE 1" PACK (10)	Pack of 10

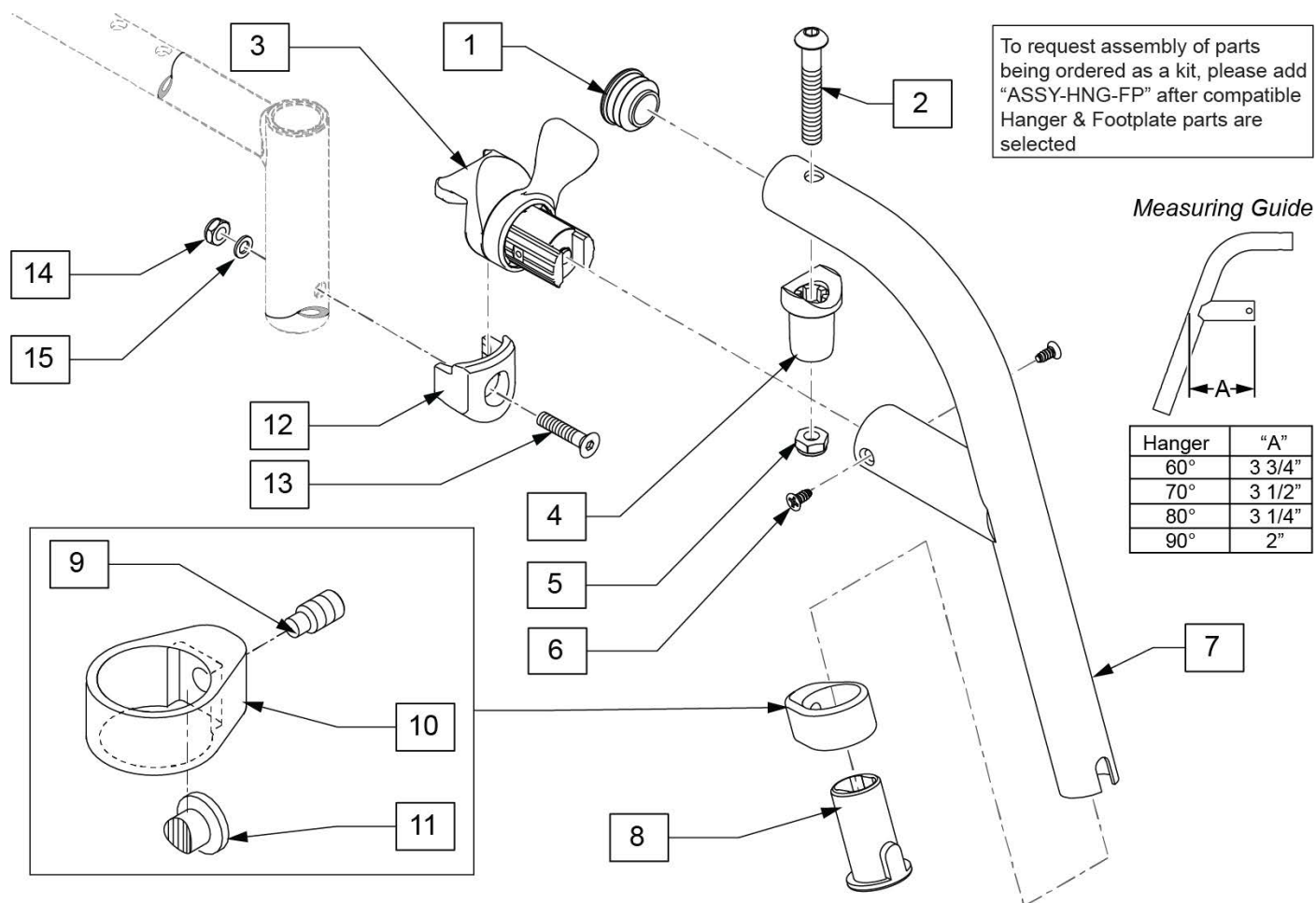


Pos.	Item Number	Description	Remarks
1	561062	1" INSERT PLUG .065 RIBS	
8	104160	BUSHING FOOTREST	
13	103500-010	FHCSM6X1X35FULLNP	
14	104381	NUT HEX NYLOCK M6X1.0	
15	104414	WAVE WASHER 6 MM	
1,2,4,5	119518	PIVOT SADDLE KIT	
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right
1-11	122699S	60 D SWING I/O HNGR PNTD L	
1-11	122700S	60 D SWING I/O HNGR PNTD R	
8-11	107485	FOOTREST CLAMP ASSM	
12-15	119514	LATCH RECEIVER KIT	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

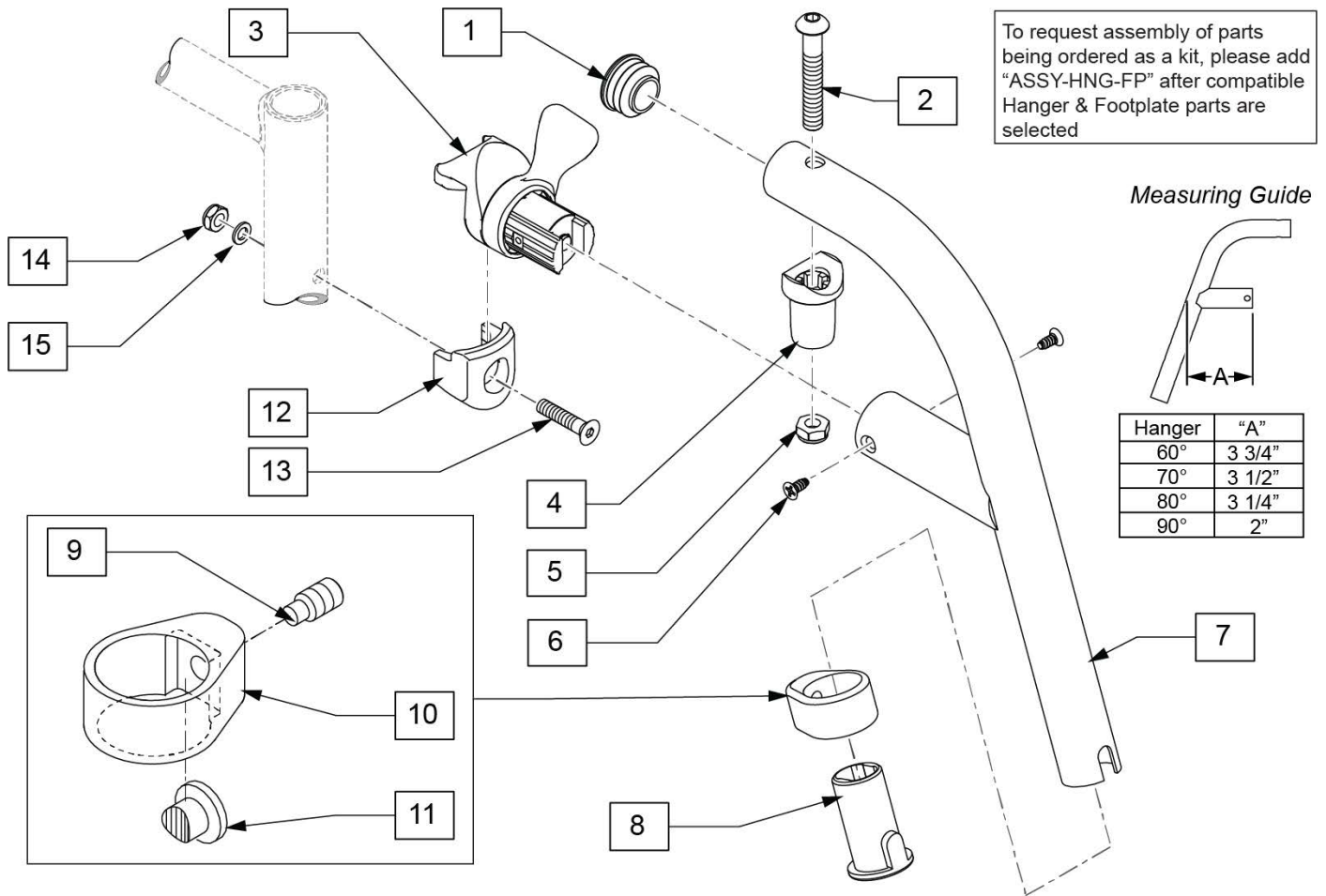


Pos.	Item Number	Description	Remarks
1	561062	1" INSERT PLUG .065 RIBS	
8	104160	BUSHING FOOTREST	
13	103500-010	FHCSM6X1X35FULLNP	
14	104381	NUT HEX NYLOCK M6X1.0	
15	104414	WAVE WASHER 6 MM	
1,2,4,5	119518	PIVOT SADDLE KIT	
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right
1-11	119458S	70 D SWING I/O HNGR PNTD L	Left
1-11	119459S	70 D SWING I/O HNGR PNTD R	Right
8-11	107485	FOOTREST CLAMP ASSM	
12-15	119514	LATCH RECEIVER KIT	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

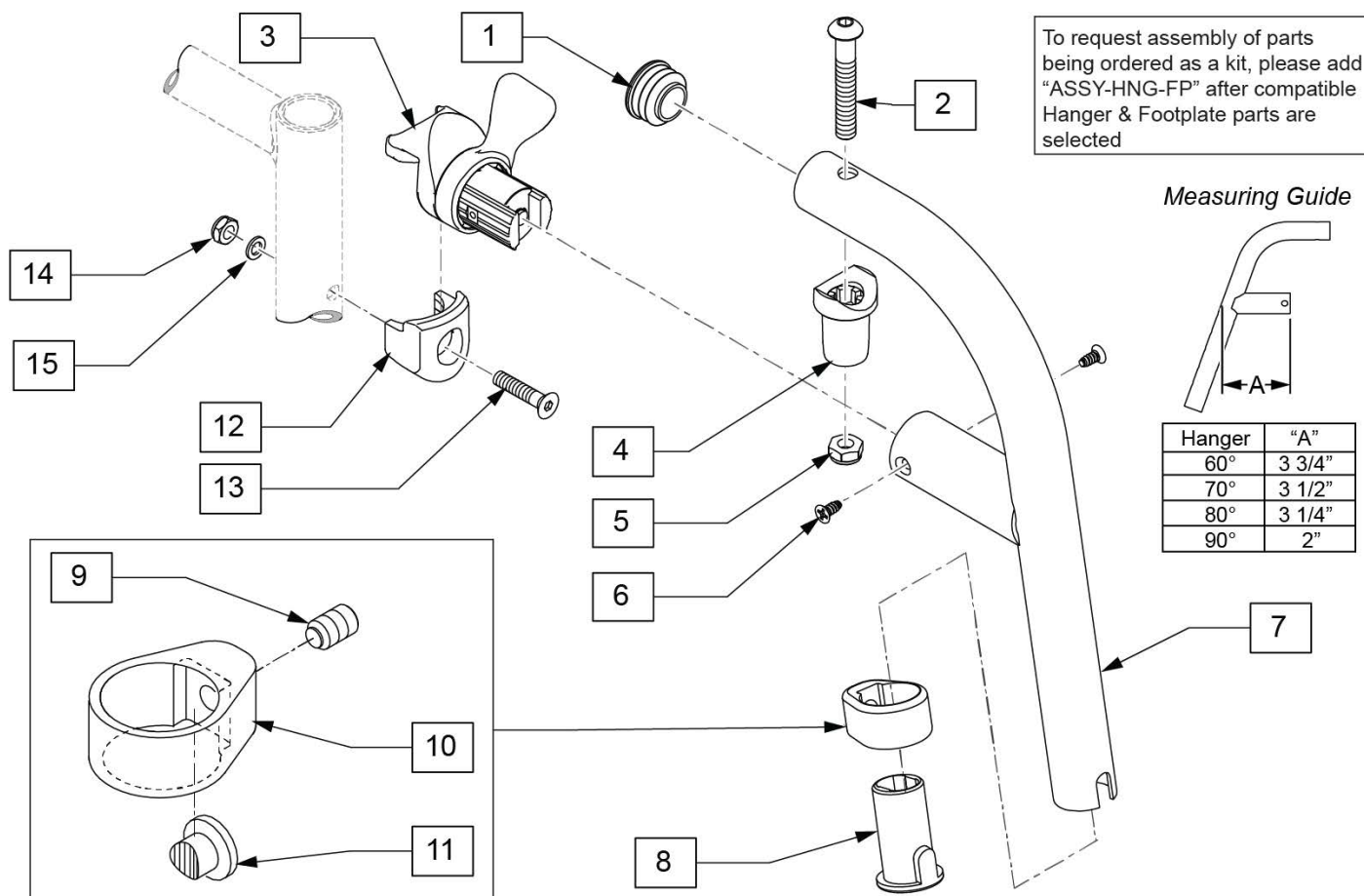

Pos.	Item Number	Description	Remarks
1	561062	1" INSERT PLUG .065 RIBS	
8	104160	BUSHING FOOTREST	
13	103500-010	FHCSM6X1X35FULLNP	
14	104381	NUT HEX NYLOCK M6X1.0	
15	104414	WAVE WASHER 6 MM	
1,2,4,5	119518	PIVOT SADDLE KIT	
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right
1-11	119466S	80 D SWING I/O HNGR PNTD L	Left
1-11	119467S	80 D SWING I/O HNGR PNTD R	Right
8-11	107485	FOOTREST CLAMP ASSM	
12-15	119514	LATCH RECEIVER KIT	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

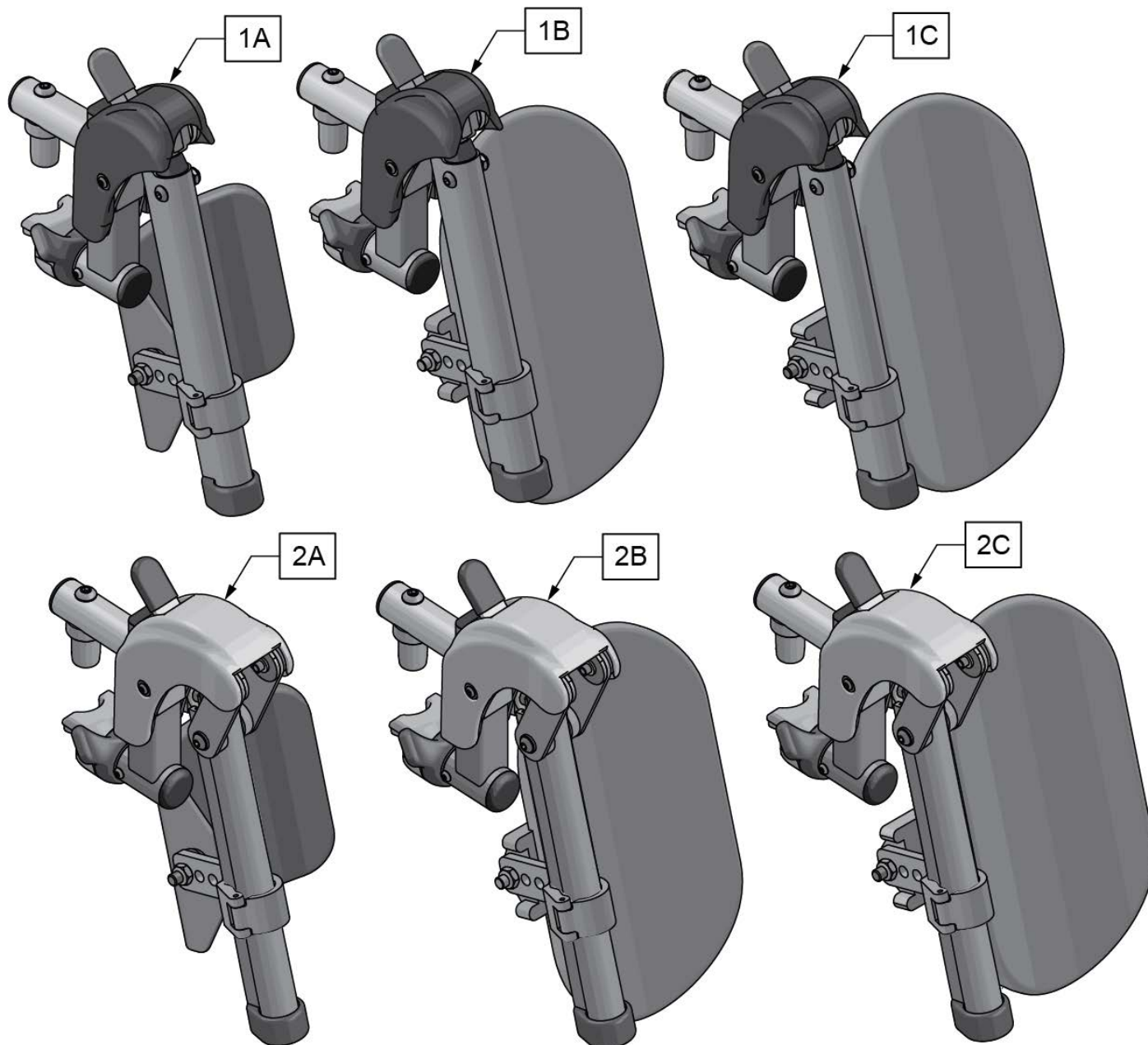


Pos.	Item Number	Description	Remarks
1A	161233S	SWNG I/O ELR ASM 14-15 W/PAD	Pair
1B	161234S	SWNG I/O ELR ASM 16-17 W/PAD	Pair
1C	161235S	SWNG I/O ELR ASM 18-20 W/PAD	Pair
2A	161236S	SWNG I/O ALR ASM 14-15 W/PAD	Pair
2B	161237S	SWNG I/O ALR ASM 16-17 W/PAD	Pair
2C	161238S	SWNG I/O ALR ASM 18-20 W/PAD	Pair
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

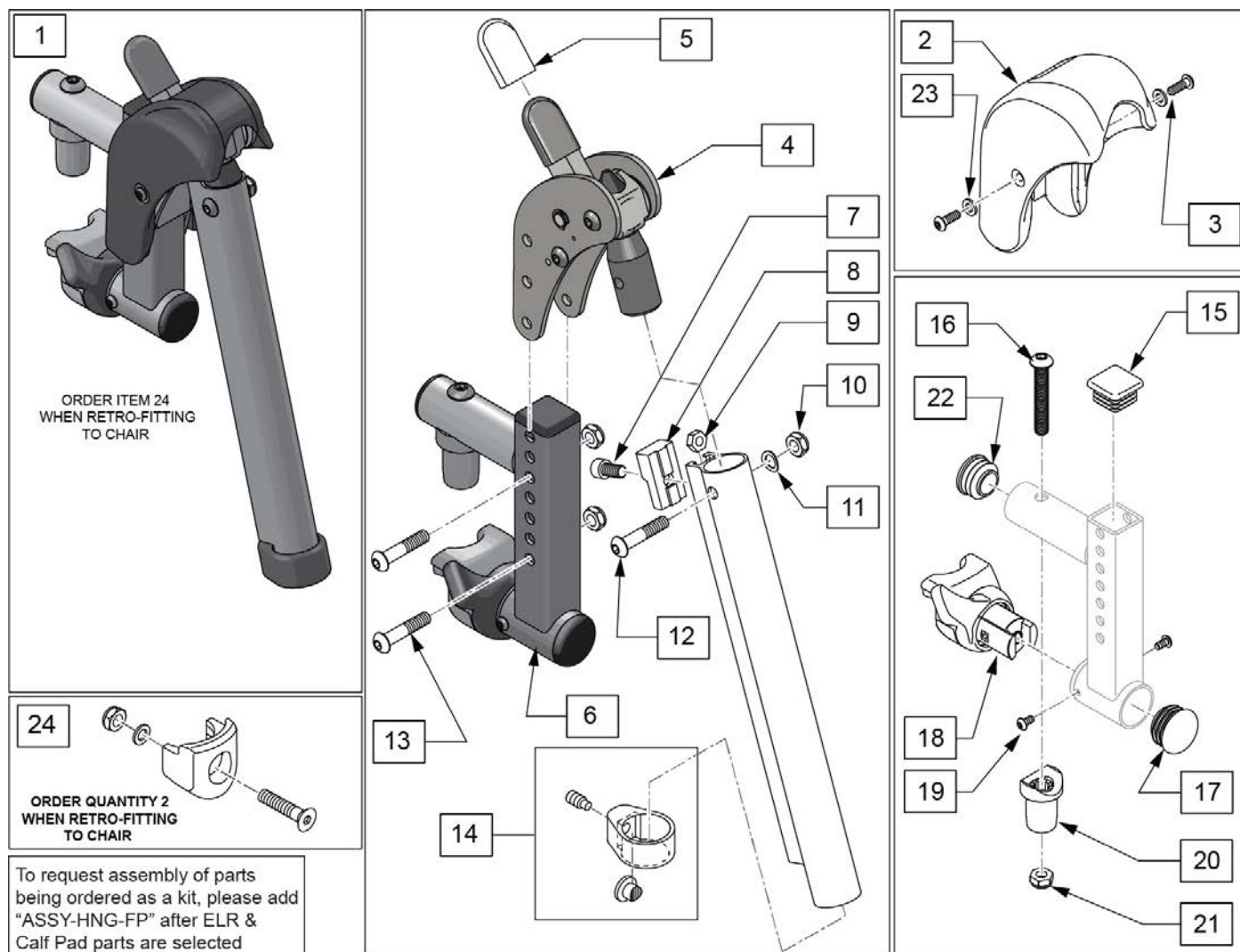


Pos.	Item Number	Description	Remarks
1	161239S	SWNG I/O ELR ASSM RH	Right
1	161240S	SWNG I/O ELR ASSM LH	Left
1	161241S	SWNG I/O ELR ASSM PR	Pair
2,3,23	129640	ELR SHROUD KIT	Kit w/ 2 screws
3	127135	BHCS 8-32 X 1/2 BLK	
4	126992	HEMI ELR SUB ASSM	
5	560501	ALR REL LEVER COVER	
6	134188	RECEIVER ASSY ELR ALR RH	
6	134186	RECEIVER ASSY ELR ALR LH	
7	107350	SHCS 1/4-28 x 1/2	
8	127129	ANGLE STOP ELR	
9	107249	NUT HEX THIN 1/4-28	
10	106935	1/4-28 NYLOCK NUT THIN BLK ZN	
11	104414	WAVE WASHER 6 MM	
12	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
13	108623	BHCS 1/4-28 x 1-1/2	
14	119495	EXTENSION CLAMP KIT	
15	010615	SQUARE END CAP CNTR LEG	
16	226217	5/16-18 X 2" BH C/S BLK	

Pos.	Item Number	Description	Remarks
17	561062	1" INSERT PLUG .065 RIBS	
18,19	161242	ELR/ALR HANGER LATCH KIT RH	Right
18,19	161243	ELR/ALR HANGER LATCH KIT LH	Left
20	561197	PLASTIC PIVOT SADDLE	
21	202102	5/16-18 NYLOCK NUT THIN	
22	560110	ENDPLUG BACKREST TNT	
23	113282	WASHER FLAT AN #10 ROHS	
24	119514	LATCH RECEIVER KIT	order qty 2 when retro-fitting to chair
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

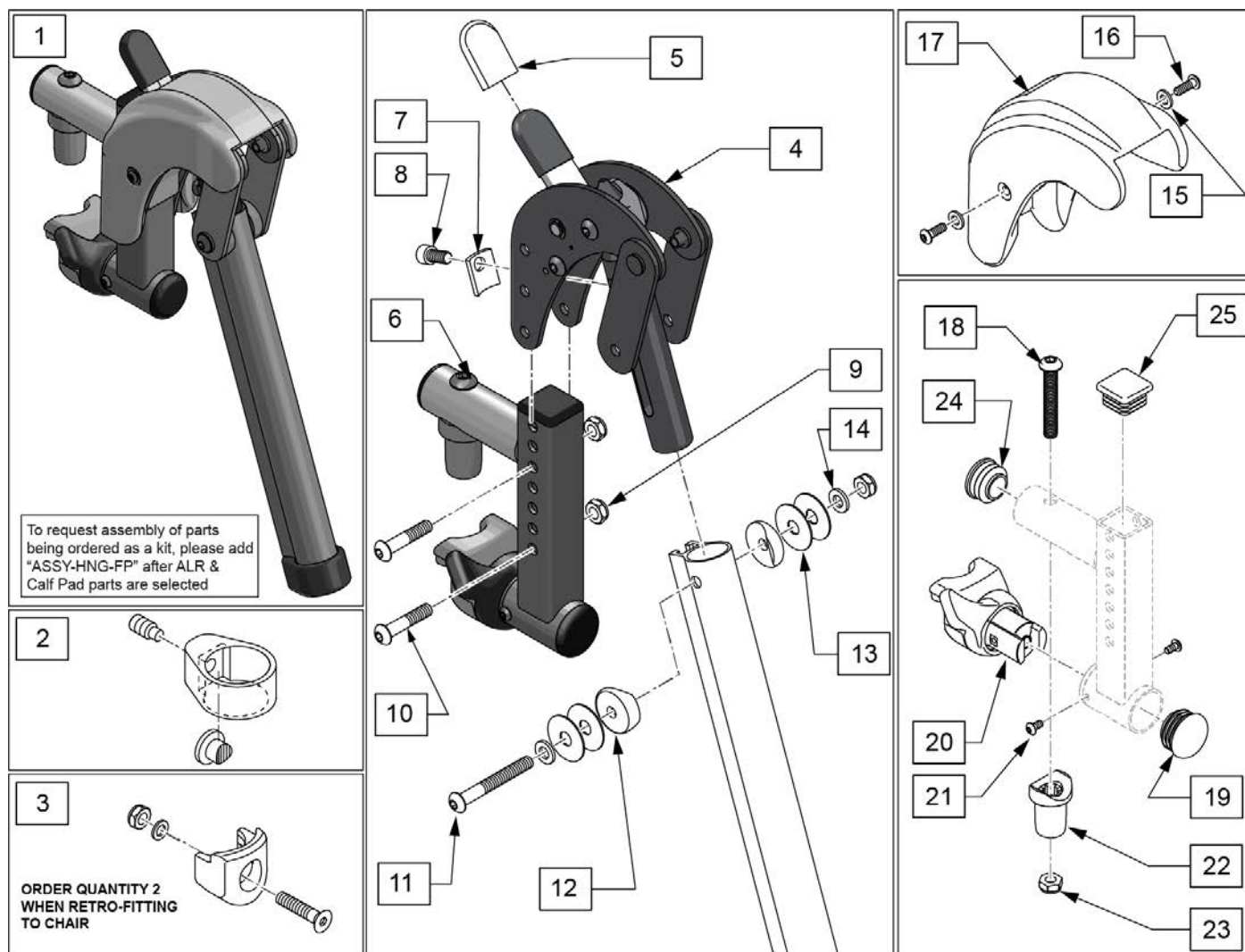


Pos.	Item Number	Description	Remarks
1	161244S	SWNG I/O ALR ASSM RH KIT	Right
1	161245S	SWNG I/O ALR ASSM LH KIT	Left
1	161246S	SWNG I/O ALR ASSM PR KIT	Pair
2	119495	EXTENSION CLAMP KIT	
3	119514	LATCH RECEIVER KIT	order qty 2 when retro-fitting to chair
4	127153	HEMI ALR SUB ASSM	
5	560501	ALR REL LEVER COVER	
6	134188	RECEIVER ASSY ELR ALR RH	
6	134186	RECEIVER ASSY ELR ALR LH	
7	127129	ANGLE STOP ELR	
8	107350	SHCS 1/4-28 x 1/2	
9	106935	1/4-28 NYLOCK NUT THIN BLK ZN	
10	108623	BHCS 1/4-28 x 1-1/2	
11	106964	BHCS 1/4-28 X 2	
12	561086	1" HALF SADDLE 1/4" HOLE	
13	113268	WASHER NYLON 1IN OD X .312 ID	
14	106943	WASHER FLAT AN 1/4 ROHS	
15	113282	WASHER FLAT AN #10 ROHS	
16	127135	BHCS 8-32 X 1/2 BLK	

Pos.	Item Number	Description	Remarks
15-17	129633	ALR SHROUD KIT	1 side
18	226217	5/16-18 X 2" BH C/S BLK	
19	560110	ENDPLUG BACKREST TNT	
20,21	161242	ELR/ALR HANGER LATCH KIT RH	Right
20,21	161243	ELR/ALR HANGER LATCH KIT LH	Left
22	561197	PLASTIC PIVOT SADDLE	
23	202102	5/16-18 NYLOCK NUT THIN	
24	561062	1" INSERT PLUG .065 RIBS	
25	010615	SQUARE END CAP CNTR LEG	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

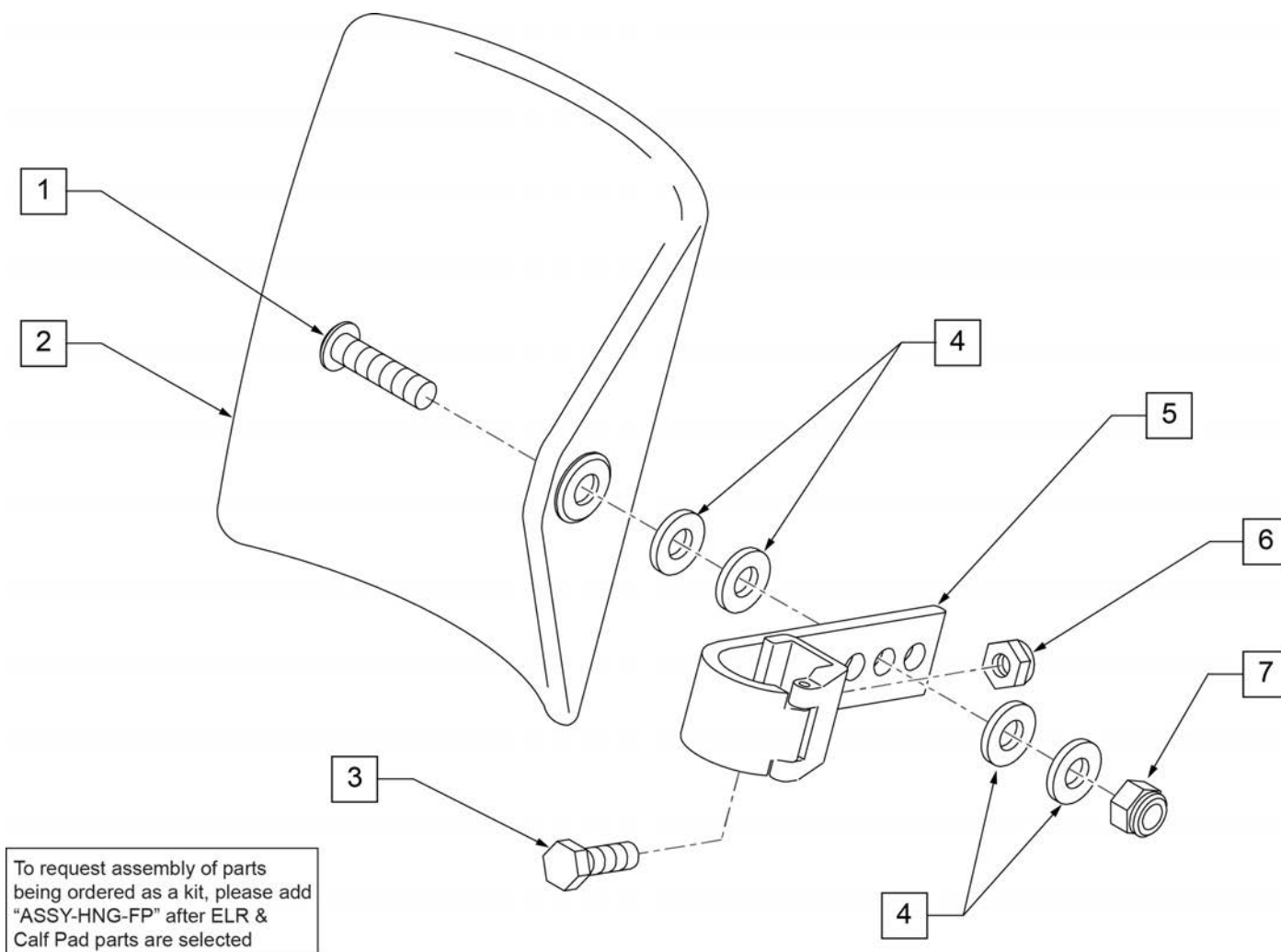


Pos.	Item Number	Description	Remarks
1	106980	BHCS 5/16-18 X 1-1/4 ROHS	
2	755014	CALF PAD TROUGH STYLE 14"/16"	
3	221097	1/4-28 X 5/8 HEX C/S BZ	
4	561016	1" WASHER 5/16" HOLE	
5	881413	CALF PAD BRACKET ASSEMBLY	
6	106934	NUT NYLOCK 1/4-28 ROHS	
7	106937	NUT NYLOCK THIN 5/16-18	
1-7	881414	CALF PAD J-HOOK 14"-15" ASSEMBLY	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

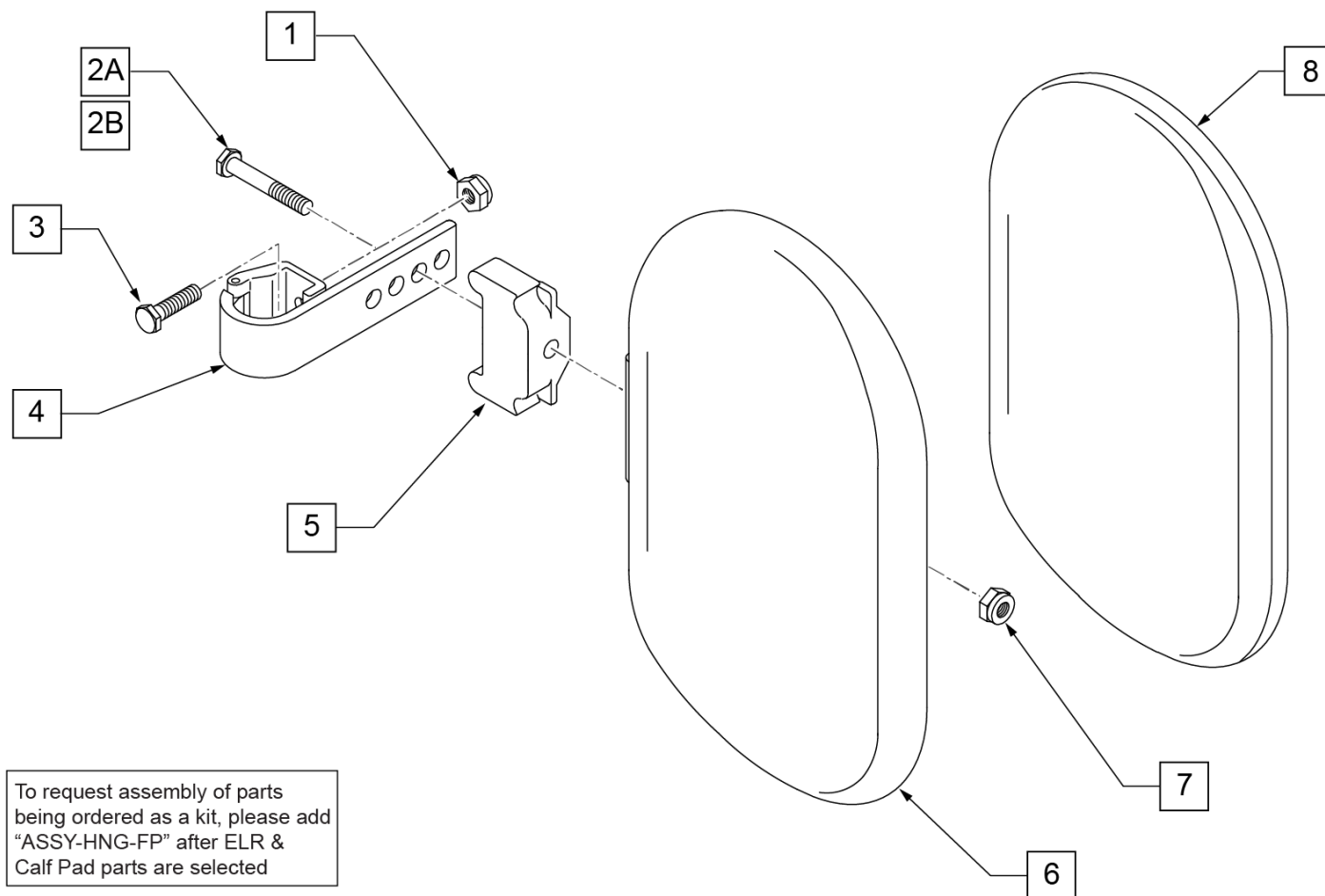

Pos.	Item Number	Description	Remarks
1	106934	NUT NYLOCK 1/4-28 ROHS	
2A	107018	HHCS 5/16-18 X 2-1/2 ROHS	16"-17" wide only
2B	107019	HHCS 5/16-18 X 3-1/2 ROHS	18"-20" wide only
3	107013	HHCS 1/4-28 X 5/8 ROHS	
4	881413	CALF PAD BRACKET ASSEMBLY	
5	560199	ELR CALF HINGE SPACER	18"-20" wide only
6	560108	CALF PAD MOLDED 16"-20"	
7	106936	NUT NYLOCK 5/16-18 ROHS	
8	751045	CALF PAD COVER	
1,2A,3,4,6-8	881416	CALF MOLDED 16"-17" ASSEMBLY	
1,2B,3-7,8	881412	CALF PAD MOLDED 18"-20" ASSEMBLY	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

Note: For installation instructions please see the informations section to the right of this parts page.

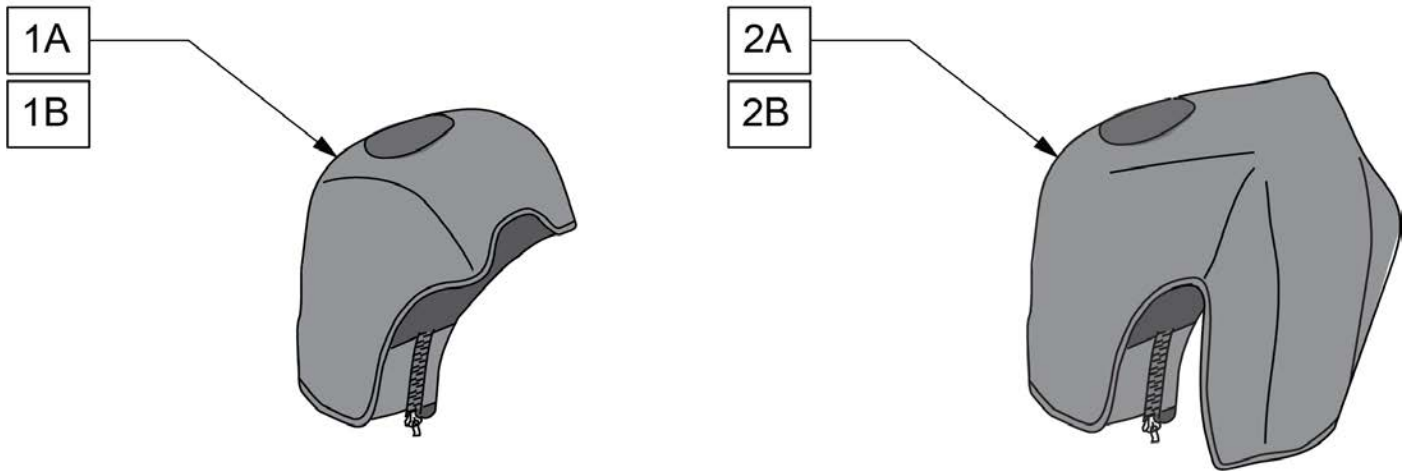

Pos.	Item Number	Description	Remarks
1A	163881	ELR NEOPRENE COVER PR	Pair
1B	163882	ELR GEL COVER PR	Pair
2A	163883	ALR NEOPRENE COVER PR	Pair
2B	163884	ALR GEL COVER PR	Pair

Note: Footplate Assemblies are set to footrest width; please see "Footplate Formula" table for seat width conversion.

Note: If using Short Ext Tube & Hemi Hanger with Composite Footplate use 1G (bent) .

Note: V Style Hangers are not available on Swing Away Style chairs effective 7/14/14; see the "Prior To" ordering guide for V Style footplate assemblies.

Note: see Informations Panel for appropriate ordering guide; for QX Series or SR45 footplate replacement parts see ordering guide marked specifically for these two models.

To have complete Footrest ship assembled, choose compatible Hanger + Footplate part numbers and add "ASSY-HNG-FP" to order

FOOTPLATE MEASURING GUIDE

PART NO	DESCRIPTION	"A"	"B"
063216-006	FOOTPLATE COMPOSITE 157 R (13"W)	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L (13"W)	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R (14"W)	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L (14"W)	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R (15"W)	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L (15"W)	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R (16"W)	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L (16"W)	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R (17"W)	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L (17"W)	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R (18"W)	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L (18"W)	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R (19-20"W)	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L (19-20"W)	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R (21-22"W)	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L (21-22"W)	10 1/8"	6 1/4"

INSET FORMULA
0" Inset (Seat Width) = Footrest Width
1" Inset (Seat Width) -2" = Footrest Width
"V" Hanger (Seat Width) -4" = Footrest Width
Offset Hanger Receiver (Seat Width) +2" = Footrest Width

Pos.	Item Number	Description	Remarks
1A,3-5	120660	SHT STRT EXT COMP HDWR	80 deg hangers + ELR only

Pos.	Item Number	Description	Remarks
1B,3-5	120661	MED STRT EXT COMP HDWR	80 deg hangers + ELR only
1C,3-5	120662	LNG STRT EXT COMP HDWR	80 deg hangers + ELR only
1D,3-5	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E,3-5	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F,3-5	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G	128078	TUBE FOOTREST EXT SHT HEMI	use for Hemi Ext Tube option
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 13" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 13" Footrest Width
2B	063216-015	FOOTPLATE COMPOSITE RIGHT	RIGHT 14" Footrest Width
2B	063216-018	FOOTPLATE COMPOSITE LEFT	LEFT 14" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2D	063216-014	FOOTPLATE COMPOSITE RIGHT	RIGHT 16" Footrest Width
2D	063216-017	FOOTPLATE COMPOSITE LEFT	LEFT 16" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2F	063216-013	FOOTPLATE COMPOSITE RIGHT	RIGHT 18" Footrest Width
2F	063216-016	FOOTPLATE COMPOSITE LEFT	LEFT 18" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" & 20" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 19" & 20" Footrest Width
2H	063216-002	FOOTPLATE COMPOSITE RIGHT	RIGHT 21" & 22" Footrest Width
2H	063216-008	FOOTPLATE COMPOSITE LEFT	LEFT 21" & 22" Footrest Width
3	104377	3/4" SADDLE THIN	
4	103175	SHCS M6 X 1 X 40 INORG	
5	132626	NUT AUTOMATION M6 X 1.0	Inserted in bottom of footplate.
6	104328	HEEL LOOP PEG	
7A	104380-001	ADJUSTABLE FOOT REST STRAP	14" wide
7B	104380-002	ADJUSTABLE FOOT REST STRAP	15-16" wide
7C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" wide
7D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" wide
8	102268-001	HHCS M6 X 1 X 25	
12	104160	BUSHING FOOTREST	
9-12	107485	FOOTREST CLAMP ASSM	
>>>>		80 DEGREE HANGERS - SHORT EXT - RIGHT	
1A,2A,3-6,7A,8	107704	COMP STRT SHT EXT ASM 13"W R	Right- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107705	COMP STRT SHT EXT ASM 14"W R	Right- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107706	COMP STRT SHT EXT ASM 15"W R	Right- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107707	COMP STRT SHT EXT ASM 16"W R	Right- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107708	COMP STRT SHT EXT ASM 17"W R	Right- 80D- 17" W- SHORT
1A,2F,3-6,7C,8	107709	COMP STRT SHT EXT ASM 18"W R	Right- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107710	COMP STRT SHT EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111522	COMP STRT SHT EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - SHORT EXT - LEFT	
1A,2A,3-6,7A,8	107711	COMP STRT SHT EXT ASM 13"W L	LEFT- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107712	COMP STRT SHT EXT ASM 14"W L	LEFT- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107713	COMP STRT SHT EXT ASM 15"W L	LEFT- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107714	COMP STRT SHT EXT ASM 16"W L	LEFT- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107715	COMP STRT SHT EXT ASM 17"W L	LEFT- 80D- 17" W- SHORT
1A,2F,3-6,7C,8	107716	COMP STRT SHT EXT ASM 18"W L	LEFT- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107717	COMP STRT SHT EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111523	COMP STRT SHT EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - MED EXT - RIGHT	
1B,2A,3-6,7A,8	107718	COMP STRT MED EXT ASM 13"W R	Right- 80D- 13" W- MED
1B,2B,3-6,7A,8	107719	COMP STRT MED EXT ASM 14"W R	Right- 80D- 14" W- MED
1B,2C,3-6,7B,8	107720	COMP STRT MED EXT ASM 15"W R	Right- 80D- 15" W- MED
1B,2D,3-6,7B,8	107721	COMP STRT MED EXT ASM 16"W R	Right- 80D- 16" W- MED
1B,2E,3-6,7C,8	107722	COMP STRT MED EXT ASM 17"W R	Right- 80D- 17" W- MED

Pos.	Item Number	Description	Remarks
1B,2F,3-6,7C,8	107723	COMP STRT MED EXT ASM 18"W R	Right- 80D- 18" W- MED
1B,2G,3-6,7D,8	107724	COMP STRT MED EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111524	COMP STRT MED EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - MED EXT - LEFT	
1B,2A,3-6,7A,8	107725	COMP STRT MED EXT ASM 13"W L	LEFT- 80D- 13" W- MED
1B,2B,3-6,7A,8	107726	COMP STRT MED EXT ASM 14"W L	LEFT- 80D- 14" W- MED
1B,2C,3-6,7B,8	107727	COMP STRT MED EXT ASM 15"W L	LEFT- 80D- 15" W- MED
1B,2D,3-6,7B,8	107728	COMP STRT MED EXT ASM 16"W L	LEFT- 80D- 16" W- MED
1B,2E,3-6,7C,8	107729	COMP STRT MED EXT ASM 17"W L	LEFT- 80D- 17" W- MED
1B,2F,3-6,7C,8	107730	COMP STRT MED EXT ASM 18"W L	LEFT- 80D- 18" W- MED
1B,2G,3-6,7D,8	107731	COMP STRT MED EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111525	COMP STRT MED EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - LONG EXT - RIGHT	
1C,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	Right- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107733	COMP STRT LONG EXT ASM 14"W R	Right- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	Right- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107735	COMP STRT LONG EXT ASM 16"W R	Right- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	Right- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107737	COMP STRT LONG EXT ASM 18"W R	Right- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111526	COMP STRT LONG EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- LONG
>>>>		80 DEGREE HANGERS - LONG EXT - LEFT	
1C,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	LEFT- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107740	COMP STRT LONG EXT ASM 14"W L	LEFT- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	LEFT- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107742	COMP STRT LONG EXT ASM 16"W L	LEFT- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	LEFT- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107744	COMP STRT LONG EXT ASM 18"W L	LEFT- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111527	COMP STRT LONG EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- LONG
>>>>		60°,70° SHORT EXT - RIGHT	
1D,2A,3-6,7A,8	107746	COMP BNT SHT EXT ASM 13" W R	Right- 70D- 13" W- SHORT
1D,2B,3-6,7A,8	107747	COMP BNT SHT EXT ASM 14" W R	Right- 70D- 14" W- SHORT
1D,2C,3-6,7B,8	107748	COMP BNT SHT EXT ASM 15" W R	Right- 70D- 15" W- SHORT
1D,2D,3-6,7B,8	107749	COMP BNT SHT EXT ASM 16" W R	Right- 70D- 16" W- SHORT
1D,2E,3-6,7C,8	107750	COMP BNT SHT EXT ASM 17" W R	Right- 70D- 17" W- SHORT
1D,2F,3-6,7C,8	107751	COMP BNT SHT EXT ASM 18" W R	Right- 70D- 18" W- SHORT
1D,2G,3-6,7D,8	107752	COMP BNT SHT EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111528	COMP BNT SHT EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- SHORT
>>>>		60°,70° - SHORT EXT - LEFT	
1D,2A,3-6,7A,8	107753	COMP BNT SHT EXT ASM 13" W L	Left- 70D- 13 W- SHORT
1D,2B,3-6,7A,8	107754	COMP BNT SHT EXT ASM 14" W L	Left- 70D- 14 W- SHORT
1D,2C,3-6,7B,8	107755	COMP BNT SHT EXT ASM 15" W L	Left- 70D- 15 W- SHORT
1D,2D,3-6,7B,8	107756	COMP BNT SHT EXT ASM 16" W L	Left- 70D- 16 W- SHORT
1D,2E,3-6,7C,8	107757	COMP BNT SHT EXT ASM 17" W L	Left- 70D- 17 W- SHORT
1D,2F,3-6,7C,8	107758	COMP BNT SHT EXT ASM 18" W L	Left- 70D- 18 W- SHORT
1D,2G,3-6,7D,8	107759	COMP BNT SHT EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111529	COMP BNT SHT EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- SHORT
>>>>		60°,70°,60°&70° HEMI HANGERS - MED EXT - RIGHT	
1E,2A,3-6,7A,8	107760	COMP BNT MED EXT ASM 13" W R	Right- 70D- 13" W- MED
1E,2B,3-6,7A,8	107761	COMP BNT MED EXT ASM 14" W R	Right- 70D- 14" W- MED
1E,2C,3-6,7B,8	107762	COMP BNT MED EXT ASM 15" W R	Right- 70D- 15" W- MED
1E,2D,3-6,7B,8	107763	COMP BNT MED EXT ASM 16" W R	Right- 70D- 16" W- MED
1E,2E,3-6,7C,8	107764	COMP BNT MED EXT ASM 17" W R	Right- 70D- 17" W- MED
1E,2F,3-6,7C,8	107765	COMP BNT MED EXT ASM 18" W R	Right- 70D- 18" W- MED
1E,2G,3-6,7D,8	107766	COMP BNT MED EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111530	COMP BNT MED EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- MED

Pos.	Item Number	Description	Remarks
>>>>		60°,70°,60°&70° HEMI HANGERS - MED EXT - LEFT	
1E,2A,3-6,7A,8	107767	COMP BNT MED EXT ASM 13" W L	Left- 70D- 13 W- MED
1E,2B,3-6,7A,8	107768	COMP BNT MED EXT ASM 14" W L	Left- 70D- 14 W- MED
1E,2C,3-6,7B,8	107769	COMP BNT MED EXT ASM 15" W L	Left- 70D- 15 W- MED
1E,2D,3-6,7B,8	107770	COMP BNT MED EXT ASM 16" W L	Left- 70D- 16 W- MED
1E,2E,3-6,7C,8	107771	COMP BNT MED EXT ASM 17" W L	Left- 70D- 17 W- MED
1E,2F,3-6,7C,8	107772	COMP BNT MED EXT ASM 18" W L	Left- 70D- 18 W- MED
1E,2G,3-6,7D,8	107773	COMP BNT MED EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111531	COMP BNT MED EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- MED
>>>>		60°,70°,60°&70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-6,7A,8	107774	COMP BNT LONG EXT ASM 13" W R	Right- 70D- 13" W- LONG
1F,2B,3-6,7A,8	107775	COMP BNT LONG EXT ASM 14" W R	Right- 70D- 14" W- LONG
1F,2C,3-6,7B,8	107776	COMP BNT LONG EXT ASM 15" W R	Right- 70D- 15" W- LONG
1F,2D,3-6,7B,8	107777	COMP BNT LONG EXT ASM 16" W R	Right- 70D- 16" W- LONG
1F,2E,3-6,7C,8	107778	COMP BNT LONG EXT ASM 17" W R	Right- 70D- 17" W- LONG
1F,2F,3-6,7C,8	107779	COMP BNT LONG EXT ASM 18" W R	Right- 70D- 18" W- LONG
1F,2G,3-6,7D,8	107780	COMP BNT LONG EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111532	COMP BNT LONG EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- LONG
>>>>		60°,70°,60°&70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-6,7A,8	107781	COMP BNT LONG EXT ASM 13" W L	Left- 70D- 13 W- LONG
1F,2B,3-6,7A,8	107782	COMP BNT LONG EXT ASM 14" W L	Left- 70D- 14 W- LONG
1F,2C,3-6,7B,8	107783	COMP BNT LONG EXT ASM 15" W L	Left- 70D- 15 W- LONG
1F,2D,3-6,7B,8	107784	COMP BNT LONG EXT ASM 16" W L	Left- 70D- 16 W- LONG
1F,2E,3-6,7C,8	107785	COMP BNT LONG EXT ASM 17" W L	Left- 70D- 17 W- LONG
1F,2F,3-6,7C,8	107786	COMP BNT LONG EXT ASM 18" W L	Left- 70D- 18 W- LONG
1F,2G,3-6,7D,8	107787	COMP BNT LONG EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111533	COMP BNT LONG EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- LONG
>>>>		60°&70° HEMI HANGER SHORT EXT- RIGHT	
1G,2A,3-6,7A,8	129724	CMP FT BNT KIT SHT HNGR 13W R	Right-Hemi-13W; add Ext behind the Hemi
1G,2B,3-6,7A,8	129726	CMP FT BNT KIT SHT HNGR 14W R	Right-Hemi-14W; add Ext behind the Hemi
1G,2C,3-6,7B,8	129728	CMP FT BNT KIT SHT HNGR 15W R	Right-Hemi-15W; add Ext behind the Hemi
1G,2D,3-6,7B,8	129730	CMP FT BNT KIT SHT HNGR 16W R	Right-Hemi-16W; add Ext behind the Hemi
1G,2E,3-6,7C,8	129732	CMP FT BNT KIT SHT HNGR 17W R	Right-Hemi-17W; add Ext behind the Hemi
1G,2F,3-6,7C,8	129734	CMP FT BNT KIT SHT HNGR 18W R	Right-Hemi-18W; add Ext behind the Hemi
1G,2G,3-6,7D,8	129736	CMP FT BNT KIT SHT HNGR 19-20W R	Right-Hemi-19-20W; add Ext behind the Hemi
1G,2H,3-6,7D,8	129738	CMP FT BNT KIT SHT HNGR 21-22W R	Right-Hemi-21-22W; add Ext behind the Hemi
>>>>		60°&70° HEMI HANGER SHORT EXT- LEFT	
1G,2A,3-6,7A,8	129725	CMP FT BNT KIT SHT HNGR 13W L	Left-Hemi-13W; add Ext behind the Hemi
1G,2B,3-6,7A,8	129727	CMP FT BNT KIT SHT HNGR 14W L	Left-Hemi-14W; add Ext behind the Hemi
1G,2C,3-6,7B,8	129729	CMP FT BNT KIT SHT HNGR 15W L	Left-Hemi-15W; add Ext behind the Hemi
1G,2D,3-6,7B,8	129731	CMP FT BNT KIT SHT HNGR 16W L	Left-Hemi-16W; add Ext behind the Hemi
1G,2E,3-6,7C,8	129733	CMP FT BNT KIT SHT HNGR 17W L	Left-Hemi-17W; add Ext behind the Hemi
1G,2F,3-6,7C,8	129735	CMP FT BNT KIT SHT HNGR 18W L	Left-Hemi-18W; add Ext behind the Hemi
1G,2G,3-6,7D,8	129737	CMP FT BNT KIT SHT HNGR 19-20W L	Left-Hemi-19-20W; add Ext behind the Hemi
1G,2H,3-6,7D,8	129739	CMP FT BNT KIT SHT HNGR 21-22W L	Left-Hemi-21-22W; add Ext behind the Hemi
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

Note: Footplate Assemblies are set to footrest width; please see "Footplate Formula" table for seat width conversion.


Note: If using Short Ext Tube & Hemi Hanger with Composite Angle Adjustable Footplate use 1D (straight) .

Note: V Style Hangers are not available on Swing Away Style chairs effective 7/14/14; see the "Prior To" ordering guide for V Style footplate assemblies.


To have complete Footrest ship assembled, choose compatible Hanger + Footplate part numbers and add "ASSY-HNG-FP" to order

ADD INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2" = Footrest Width
 2" Inset (Seat Width) -4" = Footrest Width
 "V" Hanger (Seat Width) -4" = Footrest Width
 Offset Hanger Receiver (Seat Width) +2" = Footrest Width
 Flared Frame (Seat Width) = 2" = Footrest Width

FOOTPLATE MEASURING GUIDE

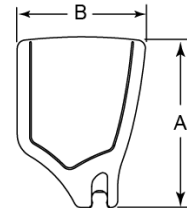

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

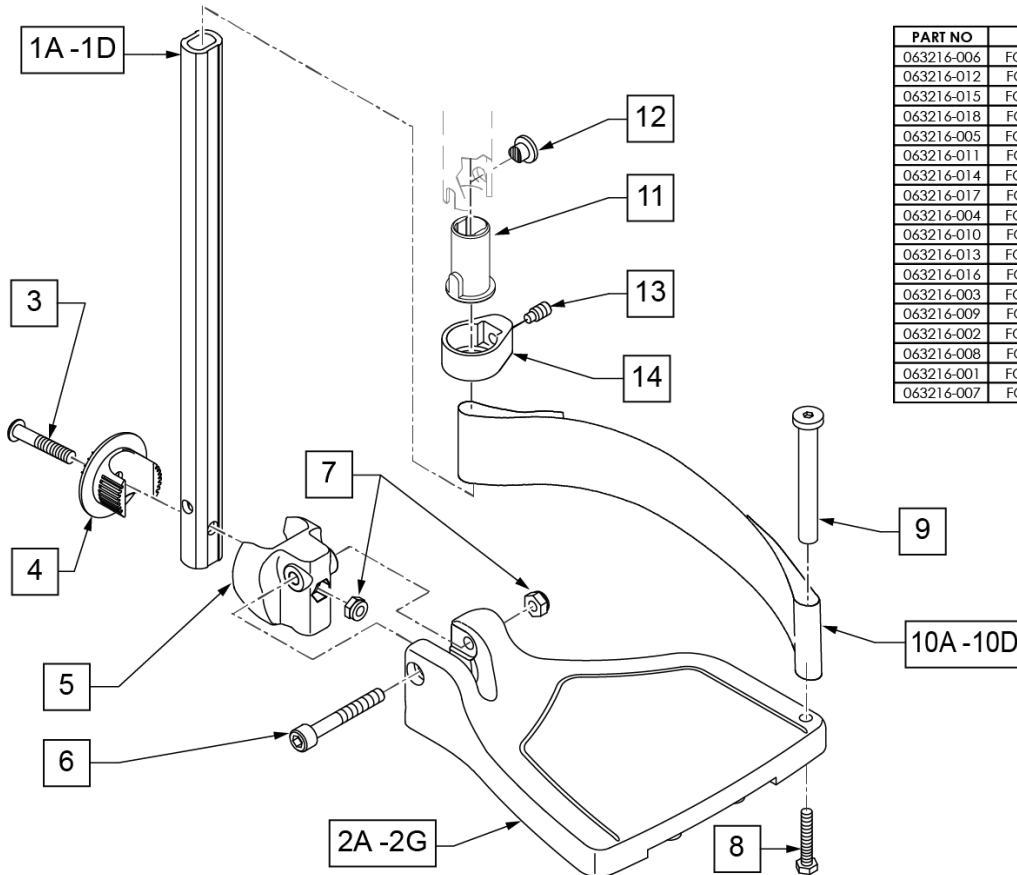

Pos.	Item Number	Description	Remarks
1A,6,7	120660	SHT STRT EXT COMP HDWR	
1B,6,7	120661	MED STRT EXT COMP HDWR	
1C,6,7	120662	LNG STRT EXT COMP HDWR	
1D	128079	TUBE FTRST EXT STRGHT SHT HEMI	Use for Hemi Ext Tube option
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width

Pos.	Item Number	Description	Remarks
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2B	063216-015	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width
2B	063216-018	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2D	063216-014	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width
2D	063216-017	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width
2F	063216-013	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width
2F	063216-016	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width
3	108233	BHCS M6 X 1 X 40	
4	054242	ANGLE COVER	
5	054241	ANGLE SEGMENT	
6	103175	SHCS M6 X 1 X 40 INORG	
7	132626	NUT AUTOMATION M6 X 1.0	
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.
9	104328	HEEL LOOP PEG	
10A	104380-001	ADJUSTABLE FOOT REST STRAP	15" Footrest Width
10B	104380-002	ADJUSTABLE FOOT REST STRAP	16" Footrest Width
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width
11	104160	BUSHING FOOTREST	
11-14	107485	FOOTREST CLAMP ASSM	
>>>>		60,70,80 HANGERS - SHORT EXT - RIGHT	
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 15 W- SHORT
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 16 W- SHORT
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 17 W- SHORT
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 18 W- SHORT
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 19 W- SHORT
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 20 W- SHORT
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 21&22 W- SHORT
>>>>		60,70,80 HANGERS - SHORT EXT - LEFT	
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 15 W- SHORT
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 16 W- SHORT
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 17 W- SHORT
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 18 W- SHORT
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 19 W- SHORT
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 20 W- SHORT
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 21&22 W- SHORT
>>>>		60,70,80& 60,70 HEMI HANGER- RIGHT	
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 15 W- MED
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 16 W- MED
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 17 W- MED
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 18 W- MED
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 19 W- MED
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 20 W- MED
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 21&22 W- MED
>>>>		60,70,80& 60,70 HEMI HANGER- LEFT	
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 15 W- MED
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 16 W- MED
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 17 W- MED
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 18 W- MED
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 19 W- MED
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 20 W- MED

Pos.	Item Number	Description	Remarks
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 21&22 W- MED
>>>>		60,70,80& 60,70 HEMI HANGER- RIGHT	
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 15 W- LONG
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 16 W- LONG
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 17 W- LONG
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 18 W- LONG
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 19 W- LONG
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 20 W- LONG
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 21&22 W- LONG
>>>>		60,70,80& 60,70 HEMI HANGER- LEFT	
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 15 W- LONG
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 16 W- LONG
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 17 W- LONG
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 18 W- LONG
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 19 W- LONG
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 20 W- LONG
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 21&22 W- LONG
>>>>		60&70 HEMI HANGERS- SHORT EXT- RIGHT	
1D,2A,3-10A	129710	CAA FOOT KIT SHT HNGR 15W R	Right-Hemi-15W; add Ext behind the Hemi
1D,2B,3-10B	129712	CAA FOOT KIT SHT HNGR 16W R	Right-Hemi-16W; add Ext behind the Hemi
1D,2C,3-10B	129714	CAA FOOT KIT SHT HNGR 17W R	Right-Hemi-17W; add Ext behind the Hemi
1D,2D,3-10C	129716	CAA FOOT KIT SHT HNGR 18W R	Right-Hemi-18W; add Ext behind the Hemi
1D,2E,3-10C	129718	CAA FOOT KIT SHT HNGR 19W R	Right-Hemi-19W; add Ext behind the Hemi
1D,2F,3-10D	129720	CAA FOOT KIT SHT HNGR 20W R	Right-Hemi-20W; add Ext behind the Hemi
1D,2G,3-10D	129722	CAA FOOT KIT SHT HNGR 22W R	Right-Hemi-21&22W; add Ext behind the Hemi
>>>>		60&70 HEMI HANGERS- SHORT EXT - LEFT	
1D,2A,3-10A	129711	CAA FOOT KIT SHT HNGR 15W L	Left-Hemi-15W; add Ext behind the Hemi
1D,2B,3-10B	129713	CAA FOOT KIT SHT HNGR 16W L	Left-Hemi-16W; add Ext behind the Hemi
1D,2C,3-10B	129715	CAA FOOT KIT SHT HNGR 17W L	Left-Hemi-17W; add Ext behind the Hemi
1D,2D,3-10C	129717	CAA FOOT KIT SHT HNGR 18W L	Left-Hemi-18W; add Ext behind the Hemi
1D,2E,3-10C	129719	CAA FOOT KIT SHT HNGR 19W L	Left-Hemi-19W; add Ext behind the Hemi
1D,2F,3-10D	129721	CAA FOOT KIT SHT HNGR 20W L	Left-Hemi-20W; add Ext behind the Hemi
1D,2G,3-10D	129723	CAA FOOT KIT SHT HNGR 22W L	Left-Hemi-21&22W; add Ext behind the Hemi
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

Note: Footplate Assemblies are set to footrest width; please see "Footplate Formula" table for seat width conversion.


Note: If using Short Ext Tube & Hemi Hanger with Adult Angle Adjustable Footplate use 1D (straight) .

Note: V Style Hangers are not available on Swing Away Style chairs effective 7/14/14; see the "Prior To" ordering guide for V Style footplate assemblies.

Note: for QX Series or SR45 footplate replacement parts see the ordering guide marked specifically for these two models in the informations panel.

Note: if chair has High Mount Ext Tubes or if switching to High Mount Ext Tubes, see AAA Footplate w/ High Mount Extension Ordering Guide.

MEASURING GUIDE

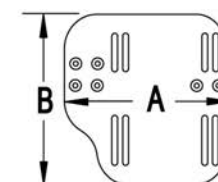


CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

21 EXT MNT HARDWARE 1 SIDE			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
3		NUT AUTOMATION M6 X 1.0 ROHS	1
4		WASHER FLAT THIN AN 1/4	2
5		HANGER EXTENSION SADDLE	1
6		3/4" SADDLE 1" OD	1
8		WASHER FLAT SAE 1/4 ROHS	1
9		SHCS M6X1X45 12MM PART THD	1

To have complete Footrest ship assembled, choose compatible Hanger + Footplate part numbers and add "ASSY-HNG-FP" to order

ADD INSET FORMULA

0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2" = Footrest Width
 2" Inset (Seat Width) -4" = Footrest Width
 "V" Hanger (Seat Width) -4" = Footrest Width
 Offset Hanger Receiver (Seat Width) +2" = Footrest Width
 Flared Frame (Seat Width) = 2" = Footrest Width

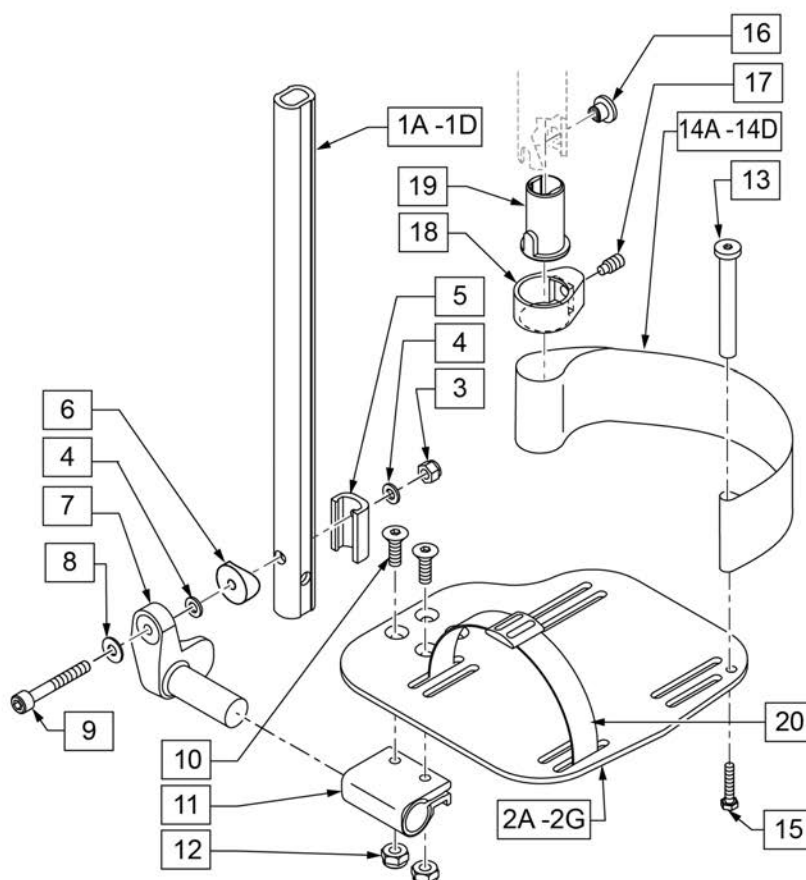


Pos.	Item Number	Description	Remarks
1A,3-6,8,9	120670	SHT STRT EXT AAA HDWR	
1B,3-6,8,9	120671	MED STRT EXT AAA HDWR	
1C,3-6,8,9	120672	LNG STRT EXT AAA HDWR	
1D	128079	TUBE FTRST EXT STRGHT SHT HEMI	Use for Hemi Ext Tube option
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width
3	132626	NUT AUTOMATION M6 X 1.0	
4	106944	WASHER FLAT THIN AN 1/4 ROHS	
5	107199	HANGER EXTENSION SADDLE	
6	561055	3/4" SADDLE 1" OD	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT

Pos.	Item Number	Description	Remarks
8	106945	WASHER FLAT SAE 1/4 ROHS	
9	104399	SHCS M6X1X45 12 MM PART THD	
10	106978	FHCS 1/4-28 X 7/8 ROHS	
11	550350	3/4" CLMP ADJ FT PLT	
12	106935	NUT NYLOCK THIN 1/4-28 ROHS	
13	104328	POST HEEL LOOP	
14A	104380-001	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE
14B	104380-002	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE
15	102268-001	HHCS M6 X 1 X 25	
16-19	107485	FOOTREST CLAMP ASSM	
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
21	120659	EXT MNT HARDWARE 1 SIDE	
>>>>		60,70,80 STD HANGERS - RIGHT	
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 12-13" W- SHORT
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 14-15" W- SHORT
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 16-17" W- SHORT
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 18-22" W- SHORT
>>>>		60,70,80 STD HANGERS - LEFT	
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 12-13" W- SHORT
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 14-15" W- SHORT
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 16-17" W- SHORT
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 18-22" W- SHORT
>>>>		60,70,80 & 60,70 HEMI HANGERS - RIGHT	
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 12-13" W- MED
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 14-15" W- MED
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 16-17" W- MED
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 18-22" W- MED
>>>>		60,70,80 & 60,70 HEMI HANGERS - LEFT	
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 12-13" W- MED
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 14-15" W- MED
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 16-17" W- MED
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 18-22" W- MED
>>>>		60,70,80 & 60,70 HEMI HANGERS - RIGHT	
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 12-13" W- LONG
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14 -15" W R	Right- 14-15" W- LONG
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16 -17" W R	Right- 16-17" W- LONG
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18 -22" W R	Right- 18-22" W- LONG
>>>>		60,70,80 & 60,70 HEMI HANGERS - LEFT	
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 12-13" W- LONG
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 14-15" W- LONG
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 16-17" W- LONG
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 18-22" W- LONG
>>>>		60&70 HEMI HANGERS - SHORT EXT - RIGHT	
1D,2B,3-14A,15	129756	AAA FOOT KIT SHT HNGR 12W R	Right- 12-13" W- SHORT; add Ext behind the Hemi
1D,2C,3-14B,15	129758	AAA FOOT KIT SHT HNGR 14W R	Right- 14-15" W- SHORT; add Ext behind the Hemi
1D,2D,3-14C,15	129760	AAA FOOT KIT SHT HNGR 16W R	Right- 16-17" W- SHORT; add Ext behind the Hemi
1D,2E,3-14D,15	129762	AAA FOOT KIT SHT HNGR 18W R	Right- 18-22" W- SHORT; add Ext behind the Hemi
>>>>		60&70 HEMI HANGERS - SHORT EXT -LEFT	
1D,2B,3-14A,15	129757	AAA FOOT KIT SHT HNGR 12W L	Left- 12-13" W-SHORT; add Ext behind the Hemi
1D,2C,3-14B,15	129759	AAA FOOT KIT SHT HNGR 14W L	Left- 14-15" W- SHORT; add Ext behind the Hemi
1D,2D,3-14C,15	129761	AAA FOOT KIT SHT HNGR 16W L	Left- 16-17" W- SHORT; add Ext behind the Hemi
1D,2E,3-14D,15	129763	AAA FOOT KIT SHT HNGR 18W L	Left- 18-22" W- SHORT; add Ext behind the Hemi
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

To request assembly of parts being ordered as a kit, please add "ASSY-HNG-FP" after ELR or ALR & Calf Pad parts are selected

ADD INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2" = Footrest Width
 2" Inset (Seat Width) -4" = Footrest Width
 "V" Hanger (Seat Width) -4" = Footrest Width
 Offset Hanger Receiver (Seat Width) +2" = Footrest Width
 Flared Frame (Seat Width) = 2" = Footrest Width

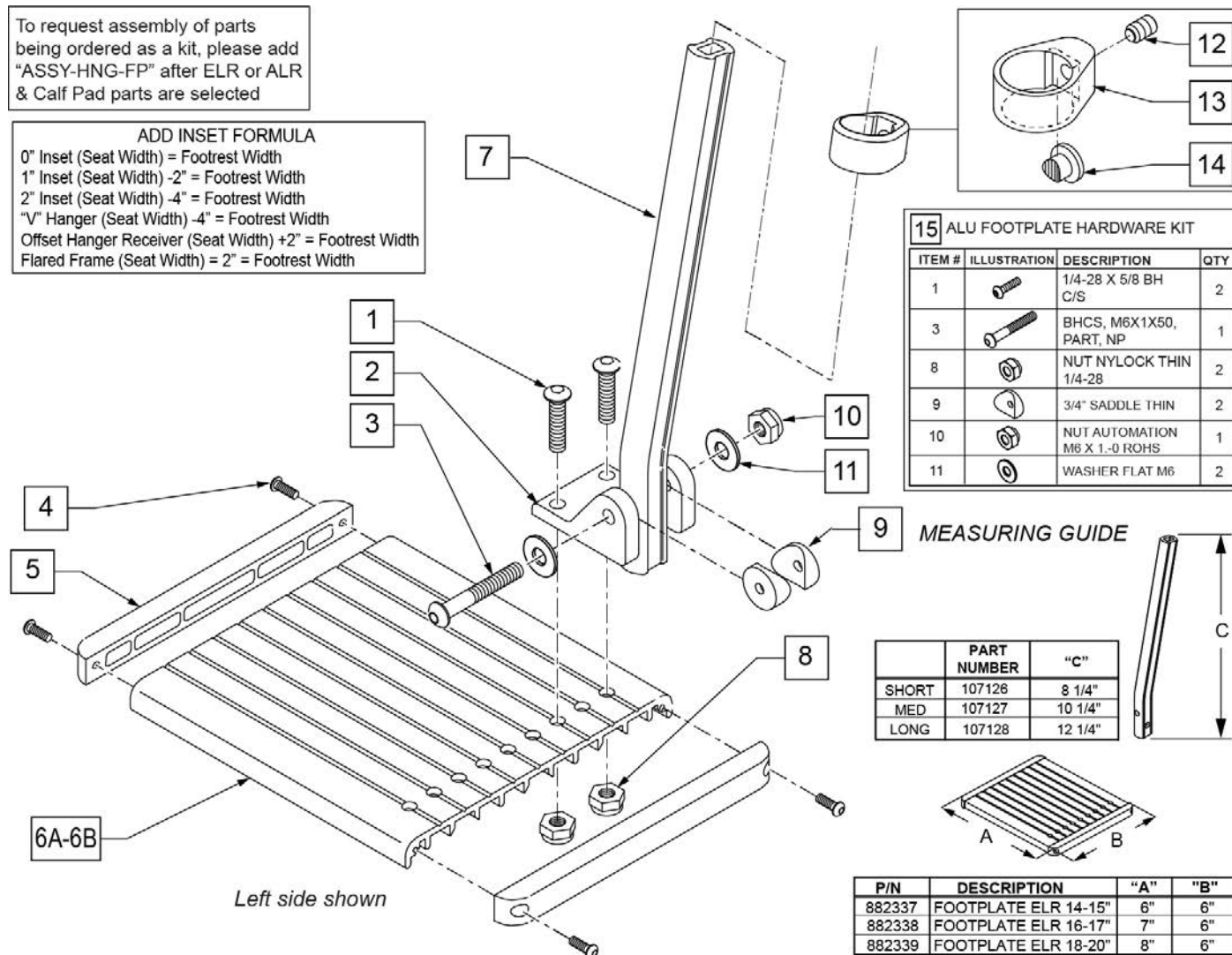

Pos.	Item Number	Description	Remarks
1	226090	1/4-28 x 5/8 BH C/S	
2	118093	BRACKET FOOTPLATE ALUM	
3	102262-008	BHCS M6 X 1 X 50 INORGANIC	
4	281043	#4 x 5/8 PAN PHIL S/TAP	
5	560101	END CAPS FOOTREST	
7	107126	SHORT BENT	
7	107127	MED BENT	
7	107128	LONG BENT	
7	128078	TUBE FOOTREST EXT SHT HEMI	Use for Hemi Extension option
8	106935	1/4-28 NYLOCK NUT THIN	
9	561003	3/4" SADDLE THIN	
10	132626	NUT AUTOMATION M6 X 1.0	
11	107383	WASHER FLAT M6 ROHS	
4,5,6A	882337	ELR FOOTPLATE ASSEMBLY 14 -15"	
4,5,6B	882338	ELR FOOTPLATE ASSEMBLY 16 -17"	
4,5,6C	882339	ELR FOOTPLATE ASSEMBLY 18 -20"	
12-14	119495	EXTENSION CLAMP KIT	
15	129775	ALU FOOTPLATE HDWARE KIT	Ext tube not included; bracket (118093) not included


Pos.	Item Number	Description	Remarks
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

NOTE: only available with Adult Angle Adjustable (AAA) or Kids Angle Adjustable (KAA) Footplates.

NOTE: if retrofitting to an existing chair with Front Mount Hangers, new Adult Angle Adjustable (AAA) or Kids Angle Adjustable (KAA) Footplates with High Mount Extension Tubes are required. See Footplate Ordering Guide.


NOTE: if retrofitting to an existing chair with extension tube mounted hangers (D2 extension tubes) with Adult Angle Adjustable (AAA) or Kids Angle Adjustable (KAA) Footplates, order 2 each of 226167 and 561055 to mount the footplates onto the round high-mount extension tubes.

NOTE: if retrofitting on a chair with extension tube mounted hangers with Composite, Composite Angle Adjustable, Foam, Locking Angle Adjustable, Platform Flip-Up, or Platform Flip-Up Locking Footplates, then new Adult Angle Adjustable (AAA) or Kids Angle Adjustable (KAA) Footplates with High Mount Extension Tubes are required. See Footplate Ordering Guide.

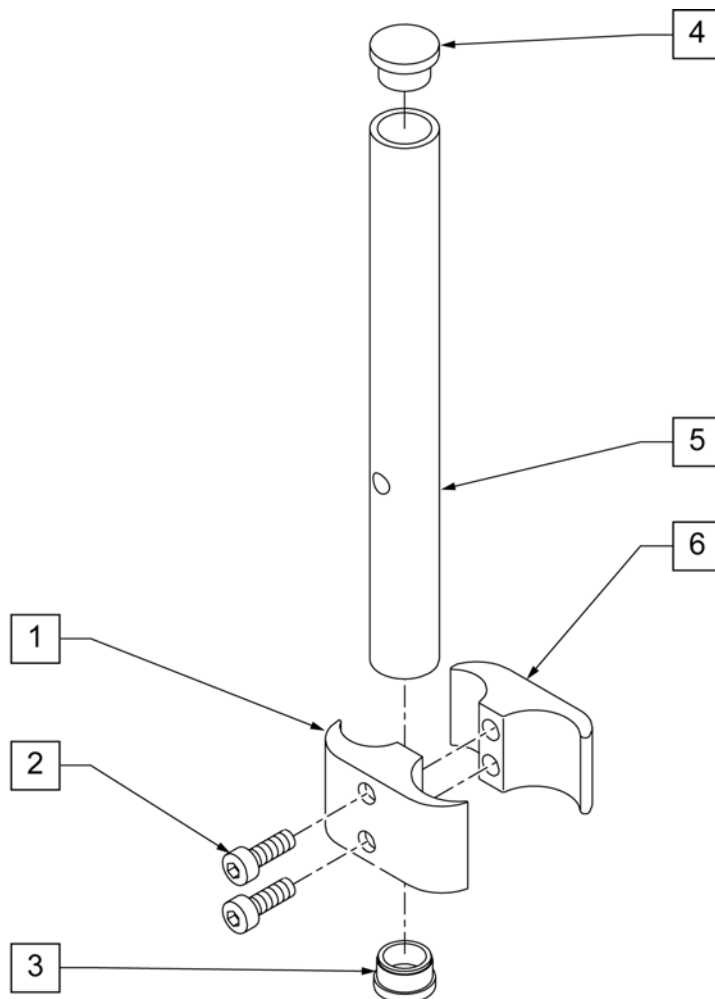


Pos.	Item Number	Description	Remarks
1	550657	CLAMP 1 1/4" RECESS FULL BLK	
2	107032	SHCS 1/4-28 X 1 ROHS	
3	561000-QU	3/4" INSERT PLUG .083	
4	482016	HIGH-MOUNT FL FT TUBE PLUG	
5	524100	H/M FL FT EXT TUBE BLK	
6	550658	CLAMP 1 1/4" THREADED BLK	
1,2,6	880538	HI-MOUNT FR CLAMP ASSEMBLY BLK	
1-6	921550S	H/M FL FT CLAMPS AND TUBES (PR)	Pair

Measuring Guide

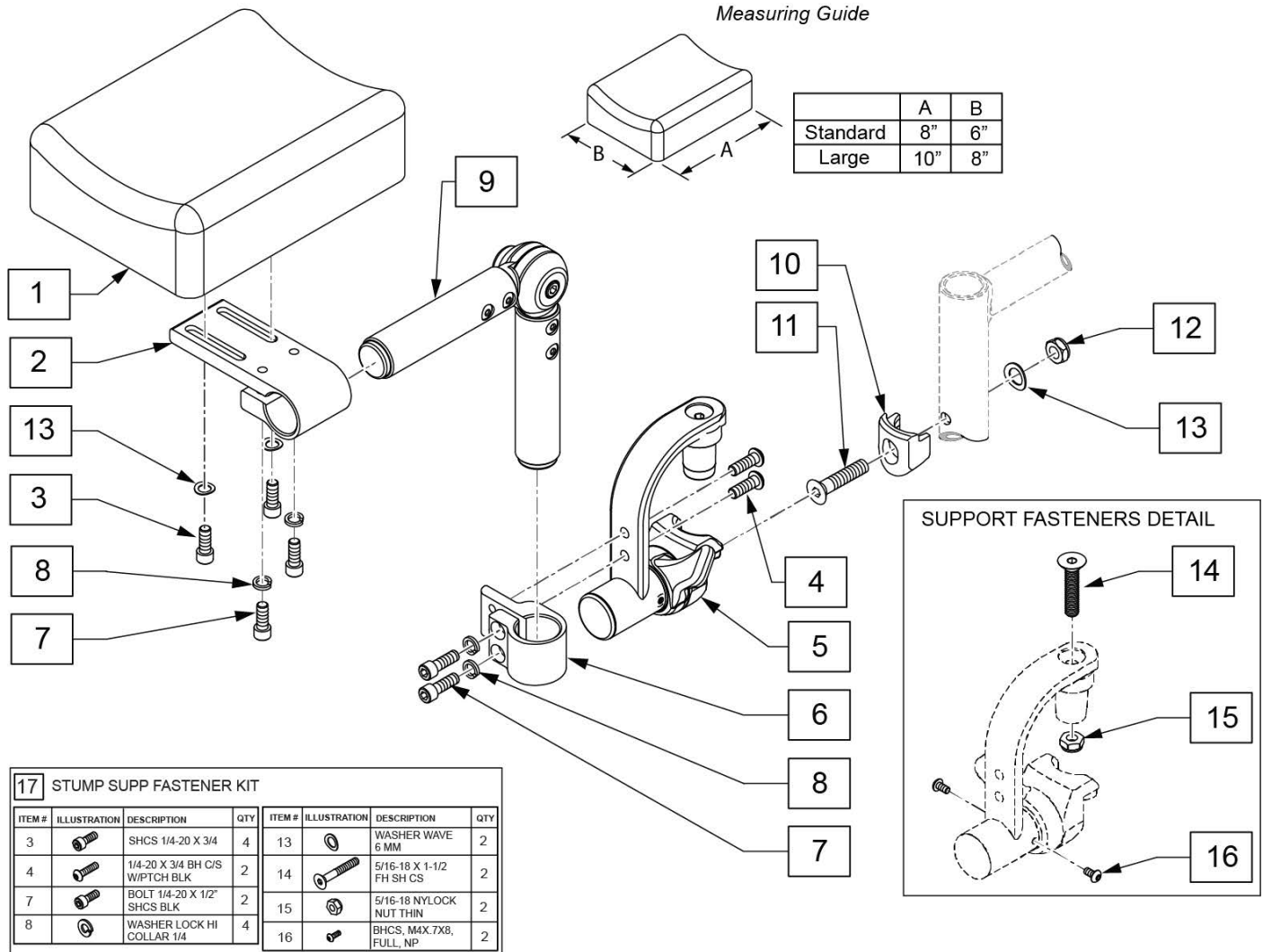


Pos.	Item Number	Description	Remarks
1,3,13	161563	PAD W/FASTENERS STD	Single Pad, 6X8
1,3,13	161564	PAD W/FASTENERS LRG	Single Pad, 8X10
1-9,13	161565	STUMP SUPP RETRO LRG PAD RH	Single Pad, 8X10
1-9,13	161566	STUMP SUPP RETRO LRG PAD LH	Single Pad, 8X10
1-9,13	161567	STUMP SUPP RETRO STD PAD RH	6X8
1-9,13	161568	STUMP SUPP RETRO STD PAD LH	6X8
2-9,13	162051	STUMP SUPPORT W/O PAD RH	No Pad
2-9,13	162052	STUMP SUPPORT W/O PAD LH	No Pad
10-13	119514	LATCH RECEIVER KIT	for 1 side
17	161569	STUMP SUPP FASTENER KIT	for 1 side

NOTE: FOR ARMRESTS PRIOR TO 7/22/96 USE RETRO FIT KIT TO REPLACE ADJUSTABLE FLIP-UP ARM CLAMP.

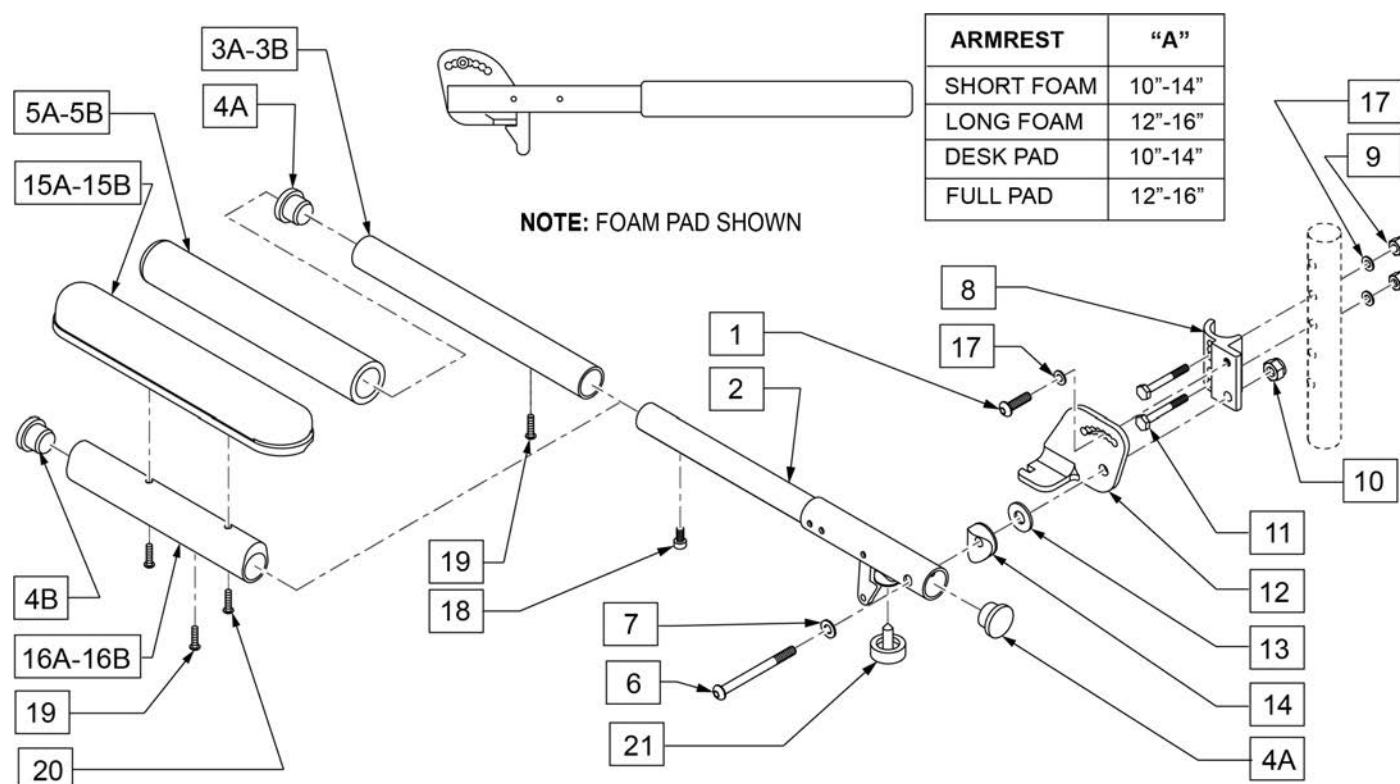


Pos.	Item Number	Description	Remarks
1	107025	BHCS 1/4-28 x 1/2 ROHS	
4A	561002	1" INSERT PLUG .083	
4B	561062	1" INSERT PLUG .065	
5A	560299	7 x .940 x .20 FOAM GRIP	SHORT PAD
5B	560640	9" X .93" X .155" FOAM GRIP	LONG PAD
6	106983	BHCS 5/16-18 x 2 SHLDR ROHS	
7	106947	WASHER FLAT THIN AN 5/16 ROHS	
8	550168	ADJUSTABLE LOCK FLIP-UP ARM MOUNT	RIGHT
8	550169	ADJUSTABLE LOCK FLIP-UP ARM MOUNT	LEFT
9	106935	NYLOCK NUT THIN 1/4 -28 ROHS	
10	202117	5/16-18 NYLOCK NUT THICK BLK	
11	106972	HHCS 1/4-28 x 1 1/2 ROHS	
12	550474	ADJUSTABLE FLIP-BACK ARM BRACKET	RIGHT
12	550475	ADJUSTABLE FLIP-BACK ARM BRACKET	LEFT
13	561016	1" WASHER 5/16" HOLE	
14	561006	1" SADDLE 5/16" HOLE	
15A	561024	ARMREST PAD 10" DESK	
15B	561031	ARMREST PAD 14" FULL	
17	106943	WASHER FLAT AN 1/4 ROHS	
18	107036	SHCS 8 -32 x 3/8 ROHS	
19	225110	10-32 X 1/2" SET SCREW	
20	107022	BHCS 10-32 X 5/8 ROHS	
21	560252	1" COVED STEM BUMPER	
3B, 4A, 5B, 19	882074	FOAM PAD LONG ADJUSTABLE LENGTH	LONG PAD ASSM ONLY

Pos.	Item Number	Description	Remarks
3A, 4A, 5A, 19	882075	FOAM PAD LONG ADJUSTABLE LENGTH	SHORT PAD ASSM ONLY
4B,15B,16B,19,20	882070	FULL PAD ADJUSTABLE LENGTH ASSM	RIGHT FULL PAD ASSM ONLY
4B,15B,16B,19,20	882071	FULL PAD ADJUSTABLE LENGTH ASSM	LEFT FULL PAD ASSM ONLY
4B,15A,16A,19,20	882072	DESK PAD ADJUSTABLE LENGTH ASSM	RIGHT DESK PAD ASSM ONLY
4B,15A,16A,19,20	882073	DESK PAD ADJUSTABLE LENGTH ASSM	LEFT DESK PAD ASSM ONLY
1,2,6-14,17,21	882088	ADJ LENGTH LOCKING FLIP-UP ASSM R	RIGHT ARM ASSM W/O PAD
1,2,6-14,17,21	882089	ADJ LENGTH LOCKING FLIP-UP ASSM R	LEFT ARM ASSM W/O PAD
1,2,3B,4A,5B,6-14,17,19 -21	900454	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	PAIR - LONG FOAM ARM ASSM
1,2,3B,4A,5B,6-14,17,19 -21	881294	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	RIGHT - LONG FOAM ARM ASSM
1,2,3B,4A,5B,6-14,17,19 -21	881295	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	LEFT - LONG FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	900455	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	PAIR - SHORT FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	881296	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	RIGHT - SHORT FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	881297	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	LEFT - SHORT FOAM ARM ASSM
1,2,4B,6-14,15B,16B,17-21	900452	ADJ LENGTH LOCKING FLIP-UP ASSM FULL	PAIR - FULL PAD ARM ASSM
1,2,4B,6-14,15B,16B,17-21	881290	ADJ LENGTH LOCKING FLIP-UP FULL PAD	RIGHT - FULL PAD ARM ASSM
1,2,4B,6-14,15B,16B,17-21	881291	ADJ LENGTH LOCKING FLIP-UP FULL PAD	LEFT - FULL PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	900453	ADJ LENGTH LOCKING FLIP-UP ASSM DESK	PAIR - DESK PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	881292	ADJ LENGTH LOCKING FLIP-UP DESK PAD	RIGHT - DESK PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	881293	ADJ LENGTH LOCKING FLIP-UP DESK PAD	LEFT - DESK PAD ARM ASSM

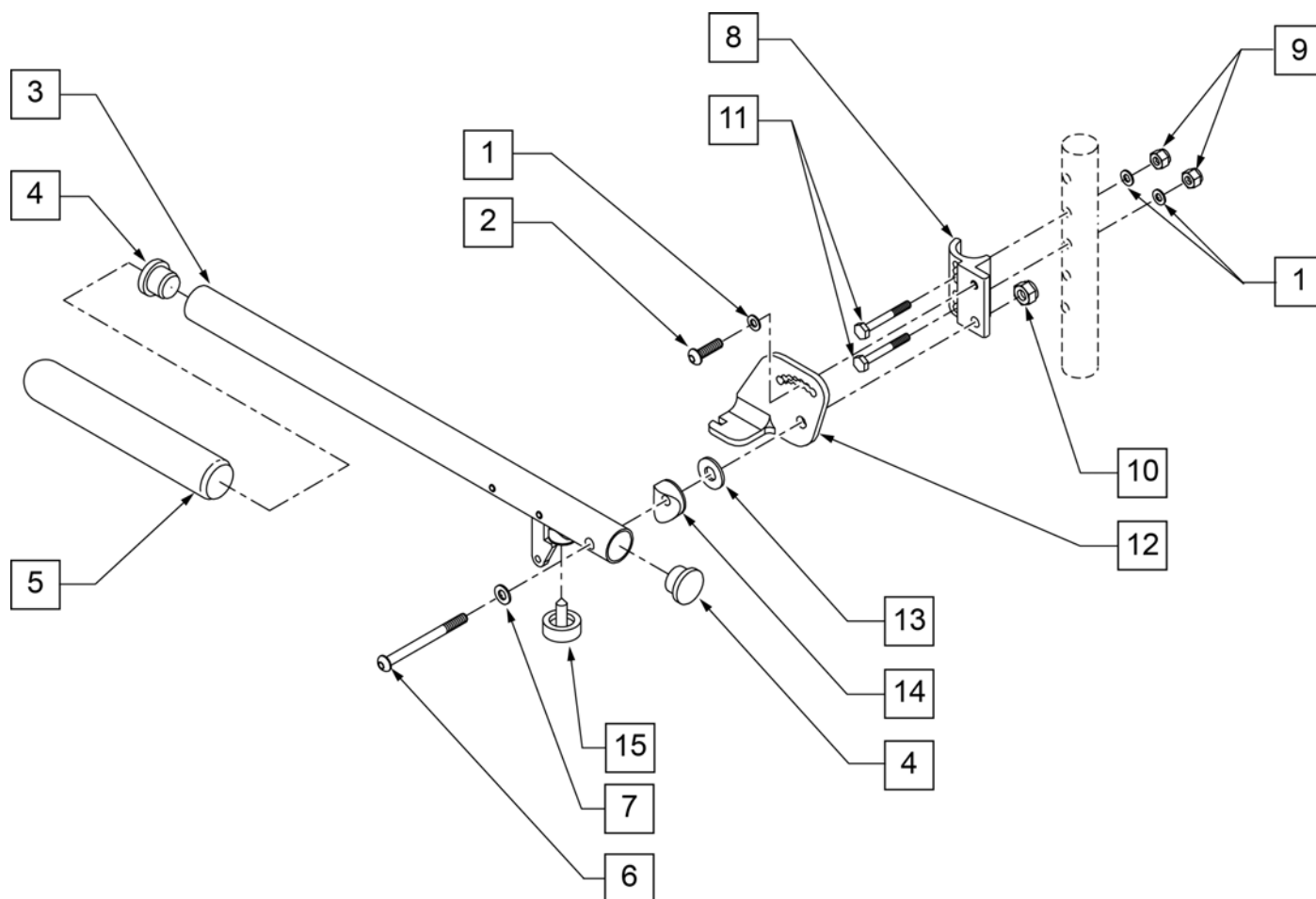


Pos.	Item Number	Description	Remarks
1	106943	1/4 AN WASHER FLAT	
2	107025	1/4-28 x 1/2 BH C/S	
4	561002	1" INSERT PLUG .083	
5	560640	9 x .93 x .155 FOAM GRIP	
6	106983	5/16-18 x 2 BH C/S SHT THREAD	
7	106947	5/16 WASHER FLAT THIN	
8	550168	ADJ LOCK FLIP-UP ARM MNT R	
8	550169	ADJ LOCK FLIP-UP ARM MNT L	
9	106935	1/4-28 NYLOCK NUT THIN	
10	202117	5/16-18 NYLOCK NUT THICK BLK	
11	106972	1/4-28 x 1 1/2 G8 HEX C/S	
12	550474	ADJ FLIP-BACK ARM BRACKET R	
12	550475	ADJ FLIP BACK ARM BRACKET L	
13	561016	1" WASHER 5/16" HOLE	
14	561006	1" SADDLE 5/16" HOLE	
15	560252	1" COVED STEM BUMPER	
3-5	880274	ADJ FLIP-BACK TUBE W/LATCH	
1,2,6,8-10	881272	ADJ LOCK F-UP ARMREST RETRO R	USE FOR ARMREST PRIOR TO 7/22/96
1,2,6,8-10	881273	ADJ LOCK F-UP ARMREST RETRO L	USE FOR ARMREST PRIOR TO 7/22/96
1-15	880272	ADJ FLIP BACK ARMREST R	
1-15	880273	ADJ FLIP BACK ARMREST L	
1-15	900449	ADJ LOCK FLIP-UP ARMREST ASSM	PAIR

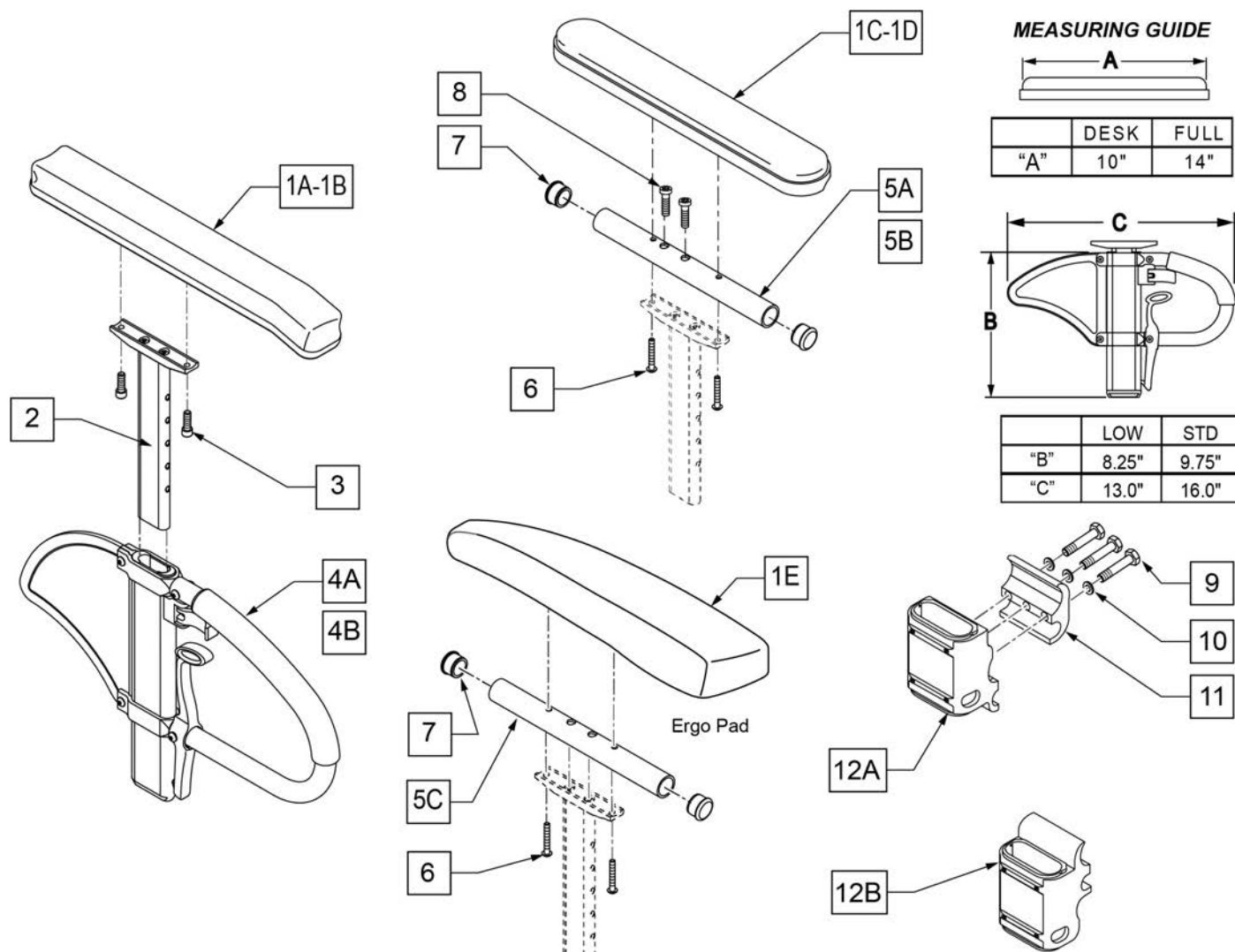

Pos.	Item Number	Description	Remarks
1A	104568-001	SPHA PAD DESK	CONTOUR DESK
1B	104568-002	SPHA PAD FULL	CONTOUR FULL
1C,8	161633	CLASSIC PAD DESK	CLASSIC DESK
1D,8	161634	CLASSIC PAD FULL	CLASSIC FULL
1E,6	161631	ERGO PAD RIGHT	With Fasteners
1E,6	161632	ERGO PAD LEFT	With Fasteners
2	116312	SNGL POST UPPR ASSM LOW	LOW/SMALL
2	116313	SNGL POST UPPR ASSM STD	STANDARD
3	106956	SHCS 10-32 X 3/4 ROHS	
2,4A	104509-001	SPHA ARMREST ASSY STD	STANDARD
2,4B	104509-002	SPHA ARMREST ASSY LOW	LOW/SMALL
5A	521812	HT ADJ PD TUBE 10IN DESK POWE	Desk
5B	521813	HT ADJ PD TUBE 14IN FULL POWE	Full
6	107024	BHCS 10-32 X 1 1/2 ROHS	
7	560735	7/8" ROUND TUBE PLUG	
8	108534	SHCS 10-32 X 1-1/4 PL	Used with Full Length & Waterfall
1C,2,3,4A,5A,6,7	108599	SPHA STD CLASIC DESK PAD R	Right
1C,2,3,4A,5A,6,7	108603	SPHA STD CLASIC DESK PAD L	Left
1D,2,3,4A,5B,6-8	108601	SPHA STD CLASIC FULL PAD R	Right


Pos.	Item Number	Description	Remarks
1D,2,3,4A,5B,6-8	108605	SPHA STD CLASIC FULL PAD L	Left
1A,2,3,4A	108595	SPHA STD CNTUR DESK PAD	
1B,2,3,4A	108597	SPHA STD CNTUR FULL PAD	
1C,2,3,4B,5A,6,7	108600	SPHA LOW CLASIC DESK PAD R	Right
1C,2,3,4B,5A,6,7	108604	SPHA LOW CLASIC DESK PAD L	Left
1D,2,3,4B,5B,6-8	108602	SPHA LOW CLASIC FULL PAD R	Right
1D,2,3,4B,5B,6-8	108606	SPHA LOW CLASIC FULL PAD L	Left
1A,2,3,4B	108596	SPHA LOW CNTUR DESK PAD	
1B,2,3,4B	108598	SPHA LOW CNTUR FULL PAD	
1C,2,5A,6,7	159600	SPHA STD CLAS DESK UPPR R	Right
1C,2,5A,6,7	159601	SPHA STD CLAS DESK UPPR L	Left
1D,2,5B,6-8	159602	SPHA STD CLAS FULL UPPR R	Right
1D,2,5B,6-8	159603	SPHA STD CLAS FULL UPPR L	Left
1C,2,5A,6,7	159604	SPHA LOW CLAS DESK UPPR R	Right
1C,2,5A,6,7	159605	SPHA LOW CLAS DESK UPPR L	Left
1D,2,5B,6-8	159606	SPHA LOW CLAS FULL UPPR R	Right
1D,2,5B,6-8	159607	SPHA LOW CLAS FULL UPPR L	Left
1E,2,4,5C,6-8	161649	SPHA FB LT ERGO R	Right
1E,2,4,5C,6-8	161650	SPHA FB LT ERGO L	Left
9-12A	161570	STANDARD SPHA RECEIVER	
9-12B	161571	LOW SPHA RECEIVER	

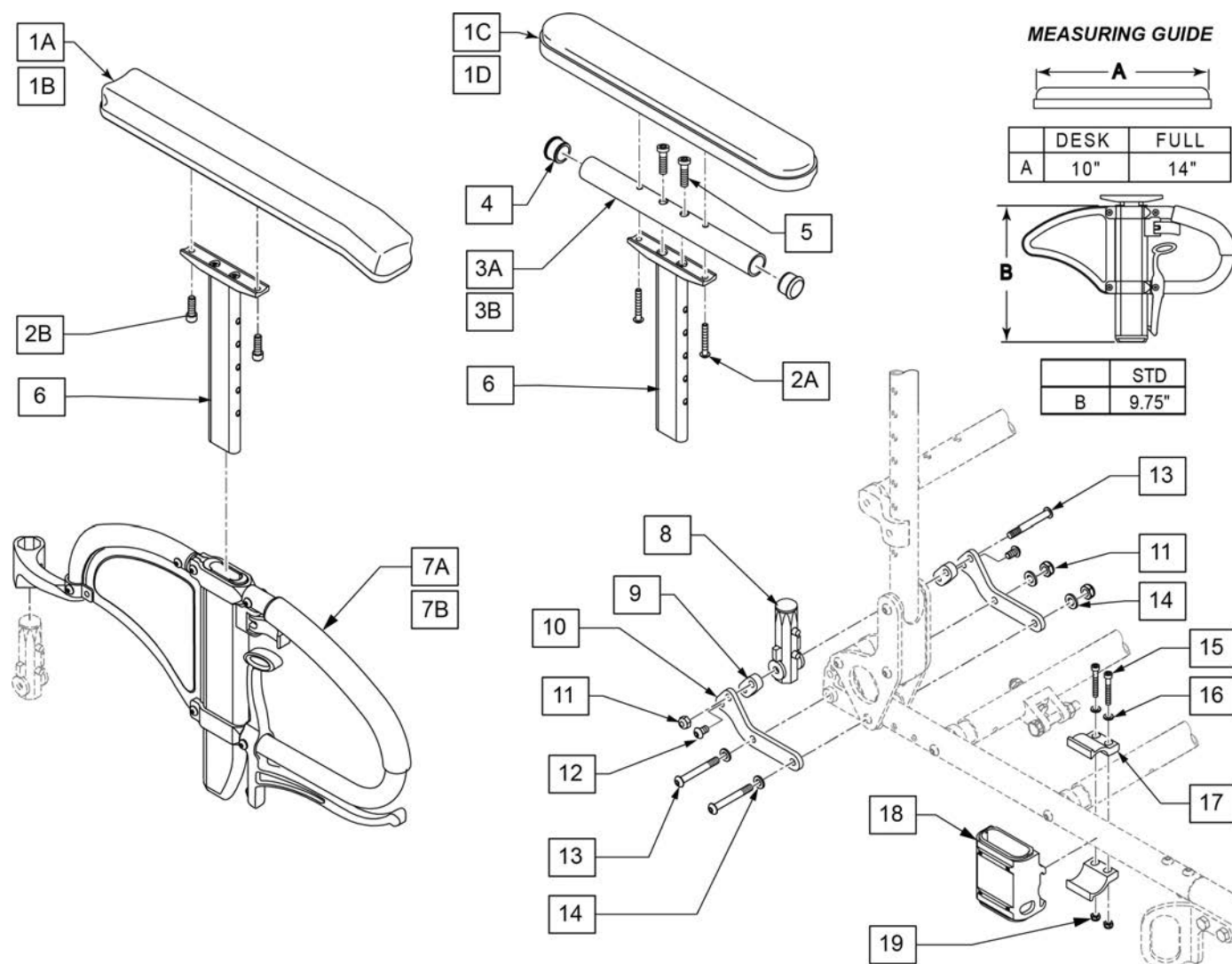


Pos.	Item Number	Description	Remarks
1A	104568-001	SPHA PAD DESK	
1B	104568-002	SPHA PAD FULL	
1C,2	161633	CLASSIC PAD DESK	With Fasteners
1D,2	161634	CLASSIC PAD FULL	With Fasteners
2A	107024	10-32 X 1 1/2 BH C/S BLK	
2B	106956	SHCS 10-32 X 3/4 ROHS	
3A	243044	HEIGHT ADJ UPPER ARM TUBE MED	
3B	521812	HT ADJ PD TUBE 10IN DESK POWE	
3C	521813	HT ADJ PD TUBE 14IN FULL POWE	
4	560735	7/8" ROUND TUBE PLUG	
5	226031	10-32 x 1 1/4 SOC C/S BLK	
6	116313	SPHA ARM PAD MOUNT	Full
6,7A	241726	Q2 FLIP UP RH	Right
6,7A	241592	Q2 FLIP UP LH	Left
6,7B	256139	LONG FLIP UP ARMREST RH	Right
6,7B	256138	LONG FLIP UP ARMREST LH	Left
15-19	161651	SPHA FB LT RECEIVER FRNT	One side, Left or Right

Pos.	Item Number	Description	Remarks
8-14	169722	DP FB HGT ADJ REAR RECIEVER	
11-14	169723	DP FB HGT ADJ FASTENERS	
1A,2B,5,6,7A,8-19	169718	DP FB HGT ADJ DESK CONT RH	Right
1A,2B,5,6,7A,8-19	169719	DP FB HGT ADJ DESK CONT LH	Left
1A,2B,5,6,7B,8-19	169720	DP FB HGT ADJ FULL CONT RH	Right
1A,2B,5,6,7B,8-19	169721	DP FB HGT ADJ FULL CONT LH	Left
1C,2A,3A,4,5,7A,8-19	169714	DP FB HGT ADJ DESK CLASS RH	Right
1C,2A,3A,4,5,7A,8-19	169715	DP FB HGT ADJ DESK CLASS LH	Left
1D,2A,3B,4,5,7A,8-19	169716	DPFB HGT ADJ FULL CLASS RH	Right
1D,2A,3B,4,5,7A,8-19	169717	DPFB HGT ADJ FULL CLASS LH	Left

NOTE: KIDS BACKPACKS DO NOT HAVE ADJUSTABLE STRAPS W/ BUCKLES.

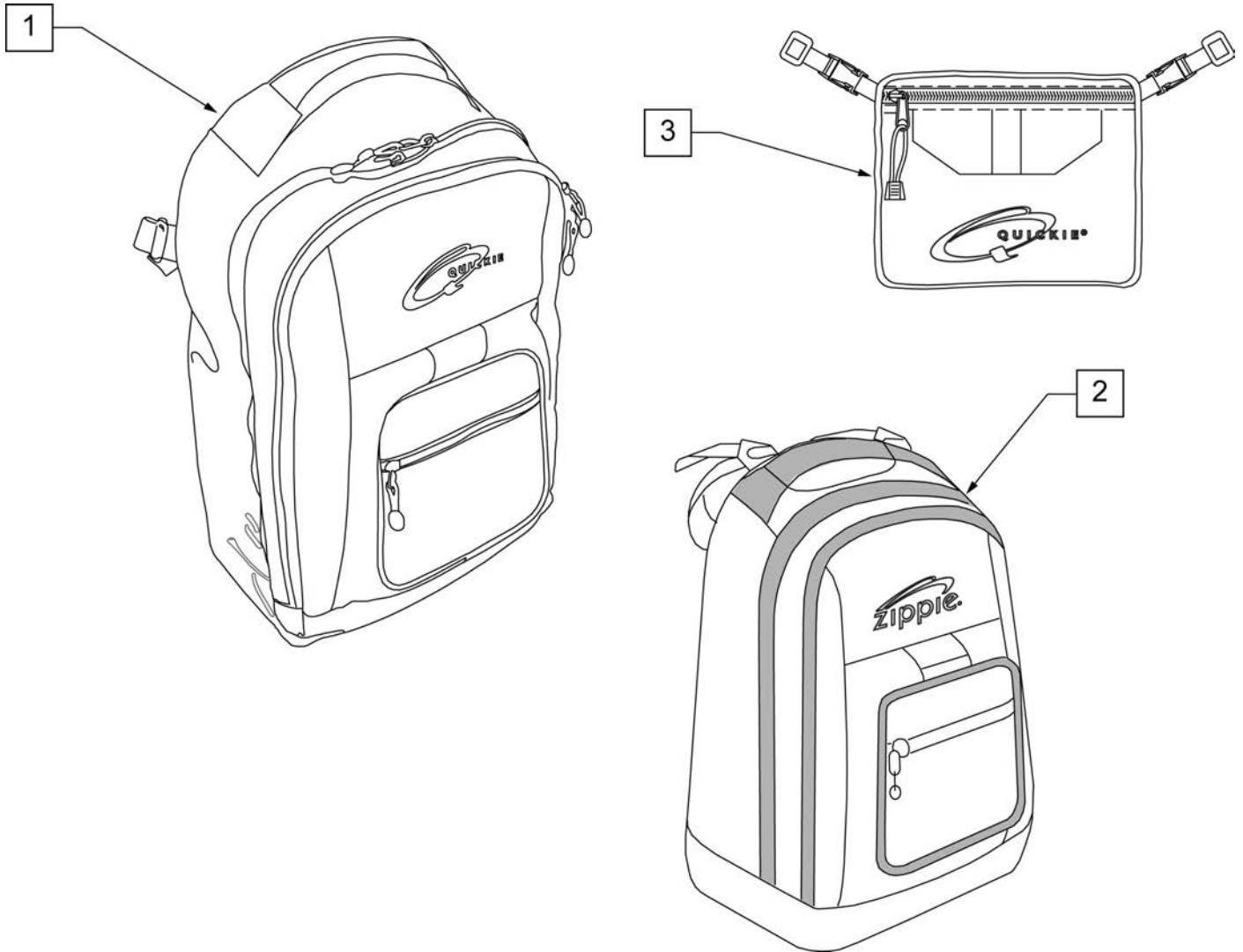


Pos.	Item Number	Description	Remarks
1	750115	BACK PACK QUICKIE	Overall dimensions:width 12.5", height 18.5", depth 6.5"
2	104572	BACKPACK ZIPPIE	Overall dimensions: width 12", height 16.5", depth 4.5"
3	750307	SEAT POUCH QUICKIE	Dimensions: width 8.5", height 6", depth top 2.5", depth bottom 3.5"

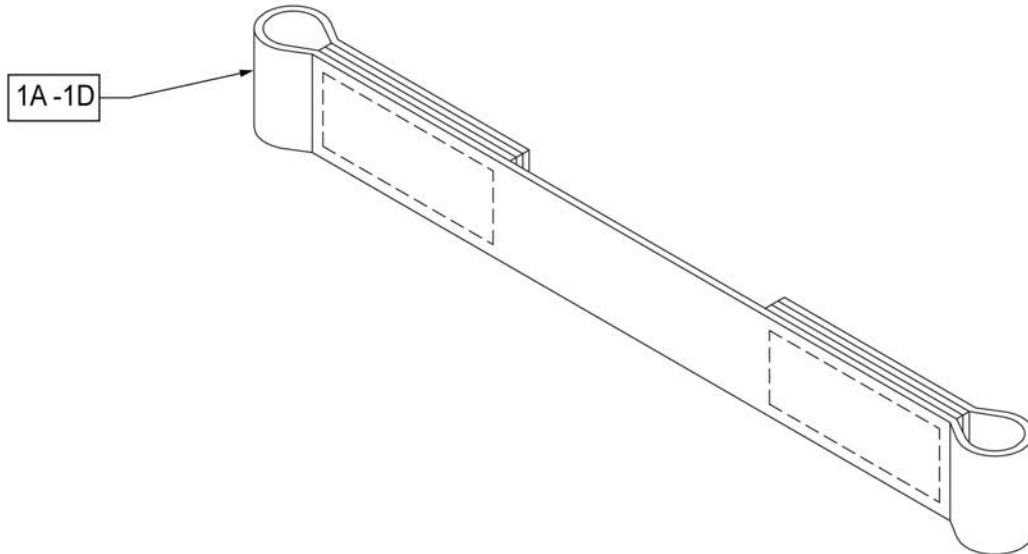

Pos.	Item Number	Description
1A	109356	LEG STRAP SOFT FEEL Q 11 -12"
1B	109357	LEG STRAP SOFT FEEL Q 13 -15"
1C	109366	LEG STRAP SOFT FEEL Q 16 -18"
1D	111487	LEG STRAP SOFT FEEL Q 19 -20"

Note: some items listed may not be available with every chair model or in conjunction with another chair feature.


NOTE: when mounting Positioning Belts to the 5R, GT or GTI, it is recommended to attach the belt by using the Backrest Support Bracket Hardware.

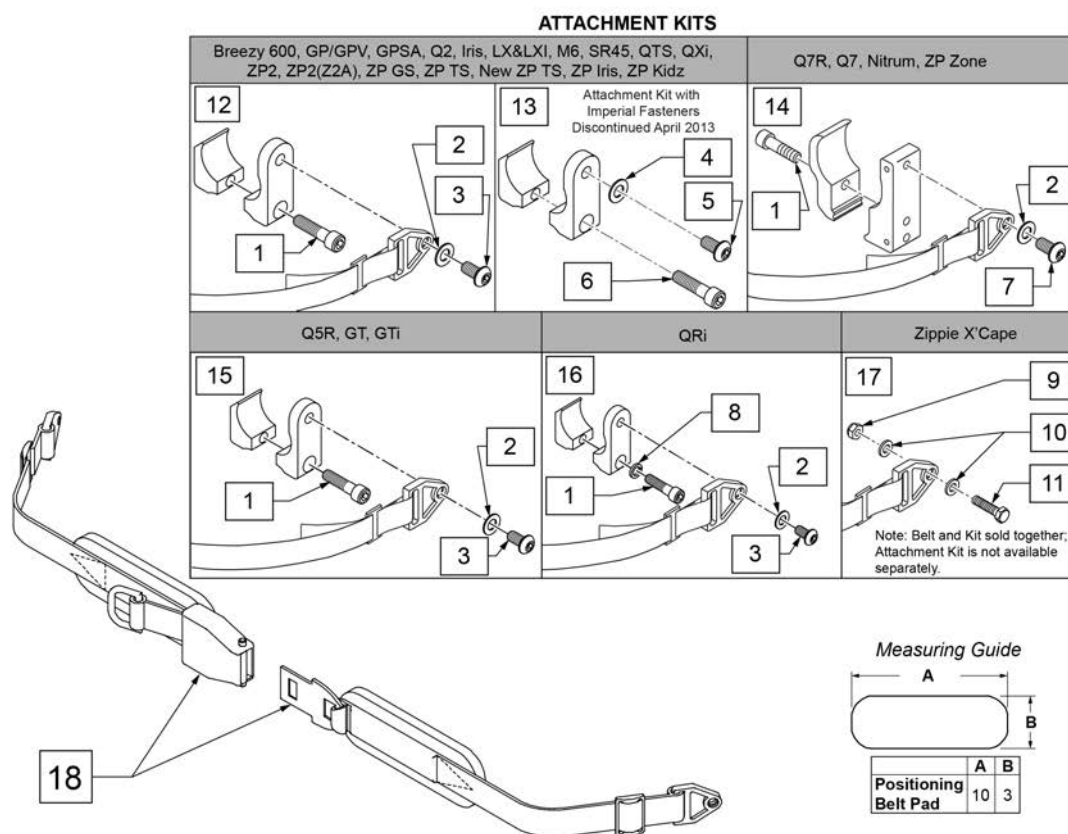

Pos.	Item Number	Description	Remarks
1	102263-003	SHCS M6 X 1 X 25 W/PL INORG	Metric
2	250104	WASHER FLAT M6	Metric
3	103185-005	BHCS M6 X 1 X 12 FULL PL	Metric
4	106943	WASHER FLAT AN 1/4 ROHS	Imperial (standard)
5	226154	1/4-20 X 1/2 BH C/S W/PATCH	Imperial (standard)
6	221116	1/4-20 X 1 SOC HD BLK	Imperial (standard)
7	230113	BHCS M6 X 1 X 12	
8	050129-001	LOCKWASHER HIGH COLLAR M6	
9	104381	NUT HEX NYLOCK M6 X 1.0	
10	107383	WASHER FLAT M6	
11	102268-002	HHCS M6 X 1 X 35 BLK	
12	120750	PSTN BELT CLAMP ASSM PR	Metric; Pair
13	900116	POSITION BELT ATTACHMENT KIT	Set of 2; Imperial (standard)
14	116268	OVAL CLAMP ASSM KIT	Kit contains 1 Clamp

Pos.	Item Number	Description	Remarks
15	159122	POSITIONING BELT CLAMPS PR	Pair
16	123534	POS BELT CLAMP KIT RPO	Set of 2
18	450418-QU	BELT 2" AIRCRAFT BUCKLE	Max length of belt 64"
18	450423-QU	BELT 2" AIRCRAFT BUCKLE W/PAD	Max length of belt 64"
18	450424	BELT 2" AIRCRAFT BKL LONG W/PAD	Max length of belt 72"
18	450425	BELT 2" AIRCRAFT BKL LONG	Max length of belt 72"
18	450419-QU	BELT 2" AIRCRAFT BKL W/QUAD LP	Max length of belt 64"
18	468419	BELT 2" AIRCRAFT BKL W/QD LPCZ	Max length of belt 64"
17,18	136464	AIRCRAFT HIP BELT 2" KIT	ZP Xcape: includes Belt & Attachment Kit
17,18	136465	AIRCRAFT HIP BELT 2" PAD KIT	ZP Xcape: includes Belt & Attachment Kit


NOTE: when mounting Positioning Belts to the 5R, GT or GTi, it is recommended to attach the belt by using the Backrest Support Bracket Hardware.

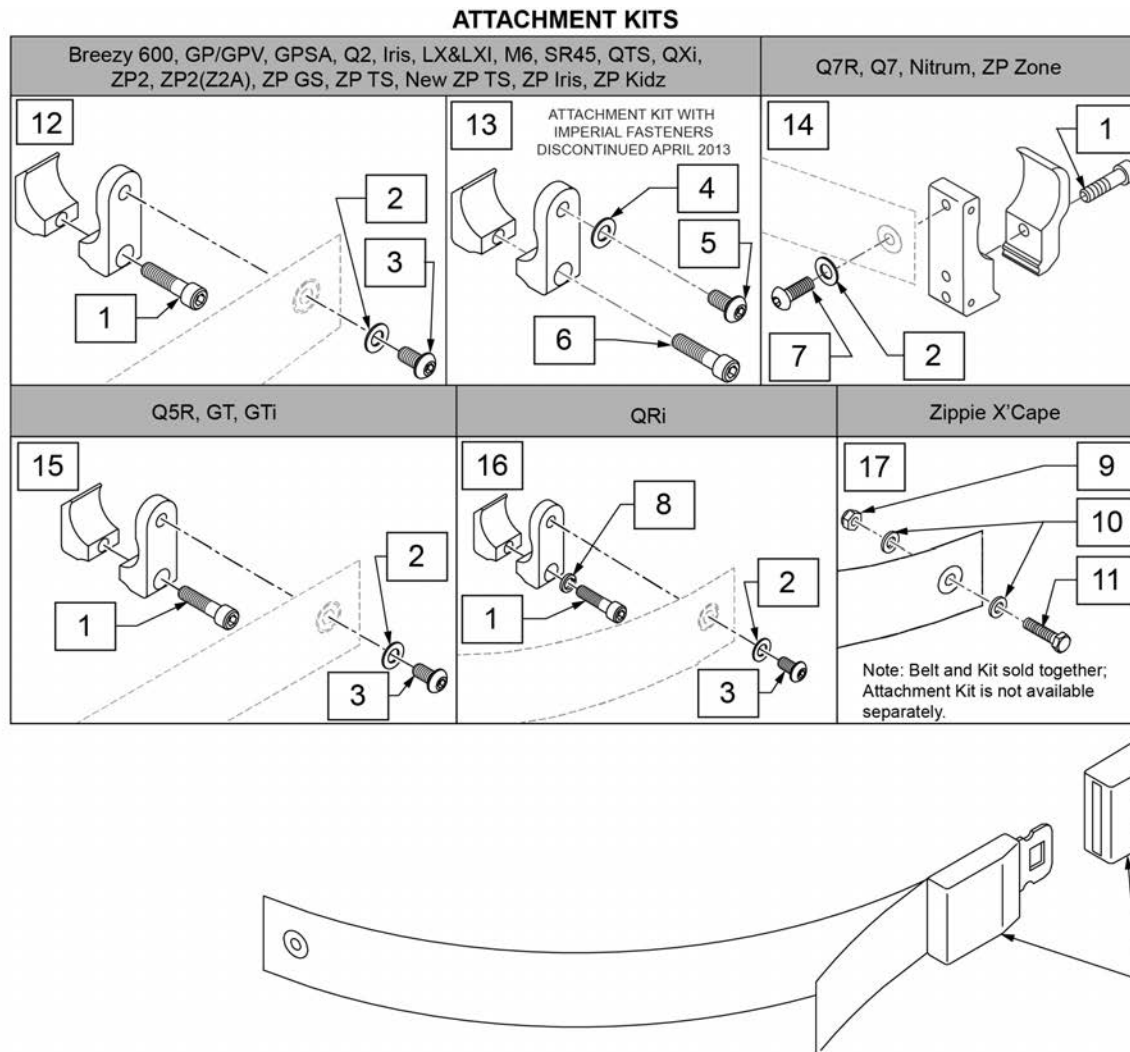


Pos.	Item Number	Description	Remarks
1	102263-003	SHCS M6 X 1 X 25 W/PL INORG	Metric
2	250104	WASHER FLAT M6	Metric
3	103185-005	BHCS M6 X 1 X 12 FULL PL	Metric
4	106943	WASHER FLAT AN 1/4 ROHS	Imperial (standard)
5	226154	1/4-20 X 1/2 BH C/S W/PATCH	Imperial (standard)
6	221116	1/4-20 X 1 SOC HD BLK	Imperial (standard)
7	230113	BHCS M6 X 1 X 12	
8	050129-001	LOCKWASHER HIGH COLLAR M6	
9	104381	NUT HEX NYLOCK M6 X 1.0	
10	107383	WASHER FLAT M6	
11	102268-002	HHCS M6 X 1 X 35 BLK	
12	120750	PSTN BELT CLAMP ASSM PR	Metric; Pair
13	900116	POSITION BELT ATTACHMENT KIT	Set of 2; Imperial (standard)
14	116268	OVAL CLAMP ASSM KIT	Kit contains 1 Clamp
15	159122	POSITIONING BELT CLAMPS PR	

Pos.	Item Number	Description	Remarks
16	123534	POS BELT CLAMP KIT RPO	Set of 2
18	750276	1 1/2" POSITION BELT	AUTOSTYLE-64" Max length of belt.
16,18	124269	AUTO POS BELT W/CLAMPS RPO	QRI: includes Belt & Attachment Kit
17,18	136463	AUTO HIP BELT 1.5" KIT	ZP Xcape: includes Belt & Attachment Kit


1A-1C

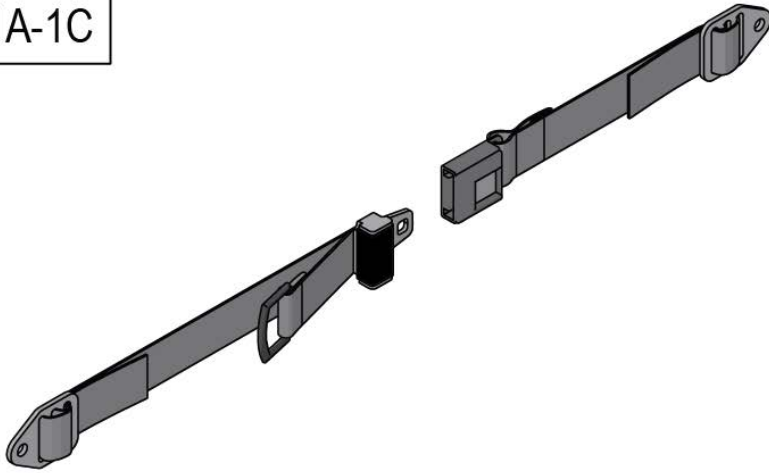

6

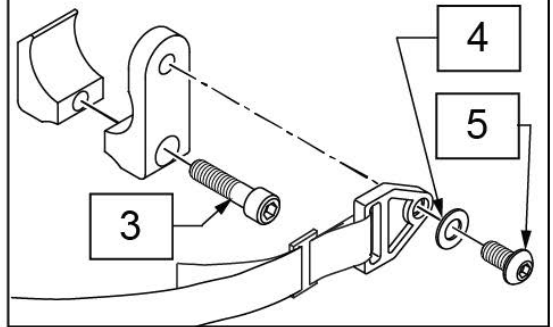


2A-2M

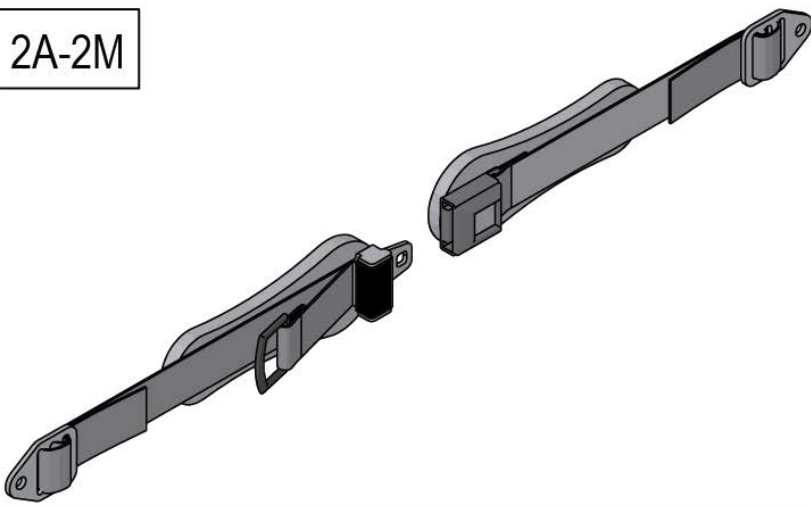

Pos.	Item Number	Description	Remarks
1A	APB5110	APB, 1" PB, SINGLE, NON PADDED	Off-chair option; Non-Padded
1B	APB7210	APB, 1.5" PB, SINGLE, NON-PAD	Non-Padded
1C	APB7310	APD, 2" PB, SINGLE, NON-PADDED	Non-Padded
2A	APB5115	APB 1" PB, SINGLE, 4.5" PAD	Zippie Iris only (4.5")
2B	APB7212	APB, 1.5" PB, SINGLE, SM PADS	Small Pad (7.5")
2C	APB7313	APB, 2" PB, SINGLE, MED PADS	Medium Pad (9.5")
2D	APB5116	APB, 1" PB, SINGLE, 5.5" PAD	Off-chair option; 5.5" Pad
2E	APB5117	APB, 1' PB, SINGLE, 7.5" PAD	Off-chair option; Small Pad (7.5")
2F	APB5125	APB, 1" PB, DUAL, 4.5" PAD	Off-chair option; 4.5" Pad
2G	APB5126	APB, 1" PB, DUAL, 5.5" PAD	Off-chair option; 5.5" Pad
2H	APB5127	APB, 1" PB, DUAL PULL, 7.5" PAD	Off-chair option; Small Pad (7.5")
2J	APB7213	APB, 1.5" PB, SINGLE, MED PADS	Off-chair option; Med Pad
2K	APB7222	APB, 1.5" PB, DUAL, SM PADS	Off-chair option; Small Pad
2L	APB7223	APB, 1.5" PB, DUAL, MED PADS	Off-chair option; Med Pad
2M	APB7314	APB, 2" PB, SINGLE, LG PADS	Off-chair option; Large Pad
3	102263-003	SHCS M6 X 1 X 25 W/PL INORG	
4	250104	WASHER FLAT M6	
5	103185-005	BHCS M6 X 1 X 12 FULL PL	
6	120750	PSTN BELT CLAMP ASSM PR	

Note: when adding a Tray to an existing chair with Desk or Full Length Arm Pads, add 1" to current seat width to achieve the best tray size.


Note: Large and X-Large Trays have a round cut-out.

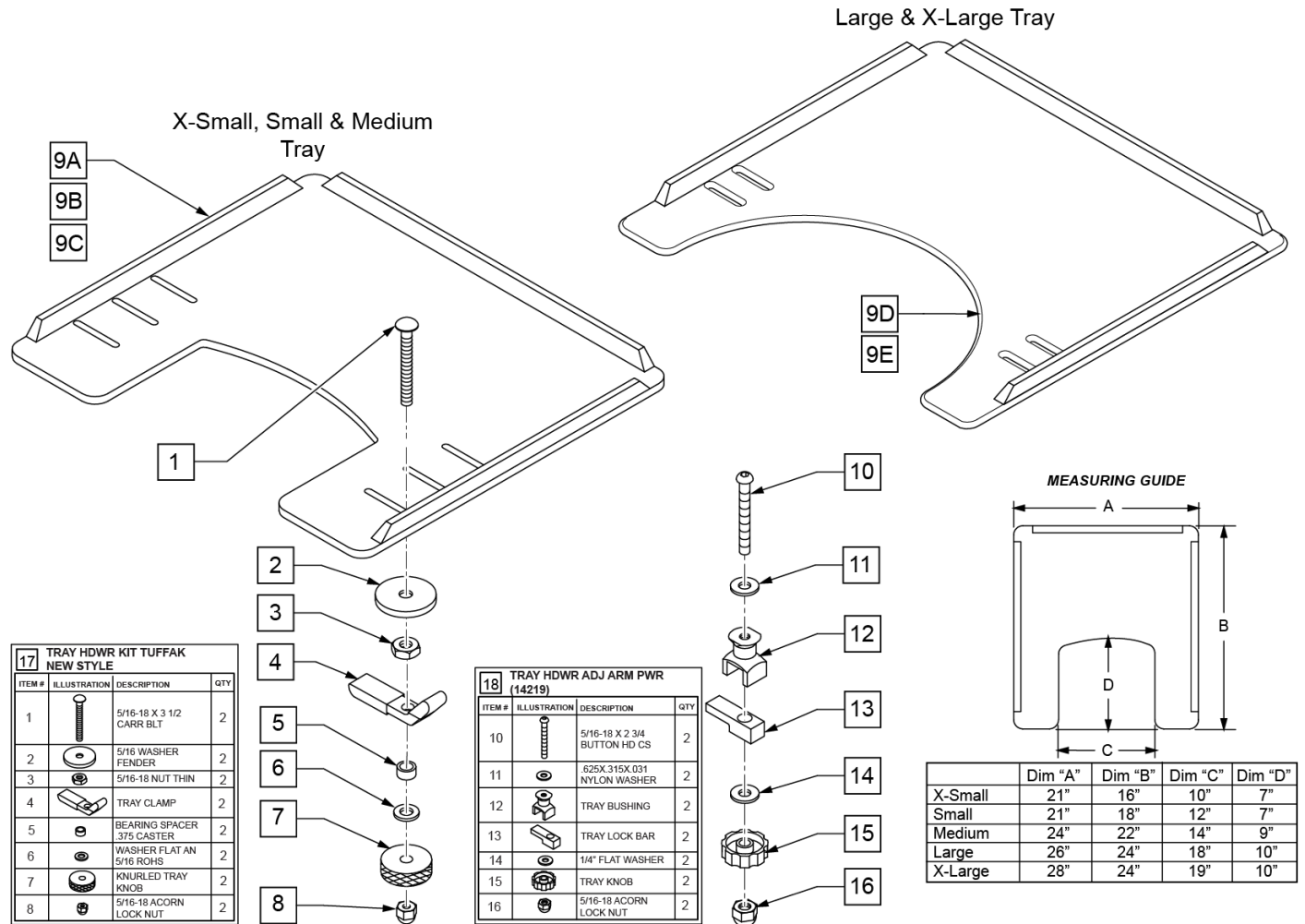


Pos.	Item Number	Description	Remarks
1	226199	5/16-18 x 3 1/2 CARR BOLT	
2	263112	5/16 WASHER FENDER	
3	202110	5/16-18 NUT THIN	
4	550400	TRAY CLAMP	
5	481113	SPACER 5/16 X .380	
6	263100	5/16 AN WASHER FLAT	
7	483026	KNURLED TRAY KNOB	
8	202114	5/16-18 ACORN LOCK NUT	
9A	562619	TRAY TOP EX-SM W/RAIL	
9B	562620	TRAY TOP SM W/RAIL	
9C	562621	TRAY TOP MED W/RAIL	
9D	562622	TRAY TOP LRG W/RAIL	
9E	562623	TRAY TOP X-LRG W/RAIL	

Pos.	Item Number	Description	Remarks
17	881051	TRAY HARDWARE KIT TUFFAK NEW STYLE	Pair
18	881053	TRAY HARDWARE ADJ ARM PWR (14219)	Pair
1-8,9A	902306	TRAY ASSEMBLY EX-SM TUFFAK (10 -12")	Fits 10"-12" wide chairs
1-8,9B	902303	TRAY ASSEMBLY SM TUFFAK (13 -14")	Fits 13"-14" wide chairs
1-8,9C	902304	TRAY ASSEMBLY MED TUFFAK (15 -17")	Fits 15"-17" wide chairs
1-8,9D	902305	TRAY ASSEMBLY LRG TUFFAK (18 -20")	Fits 18"-20" wide chairs
1-8,9E	902309	TRAY ASSEMBLY X-LRG TUFFAK (21 -22")	Fits 21"-22" wide chairs
9A,10-16	902330	TRAY ASSM X-SM TFK 10-12/CAM	Fits 10"-12" wide chairs
9B,10-16	902331	TRAY ASSM SM TUFFAK 13-14/CAM	Fits 13"-14" wide chairs
9C,10-16	902332	TRAY ASSM MED TUFFAK 15-17/CAM	Fits 15"-17" wide chairs
9D,10-16	902333	TRAY ASSM LRG TUFFAK 18-20/CAM	Fits 18"-20" wide chairs


Note: Pad cannot be added to an existing tray as snaps must be installed at the factory to function correctly.

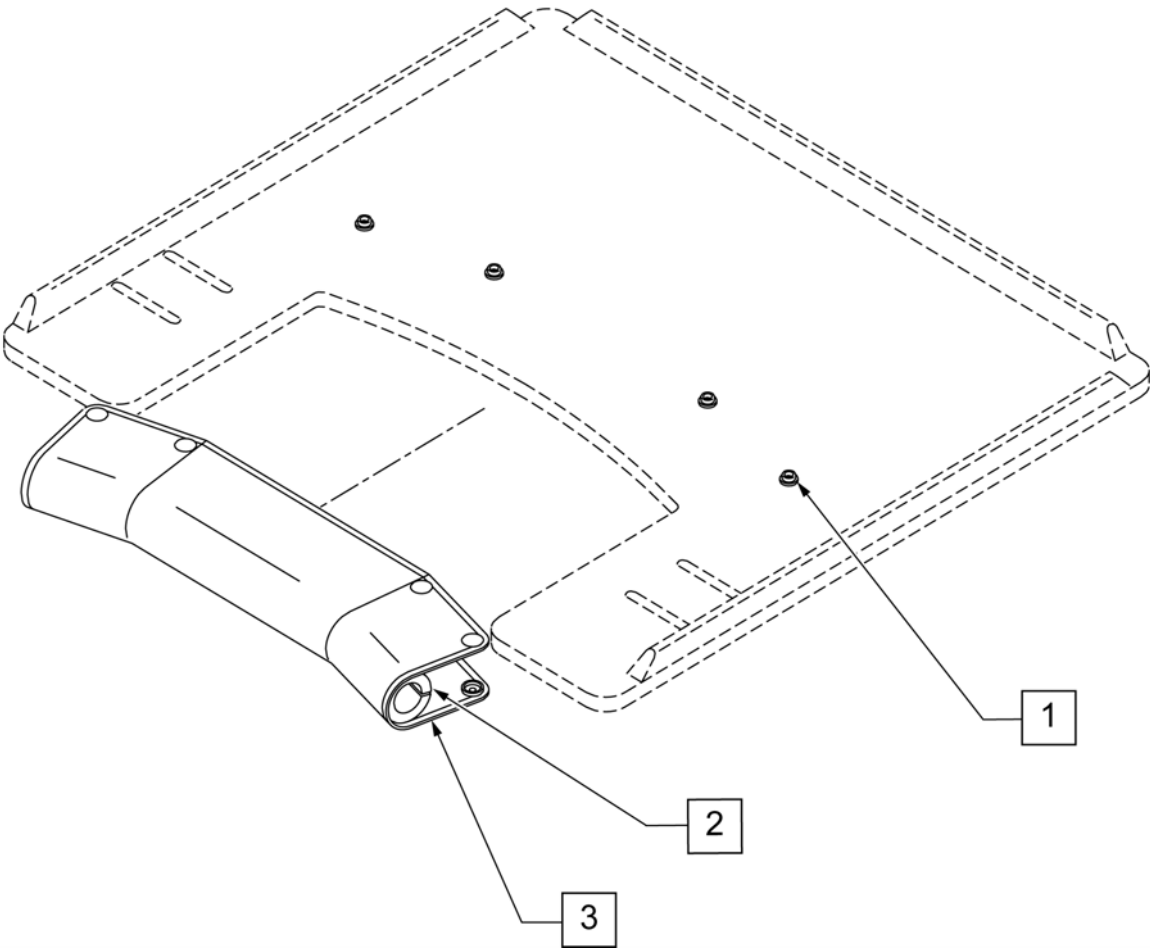


Pos.	Item Number	Description	Remarks
1-3	9PS	PADDED STOMACH CUT OUT	(SPECIFY COLOR)

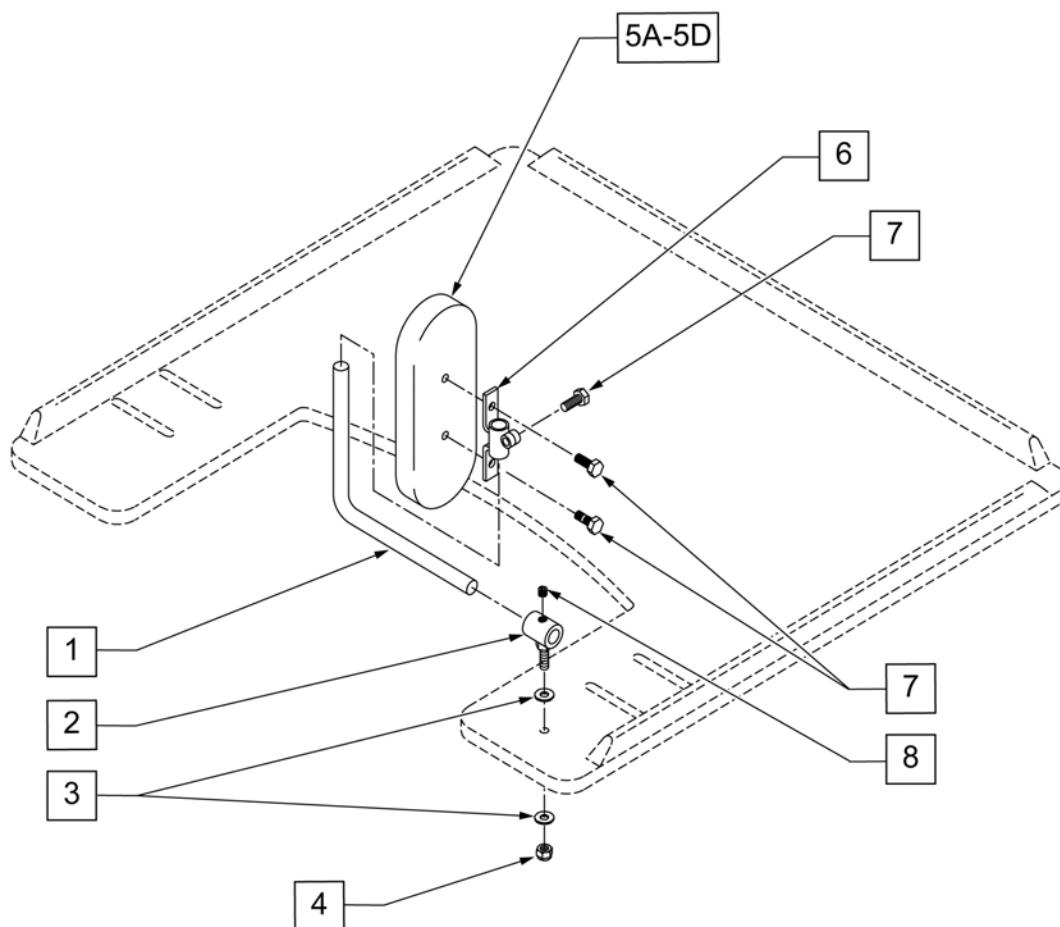

Pos.	Item Number	Description	Remarks
1	450752	HEADREST ADJUSTMENT BAR	
2	450751	HEADREST PAD BUSHING	
3	2702	#10 SAE FLAT WASHER	
4	106932	1/4 -20 NYLOCK NUT HIGH	
5A	15022	4X6 RECTANGULAR PAD	
5B	15013	5 x 7 OVAL PAD	
5C	755020	4 x 5 OVAL PAD	
5D	15018	4 x 6 OVAL PAD	
6	450750	HEADREST PAD BRACKET	
7	221105-QU	1/4-20 x 3/4" HEX C/S	
8	2270	5/16-18 x 1/4" SOC SET SCREW	
1-4,5A,6-8	14980A	TRAY PROTRACTION ASSM-4X6 RECT	PAIR
1-4,5B,6-8	14980B	TRAY PROTRACTION ASSM-5X7 OVAL	PAIR
1-4,5C,6-8	14980D	TRAY PROTRACTION ASSM-4X5 OVAL	PAIR
1-4,5D,6-8	14980E	TRAY PROTRACTION ASSM-4X6 OVAL	PAIR

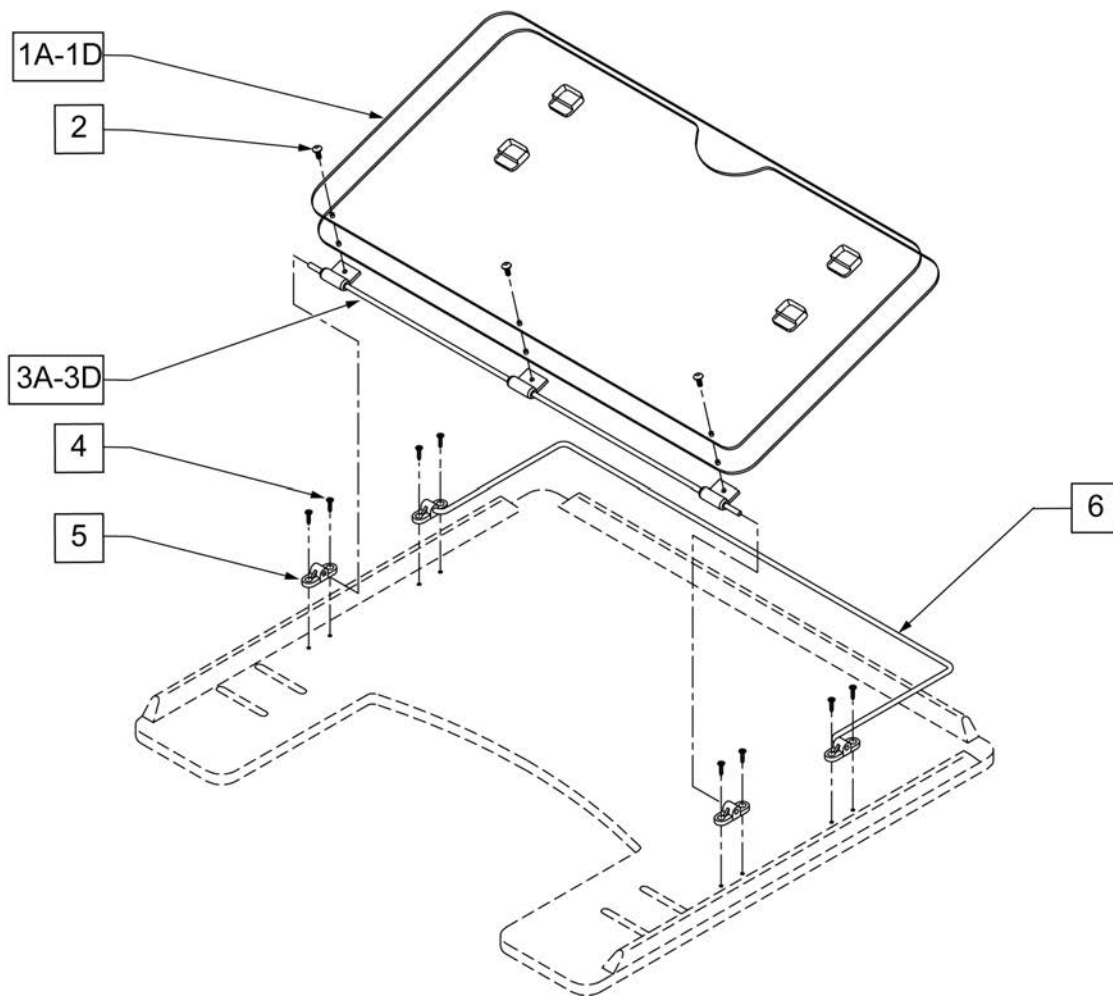


Pos.	Item Number	Description
1A,2	17916	9-1 STYLE 3 OVERLAY ONLY
1B,2	17917	9-2 STYLE 3 OVERLAY ONLY
1C,2	17918	9-5 STYLE 3 OVERLAY ONLY
1D,2	17939	9-6 STYLE 3 OVERLAY ONLY
2	226017	10-32 x 3/8 BH C/S BLK
3A	17922	9-1 HINGED OVERLAY PIVOT BAR
3B	17923	9-2 HINGED OVERLAY PIVOT BAR
3C	17924	9-5 HINGED OVERLAY PIVOT BAR
3D	17929	9-6 HINGED OVERLAY PIVOT BAR
4	2266	#6 x 1/2" FL DH PHL SCREW
5	4061	TRAY OVERLAY MOUNTING BRACKET
6A	17925	TRAY OVERLAY ROD
6B	17926	9-2 OVERLAY ROD
6C	17927	9-5 OVERLAY ROD
6D	17928	9-6 TRAY OVERLAY ROD
1A,2,3A,4,5,6A	17919	9-1 STYLE 3 OVERLAY ASSM
1B,2,3B,4,5,6B	17920	9-2 STYLE 3 OVERLAY ASSM
1C,2,3C,4,5,6C	17921	9-5 STYLE 3 OVERLAY ASSM
1D,2,3D,4,5,6D	17902	9-6 STYLE 3 OVERLAY ASSY

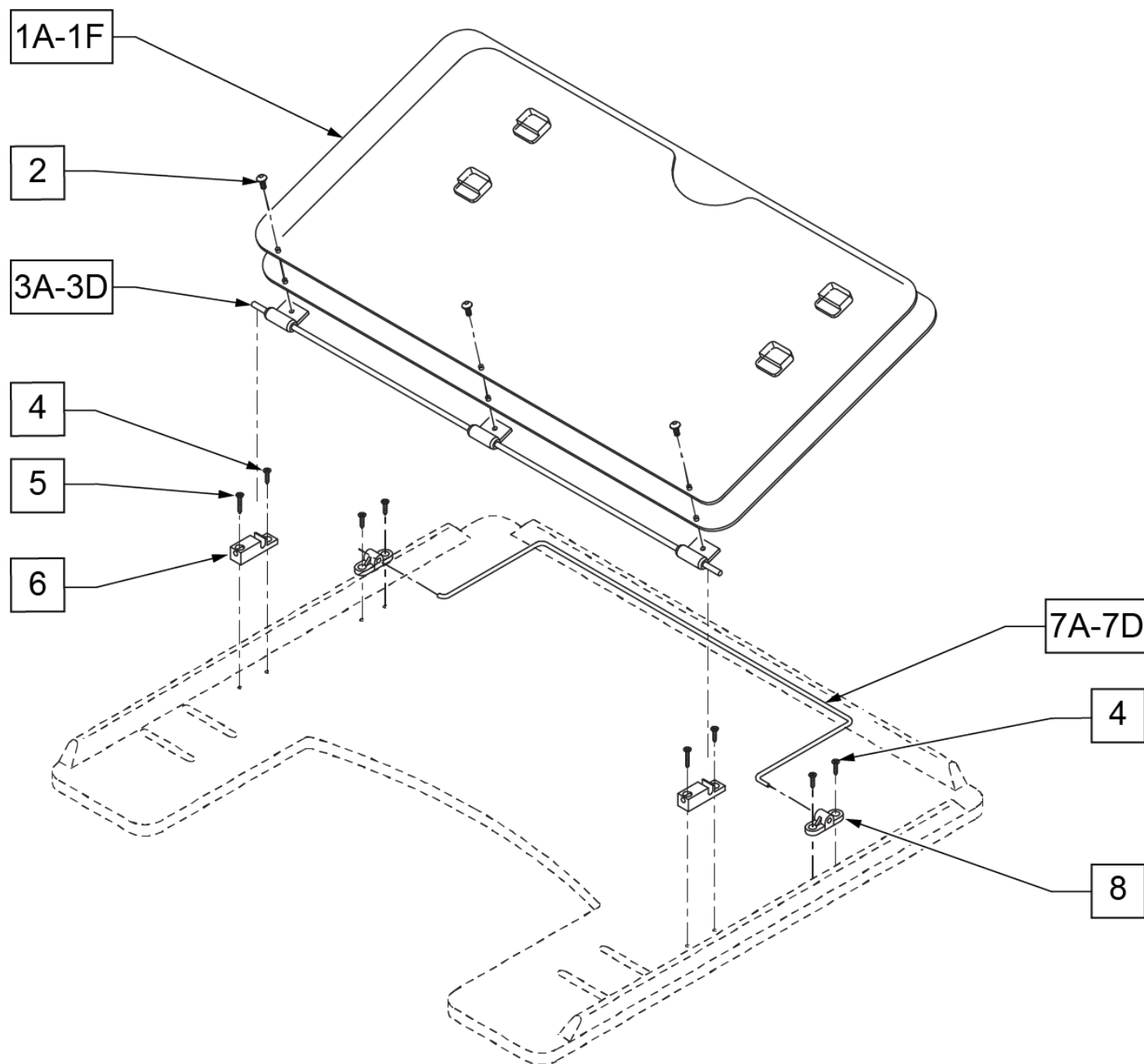


Pos.	Item Number	Description
1A	17935	9-1 STYLE 2 OVERLAY
1B	17936	9-2 STYLE 2 OVERLAY
1C	17937	9-5 STYLE 2 OVERLAY
1D	17938	9-6 STYLE 2 OVERLAY
1E	17930	STYLE #1 LEARNING BOARD
1F	17931	STYLE #1 FULL LEARNING BOARD
1A,2	17916	9-1 STYLE 3 OVERLAY ONLY
1B,2	17917	9-2 STYLE 3 OVERLAY ONLY
1C,2	17918	9-5 STYLE 3 OVERLAY ONLY
1D,2	17939	9-6 STYLE 3 OVERLAY ONLY
2	226017	10-32 x 3/8 BH C/S BLK
3A	17922	9-1 HINGED OVERLAY PIVOT BAR

Pos.	Item Number	Description
3B	17923	9-2 HINGED OVERLAY PIVOT BAR
3C	17924	9-5 HINGED OVERLAY PIVOT BAR
3D	17929	9-6 HINGED OVERLAY PIVOT BAR
4	2266	#6 x 1/2" FL DH PHL SCREW
5	230047	#6 x 3/4" FH PHIL S/M SCREW
6	14232	TRAY OVERLAY - QUICK RELEASE
7A	17925	TRAY OVERLAY ROD
7B	17926	9-2 OVERLAY ROD
7C	17927	9-5 OVERLAY ROD
7D	17928	9-6 TRAY OVERLAY ROD
8	4061	TRAY OVERLAY MOUNTING BRACKET
1A,2,3A,4-6,7A,8	17903	9-1 STYLE 3A OVERLAY ASSEMBLY
1B,2,3B,4-6,7B,8	17904	9-2 STYLE 3A OVERLAY ASSEMBLY
1,2,3C,4-6,7C,8	17905	9-5 STYLE 3A OVERLAY ASSEMBLY
1,2,3D,4-6,7D,8	17906	9-6 STYLE 3A OVERLAY ASSEMBLY

NOTE: ONE 19" RIB AND TWO 15 1/2" RIBS FIT THE EXTRA-SMALL, SMALL, 9-0 AND 9-1 TRAY TABLES. ONE 21" RIB AND TWO 19" RIBS FIT THE MEDIUM AND 9-2 TRAY TABLES. ONE 23 1/2" RIB AND TWO 21" RIBS FIT THE LARGE, 9-2, 9-5, AND 9-6 TRAY TABLES.

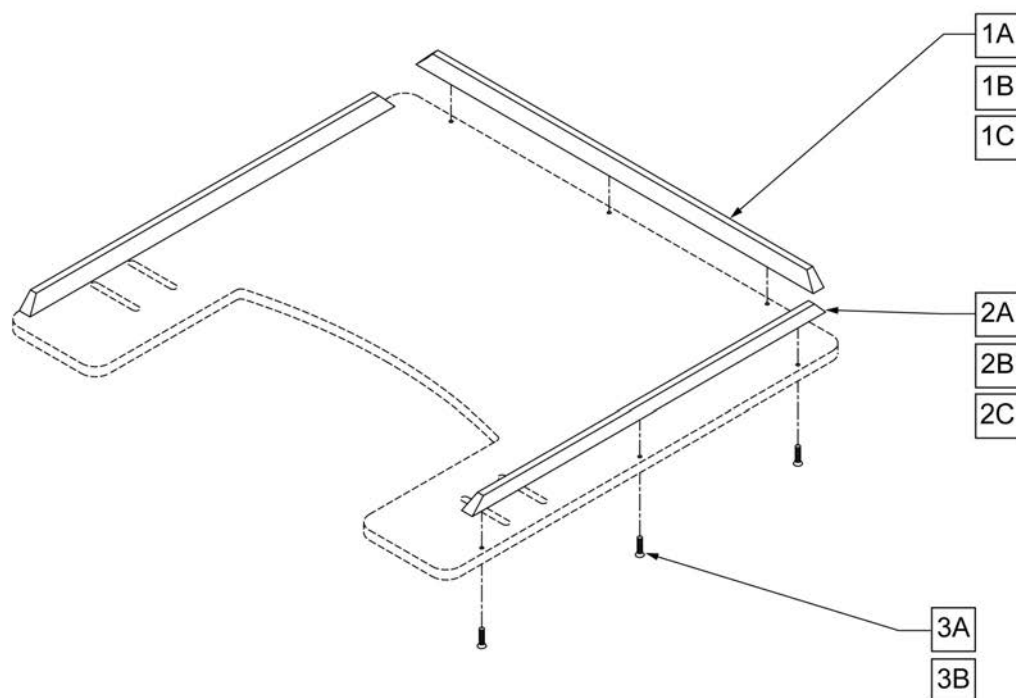


Pos.	Item Number	Description	Pos.	Item Number	Description
1A	9004	19" WOOD TRAY RIBS	2A	9004	19" WOOD TRAY RIBS
1A	9005	21" WOOD TRAY RIBS	2A	9005	21" WOOD TRAY RIBS
1A	9006-QU	23 1/2" WOOD TRAY RIBS	2B	9007-QU	15 1/2" HEAVY DUTY RIB FOR WOOD TRAY
1B	9010	19" HEAVY DUTY RIB FOR WOOD TRAY	2B	9010	19" HEAVY DUTY RIB FOR WOOD TRAY
1B	9013	21" HEAVY DUTY RIB FOR WOOD TRAY	2B	9013	21" HEAVY DUTY RIB FOR WOOD TRAY
1B	9016	23 1/2" HEAVY DUTY RIB FOR WOOD TRAY	2C	9008-QU	15 1/2" HEAVY DUTY RIB FOR 3/8" TUFFAK
1C	9011-QU	19" HEAVY DUTY RIB FOR 3/8" TUFFAK	2C	9011-QU	19" HEAVY DUTY RIB FOR 3/8" TUFFAK
1C	9014	21" HEAVY DUTY RIB FOR 3/8" TUFFAK	2C	9014	21" HEAVY DUTY RIB FOR 3/8" TUFFAK
1C	9017	23 1/2" HEAVY DUTY RIB FOR 3/8" TUFFAK	3A	2268	#6 x 5/8" OVAL PHILLIPS SCREW
2A	9003	15 1/2" WOOD TRAY RIBS	3B	230047	#6 x 3/4" FH PHIL S/M SCREW

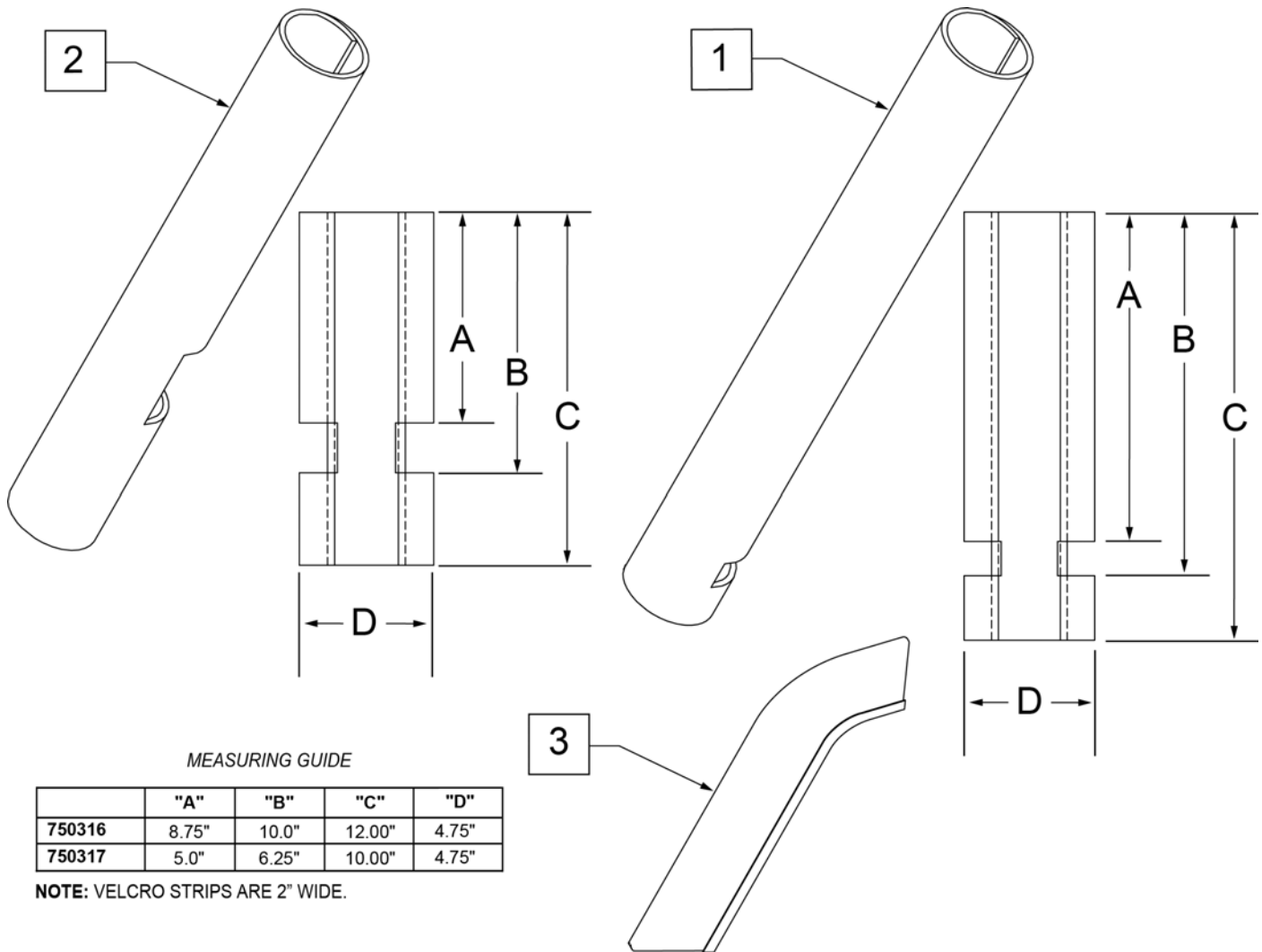



Pos.	Item Number	Description	Remarks
1	750316	IMPACT GUARDS NEOPRENE FIXED (PR)	PAIR-Fixed Front Frame
2	750317	IMPACT GUARDS NEOPRENE S/A	PAIR-S/A Footrest & Q7
3	109231	GEL VELCRO PAD HANGER/FRAME	PAIR- fits S/A & Fixed Front Frame
4	108315	GEL PAD SA ARM	Single, fits Padded SA Armrest

Measuring Guide

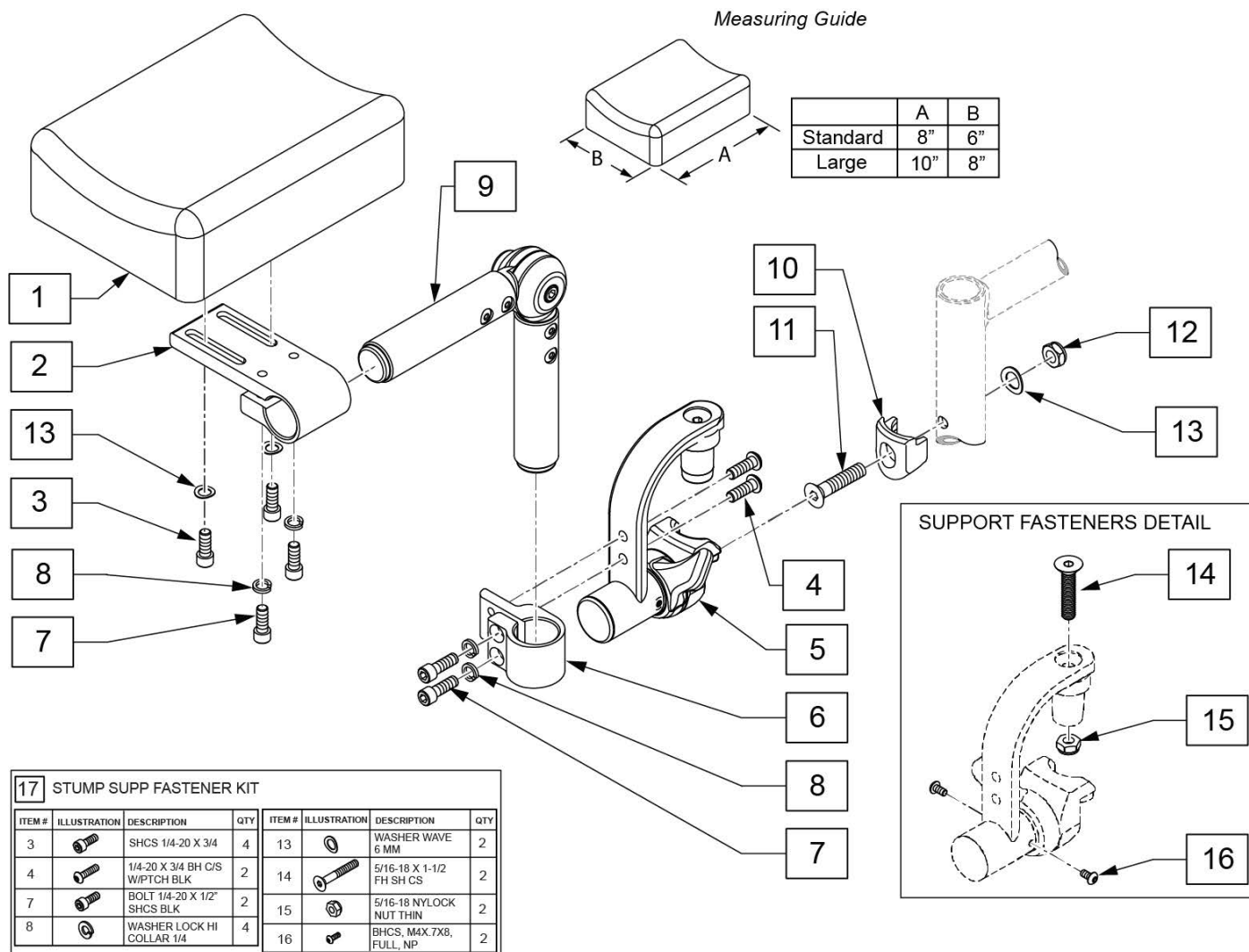


Pos.	Item Number	Description	Remarks
1,3,13	161563	PAD W/FASTENERS STD	Single Pad, 6X8
1,3,13	161564	PAD W/FASTENERS LRG	Single Pad, 8X10
1-9,13	161565	STUMP SUPP RETRO LRG PAD RH	Single Pad, 8X10
1-9,13	161566	STUMP SUPP RETRO LRG PAD LH	Single Pad, 8X10
1-9,13	161567	STUMP SUPP RETRO STD PAD RH	6X8
1-9,13	161568	STUMP SUPP RETRO STD PAD LH	6X8
2-9,13	162051	STUMP SUPPORT W/O PAD RH	No Pad
2-9,13	162052	STUMP SUPPORT W/O PAD LH	No Pad
10-13	119514	LATCH RECEIVER KIT	for 1 side
17	161569	STUMP SUPP FASTENER KIT	for 1 side

NOTE: THE ANGLE ADJUSTABLE IV POLE IS NOT COMPATIBLE WITH THE TRANSIT OPTION.

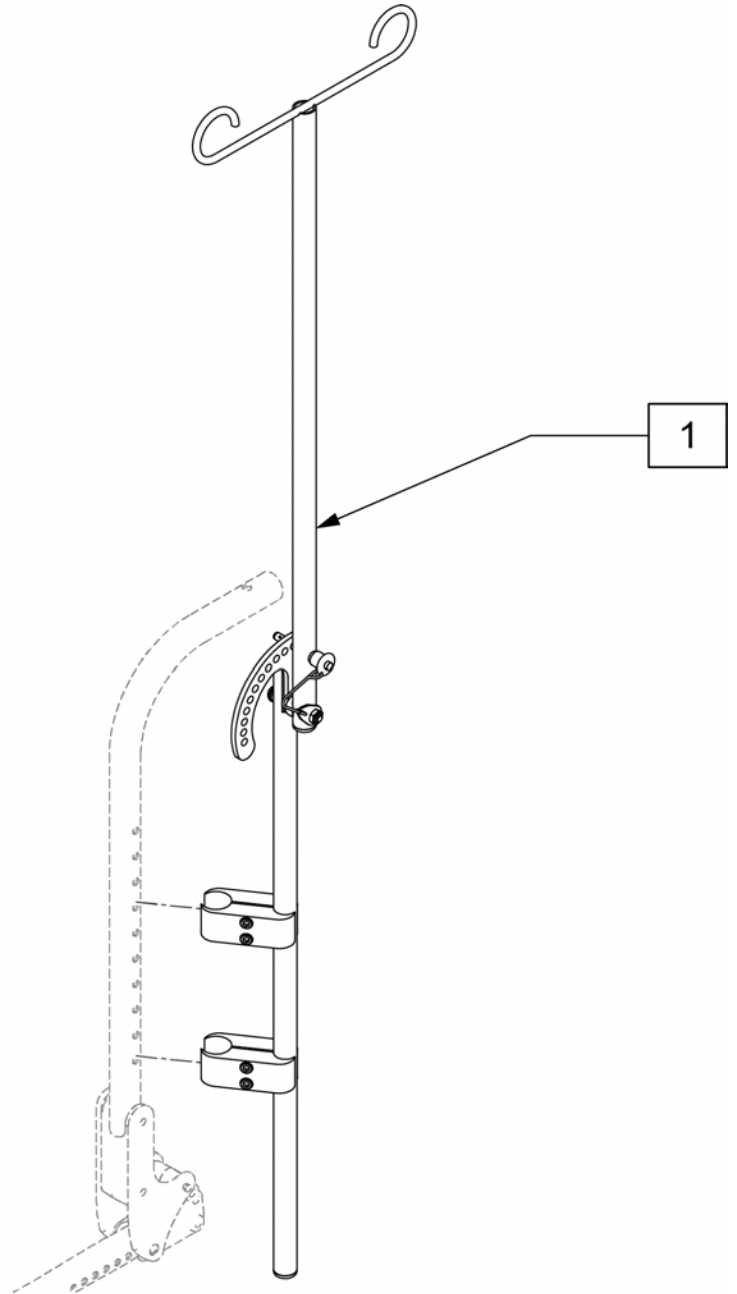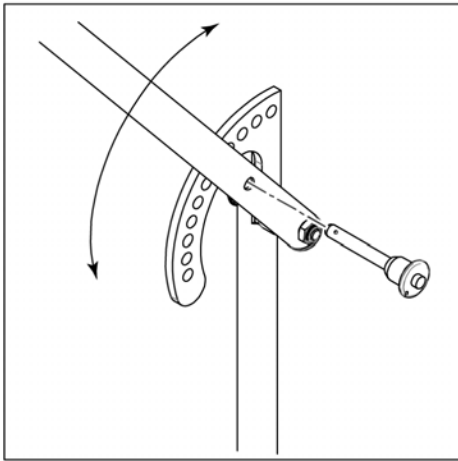

Pos.	Item Number	Description
1	884140	ANGLE ADJUSTABLE IV POLE


NOTE: AVAILABLE WITH STANDARD BACKREST ONLY AND WILL NOT BE COMPATIBLE WITH CHAIRS THAT ARE EQUIPPED WITH S/A HEIGHT ADJ STROLLER HANDLES, TRANSIT OPTION, JAY 3 BACK OR RECLINING BACKREST. ACCOMMODATES SIZE D & E OXYGEN TANKS.

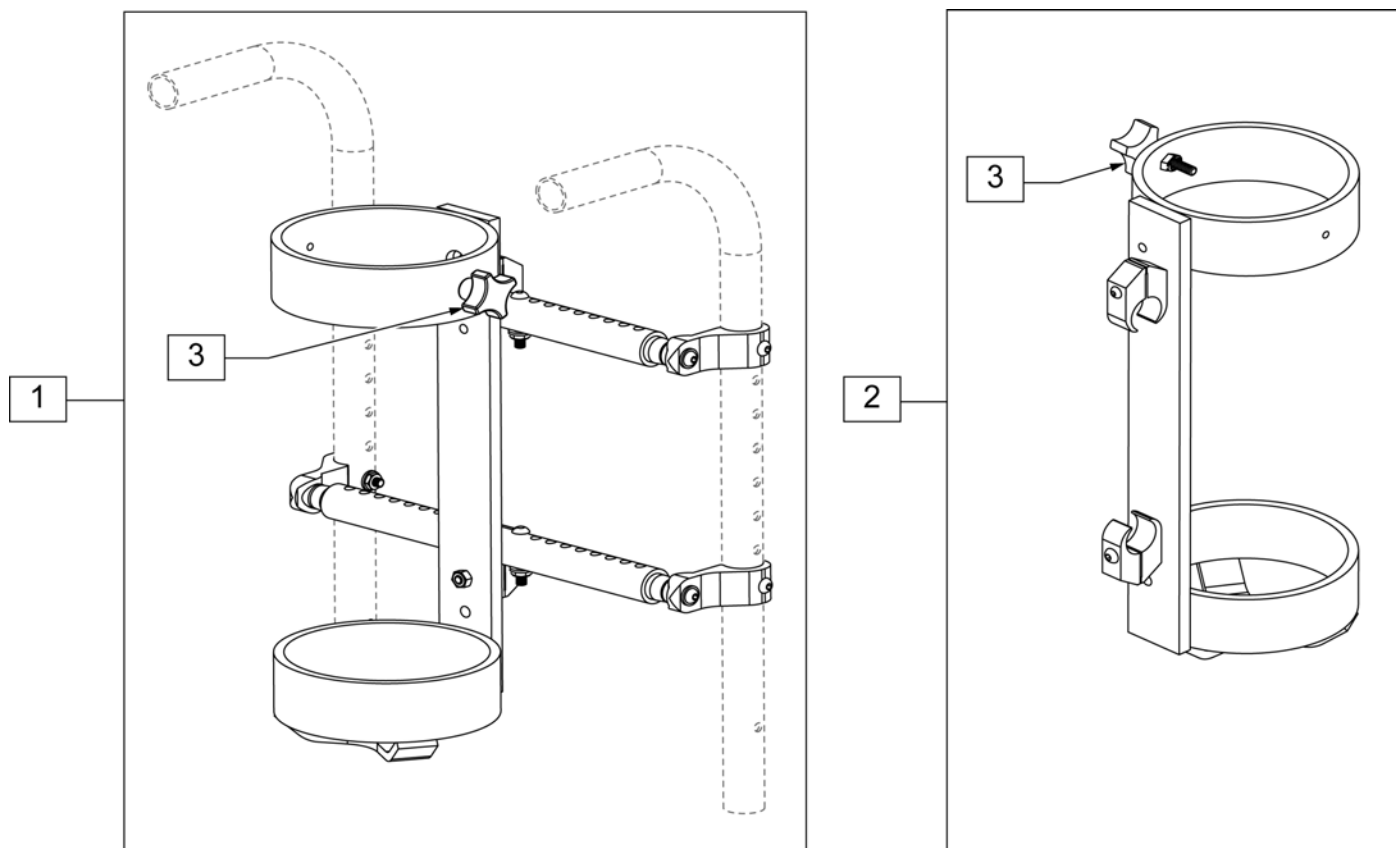


Pos.	Item Number	Description	Remarks
1	884136	O2 HOLDER W/ BACKREST MOUNT 14" - 22" WIDE	ACCOMMODATES SIZE D & E OXYGEN TANKS
2	884135	O2 HOLDER W/ 1" CLAMPS	ACCOMMODATES SIZE D & E OXYGEN TANKS
3	343080	FOUR-PRONG KNOB STUD 1/4 -20	

Note: Water Bottle Cage clamp fits 3/4" to 1 1/2" round tubing, 7R/Q7 and Xenon oval tubing as well as Double "D" tubing; Max bottle diameter is 3 3/4".


Note: Use rubbing alcohol to clean area of frame where adhesive rubber strip (item 3) is to be located prior to instalation to insure best adhesion.

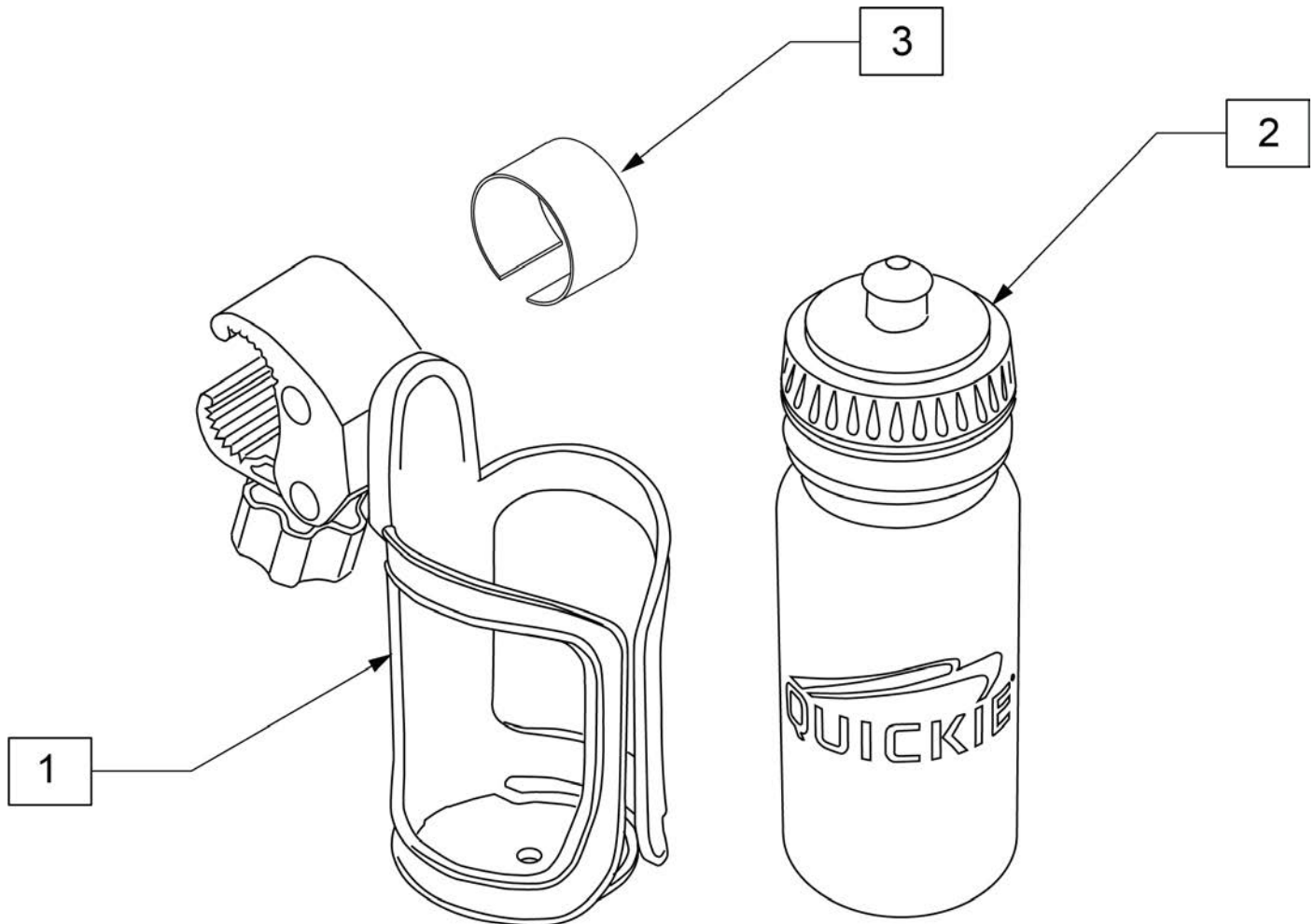

Pos.	Item Number	Description	Remarks
1	252135	WATER BOTTLE CAGE W/ CLAMP	Black
2	252136	QUICKIE WATER BOTTLE	Black
3	123526	ADHESIVE RUBBER PAD	
1-3	163680	WATER BOTTLE W/ CAGE AND CLAMP	

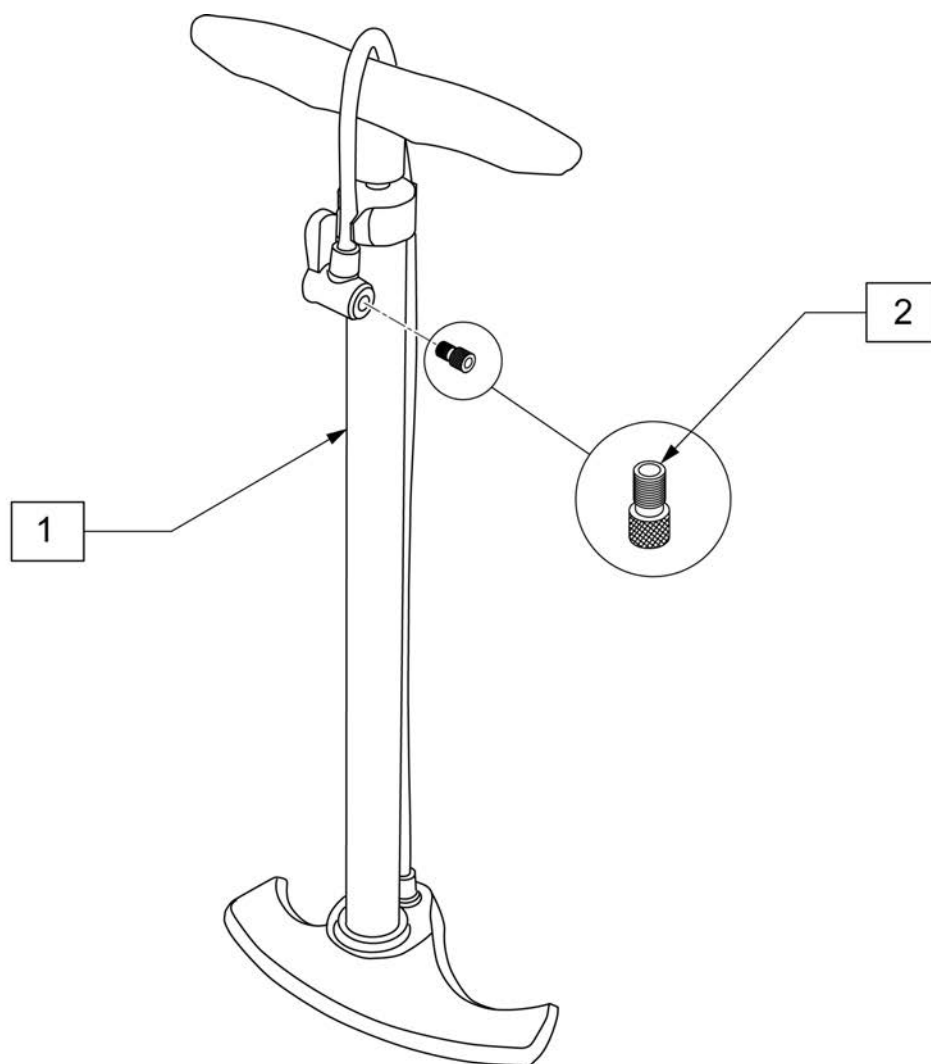

Pos.	Item Number	Description
1	389050	TIRE PUMP
2	381230	VALVE ADAPTER PRESTA

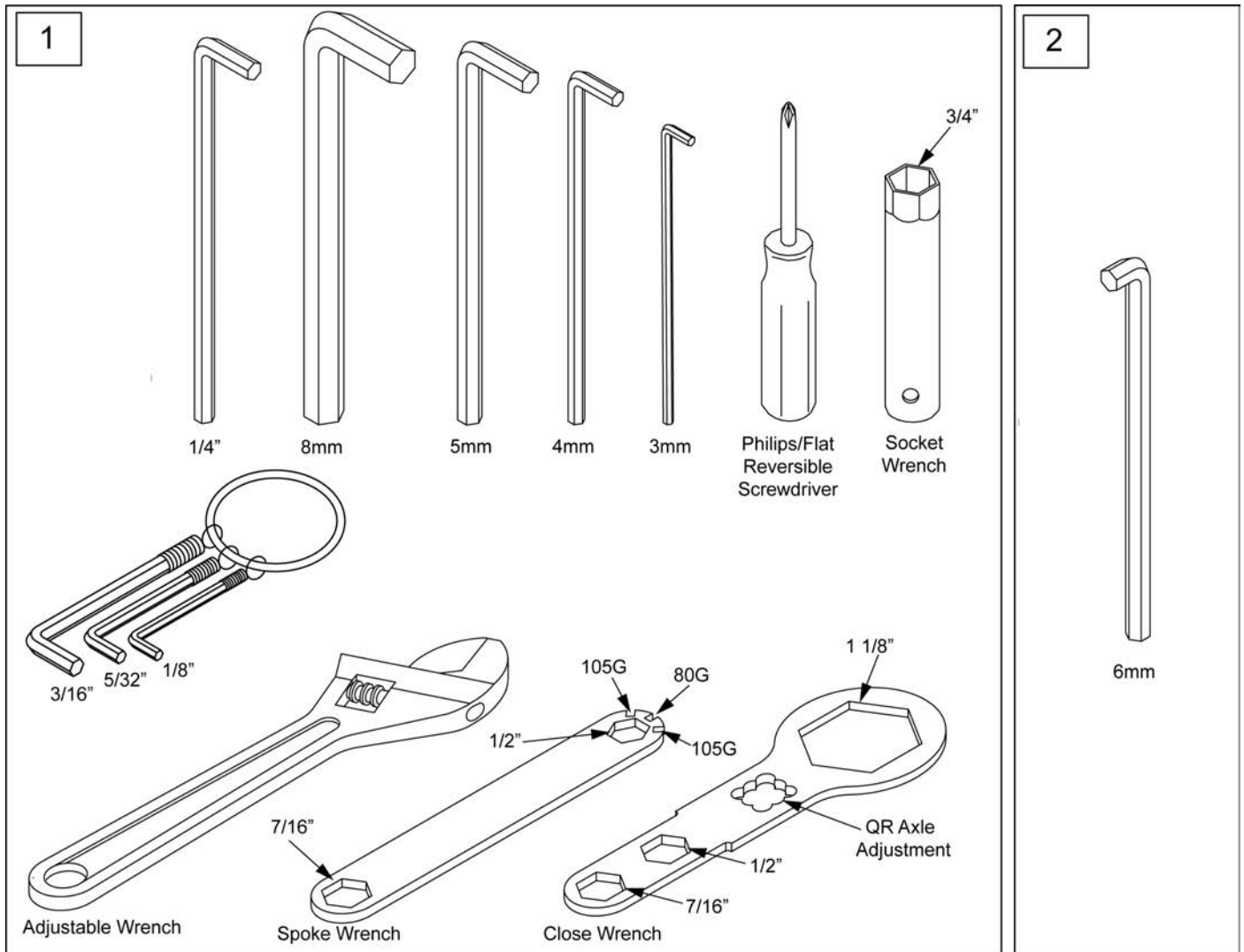

Pos.	Item Number	Description	Remarks
1	108314	QUICKIE COMBO TOOL KIT	Includes pouch
2	113403	6MM ALLEN WRENCH L, 7MM SHORT	caster spline chairs only