

INSTALLATION AND ADJUSTMENT OF THE LATERAL HIP SUPPORT

The left and right hip supports can be mounted to the backshell using 1x 2-hole bracket, 2x washers, and 2x M6x1x12 bolts per side.

1. Attachment location

1 2

- The hip support mounting points can be found behind the back support padding on the seating shell in locations shown (A)

2. Attaching the hip support pad/bracket

2 3

- Mount the lateral hip support (B) to the backshell by placing the lateral bracket on the inside of the backshell, and then the 2-hole nut (C) on top of the lateral bracket.
- Thread the M6 screws thru the back and into the 2-hole nut (C).
- Using a 4mm hex key, snug the screws.

3. Hip support pad adjustment

3

- You can move the pad and bracket horizontally in and out for optimum fit.
- After adjusting to proper fit, use the 4mm Hex Key and tighten the screws on each side
- If necessary repeat the steps for the other side.

