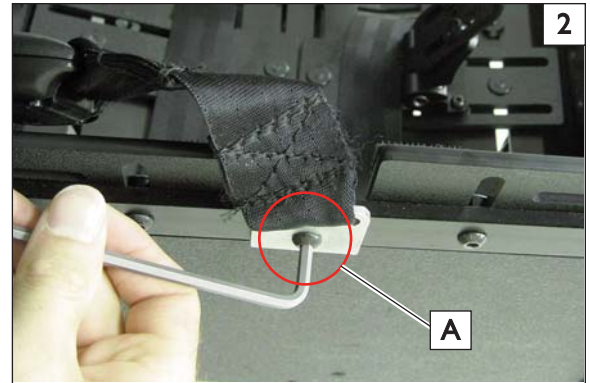
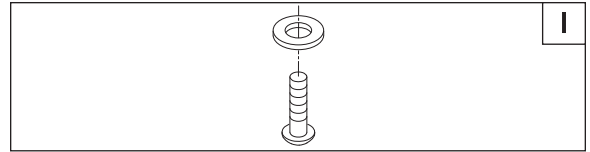


INSTALLATION AND ADJUSTMENT OF THE MEDIAL SUPPORT


The Medial support can be mounted to the seatpan using 1x M6x1x16 bolt, and 1x washer.

1. Remove the Lower Transit Strap **2**

- The medial support mounting point can be found under the front of the seat pan in location shown (A).
- Using the 4mm hex key, remove the screw holding the lower transit strap, and discard it. This is where the medial support bracket will attach.

2. Assembly of parts before attaching the bracket **1** **3**

- Assemble the longer screw (B), washer (C), medial bracket (D), and seat belt (E) as shown in (figure 3).

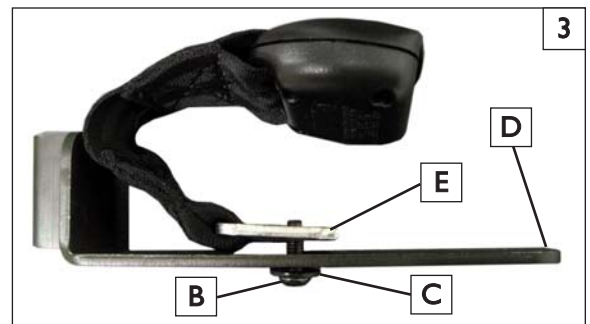2. Attach and adjust the medial support pad/bracket **4** **5**

- Using the 4mm hex key, mount the medial support bracket at the same location as the belt attachment point (A) on the seat pan.
- Make sure the bracket (D) is straight, and using the 4mm hex key, tighten the screw (B) to a torque of 40-45 in-lbs.
- Insert the hip abductor/Medial pad post (F) into the receiver (G).

