

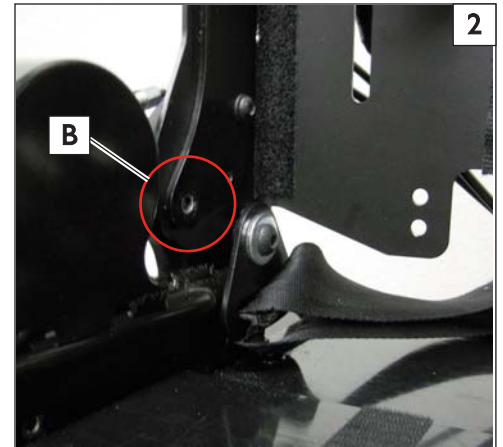
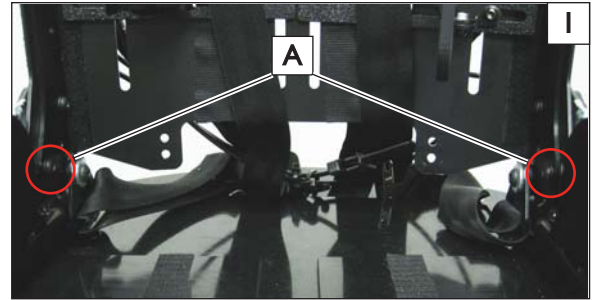
INSTALLING THE PELVIC POSITIONING BELT

⚠ WARNING

The pelvic belt is not transit approved and should not be used as a substitute for the Anterior Trunk H strap support when in a transit situation.

1. Location and Attachment 1 2

- a. Using a 4mm hex key and 10mm wrench, remove the bolts(A) circled in (Figure 1). These will be the lower mounting points.
- b. Bolt removed(B) (Figure 2)

**2. Attaching the Pelvic belt** 3 4

- a. Reattach the bolt (A) for each side, placing the washer (C) in front of the pelvic positioning belt bracket (D) as shown.
- b. The completed pelvic positioning belt is shown in (Figure 4).

3. NOTE: For chairs with recline.

- a. Check to see if the backshell is difficult to recline after tightening the bolts. If the recline mechanism is stiff, you may loosen the screw(A) slightly until the recline functions freely.

