

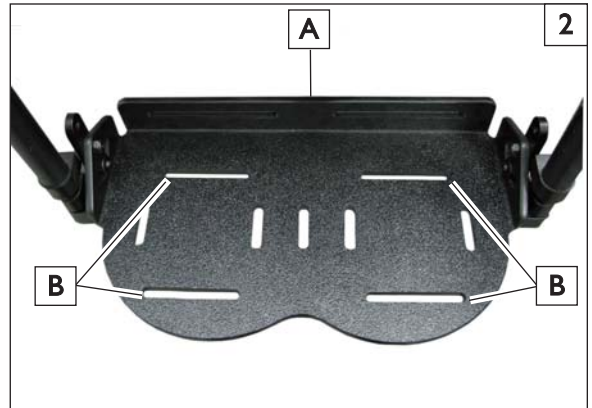
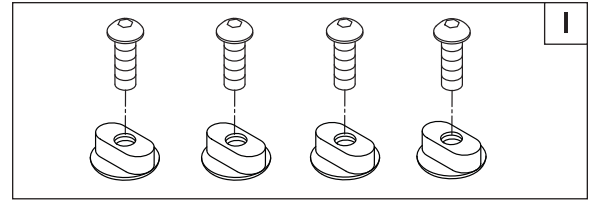
INSTALLATION AND ADJUSTMENT OF THE SHOE HOLDER

The left and right shoe holders can mount to the footrest plate using 2x T-nuts and 2x M6x1x12 bolts per side.

1. Mount the shoe holder

1 **2**

- Make sure that height and angle adjustment of the footplate (A) is optimal before proceeding to the next steps.
- While holding the T-nuts in the slots (B), thread the button head screws into the T-nuts.
- Using a 4mm hex key, snug but do not tighten the screws so that the feet can be placed in the best location for support and posture.

**2. Adjust the shoe holder**

3 **4**

- The shoe holder can slide left or right in the slot (Figure 3) and can be set at an angle as shown in (Figure 4)
- Once the correct placement of the shoe holder is determined, use the 4mm hex key to tighten the M6 screws.
- If necessary, repeat the steps for the other side.

