



PARTS MANUAL

JANUARY 2023

Cover	1
cover	1
ZIPPIE XCAPE	6
GLOSSARY OF ABBREVIATIONS	6
BASE FRAME	6
FRAME	6
STANDARD SWING-AWAY SIDE FRAME	7
FIXED SIDE FRAME	9
HEMI SWING-AWAY SIDE FRAME	11
CROSS FRAME	13
WC19 TRANSIT KIT-X'CAPE	14
AXLE PLATES AND AXLES	16
AXLE PLATE	16
QUICK RELEASE AXLES	18
CASTERS AND FORKS	18
CASTER PLATE AND MULTI-POSITION FORK ASSEMBLY	19
MICRO CASTERS	21
ALUMINUM SOFT ROLL CASTERS	22
THREE SPOKE CASTERS	23
ANTI-TIPS	24
ANTI-TIP X'CAPE	25
FRONT ANTI-TIP RECEIVER X'CAPE	27
WHEEL LOCKS	27
WHEEL LOCK PUSH X'CAPE	28
WHEEL LOCK PULL X'CAPE	29
WHEEL LOCK COMPACT X'CAPE	30
HUB LOCK FOOT RELEASE MAG WHEEL X'CAPE	31
HUB LOCK FOOT RELEASE SPOKE WHEEL X'CAPE	33
HUB LOCK LEVER RELEASE MAG WHEEL X'CAPE	35
HUB LOCK LEVER RELEASE SPOKE WHEEL X'CAPE	37
WHEELS AND TIRES	38
WHEELS	38
12" MAG	39
16" MAG	40
TRADITIONAL 6 SPOKE MAG	41
5 SPOKE MAG	42
LITE SPOKE	44
ULTRA LITE SPOKE	46
SPINERGY SPOX (18 SPOKE)	47
SPINERGY LX (12 SPOKE)	48
SPINERGY SPORT LT EX (18 SPOKE)	50
TIRES	51
PNEUMATIC & PNEU. AIRLESS TIRE	52
FULL POLY TIRE	53
LOW POLY TIRE	54
SHOX TIRE	55
SCHWALBE MARATHON PLUS TIRE	56
SCHWALBE ONE TIRE	57
HANDRIMS	57
ALUMINUM ANODIZED (SCREW MOUNT)	58
ALUMINUM ANODIZED (TAB MOUNT)	59
PLASTIC COATED (REG TAB MOUNT)	60
PLASTIC COATED (SCREW MOUNT)	61
NATURAL FIT (SCREW MOUNT)	62
NATURAL FIT (TAB MOUNT)	64
PLASTIC COATED 5 SPOKE MAG (SCREW MOUNT)	66
ALUMINUM ANODIZED 5 SPOKE MAG (SCREW MOUNT)	67
TUBES	67
TUBES AND AIRLESS INSERTS	68
RIMSTRIPS	68
RIM STRIP	69
SEATING AND POSTITIONING	69
SEAT	69
SEAT SLING	70
1" SEAT CUSHION	73
2" SEAT CUSHION	75
BACK	77
BACK FRAMES	77
FOLDING BACK FRAME (HEIGHT ADJUSTABLE)	78
BACKREST STABILIZER BAR X'CAPE	80

DYNAMIC BACK FRAME.....	82
FIXED HEIGHT FOLDING PUSH HANDLE BACK FRAME	84
FIXED HEIGHT PUSH HANDLE BACK FRAME	86
BACK UPHOLSTERY	87
STANDARD BACK UPHOLSTERY ZIPPIE	88
INTEGRAL PUSH HANDLE BACKREST UPHOLSTERY ZIPPIE.....	90
TENSION ADJUSTABLE BALLISTIC NYLON UPHOLSTERY ZIPPIE	92
TENSION ADJUSTABLE 3DX VENTED UPHOLSTERY PED	95
HANGERS & FOOTRESTS	96
Z-FINITY HANGERS.....	96
Z-FINITY SWING I/O EXTENSION MOUNT HANGERS.....	97
Z-FINITY SWING I/O DIRECT MOUNT HANGERS	98
DIVIDED FOOTREST EXTENSION MOUNT	99
DIVIDED FOOTREST DIRECT MOUNT.....	103
FLIP-UP FOOTREST EXTENSION MOUNT	105
FLIP-UP FOOTREST DIRECT MOUNT.....	108
FRONT MOUNT HANGERS	109
80 DEG FRONT MOUNT SWING IN/OUT HANGER	110
90 DEG FRONT MOUNT SWING IN/OUT HANGER	112
KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT	114
ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT.....	117
FRONT MOUNT HANGER EXT.....	119
SWING AWAY ELR EXT MOUNT (LATCH STUD STYLE) (DISC. 2/6/17)	119
EXTENSION MOUNT ELR RETROFIT (LATCH STUD STYLE) (DISC. 2/6/17)	120
EXTENSION MOUNT ELR REPLACEMENT PARTS (LATCH STUD STYLE) (DISC. 2/6/17)	121
ELR & ALR CALF PAD 14 -15"	123
ELR & ALR CALF PAD 16-20"	124
SWING IN-OUT ELR & ALR (PLUNGER STYLE) (EFFECTIVE 2/6/17)	124
EXTENSION MOUNT ELR & ALR (PLUNGER STYLE) (EFFECTIVE 2/6/17).....	124
SWING IN-OUT EXT MOUNT ELR & ALR RETROFIT (PLUNGER STYLE) (EFFECTIVE 2/6/17)	125
SWING IN-OUT EXT MOUNT ELR REPLACEMENT PARTS (PLUNGER STYLE) (EFFECTIVE 2/6/17)	126
SWING IN-OUT ALR EXT MNT REPLACEMENT PARTS (PLUNGER STYLE)(EFFECTIVE 2/6/17)	128
ELR & ALR CALF PAD 14 -15"	130
ELR & ALR CALF PAD 16-20"	131
EXTENSION MOUNT FOOTPLATES.....	131
COMPOSITE FOOTPLATE.....	132
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE	136
ADULT ALUMINUM ANGLE-ADJ FOOTPLATE EXT MOUNT.....	139
KIDS ANGLE ADJUSTABLE FOOTPLATE EXT MOUNT	142
ELR/ALR ALUMINUM FOOTPLATE	144
ARMRESTS AND RECEIVERS	145
ARMRESTS.....	145
KIDS T POST ARMREST AND SIDEGUARD.....	146
SINGLE POST HEIGHT ADJUSTABLE ARMREST	147
FLIP UP ARMREST ADJUSTABLE HT BACK.....	148
ADJ LOCKING & LENGTH ADJ LOCKING FLIP-UP ARMREST for HEIGHT ADJ BACK.....	149
FLIP-UP ARMREST for HEIGHT ADJUSTABLE BACKREST	151
FLIP UP ARMREST FIXED HT BACK	152
FLIP-UP ARMREST	153
ADJUSTABLE LOCKING FLIP-UP ARMREST (USE ON DAB FRAMES ONLY)	154
LENGTH ADJUSTABLE LOCKING FLIP-UP ARMREST (USE ON DAB FRAMES ONLY).....	155
ARMREST RECEIVERS	156
SINGLE POST ARMREST RECEIVER X'CAPE.....	157
ACCESSORIES	157
BACKPACKS AND SEAT POUCHES.....	157
BACKPACKS AND SEAT POUCHES.....	158
POSITIONING BELTS	158
AIRCRAFT BUCKLE POSITIONING BELTS & ATTACHMENT KITS	159
AUTO BUCKLE POSITIONING BELT & ATTACHMENT KITS.....	161
1 INCH PADDED DOUBLE PULL HIP BELT	163
1 INCH COMFORT FIT POSITIONING BELT	164
SIDE GUARDS.....	164
PLASTIC SIDE GUARD X'CAPE	165
SPOKE GUARDS.....	165
DESIGNER SPOKE GUARDS.....	166
SPOKE GUARDS ZIPPIE (Black & Clear)	168
TRAYS	169
TUFFAK TRAY (MANUAL & PEDIATRIC CHAIRS)	170
XTENDER JWX-2.....	171
GEN 2 XTENDER AXLE PLATE & RETRO KIT XCAPE	172


XTENDER GEN 2 WHEEL PARTS.....	174
DRIVE UNIT INTERNAL PARTS	175
XTENDER RIGHT HAND CONTROL SPARE PARTS	176
BATTERY AND CHARGER	177
BATTERY BACKPACK PARTS	178
XTENDER SPOKE GUARD	179
CANOPY	180
QUICKIE CADDY	181
CRUTCH HOLDER X'CAPE	182
IMPACT GUARD	183
ONE ARM DRIVE X'CAPE	184
TOOL KIT	186

Note: see Informations Panel for file.

1AD	One Arm Drive	CZ	Cozy	INH	Inhibit	PCB	Pediatric Center of Gravity (AKA: Zippie Iris)	S/A	Swing Away
3DCR	3DX Microclimatic cover for Cryo style inserts	DAB	Depth-adjustable Back	INORG	Inorganic coating	PCM	Power Center Mount	S/F	Side Frame
AAA	Adult Angle Adjustable	DB	Dynamic Back	INSRT	Insert	PED	Pediatric	SCW	Screw
AACM	Angle Adjustable Center Mount	DBL	Double	I/O	Swing In Out	PEL	Pelvic	SDLE	Saddle
ABD	Abductor	DC	Deep Contour	IRIS	Intelligent Rotation in Space	PH	Push Handle	SG	Side Guard
AC	All Court	DL	Desk Length	ISM	Intelligent Seating Module	PHIL	Phillips	SGTG	Super Grip Thumb Guard
AD	Arm Drive	DP	Dual Post Armrest	JA	Jay Active	P/L	Pin Lock	SHCS	Socket Head Cap Screw
ADJ	Adjustable	D-Post	Dual Post Armrest	JNR	Junior	PL	Plated	SHSS	Socket Head Set Screw
ADPT	Adapter	D-PST	Dual Post Armrest	KAA	Kids Angle Adjustable	PLL	Pull	SHLDR	Shoulder
AE	Air Exchange	DRP	Drop (seat)	KDS	Kids	PLT	Plate	SHT	Short
AHAB	ASAP II	DRV	Drive	KT	Kit (usually consists of all mounting hardware for one side)	PLST	Plastic	SLD	Solid
ANG	Angle	DTT	Diagnostic Test Tool	KVLR	Kevlar	PLY	Plywood	SLNG	Sling
ALR	Articulating Legrest	DY	Dynamic Back	KZ	Quickie Kidz/Zippie Kidz	PNEU	Pneumatic	SLTD	Slotted
ALUM	Aluminum	EA	Each	L	Left	PNK	Pink	SLV	Sleeve
AMH	Adjustable Mounting Hardware	ELR	Elevating Legrest	LAT	Lateral	PNT	Paint	SM	Small
ANO	Anodized	EXT	Extension	LCK	Lock	PNTD	Painted	SNGL	Single
ASAP	Adjustable Seating and Positioning	EXTND	Extended	LFH	Left Front Horizontal	POLY	Polyurethane	SOC	Socket
ASL	Adaptive Switch Laboratories	F	Folding Frame	LFT	Left	POST	Posterior	SPCR	Spacer
ASM	Assembly (includes other parts)	FHCS	Flat Head Cap Screw	LFV	Left Front Vertical	PR	Pressure Relieving	SPEC CLR	Specify Color
ASSM	Assembly	FL	Full Length	LH	Left Hand	PR	Power Recline	SPK	Spoke
ASSMLY	Assembly	FLD	Folding	LH	Left Horizontal	PR	Pair	SPR	Super Grip Handrim
ASSY	Assembly	FL FT	Flip-up Footrest	LK	Lock	(PR)	Pair	SQZ	Squeeze
AT	Anti-tip	FLP	Flip	LNG	Long	PRCR	Premium Leatherette cover for Cryo style inserts	SS	Stainless Steel
ATT	Attachment	FLRD	Flared	LO	Lift Off	PRE-ISC	Pre-Ischial	ST	Seat
BB	Basketball	FM	Foam	L/O	Lift Off	PREM	Premium Leatherette cover material	S/TAP	Self Tapping
BC	Breezy Chrome	FM	Front Mount	LP	Loop	PROJ	Projection	STCR	Stretch Cover for Cryo Style inserts
BCK	Back	FNDR	Fender	LPS	Loops	PRPL	Purple	STD	Standard
BCKBRD	Backboard	FPH	Folding Push Handle	LRG	Large	PTL	Partial	STL	Style
BH	Button Head	FT	Foot	LRH	Left Rear Horizontal	PSH	Push	STR	Stretch cover material
BHCS	Button Head Cap Screw	F/REST	Footrest	LRV	Left Rear Vertical	PT	Point	STRLR	Stroller
BKL	Buckle	FRNT	Front	LS	Low Sheer	PTTRN	Pattern	STRT	Straight
BKRST	Backrest	FT PLT	Footplate	LT	Long Tab	PWR	Power	SUP	Support
B-LOCK	Back Lock	FTRST	Footrest	LT	Natural Fit Lite Handrims	Q1	Quickie 1	SYS	System
BLD	Build Up	F/UP	Flip-up	LTRL	Lateral	Q2	Quickie 2	T	Tab
BLK	Black	FWD	Front Wheel Drive	LUMB	Lumbar	Q2HP	Quickie 2 High Performance	TG	Thumb Guard (std)
BLU	Blue	FXT	Fixed	LV	Left Vertical	QC	Quickie Carbon	THRD	Threaded
BR	Breezy	G8	Grade 8 higher grade = stronger	LW	Low	QR	Quick Release	Ti	Titanium
BRD	Board	GP	Quickie GP	LX	Quickie LX	QRx	Quickie RX	TL	Tail
BRNG	Bearing	GPV	Quickie GPV	LXI	Quickie LXi	QTI	Quickie Titanium	TLT	Tilt Lift Comb
BRKT	Bracket	GNRC	Generic	MACH	Machine	QT/QTs	Quickie Tilt & Space	TNNS	Tennis
BSHNG	Bushing	H	Hole	MECH	Mechanism	QUAD	Quadrant Pad	TIS	Tilt-in-Space
BTTN	Button	HB	High Brake	MED	Medium	R	Right	TIS	Tilt-in-Space
BTRY	Battery	HD	Head or Heavy Duty	mm	Millimeter	RCLNR	Recliner	UNIV	Universal
BZ	Breezy	HVD	Heavy Duty	MNT	Mount	REC	Receiver	UNTHRD	Unthreaded
CAA	Composite Angle Adjustable	HDRST	Headrest	MNTG	Mounting	REG	Regular	UPH	Upholstery
CG	Center of Gravity	IDWR	Hardware	MOD	Modify or Modification	REL	Release	VERT	Vertical
CGT	Center of Gravity Tilt (AKA: IRIS)	HEX	Hexagon	M/S	Machine Screw	REV	Reverse	W	With
CHRG	Charger	HH	Hem-Height or Hex Head	MWD	Mid Wheel Drive	RH	Right Hand	W/	With
CJSM	Color Joystick Module	HK	Hook	Nat.Fit	Natural Fit Handrim	RH	Right Horizontal	WD	Wide
CLMP	Clamp	HL	Heel Loops	NAUG	Naugahyde	RFH	Right Front Horizontal	W/F	Water Fall
CLNCHR	Clincher	HM	High Mount	NF	Non-folding	RFV	Right Front Vertical	W/O	Without
CLR	Clear	HNDL	Handle	NF	Natural Fit Handrim	RNUT(S)	Rivnut(s)	WHL	Wheel
COMP	Complete	HNGR	Hanger	NFLT	Natural Fit Lite Handrim	RoHS	Restriction of Hazardous Substances	WDMNT	Weldment
CMPT	Composite	HP	High Pressure	NR	Non-recessed	RR	Rear	XS	Cross Spoke
CRM	Chrome	HR	Handrim	OBLIQ	Obliquity	RRH	Right Rear Horizontal	YLW	Yellow
C/S	Cap Screw	HRDWR	Hardware	OD	Outer/Outside Dimension	RRV	Right Rear Vertical	Z2	Zippie 2
C-S	Countersunk	HRNSS	Harness	O/S	Old Style	RT	Right	ZN	Zinc
CSTR	Caster	HS	High Speed	OUTBRD	Outboard	RV	Right Vertical	ZP	Zippie
CUSH	Cushion	HT	Height or Heat Treated	P1	Quickie P100	RV	Quickie Revolution	ZP2	Zippie 2
CV	Conventional	ID	Inner/Inside Dimension Diameter	P2	Quickie P200	RWD	Rear Wheel Drive		
CVR	Cover	IN	In/Out (for swing in/out hangers)	P3	Quickie P300	SA	Swing Away		
CVRS	Covers	INBRD	Inboard	P5	Quickie/Zippie P500				
CYL	Cylinder	INC	Incontinent						

Pos.	Item Number	Description	Remarks
		GLOSSARY OF ABBREVIATIONS	see attachment in Informations Panel

Note: ORDER PART 14 (TUBE CAP QTY 2) WHEN SWING-AWAY FOOTRESTS ARE NOT INSTALLED.

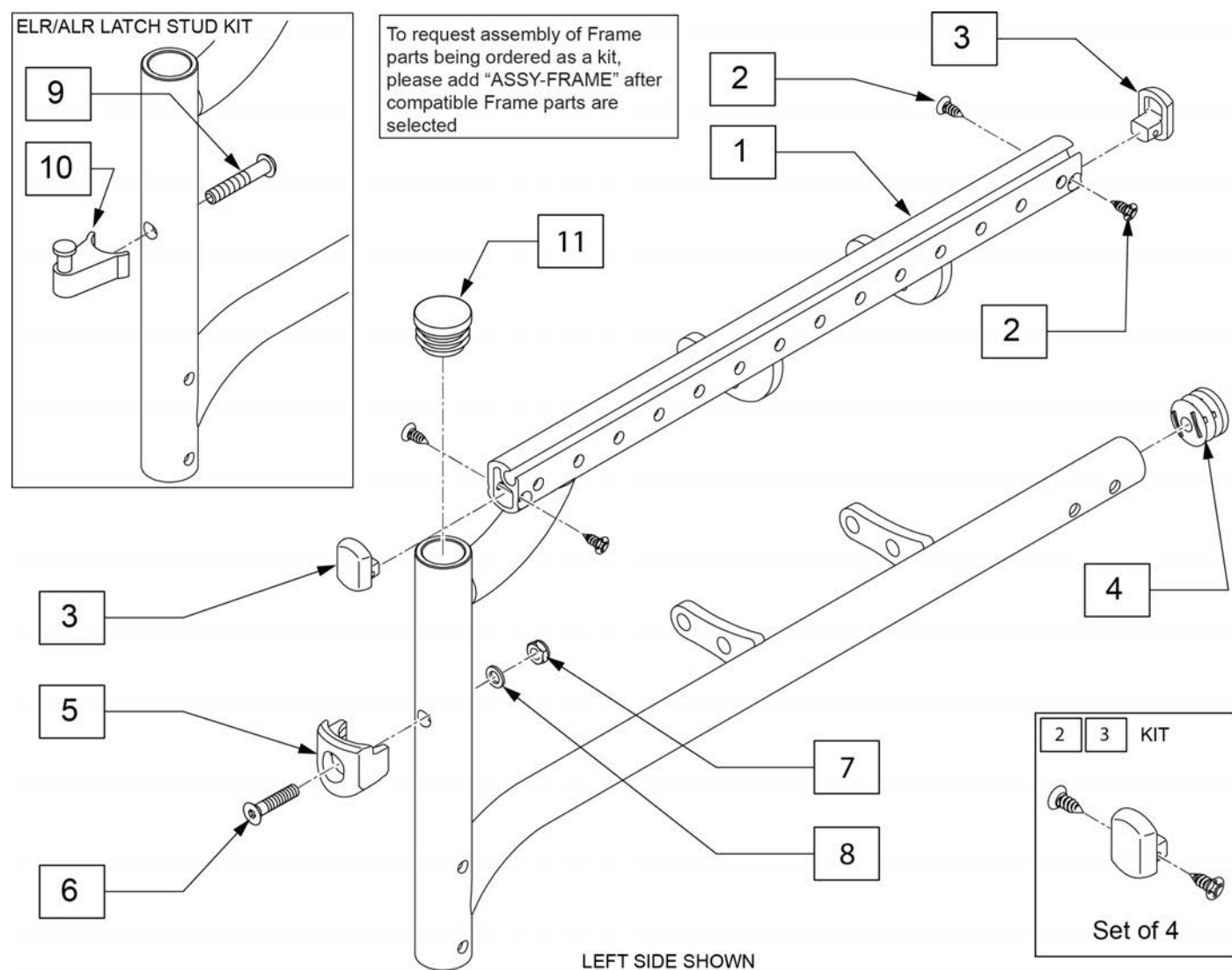


Pos.	Item Number	Description	Remarks
2,3	128828	FRAME END CAP KIT	includes 4 caps and 8 screws
4	129271	7/8 TUBE END CAP	
5-8	119514	LATCH RECEIVER KIT	Used with Swing in-out Hangers
1-8	128744S	XCAPE STD SIDE FRM R 8 DEEP	Right
1-8	128745S	XCAPE STD SIDE FRM L 8 DEEP	Left
1-8	128746S	XCAPE STD SIDE FRM R 9 DEEP	Right
1-8	128747S	XCAPE STD SIDE FRM L 9 DEEP	Left
1-8	128748S	XCAPE STD SIDE FRM R 10 DEEP	Right
1-8	128749S	XCAPE STD SIDE FRM L 10 DEEP	Left
1-8	128750S	XCAPE STD SIDE FRM R 11 DEEP	Right
1-8	128751S	XCAPE STD SIDE FRM L 11 DEEP	Left
1-8	128752S	XCAPE STD SIDE FRM R 12 DEEP	Right
1-8	128753S	XCAPE STD SIDE FRM L 12 DEEP	Left
1-8	128754S	XCAPE STD SIDE FRM R 13 DEEP	Right

Pos.	Item Number	Description	Remarks
1-8	128755S	XCAPE STD SIDE FRM L 13 DEEP	Left
1-8	128756S	XCAPE STD SIDE FRM R 14 DEEP	Right
1-8	128757S	XCAPE STD SIDE FRM L 14 DEEP	Left
1-8	128758S	XCAPE STD SIDE FRM R 15 DEEP	Right
1-8	128759S	XCAPE STD SIDE FRM L 15 DEEP	Left
1-8	128760S	XCAPE STD SIDE FRM R 16 DEEP	Right
1-8	128761S	XCAPE STD SIDE FRM L 16 DEEP	Left
1-8	128762S	XCAPE STD SIDE FRM R 17 DEEP	Right
1-8	128763S	XCAPE STD SIDE FRM L 17 DEEP	Left
1-8	128764S	XCAPE STD SIDE FRM R 18 DEEP	Right
1-8	128765S	XCAPE STD SIDE FRM L 18 DEEP	Left
9-10	107491	LATCH STUD KIT	Used with ELR/ALR hangers
11	560735	7/8" ROUND TUBE PLUG	No SA Hanger SEE NOTE
		Ship assembled part #	Use with multiple assemblies
	ASSY-FRAME	FRAME ASSY REQUIRED	add with Frame kit

Note: LEFT SIDE SHOWN

To request assembly of Frame parts being ordered as a kit, please add "ASSY-FRAME" after compatible Frame parts are selected

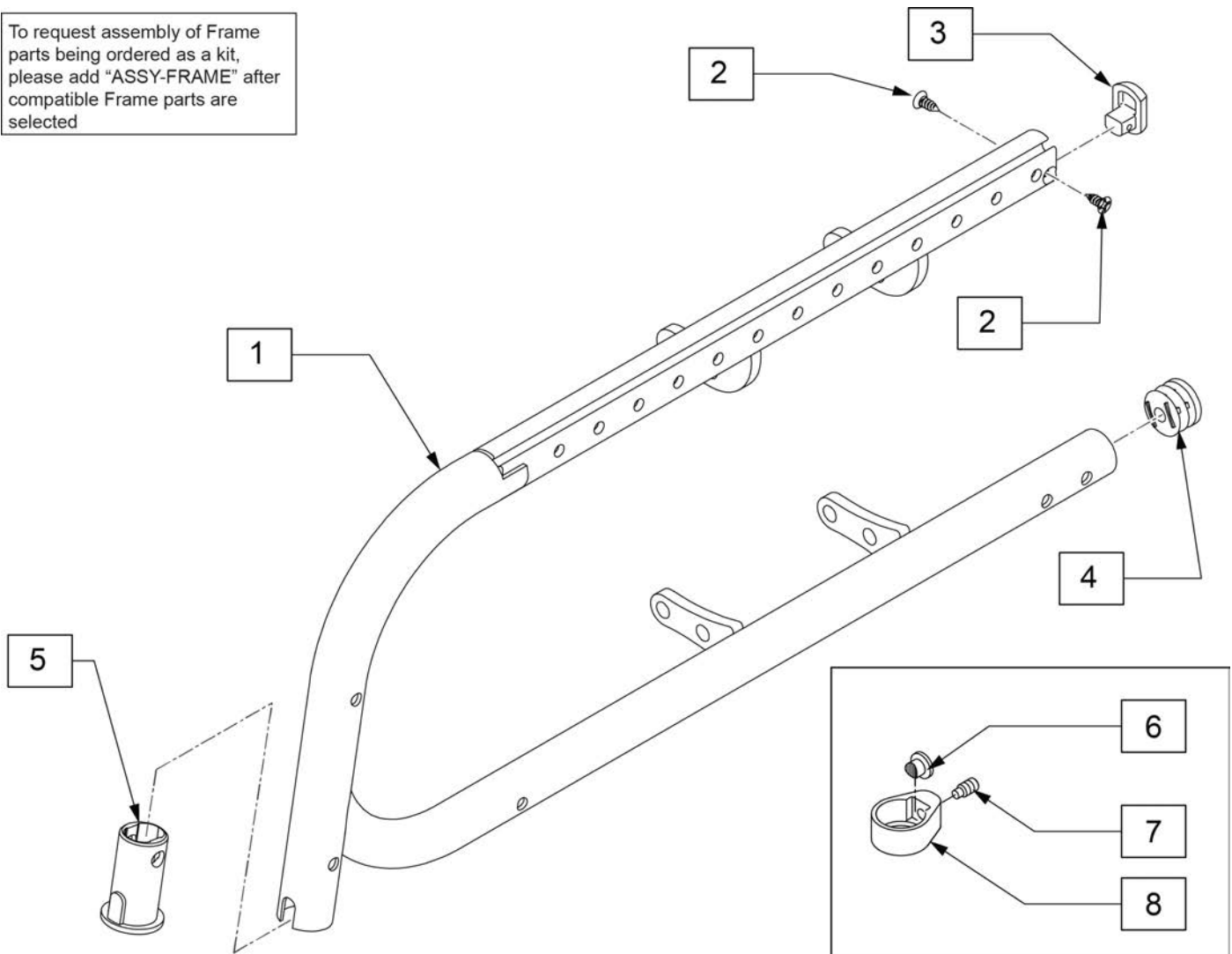


Pos.	Item Number	Description	Remarks
2-3	128828	FRAME END CAP KIT	includes 4 caps and 8 screws
4	129271	7/8 TUBE END CAP	
5-8	107485	FOOTREST CLAMP ASSM	
1-5	128766S	XCAPE FIXED SIDE FRM R 8 DEEP	Right
1-5	128767S	XCAPE FIXED SIDE FRM 8 DEEP	Left
1-5	128768S	XCAPE FIXED SIDE FRM R 9 DEEP	Right
1-5	128769S	XCAPE FIXED SIDE FRM L 9 DEEP	Left
1-5	128770S	XCAPE FIXED SIDE FRM R 10 DEEP	Right
1-5	128771S	XCAPE FIXED SIDE FRM L 10 DEEP	Left
1-5	128772S	XCAPE FIXED SIDE FRM R 11 DEEP	Right
1-5	128773S	XCAPE FIXED SIDE FRM L 11 DEEP	Left
1-5	128774S	XCAPE FIXED SIDE FRM R 12 DEEP	Right
1-5	128775S	XCAPE FIXED SIDE FRM L 12 DEEP	Left
1-5	128776S	XCAPE FIXED SIDE FRM R 13 DEEP	Right

Pos.	Item Number	Description	Remarks
1-5	128777S	XCAPE FIXED SIDE FRM L 13 DEEP	Left
1-5	128778S	XCAPE FIXED SIDE FRM R 14 DEEP	Right
1-5	128779S	XCAPE FIXED SIDE FRM L 14 DEEP	Left
1-5	128820S	XCAPE FIXED SIDE FRM R 15 DEEP	Right
1-5	128821S	XCAPE FIXED SIDE FRM L 15 DEEP	Left
1-5	128822S	XCAPE FIXED SIDE FRM R 16 DEEP	Right
1-5	128823S	XCAPE FIXED SIDE FRM L 16 DEEP	Left
1-5	128824S	XCAPE FIXED SIDE FRM R 17 DEEP	Right
1-5	128825S	XCAPE FIXED SIDE FRM L 17 DEEP	Left
1-5	128826S	XCAPE FIXED SIDE FRM R 18 DEEP	Right
1-5	128827S	XCAPE FIXED SIDE FRM L 18 DEEP	Left
		Ship assembled part #	Use with multiple assemblies
	ASSY-FRAME	FRAME ASSY REQUIRED	add with Frame kit


Note: LEFT SIDE SHOWN

Note: ORDER PART 14 (TUBE CAP QTY 2) WHEN SWING-AWAY FOOTRESTS ARE NOT INSTALLED.

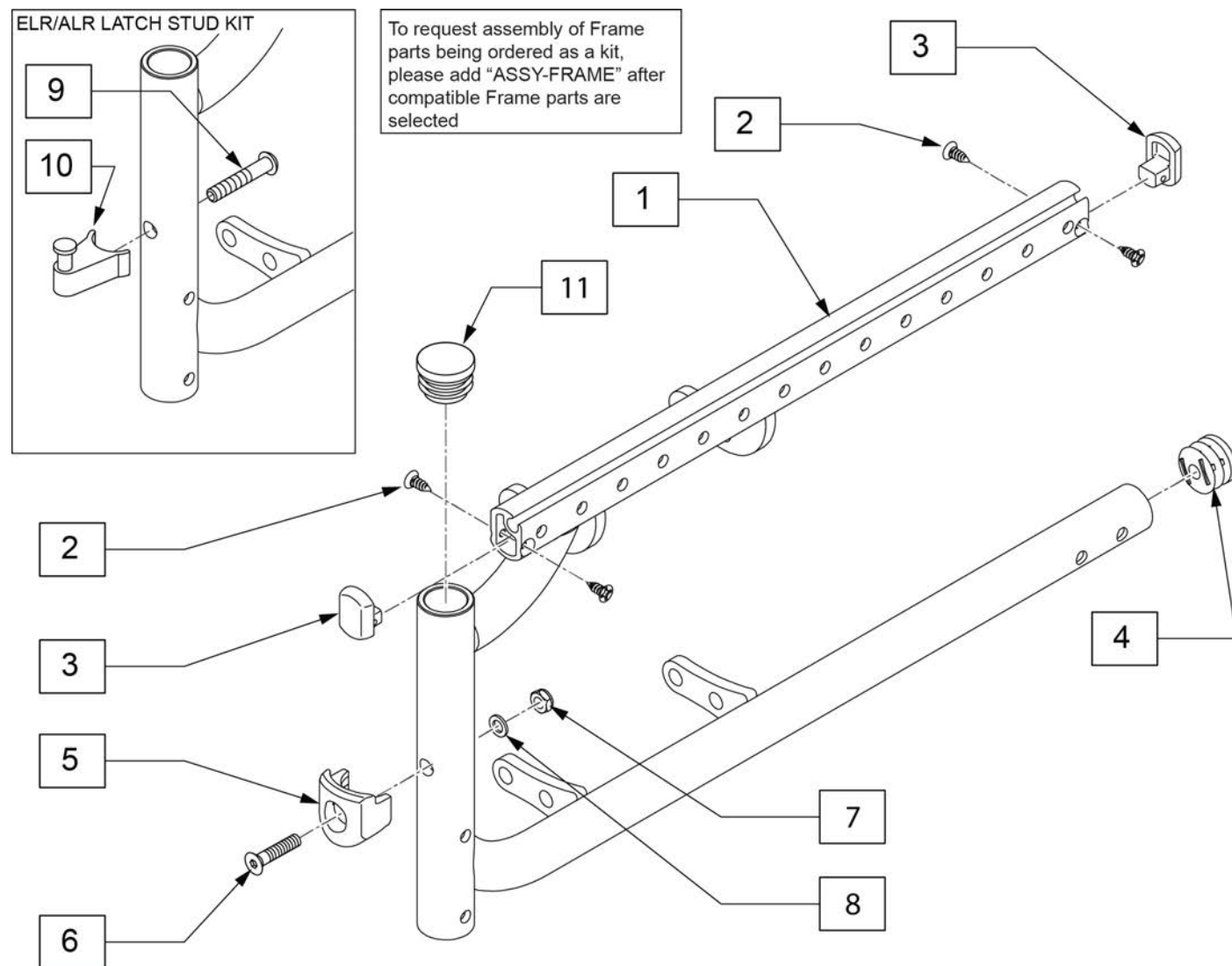


Pos.	Item Number	Description	Remarks
2,3	128828	FRAME END CAP KIT	includes 4 caps and 8 screws
4	129271	7/8 TUBE END CAP	
5-8	119514	LATCH RECEIVER KIT	Used with Swing in-out Hangers
1-8	128829S	XCAPE HEMI SIDE FRM R 8 DEEP	Right
1-8	128830S	XCAPE HEMI SIDE FRM L 8 DEEP	Left
1-8	128831S	XCAPE HEMI SIDE FRM R 9 DEEP	Right
1-8	128832S	XCAPE HEMI SIDE FRM L 9 DEEP	Left
1-8	128833S	XCAPE HEMI SIDE FRM R 10 DEEP	Right
1-8	128834S	XCAPE HEMI SIDE FRM L 10 DEEP	Left
1-8	128835S	XCAPE HEMI SIDE FRM R 11 DEEP	Right

Pos.	Item Number	Description	Remarks
1-8	128836S	XCAPE HEMI SIDE FRM L 11 DEEP	Left
1-8	128837S	XCAPE HEMI SIDE FRM R 12 DEEP	Right
1-8	128838S	XCAPE HEMI SIDE FRM L 12 DEEP	Left
1-8	128839S	XCAPE HEMI SIDE FRM R 13 DEEP	Right
1-8	128840S	XCAPE HEMI SIDE FRM L 13 DEEP	Left
1-8	128841S	XCAPE HEMI SIDE FRM R 14 DEEP	Right
1-8	128842S	XCAPE HEMI SIDE FRM L 14 DEEP	Left
1-8	128990S	XCAPE HEMI SIDE FRM R 15 DEEP	Right
1-8	128991S	XCAPE HEMI SIDE FRM L 15 DEEP	Left
1-8	128992S	XCAPE HEMI SIDE FRM R 16 DEEP	Right
1-8	128993S	XCAPE HEMI SIDE FRM L 16 DEEP	Left
1-8	128994S	XCAPE HEMI SIDE FRM R 17 DEEP	Right
1-8	128995S	XCAPE HEMI SIDE FRM L 17 DEEP	Left
1-8	128996S	XCAPE HEMI SIDE FRM R 18 DEEP	Right
1-8	128997S	XCAPE HEMI SIDE FRM L 18 DEEP	Left
9-10	107491	LATCH STUD KIT	Used with ELR/ALR hangers
11	560735	7/8" ROUND TUBE PLUG	No SA Hanger SEE NOTE
		Ship assembled part #	Use with multiple assemblies
	ASSY-FRAME	FRAME ASSY REQUIRED	add with Frame kit

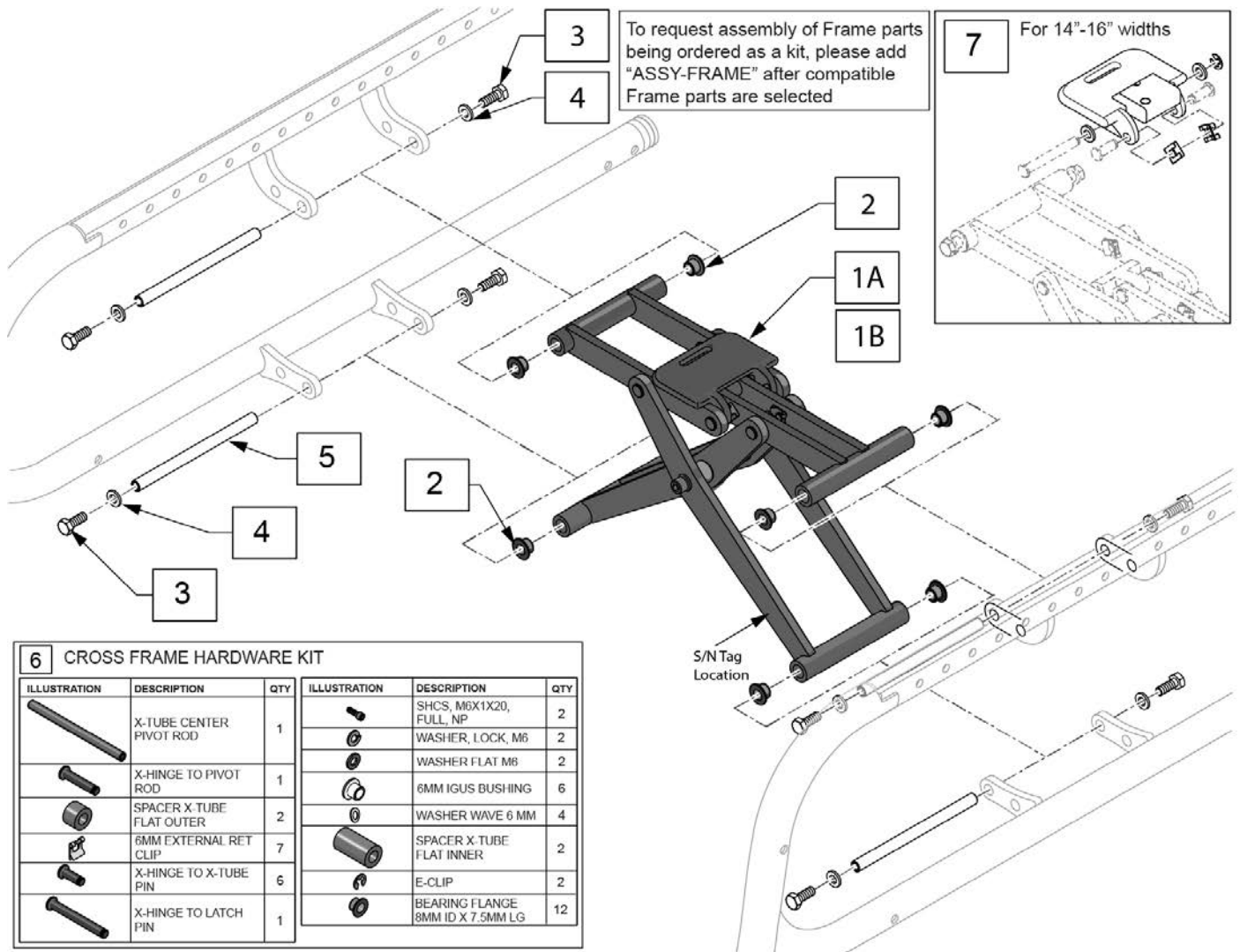

Pos.	Item Number	Description	Remarks
1A-5	136880S	CROSS FRAME KIT 8"	complete kit w/ attachment hardware
1A-5	136881S	CROSS FRAME KIT 9"	complete kit w/ attachment hardware
1A-5	136882S	CROSS FRAME KIT 10"	complete kit w/ attachment hardware
1B-5	136883S	CROSS FRAME KIT 11"	complete kit w/ attachment hardware
1B-5	136884S	CROSS FRAME KIT 12"	complete kit w/ attachment hardware
1B-5	136885S	CROSS FRAME KIT 13"	complete kit w/ attachment hardware
1B-5,7	136886S	CROSS FRAME KIT 14"	complete kit w/ attachment hardware
1B-5,7	136888S	CROSS FRAME KIT 15"	complete kit w/ attachment hardware
1B-5,7	136889S	CROSS FRAME KIT 16"	complete kit w/ attachment hardware
2	050139	BEARING FLANGE 8MMIDX7.5MM LG	
3	106995	HHCS M6X1X16 PL	
4	107383	WASHER FLAT M6	
3-5	136080	CROSS FRAME ATTACHMENT KIT	4 pins, 8 washers, 8 hex screws
6	136081	CROSS FRAME HARDWARE KIT	includes parts for Cross Frame inner pivots only
7	161572	CROSS BRACE LATCH ASSEMBLY	if growing to 14" wide, this item is required
		Ship assembled part #	Use with multiple assemblies
	ASSY-FRAME	FRAME ASSY REQUIRED	add with Frame kit


Note: LEFT SIDE SHOWN

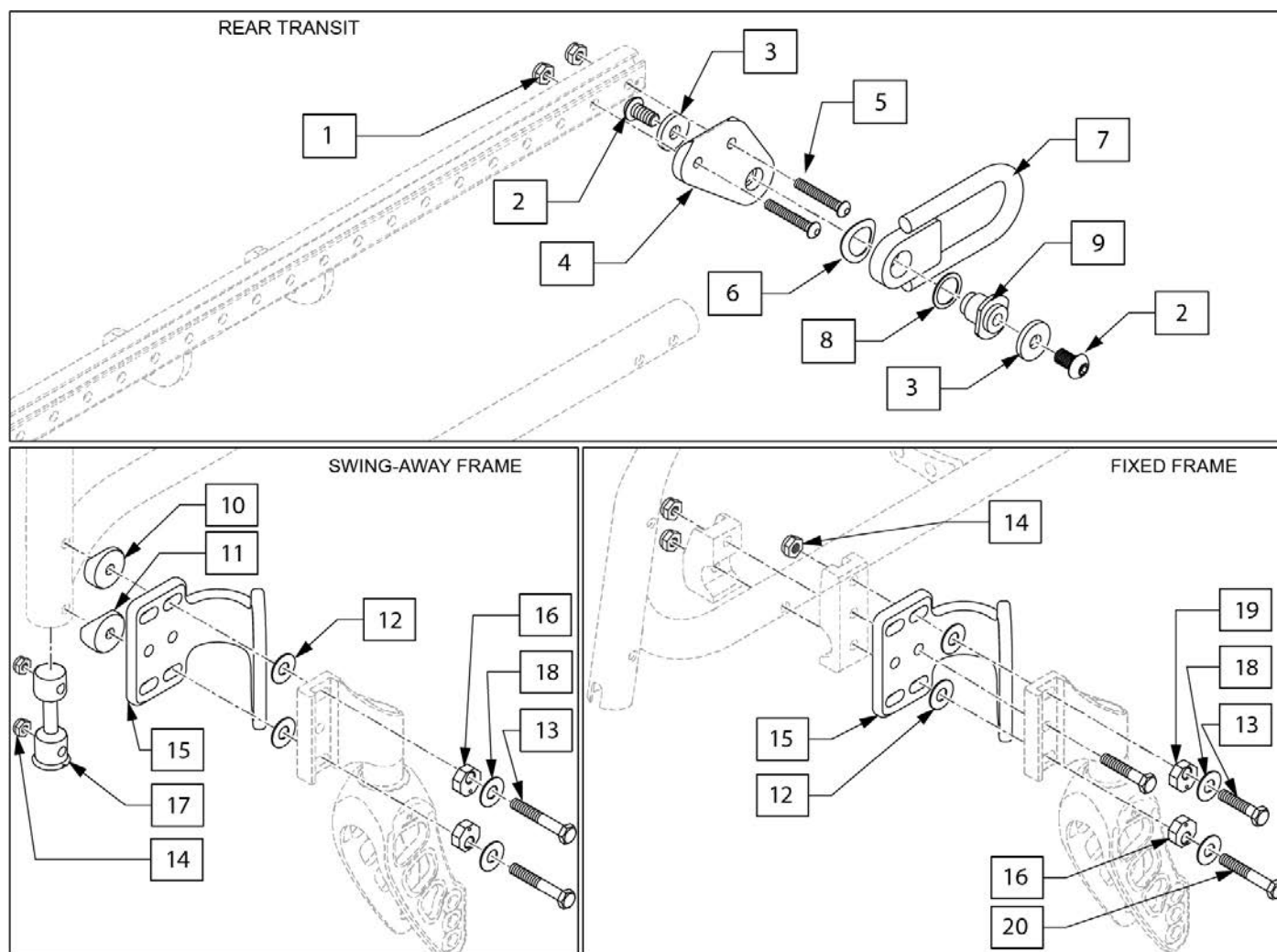










Pos.	Item Number	Description
1	104381	NUT HEX NYLOCK M6X1.0 INORG
2	230120	M8 X 1.25 X 14 BHCS
3	263113	5/16 WASHER 1/8" THICK
5	112375	BHCS M6 X 1.0 X 35MM
10	561086	1" HALF SADDLE 1/4" HOLE
11	561004	1" SADDLE 1/4" HOLE
12	122093	NYLON WASHER THIN
13	234813	HHCS M8X1X50 SHLDR PL
14	104381	NUT HEX NYLOCK M6X1.0
16	122423	ECCENTRIC WASH LOCTR
17	119029	CASTER MOUNT NUT RETAINER
18	107383	HHCS M6X1X55 PART NP
19	230063	HHCS M6 X 1.0 X 70
20	129105	HHCS M6X1X65 SHLDR NP
1-18	136082	TRANSIT KIT SA XCP
1-9,13-16,19,20	136083	TRANSIT KIT FIXED XCP

Note: see Zippie X'Cape Center of Gravity & Wheel Camber Adjustment instruction sheet to adjust Center of Gravity or Wheel Camber.


Note: 3D of camber is not available with Single Post Height Adjustable Armrests.

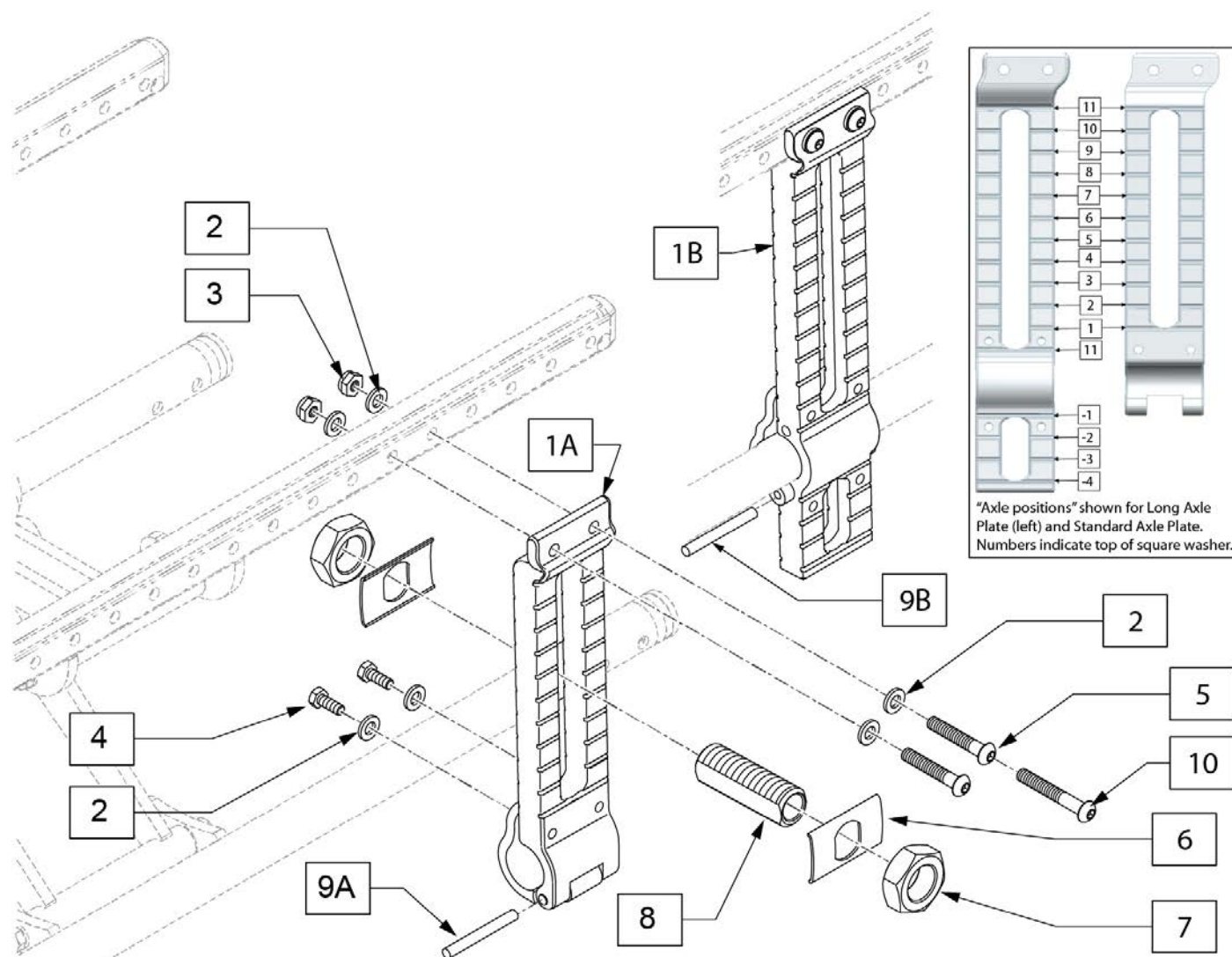

Pos.	Item Number	Description	Remarks
2	107383	M6 WASHER	
3	104381	NUT HEX NYLOCK M6X1.0 INORG	
4	118192	M6-1.0 X 20MM HHCS PL	
5	112375	M6-1.0 X 35MM BHCS PL	
6-8	136088	AXLE SLEEVE INDEX KIT	
9A	128524	5MM X 45MM DOWELPIN	
9B	128093	5MM X 60MM DOWEL PIN	
1A,2-8,9A	136084	VERTICAL AXLE PLATE KIT A STD	Right
1A,2-8,9A	136085	VERTICAL AXLE PLATE KIT B STD	Left
1B,2-8,9B	136086	VERTICAL AXLE PLATE KIT A LOW	Right
1B,2-8,9B	136087	VERTICAL AXLE PLATE KIT B LOW	Left
10	102262-008	BHCS M6X1X50 PART PL	Used when aligned with backrest bracket

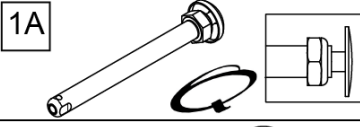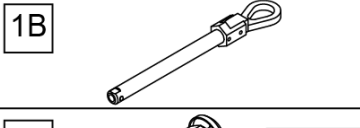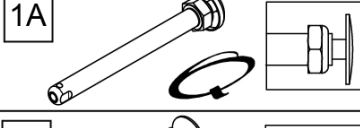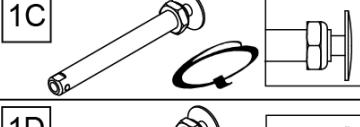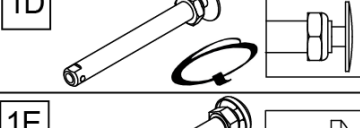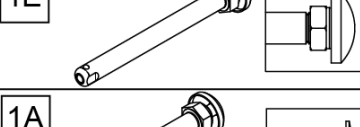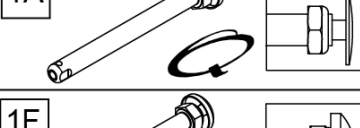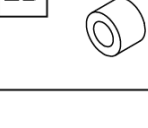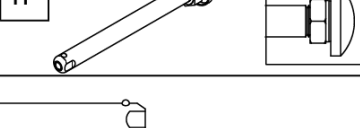
Wheel Type	Axle Type		Axle P/N	Spacer P/N
5 Spoke Mag Wheels	Quick Release	1A 	109385 A=4 1/2"	N/A
	Quad Quick Release	1B 	922170 A=4 1/8"	N/A
Spinergy Wheels & Mountain Wheels	Quick Release	1A 	109385 A=4 1/2"	N/A
Lite Spoke Wheels	Quick Release	1C 	107268 A=4 1/4"	N/A
Ultra Lite Spoke Wheels	Quick Release	1D 	111843 A=4"	N/A
Hub Lock Option Mag Wheels	Quick Release	1E 	102065 A=4 3/4"	2A 
Hub Lock Option Spoke Wheels	Quick Release	1A 	109385 A=4 1/2"	2B 
12" & 16" Mag Wheels	Quick Release	1F 	102063 A=4 1/8"	N/A

Quick Release
Measuring Guide


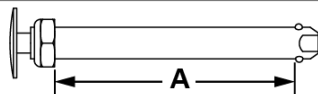


Pos.	Item Number	Description
1A	109385	QR AXLE/NUT LONG BG BTTN
1B	922170	QR AXLE MAG W/QUAD NUT
1C	107268	QR AXLE AND NUT SHORT BIG BTTN
1D	111843	QR AXLE EX SHORT BIG BTN
1E	102065	QR AXLE W/M12 NUT
1F	102063	QR AXLE MAG W/ M-12 NUT
2A	920117	MAG SPACER 6S W/SET SCREW
2B	131810	SPOKE HUB LOCK SPACER

Note: For chairs prior to S/N FZ1-001703 when switching from the obsolete Carbotecture Forks to the Multi-Position Forks, order: item 11A-11D (in pairs), qty 2 of 102259-001, qty 4 of 107357, qty 2 of 15A/15B, qty 4 of 17A/17B.

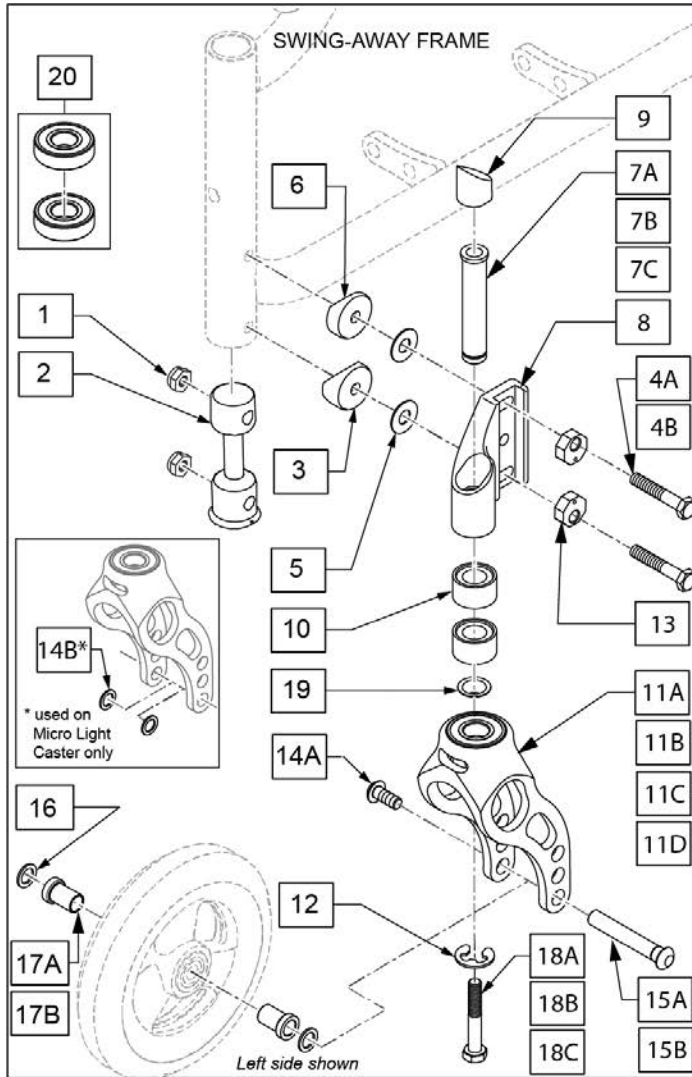


FORK MEASURING GUIDE

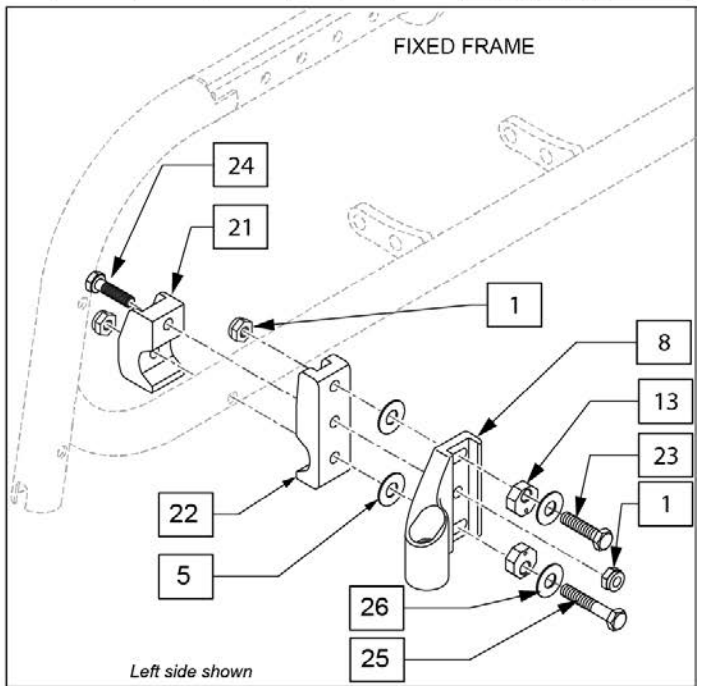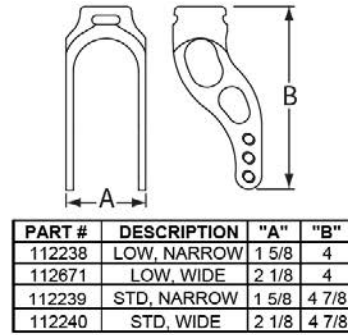

Pos.	Item Number	Description	Remarks
1	104381	NUT HEX NYLOCK M6X1.0	
2	119029	CASTER MOUNT NUT RETAINER	
3	561004	1" SADDLE 1/4" HOLE	
4A	118665	HHCS M6 X 1 X 45 PART NP	Without Transit
4B	230502	HHCS M6 X 1 X 50 BLK	With Transit
5	122093	NYLON WASHER THIN	
6	561086	1" HALF SADDLE 1/4" HOLE	
7A	127639	CASTER STEM STD	No Spacer
7B	127643	CASTER STEM +.5"	1 Spacer = 1/2"
7C	127645	CASTER STEM +1.0"	2 Spacers = 1"
8	136097	CASTER HOUSING B RT	Forward, Right
8	136098	CASTER HOUSING A LFT	Forward, Left
9	127627	CASTER PLUG	
10	127644	CASTER STEM 0.5" SPACER	


Pos.	Item Number	Description	Remarks
11A	112238	ASSM FORK LOW NARROW INTG BRG	See note
11B	112671	ASSM FORK LOW WIDE INTG BRG	See note
11C	112239	ASSM FORK STD NARROW INTG BRG	See note
11D	112240	ASSM FORK STD WIDE INTG BRG	See note
12	114479	E-STYLE BOWED CLIP	
13	122423	ECCENTRIC WASH LOCTR	
14A	102259-001	BHCS M5 x .8 x 12 W/ PL INORG	
14B	106947	WASHER FLAT THIN AN 5/16 ROH	Used w/Micro Light Caster only
15A	112783	AXLE CASTER NARROW 1/4"	Used with Narrow Forks Only
15B	112782	AXLE CASTER WIDE 1/4"	Used with Wide Forks Only
16	107357	WASHER SPLIT LOCK M5 ROHS	
17A	112781	SPACER, CASTER NARROW, INTEGRAL	Used with Narrow Forks Only
17B	112780	SPACER, CASTER WIDE, INTEGRAL	Used with Wide Forks Only
18A	129106	HHCS M8X1X50 SHLDR PL	No Spacer
18B	129107	HHCS M8X1X60 SHLDR PL	1 Spacer = 1/2"
18C	129108	HHCS M8X1X75 SHLDR PL	2 Spacers = 1"
19	130895	12MM METRIC SHIM	Used with spacers ONLY; required if adding spacers to chair
20	159434	BEARING KIT MULTI POS FORK	Includes 2 bearings (28mm OD, 12mm ID, 8mm THK)
23	118195	HHCS M6X1X30 PART PL	Used for Fixed Frame only
24	118665	HHCS M6X1X45 PART NP	Used for Fixed Frame only
25	130884	HHCS M6X1X60 SHLDR NP	Used for Fixed Frame only
26	106943	WASHER FLAT AN 1/4	Used for Fixed Frame only
27	120957	ANTI FLUTTER KIT W/ INSTR	Pair - Used on Standard Wide and Standard Narrow Fork only.
1-6,8,13	136091	S-A CASTER HOUSING KIT A	Forward,Left
1-6,8,13	136092	S-A CASTER HOUSING KIT B	Forward,Right
1,5,8,13,21-26	136093	FXD FRAME CASTER HOUSING KIT A	Forward,Left
1,5,8,13,21-26	136094	FXD FRAME CASTER HOUSING KIT B	Forward,Right
1-3,4A,4B,5,6,13	136095	HOUSING MOUNT KIT SA	Replacement hardware only
1,5,13,21-26	136096	HOUSING MOUNT KIT FXD	Replacement hardware & clamps only
7A-7C,9,10,12,18A-18C	136099	CASTER STEM ALL POSTN KIT	Pair, Stem kit for all sizes

Note: to ensure that the lights will function in the Micro Light Up Caster, qty 2 each of p/n 106947 will be required per caster when retrofitting Micro Light Up Casters to the chair.

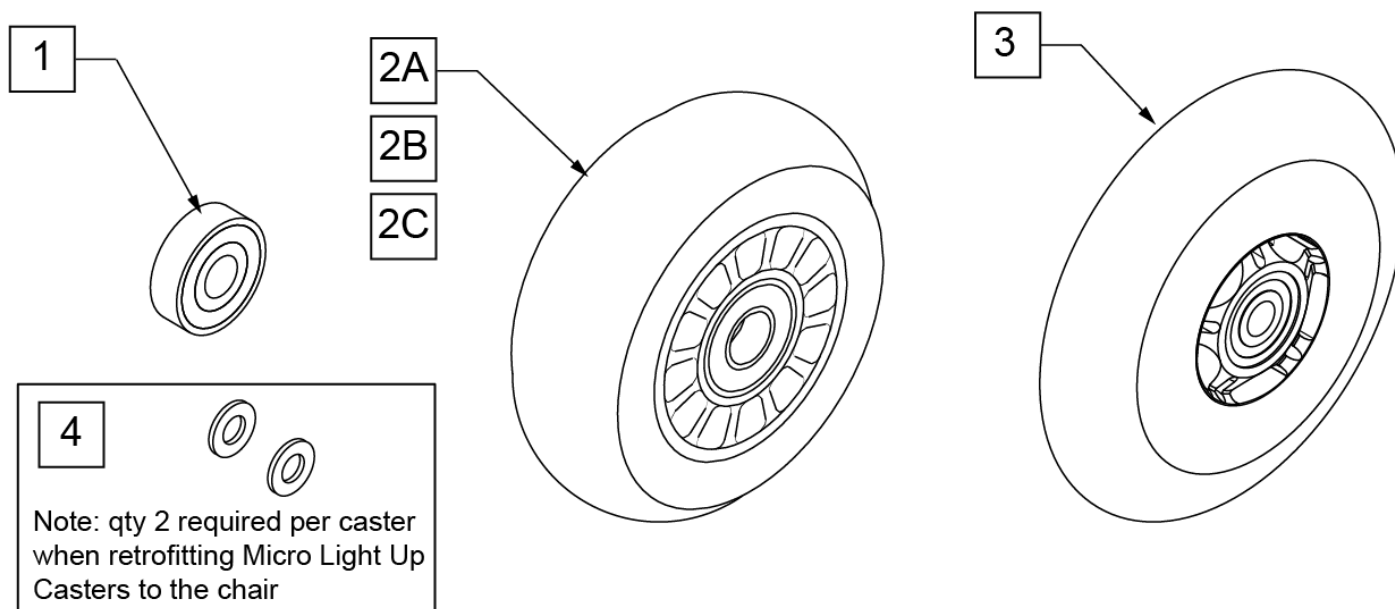

CASTER MEASURING GUIDE

PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH
400108	4.0	1.0
400107	3.0	1.0
052129	4.7	1.0

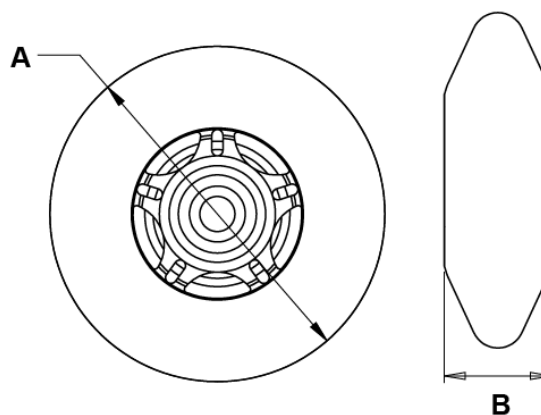


Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	400108	4" MICRO CASTER LIGHTED	K0077	4" MICRO LIGHTED
2B	920025	3" MICRO CASTER W/ BEARING -PAIR	K0077	PAIR 3" MICRO
2C	400107	3" MICRO CASTER LIGHTED	K0077	3" MICRO LIGHTED
3	052129	5" MICRO CASTER LIGHTED	K0077	5" MICRO LIGHTED
4	106947	WASHER FLAT THIN AN 5/16 ROH		

Note: WHEEL CASTER 3" ULTRALITE SFTRLL no longer available as of April 22, 2020




SIDE VIEW

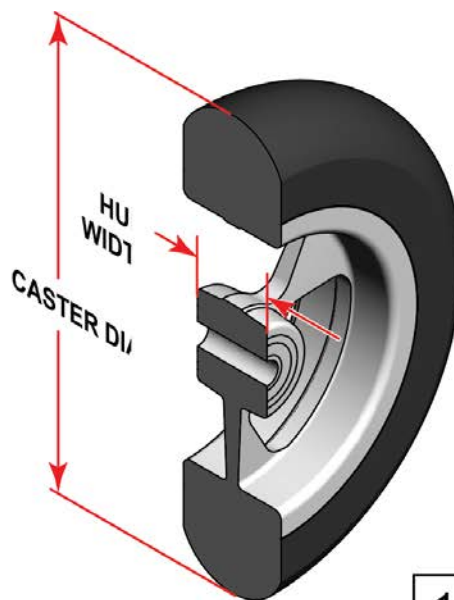


**CASTER
MEASURING GUIDE**

TIRE DIAMETER	HUB WIDTH
3	1
4 1/8	1
5	1
6	1

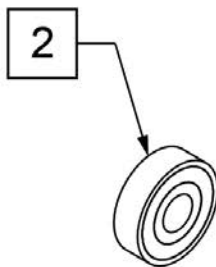


Pos.	Item Number	Description	HCPC	Remarks
1	251732	CASTR ASSM ULTRLT SFTRL 4X1.5"	K0077	4" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
1	251733	CASTR ASSM ULTRLT SFTRL 5X1.5"	K0077	5" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
1	251734	CASTR ASSM ULTRLT SFTRL 6X1.5"	K0077	6" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
		ANODIZED		
1	162609S	WHL CSTR 4" ULTRALT SFTRL ANO		4" x 1.5" Aluminum Soft Roll - Black Tire - Anodized Hub
1	162611S	WHL CSTR 5" ULTRALT SFTRLL ANO		5" x 1.5" Aluminum Soft Roll - Black Tire - Anodized Hub
1	162613S	WHL CSTR 6" ULTRALT SFTRLL ANO		6" x 1.5" Aluminum Soft Roll - Black Tire - Anodized Hub
2	349502	7/8" X 5/16" BEARING		

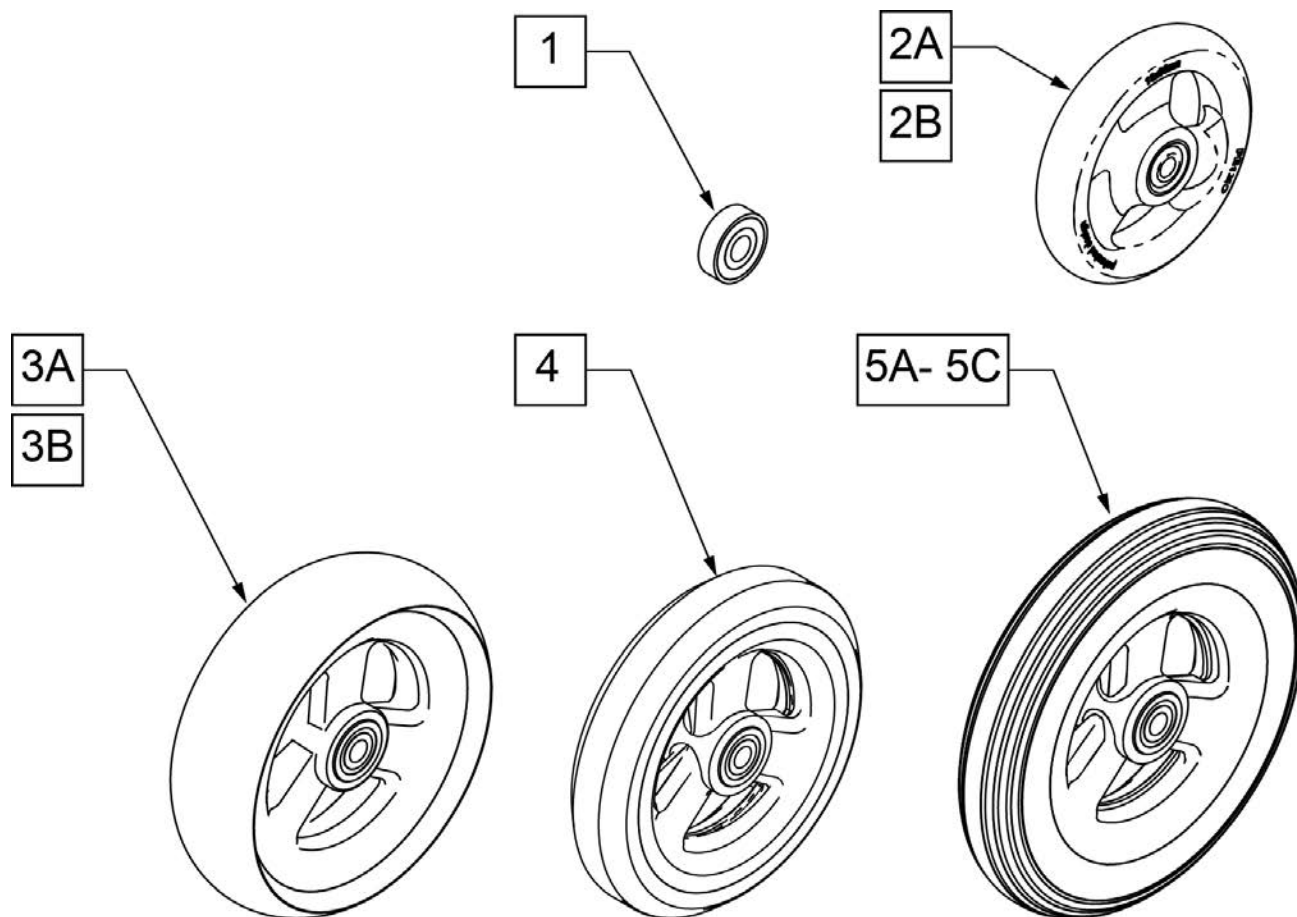


**CASTER
MEASURING GUIDE**

PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH
101957	4.3	1.2
101098	4.0	.8
101259	4.9	1.5
101255	5.7	1.5
101254	4.9	1.0
101256	5.75	1.0
101257	5.9	1.25

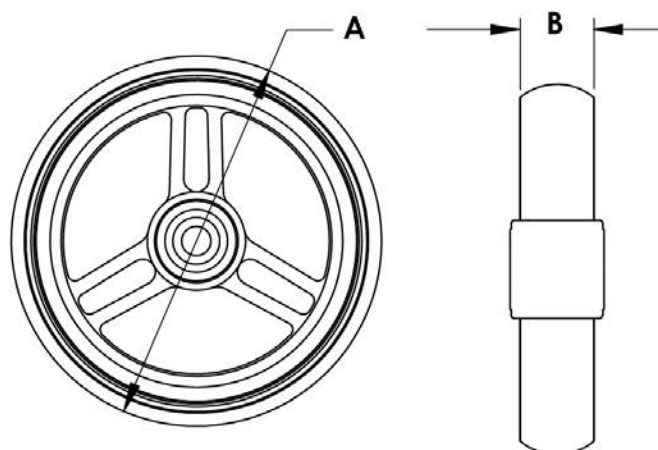


Pos.	Item Number	Description	Remarks
1	349502	7/8"x 5/16" BEARING	
2A	101957	4"X1-1/4 POLY 3 SPOKE CSTR	4" X 1.25 SEMI-PNEUMATIC
2B	101098	CASTER 4" PLASTIC/POLY W/BRG	4" LOW PROFILE POLY
3A	101259	5"X1-1/2" POLY 3 SPOKE CSTR	5" x 1.5" SEMI-PNEUMATIC
3B	101255	6"X1-1/2" POLY 3 SPOKE CSTR	6" x 1.5" SEMI-PNEUMATIC
4	101254	5" LOW POLY 3 SPOKE CASTER	5" LOW PROFILE POLY
5A	101256	6" SEMI PNEU 3 SPOKE CASTER	6" POLY
5B	101257	6" PNEUMATIC 3 SPOKE CASTER	6" PNEUMATIC
5C	383245	6" PNEUMATIC TIRE & TUBE	

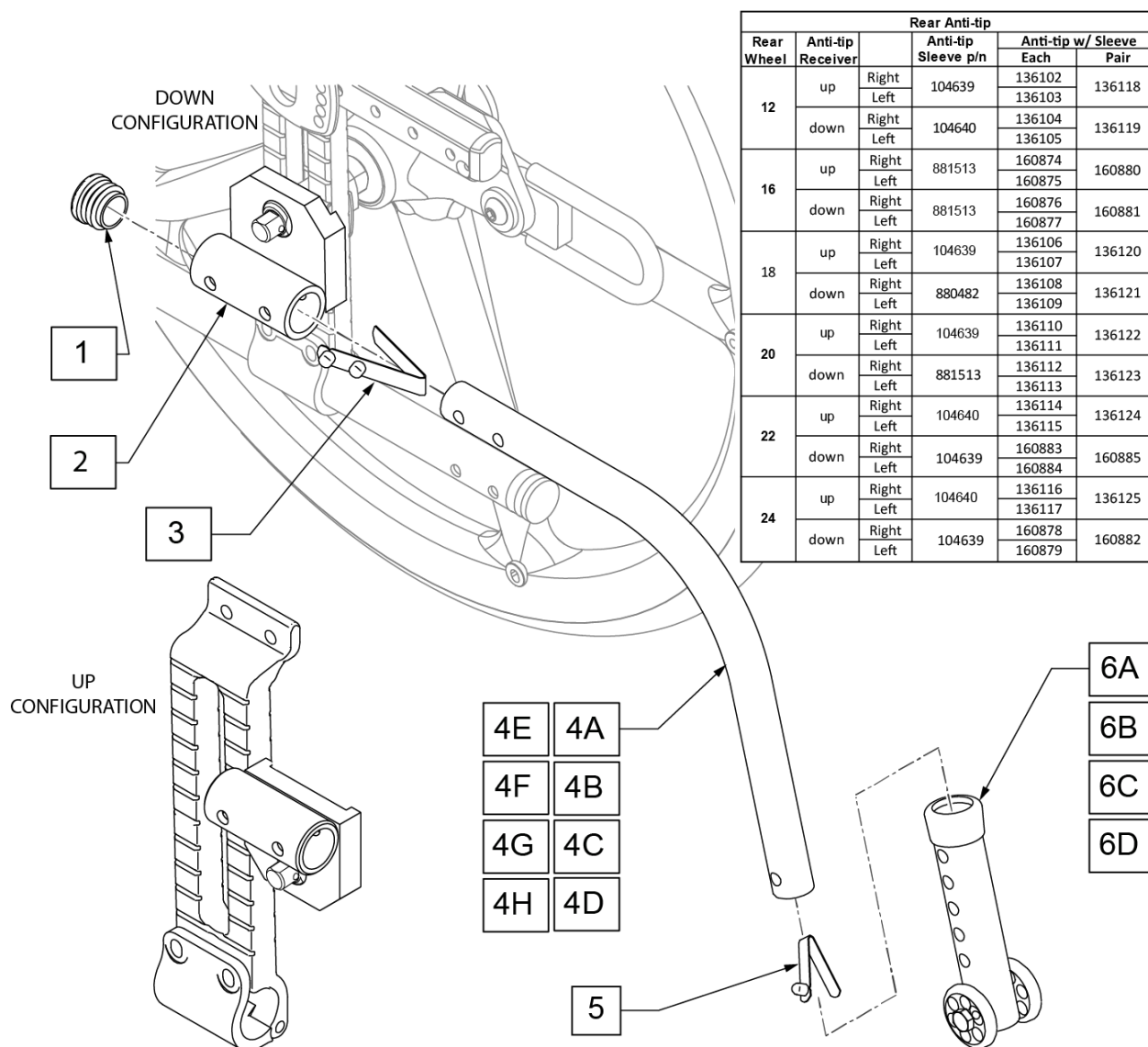




Pos.	Item Number	Description	Remarks
1	560735	7/8 ROUND TUBE PLUG	
2	129373	REAR ANTITIP TUBE ASSY	Receiver only
3	301109	SNAP BUTTON DUAL	
5	301100	SNAP BUTTON SINGLE	
6A	104639	ANTI-TIP ARM ADJUSTER MED	6 Hole Med
6B	880482	ANTI-TIP SLEEVE W/WHLS 4H	3 Hole
6C	104640	ANTI-TIP ARM ADJUSTER LONG	5 Hole
6D	881513	ANTI-TIP SLV W/WHLS BLK 5H	6 Hole Long
<< SINGLE RIGHT OR LEFT >>			
3,4A,5,6A	136102	ANTI-TIP KIT MED 6 HOLE R	Right
3,4A,5,6A	136103	ANTI-TIP KIT MED 6 HOLE L	Left
3,4B,5,6C	136104	ANTI-TIP KIT SHT 6 HOLE R	Right
3,4B,5,6C	136105	ANTI-TIP KIT SHT 6 HOLE L	Left
3,4C,5,6A	136106	ANTI-TIP KIT LNG 6 HOLE R	Right
3,4C,5,6A	136107	ANTI-TIP KIT LNG 6 HOLE L	Left
3,4C,5,6B	136108	ANTI-TIP KIT LNG 3 HOLE R	Right
3,4C,5,6B	136109	ANTI-TIP KIT LNG 3 HOLE L	Left

Pos.	Item Number	Description	Remarks
3,4F,5,6A	136110	ANTI-TIP KIT XLNG 6 HOLE R	Right
3,4F,5,6A	136111	ANTI-TIP KIT XLNG 6 HOLE L	Left
3,4G,5,6D	136112	ANTI-TIP KIT LNG 5 HOLE R	Right
3,4G,5,6D	136113	ANTI-TIP KIT LNG 5 HOLE L	Left
3,4F,5,6C	136114	ANTI-TIP KIT LNG 6 HOLE LNG R	Right
3,4F,5,6C	136115	ANTI-TIP KIT LNG 6 HOLE LNG L	Left
3,4C,5,6C	136116	ANTI-TIP KIT XL 6 HOLE LNG R	Right
3,4C,5,6C	136117	ANTI-TIP KIT XL 6 HOLE LNG L	Left
3,4C,5,6D	160874	ANTI-TIP KIT MED 5 HOLE R	Right
3,4C,5,6D	160875	ANTI-TIP KIT MED 5 HOLE L	Left
3,4E,5,6D	160876	ANTI-TIP KIT SHT 5 HOLE R	Right
3,4E,5,6D	160877	ANTI-TIP KIT SHT 5 HOLE L	Left
3,4H,5,6A	160878	ANTI-TIP KIT XLNG 6 HOLE LNG R	Right
3,4H,5,6A	160879	ANTI-TIP KIT XLNG 6 HOLE LNG L	Left
3,4D,5,6A	160883	ANTI-TIP KIT MED 6 HOLE MED R	Right
3,4D,5,6A	160884	ANTI-TIP KIT MED 6 HOLE MED L	Left
<< PAIRS >>			
3,4A,5,6A	136118	ANTI-TIP KIT MED 6 HOLE PR	Pair
3,4B,5,6C	136119	ANTI-TIP KIT SHT 6 HOLE PR	Pair
3,4C,5,6A	136120	ANTI-TIP KIT LNG 6 HOLE PR	Pair
3,4C,5,6B	136121	ANTI-TIP KIT LNG 3 HOLE PR	Pair
3,4F,5,6A	136122	ANTI-TIP KIT XLNG 6 HOLE PR	Pair
3,4G,5,6D	136123	ANTI-TIP KIT LNG 5 HOLE PR	Pair
3,4F,5,6C	136124	ANTI-TIP KIT LNG 6 HOLE LNG RP	Pair
3,4C,5,6C	136125	ANTI-TIP KIT XL 6 HOLE LNG PR	Pair
3,4D,5,6A	160885	ANTI-TIP KIT MED 6 HOLE MED PR	Pair
3,4C,5,6D	160880	ANTI-TIP KIT MED 5 HOLE PR	Pair
3,4E,5,6D	160881	ANTI-TIP KIT SHT 5 HOLE PR	Pair
3,4H,5,6A	160882	ANTI-TIP KIT XLNG 6 HOLE LNG PR	Pair




		0" spacer	0.5" spacer	1" spacer
Caster	Part	P/N	P/N	P/N
3"	tube	104219-001, 104219-002		
	sleeve	104157		
4"	tube	104219-001, 104219-002		
	sleeve	104157		104639
5"	tube	104219-001, 104219-002		
	sleeve	104639		
6"	tube	882967-882966		
	sleeve	104157		

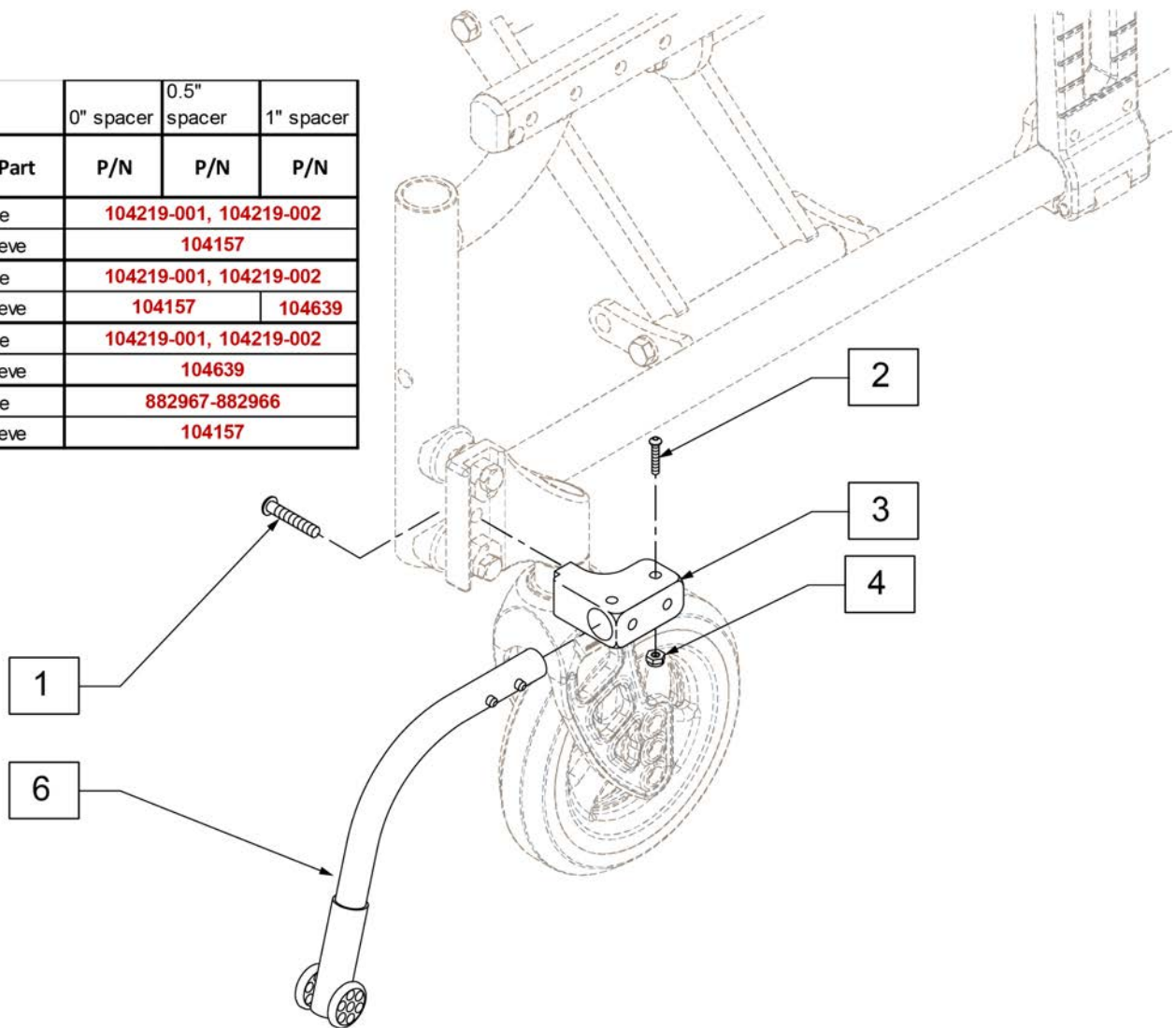


Pos.	Item Number	Description
1	106961	1/4-28 x 1 1/4 BH C/S
2	226029-QU	10 -32 x 1 1/8 BH C/S BLK
4	200100-QU	10 -32 NYLOCK NUT
1-4	922050	FRONT ANTI-TIP REC H/A CASTER Q2
6	136126	ANTI-TIP KIT CURVE 5 HOLE RT
6	136127	ANTI-TIP KIT CURVE 5 HOLE LFT
6	136128	ANTI-TIP KIT CURVE 6 HOLE RT
6	136129	ANTI-TIP KIT CURVE 6 HOLE LFT
6	136130	ANTI-TIP KIT X/S 5 HOLE RT
6	136131	ANTI-TIP KIT X/S 5 HOLE LFT
6	136132	ANTI-TIP KIT CURVE 5 HOLE PR
6	136133	ANTI-TIP KIT CURVE 6 HOLE PR
6	136134	ANTI-TIP KIT X/S 5 HOLE PR

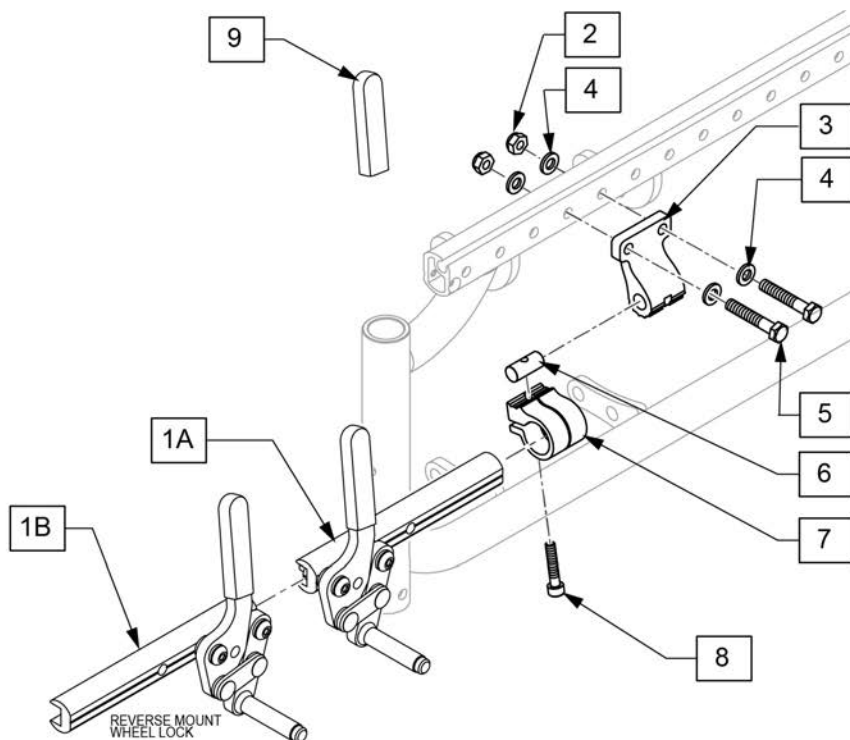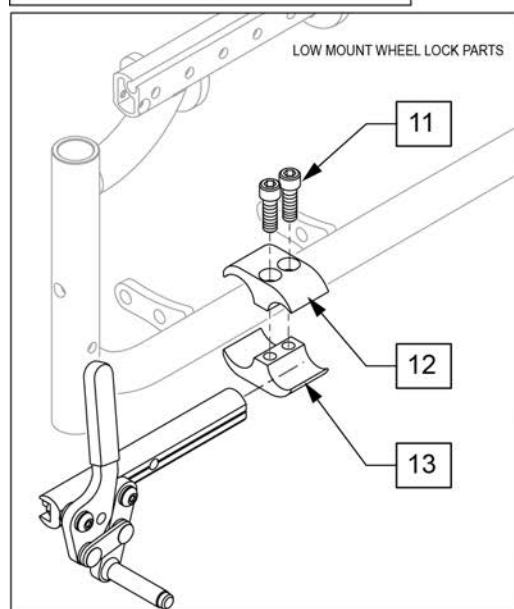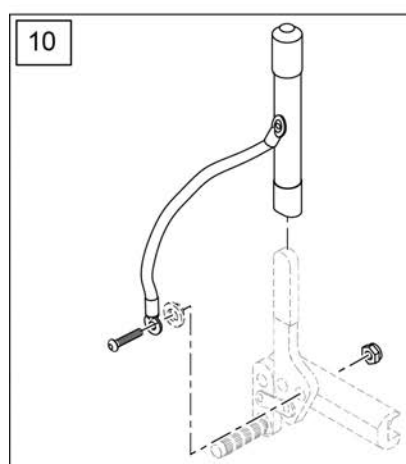

Pos.	Item Number	Description	Remarks
1A	881946	WHL LCK PUSH LG R W/O CLMP	
1A	881947	WHL LCK PUSH LG L W/O CLMP	
1B	881960	WHL LCK PUSH LG R RVS W/O CLMP	
1B	881961	WHL LCK PUSH LG L RVS W/O CLMP	
2	104381	NUT HEX NYLOCK M6X1.0	
4	250104	WASHER, FLAT, M6	
5	118197	HHCS M6X1X35 PART PL	
3,6-8	127669	WHEEL LOCK CLAMP ASSEMBLY	
9	560500-QU	HB PLASTIC COVER	single replacement cover
10	161931	WHL LOCK 6" EXT TUBE ASSM (EA)	EACH
10	920032	WHEEL LOCK 6" EXT TUBES (PR)	PAIR
1A-9	136135	WHL LOCK PUSH LNG FWD KIT	PAIR
1B-9	136136	WHL LOCK PUSH LNG REV KIT	PAIR
11	102263-003	SHCS, M6X1X25, FULL, PL	Low Mount Parts
12	128098	1 AD CLAMP 7/8" TO 3/4"	Low Mount Parts
13	128099	1 AD THREADED CLAMP 7/8" TO 3/4"	Low Mount Parts
1A,11-13	135274	WHL LOCK PUSH LNG FWD LOW KIT	PAIR
1B,11-13	135275	WHL LOCK PUSH LNG REV LOW KIT	PAIR

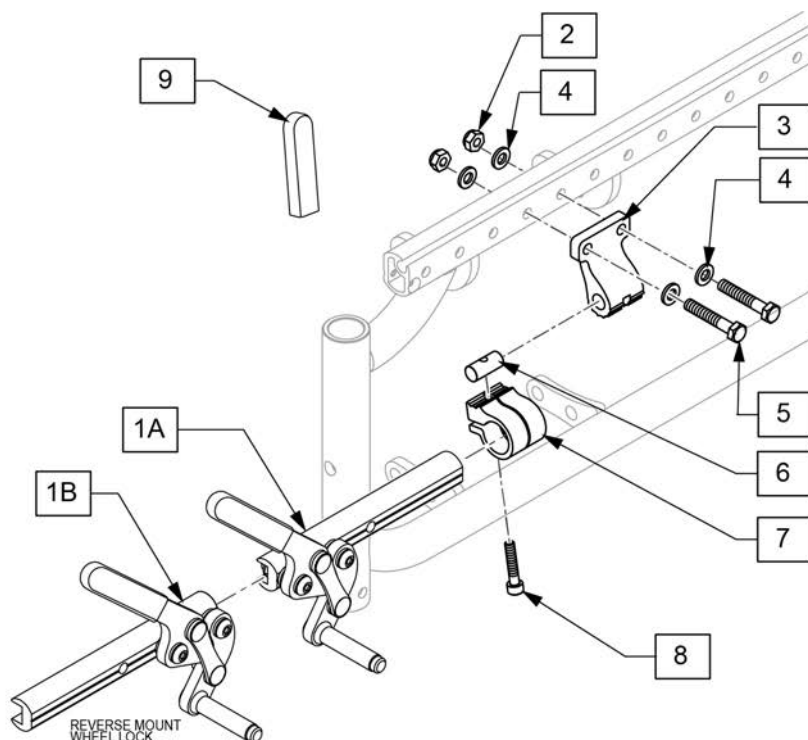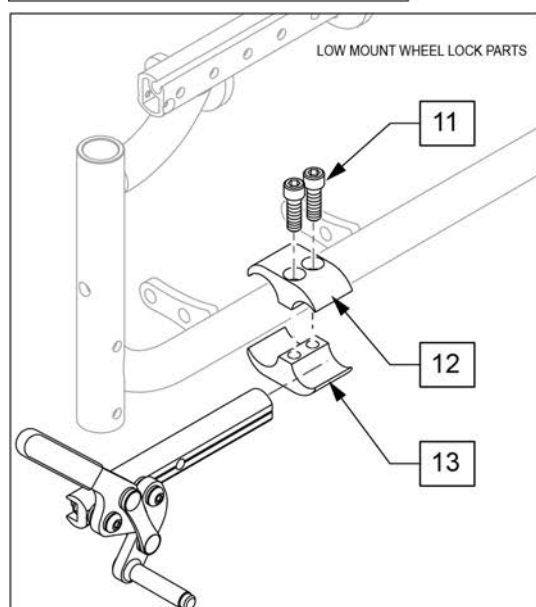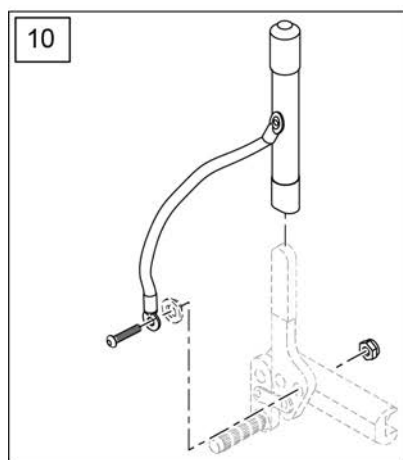


Pos.	Item Number	Description	Remarks
1A	881948	WHL LCK PULL LG R W/O CLMP	Right
1A	881949	WHL LCK PULL LG L W/O CLMP	Left
1B	120043	WHL LCK PULL LG R REV	Right
1B	120044	WHL LCK PULL LG L REV	Left
2	104381	NUT HEX NYLOCK M6X1.0	
4	250104	WASHER, FLAT, M6	
5	118197	HHCS M6X1X35 PART PL	
3,6-8	127669	WHEEL LOCK CLAMP ASSEMBLY	
9	560500-QU	HB PLASTIC COVER	single replacement cover
10	161931	WHL LOCK 6" EXT TUBE ASSM (EA)	Each
10	920032	WHEEL LOCK 6" EXT TUBES (PR)	Pair
11	102263-003	SHCS, M6X1X25, FULL, PL	Low Mount Parts
12	128098	1 AD CLAMP 7/8" TO 3/4"	Low Mount Parts
13	128099	1 AD THREADED CLAMP 7/8" TO 3/4"	Low Mount Parts
1A-9	136137	WHL LOCK PULL LNG FWD KIT	Std mount Complete kit pair
1B-9	136138	WHL LOCK PULL LNG RWD KIT	Reverse Mount Complete kit pair
1A,11-13	135276	WHL LOCK PULL LNG FWD LOW KIT	Low Mount Complete kit pair
1B,11-13	135277	WHL LOCK PULL LNG REV LOW KIT	Low Mount Complete kit pair

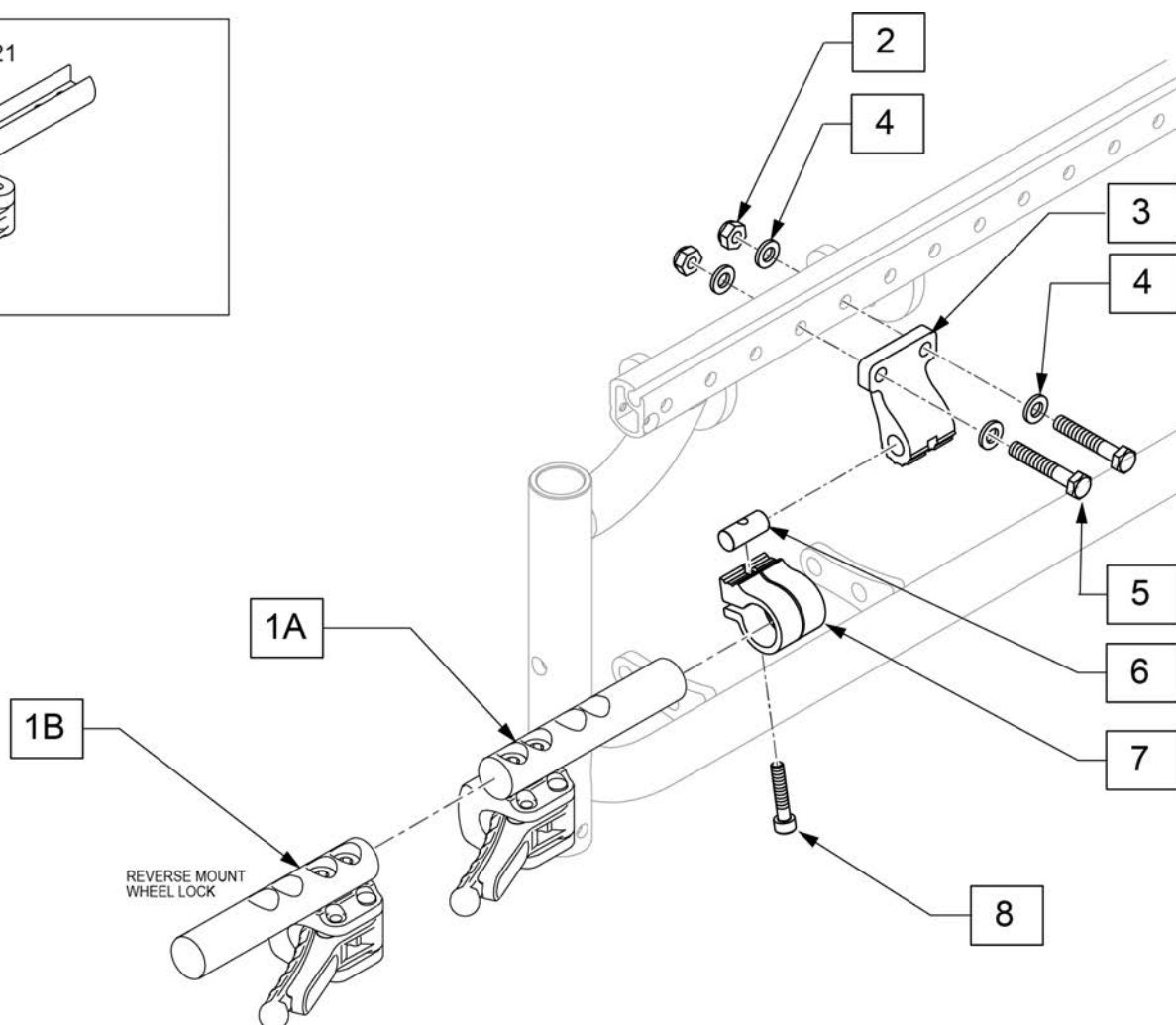

Pos.	Item Number	Description	Remarks
1A	66006100-301	COMPACT BRAKE	Left side Forward Mounted
1A	66006100-302	COMPACT BRAKE	Right side Forward Mounted
1B	66006100-401	COMPACT BRAKE STD BACK LEFT	Reverse Mounted
1B	66006100-402	COMPACT BRAKE STD BACK RIGHT	Reverse Mounted
2	104381	NUT HEX NYLOCK M6X1.0	
4	250104	WASHER, FLAT, M6	
5	118197	HHCS M6X1X35 PART PL	
3,6-8	127669	WHEEL LOCK CLAMP ASSEMBLY	
1-8	136139	WHL LOCK COMPACT KIT FORWARD	Pair
1-8	136140	WHL LOCK COMPACT KIT REVERSE	Pair


Note: see AXLE PLATE parts page for notch axle positions.

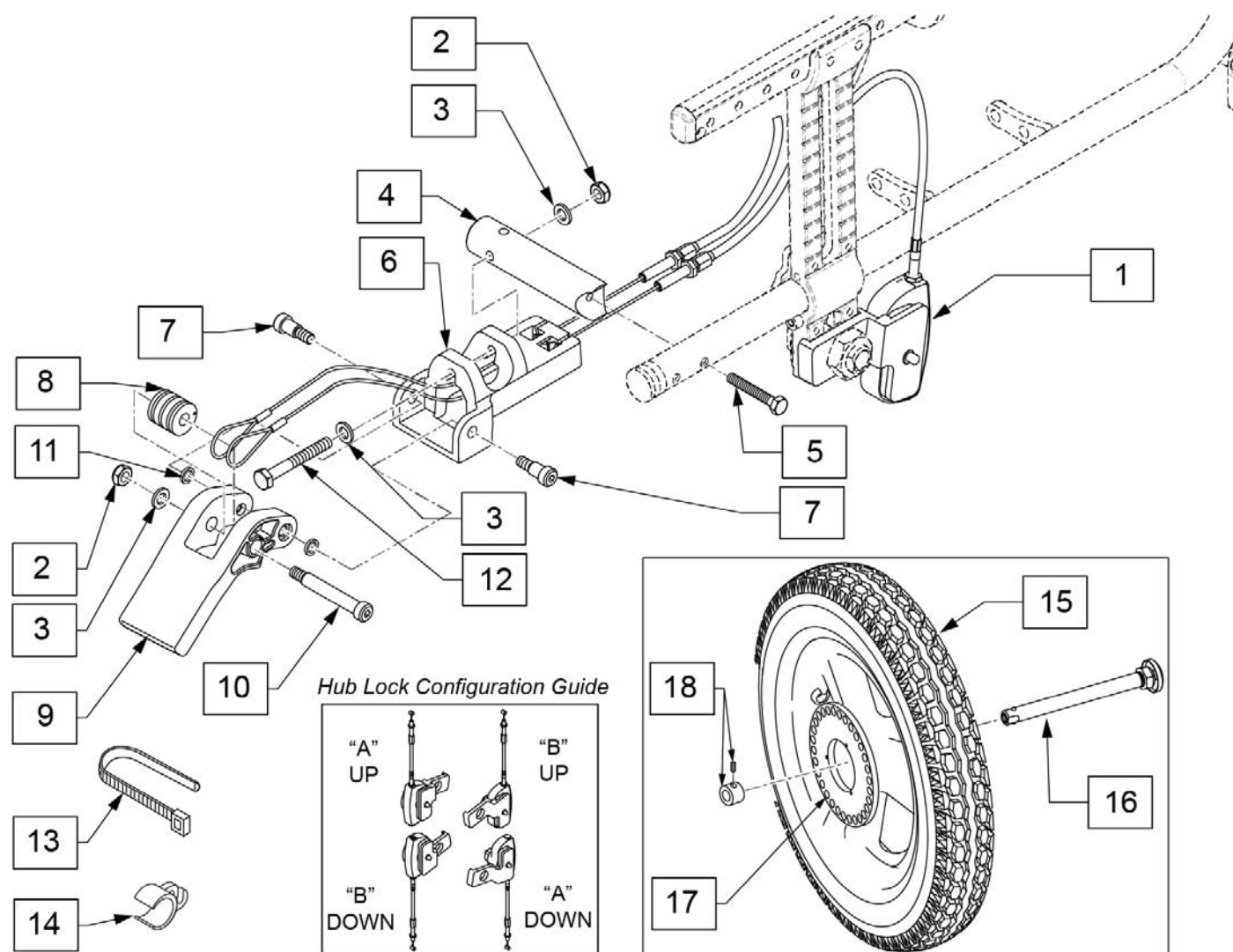

Pos.	Item Number	Description	Remarks
1	130376	FOOT LOCK HUB ASSM SHORT A	For frame depths or frame lengths less than 14"
1	130377	FOOT LOCK HUB ASSM SHORT B	For frame depths or frame lengths less than 14"
1	130378	FOOT LOCK HUB ASSM LONG A	For frame depths or frame lengths greater than 14"
1	130379	FOOT LOCK HUB ASSM LONG B	For frame depths or frame lengths greater than 14"
2	106932	NUT NYLOCK 1/4 - 20 ROHS	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	127868	7/8" FOOT LOCK MOUNT PLATE	
5	221101	HHCS 1/4-28X1 1/2" PL	
6	123517	ATTENDT WHL LCK MT BRKT	
7	281048	SHOULDER SCREW .312 DIA X .375 LONG	
8	506185	BEARING ATTENDANT WHEEL LOCK CABLE	
9	506182	ATTENDANT W.L. ACTUATION PEDAL	
10	281041	SHOULDER SCREW .312 DIA X 1.75 LONG	
11	481353	SPACER, BEARING	
12	106992	HHCS 1/4 -20 X 2 HEX C/S	


Pos.	Item Number	Description	Remarks
13	560410	ZIP TIE BLACK	
14	105850-001	CLIP CABLE 1"	
15	112226-001	WHEEL HUB LOCK 12" PNEU	
15	112226-002	WHEEL HUB LOCK 12" AIRLESS	
15	112226-003	WHEEL HUB LOCK 12" LOW POLY	
15	112227-001	WHEEL HUB LOCK 16" PNEU	
15	112227-002	WHEEL HUB LOCK 16" AIRLESS	
16	102065	QR ALXE W/ M12 NUT	
17	111845	INDEX WHEEL 12IN	
18	920117	MAG SPACER 6S W/SET SCREW	5/8" wide
1-14	136151	FOOT LOCK KIT SHORT	
1-14	136152	FOOT LOCK KIT LONG	

Note: Hub Lock requires Lite Spoke Wheel
for 20", 22" and 24" Wheels

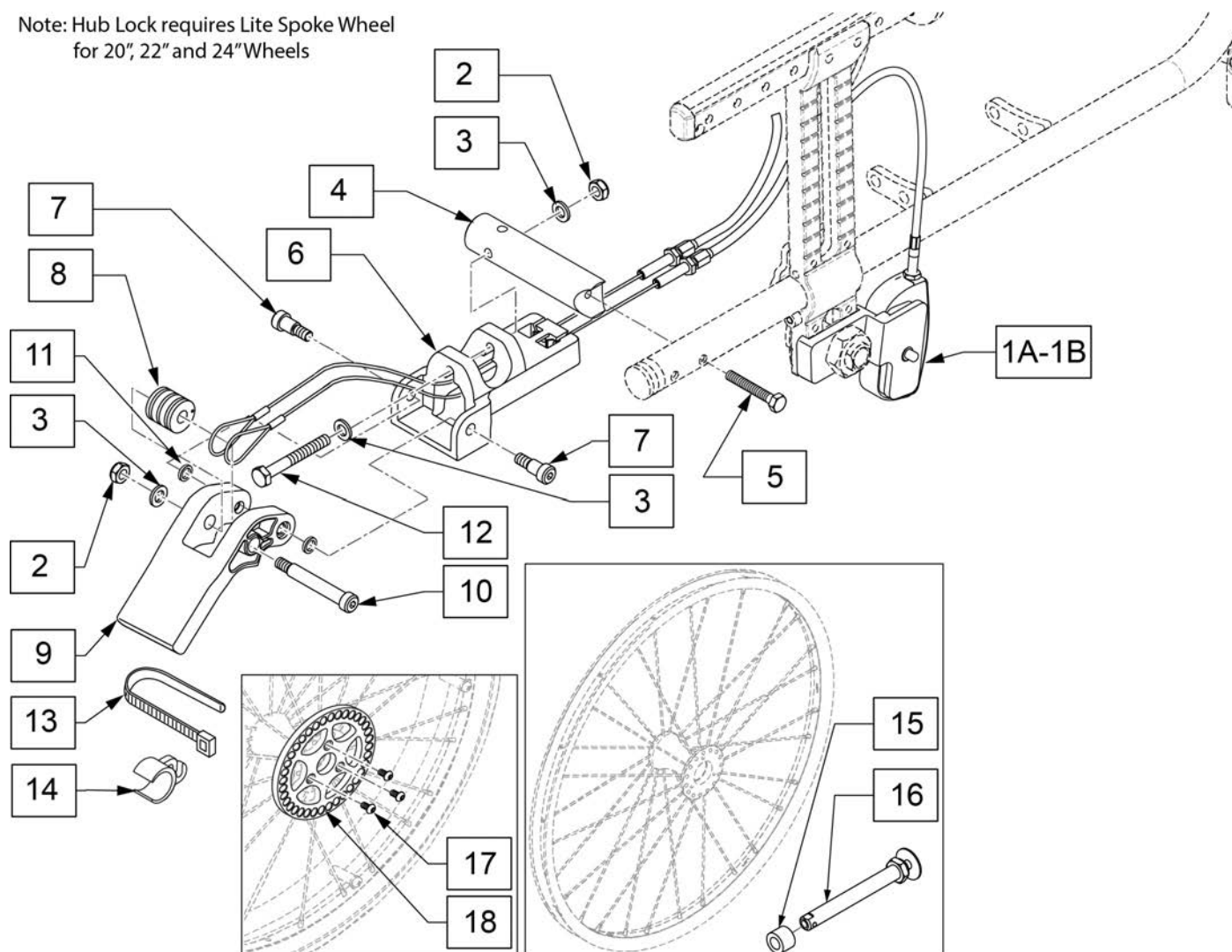

Pos.	Item Number	Description	Remarks
1A	131806	FOOT LOCK SPOKE HUB ASSY SHRT A	For frame depths or frame lengths less than 14"
1A	131807	FOOT LOCK SPOKE HUB ASSY SHRT B	For frame depths or frame lengths less than 14"
1B	131808	FOOT LOCK SPOKE HUB ASSY LONG A	For frame depths or frame lengths greater than 14"
1B	131809	FOOT LOCK SPOKE HUB ASSY LONG B	For frame depths or frame lengths greater than 14"
2	106932	NUT NYLOCK 1/4 - 20 ROHS	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	127868	7/8" FOOT LOCK MOUNT PLATE	
5	221101	HHCS 1/4-28X1 1/2" PL	
6	123517	ATTENDT WHL LCK MT BRKT	
7	281048	SHOULDER SCREW .312 DIA X .375 LONG	
8	506185	BEARING ATTENDANT WHEEL LOCK CABLE	
9	506182	ATTENDANT W.L. ACTUATION PEDAL	
10	281041	SHOULDER SCREW .312 DIA X 1.75 LONG	
11	481353	SPACER, BEARING	
12	106992	HHCS 1/4 -20 X 2 HEX C/S	
13	560410	ZIP TIE BLACK	
14	105850-001	CLIP CABLE 1"	
15	131810	SPOKE HUB LOCK SPACER	
16	109385	QR AXLE NUT LONG BG BTTN	

Pos.	Item Number	Description	Remarks
17	103184-002	BHCS M5X0.8X8 FULL PL	
18	113347	LOCK PLATE SPOKE WHEEL	Fits on lite spoke wheel
1A,2-18	159732	FOOT LOCK SPOKE KIT SHORT	
1B,2-18	159733	FOOT LOCK SPOKE KIT LONG	

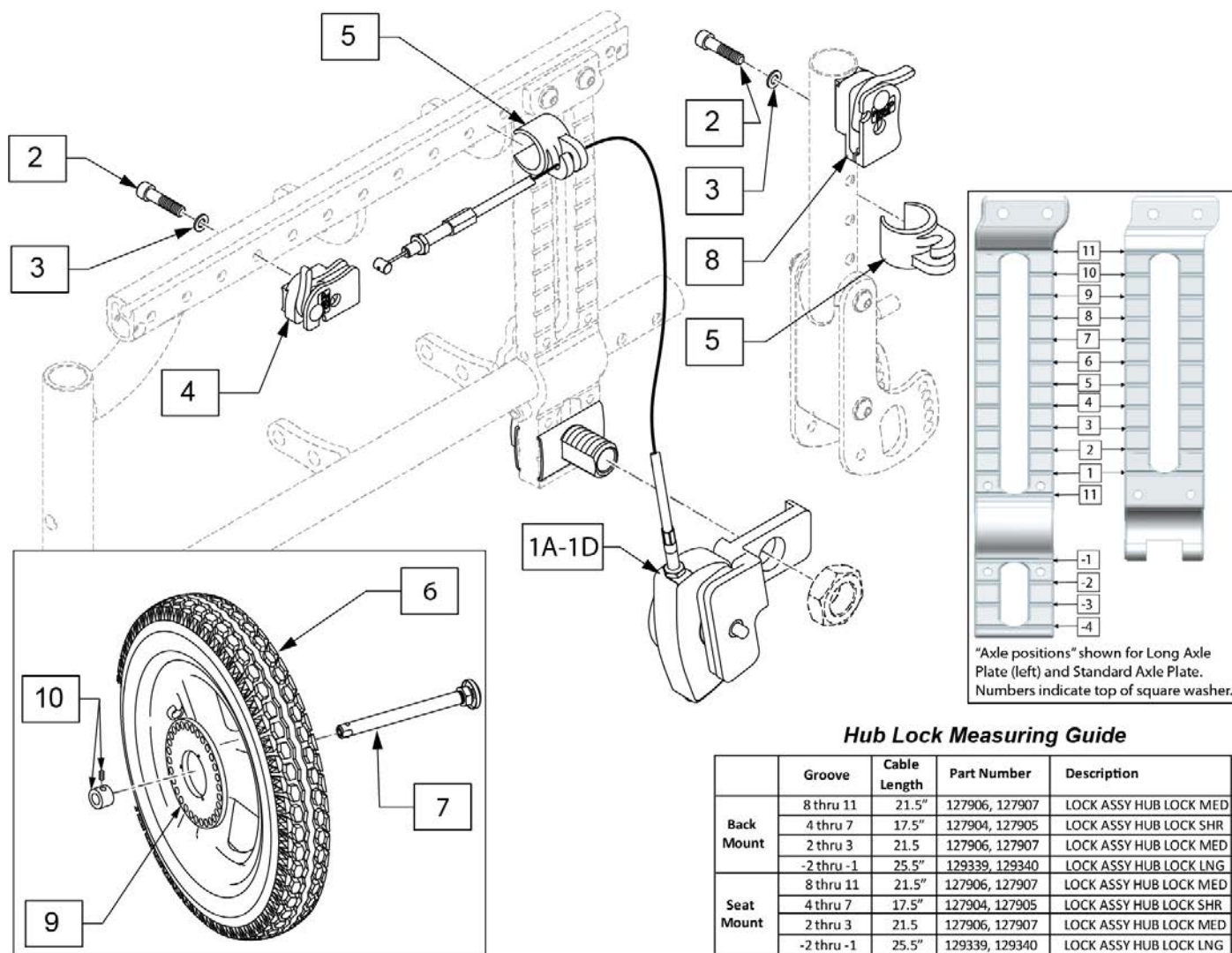


Hub Lock Measuring Guide

	Groove	Cable Length	Part Number	Description
Back Mount	8 thru 11	21.5"	127906, 127907	LOCK ASSY HUB LOCK MED
	4 thru 7	17.5"	127904, 127905	LOCK ASSY HUB LOCK SHR
	2 thru 3	21.5	127906, 127907	LOCK ASSY HUB LOCK MED
	-2 thru -1	25.5"	129339, 129340	LOCK ASSY HUB LOCK LNG
Seat Mount	8 thru 11	21.5"	127906, 127907	LOCK ASSY HUB LOCK MED
	4 thru 7	17.5"	127904, 127905	LOCK ASSY HUB LOCK SHR
	2 thru 3	21.5	127906, 127907	LOCK ASSY HUB LOCK MED
	-2 thru -1	25.5"	129339, 129340	LOCK ASSY HUB LOCK LNG

Pos.	Item Number	Description	Remarks
1A	127904	LOCK ASSY HUB LOCK SHRT A	Left
1A	127905	LOCK ASSY HUB LOCK SHRT B	Right
1B	127906	LOCK ASSY HUB LOCK MED A	Left
1B	127907	LOCK ASSY HUB LOCK MED B	Right
1C	129339	LOCK ASSY HUB LOCK LNG A	Left
1C	129340	LOCK ASSY HUB LOCK LNG B	Right
1D	129341	LOCK ASSY HUB LOCK XL A	Left
1D	129342	LOCK ASSY HUB LOCK XL B	Right
2	106976	SHCS 1/4-28 X 1-1/2	
3	106943	WASHER FLAT AN 1/4	
4	127869	HUB LOCK LEVER ASSY R	used for Frame Mount
4	127870	HUB LOCK LEVER ASSY L	used for Frame Mount
5	108134	CABLE CLIP 1" PACK (10)	
6	112226-001	WHEEL HUB LOCK 12" PNEU	
6	112226-002	WHEEL HUB LOCK 12" AIRLESS	
6	112226-003	WHEEL HUB LOCK 12" LOW POLY	
6	112227-001	WHEEL HUB LOCK 16" PNEU	
6	112227-002	WHEEL HUB LOCK 16" AIRLESS	
7	102065	QR ALXE W/ M12 NUT	

Pos.	Item Number	Description	Remarks
8	112426	ASSY LEVER HUB LOCK RHS	used for Back Mount
8	112427	ASSY LEVER HUB LOCK LHS	used for Back Mount
9	111845	INDEX WHEEL 12IN	
10	920117	MAG SPACER 6S S/SET SCREW	5/8" wide
1A,2-5	136142	LVR HUB LOC KIT SHRT CABLE	Frame Mount Kit (pair)
1B,2-5	136143	LVR HUB LOC KIT MED CABLE	Frame Mount Kit (pair)
1C,2-5	136144	LVR HUB LOC KIT LONG CABLE	Frame Mount Kit (pair)
1D,2-5	136145	LVR HUB LOC KIT XLONG CABLE	Frame Mount Kit (pair)
1A,2,3,5,8	136147	LVR HUB LOC KIT SHT CABLE BK	Back Mount Kit (pair)
1B,2,3,5,8	136148	LVR HUB LOC KIT MED CABLE BK	Back Mount Kit (pair)
1C,2,3,5,8	136149	LVR HUB LOC KIT LNG CABLE BK	Back Mount Kit (pair)
1D,2,3,5,8	136150	LVR HUB LOC KIT XL CABLE BK	Back Mount Kit (pair)

Note: Hub Lock requires Lite Spoke Wheel for 20", 22" and 24" Wheels

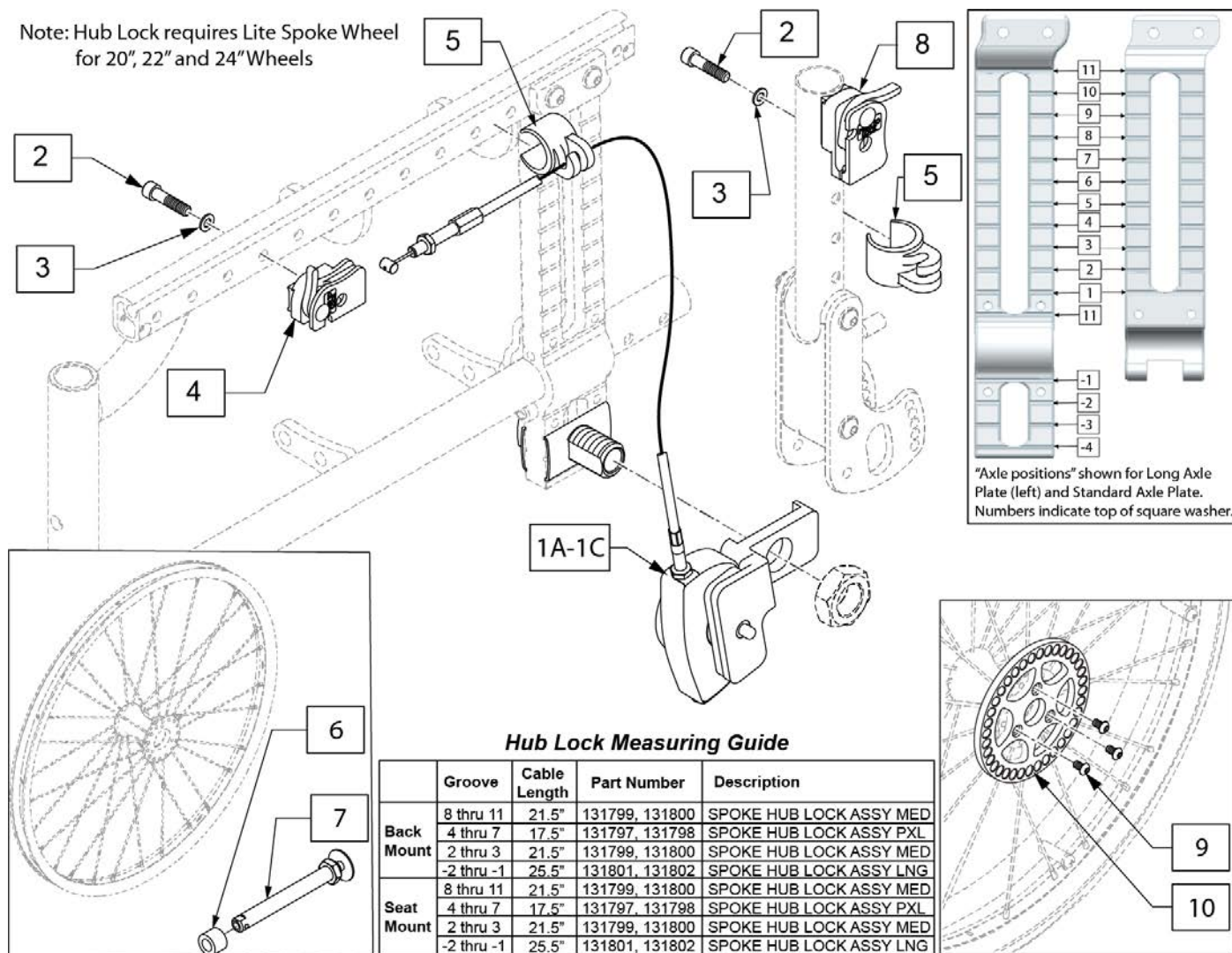


Pos.	Item Number	Description	Remarks
1A	131799	SPOKE HUB LOCK ASSY MED A	Med, Left
1A	131800	SPOKE HUB LOCK ASSY MED B	Med, Right
1B	131797	SPOKE HUB LOCK ASSY PXL A	Pxl, Left
1B	131798	SPOKE HUB LOCK ASSY PXL B	Pxl Right
1C	131801	SPOKE HUB LOCK ASSY XL A	Long, Left
1C	131802	SPOKE HUB LOCK ASSY XL B	Long, Right
2	106976	SHCS 1/4-28 X 1-1/2	
3	106943	WASHER FLAT AN 1/4	
4	127869	HUB LOCK LEVER ASSY R	Frame Mount
4	127870	HUB LOCK LEVER ASSY L	Frame Mount
5	108134	CABLE CLIP 1" PACK (10)	
6	131810	SPOKE HUB LOCK SPACER	
7	109385	QR AXLE NUT LONG BG BTTN	
8	112426	ASSY LEVER HUB LOCK RHS	Back Mount
8	112427	ASSY LEVER HUB LOCK LHS	Back Mount
9	103184-002	BHCS M5X0.8X8 FULL PL	
10	113347	LOCK PLATE SPOKE WHEEL	Fits on lite spoke wheel
1A,2-7,9,10	159734	LVR HUB LOC SPOKE KIT MED CABLE	Frame Mount Kit (pair)
1B,2-7,9,10	159735	LVR HUB LOC SPOKE KIT PXL CABLE	Frame Mount Kit (pair)

Pos.	Item Number	Description	Remarks
1C,2-7,9,10	159736	LVR HUB LOC SPOKE KIT LONG CABLE	Frame Mount Kit (pair)
1A,2,3,5-10	159737	LVR HUB LOC SPOKE KIT MED CABLE BK	Back Mount Kit (pair)
1B,2,3,5-10	159738	LVR HUB LOC SPOKE KIT PXL CABLE BK	Back Mount Kit (pair)
1C,2,3,5-10	159739	LVR HUB LOC SPOKE KIT LONG CABLE BK	Back Mount Kit (pair)


Note: Black Tires are not sold separately; please order the appropriate assembly part number.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

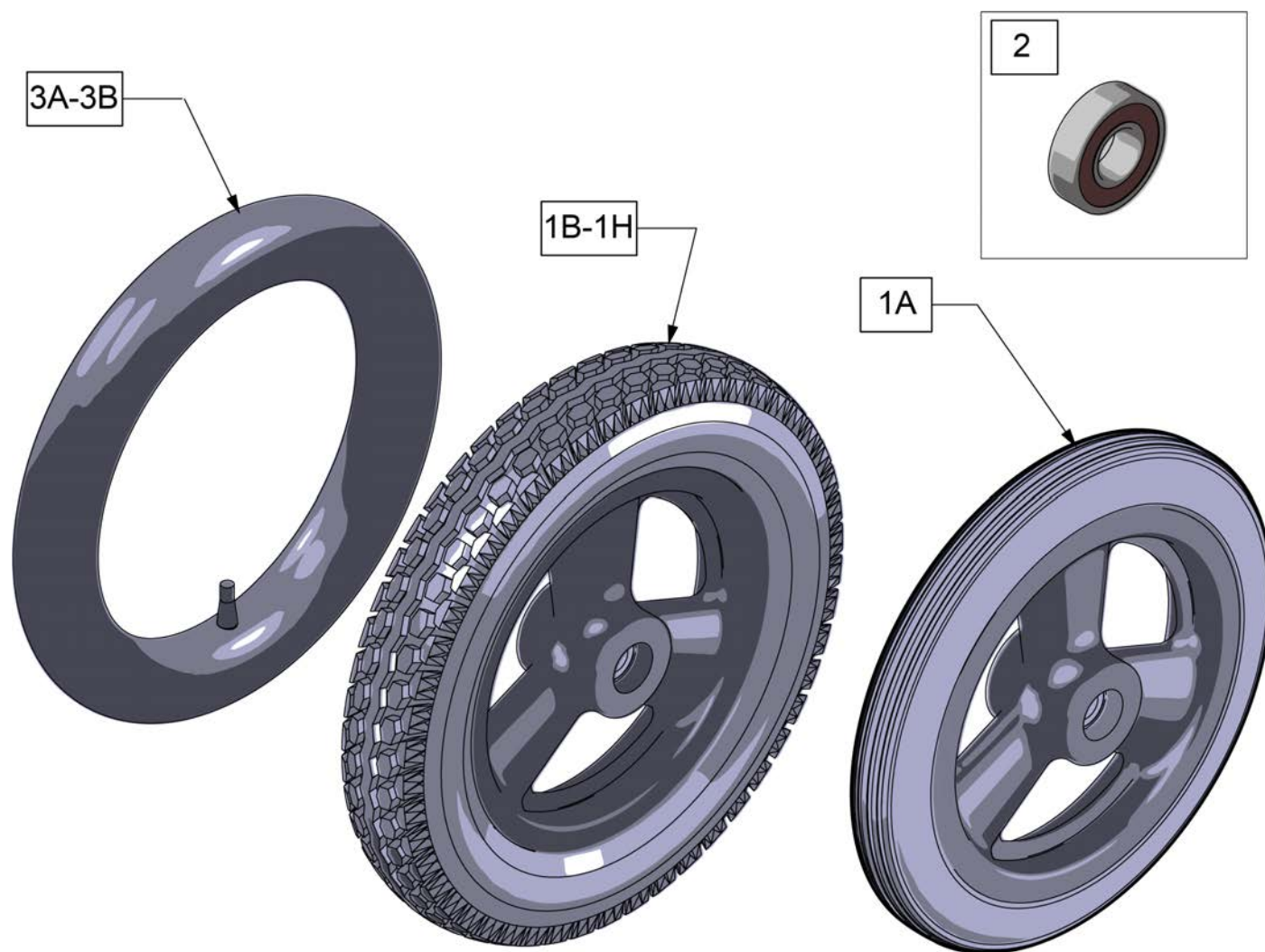

Pos.	Item Number	Description	Remarks
1A	101466	12 IN MAG POLY TIRE 3 SPOKE	Low Poly
1B	101661	12" MAG PNEU 3 SPOKE	Grey Tire; for Quickie & Zippie Iris see Black Wheel Assm p/n's below
1C	101662	12" MAG PNEU INSRT 3 SPK	Grey Tire; for Quickie & Zippie Iris see Black Wheel Assm p/n's below
1D	384706	TIRE PNEU 12-1/2" X 2-1/4"	Grey Tire only
1E	243593	12IN MAG PNEU 3 SPOKE BLK	Black tire, only available on Quickie and Zippie IRIS at this time
1F	243594	12IN MAG PNEU INSRT 3 SPK BLK	Black tire, only available on Quickie and Zippie IRIS at this time
1G	112226-004	WHEEL HUB LOCK 12IN PNEU BLK	Black tire for Hub lock, only available on Quickie and Zippie IRIS at this time
1H	112226-005	WHEEL HUB LOCK 12IN INSRT BLK	Black tire for Hub lock, only available on Quickie and Zippie IRIS at this time
2	349500	1 1/8 X 1/2 BEARING	
3A	384736	INNER TUBE 12-1/2" X 2-1/4"	
3B	102972-001	12 1/2 IN AIRLESS INSERT	

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

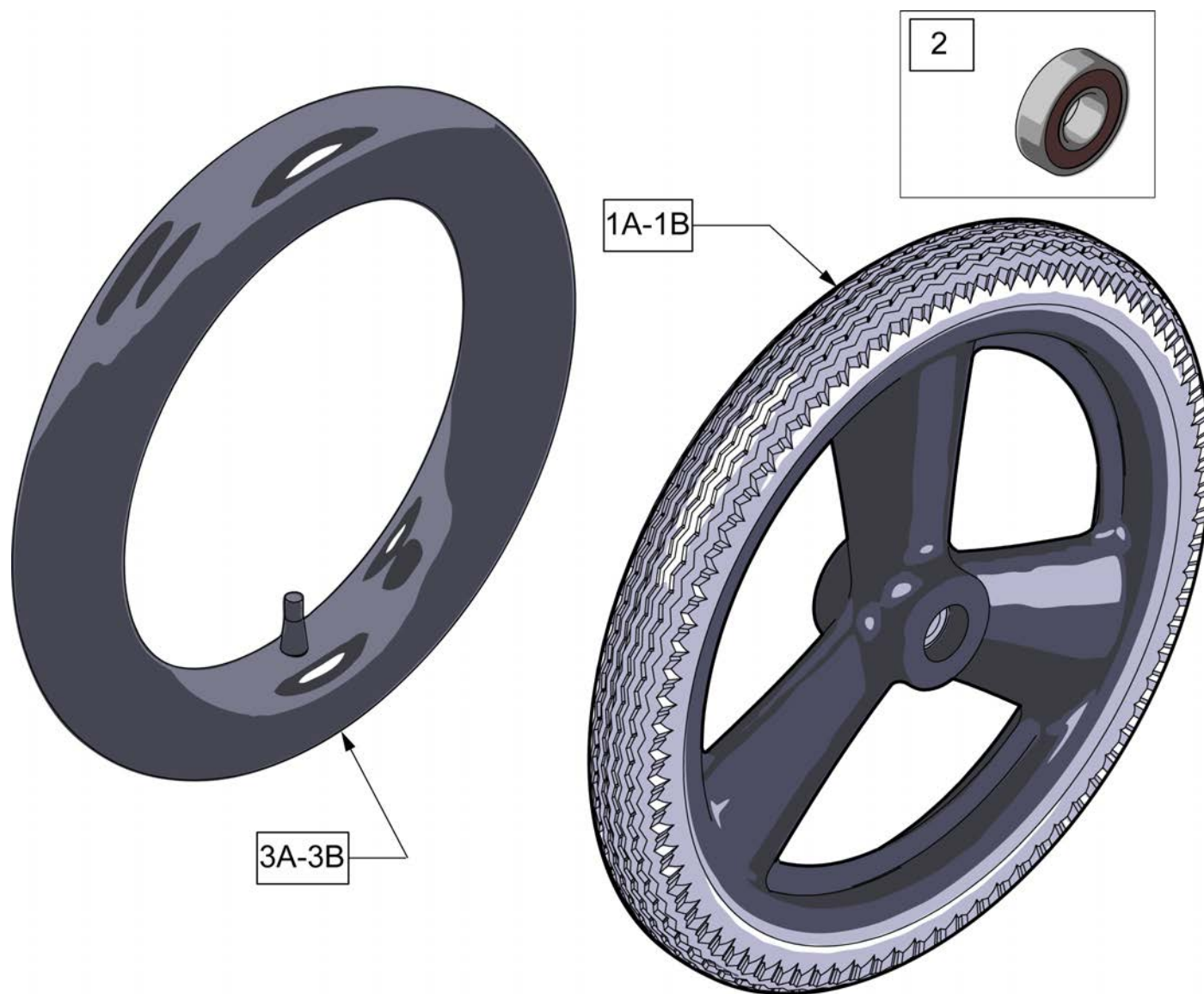

Pos.	Item Number	Description	Remarks
1A	101667	16 IN MAG PNEU 3 SPOKE	
1B	101668	16 IN MAG PNEU W/INSERT	
1C	385237	16" X 1.75" PNEU TIRE	Tire Only
2	349500	1 1/8 X 1/2 BEARING	
3A	384738	16" X 1.75" INNER TUBE	
3B	102973-001	16IN AIRLESS INSERT	

Note: this wheel utilizes Screw Mount Handrims - the handrim has a threaded hole that mounts the handrim to the wheel through a spacer and fastener.

Note: if Mag Wheel Assembly was ordered without Handrim and a Handrim is added to the wheel, qty 6 of 101932 and 281175 ARE REQUIRED to properly mount the handrim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

Note: for Zippie X'Cape 6 Spoke Mags are only available with the One Arm Drive.




Pos.	Item Number	Description	Remarks
1-3	401094	20" MAG W/BRNG 6 SPOKE	20"
1-3	401093	22" MAG W/BRNG 6 SPOKE	22"(501mm)
1-3	401092	24" MAG W/BRNG 6 SPOKE	24"(540mm)
2	349500	1 1/8 X 1/2 BEARING	
3	521564	MAG BEARING SPACER	
4	101932	SPCR .194ID X .375 OD X .750	Used when previously ordered with Omit handrims or Natural Fit Handrims

Note: 5 Spoke Mag is not compatible with One Arm Drive or Projection Handrims.

Note: 5 Spoke Mag utilizes exclusive screw mounted handrims.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

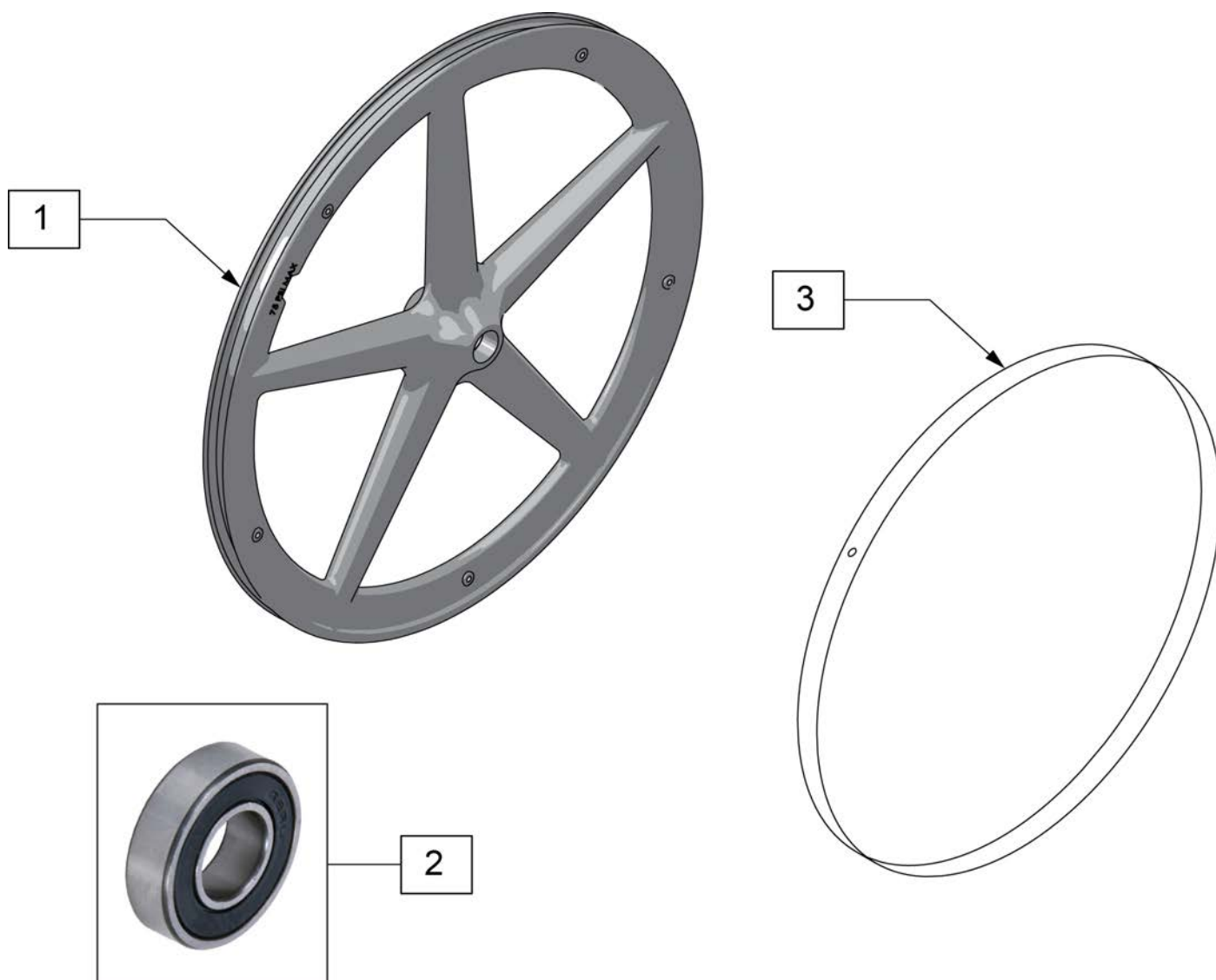

Pos.	Item Number	Description	Remarks
1,2	128985	20" 5 SPOKE MAG WHEEL	20" (451mm)
1,2	128986	22" 5 SPOKE MAG WHEEL	22" (501mm)
1,2	126405	24" MAG W/BRNG 5 SPOKE	24" (540mm)
2	349500	1 1/8 X 1/2 BEARING	
3	129325	20" MAG RIM STRIP	
3	129324	22" MAG RIM STRIP	
3	127912	24" MAG RIM STRIP	

Note: this Wheel utilizes Screw Mount Handrims; the Handrim has a threaded hole that mounts the Handrim to the Wheel through a spacer and fastener.

Note: the Lite Spoke Wheel has a total of 30 spokes.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

Note: when ordering replacement bearings, if dust shield on bearing has the following markings: MBL-Z2 6001 2RS, then order P/N 114097.



Pos.	Item Number	Description	Remarks
1A	118755	WHEEL ASSM GALLOP 20IN	20" (451)
1B	118756	WHEEL ASSM GALLOP 22IN	22" (501)
1C	107131	WHEEL RIM/SPOKE 24" ASSM	24" (540)
1D	118764	25 LT SPK W-HNDRM ANO ASSM GLP	25" (559)
1E	118760	WHEEL ASSM GALLOP 26IN	26" (590)
2A	114097	BEARING, REPLACEMENT, MBL	12.7mm i.d. - 28mm o.d. - 8mm wide; used prior to 11/29/10
2B	349500	1_125 X .5 BEARING	Beginning 11/29/10
3A	118100	MBL LT 20IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3B	118101	MBL LT 22IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3C	118102	MBL LT 24IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3D	118103	MBL LT 25IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3E	118104	MBL LT 26IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes

Note: this wheel utilizes Tab Mount Handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: 24 spoke count on all Ultra Light Spoke Wheels; spokes will be silver.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.



Pos.	Item Number	Description	Remarks
1	100798-001	WHL RIM 22" MBL ULTRALITE	22"(489mm)
1	100798-002	WHL RIM 24" MBL ULTRALITE	24"(540mm)
1	100798-003	WHL RIM 26" MBL ULTRALITE	26"(590mm)
2	349500	1 1/8 X 1/2 BEARING	

Note: this wheel utilizes tab mount handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: please see the Rear Wheel Finder for the complete Wheel Assembly with the Flex Rim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.




Pos.	Item Number	Description	Remarks
1	109242	SPINERGY STD WHL 22" BLK SPK	22"(489mm)
1	762438	SPINERGY STD WHEEL 540 BLK SPK	24"(540mm)
1	762441	SPINERGY STD WHL 559 BLK SPK	25"(559mm)
1	762437	SPINERGY STD WHEEL 590 BLK SPK	26"(590mm)
1	762442	SPINERGY STD WHEEL 540 BLUE	24"(540mm)
1	762444	SPINERGY STD WHEEL 559 BLU	25"(559mm)
1	762446	SPINERGY STD WHEEL 590 BLUE	26"(590mm)
1	762443	SPINERGY STD WHEEL 540 RED	24"(540mm)
1	762445	SPINERGY STD WHEEL 559 RED	25"(559mm)
1	762447	SPINERGY STD WHEEL 590 RED	26"(590mm)
1	762456	SPINERGY STD WHEEL 540 WHT	24"(540mm)
1	762433	SPINERGY STD WHEEL 540 YELLOW	24"(540mm)
1	762462	SPINERGY STD WHL 559 YLW SPK	25"(559mm)
1	762431	SPINERGY STD WHEEL 590 YELLOW	26"(590mm)
2	349500	1 1/8 X 1/2 BEARING	

Note: this wheel utilizes tab mount handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: please see the Rear Wheel Finder for the complete Wheel Assembly with the Flex Rim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

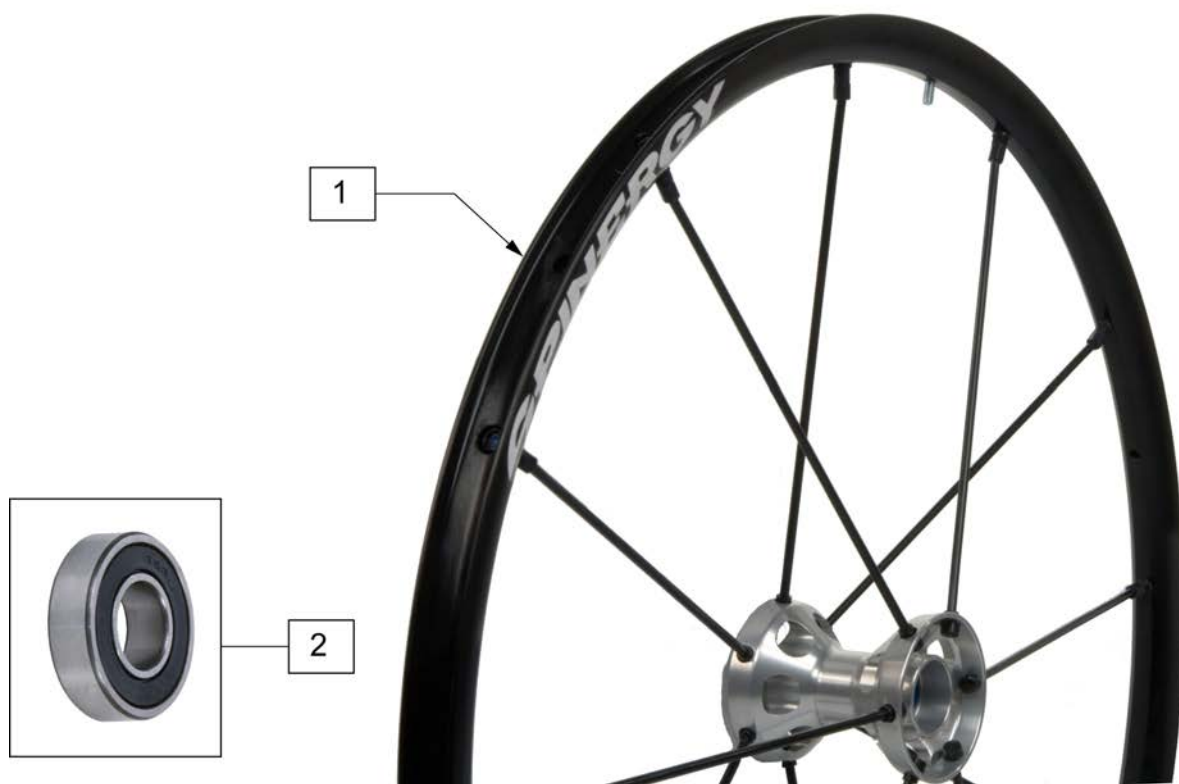

Pos.	Item Number	Description	Remarks
1	108858	SPINERGY LT EX WHL 22" BLK	22"(489mm); 12 Spoke
1	109215	SPINERGY LT EX WHL 22" BLU	22"(489mm); 12 Spoke
1	109216	SPINERGY LT EX WHL 22" RED	22"(489mm); 12 Spoke
1	109217	SPINERGY LT EX WHL 22" WHT	22"(489mm); 12 Spoke
1	109218	SPINERGY LT EX WHL 22" YLW	22"(489mm); 12 Spoke
1	101989-001	SPINERGY LT EX WHEEL 540 BLK	24"(540mm); 12 Spoke
1	101989-002	SPINERGY LT EX WHEEL 540 BLU	24"(540mm); 12 Spoke
1	101989-003	SPINERGY LT EX WHEEL 540 RED	24"(540mm); 12 Spoke
1	101989-005	SPINERGY LT EX WHEEL 540 WHT	24"(540mm); 12 Spoke
1	101989-004	SPINERGY LT EX WHEEL 540 YLW	24"(540mm); 12 Spoke
1	101991-001	SPINERGY LT EX WHEEL 559 BLK	25"(559mm); 12 Spoke
1	101991-002	SPINERGY LT EX WHEEL 559 BLU	25"(559mm); 12 Spoke
1	101991-003	SPINERGY LT EX WHEEL 559 RED	25"(559mm); 12 Spoke
1	101991-004	SPINERGY LT EX WHEEL 559 YLW	25"(559mm); 12 Spoke
1	101993-001	SPINERGY LT EX WHEEL 590 BLK	26"(590mm); 12 Spoke
1	101993-004	SPINERGY LT EX WHEEL 590 YLW	26"(590mm); 12 Spoke
2	349500	1 1/8 X 1/2 BEARING	

Note: this wheel utilizes tab mount handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: please see the Rear Wheel Finder for the complete Wheel Assembly with the Flex Rim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.




Pos.	Item Number	Description	Remarks
1	101988-001	SPINRGY SPT LT EX WHL 540 BLK	24"(540mm), 18 spoke
1	101988-002	SPINRGY SPT LT EX WHL 540 BLU	24"(540mm), 18 spoke
1	101988-003	SPINRGY SPT LT EX WHL 540 RED	24"(540mm), 18 spoke
1	101988-004	SPINRGY SPT LT EX WHL 540 YLW	24"(540mm), 18 spoke
1	101988-005	SPINRGY SPT LT EX WHL 540 WHT	24"(540mm), 18 spoke
1	101990-001	SPINRGY SPT LT EX WHL 559 BLK	25"(559mm), 18 spoke
1	101990-002	SPINRGY SPT LT EX WHL 559 BLU	25"(559mm), 18 spoke
1	101990-003	SPINRGY SPT LT EX WHL 559 RED	25"(559mm), 18 spoke
1	101990-004	SPINRGY SPT LT EX WHL 559 YLW	25"(559mm), 18 spoke
1	101990-005	SPINRGY SPT LT EX WHL 559 WHT	25"(559mm), 18 spoke
1	101992-001	SPINRGY SPT LT EX WHL 590 BLK	26"(590mm), 18 spoke
1	101992-002	SPINRGY SPT LT EX WHL 590 BLU	26"(590mm), 18 spoke
1	101992-003	SPINRGY SPT LT EX WHL 590 RED	26"(590mm), 18 spoke
1	101992-004	SPINRGY SPT LT EX WHL 590 YLW	26"(590mm), 18 spoke
1	101992-005	SPINRGY SPT LT EX WHL 590 WHT	26"(590mm), 18 spoke
2	349500	1 1/8 X 1/2 BEARING	



Pos.	Item Number	Description	Remarks
1	385237	16" X 1.75" PNEU TIRE	
1	385250	20" PNEUMATIC TIRE	
1	385275	22" PNEUMATIC TIRE	22" x 1 3/8 (501mm)
1	385300	24" PNEUMATIC TIRE	24" x 1 3/8 (540mm)
1	385325	26" PNEUMATIC TIRE	26" x 1 3/8 (590mm)
1	384614	24" x 2.125 PNEUMATIC TIRE	24"x2 1/8 (540mm) used w/ M6




Pos.	Item Number	Description	Remarks
1	386232	20" POLY TIRE STD	20"
1	386242	22" POLY TIRE STD	22" (501mm)
1	386252	24" POLY TIRE STD	24" (540mm)

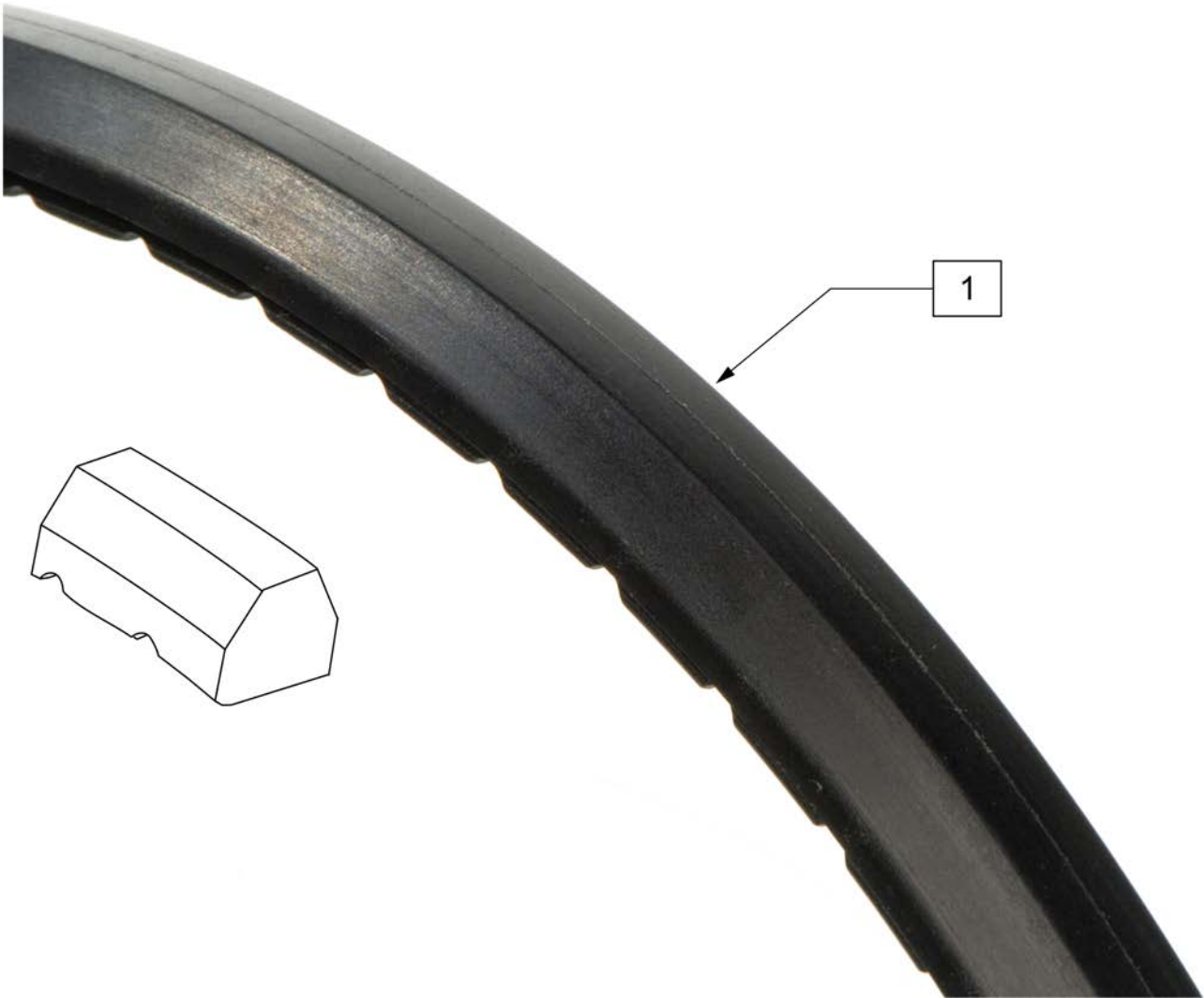

Pos.	Item Number	Description	Remarks
1	386262	20" LP POLY TIRE BR	20" =16" dia
1	386263	22" LP POLY TIRE BR	22"(501mm) = 18"dia
1	386254	24" POLY LOW PROFILE	24"(540mm)= 23" dia

Note: Shox Tire do not fit Topolino Wheels due to width.

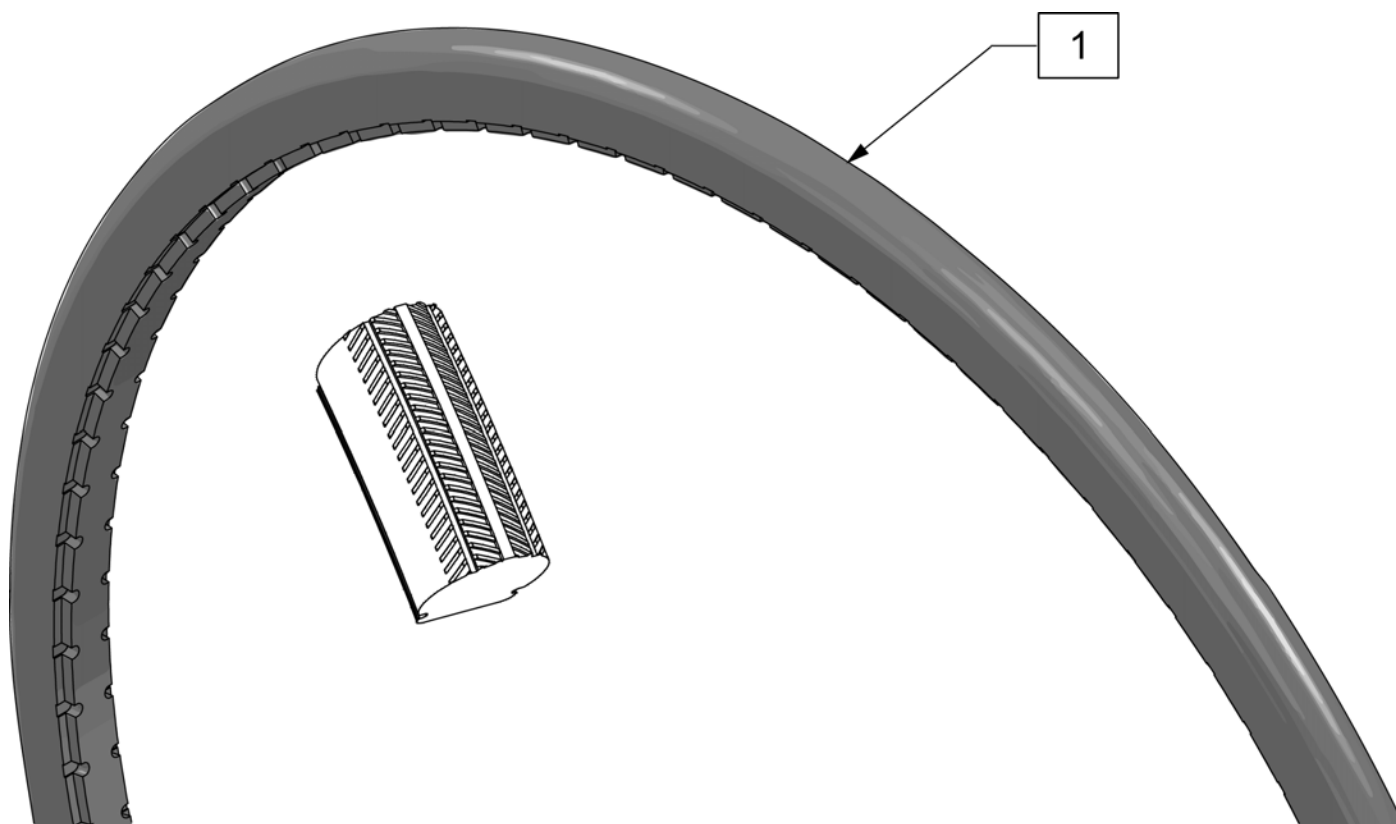


Pos.	Item Number	Description	Remarks
1	129924	TIRE SHOX CEW20"	20"(457mm) Black
1	129925	TIRE SHOX CEW22"	22"(501mm) Black
1	129926	TIRE SHOX CEW24"	24"(540mm) Black
1	129927	TIRE SHOX CEW25"	25"(559mm) Black





Pos.	Item Number	Description	Remarks
1	102066-004	TIRE SCHWALBE MP 489 BLK	22" (489mm)
1	102066-001	TIRE SCHWBL MPE 540 BLK	24" (540mm)
1	102066-002	TIRE SCHWBL MPE 559 BLK	25" (559mm)
1	102066-003	TIRE SCHWBL MPE 590 BLK	26" (590mm)




Pos.	Item Number	Description	Remarks
1	113247	TIRE SCHWALBE ONE 24 IN	24"(540mm)
1	113248	TIRE SCHWALBE ONE 25 IN	25"(559mm)

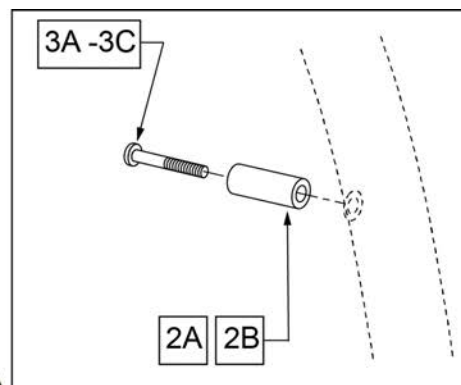


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	LITE SPOKE	>>>>>
1,2A,3A	159508	20IN LT SPK AA HNDRIM W/HDWR	20" LITE SPOKE
1,2A,3A	159509	22IN LT SPK AA HNDRIM W/HDWR	22" (501) LITE SPOKE
1,2A,3A	159510	24IN LT SPK AA HNDRIM W/HDWR	24" (540) LITE SPOKE
1,2A,3A	159511	25IN LT SPK AA HNDRIM W/HDWR	25" (559) LITE SPOKE
1,2A,3A	159512	26IN LT SPK AA HNDRIM W/HDWR	26" (590) LITE SPOKE
2A,3A	159513	LT SPK AL AN HNDRIM HDWR	LITE SPOKE, SET OF 6
>>>>>	>>>>>>>>>	SPOKE	>>>>>
1,2B,3B	425931	20" HR 6SMT W/HDWR	20" SPOKE
1,2B,3B	425941	22" HR 6SMT W/HDWR	22" (501) SPOKE
1,2B,3B	426000	24" HR 6SMT W/ HDWR	24" (540) SPOKE
2B,3B	159514	SPK AL AN HNDRIM HDWR	SPOKE, SET OF 6
>>>>>	>>>>>>>>>	MAG	>>>>>
1,3C	425934	20" HR 6SMT 6MAG W/HDWR	20" MAG
1,3C	425951	22" HR 6SMT 6MAG W/HDWR	22" (501) MAG
1,3C	426009	24" HR 6SMT 6MAG W/HDWR	24" (540) MAG
3C	281175	10-32 X 2 1/8 PAN PHIL SS	MAG
>>>>>	>>>>>>>>>	M6 SPOKE	>>>>>
1,2B,3B	425941	22" HR 6SMT W/HDWR	24" (540) SPOKE

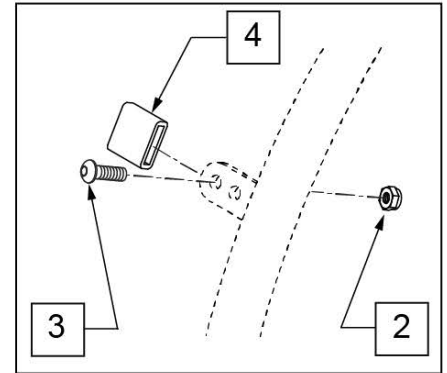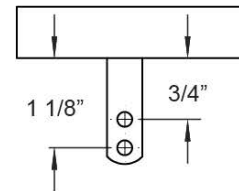**MEASURING GUIDE**

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	CROSS SPOKE & SPOKE	>>>>>
1 -4	425095	HANDRIM 18" 4T W/HDWR ANO	18" SPOKE
1	529529	QP 24" HR .065 (6) TAB ANO	24" (540) CROSS SPOKE -SHORT & REG TAB
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (590) CROSS SPOKE
>>>>>	>>>>>>>>>>	CROSS SPOKE QP (QUICKIE PERFORMANCE)	>>>>>
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) CROSS SPOKE TENNIS & BBALL
1 -4	425045	24" HR SHT(6T)ANO W/X/S HDWR	25" (590) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	425048	26" HR (6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL -REG TAB
1 -4	425055	26" HR SHT(6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	426015	QP 24" HR .049 W/HDWR SMT	24" (540) TENNIS
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (559) TENNIS & BASKETBALL
1 -4	426050	QP 26" HR .065 W/HDWR (6T)	26" (700) TENNIS & BASKETBALL
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) TENNIS & BASKETBALL
>>>>>	>>>>>>>>>>	SPINERGY & PROTON	>>>>>
1	529529	QP 24" HR .065 (6) TAB ANO	24" (540) SPINERGY
1 -4	425038	QP 24" HR .049 W/HDWR (6T)	25" (559) SPINERGY
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) SPINERGY
>>>>>	>>>>>>>>>>	MOUNTAIN	>>>>>
1 -4	425020	22" HR 4T W/HDWR	24" (540) MOUNTAIN
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	26" (590) MOUNTAIN
2-4	159515	AA TAB MOUNT HANDRIM HARDWARE	SET OF 6

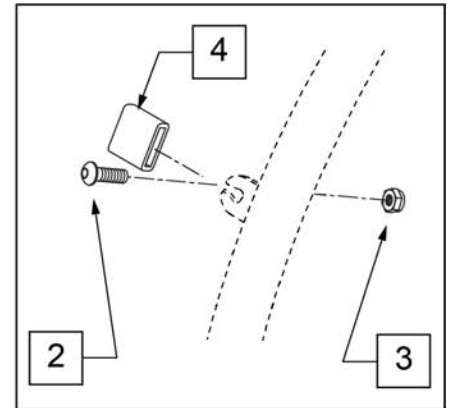


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	MOUNTAIN	>>>>>
1,2	425022	22" HR 4T PLST W/ HDWR	24" (540)
>>>>>	>>>>>>>>>	SPINERGY & PROTON	>>>>>
1,2	425039	24"HR 6T BLK PLST W/HDWR	24" (540)
1,2	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
>>>>>	>>>>>>>>>	QP (QUICKIE PERFORMANCE)	>>>>>
1,2	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
1,2	426060	QP 26" HR W/ HDWR (6T) PLASTIC	26" (700)
2	159535	PLST COATED TAB HNDRM HDWR	

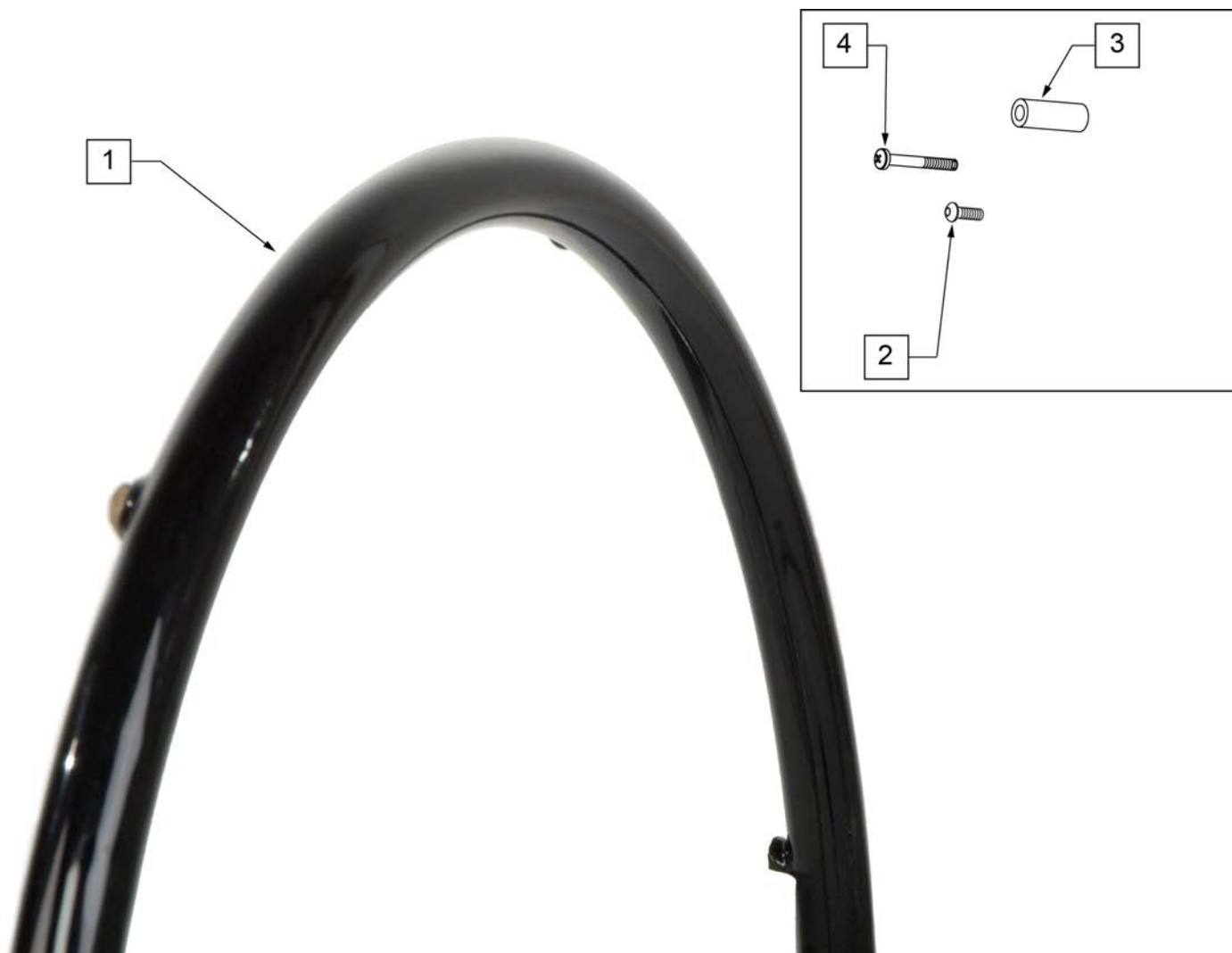

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	SPOKE & LITE SPOKE	>>>>>
1-3	425933	20" HR 6SMT PLST W/HDWR	20" SPOKE
1-3	425943	22" HR 6SMT PLST W/HDWR	22" (501) SPOKE
1-3	426004	24" HR 6SMT PLST W/HDWR	24" (540) SPOKE
>>>>>	>>>>>>>>>>	MAG	>>>>>
1,4	425935	20" HR 6SMT 6MAG PLST W/HDWR	20" MAG
1,4	425953	22" HR 6SMT 6MAG PLST W/HDWR	22" (501) MAG
1,4	426013	24" HR 6SMT 6MAG PLST W/HDWR	24" (540) MAG
>>>>>	>>>>>>>>>>	M6 SPOKE	>>>>>
1-3	425943	22" HR 6SMT 6 MAG PLST W/ HDWR	24" (540) M6 SPOKE
2,3	159534	PLST COATED HNDRM HARDWARE	SPOKE; SET OF 6
4	281175	10 -32 x 2 1/8 PAN PHIL SS	6 SPOKE MAG


Note: Screw Mount Handrims are utilized with Lite Spoke, Spoke and 6 Spoke Mag Wheels.

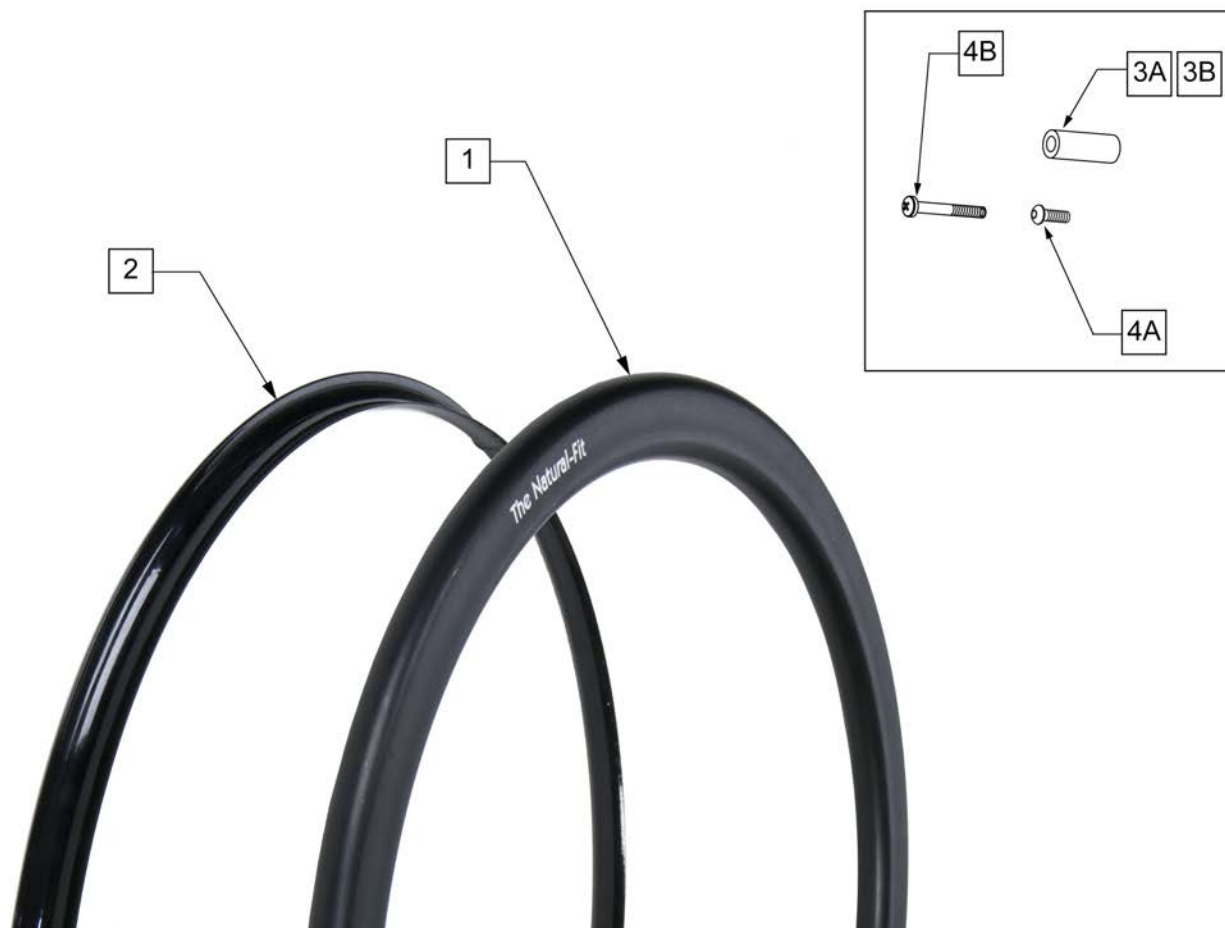

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	STD GRIP THUMB GUARD	>>>>>
2	102168-001	GUARD THUMB NAT FIT 22 PC BLK	22" (501)- Std Grip Thumb Guard
2	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)- Std Grip Thumb Guard
2	102168-005	GUARD THUMB NAT FIT 25 PC BLK	25" (559)- Std Grip Thumb Guard
2	102168-007	GUARD THUMB NAT FIT 26 PC BLK	26" (590)- Std Grip Thumb Guard
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
2	102168-002	GUARD THUMB NAT FIT 22 SG BLK	22" (501)- Super Grip Thumb Guard
2	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)- Super Grip Thumb Guard
2	102168-006	GUARD THUMB NAT FIT 25 SG BLK	25" (559)- Super Grip Thumb Guard
2	102168-008	THUMB GUARD NAT FIT 26 SG BLK	26" (590)- Super Grip Thumb Guard
>>>>>	>>>>>>>>>	NATURAL FIT LITE HANDRIM	>>>>>
1,3A,4A	159516	22 NAT-FIT LT HNDRIM W/HDWR	22" (501)
1,3A,4A	159517	24 NAT-FIT LT HNDRIM W/HDWR	24" (540)
1,3A,4A	159518	25 NAT-FIT LT HNDRIM W/HDWR	25" (559)
1,3A,4A	159519	26 NAT-FIT LT HNDRIM W/HDWR	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT HANDRIM	>>>>>
1,3A,4A	159520	22 NAT-FIT HNDRIM W/HDWR	22" (501)
1,3A,4A	159521	24 NAT-FIT HNDRIM W/HDWR	24" (540)

Pos.	Item Number	Description	Remarks
1,3A,4A	159522	25 NAT-FIT HNDRIM W/HDWR	25" (559)
1,3A,4A	159523	26 NAT-FIT HNDRIM W/HDWR	26" (590)
3A,4A	159524	NAT-FIT HNDRIM LT HDWR SET 6	USED W/ LITE SPOKE & SPOKE; SET OF 6
3B	101932	SPCR .194ID X .375 OD X .750	USED W/ MAG
4B	281175	10-32 X 2 1/8 PAN PHIL SS	USED W/ MAG

Note: Tab Mount Hand Rims are utilized with Spinergy & Toplino Wheels.

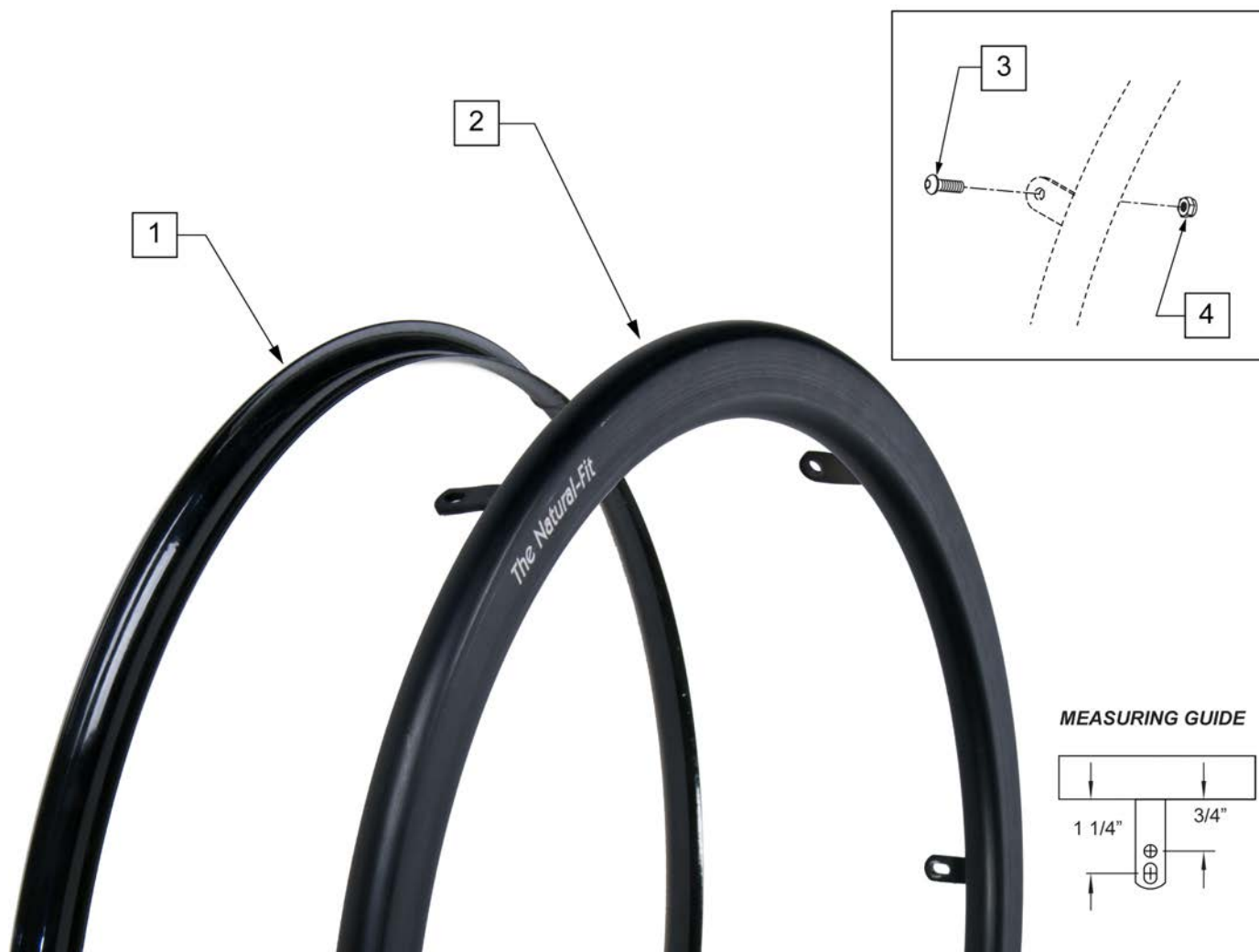


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	STD GRIP THUMB GRIP	>>>>>
1	102168-001	GUARD THUMB NAT FIT 22 PC BLK	22" (501)
1	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)
1	102168-005	GUARD THUMB NAT FIT 25 PC BLK	25" (559)
1	102168-007	GUARD THUMB NAT FIT 26 PC BLK	26" (590)
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
1	102168-002	GUARD THUMB NAT FIT 22 SG BLK	22" (501)
1	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)
1	102168-006	GUARD THUMB NAT FIT 25 SG BLK	25" (559)
1	102168-008	THUMB GUARD NAT FIT 26 SG BLK	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT LITE HANDRIM	>>>>>
2-4	159525	22 NAT-FIT LT TB HNDRIM W/HDWR	22" (501)
2-4	159526	24 NAT-FIT LT TB HNDRIM W/HDWR	24" (540)
2-4	159527	25 NAT-FIT LT TB HNDRIM W/HDWR	25" (559)

Pos.	Item Number	Description	Remarks
2-4	159528	26 NAT-FIT LT TB HNDRIM W/HDWR	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT -W/O THUMB GRIP	>>>>>
2-4	159529	22 NAT-FIT TB HNDRIM W/HDWR	22" (501)
2-4	159530	24 NAT-FIT TB HNDRIM W/HDWR	24" (540)
2-4	159531	25 NAT-FIT TB HNDRIM W/HDWR	25" (559)
2-4	159532	26 NAT-FIT TB HNDRIM W/HDWR	26" (590)
3,4	159533	NAT-FIT TAB MNT HARDWARE	SET OF 6




Pos.	Item Number	Description
2,3	159507	5 SPK HNDRM HDWR
1-3	136155	20" PLSTC COATED HND RIM KIT
1-3	136154	22" PLSTC COATED HND RIM KIT
1-3	136153	24" PLSTC COATED HND RIM KIT

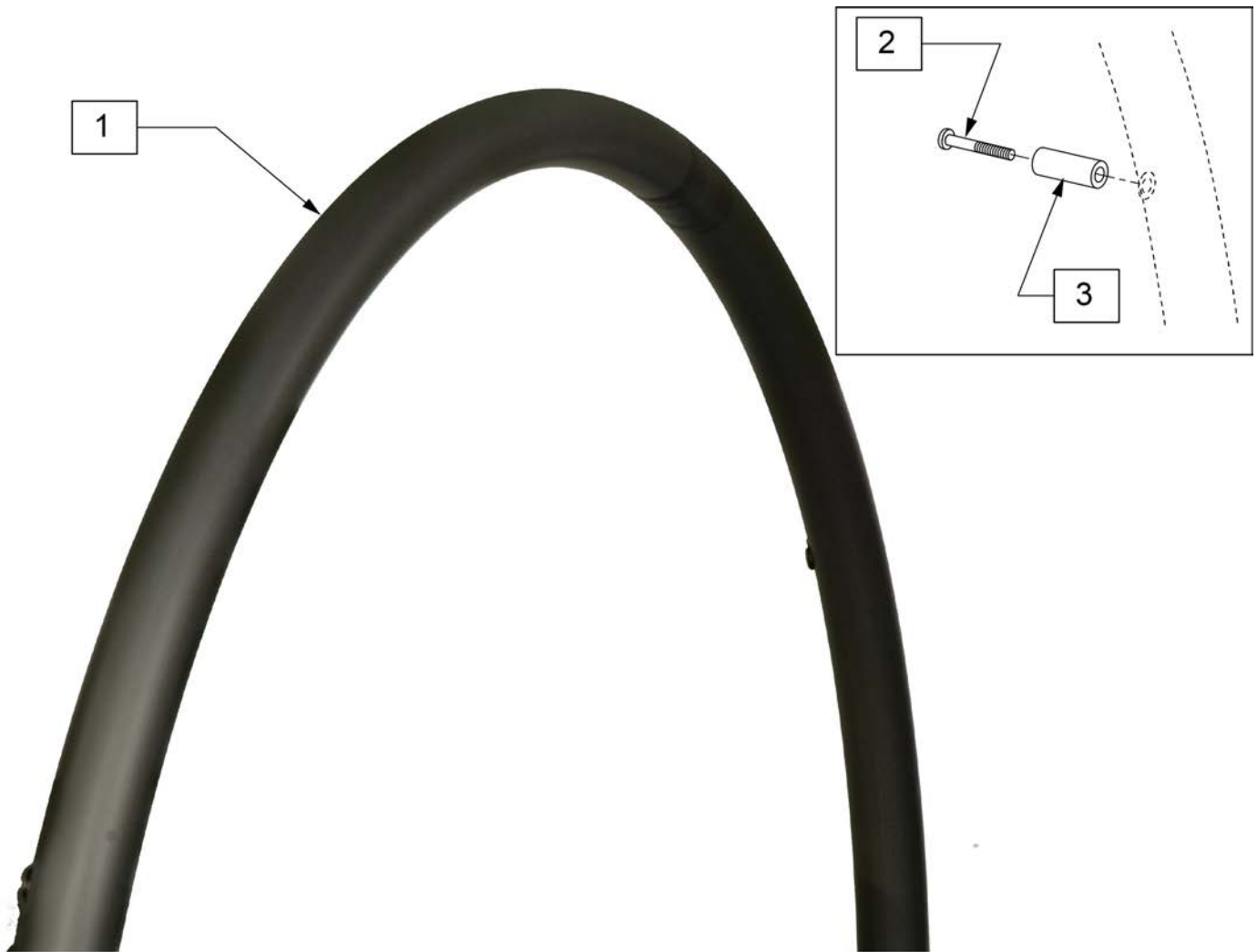

Pos.	Item Number	Description	Remarks
1-3	136158	20" ALU ANO HND RIM KIT	
1-3	136157	22" ALU ANO HND RIM KIT	
1-3	136156	24" ALU ANO HND RIM KIT	
2,3	159507	5 SPK HNDRM HDWR	set of 5




Pos.	Item Number	Description	Remarks
1A	Discontinued	Presta Valves Discontinued	
1A	384736	INNER TUBE 12-1/2" X 2-1/4"	
1A	384738	16" X 1.75" INNER TUBE	
1A	382255	TUBE 18" TIRE	18" X 1-3/4" TUBE
1A	382275	20" TUBE	20" 1-3/8" TUBE
1A	382300	22" TUBE	22" X 1-3/8" TUBE
1A	382336	TUBE PRIMO 24" 25-540 SV	24" X 1" TUBE
1A	382325	24 X 1.3/8IN TUBE REG	
1A	382334	TUBE INLINE 24 X 1.75	
1A	384551	26" X 1" TUBE SCHRADER	
1A	382350	26" TUBE	26" X 1-3/8" TUBE
1A	382361	TUBE 26 X 1.5/1.95/ S/V	
1A	382365	26 X 1.25/1.5 TUBE SHRADER	
1A	382375	24" x 2.125 TUBE	
1B	102972-001	12 1/2 IN AIRLESS INSERT	12-1/2" X 2-1/4"
1B	102973-001	16IN AIRLESS INSERT	16" X 1-3/4"
1B	102973-002	18IN AIRLESS INSERT	18" X 1-3/4"
1B	102973-003	20IN AIRLESS INSERT	20" X 1-3/8"
1B	102972-002	AIRLESS INSERT 20 X 2-1/8 P3	
1B	102973-004	22IN AIRLESS INSERT	22" X 1-3/8"
1B	102973-005	24IN AIRLESS INSERT	24" X 1-3/8"
1B	102972-003	AIRLESS INSERT 24 X 2-1/8	
1B	108259	26" AIRLESS INSERT	26" X 1-3/8"

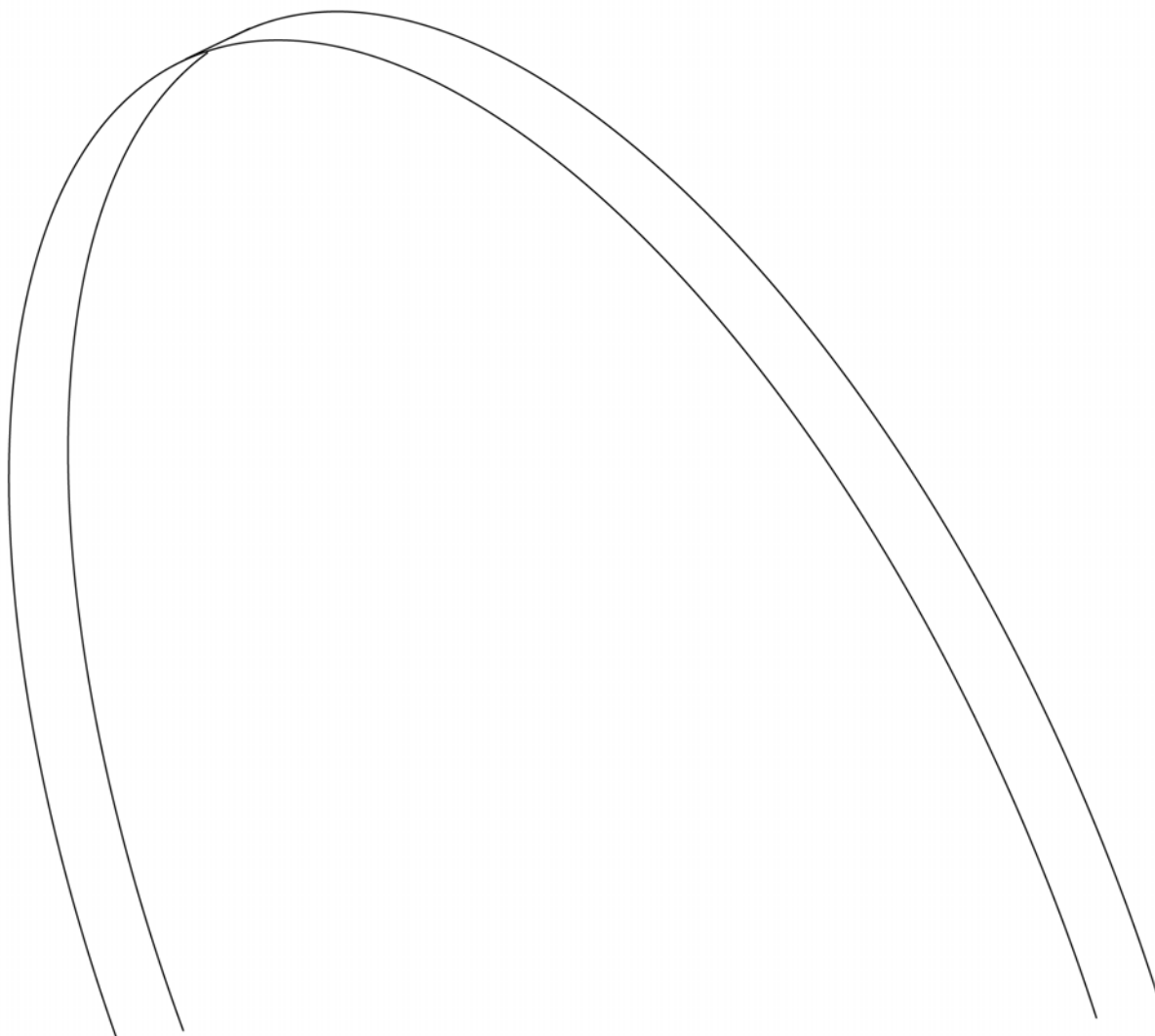



Pos.	Item Number	Description	Remarks
1	381235	STRIP RIM 18"	18"
1	381250	20" RIM STRIP	20"
1	381265	22" RIM STRIP	22" (501)
1	381275	24" RIM STRIP	24" (540)
1	381350	QP 26" HT RIM STRIP	25" (559)
1	381300	26" RIM STRIP	26" (590)
1	381301	RIM STRIP 24 x 1.75/2.125	24" X 2.125" (USED ON 26" MT)
1	381210	RIMSTRIP 16mm RL	27" (700)
1	129325	20" MAG RIM STRIP	5 Spoke mag 20"
1	129324	22" MAG RIM STRIP	5 Spoke mag 22"
1	127912	24" MAG RIM STRIP	5 Spoke mag 24"

Note: for proper fit open the velcro flap on the under side of the sling for installation; the sling may measure short, but by opening the flap it will adjust to fit the frame.

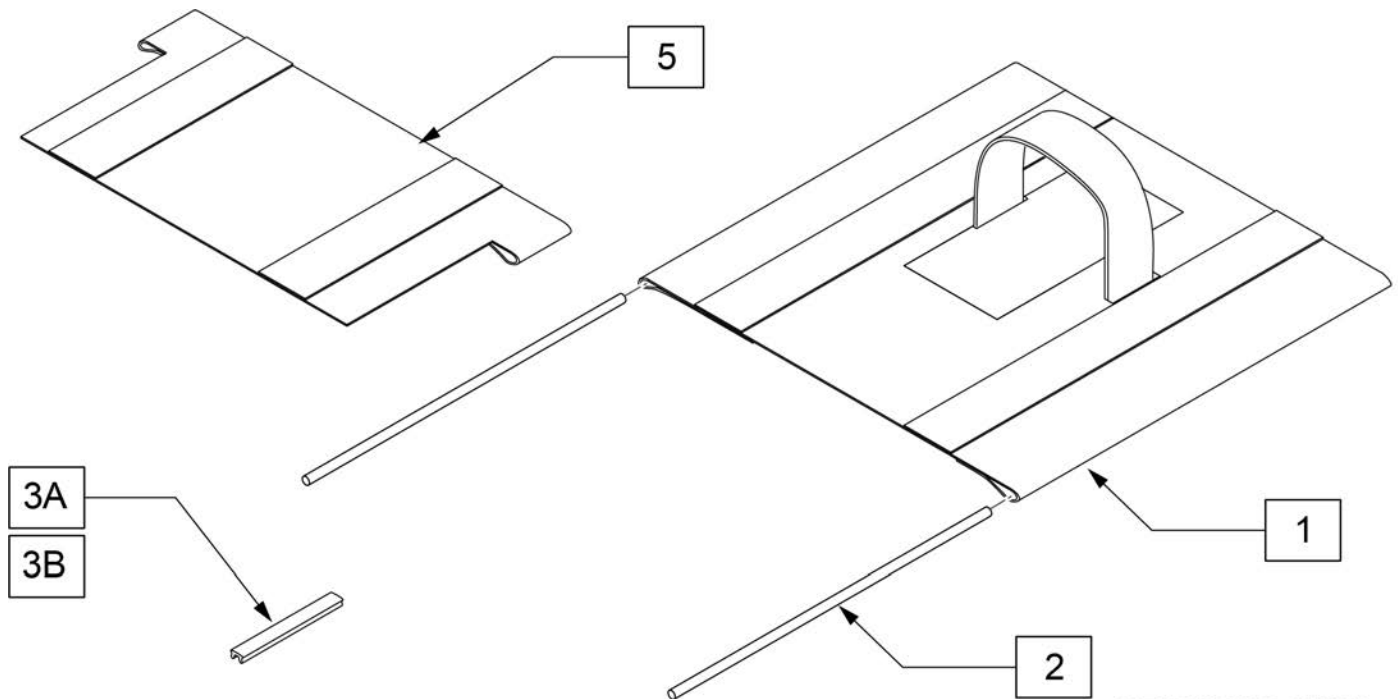

MEASURING GUIDE


Chair Width	Dim "A"	Dim "B"
8"	11"	8"
9"	12"	9"
10"	13"	10"
11"	14"	11"
12"	15"	12"
13"	16"	13"
14"	17"	14"
15"	18"	15"
16"	19"	16"
17"	20"	17"
18"	21"	18"

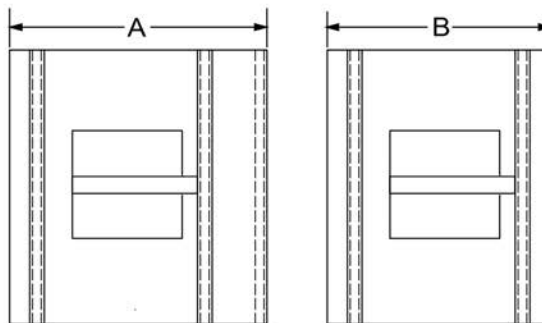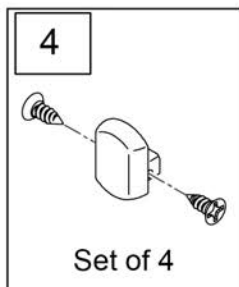

Pos.	Item Number	Description	Remarks
1	128658	8 X 8 SS WINDOW	
1	128704	8 X 9 SS WINDOW	
1	128659	8 X 10 SS WINDOW	
1	128660	8 X 11 SS WINDOW	
1	128661	8 X 12 SS WINDOW	
1	128662	8 X 13 SS WINDOW	
1	128663	8 X 14 SS WINDOW	
1	128664	9 X 8 SS WINDOW	
1	128665	9 X 9 SS WINDOW	
1	128666	9 X 10 SS WINDOW	
1	128667	9 X 11 SS WINDOW	
1	128668	9 X 12 SS WINDOW	
1	128669	9 X 13 SS WINDOW	

Pos.	Item Number	Description	Remarks
1	128670	9 X 14 SS WINDOW	
1	128671	9 X 15 SS WINDOW	
1	128672	10 X 8 SS WINDOW	
1	128673	10 X 9 SS WINDOW	
1	128674	10 X 10 SS WINDOW	
1	128675	10 X 11 SS WINDOW	
1	128676	10 X 12 SS WINDOW	
1	128677	10 X 13 SS WINDOW	
1	128678	10 X 14 SS WINDOW	
1	128679	10 X 15 SS WINDOW	
1	128680	11 X 8 SS WINDOW	
1	128681	11 X 9 SS WINDOW	
1	128682	11 X 10 SS WINDOW	
1	128683	11 X 11 SS WINDOW	
1	128684	11 X 12 SS WINDOW	
1	128685	11 X 13 SS WINDOW	
1	128686	11 X 14 SS WINDOW	
1	128687	11 X 15 SS WINDOW	
1	128688	11 X 16 SS WINDOW	
1	128689	12 X 9 SS WINDOW	
1	128690	12 X 10 SS WINDOW	
1	128691	12 X 11 SS WINDOW	
1	128692	12 X 12 SS WINDOW	
1	128693	12 X 13 SS WINDOW	
1	128694	12 X 14 SS WINDOW	
1	128695	12 X 15 SS WINDOW	
1	128696	12 X 16 SS WINDOW	
1	128697	12 X 17 SS WINDOW	
1	128698	13 X 10 SS WINDOW	
1	128699	13 X 11 SS WINDOW	
1	128700	13 X 12 SS WINDOW	
1	128701	13 X 13 SS WINDOW	
1	128702	13 X 14 SS WINDOW	
1	128703	13 X 15 SS WINDOW	
1	128843	13 X 16 SS WINDOW	
1	128705	13 X 17 SS WINDOW	
1	128706	13 X 18 SS WINDOW	
1	128707	14 X 11 SS WINDOW	
1	128708	14 X 12 SS WINDOW	
1	128709	14 X 13 SS WINDOW	
1	128710	14 X 14 SS WINDOW	
1	128711	14 X 15 SS WINDOW	
1	128712	14 X 16 SS WINDOW	
1	128713	14 X 17 SS WINDOW	
1	128714	14 X 18 SS WINDOW	
1	128715	15 X 11 SS WINDOW	
1	128716	15 X 12 SS WINDOW	
1	128717	15 X 13 SS WINDOW	
1	128718	15 X 14 SS WINDOW	
1	128719	15 X 15 SS WINDOW	
1	128720	15 X 16 SS WINDOW	
1	128721	15 X 17 SS WINDOW	
1	128722	15 X 18 SS WINDOW	
1	128723	16 X 12 SS WINDOW	
1	128724	16 X 13 SS WINDOW	
1	128725	16 X 14 SS WINDOW	
1	128726	16 X 15 SS WINDOW	


Pos.	Item Number	Description	Remarks
1	128727	16 X 16 SS WINDOW	
1	128728	16 X 17 SS WINDOW	
1	128729	16 X 18 SS WINDOW	
1	128730	17 X 12 SS WINDOW	
1	128731	17 X 13 SS WINDOW	
1	128732	17 X 14 SS WINDOW	
1	128733	17 X 15 SS WINDOW	
1	128734	17 X 16 SS WINDOW	
1	128735	17 X 17 SS WINDOW	
1	128736	17 X 18 SS WINDOW	
1	128737	18 X 12 SS WINDOW	
1	128738	18 X 13 SS WINDOW	
1	128739	18 X 14 SS WINDOW	
1	128740	18 X 15 SS WINDOW	
1	128741	18 X 16 SS WINDOW	
1	128742	18 X 17 SS WINDOW	
1	128743	18 X 18 SS WINDOW	
2	127592	SEAT SLING ROD 8" X 3/16"	13.92" Cut to sling depth length during install
2	127596	SEAT SLING ROD 9" X 3/16"	14.92" Cut to sling depth length during install
2	127597	SEAT SLING ROD 10" X 3/16"	15.92" Cut to sling depth length during install
2	127598	SEAT SLING ROD 11" X 3/16"	16.92" Cut to sling depth length during install
2	127599	SEAT SLING ROD 12" X 3/16"	17.92" Cut to sling depth length during install
2	127600	SEAT SLING ROD 13" X 3/16"	18.92" Cut to sling depth length during install
2	127601	SEAT SLING ROD 14" X 3/16"	19.92" Cut to sling depth length during install
2	127602	SEAT SLING ROD 15" X 3/16"	20.92" Cut to sling depth length during install
2	127603	SEAT SLING ROD 16" X 3/16"	21.92" Cut to sling depth length during install
2	127604	SEAT SLING ROD 17" X 3/16"	22.92" Cut to sling depth length during install
2	127605	SEAT SLING ROD 18" X 3/16"	23.92" Cut to sling depth length during install
3A	127594	SEAT SLING STOP	6.5" Cut to length to fill gap between edge of sling to front of the frame during install.
3B	127606	OMIT SEAT SLING STOP	26" for omit seat sling option ONLY
4	128828	FRAME END CAP KIT	includes 4 caps and 8 screws
5	130984	ZIPPIE GROWTH SLING EXT 8W	
5	130994	ZIPPIE GROWTH SLING EXT 9W	
5	130995	ZIPPIE GROWTH SLING EXT 10W	
5	130996	ZIPPIE GROWTH SLING EXT 11W	
5	130997	ZIPPIE GROWTH SLING EXT 12W	
5	130998	ZIPPIE GROWTH SLING EXT 13W	
5	130999	ZIPPIE GROWTH SLING EXT 14W	
5	131038	ZIPPIE GROWTH SLING EXT 15W	
5	131039	ZIPPIE GROWTH SLING EXT 16W	
5	131040	ZIPPIE GROWTH SLING EXT 17W	
5	131041	ZIPPIE GROWTH SLING EXT 18W	

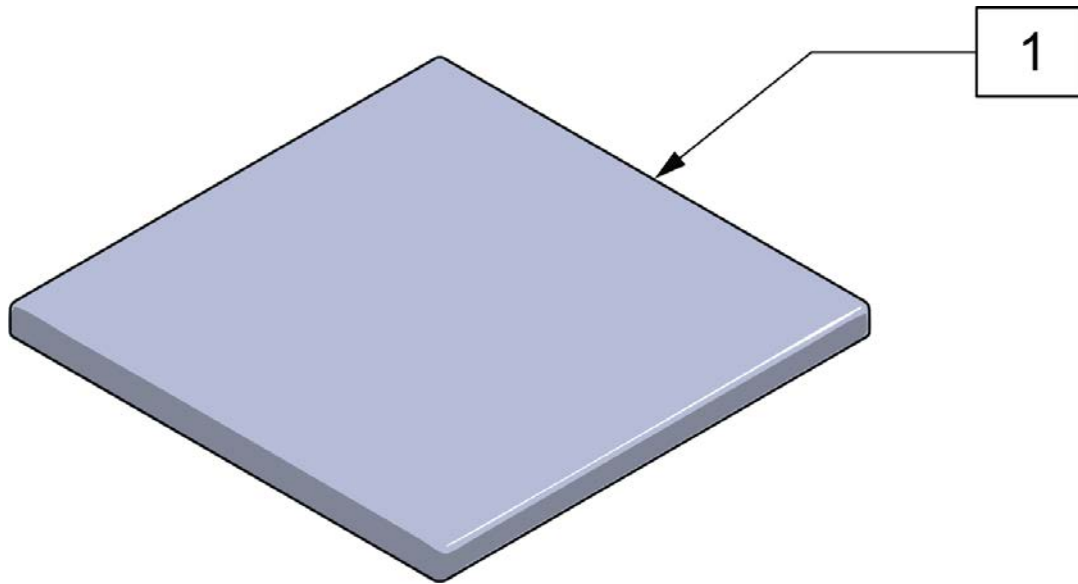

Pos.	Item Number	Description	Pos.	Item Number	Description
		8" WIDE SEAT CUSHIONS			
1	115444	Seat Cushion 1 Thk, 8x8	1	115484	Seat Cushion 1 Thk, 11x15
1	115445	Seat Cushion 1 Thk, 8x9	1	115485	Seat Cushion 1 Thk, 11x16
1	115446	Seat Cushion 1 Thk, 8x10			12" WIDE SEAT CUSHIONS
1	115447	Seat Cushion 1 Thk, 8x11	1	115448	Seat Cushion 1 Thk, 12X8
1	115448	Seat Cushion 1 Thk, 8x12	1	115459	Seat Cushion 1 Thk, 12X9
1	115449	Seat Cushion 1 Thk, 8x13	1	115470	Seat Cushion 1 Thk, 12X10
1	115450	Seat Cushion 1 Thk, 8x14	1	115481	Seat Cushion 1 Thk, 12X11
		9" WIDE SEAT CUSHIONS	1	115492	Seat Cushion 1 Thk, 12x12
1	115445	Seat Cushion 1 Thk, 9X8	1	115493	Seat Cushion 1 Thk, 12x13
1	115456	Seat Cushion 1 Thk, 9x9	1	115494	Seat Cushion 1 Thk, 12x14
1	115457	Seat Cushion 1 Thk, 9x10	1	115495	Seat Cushion 1 Thk, 12x15
1	115458	Seat Cushion 1 Thk, 9x11	1	115496	Seat Cushion 1 Thk, 12x16
1	115459	Seat Cushion 1 Thk, 9x12	1	115497	Seat Cushion 1 Thk, 12x17
1	115460	Seat Cushion 1 Thk, 9x13			13" WIDE SEAT CUSHIONS
1	115461	Seat Cushion 1 Thk, 9x14	1	115449	Seat Cushion 1 Thk, 13X8
1	115462	Seat Cushion 1 Thk, 9x15	1	115460	Seat Cushion 1 Thk, 13X9
		10" WIDE SEAT CUSHIONS	1	115471	Seat Cushion 1 Thk, 13X10
1	115446	Seat Cushion 1 Thk, 10X8	1	115482	Seat Cushion 1 Thk, 13X11
1	115457	Seat Cushion 1 Thk, 10X9	1	115493	Seat Cushion 1 Thk, 13X12
1	115468	Seat Cushion 1 Thk, 10x10	1	115504	Seat Cushion 1 Thk, 13x13
1	115469	Seat Cushion 1 Thk, 10x11	1	115505	Seat Cushion 1 Thk, 13x14
1	115470	Seat Cushion 1 Thk, 10x12	1	115506	Seat Cushion 1 Thk, 13x15
1	115471	Seat Cushion 1 Thk, 10x13	1	115507	Seat Cushion 1 Thk, 13x16
1	115472	Seat Cushion 1 Thk, 10x14	1	115508	Seat Cushion 1 Thk, 13x17
1	115473	Seat Cushion 1 Thk, 10x15	1	115509	Seat Cushion 1 Thk, 13x18
		11" WIDE SEAT CUSHIONS			14" WIDE SEAT CUSHIONS
1	115447	Seat Cushion 1 Thk, 11X8	1	115450	Seat Cushion 1 Thk, 14X8
1	115458	Seat Cushion 1 Thk, 11X9	1	115461	Seat Cushion 1 Thk, 14X9
1	115469	Seat Cushion 1 Thk, 11X10	1	115472	Seat Cushion 1 Thk, 14X10
1	115480	Seat Cushion 1 Thk, 11x11	1	115483	Seat Cushion 1 Thk, 14X11
1	115481	Seat Cushion 1 Thk, 11x12	1	115494	Seat Cushion 1 Thk, 14X12
1	115482	Seat Cushion 1 Thk, 11x13	1	115505	Seat Cushion 1 Thk, 14X13
1	115483	Seat Cushion 1 Thk, 11x14	1	115516	Seat Cushion 1 Thk, 14x14
			1	115517	Seat Cushion 1 Thk, 14x15

Pos.	Item Number	Description	Pos.	Item Number	Description
1	115518	Seat Cushion 1 Thk, 14x16	1	115507	Seat Cushion 1 Thk, 16X13
1	115519	Seat Cushion 1 Thk, 14x17	1	115518	Seat Cushion 1 Thk, 16X14
1	115520	Seat Cushion 1 Thk, 14x18	1	115529	Seat Cushion 1 Thk, 16X15
		15" WIDE SEAT CUSHIONS	1	115540	Seat Cushion 1 Thk, 16x16
1	115462	Seat Cushion 1 Thk, 15X9	1	115541	Seat Cushion 1 Thk, 16x17
1	115473	Seat Cushion 1 Thk, 15X10	1	115542	Seat Cushion 1 Thk, 16x18
1	115484	Seat Cushion 1 Thk, 15X11			17" WIDE SEAT CUSHIONS
1	115495	Seat Cushion 1 Thk, 15X12	1	115497	Seat Cushion 1 Thk, 17X12
1	115506	Seat Cushion 1 Thk, 15X13	1	115508	Seat Cushion 1 Thk, 17X13
1	115517	Seat Cushion 1 Thk, 15X14	1	115519	Seat Cushion 1 Thk, 17X14
1	115528	Seat Cushion 1 Thk, 15x15	1	115530	Seat Cushion 1 Thk, 17X15
1	115529	Seat Cushion 1 Thk, 15x16	1	115541	Seat Cushion 1 Thk, 17X16
1	115530	Seat Cushion 1 Thk, 15x17			18" WIDE SEAT CUSHIONS
1	115531	Seat Cushion 1 Thk, 15x18	1	115509	Seat Cushion 1 Thk, 18X13
		16" WIDE SEAT CUSHIONS	1	115520	Seat Cushion 1 Thk, 18X14
1	115485	Seat Cushion 1 Thk, 16X11	1	115531	Seat Cushion 1 Thk, 18X15
1	115496	Seat Cushion 1 Thk, 16X12	1	115542	Seat Cushion 1 Thk, 18X16


NOTE: On all Quickie 2 and GP Family of Chairs an inch deeper cushion is utilized on all seat depths.

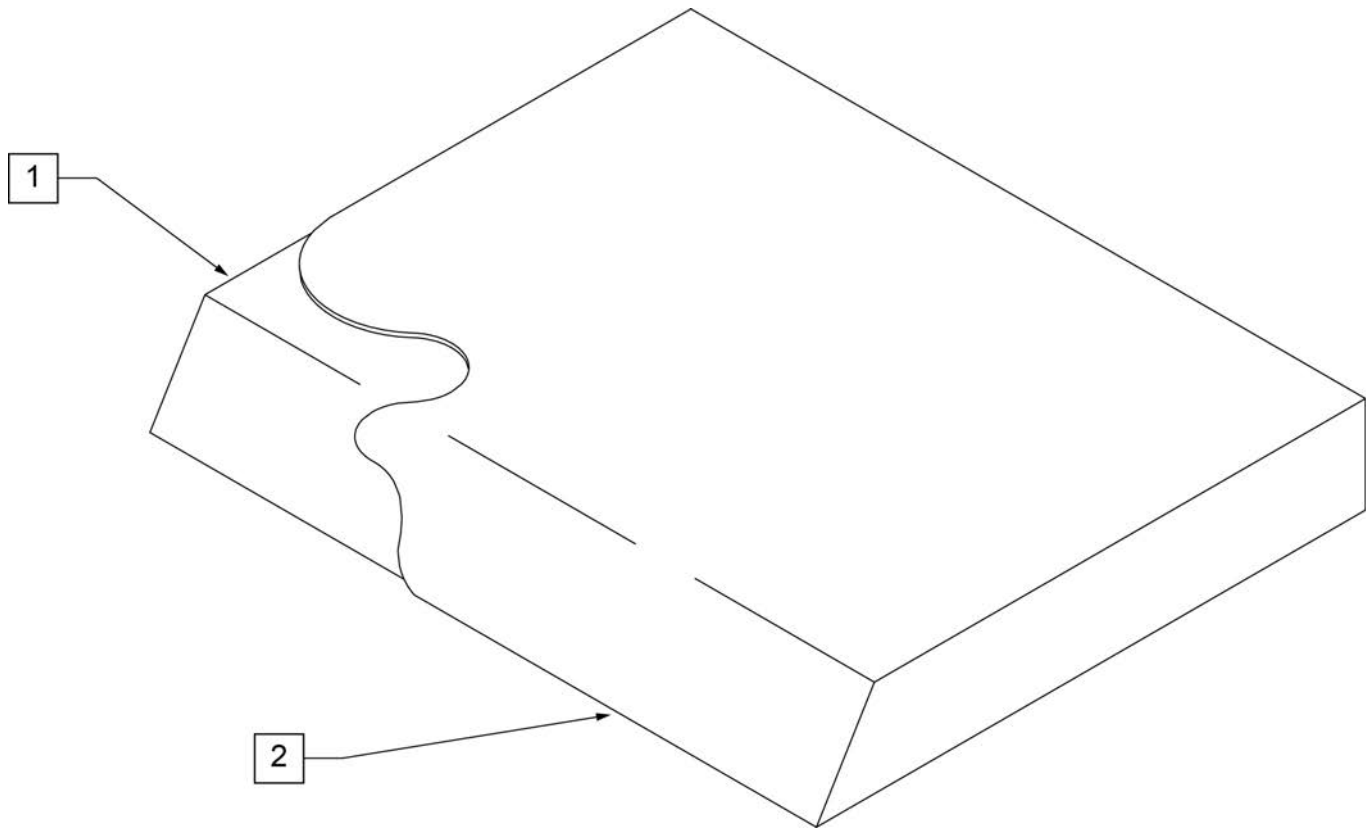

Pos.	Item Number	Description	Pos.	Item Number	Description
>>>>		10" WIDE SEAT CUSHION	1,2	700102	SEAT CUSHION 12" X 10" X 2"
1,2	700025	SEAT CUSHION 10" X 8" X 2"	1,2	700112	SEAT CUSHION 12" X 11" X 2"
1,2	700026	SEAT CUSHION 10" X 9" X 2"	1,2	700122	SEAT CUSHION 12" X 12" X 2"
1,2	700027	SEAT CUSHION 10" X 10" X 2"	1,2	700132	SEAT CUSHION 12" X 13" X 2"
1,2	700028	SEAT CUSHION 10" X 11" X 2"	1,2	700142	SEAT CUSHION 12" X 14" X 2"
1,2	700029	SEAT CUSHION 10" X 12" X 2"	1,2	700152	SEAT CUSHION 12" X 15" X 2"
1,2	700035	SEAT CUSHION 10" X 13" X 2"	1,2	700162	SEAT CUSHION 12" X 16" X 2"
1,2	700045	SEAT CUSHION 10" X 14" X 2"	1,2	700172	SEAT CUSHION 12" X 17" X 2"
>>>>		11" WIDE SEAT CUSHION	1,2	700182	SEAT CUSHION 12" X 18" X 2"
1,2	700055	SEAT CUSHION 11" X 8" X 2"	1,2	700192	SEAT CUSHION 12" X 19" X 2"
1,2	700056	SEAT CUSHION 11" X 9" X 2"	>>>>		13" WIDE SEAT CUSHION
1,2	700002	SEAT CUSHION 11" X 10" X 2"	1,2	700207	SEAT CUSHION 13" X 8" X 2"
1,2	700012	SEAT CUSHION 11" X 11" X 2"	1,2	700208	SEAT CUSHION 13" X 9" X 2"
1,2	700022	SEAT CUSHION 11" X 12" X 2"	1,2	700202	SEAT CUSHION 13" X 10" X 2"
1,2	700032	SEAT CUSHION 11" X 13" X 2"	1,2	700212	SEAT CUSHION 13" X 11" X 2"
1,2	700042	SEAT CUSHION 11" X 14" X 2"	1,2	700222	SEAT CUSHION 13" X 12" X 2"
1,2	700052	SEAT CUSHION 11" X 15" X 2"	1,2	700232	SEAT CUSHION 13" X 13" X 2"
1,2	700062	SEAT CUSHION 11" X 16" X 2"	1,2	700242	SEAT CUSHION 13" X 14" X 2"
1,2	700072	SEAT CUSHION 11" X 17" X 2"	1,2	700252	SEAT CUSHION 13" X 15" X 2"
1,2	700082	SEAT CUSHION 11" X 18" X 2"	1,2	700262	SEAT CUSHION 13" X 16" X 2"
>>>>		12" WIDE SEAT CUSHION	1,2	700272	SEAT CUSHION 13" X 17" X 2"
1,2	700107	SEAT CUSHION 12" X 8" X 2"	1,2	700277	SEAT CUSHION 13" X 18" X 2"
1,2	700108	SEAT CUSHION 12" X 9" X 2"	1,2	700267	SEAT CUSHION 13" X 19" X 2"

Pos.	Item Number	Description	Pos.	Item Number	Description
>>>>		14" WIDE SEAT CUSHION	1,2	700692	SEAT CUSHION 18" X 13" X 2"
1,2	700287	SEAT CUSHION 14" X 8" X 2"	1,2	700702	SEAT CUSHION 18" X 14" X 2"
1,2	700288	SEAT CUSHION 14" X 9" X 2"	1,2	700712	SEAT CUSHION 18" X 15" X 2"
1,2	700282	SEAT CUSHION 14" X 10" X 2"	1,2	700722	SEAT CUSHION 18" X 16" X 2"
1,2	700292	SEAT CUSHION 14" X 11" X 2"	1,2	700732	SEAT CUSHION 18" X 17" X 2"
1,2	700302	SEAT CUSHION 14" X 12" X 2"	1,2	700742	SEAT CUSHION 18" X 18" X 2"
1,2	700312	SEAT CUSHION 14" X 13" X 2"	1,2	700752	SEAT CUSHION 18" X 19" X 2"
1,2	700322	SEAT CUSHION 14" X 14" X 2"	1,2	700762	SEAT CUSHION 18" X 20" X 2"
1,2	700332	SEAT CUSHION 14" X 15" X 2"	1,2	700772	SEAT CUSHION 18" X 21" X 2"
1,2	700342	SEAT CUSHION 14" X 16" X 2"	1,2	702114	SEAT CUSHION 18" X 22" X 2"
1,2	700352	SEAT CUSHION 14" X 17" X 2"	>>>>		19" WIDE SEAT CUSHION
1,2	700362	SEAT CUSHION 14" X 18" X 2"	1,2	700782	SEAT CUSHION 19" X 12" X 2"
1,2	700372	SEAT CUSHION 14" X 19" X 2"	1,2	700792	SEAT CUSHION 19" X 13" X 2"
1,2	700382	SEAT CUSHION 14" X 20" X 2"	1,2	700802	SEAT CUSHION 19" X 14" X 2"
1,2	700392	SEAT CUSHION 14" X 21" X 2"	1,2	700812	SEAT CUSHION 19" X 15" X 2"
1,2	702110	SEAT CUSHION 14" X 22" X 2"	1,2	700822	SEAT CUSHION 19" X 16" X 2"
>>>>		15" WIDE SEAT CUSHION	1,2	700832	SEAT CUSHION 19" X 17" X 2"
1,2	700398	SEAT CUSHION 15" X 11" X 2"	1,2	700842	SEAT CUSHION 19" X 18" X 2"
1,2	700402	SEAT CUSHION 15" X 12" X 2"	1,2	700852	SEAT CUSHION 19" X 19" X 2"
1,2	700412	SEAT CUSHION 15" X 13" X 2"	1,2	700862	SEAT CUSHION 19" X 20" X 2"
1,2	700422	SEAT CUSHION 15" X 14" X 2"	1,2	700872	SEAT CUSHION 19" X 21" X 2"
1,2	700432	SEAT CUSHION 15" X 15" X 2"	1,2	702115	SEAT CUSHION 19" X 22" X 2"
1,2	700442	SEAT CUSHION 15" X 16" X 2"	>>>>		20" WIDE SEAT CUSHION
1,2	700452	SEAT CUSHION 15" X 17" X 2"	1,2	700882	SEAT CUSHION 20" X 12" X 2"
1,2	700462	SEAT CUSHION 15" X 18" X 2"	1,2	700892	SEAT CUSHION 20" X 13" X 2"
1,2	700472	SEAT CUSHION 15" X 19" X 2"	1,2	700902	SEAT CUSHION 20" X 14" X 2"
1,2	700482	SEAT CUSHION 15" X 20" X 2"	1,2	700912	SEAT CUSHION 20" X 15" X 2"
1,2	700492	SEAT CUSHION 15" X 21" X 2"	1,2	700922	SEAT CUSHION 20" X 16" X 2"
1,2	702111	SEAT CUSHION 15" X 22" X 2"	1,2	700932	SEAT CUSHION 20" X 17" X 2"
>>>>		16" WIDE SEAT CUSHION	1,2	700942	SEAT CUSHION 20" X 18" X 2"
1,2	700498	SEAT CUSHION 16" X 11" X 2"	1,2	700952	SEAT CUSHION 20" X 19" X 2"
1,2	700502	SEAT CUSHION 16" X 12" X 2"	1,2	700962	SEAT CUSHION 20" X 20" X 2"
1,2	700512	SEAT CUSHION 16" X 13" X 2"	1,2	700972	SEAT CUSHION 20" X 21" X 2"
1,2	700522	SEAT CUSHION 16" X 14" X 2"	1,2	702116	SEAT CUSHION 20" X 22" X 2"
1,2	700532	SEAT CUSHION 16" X 15" X 2"	>>>>		21" WIDE SEAT CUSHION
1,2	700542	SEAT CUSHION 16" X 16" X 2"	1,2	702080	SEAT CUSHION 21" X 13" X 2"
1,2	700552	SEAT CUSHION 16" X 17" X 2"	1,2	702081	SEAT CUSHION 21" X 14" X 2"
1,2	700562	SEAT CUSHION 16" X 18" X 2"	1,2	702082	SEAT CUSHION 21" X 15" X 2"
1,2	700572	SEAT CUSHION 16" X 19" X 2"	1,2	702083	SEAT CUSHION 21" X 16" X 2"
1,2	700582	SEAT CUSHION 16" X 20" X 2"	1,2	702084	SEAT CUSHION 21" X 17" X 2"
1,2	700592	SEAT CUSHION 16" X 21" X 2"	1,2	702085	SEAT CUSHION 21" X 18" X 2"
1,2	702112	SEAT CUSHION 16" X 22" X 2"	1,2	702086	SEAT CUSHION 21" X 19" X 2"
>>>>		17" WIDE SEAT CUSHION	1,2	702087	SEAT CUSHION 21" X 20" X 2"
1,2	700585	SEAT CUSHION 17" X 11" X 2"	1,2	702088	SEAT CUSHION 21" X 21" X 2"
1,2	700606	SEAT CUSHION 17" X 12" X 2"	1,2	702089	SEAT CUSHION 21" X 22" X 2"
1,2	700597	SEAT CUSHION 17" X 13" X 2"	>>>>		22" WIDE SEAT CUSHION
1,2	700602	SEAT CUSHION 17" X 14" X 2"	1,2	702090	SEAT CUSHION 22" X 13" X 2"
1,2	700612	SEAT CUSHION 17" X 15" X 2"	1,2	702091	SEAT CUSHION 22" X 14" X 2"
1,2	700622	SEAT CUSHION 17" X 16" X 2"	1,2	702092	SEAT CUSHION 22" X 15" X 2"
1,2	700632	SEAT CUSHION 17" X 17" X 2"	1,2	702093	SEAT CUSHION 22" X 16" X 2"
1,2	700642	SEAT CUSHION 17" X 18" X 2"	1,2	702094	SEAT CUSHION 22" X 17" X 2"
1,2	700652	SEAT CUSHION 17" X 19" X 2"	1,2	702095	SEAT CUSHION 22" X 18" X 2"
1,2	700662	SEAT CUSHION 17" X 20" X 2"	1,2	702096	SEAT CUSHION 22" X 19" X 2"
1,2	700672	SEAT CUSHION 17" X 21" X 2"	1,2	702097	SEAT CUSHION 22" X 20" X 2"
1,2	702113	SEAT CUSHION 17" X 22" X 2"	1,2	702098	SEAT CUSHION 22" X 21" X 2"
>>>>		18" WIDE SEAT CUSHION	1,2	702117	SEAT CUSHION 22" X 22" X 2"
1,2	700682	SEAT CUSHION 18" X 12" X 2"	>>>>		24" WIDE SEAT CUSHION


Pos.	Item Number	Description	Pos.	Item Number	Description
1,2	702099	SEAT CUSHION 24" X 14" X 2	1,2	702224	SEAT CUSHION 26" X 22" X 2"
1,2	702100	SEAT CUSHION 24" X 15" X 2	>>>>		28" WIDE SEAT CUSHION
1,2	702101	SEAT CUSHION 24" X 16" X 2	1,2	702225	SEAT CUSHION 28" X 18" X 2"
1,2	702102	SEAT CUSHION 24" X 17" X 2	1,2	702226	SEAT CUSHION 28" X 19" X 2"
1,2	702103	SEAT CUSHION 24" X 18" X 2	1,2	702227	SEAT CUSHION 28" X 20" X 2"
1,2	702104	SEAT CUSHION 24" X 19" X 2	1,2	702228	SEAT CUSHION 28" X 21" X 2"
1,2	702105	SEAT CUSHION 24" X 20" X 2"	1,2	702229	SEAT CUSHION 28" X 22" X 2"
1,2	702106	SEAT CUSHION 24" X 21" X 2"	>>>>		30" WIDE SEAT CUSHION
1,2	702118	SEAT CUSHION 24" X 22" X 2"	1,2	702230	SEAT CUSHION 30" X 18" X 2"
>>>>		26" WIDE SEAT CUSHION	1,2	702231	SEAT CUSHION 30" X 19" X 2"
1,2	702220	SEAT CUSHION 26" X 18" X 2"	1,2	702232	SEAT CUSHION 30" X 20" X 2"
1,2	702221	SEAT CUSHION 26" X 19" X 2"	1,2	702233	SEAT CUSHION 30" X 21" X 2"
1,2	702222	SEAT CUSHION 26" X 20" X 2"	1,2	702234	SEAT CUSHION 30" X 22" X 2"
1,2	702223	SEAT CUSHION 26" X 21" X 2"			

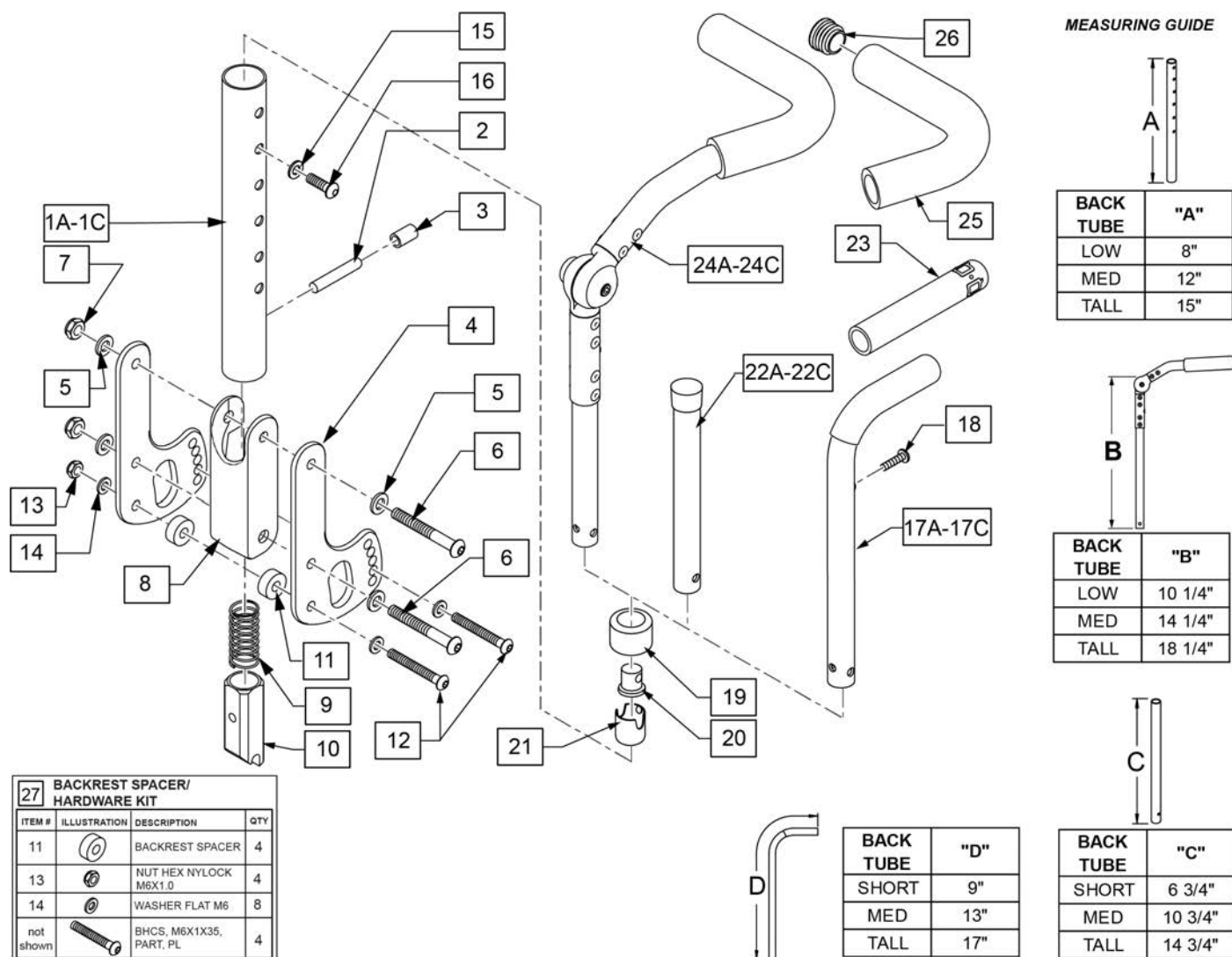

Pos.	Item Number	Description	Remarks
2	481084	BACKREST RELEASE PIN	
3	560502	ANTI-TIP LEVER COVER	
5	106943	WASHER FLAT AN 1/4 ROHS	
6	106965	BHCS 1/4-28 X 2 SHLDR ROHS	
7	106935	NUT NYLOCK THIN 1/4-28 ROHS	
9	321110	SWING AWAY LATCH SPRING	
11	127554	BACKREST SPACER	used when brackets don't line up w/ axle plates
12	102262-008	BHCS M6X1X50 PART PL	
13	104381	NUT HEX NYLOCK M6X1.0 INORG	
14	106943	FLAT AN 1/4 ROHS	
15	104414	WAVE WASHER 6 MM	
16	108107	BHCS 1/4-20 X 3/4 ROHS	

Pos.	Item Number	Description	Remarks
15,16,17A,18-21	136168	PUSH HANDLE BACK KIT SHT	Short; Single side
15,16,17B,18-21	136169	PUSH HANDLE BACK KIT MED	Medium; Single side
15,16,17C,18-21	136170	PUSH HANDLE BACK KIT TLL	Tall; Single side
18	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
19	561018	1" TUBE END BUSHING THICK	
20	483001	Q2 FRAME/FOOT RETAINING PLUG	
21	104187	BUSHING BCKRST NONFLDNG	
15,16,19-21,22A	136165	OMIT PH BACK KIT SHT	Short; Single side
15,16,19-21,22B	136166	OMIT PH BACK KIT MED	Medium; Single side
15,16,19-21,22C	136167	OMIT PH BACK KIT TLL	Tall; Single side
23	920026	QUICKIE GRIP 3/4" W/INSTR SH	Replacement Grip
1A,2-16	136159	BACKREST LOWER KIT L SHORT	Left; Short
1A,2-16	136160	BACKREST LOWER KIT R SHORT	Right; Short
1B,2-16	136161	BACKREST LOWER KIT L STD	Left, Std/Med
1B,2-16	136162	BACKREST LOWER KIT R STD	Right, Std/Med
1C,2-16	136163	BACKREST LOWER KIT L TALL	Left; Tall
1C,2-16	136164	BACKREST LOWER KIT R TALL	Right; Tall
15,16,18-21,24A	136171	HEIGHT ADJ FOLDING BACKREST LOW RT	Right; Low
15,16,18-21,24A	136172	HEIGHT ADJ FOLDING BACKREST LOW LFT	Left; Low
15,16,18-21,24B	136173	HEIGHT ADJ FOLDING BACKREST MED RT	Right; Medium
15,16,18-21,24B	136174	HEIGHT ADJ FOLDING BACKREST MED LFT	Left; Medium
15,16,18-21,24C	136175	HEIGHT ADJ FOLDING BACKREST TALL RT	Right; Tall
15,16,18-21,24C	136176	HEIGHT ADJ FOLDING BACKREST TALL LFT	Left; Tall
25	560638	FOAM GRIP	Replacement Foam Grip-one side
26	560735	7/8 ROUND TUBE PLUG	
27	131262	BACKREST SPACER/HARDWARE KIT	used when brackets don't line up w/ axle plates; includes extra fasteners


Note: Not available with the Dynamic Backrest or Fixed Style backrests. ONLY available with Folding Height Adjustable with integral push handles. REQUIRES integral push handles with a 3/4" tubing diameter.

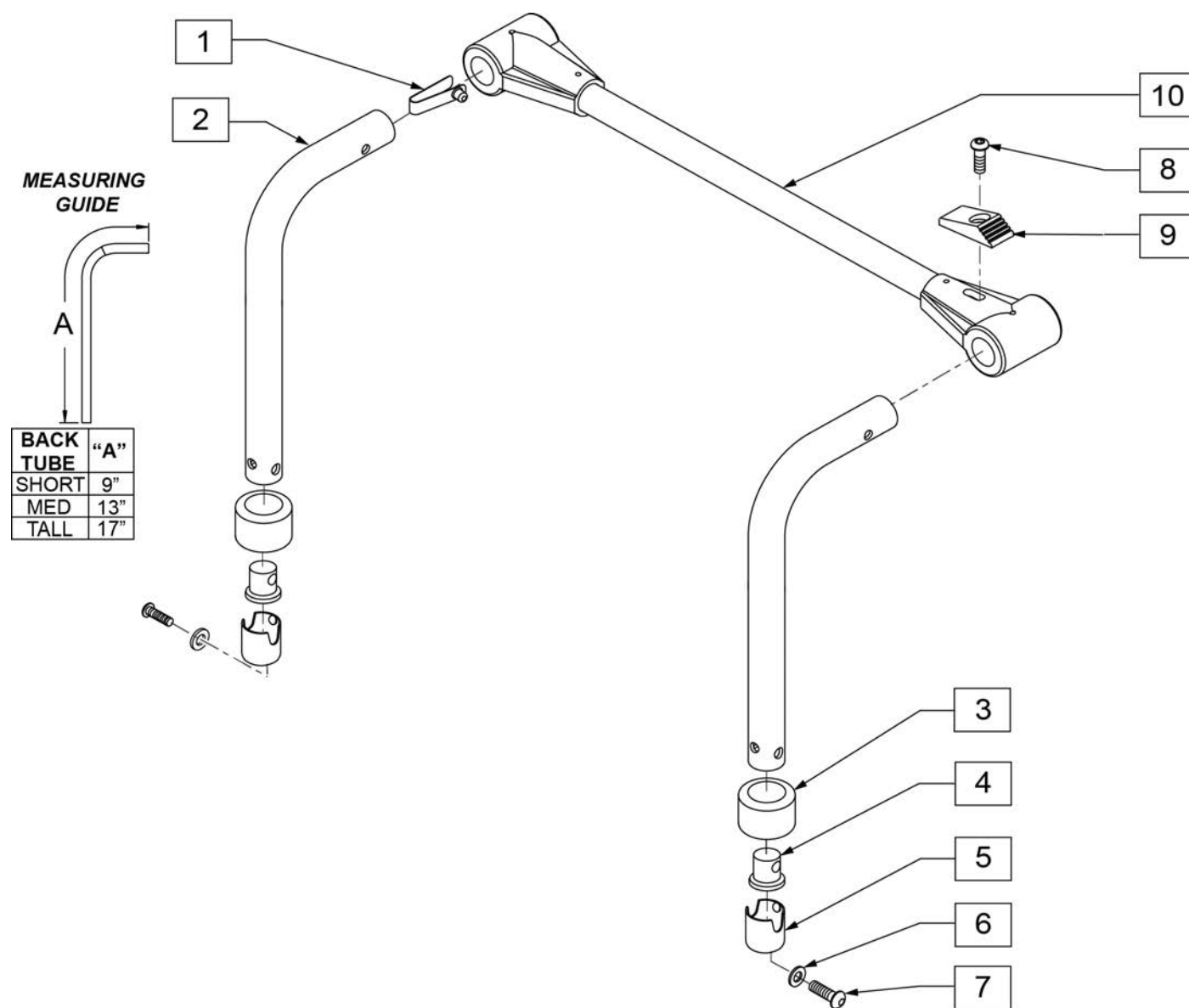

Pos.	Item Number	Description	Pos.	Item Number	Description
1	301100	1/4" SNAP BUTTON SINGLE	8-10	136178	STABILIZER CROSS BAR ASSM 9"
2	129291	STAB PH ASSY SHORT	8-10	136179	STABILIZER CROSS BAR ASSM 10"
2	129292	STAB PH ASSY MED	8-10	820692	STABILIZER CROSS BAR ASSM 11"
2	129293	STAB PH ASSY TALL	8-10	820693	STABILIZER CROSS BAR ASSM 12"
3	561018	1" TUBE END BUSHING THICK	8-10	820694	STABILIZER CROSS BAR ASSM 13"
4	483001	Q2 FRAME/FOOT RETAINING PLUG	8-10	820695	STABILIZER CROSS BAR ASSM 14"
5	104187	BUSHING BCKRST NONFLDNG	8-10	820696	STABILIZER CROSS BAR ASSM 15"
6	104414	WAVE WASHER 6 MM	8-10	820697	STABILIZER CROSS BAR ASSM 16"
7	108107	BHCS 1/4-20 X 3/4 ROHS	8-10	820698	STABILIZER CROSS BAR ASSM 17"
9	560143	STABILIZER LATCH RELEASE BR	8-10	820700	STABILIZER CROSS BAR ASSM 18"
8-10	136177	STABILIZER CROSS BAR ASSM 8"	8-10	820701	STABILIZER CROSS BAR ASSM 19"

Pos.	Item Number	Description
8-10	820702	STABILIZER CROSS BAR ASSM 20"



[12/2014]

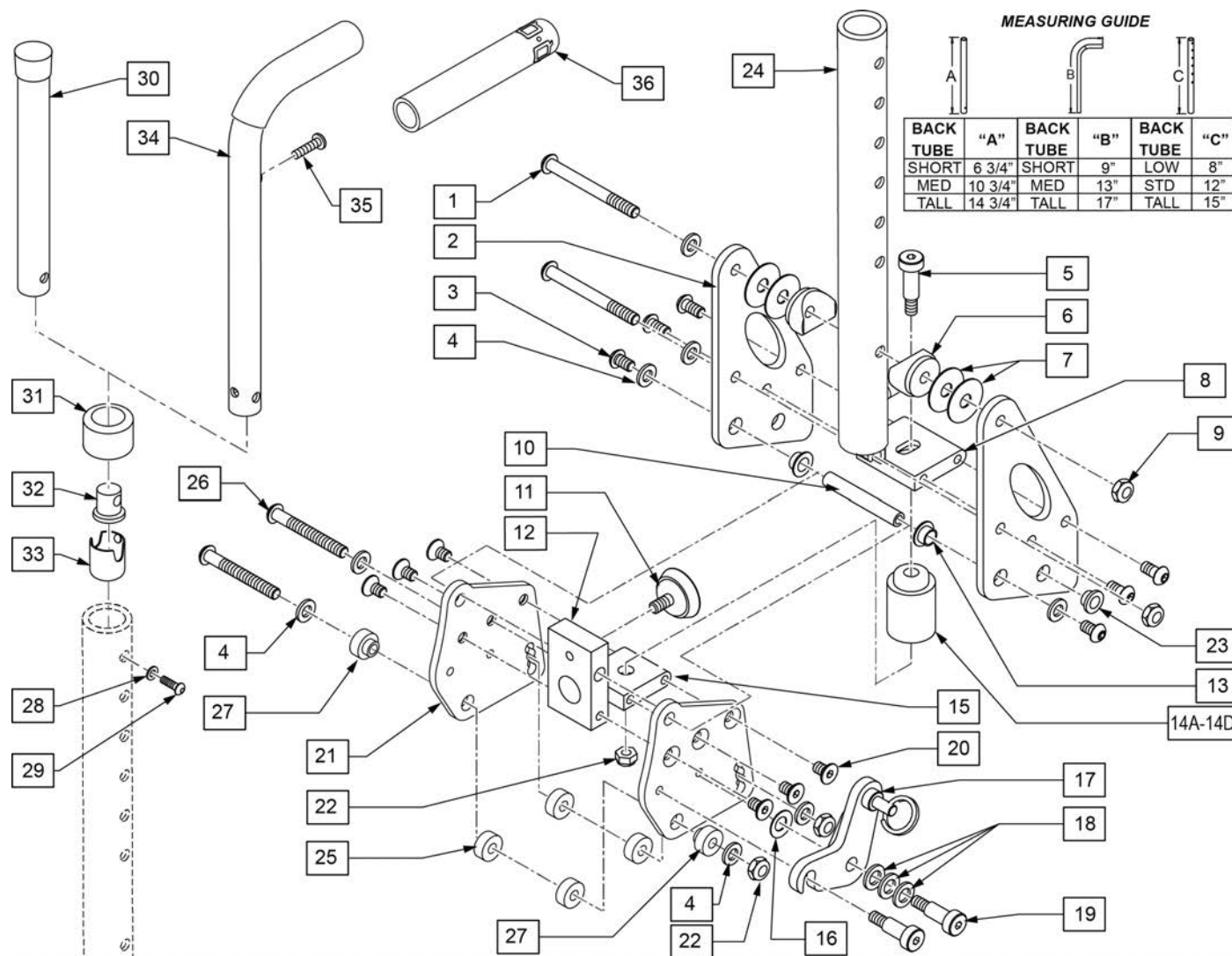

Pos.	Item Number	Description	Remarks
1	106967	BHCS 1/4-28 X 2-1/2 SHLDR ROHS	
3	103185-005	BHCS M6 X 1 X 12 FULL PL	
4	106943	WASHER FLAT AN 1/4 ROHS	
5	011649-003	SHLDR SCREW M6 X 8MM OD X 45MM	
6	561087	BACK REL SADDLE RIGID	
7	113268	WASHER NYLON 1IN OD X .312 ID	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	119410	PIVOT DYNAMIC BACKREST	
11	349560	ZP RUBBER STOP	
13	119418	IGLIDE BUSHING .22 DEEP	
14A	124044	SOFT RED SUSPENSION KIT	
14B	124060	MED BLUE SUSPENSION KIT	
14C	124064	FIRM GREEN SUSPENSION KIT	
14D	124082	X-FIRM BLACK SUSPENSION KIT	
16	262134	5/16 WASHER SPRING DISC	
18	106947	WASHER FLAT THIN AN 5/16 ROHS	
19	011649-001	SHLDR SCREW 8MM OD X 16MM LG	
20	013450	FHCS M6 X 1 X 10MM W/PCH	

Pos.	Item Number	Description	Remarks
22	104381	NUT HEX NYLOCK M6X1.0 INORG	
16-19	120943	DB LATCH REPLACEMENT KIT	Left & Right
1-24	129098	DYBCK ASSM ZP LOW HGT ADJ A	Right-no mntg hardware
1-24	129099	DYBCK ASSM ZP LOW HGT ADJ B	Left-no mntg hardware
1-24	129100	DYBCK ASSM ZP STD HGT ADJ A	Right-no mntg hardware
1-24	129101	DYBCK ASSM ZP STD HGT ADJ B	Left-no mntg hardware
1-24	129102	DYBCK ASSM ZP TALL HGT ADJ A	Right-no mntg hardware
1-24	129103	DYBCK ASSM ZP TALL HGT ADJ B	Left-no mntg hardware
1-27	136181	DYNAMIC BACK KIT SHT XCP	Pair-w/ mntg hardware
1-27	136182	DYNAMIC BACK KIT MED XCP	Pair-w/ mntg hardware
1-27	136183	DYNAMIC BACK KIT MTALL XCP	Pair-w/ mntg hardware
25	127554	BACKREST SPACER	
26	102262-008	BHCS M6X1X50 PART PL	
27	129104	DYNAMIC BACK SPACER	
28	104414	WAVE WASHER 6 MM	
29	108107	BHCS 1/4-20 X 3/4 ROHS	
30	104236-001	TUBE BACKREST STRAIGHT SHT	
30	104236-002	TUBE BACKREST STRAIGHT MED	
30	104236-003	TUBE BACKREST STRAIGHT STD. 17-20"	
31	561018	1" TUBE END BUSHING THICK	
32	483001	Q2 FRAME/FOOT RETAINING PLUG	
33	104187	BUSHING BCKRST NONFLDNG	
34	104093-001	NON FOLDING PH ASSY SHORT	
34	104093-002	NON FOLDING PH ASSY MED	
34	104093-003	NON FOLDING PH ASSY TALL	
35	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
36	920026	QUICKIE GRIP 3/4" W/INSTR SH	
37	234900	SHCS,M6X1X60,PART,NP	


MEASURING GUIDE

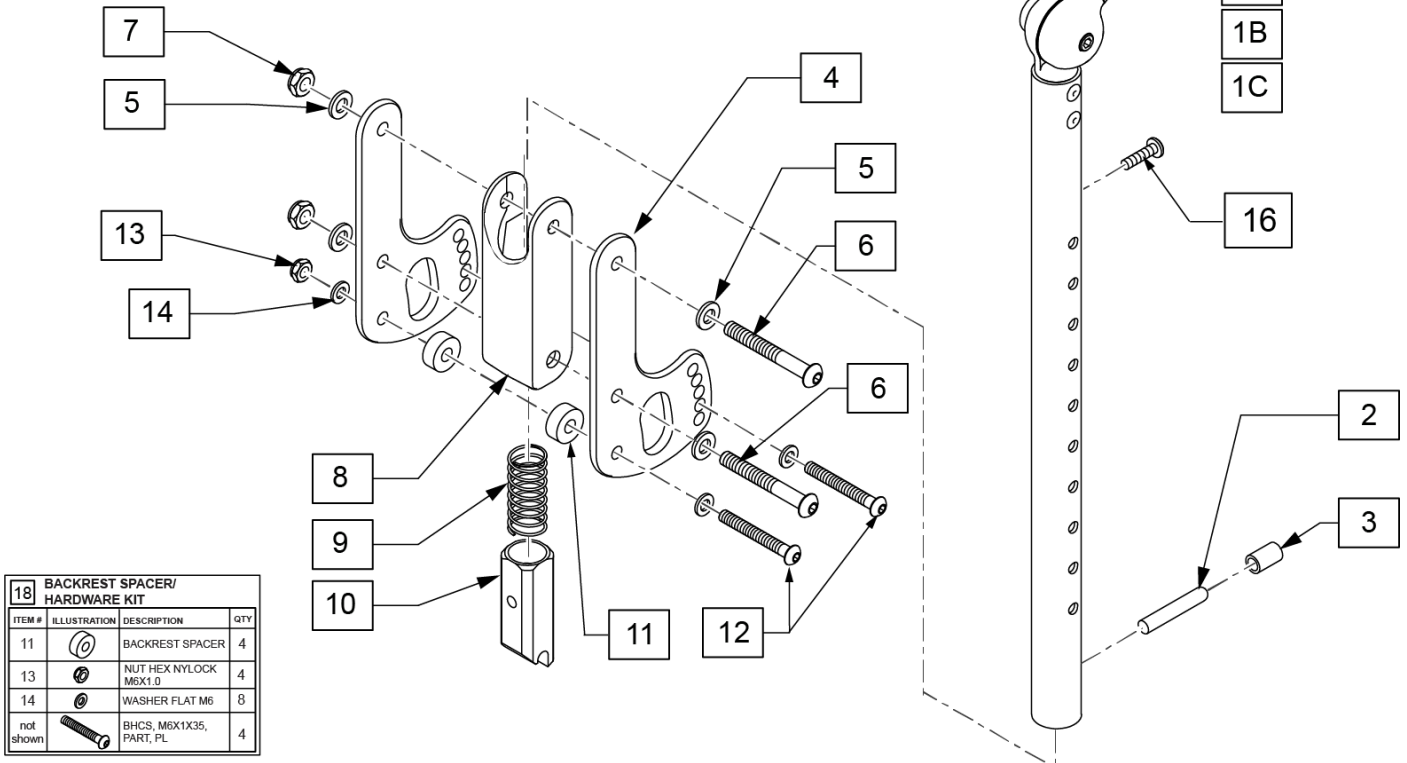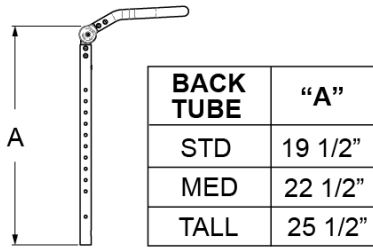

Pos.	Item Number	Description	Remarks
2	481084	BACKREST RELEASE PIN	
3	560502	ANTI-TIP LEVER COVER	
5	106943	WASHER FLAT AN 1/4 ROHS	
6	106965	BHCS 1/4-28 X 2 SHLDR ROHS	
7	106935	NUT NYLOCK THIN 1/4-28 ROHS	
9	321110	SWING AWAY LATCH SPRING	
11	127554	BACKREST SPACER	used when brackets don't line up w/ axle plates
12	102262-008	BHCS M6X1X50 PART PL	
13	104381	NUT HEX NYLOCK M6X1.0 INORG	
14	106943	FLAT AN 1/4 ROHS	
15	560638	FOAM GRIP	Replacement Faom Grip-one side
16	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
17	560735	.875 ROUND TUBE PLUG	
18	131262	BACKREST SPACER/HARDWARE KIT	used when brackets don't line up w/ axle plates includes extra fasteners
1A-17	158626	FIXED HT BACK W/FLD PH STD R	Right
1A-17	158627	FIXED HT BACK W/FLD PH STD L	Left
1B-17	158628	FIXED HT BACK W/FLD PH MED R	Right

Pos.	Item Number	Description	Remarks
1B-17	158629	FIXED HT BACK W/FLD PH MED L	Left
1C-17	158630	FIXED HT BACK W/FLD PH TALL R	Right
1C-17	158631	FIXED HT BACK W/FLD PH TALL L	Left


MEASURING GUIDE

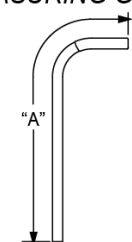

BACK TUBE	"A"
STD	25 3/4"
MED	28 3/4"
TALL	31 3/4"

NOTE: ABOVE DIMENSIONS ARE FOR BACK TUBE ONLY AND DO NOT INCLUDE FOAM GRIP.

18 BACKREST SPACER/HARDWARE KIT			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
11		BACKREST SPACER	4
13		NUT HEX NYLOCK M6X1.0	4
14		WASHER FLAT M6	8
not shown		BHCS, M6X1X35, PART, PL	4

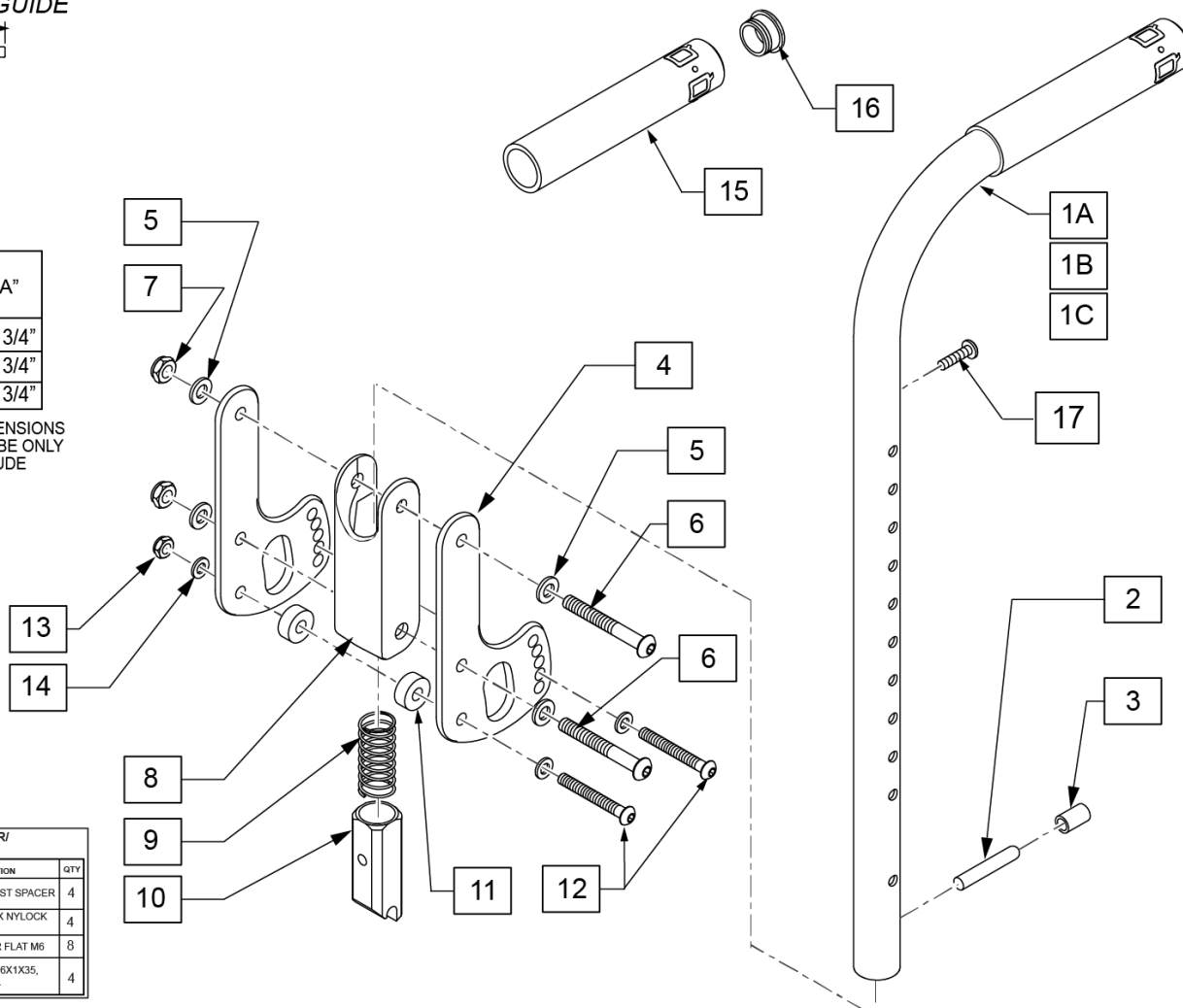


Pos.	Item Number	Description	Remarks
1A	842067	EXT STROLLER HANDLE STD	Std (AKA Short); includes grip
1B	842066	EXT STROLLER HANDLE MED	Med; includes grip
1C	842068	EXT STROLLER HANDLE TALL	Tall; includes grip
2	481084	BACKREST RELEASE PIN	
3	560502	ANTI-TIP LEVER COVER	
5	106943	WASHER FLAT AN 1/4 ROHS	
6	106965	BHCS 1/4-28 X 2 SHLDR ROHS	
7	106935	NUT NYLOCK THIN 1/4-28 ROHS	
9	321110	SWING AWAY LATCH SPRING	
10	483019	BACK RELEASE LATCH	
11	127554	BACKREST SPACER	used when brackets don't line up w/ axle plates
12	102262-008	BHCS M6X1X50 PART PL	
13	104381	NUT HEX NYLOCK M6X1.0 INORG	
14	106943	FLAT AN 1/4 ROHS	
15	920023	QUICKIE GRIP 1" W/HOLE W/INSTR	
16	561002	1" INSERT PLUG .083	
17	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
18	131262	BACKREST SPACER/HARDWARE KIT	used when brackets don't line up w/ axle plates; includes extra fasteners
1A,2-16	158620	FIXED HT BACK STD R	Right; Std (AKA Short)

Pos.	Item Number	Description	Remarks
1A,2-16	158621	FIXED HT BACK STD L	Left; Std (AKA Short)
1B,2-16	158622	FIXED HT BACK MED R	Right; Med
1B,2-16	158623	FIXED HT BACK MED L	Left; Med
1C,2-16	158624	FIXED HT BACK TALL R	Right; Tall
1C,2-16	158625	FIXED HT BACK TALL L	Left; Tall

Note: For chairs equipped with Inset Backrest, upholstery must be 2" narrower than listed width

Note: 17" & 18" Chair Widths not available with 6-8" Back Heights.

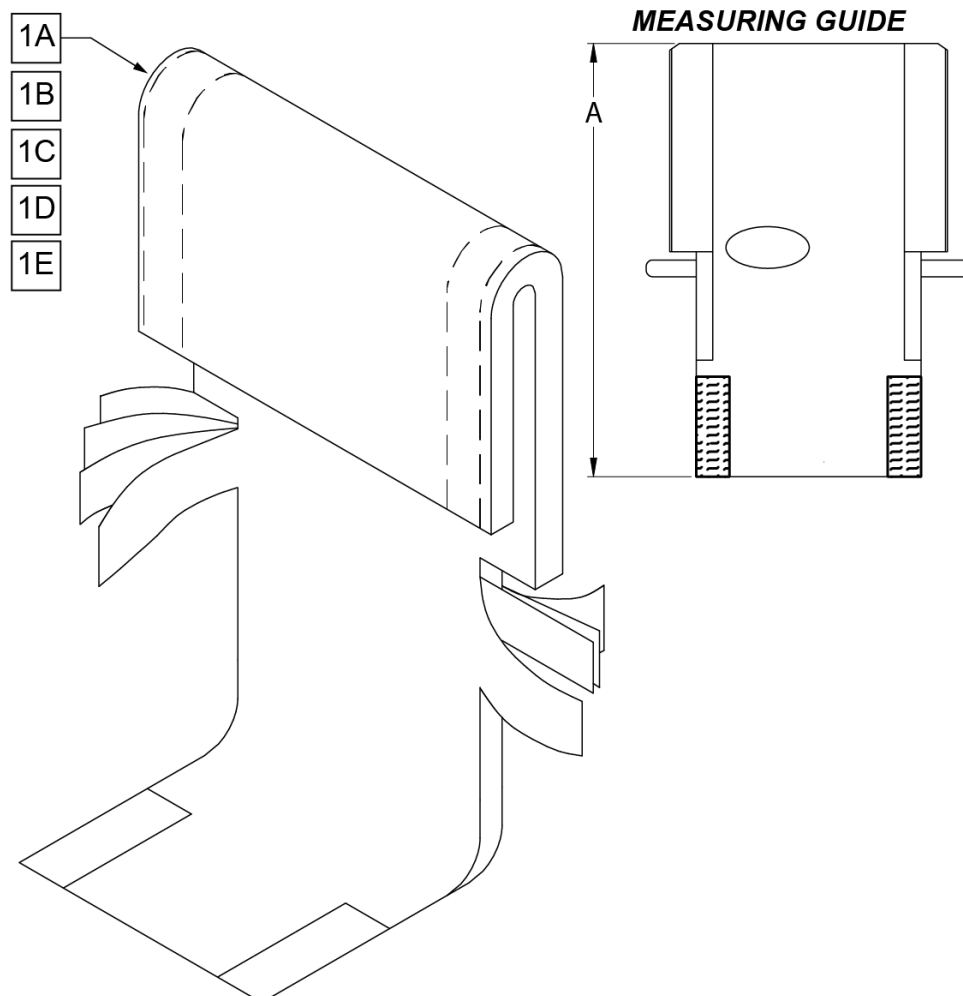


PART #	DESCRIPTION	DIM "A"
116012	BCK UPH 8W, 6-8T	13"
116013	BCK UPH 9W, 6-8T	13"
116014	BCK UPH 10W, 6-8T	13"
116015	BCK UPH 11W, 6-8T	13"
116016	BCK UPH 12W, 6-8T	13"
116017	BCK UPH 13W, 6-8T	13"
116018	BCK UPH 14W, 6-8T	13"
116019	BCK UPH 15W, 6-8T	13"
116020	BCK UPH 16W, 6-8T	13"
115572	BACK UPH 8" 9-14" W/TAIL	26"
115575	BACK UPH 8" 13-18" W/TAIL	30"
115577	BACK UPH 9" 9-14" W/TAIL	26"
115580	BACK UPH 9" 13-18" W/TAIL	30"
115582	BACK UPH 10" 9-14" W/TAIL	26"
115585	BACK UPH 10" 13-18" W/TAIL	30"
115587	BACK UPH 11" 9-14" W/TAIL	26"
115590	BACK UPH 11" 13-18" W/TAIL	30"
115592	BACK UPH 12" 9-14" W/TAIL	26"
115595	BACK UPH 12" 13-18" W/TAIL	30"
115597	BACK UPH 13" 9-14" W/TAIL	26"
115600	BACK UPH 13" 13-18" W/TAIL	30"
115602	BACK UPH 14" 9-14" W/TAIL	26"
115605	BACK UPH 14" 13-18" W/TAIL	30"
115607	BACK UPH 15" 9-14" W/TAIL	26"
115610	BACK UPH 15" 13-18" W/TAIL	30"
115612	BACK UPH 16" 9-14" W/TAIL	26"
115615	BACK UPH 16" 13-18" W/TAIL	30"
680601	BACK UPH 17" 11-15" W/TAIL	28"
680701	BACK UPH 18" 11-15" W/TAIL	28"
680605	BACK UPH 17" 14-18" W/TAIL	31"
680705	BACK UPH 18" 14-18" W/TAIL	31"

Pos.	Item Number	Description	Remarks
1A	116012	BCK UPH 8W, 6-8T W/TAIL	(8" WIDE) 6-8" Tall for Ht Adj Back
1A	116013	BCK UPH 9W, 6-8T W/TAIL	(9" WIDE) 6-8" Tall for Ht Adj Back
1A	116014	BCK UPH 10W, 6-8T W/TAIL	(10" WIDE) 6-8" Tall for Ht Adj Back
1A	116015	BCK UPH 11W, 6-8T W/TAIL	(11" WIDE) 6-8" Tall for Ht Adj Back
1A	116016	BCK UPH 12W, 6-8T W/TAIL	(12" WIDE) 6-8" Tall for Ht Adj Back
1A	116017	BCK UPH 13W, 6-8T W/TAIL	(13" WIDE) 6-8" Tall for Ht Adj Back
1A	116018	BCK UPH 14W, 6-8T W/TAIL	(14" WIDE) 6-8" Tall for Ht Adj Back
1A	116019	BCK UPH 15W, 6-8T W/TAIL	(15" WIDE) 6-8" Tall for Ht Adj Back
1A	116020	BCK UPH 16W, 6-8T W/TAIL	(16" WIDE) 6-8" Tall for Ht Adj Back
1B	115572	BACK UPH 8" 9-14" W/TAIL	(8" WIDE) 9-14" Tall for Ht Adj Back


Pos.	Item Number	Description	Remarks
1B	115577	BACK UPH 9" 9-14" W/TAIL	(9" WIDE) 9-14" Tall for Ht Adj Back
1B	115582	BACK UPH 10" 9-14" W/TAIL	(10" WIDE) 9-14" Tall for Ht Adj Back
1B	115587	BACK UPH 11" 9-14" W/TAIL	(11" WIDE) 9-14" Tall for Ht Adj Back
1B	115592	BACK UPH 12" 9-14" W/TAIL	(12" WIDE) 9-14" Tall for Ht Adj Back
1B	115597	BACK UPH 13" 9-14" W/TAIL	(13" WIDE) 9-14" Tall for Ht Adj Back
1B	115602	BACK UPH 14" 9-14" W/TAIL	(14" WIDE) 9-14" Tall for Ht Adj Back
1B	115607	BACK UPH 15" 9-14" W/TAIL	(15" WIDE) 9-14" Tall for Ht Adj Back
1B	115612	BACK UPH 16" 9-14" W/TAIL	(16" WIDE) 9-14" Tall for Ht Adj Back
1B	680601	BACK UPH 17" 11-15" W/TAIL	(17" WIDE) 11-15" Tall for Ht Adj Back
1B	680701	BACK UPH 18" 11-15" W/TAIL	(18" WIDE) 11-15" Tall for Ht Adj Back
1C	115575	BACK UPH 8" 13-18" W/TAIL	(8" WIDE) 13-18" Tall for Ht Adj Back
1C	115580	BACK UPH 9" 13-18" W/TAIL	(9" WIDE) 13-18" Tall for Ht Adj Back
1C	115585	BACK UPH 10" 13-18" W/TAIL	(10" WIDE) 13-18" Tall for Ht Adj Back
1C	115590	BACK UPH 11" 13-18" W/TAIL	(11" WIDE) 13-18" Tall for Ht Adj Back
1C	115595	BACK UPH 12" 13-18" W/TAIL	(12" WIDE) 13-18" Tall for Ht Adj Back
1C	115600	BACK UPH 13" 13-18" W/TAIL	(13" WIDE) 13-18" Tall for Ht Adj Back
1C	115605	BACK UPH 14" 13-18" W/TAIL	(14" WIDE) 13-18" Tall for Ht Adj Back
1C	115610	BACK UPH 15" 13-18" W/TAIL	(15" WIDE) 13-18" Tall for Ht Adj Back
1C	115615	BACK UPH 16" 13-18" W/TAIL	(16" WIDE) 13-18" Tall for Ht Adj Back
1C	680605	BACK UPH 17" 14-18" W/TAIL	(17" WIDE) 14-18" Tall for Ht Adj Back
1C	680705	BACK UPH 18" 14-18" W/TAIL	(18" WIDE) 14-18" Tall for Ht Adj Back

Note: For Inset Backrest Upholstery must be 2" narrower than listed width

Note: 17" & 18" wide chairs are not available with 9-13" Back Heights.

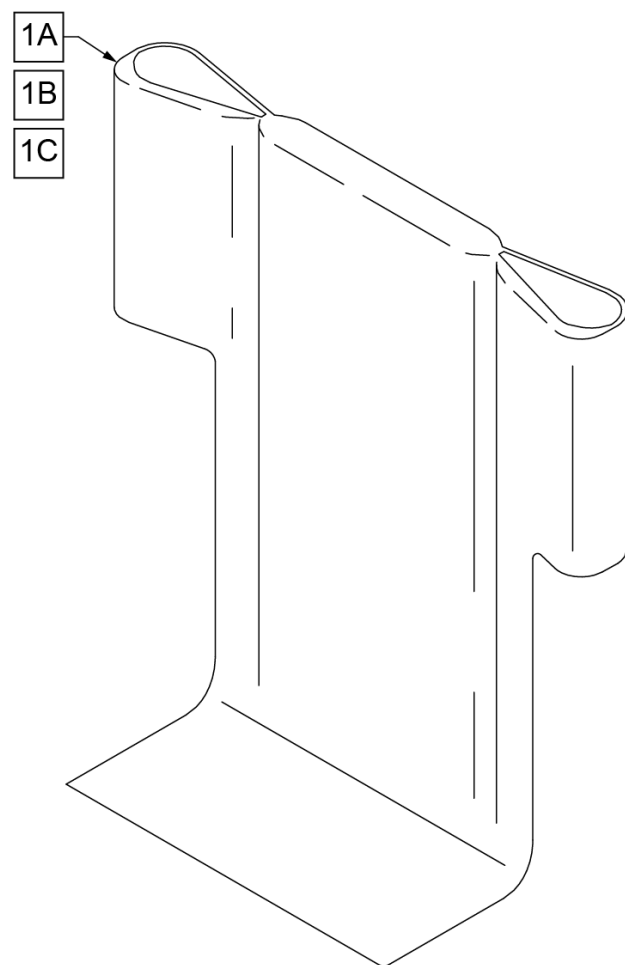


MEASURING GUIDE

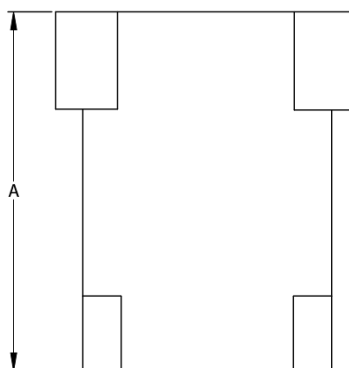

PART #	DESCRIPTION	DIM "A"
115616	BCK UPH 8", 9-13	16"
115617	BCK UPH 8", 11-14/12-16	19"
115618	BACK UPH 8" WID, 14-18	21"
115619	BCK UPH 9", 9-13	16"
115620	BCK UPH 9", 11-14/12-16	19"
115621	BACK UPH 9" WID, 14-18	21"
115622	BCK UPH 10", 9-13	16"
115623	BCK UPH 10", 11-14/12-16	19"
115624	BACK UPH 10" WID, 14-18	21"
115625	BCK UPH 11", 9-13	16"
115626	BCK UPH 11", 11-14/12-16	19"
115627	BACK UPH 11" WID, 14-18	21"
115628	BCK UPH 12", 9-13	16"
115629	BCK UPH 12", 11-14/12-16	19"
115630	BACK UPH 12" WID, 14-18	21"
115631	BCK UPH 13", 9-13	16"
115632	BCK UPH 13", 11-14/12-16	19"
115633	BACK UPH 13" WID, 14-18	21"
115634	BCK UPH 14", 9-13	16"
115635	BCK UPH 14", 11-14/12-16	19"
115636	BACK UPH 14" WID, 14-18	21"
115637	BCK UPH 15", 9-13	16"
115638	BCK UPH 15", 11-14/12-16	19"
115639	BACK UPH 15" WID, 14-18	21"
115640	BCK UPH 16", 9-13	16"
115641	BCK UPH 16", 11-14/12-16	19"
115642	BACK UPH 16" WID, 14-18	21"
680457	BACK UPH 17", 11-14/12-16	19"
680458	BACK UPH 18", 11-14/12-16	19"
680467	BACK UPH 17" WID, 14-18	21"
680468	BACK UPH 18" WID, 14-18	21"

Pos.	Item Number	Description	Remarks
1A	115616	BCK UPH 8", 9-13	8" Wide 9-13" BACK HEIGHT
1A	115619	BCK UPH 9", 9-13	9" Wide 9-13" BACK HEIGHT
1A	115622	BCK UPH 10", 9-13	10" Wide 9-13" BACK HEIGHT
1A	115625	BCK UPH 11", 9-13	11" Wide 9-13" BACK HEIGHT
1A	115628	BCK UPH 12", 9-13	12" Wide 9-13" BACK HEIGHT
1A	115631	BCK UPH 13", 9-13	13" Wide 9-13" BACK HEIGHT
1A	115634	BCK UPH 14", 9-13	14" Wide 9-13" BACK HEIGHT
1A	115637	BCK UPH 15", 9-13	15" Wide 9-13" BACK HEIGHT
1A	115640	BCK UPH 16", 9-13	16" Wide 9-13" BACK HEIGHT
1B	115617	BCK UPH 8", 11-14/12-16	8" Wide 11 -14" & 12 -16" BACK HEIGHT
1B	115620	BCK UPH 9", 11-14/12-16	9" Wide 11 -14" & 12 -16" BACK HEIGHT
1B	115623	BCK UPH 10", 11-14/12-16	10" Wide 11 -14" & 12 -16" BACK HEIGHT
1B	115626	BCK UPH 11", 11-14/12-16	11" Wide 11 -14" & 12 -16" BACK HEIGHT
1B	115629	BCK UPH 12", 11-14/12-16	12" Wide 11 -14" & 12 -16" BACK HEIGHT

Pos.	Item Number	Description	Remarks
1B	115632	BCK UPH 13", 11-14/12-16	13" Wide 11 -14" & 12 -16" BACK HEIGHT
1B	115635	BCK UPH 14", 11-14/12-16	14" Wide 11 -14" & 12 -16" BACK HEIGHT
1B	115638	BCK UPH 15", 11-14/12-16	15" Wide 11 -14" & 12 -16" BACK HEIGHT
1B	115641	BCK UPH 16", 11-14/12-16	16" Wide 11 -14" & 12 -16" BACK HEIGHT
1B	680457	BACK UPH 17" 11-14/12-16 RDG	17" Wide 11-14" & 12-16" BACK HEIGHT
1B	680458	BACK UPH 18" 11-14/12-16 RDG	18" Wide 11-14" & 12-16" BACK HEIGHT
1C	115618	BACK UPH 8" WID, 14-18	8" Wide 14-18" BACK HEIGHT
1C	115621	BACK UPH 9" WID, 14-18	9" Wide 14-18" BACK HEIGHT
1C	115624	BACK UPH 10" WID, 14-18	10" Wide 14-18" BACK HEIGHT
1C	115627	BACK UPH 11" WID, 14-18	11" Wide 14-18" BACK HEIGHT
1C	115630	BACK UPH 12" WID, 14-18	12" Wide 14-18" BACK HEIGHT
1C	115633	BACK UPH 13" WID, 14-18	13" Wide 14-18" BACK HEIGHT
1C	115636	BACK UPH 14" WID, 14-18	14" Wide 14-18" BACK HEIGHT
1C	115639	BACK UPH 15" WID, 14-18	15" Wide 14-18" BACK HEIGHT
1C	115642	BACK UPH 16" WID, 14-18	16" Wide 14-18" BACK HEIGHT
1C	680467	BACK UPH 17" 14-18 RDG	17" Wide 14-18" BACK HEIGHT
1C	680468	BACK UPH 18" 14-18 RDG	18" Wide 14-18" BACK HEIGHT

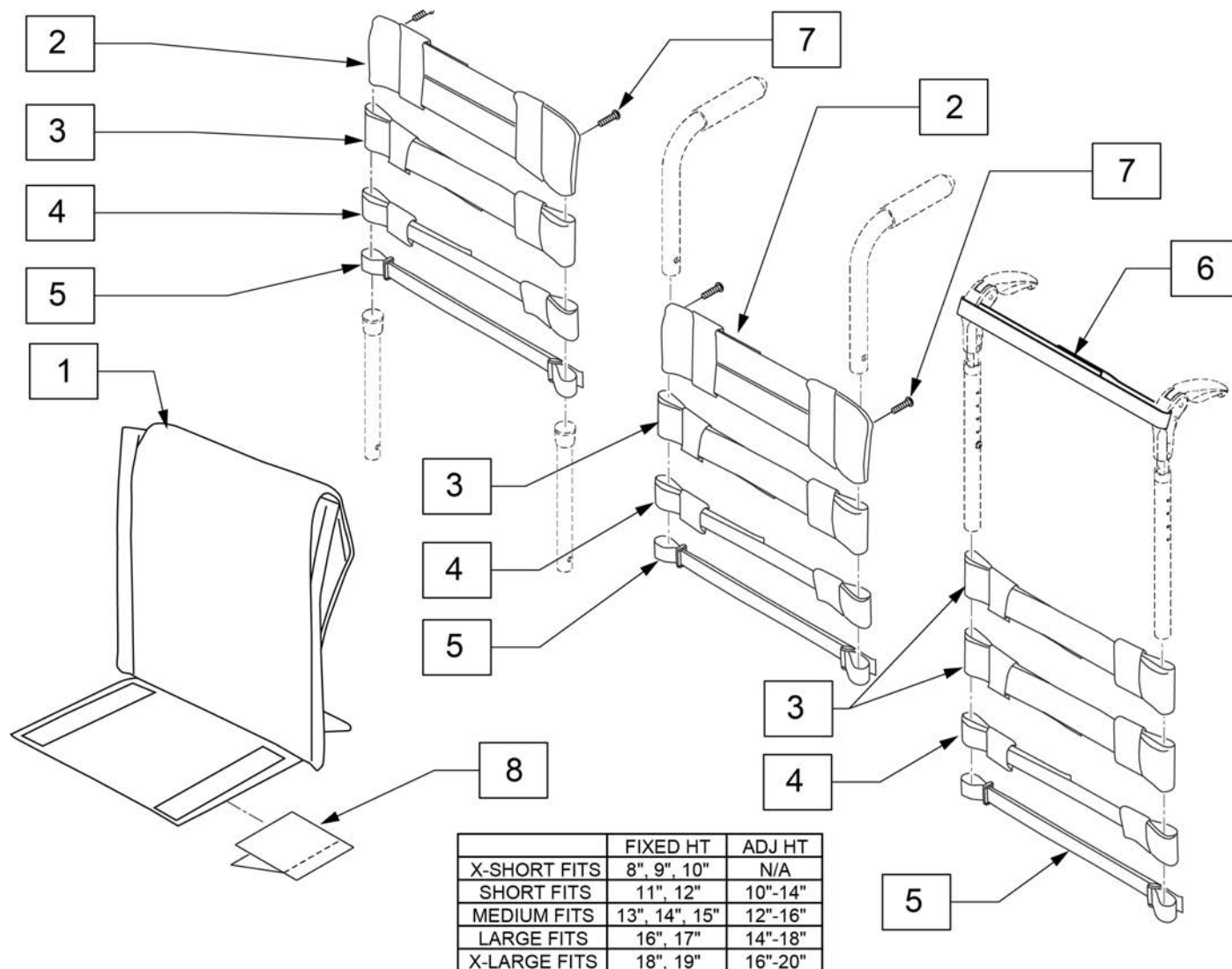

Pos.	Item Number	Description	Remarks
2	115285	TOP CAP, STD, 9-10	Used for omit push handle & Integral Push Handle only
2	113712	TOP CAP, STD, 11-12	Used for omit push handle & Integral Push Handle only
2	113713	TOP CAP, STD, 13-14	Used for omit push handle & Integral Push Handle only
2	113714	TOP CAP, STD, 15-16	Used for omit push handle & Integral Push Handle only
2	113715	TOP CAP, STD, 17-18	Used for omit push handle & Integral Push Handle only
3	115287	STRAP PADDED 2IN, 9-10	
3	113727	STRAP PADDED 2IN, 11-12	
3	113728	STRAP PADDED 2IN, 13-14	
3	113729	STRAP PADDED 2IN, 15-16	
3	113730	STRAP PADDED 2IN, 17-18	
4	115288	STRAP PADDED 1IN, 9-10	
4	113732	STRAP PADDED 1IN, 11-12	
4	113733	STRAP PADDED 1IN, 13-14	
4	113734	STRAP PADDED 1IN, 15-16	
4	113735	STRAP PADDED 1IN, 17-18	
4	113736	STRAP PADDED 1IN, 19-20	
5	115775	POSITION STRAP 1IN, 9-10	
5	113717	POSITION STRAP 1IN, 11-12	

Pos.	Item Number	Description	Remarks
5	113718	POSITION STRAP 1IN, 13-14	
5	113719	POSITION STRAP 1IN, 15-16	
5	113720	POSITION STRAP 1IN, 17-18	
6	052435-006	UPH VNTD ADJ FPH STRP 10-11"W	Fold Down Push Handles only
6	052435-001	UPH VNTD ADJ FPH STRP 12-13W	Fold Down Push Handles only
6	052435-002	UPH VNTD ADJ FPH STRP 14-15W	Fold Down Push Handles only
6	052435-003	UPH VNTD ADJ FPH STRP 16-17W	Fold Down Push Handles only
6	052435-004	UPH,VNTD ADJ.,FPH STRP,18-20"W	Fold Down Push Handles only
7	107384	PAN HEAD PHIL 10-32 X 5/8 ROHS	Used for omit push handle & Integral Push Handle only
8	116520	HOOK AND LOOP INSET BACK	Used for Inset Back only.
2-7	116852	ADJ UPH 9-10W STRAP KIT	
2-7	114879	ADJ UPH 11-12W STRAP KIT	
2-7	114880	ADJ UPH 13-14W STRAP KIT	
2-7	114881	ADJ UPH 15-16W STRAP KIT	
2-7	114882	ADJ UPH 17-18W STRAP KIT	
1-7	116853	ZIP ADJ UPH 10W X-SHT ASM	
1-7	116854	ZIP ADJ UPH 10W SHT ASM	
1-7	116855	ZIP ADJ UPH 10W MED ASM	
1-7	116856	ZIP ADJ UPH 10W TAL ASM	
1-7	116857	ZIP ADJ UPH 11W X-SHT ASM	
1-7	116858	ZIP ADJ UPH 11W SHT ASM	
1-7	116859	ZIP ADJ UPH 11W MED ASM	
1-7	116860	ZIP ADJ UPH 11W TAL ASM	
1-7	116861	ZIP ADJ UPH 12W X-SHT ASM	
1-7	116862	ZIP ADJ UPH 12W SHT ASM	
1-7	116863	ZIP ADJ UPH 12W MED ASM	
1-7	116864	ZIP ADJ UPH 12W TAL ASM	
1-7	116865	ZIP ADJ UPH 13W X-SHT ASM	
1-7	116866	ZIP ADJ UPH 13W SHT ASM	
1-7	116867	ZIP ADJ UPH 13W MED ASM	
1-7	116868	ZIP ADJ UPH 13W TAL ASM	
1-7	116869	ZIP ADJ UPH 14W X-SHT ASM	
1-7	116870	ZIP ADJ UPH 14W SHT ASM	
1-7	116871	ZIP ADJ UPH 14W MED ASM	
1-7	116872	ZIP ADJ UPH 14W TAL ASM	
1-7	116873	ZIP ADJ UPH 15W X-SHT ASM	
1-7	116874	ZIP ADJ UPH 15W SHT ASM	
1-7	116875	ZIP ADJ UPH 15W MED ASM	
1-7	116876	ZIP ADJ UPH 15W TAL ASM	
1-7	116877	ZIP ADJ UPH 16W X-SHT ASM	
1-7	116878	ZIP ADJ UPH 16W SHT ASM	
1-7	116879	ZIP ADJ UPH 16W MED ASM	
1-7	116880	ZIP ADJ UPH 16W TAL ASM	
1-7	113321-026	ADJ UPH 17W X-SHT ASM	
1-7	113321-027	ADJ UPH 17W SHT ASM	
1-7	113321-028	ADJ UPH 17W MED ASM	
1-7	113321-029	ADJ UPH 17W TALL ASM	
1-7	113321-031	ADJ UPH 18W X-SHT ASM	
1-7	113321-032	ADJ UPH 18W SHT ASM	
1-7	113321-033	ADJ UPH 18W MED ASM	
1-7	113321-034	ADJ UPH 18W TALL ASM	

(12/2017)

Note: each back height uses a different variation of back straps; replacement strap kits contain more straps than needed to support all back heights.


Note: this upholstery is not offered on 8"-9" Std Back widths and 8"-11" chair widths with Inset Back.

Note: for Inset Backrest upholstery must be 2" narrower than listed width.

Note: each back height uses a different variation of back straps; replacement strap kits contain more straps than needed to support all back heights.


Note: this upholstery is not offered on 8"-11" Std Back Width and 8"-13" chair widths with Inset Backrest option.

Note: for Inset Backrest option, upholstery must be 2" narrower than listed width.

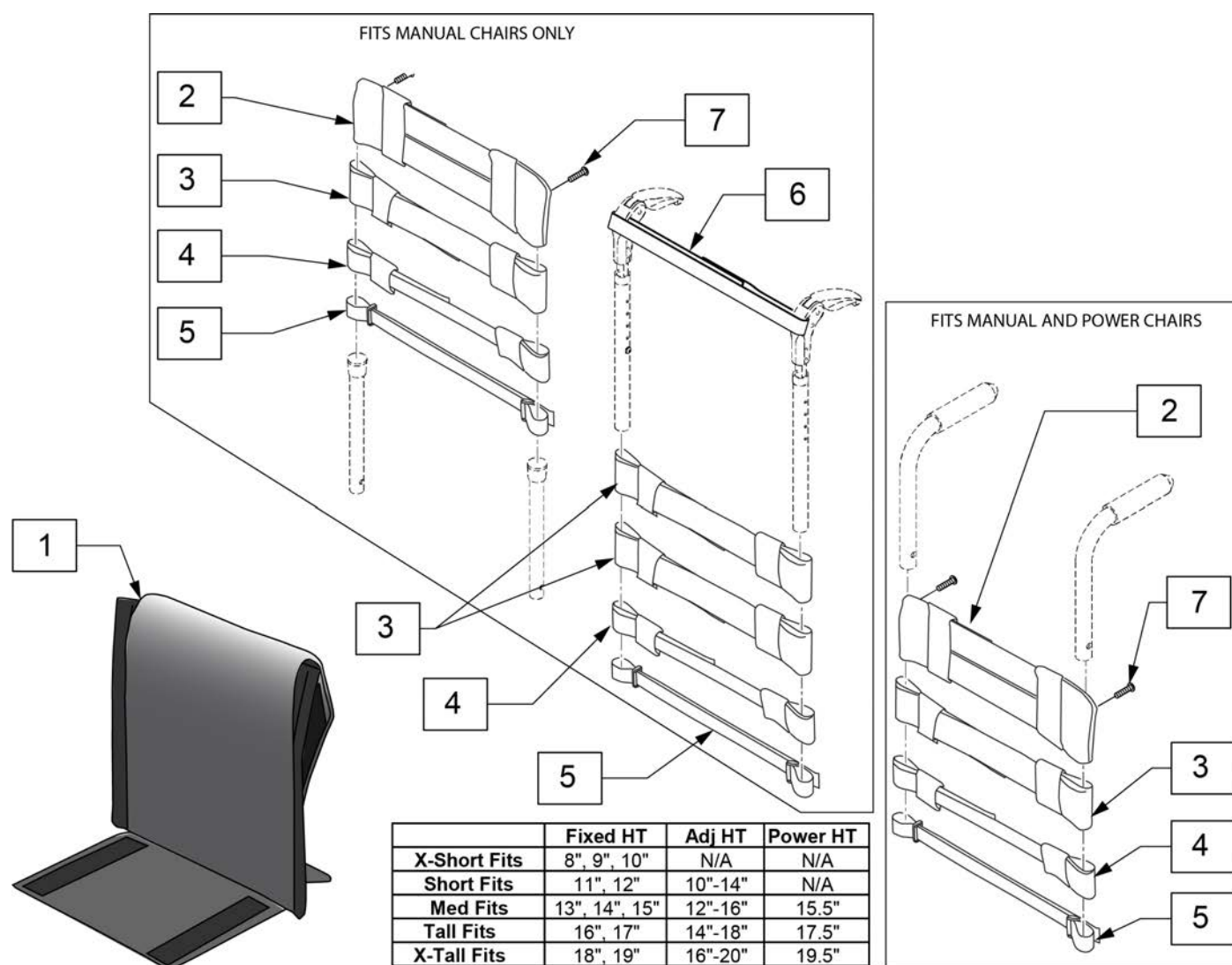


Pos.	Item Number	Description	Remarks
2	113712	TOP CAP, STD, 11 -12"	Used for omit push handle & Integral Push Handle only
2	113713	TOP CAP, STD, 13 -14"	Used for omit push handle & Integral Push Handle only
2	113714	TOP CAP, STD, 15 -16"	Used for omit push handle & Integral Push Handle only
2	113715	TOP CAP, STD, 17 -18"	Used for omit push handle & Integral Push Handle only
3	113727	STRAP PADDED 2", 11 -12"	
3	113728	STRAP PADDED 2", 13 -14"	


Pos.	Item Number	Description	Remarks
3	113729	STRAP PADDED 2", 15 -16"	
3	113730	STRAP PADDED 2", 17 -18"	
4	113732	STRAP PADDED 1", 11 -12"	
4	113733	STRAP PADDED 1", 13 -14"	
4	113734	STRAP PADDED 1", 15 -16"	
4	113735	STRAP PADDED 1", 16 -17"	
4	113736	STRAP PADDED 1", 18 -19"	
5	113717	POSITION STRAP 1", 11 -12	
5	113718	POSITION STRAP 1", 13 -14	
5	113719	POSITION STRAP 1", 15 -16	
5	113720	POSITION STRAP 1", 17 -18	
6	052435-001	UPH,VNTD ADJ.,FPH STRP,12-13"W	Fold Down Push Handles only
6	052435-002	UPH,VNTD ADJ.,FPH STRP,14-15"W	Fold Down Push Handles only
6	052435-003	UPH,VNTD ADJ.,FPH STRP,16-17"W	Fold Down Push Handles only
6	052435-004	UPH,VNTD ADJ.,FPH STRP,18-20"W	Fold Down Push Handles only
7	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
2-7	114879	ADJ UPH 11-12W STRAP KIT	
2-7	114880	ADJ UPH 13-14W STRAP KIT	
2-7	114881	ADJ UPH 15-16W STRAP KIT	
2-7	114882	ADJ UPH 17-18W STRAP KIT	
1A-7	113322-001	VNT ADJ UP 12W X-SHT ASM	
1B-7	113322-002	VNT ADJ UP 12W SHT ASM	
1C-7	113322-003	VNT ADJ UP 12W MED ASM	
1D-7	113322-004	VNT ADJ UP 12W TAL ASM	
1E-7	113322-005	VNT ADJ UP 12W X-TAL ASM	
1A-7	113322-006	VNT ADJ UP 13W X-SHT ASM	
1B-7	113322-007	VNT ADJ UP 13-W SHT ASM	
1C-7	113322-008	VNT ADJ UP 13W MED ASM	
1D-7	113322-009	VNT ADJ UP 13W TAL ASM	
1E-7	113322-010	VNT ADJ UP 13W X-TAL ASM	
1A-7	113322-011	VNT ADJ UP 14W X-SHT ASM	
1B-7	113322-012	VNT ADJ UP 14W SHT ASM	
1C-7	113322-013	VNT ADJ UP 14W MED ASM	
1D-7	113322-014	VNT ADJ UP 14W TAL ASM	
1E-7	113322-015	VNT ADJ UP 14W X-TAL ASM	
1A-7	113322-016	VNT ADJ UP 15W X-SHT ASM	
1B-7	113322-017	VNT ADJ UP 15W SHT ASM	
1C-7	113322-018	VNT ADJ UP 15W MED ASM	
1D-7	113322-019	VNT ADJ UP 15W TAL ASM	
1E-7	113322-020	VNT ADJ UP 15W X-TAL ASM	
1A-7	113322-021	VNT ADJ UP 16W X-SHT ASM	
1B-7	113322-022	VNT ADJ UP 16W SHT ASM	
1C-7	113322-023	VNT ADJ UP 16W MED ASM	
1D-7	113322-024	VNT ADJ UP 16W TAL ASM	
1E-7	113322-025	VNT ADJ UP 16W X-TAL ASM	
1A-7	113322-026	VNT ADJ UP 17W X-SHT ASM	
1B-7	113322-027	VNT ADJ UP 17W SHT ASM	
1C-7	113322-028	VNT ADJ UP 17W MED ASM	
1D-7	113322-029	VNT ADJ UP 17W TAL ASM	
1E-7	113322-030	VNT ADJ UP 17W X-TAL ASM	
1A-7	113322-031	VNT ADJ UP 18W X-SHT ASM	
1B-7	113322-032	VNT ADJ UP 18W SHT ASM	
1C-7	113322-033	VNT ADJ UP 18W MED ASM	
1D-7	113322-034	VNT ADJ UP 18W TAL ASM	
1E-7	113322-035	VNT ADJ UP 18W X-TAL ASM	

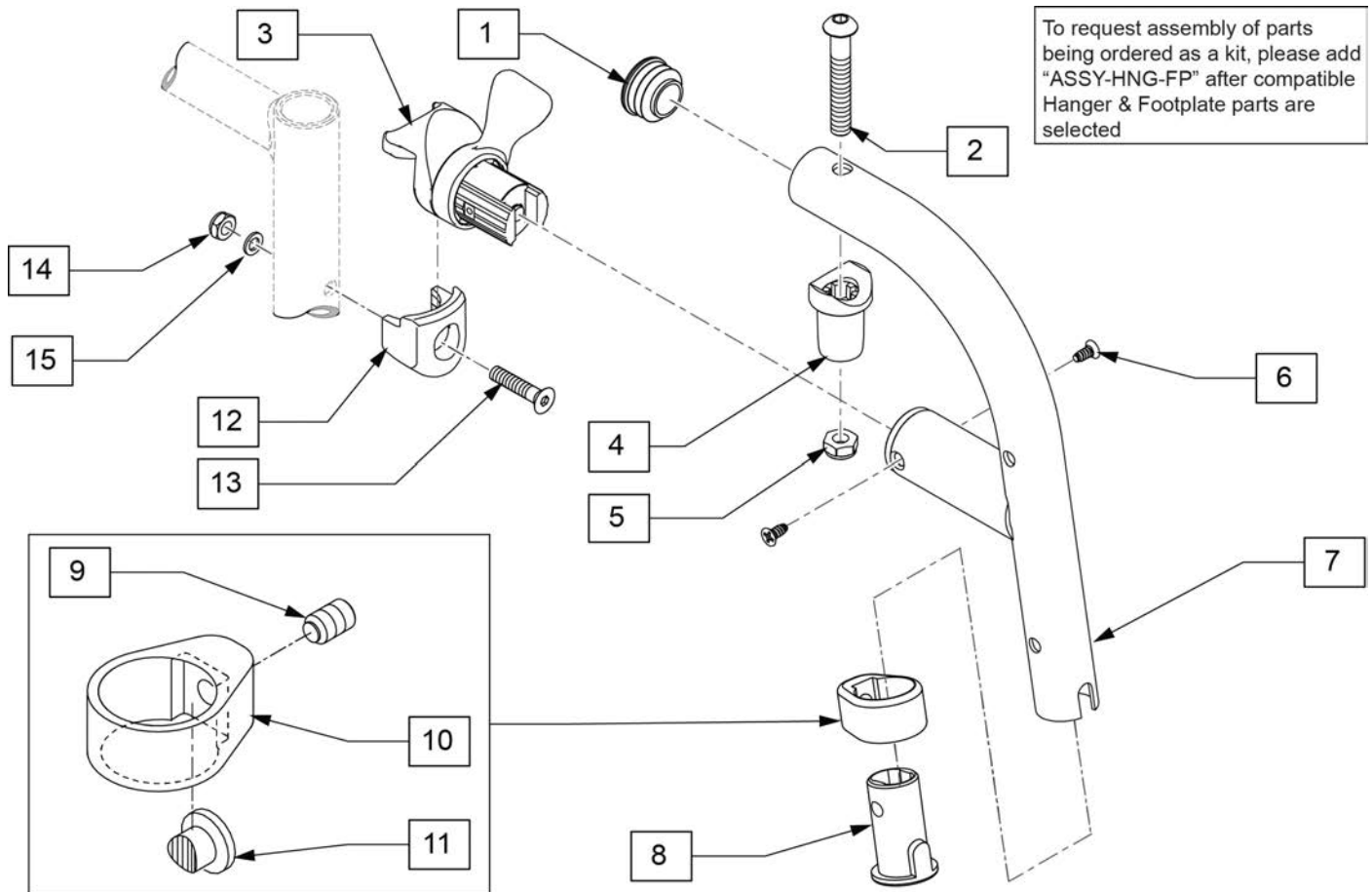

Pos.	Item Number	Description	Remarks
1	561062	1" INSERT PLUG .065 RIBS	
8	128357	BUSHING, FOOTREST, DIRECT	
13	103500-010	FHCSM6X1X35FULLNP	
14	104381	NUT HEX NYLOCK M6X1.0	
15	104414	WAVE WASHER 6 MM	
1,2,4,5	119518	PIVOT SADDLE KIT	
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right
1-11	136186S	80D HANGER R EXT	Right-hangers that use extension
1-11	136187S	80D HANGER L EXT	Left-hangers that use extension
8-11	136188	FOOTREST CLAMP KIT	
12-15	119514	LATCH RECEIVER KIT	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

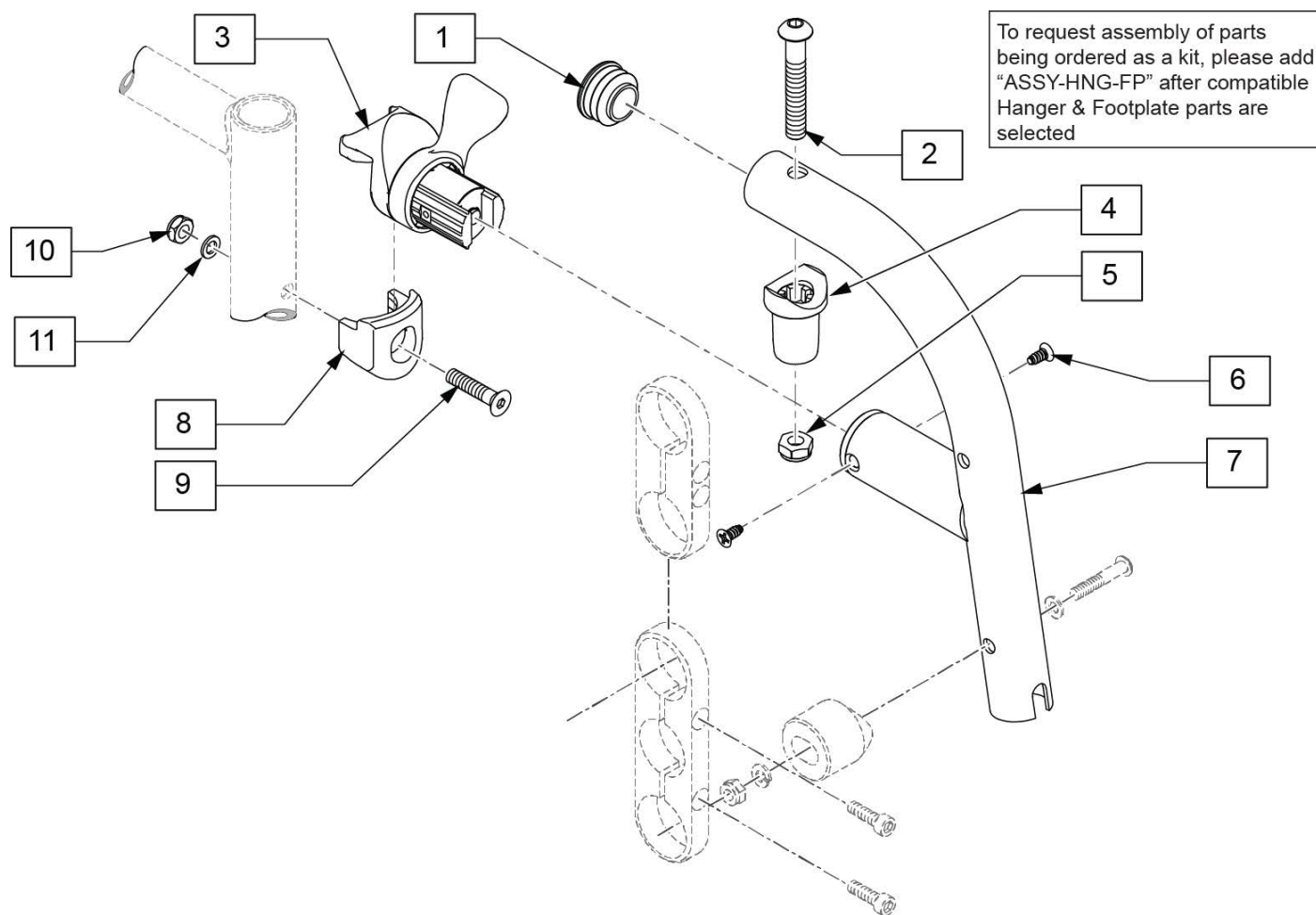

Pos.	Item Number	Description	Remarks
1	561062	1" INSERT PLUG .065 RIBS	
9	103500-010	FHCSM6X1X35FULLNP	
10	104381	NUT HEX NYLOCK M6X1.0	
11	104414	WAVE WASHER 6 MM	
1,2,4,5	119518	PIVOT SADDLE KIT	
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right
1-7	136184	80D HANGER R DIRECT	Right
1-7	136185	80D HANGER L DIRECT	Left
8-11	119514	LATCH RECEIVER KIT	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

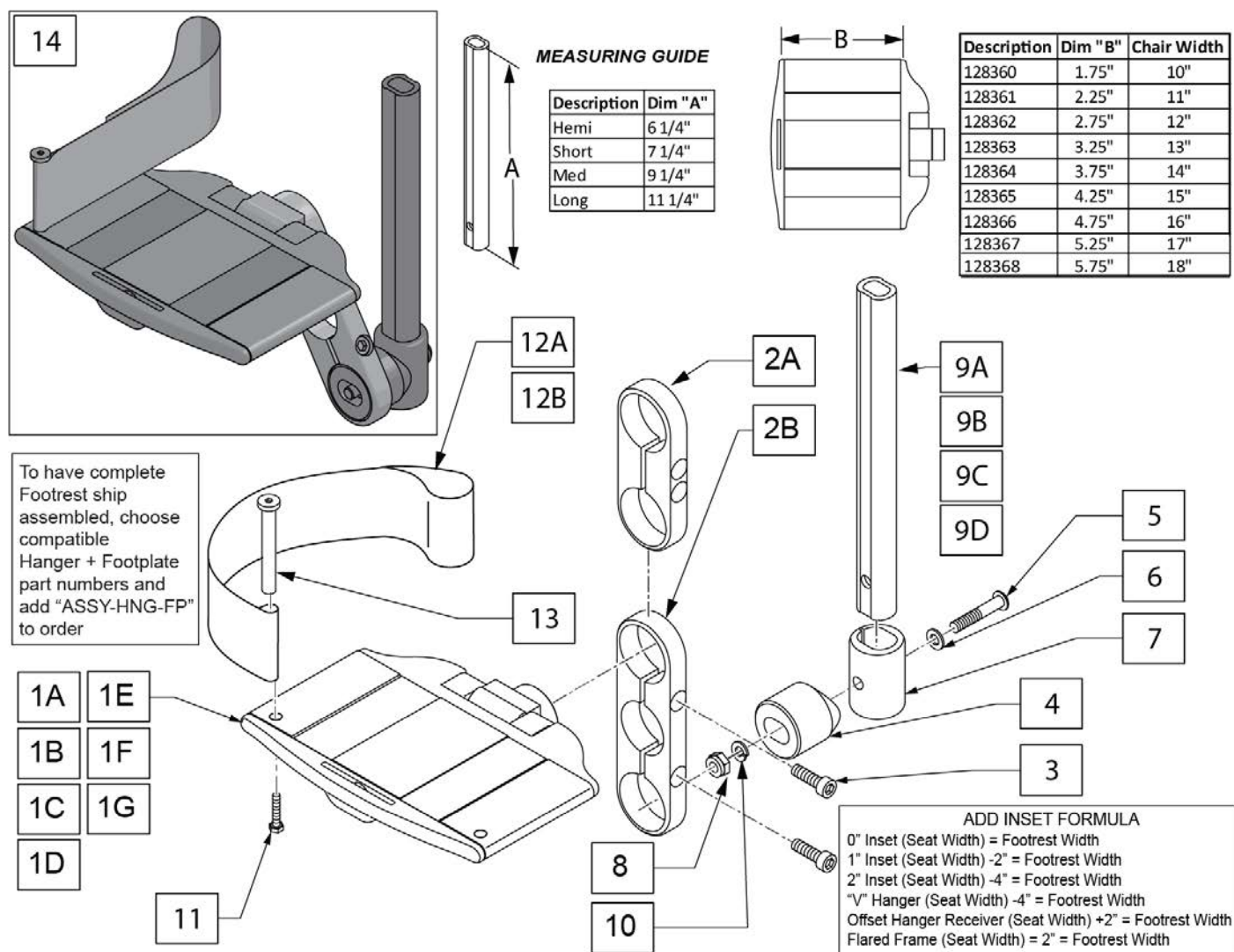


Pos.	Item Number	Description	Remarks
1A	128360	10" ALUM DIVIDED FTRST	
1B	128361	11" ALUM DIVIDED FTRST	
1C	128362	12" ALUM DIVIDED FTRST	
1D	128363	13" ALUM DIVIDED FTRST	
1E	128364	14" ALUM DIVIDED FTRST	
1F	128365	15" ALUM DIVIDED FTRST	
1G	128366	16" ALUM DIVIDED FTRST	
1H	128367	17" ALUM DIVIDED FTRST	
1I	128368	18" ALUM DIVIDED FTRST	
2A	128402	SHORT FOOTPLATE CLAMP	
2B	128403	LONG FOOTPLATE CLAMP	
3	102263-003	SHCS, M6X1X25, FULL, PL	
4	129058	HALF EXTEN TUBE ADAPTOR	
5	106964	BHCS 1/4-28 X 2	
6	107383	WASHER FLAT M6	
7	128229	EXTEN MOUNT LOWER ADAPTOR	
8	106934	NUT NYLOCK 1/4-28	
9A	129054	EXT TUBE FTREST SRT HEMI	

Pos.	Item Number	Description	Remarks
9B	129055	EXT TUBE FTREST SRT SHT	
9C	129056	EXT TUBE FTREST SRT MED	
9D	129057	COMBO EXT TUBE LNG	
10	107367	WASHER LOCK HI COLLAR 1/4 ROHS	
11	102268-001	HHCS M6 X 1 X 25	
12A	104380-001	ADJ FOOTREST STRAP 14W	10" to 14" wide
12B	104380-002	ADJ FOOTREST STRAP 15-16W	15' to 18" wide
13	104328	HEEL LOOP PEG	
		HEMI EXTENSION / 2 HOLE CLAMP	
14	136189	10" FTPLT HEMI SHORT RT	Right
14	136190	10" FTPLT HEMI SHORT LFT	Left
14	136191	11" FTPLT HEMI SHORT RT	Right
14	136192	11" FTPLT HEMI SHORT LFT	Left
14	136193	12" FTPLT HEMI SHORT RT	Right
14	136194	12" FTPLT HEMI SHORT LFT	Left
14	136195	13" FTPLT HEMI SHORT RT	Right
14	136196	13" FTPLT HEMI SHORT LFT	Left
14	136197	14" FTPLT HEMI SHORT RT	Right
14	136198	14" FTPLT HEMI SHORT LFT	Left
14	136199	15" FTPLT HEMI SHORT RT	Right
14	136200	15" FTPLT HEMI SHORT LFT	Left
14	136201	16" FTPLT HEMI SHORT RT	Right
14	136202	16" FTPLT HEMI SHORT LFT	Left
14	136203	17" FTPLT HEMI SHORT RT	Right
14	136204	17" FTPLT HEMI SHORT LFT	Left
14	136205	18" FTPLT HEMI SHORT RT	Right
14	136206	18" FTPLT HEMI SHORT LFT	Left
		SHORT EXTENSION / 2 HOLE CLAMP	
14	136207	10" FTPLT SHORT SHORT RT	Right
14	136208	10" FTPLT SHORT SHORT LFT	Left
14	136209	11" FTPLT SHORT SHORT RT	Right
14	136210	11" FTPLT SHORT SHORT LFT	Left
14	136211	12" FTPLT SHORT SHORT RT	Right
14	136212	12" FTPLT SHORT SHORT LFT	Left
14	136213	13" FTPLT SHORT SHORT RT	Right
14	136214	13" FTPLT SHORT SHORT LFT	Left
14	136215	14" FTPLT SHORT SHORT RT	Right
14	136216	14" FTPLT SHORT SHORT LFT	Left
14	136217	15" FTPLT SHORT SHORT RT	Right
14	136218	15" FTPLT SHORT SHORT LFT	Left
14	136219	16" FTPLT SHORT SHORT RT	Right
14	136220	16" FTPLT SHORT SHORT LFT	Left
14	136221	17" FTPLT SHORT SHORT RT	Right
14	136222	17" FTPLT SHORT SHORT LFT	Left
14	136223	18" FTPLT SHORT SHORT RT	Right
14	136224	18" FTPLT SHORT SHORT LFT	Left
		MED EXTENSION / 2 HOLE CLAMP	
14	136225	10" FTPLT MED SHORT RT	Right
14	136226	10" FTPLT MED SHORT LFT	Left
14	136227	11" FTPLT MED SHORT RT	Right
14	136228	11" FTPLT MED SHORT LFT	Left
14	136229	12" FTPLT MED SHORT RT	Right
14	136230	12" FTPLT MED SHORT LFT	Left
14	136231	13" FTPLT MED SHORT RT	Right
14	136232	13" FTPLT MED SHORT LFT	Left
14	136233	14" FTPLT MED SHORT RT	Right
14	136234	14" FTPLT MED SHORT LFT	Left


Pos.	Item Number	Description	Remarks
14	136235	15" FTPLT MED SHORT RT	Right
14	136236	15" FTPLT MED SHORT LFT	Left
14	136237	16" FTPLT MED SHORT RT	Right
14	136238	16" FTPLT MED SHORT LFT	Left
14	136239	17" FTPLT MED SHORT RT	Right
14	136240	17" FTPLT MED SHORT LFT	Left
14	136241	18" FTPLT MED SHORT RT	Right
14	136242	18" FTPLT MED SHORT LFT	Left
		LONG EXTENSION / 2 HOLE CLAMP	
14	136243	10" FTPLT LONG SHORT RT	Right
14	136244	10" FTPLT LONG SHORT LFT	Left
14	136245	11" FTPLT LONG SHORT RT	Right
14	136246	11" FTPLT LONG SHORT LFT	Left
14	136247	12" FTPLT LONG SHORT RT	Right
14	136248	12" FTPLT LONG SHORT LFT	Left
14	136249	13" FTPLT LONG SHORT RT	Right
14	136250	13" FTPLT LONG SHORT LFT	Left
14	136251	14" FTPLT LONG SHORT RT	Right
14	136252	14" FTPLT LONG SHORT LFT	Left
14	136253	15" FTPLT LONG SHORT RT	Right
14	136254	15" FTPLT LONG SHORT LFT	Left
14	136255	16" FTPLT LONG SHORT RT	Right
14	136256	16" FTPLT LONG SHORT LFT	Left
14	136257	17" FTPLT LONG SHORT RT	Right
14	136258	17" FTPLT LONG SHORT LFT	Left
14	136259	18" FTPLT LONG SHORT RT	Right
14	136260	18" FTPLT LONG SHORT LFT	Left
		HEMI EXTENSION / 3 HOLE CLAMP	
14	136261	10" FTPLT HEMI LONG RT	Right
14	136262	10" FTPLT HEMI LONG LFT	Left
14	136263	11" FTPLT HEMI LONG RT	Right
14	136264	11" FTPLT HEMI LONG LFT	Left
14	136265	12" FTPLT HEMI LONG RT	Right
14	136266	12" FTPLT HEMI LONG LFT	Left
14	136267	13" FTPLT HEMI LONG RT	Right
14	136268	13" FTPLT HEMI LONG LFT	Left
14	136269	14" FTPLT HEMI LONG RT	Right
14	136270	14" FTPLT HEMI LONG LFT	Left
14	136271	15" FTPLT HEMI LONG RT	Right
14	136272	15" FTPLT HEMI LONG LFT	Left
14	136273	16" FTPLT HEMI LONG RT	Right
14	136274	16" FTPLT HEMI LONG LFT	Left
14	136275	17" FTPLT HEMI LONG RT	Right
14	136276	17" FTPLT HEMI LONG LFT	Left
14	136277	18" FTPLT HEMI LONG RT	Right
14	136278	18" FTPLT HEMI LONG LFT	Left
		SHORT EXTENSION / 3 HOLE CLAMP	
14	136279	10" FTPLT SHORT LONG RT	Right
14	136280	10" FTPLT SHORT LONG LFT	Left
14	136281	11" FTPLT SHORT LONG RT	Right
14	136282	11" FTPLT SHORT LONG LFT	Left
14	136283	12" FTPLT SHORT LONG RT	Right
14	163269	12" FTPLT SHORT LONG LFT	Left
14	136284	13" FTPLT SHORT LONG RT	Right
14	136285	13" FTPLT SHORT LONG LFT	Left
14	136286	14" FTPLT SHORT LONG RT	Right
14	136287	14" FTPLT SHORT LONG LFT	Left

Pos.	Item Number	Description	Remarks
14	136288	15" FTPLT SHORT LONG RT	Right
14	136289	15" FTPLT SHORT LONG LFT	Left
14	136290	16" FTPLT SHORT LONG RT	Right
14	136291	16" FTPLT SHORT LONG LFT	Left
14	136292	17" FTPLT SHORT LONG RT	Right
14	136293	17" FTPLT SHORT LONG LFT	Left
14	136294	18" FTPLT SHORT LONG RT	Right
14	136295	18" FTPLT SHORT LONG LFT	Left
		MED EXTENSION / 3 HOLE CLAMP	
14	136296	10" FTPLT MED LONG RT	Right
14	136297	10" FTPLT MED LONG LFT	Left
14	136298	11" FTPLT MED LONG RT	Right
14	136299	11" FTPLT MED LONG LFT	Left
14	136300	12" FTPLT MED LONG RT	Right
14	136301	12" FTPLT MED LONG LFT	Left
14	136302	13" FTPLT MED LONG RT	Right
14	136303	13" FTPLT MED LONG LFT	Left
14	136304	14" FTPLT MED LONG RT	Right
14	136305	14" FTPLT MED LONG LFT	Left
14	136306	15" FTPLT MED LONG RT	Right
14	136307	15" FTPLT MED LONG LFT	Left
14	136308	16" FTPLT MED LONG RT	Right
14	136309	16" FTPLT MED LONG LFT	Left
14	136310	17" FTPLT MED LONG RT	Right
14	136311	17" FTPLT MED LONG LFT	Left
14	136312	18" FTPLT MED LONG RT	Right
14	136313	18" FTPLT MED LONG LFT	Left
		LONG EXTENSION / 3 HOLE CLAMP	
14	136314	10" FTPLT LONG LONG RT	Right
14	136315	10" FTPLT LONG LONG LFT	Left
14	136316	11" FTPLT LONG LONG RT	Right
14	136317	11" FTPLT LONG LONG LFT	Left
14	136318	12" FTPLT LONG LONG RT	Right
14	136319	12" FTPLT LONG LONG LFT	Left
14	136320	13" FTPLT LONG LONG RT	Right
14	136321	13" FTPLT LONG LONG LFT	Left
14	136322	14" FTPLT LONG LONG RT	Right
14	136323	14" FTPLT LONG LONG LFT	Left
14	136324	15" FTPLT LONG LONG RT	Right
14	136325	15" FTPLT LONG LONG LFT	Left
14	136326	16" FTPLT LONG LONG RT	Right
14	136327	16" FTPLT LONG LONG LFT	Left
14	136328	17" FTPLT LONG LONG RT	Right
14	136329	17" FTPLT LONG LONG LFT	Left
14	136330	18" FTPLT LONG LONG RT	Right
14	136331	18" FTPLT LONG LONG LFT	Left
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

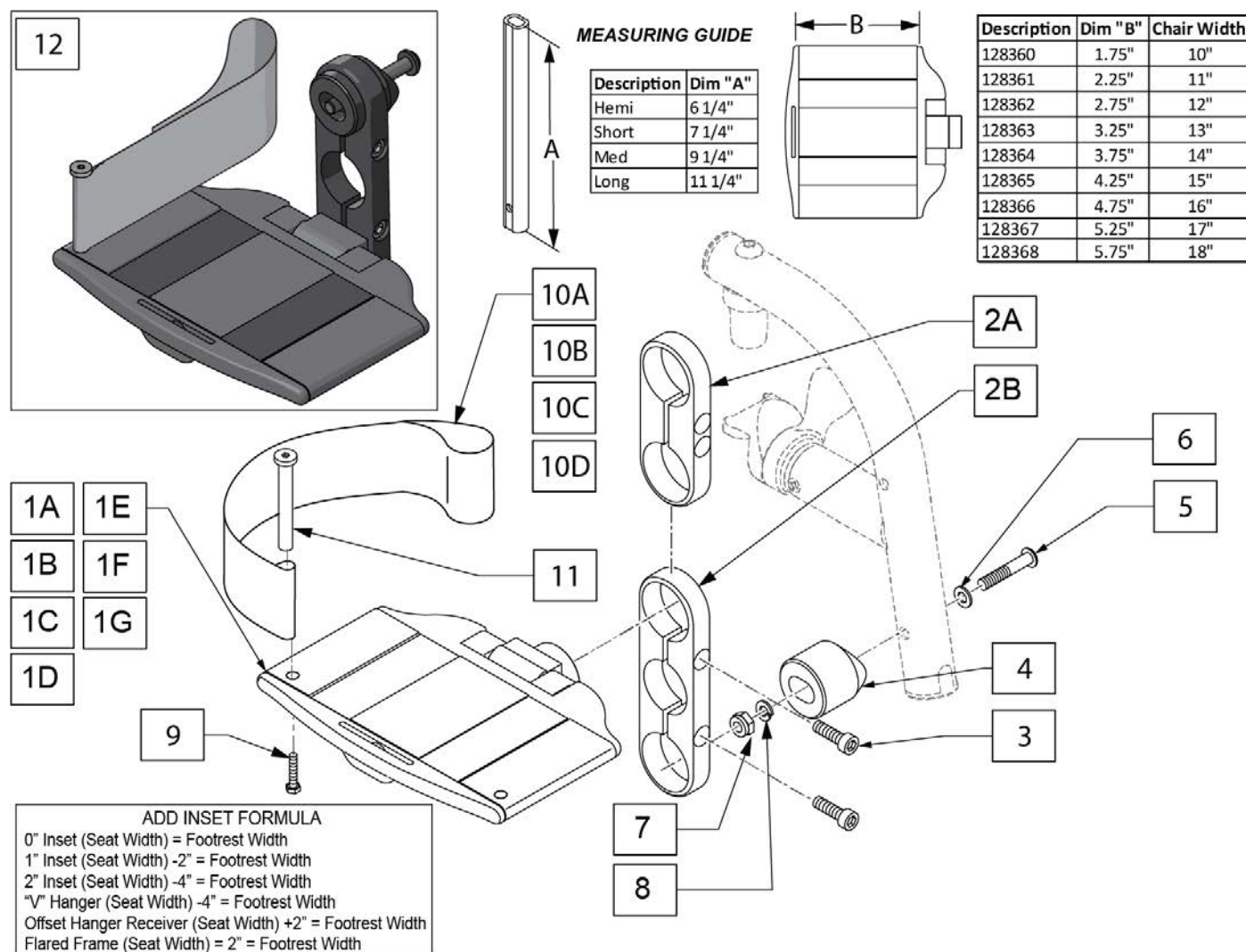

Pos.	Item Number	Description	Remarks
1A	128360	10" ALUM DIVIDED FTRST	
1B	128361	11" ALUM DIVIDED FTRST	
1C	128362	12" ALUM DIVIDED FTRST	
1D	128363	13" ALUM DIVIDED FTRST	
1E	128364	14" ALUM DIVIDED FTRST	
1F	128365	15" ALUM DIVIDED FTRST	
1G	128366	16" ALUM DIVIDED FTRST	
1H	128367	17" ALUM DIVIDED FTRST	
1I	128368	18" ALUM DIVIDED FTRST	
2A	128402	SHORT FOOTPLATE CLAMP	
2B	128403	LONG FOOTPLATE CLAMP	
3	102263-003	SHCS, M6X1X25, FULL, PL	
4	129058	HALF EXTEN TUBE ADAPTOR	
5	106964	BHCS 1/4-28 X 2	
6	107383	WASHER FLAT M6	
7	106934	NUT NYLOCK 1/4-28	
8	107367	WASHER LOCK HI COLLAR 1/4 ROHS	
9	102268-001	HHCS M6 X 1 X 25	
10A	104380-001	ADJ FOOTREST STRAP 14W	10" to 14" wide


Pos.	Item Number	Description	Remarks
10B	104380-002	ADJ FOOTREST STRAP 15-16W	15' to 18" wide
11	104328	HEEL LOOP PEG	
		2 HOLE CLAMP NO EXT	
12	136332	10" FTPLT SHORT RT	Right
12	136333	10" FTPLT SHORT LFT	Left
12	136334	11" FTPLT SHORT RT	Right
12	136335	11" FTPLT SHORT LFT	Left
12	136336	12" FTPLT SHORT RT	Right
12	136337	12" FTPLT SHORT LFT	Left
12	136338	13" FTPLT SHORT RT	Right
12	136339	13" FTPLT SHORT LFT	Left
12	136340	14" FTPLT SHORT RT	Right
12	136341	14" FTPLT SHORT LFT	Left
12	136342	15" FTPLT SHORT RT	Right
12	136343	15" FTPLT SHORT LFT	Left
12	136344	16" FTPLT SHORT RT	Right
12	136345	16" FTPLT SHORT LFT	Left
12	136346	17" FTPLT SHORT RT	Right
12	136347	17" FTPLT SHORT LFT	Left
12	136348	18" FTPLT SHORT RT	Right
12	136349	18" FTPLT SHORT LFT	Left
		3 HOLE CLAMP NO EXT	
12	136350	10" FTPLT LONG RT	Right
12	136351	10" FTPLT LONG LFT	Left
12	136352	11" FTPLT LONG RT	Right
12	136353	11" FTPLT LONG LFT	Left
12	136354	12" FTPLT LONG RT	Right
12	136355	12" FTPLT LONG LFT	Left
12	136356	13" FTPLT LONG RT	Right
12	136357	13" FTPLT LONG LFT	Left
12	136358	14" FTPLT LONG RT	Right
12	136359	14" FTPLT LONG LFT	Left
12	136360	15" FTPLT LONG RT	Right
12	136361	15" FTPLT LONG LFT	Left
12	136362	16" FTPLT LONG RT	Right
12	136363	16" FTPLT LONG LFT	Left
12	136364	17" FTPLT LONG RT	Right
12	136365	17" FTPLT LONG LFT	Left
12	136366	18" FTPLT LONG RT	Right
12	136367	18" FTPLT LONG LFT	Left
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

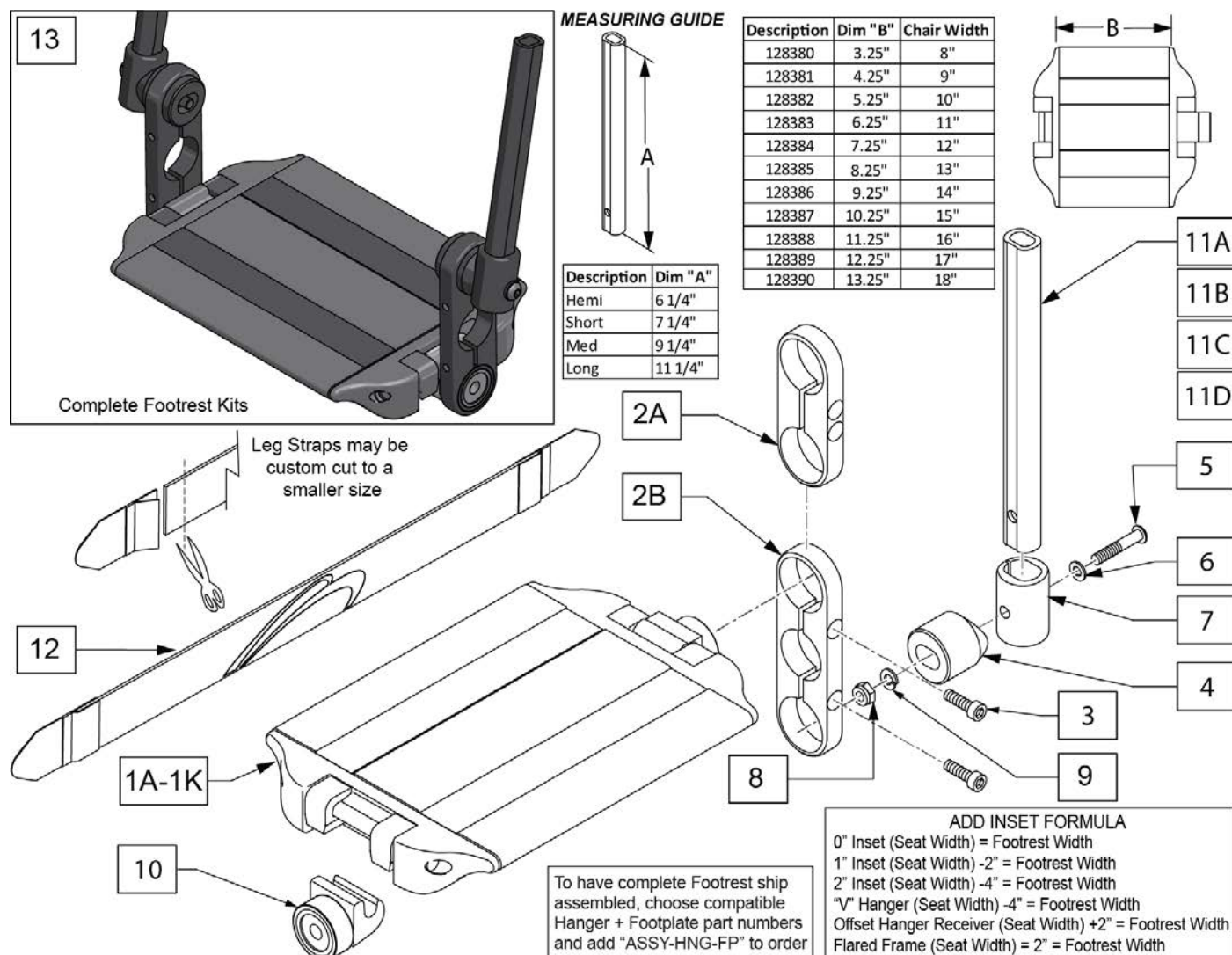

Pos.	Item Number	Description	Remarks
1A	128380	8" ALUM CONT FTRST	
1B	128381	9" ALUM CONT FTRST	
1C	128382	10" ALUM CONT FTRST	
1D	128383	11" ALUM CONT FTRST	
1E	128384	12" ALUM CONT FTRST	
1F	128385	13" ALUM CONT FTRST	
1G	128386	14" ALUM CONT FTRST	
1H	128387	15" ALUM CONT FTRST	
1I	128388	16" ALUM CONT FTRST	
1J	128389	17" ALUM CONT FTRST	
1K	128390	18" ALUM CONT FTRST	
2A	128402	SHORT FOOTPLATE CLAMP	
2B	128403	LONG FOOTPLATE CLAMP	
3	102263-003	SHCS, M6X1X25, FULL, PL	
4	129058	HALF EXTEN TUBE ADAPTOR	
5	106964	BHCS 1/4-28 X 2	
6	107383	WASHER FLAT M6	
7	128229	EXTEN MOUNT LOWER ADAPTOR	


Pos.	Item Number	Description	Remarks
8	106935	NUT NYLOCK THIN 1/4-28	
9	107367	WASHER LOCK HI COLLAR 1/4	
10	128410	ALUM CONT FTRST CATCH	
11A	129054	EXT TUBE FTREST SRT HEMI	
11B	129055	EXT TUBE FTREST SRT SHT	
11C	129056	EXT TUBE FTREST SRT MED	
11D	129057	COMBO EXT TUBE LNG	
12	113255	LEG STRAP 11"-12" BLK	
12	113256	LEG STRAP 13"-15" BLK	
12	113257	LEG STRAP 16"-18" BLK	
		HEMI EXTENSION / 2 HOLE CLAMP	
13	136368	10" FLIP-UP FTPLT HEMI SHORT	
13	136369	11" FLIP-UP FTPLT HEMI SHORT	
13	136370	12" FLIP-UP FTPLT HEMI SHORT	
13	136371	13" FLIP-UP FTPLT HEMI SHORT	
13	136372	14" FLIP-UP FTPLT HEMI SHORT	
13	136373	15" FLIP-UP FTPLT HEMI SHORT	
13	136374	16" FLIP-UP FTPLT HEMI SHORT	
13	136375	17" FLIP-UP FTPLT HEMI SHORT	
13	136376	18" FLIP-UP FTPLT HEMI SHORT	
		SHORT EXTENSION / 2 HOLE CLAMP	
13	136377	10" FLIP-UP FTPLT SHORT SHORT	
13	136378	11" FLIP-UP FTPLT SHORT SHORT	
13	136379	12" FLIP-UP FTPLT SHORT SHORT	
13	136380	13" FLIP-UP FTPLT SHORT SHORT	
13	136381	14" FLIP-UP FTPLT SHORT SHORT	
13	136382	15" FLIP-UP FTPLT SHORT SHORT	
13	136383	16" FLIP-UP FTPLT SHORT SHORT	
13	136384	17" FLIP-UP FTPLT SHORT SHORT	
13	136385	18" FLIP-UP FTPLT SHORT SHORT	
		MED EXTENSION / 2 HOLE CLAMP	
13	136386	10" FLIP-UP FTPLT MED SHORT	
13	136387	11" FLIP-UP FTPLT MED SHORT	
13	136388	12" FLIP-UP FTPLT MED SHORT	
13	136389	13" FLIP-UP FTPLT MED SHORT	
13	136390	14" FLIP-UP FTPLT MED SHORT	
13	136391	15" FLIP-UP FTPLT MED SHORT	
13	136392	16" FLIP-UP FTPLT MED SHORT	
13	136393	17" FLIP-UP FTPLT MED SHORT	
13	136394	18" FLIP-UP FTPLT MED SHORT	
		LONG EXTENSION / 2 HOLE CLAMP	
13	136395	10" FLIP-UP FTPLT LONG SHORT	
13	136396	11" FLIP-UP FTPLT LONG SHORT	
13	136397	12" FLIP-UP FTPLT LONG SHORT	
13	136398	13" FLIP-UP FTPLT LONG SHORT	
13	136399	14" FLIP-UP FTPLT LONG SHORT	
13	136400	15" FLIP-UP FTPLT LONG SHORT	
13	136401	16" FLIP-UP FTPLT LONG SHORT	
13	136402	17" FLIP-UP FTPLT LONG SHORT	
13	136403	18" FLIP-UP FTPLT LONG SHORT	
		HEMI EXTENSION / 3 HOLE CLAMP	
13	136404	10" FLIP-UP FTPLT HEMI LONG	
13	136405	11" FLIP-UP FTPLT HEMI LONG	
13	136406	12" FLIP-UP FTPLT HEMI LONG	
13	136407	13" FLIP-UP FTPLT HEMI LONG	
13	136408	14" FLIP-UP FTPLT HEMI LONG	
13	136409	15" FLIP-UP FTPLT HEMI LONG	

Pos.	Item Number	Description	Remarks
13	136410	16" FLIP-UP FTPLT HEMI LONG	
13	136411	17" FLIP-UP FTPLT HEMI LONG	
13	136412	18" FLIP-UP FTPLT HEMI LONG	
		SHORT EXTENSION / 3 HOLE CLAMP	
13	136413	10" FLIP-UP FTPLT SHORT LONG	
13	136414	11" FLIP-UP FTPLT SHORT LONG	
13	136415	12" FLIP-UP FTPLT SHORT LONG	
13	136416	13" FLIP-UP FTPLT SHORT LONG	
13	136417	14" FLIP-UP FTPLT SHORT LONG	
13	136418	15" FLIP-UP FTPLT SHORT LONG	
13	136419	16" FLIP-UP FTPLT SHORT LONG	
13	136420	17" FLIP-UP FTPLT SHORT LONG	
13	136421	18" FLIP-UP FTPLT SHORT LONG	
		MED EXTENSION / 3 HOLE CLAMP	
13	136422	10" FLIP-UP FTPLT MED LONG	
13	136423	11" FLIP-UP FTPLT MED LONG	
13	136424	12" FLIP-UP FTPLT MED LONG	
13	136425	13" FLIP-UP FTPLT MED LONG	
13	136426	14" FLIP-UP FTPLT MED LONG	
13	136427	15" FLIP-UP FTPLT MED LONG	
13	136428	16" FLIP-UP FTPLT MED LONG	
13	136429	17" FLIP-UP FTPLT MED LONG	
13	136430	18" FLIP-UP FTPLT MED LONG	
		LONG EXTENSION / 3 HOLE CLAMP	
13	136431	10" FLIP-UP FTPLT LONG LONG	
13	136432	11" FLIP-UP FTPLT LONG LONG	
13	136433	12" FLIP-UP FTPLT LONG LONG	
13	136434	13" FLIP-UP FTPLT LONG LONG	
13	136435	14" FLIP-UP FTPLT LONG LONG	
13	136436	15" FLIP-UP FTPLT LONG LONG	
13	136437	16" FLIP-UP FTPLT LONG LONG	
13	136438	17" FLIP-UP FTPLT LONG LONG	
13	136439	18" FLIP-UP FTPLT LONG LONG	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

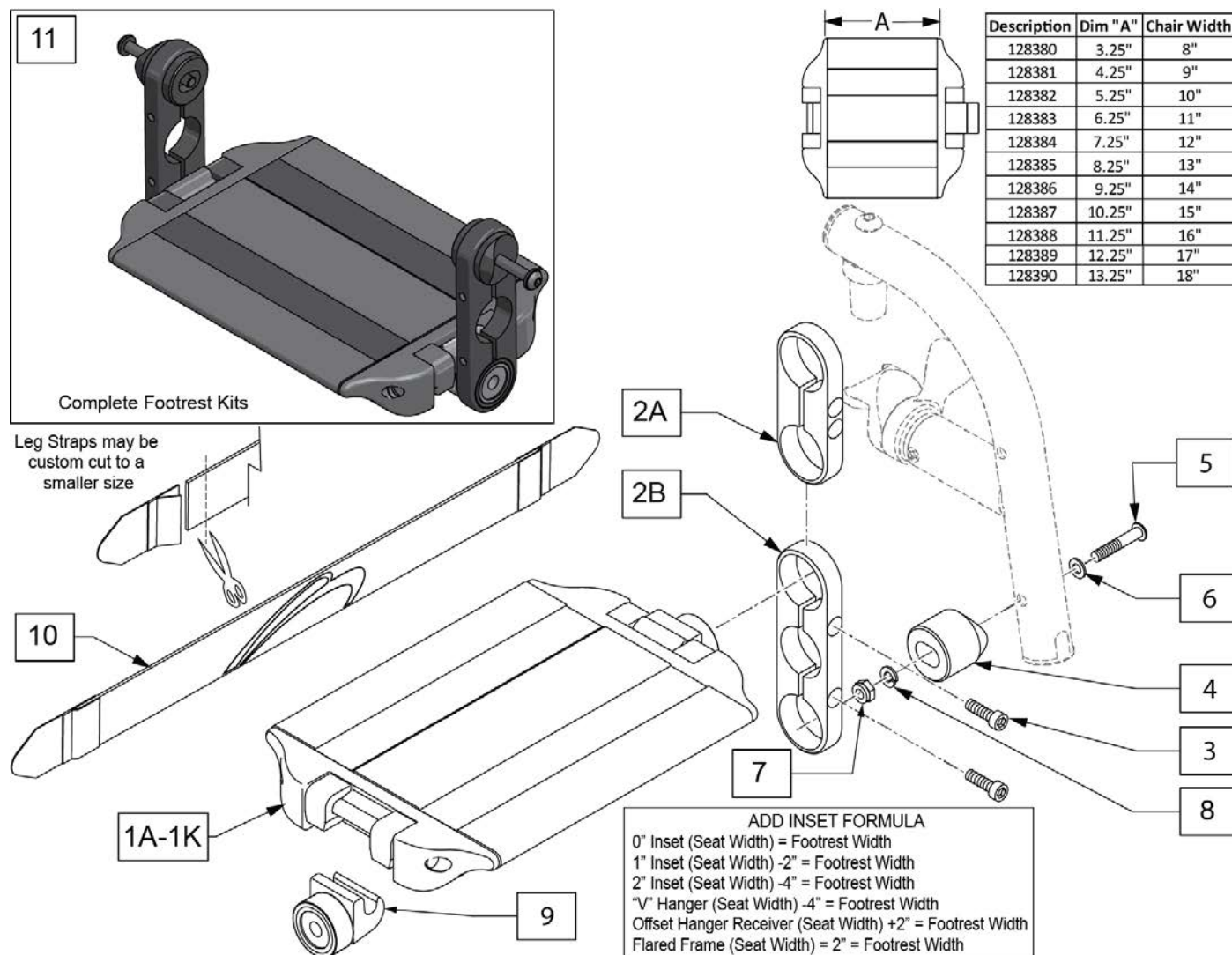


Pos.	Item Number	Description	Remarks
1A	128380	8" ALUM CONT FTRST	
1B	128381	9" ALUM CONT FTREST	
1C	128382	10" ALUM CONT FTRST	
1D	128383	11" ALUM CONT FTRST	
1E	128384	12" ALUM CONT FTRST	
1F	128385	13" ALUM CONT FTRST	
1G	128386	14" ALUM CONT FTRST	
1H	128387	15" ALUM CONT FTRST	
1I	128388	16" ALUM CONT FTRST	
1J	128389	17" ALUM CONT FTRST	
1K	128390	18" ALUM CONT FTRST	
2A	128402	SHORT FOOTPLATE CLAMP	
2B	128403	LONG FOOTPLATE CLAMP	
3	102263-003	SHCS, M6X1X25, FULL, PL	
4	129058	HALF EXTEN TUBE ADAPTOR	
5	106964	BHCS 1/4-28 X 2	
6	107383	WASHER FLAT M6	
7	106935	NUT NYLOCK THIN 1/4-28	

Pos.	Item Number	Description	Remarks
8	107367	WASHER LOCK HI COLLAR 1/4	
9	128410	ALUM CONT FTRST CATCH	
10	113255	LEG STRAP 11"-12" BLK	
10	113256	LEG STRAP 13"-15" BLK	
10	113257	LEG STRAP 16"-18" BLK	
		2 HOLE CLAMP NO EXT	
11	136440	10" FLIP-UP FTPLT SHORT	
11	136441	11" FLIP-UP FTPLT SHORT	
11	136442	12" FLIP-UP FTPLT SHORT	
11	136443	13" FLIP-UP FTPLT SHORT	
11	136444	14" FLIP-UP FTPLT SHORT	
11	136445	15" FLIP-UP FTPLT SHORT	
11	136446	16" FLIP-UP FTPLT SHORT	
11	136447	17" FLIP-UP FTPLT SHORT	
11	136448	18" FLIP-UP FTPLT SHORT	
		3 HOLE CLAMP NO EXT	
11	136449	10" FLIP-UP FTPLT LONG	
11	136450	11" FLIP-UP FTPLT LONG	
11	136451	12" FLIP-UP FTPLT LONG	
11	136452	13" FLIP-UP FTPLT LONG	
11	136453	14" FLIP-UP FTPLT LONG	
11	136454	15" FLIP-UP FTPLT LONG	
11	136455	16" FLIP-UP FTPLT LONG	
11	136456	17" FLIP-UP FTPLT LONG	
11	136457	18" FLIP-UP FTPLT LONG	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

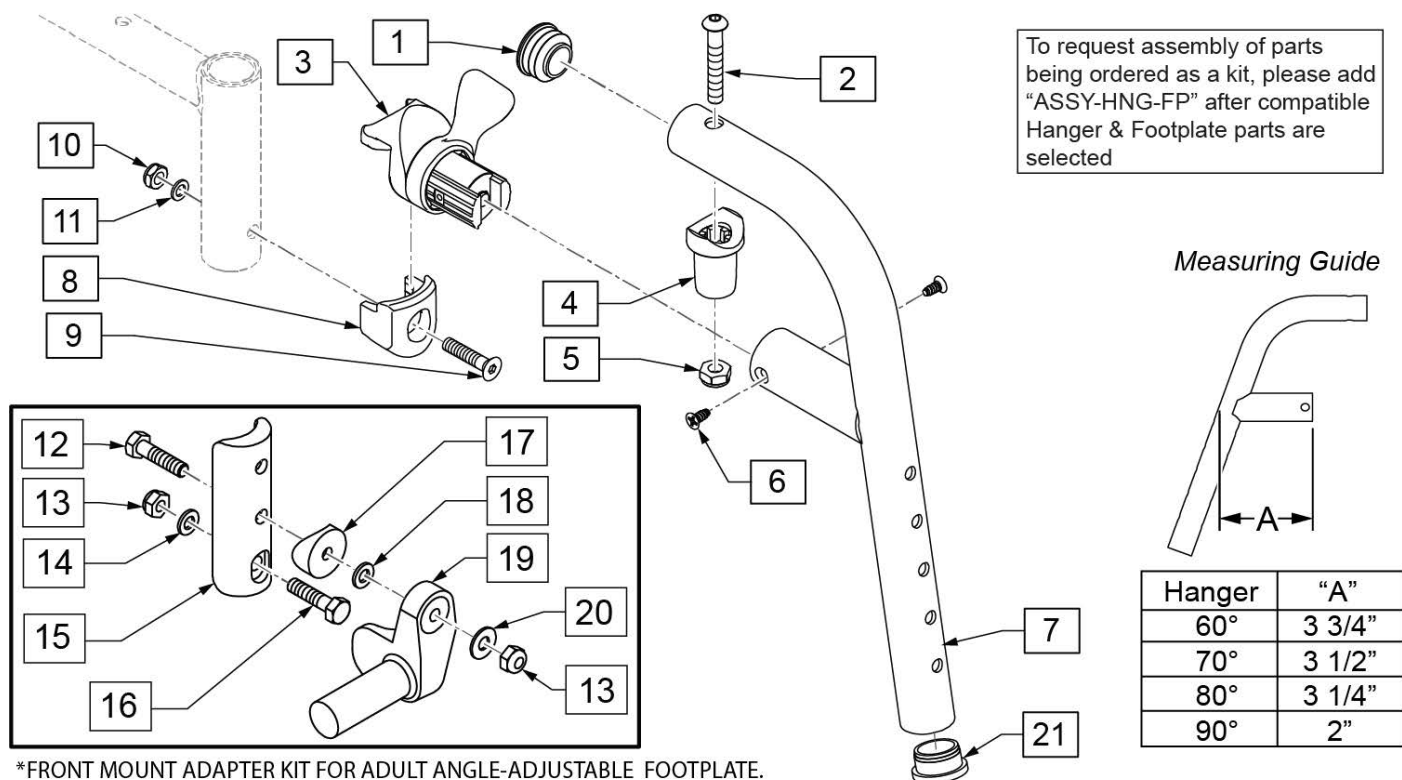

Pos.	Item Number	Description	Remarks
1	561062	1" INSERT PLUG .065 RIBS	
2	226217	5/16-18 X 2" BH C/S BLK	
4	561197	PLASTIC PIVOT SADDLE	
5	202102	5/16-18 NYLOCK NUT THIN	
6	118695	#8 LOW FLT HD TAP SCREW	
9	103500-010	FHCSM6X1X35FULLNP	
10	104381	NUT HEX NYLOCK M6X1.0	
11	104414	WAVE WASHER 6 MM	
12	102268-001	HHCS M6 X 1 X 25	
13	104381	NUT HEX NYLOCK M6 X 1	
14	107383	WASHER FLAT M6 ROHS	
15	119240	FOOTPLATE ADAPTER FM	3" tall
16	118198	HHCS M6X1X40 PART NP	
17	561004	1" SADDLE 1/4" HOLE	
18	106944	WASHER FLAT AN 1/4 ROHS	
19	541000	FOOTREST MOUNTING BRKT 1/2" RAD R	Right
19	541001	FOOTREST MOUNTING BRKT 1/2" RAD L	Left
20	106944	WASHER FLAT SAE 1/4 ROHS	
1,2,4,5	119518	PIVOT SADDLE KIT	
1-7	119464S	80 D FRNT MNT SWING IN/OUT HANGER L	Left
1-7	119465S	80 D FRNT MNT SWING IN/OUT HANGER R	Right
8-11	119514	LATCH RECEIVER KIT	
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right
21	561002	1" INSERT PLUG .083	
12-20	129764	AAA FRNT MNT ADAPTER KIT PR	Pair
21	561002	1" INSERT PLUG .083	
		Ship assembled part #	Use with multiple assemblies


Pos.	Item Number	Description	Remarks
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

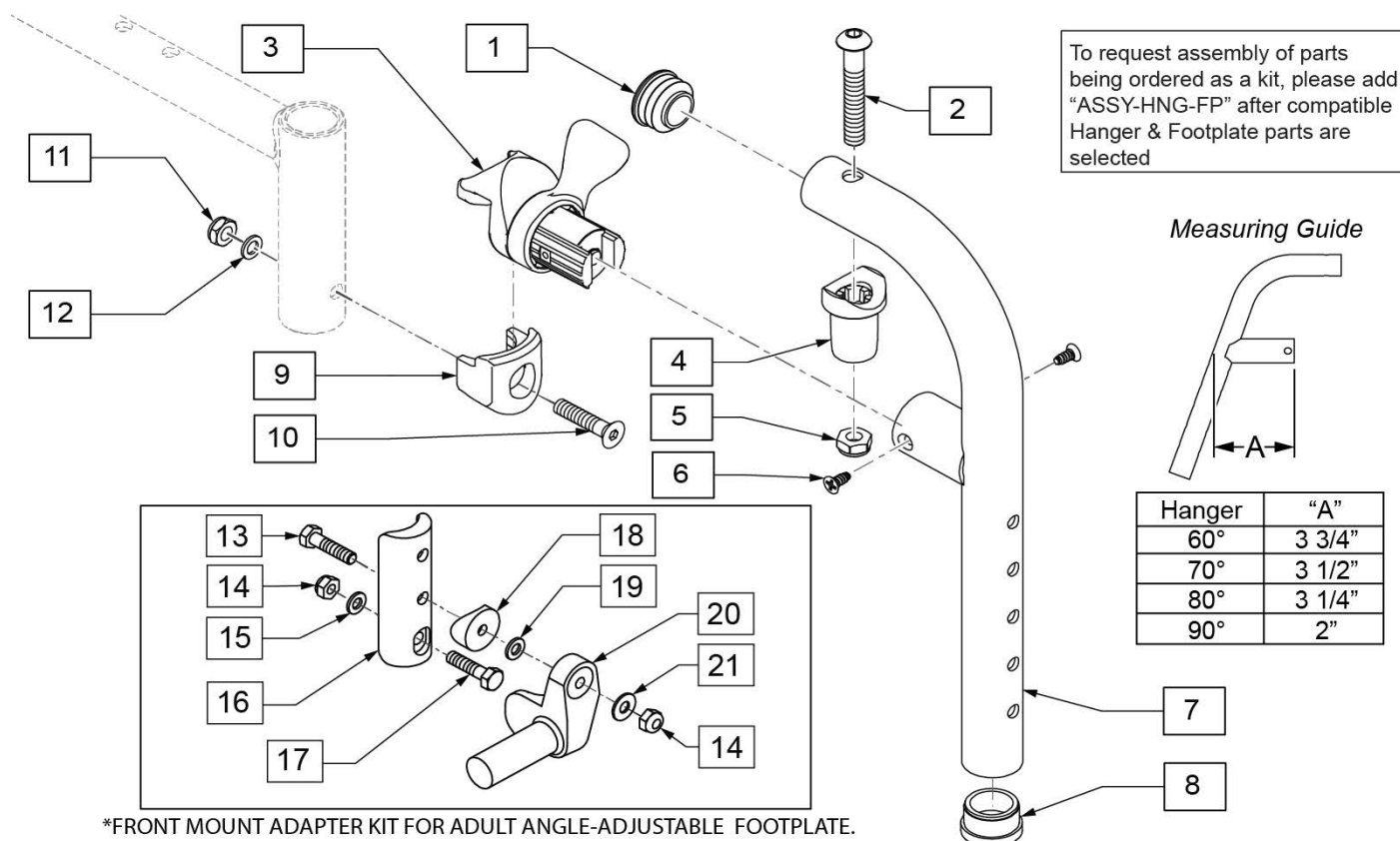

Pos.	Item Number	Description	Remarks
1	561062	1" INSERT PLUG .065 RIBS	
2	226217	5/16-18 X 2" BH C/S BLK	
4	561197	PLASTIC PIVOT SADDLE	
5	202102	5/16-18 NYLOCK NUT THIN	
6	118695	#8 LOW FLT HD TAP SCREW	
8	561002	1" INSERT PLUG .083	
10	103500-010	FHCSM6X1X35FULLNP	
11	104381	NUT HEX NYLOCK M6X1.0	
12	104414	WAVE WASHER 6 MM	
13	102268-001	HHCS M6 X 1 X 25	
14	104381	NUT HEX NYLOCK M6 X 1	
15	107383	WASHER FLAT M6 ROHS	
16	119240	FOOTPLATE ADAPTER FM	3" tall
17	118198	HHCS M6X1X40 PART NP	
18	561004	1" SADDLE 1/4" HOLE	
19	106943	WASHER FLAT AN 1/4 ROHS	
20	541000	FOOTREST MOUNTING BRKT 1/2" RAD R	Right
20	541001	FOOTREST MOUNTING BRKT 1/2" RAD L	Left
21	106945	WASHER FLAT SAE 1/4 ROHS	
1,2,4,5	119518	PIVOT SADDLE KIT	
1-7	129644S	90 D FRNT MNT SWING IN/OUT HANGER R	Right
1-7	129645S	90 D FRNT MNT SWING IN/OUT HANGER L	Left
9-12	119514	LATCH RECEIVER KIT	
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right
13-21	129764	AAA FRNT MNT ADAPTER KIT PR	Pair

Pos.	Item Number	Description	Remarks
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

To have complete Footrest ship assembled, choose compatible Hanger + Footplate part numbers and add "ASSY-HNG-FP" to order

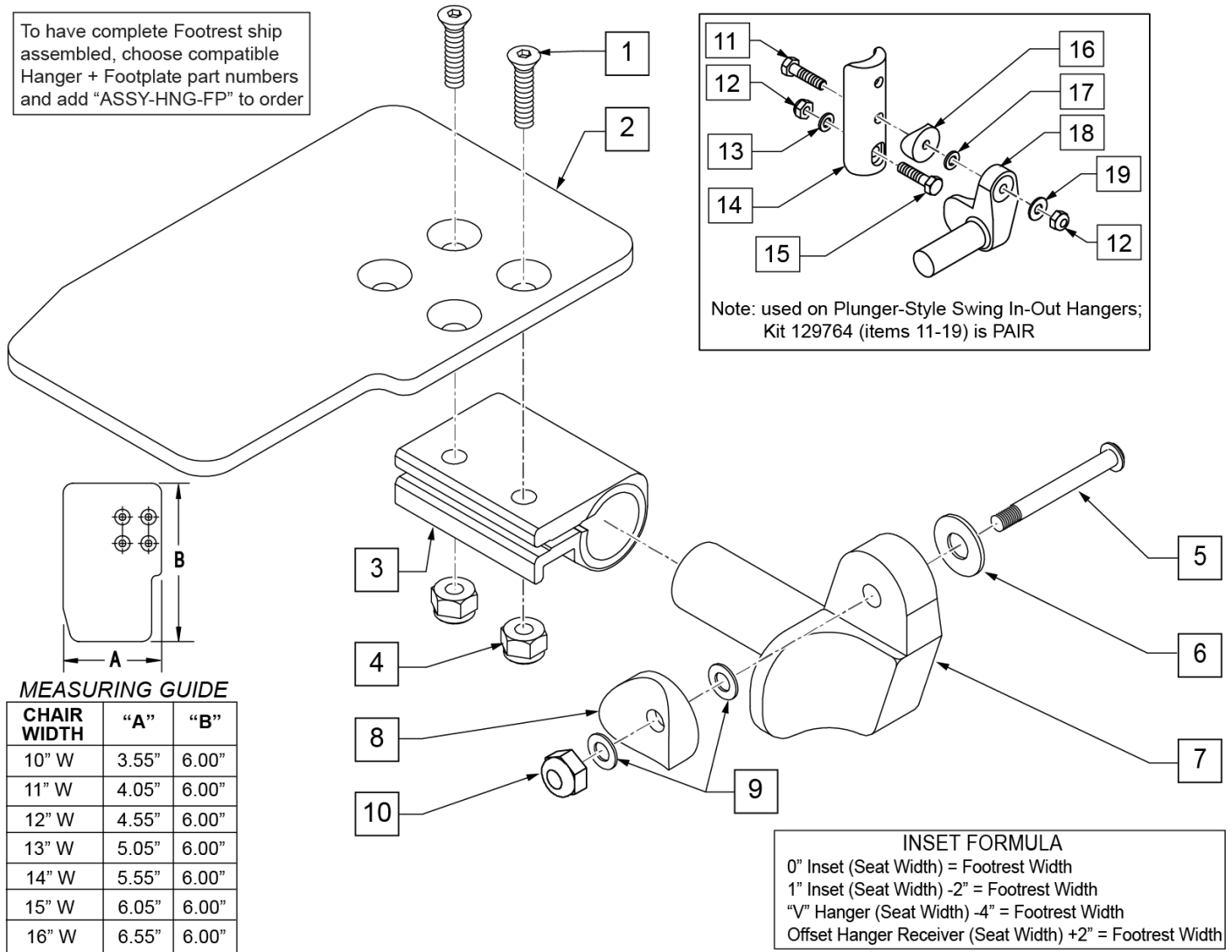


Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8 ROHS	
2	500635	KIDS ANGL ADJ F/PLT 10	RIGHT
2	500636	KIDS ANGL ADJ F/PLT 10	LEFT
2	500637	KIDS ANGL ADJ F/PLT 11	RIGHT
2	500638	KIDS ANGL ADJ F/PLT 11	LEFT
2	500639	KIDS ANGL ADJ F/PLT 12	RIGHT
2	500640	KIDS ANGL ADJ F/PLT 12	LEFT
2	500641	KIDS ANGL ADJ F/PLT 13	RIGHT
2	500642	KIDS ANGL ADJ F/PLT 13	LEFT
2	500643	KIDS ANGL ADJ F/PLT 14	RIGHT
2	500644	KIDS ANGL ADJ F/PLT 14	LEFT
2	500645	KIDS ANGL ADJ F/PLT 15	RIGHT
2	500646	KIDS ANGL ADJ F/PLT 15	LEFT
2	500647	KIDS ANGL ADJ F/PLT 16	RIGHT
2	500648	KIDS ANGL ADJ F/PLT 16	LEFT
3	550350	3/4 CLAMP ADJ FOOTPLATE	
4	106935	NUT NYLOCK THIN 1/4-28 ROHS	
5	226147	1/4-28 X 1.85 BH C/S BLK ZN	

Pos.	Item Number	Description	Remarks
6	106945	WASHER FLAT SAE 1/4 ROHS	
7	541000	FTRST MOUNTING BRKT 1/2" RAD R	RIGHT
7	541001	FTRST MOUNTING BRKT 1/2" RAD L	LEFT
8	561004	1 SADDLE 1/4 HOLE	
9	106943	WASHER FLAT THIN AN 1/4	
10	106941	NUT AUTOMATION 1/4-28 ROHS	
11	102268-001	HHCS M6 X 1 X 25	
12	132626	NUT AUTOMATION M6 X 1.0 ROHS	
13	107383	WASHER FLAT M6 ROHS	
15	118198	HHCS M6 X 1 X 40 PART NP	
16	561004	1" SADDLE 1/4" HOLE	
17	106943	WASHER FLAT AN 1/4 ROHS	
19	106945	WASHER FLAT SAE 1/4 ROHS	
1-4	158632	KAA F/PLT KIT 10 RT	RIGHT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158633	KAA F/PLT KIT 10 LT	LEFT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158634	KAA F/PLT KIT 11 RT	RIGHT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158635	KAA F/PLT KIT 11 LT	LEFT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158636	KAA F/PLT KIT 12 RT	RIGHT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158637	KAA F/PLT KIT 12 LT	LEFT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158638	KAA F/PLT KIT 13 RT	RIGHT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158639	KAA F/PLT KIT 13 LT	LEFT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158640	KAA F/PLT KIT 14 RT	RIGHT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158641	KAA F/PLT KIT 14 LT	LEFT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158642	KAA F/PLT KIT 15 RT	RIGHT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158643	KAA F/PLT KIT 15 LT	LEFT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158644	KAA F/PLT KIT 16 RT	RIGHT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-4	158645	KAA F/PLT KIT 16 LT	LEFT; used with Front Mount Adapter Kit on Plunger-Style Swing In/Out Hangers
1-10	828821	KIDS ANG ADJ F/PLT ASSM 10	RIGHT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828822	KIDS ANG ADJ F/PLT ASSM 10	LEFT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828823	KIDS ANG ADJ F/PLT ASSM 11	RIGHT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828824	KIDS ANG ADJ F/PLT ASSM 11	LEFT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828825	KIDS ANG ADJ F/PLT ASSM 12	RIGHT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828826	KIDS ANG ADJ F/PLT ASSM 12	LEFT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828827	KIDS ANG ADJ F/PLT ASSM 13	RIGHT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828828	KIDS ANG ADJ F/PLT ASSM 13	LEFT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828829	KIDS ANG ADJ F/PLT ASSM 14	RIGHT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828830	KIDS ANG ADJ F/PLT ASSM 14	LEFT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828831	KIDS ANG ADJ F/PLT ASSM 15	RIGHT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828832	KIDS ANG ADJ F/PLT ASSM 15	LEFT; used with Old-Style Latch Block Swing In/Out & S/A Hangers


Pos.	Item Number	Description	Remarks
1-10	828833	KIDS ANG ADJ F/PLT ASSM 16	RIGHT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	828834	KIDS ANG ADJ F/PLT ASSM 16	LEFT; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	921226	KIDS ANG ADJ F/PLT ASSM 10	PAIR; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	921227	KIDS ANG ADJ F/PLT ASSM 11	PAIR; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	921228	KIDS ANG ADJ F/PLT ASSM 12	PAIR; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	921229	KIDS ANG ADJ F/PLT ASSM 13	PAIR; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	921230	KIDS ANG ADJ F/PLT ASSM 14	PAIR; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	921231	KIDS ANG ADJ F/PLT ASSM 15	PAIR; used with Old-Style Latch Block Swing In/Out & S/A Hangers
1-10	921232	KIDS ANG ADJ F/PLT ASSM 16	PAIR; used with Old-Style Latch Block Swing In/Out & S/A Hangers
11-19	129764	AAA FRNT MNT ADAPTER KIT PR	AAA FRNT MNT fits both Kids and Adult Angle Adjustable Footplates
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

Note: for QX Series or SR45 footplate replacement parts see the ordering guide marked specifically for these two models in the informations panel.


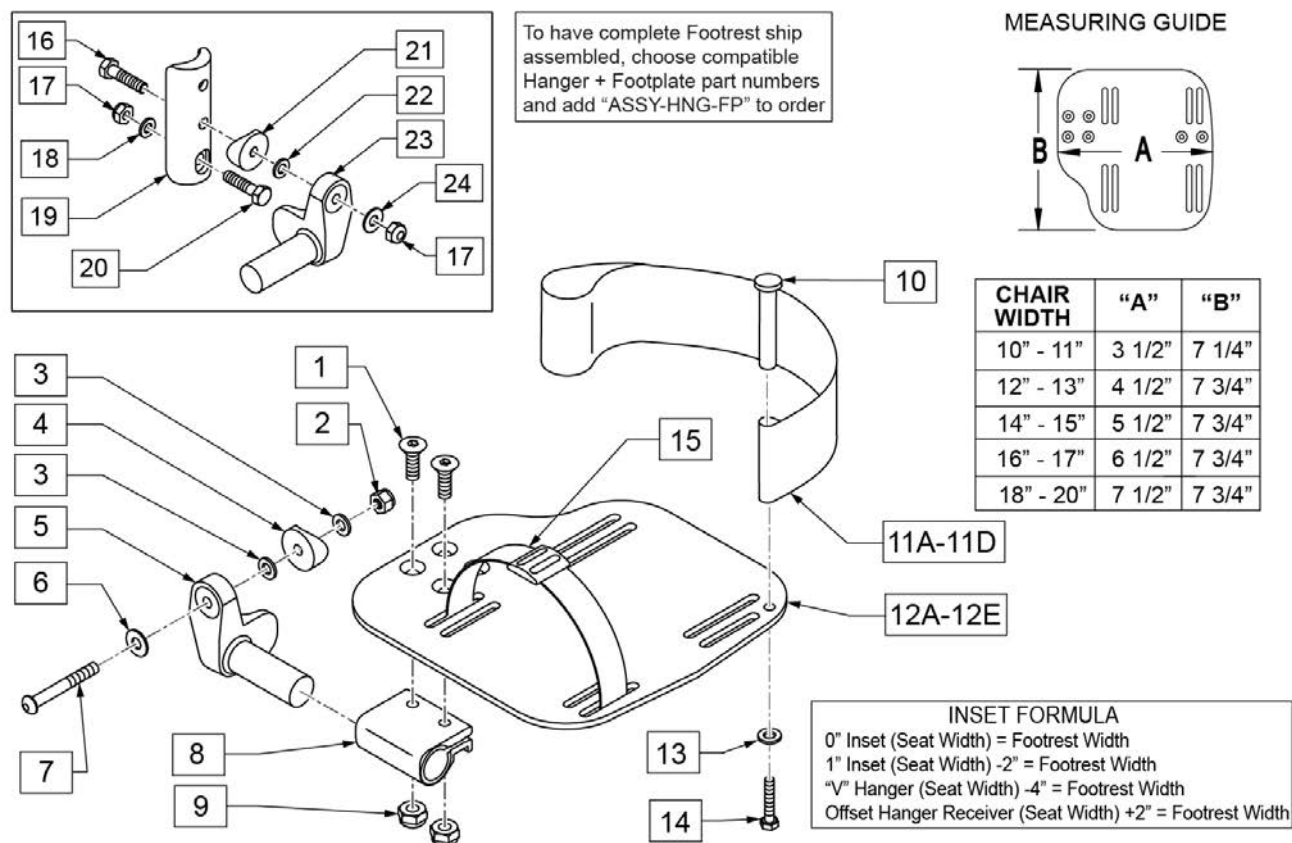


Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8	
2	106934	NUT NYLOCK 1/4-28	
3	106944	WASHER FLAT THIN AN 1/4	
4	561004	Q2 SADDLE	
5	541000	FTRST MOUNTING BRKT 1/2" RAD R	Right
5	541001	FTRST MOUNTING BRKT 1/2" RAD L	Left
6	106945	WASHER FLAT SAE 1/4 ROHS	
7	106964	BHCS 1/4-28 X 2	
8	550350	3/4 CLMP ADJ FT PLT	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	561040	HEEL LOOP PEG	
11A	104380-001	ADJ FOOTREST STRAP 14W	Used on 10"-13" wide
11B	104380-002	ADJ FOOTREST STRAP 15-16W	Used on 14"-15" wide
11C	104380-003	ADJ FOOTREST STRAP 17-18W	Used on 16"-17" wide
11D	104380-004	ADJ FOOTREST STRAP 19-20W	Used on 18"-20" wide
12A	500656	ADULT ANGL ADJ F/PLT 10/11	Right; 10"-11" wide
12A	500657	ADULT ANGL ADJ F/PLT 10/11	Left; 10"-11" wide

Pos.	Item Number	Description	Remarks
12B	500658	ADULT ANGL ADJ F/PLT 12/13	Right; 12"-13" wide
12B	500659	ADULT ANGL ADJ F/PLT 12/13	Left; 12"-13" wide
12C	500660	ADULT ANGL ADJ F/PLT 14/15	Right; 14"-15" wide
12C	500661	ADULT ANGL ADJ F/PLT 14/15	Left; 14"-15" wide
12D	500662	ADULT ANGL ADJ F/PLT 16/17	Right; 16"-17" wide
12D	500663	ADULT ANGL ADJ F/PLT 16/17	Left; 16"-17" wide
12E	500664	ADULT ANGL ADJ F/PLT 18-20	Right; 18"-22" wide
12E	500665	ADULT ANGL ADJ F/PLT 18-20	Left; 18"-22" wide
13	107383	WASHER FLAT M6 ROHS	
14	102268-001	HHCS M6 X 1 X 25	
15	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
16	102268-001	HHCS M6 X 1 X 25	
17	132626	NUT AUTOMATION M6 X 1	
18	107383	WASHER FLAT M6 ROHS	
20	118198	HHCS M6X1X40 PART NP	
21	561004	Q2 SADDLE	
22	106943	WASHER FLAT AN 1/4 ROHS	
24	106945	WASHER FLAT SAE 1/4 ROHS	
16-24	129764	AAA FRNT MNT ADAPTER KIT PR	Pair; fits both Kids & Adult Angle Adj Footplates
1,8-10,11A,12A,13,14	129765	AAA F/PLT KIT 10-11 RT	Right; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1,8-10,11A,12A,13,14	129766	AAA F/PLT KIT 10-11 LFT	Left; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1,8-10,11A,12B,13,14	129767	AAA F/PLT KIT 12-13 RT	Right; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1,8-10,11A,12B,13,14	129768	AAA F/PLT KIT 12-13 LFT	Left; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1,8-10,11B,12C,13,14	129769	AAA F/PLT KIT 14-15 RT	Right; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1,8-10,11B,12C,13,14	129770	AAA F/PLT KIT 14-15 LFT	Left; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1,8-10,11C,12D,13,14	129771	AAA F/PLT KIT 16-17 RT	Right; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1,8-10,11C,12D,13,14	129772	AAA F/PLT KIT 16-17 LFT	Left; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1,8-10,11D,12E,13,14	129773	AAA F/PLT KIT 18-20 RT	Right; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1,8-10,11D,12E,13,14	129774	AAA F/PLT KIT 18-20 LFT	Left; used w/ 129764 on Plunger-Style Swing In/Out Hangers
1-10,11A,12B	111514	AAA FTPLT 90D 12/13" R	Right; used w/ Old-Style Latch Block Swing In/Out & S/A Hngrs
1-10,11A,12B	111515	AAA FTPLT 90D 12/13" L	Left; used w/ Old-Style Latch Block Swing IO & S/A Hngrs
1-10,11B,12C	111516	AAA FTPLT 90D 14/15" R	Right; used w/ Old-Style Latch Block Swing In/Out & S/A Hngrs
1-10,11B,12C	111517	AAA FTPLT 90D 14/15" L	Left; used w/ Old-Style Latch Block Swing IO & S/A Hngrs
1-10,11C,12D	111518	AAA FTPLT 90D 16/17" R	Right; used w/ Old-Style Latch Block Swing In/Out & S/A Hngrs
1-10,11C,12D	111519	AAA FTPLT 90D 16/17" L	Left; used w/ Old-Style Latch Block Swing IO & S/A Hngrs
1-10,11D,12E	111520	AAA FTPLT 90D 18/20" R	Right; used w/ Old-Style Latch Block Swing In/Out & S/A Hngrs
1-10,11D,12E	111521	AAA FTPLT 90D 18/20" L	Left; used w/ Old-Style Latch Block Swing IO & S/A Hngrs
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

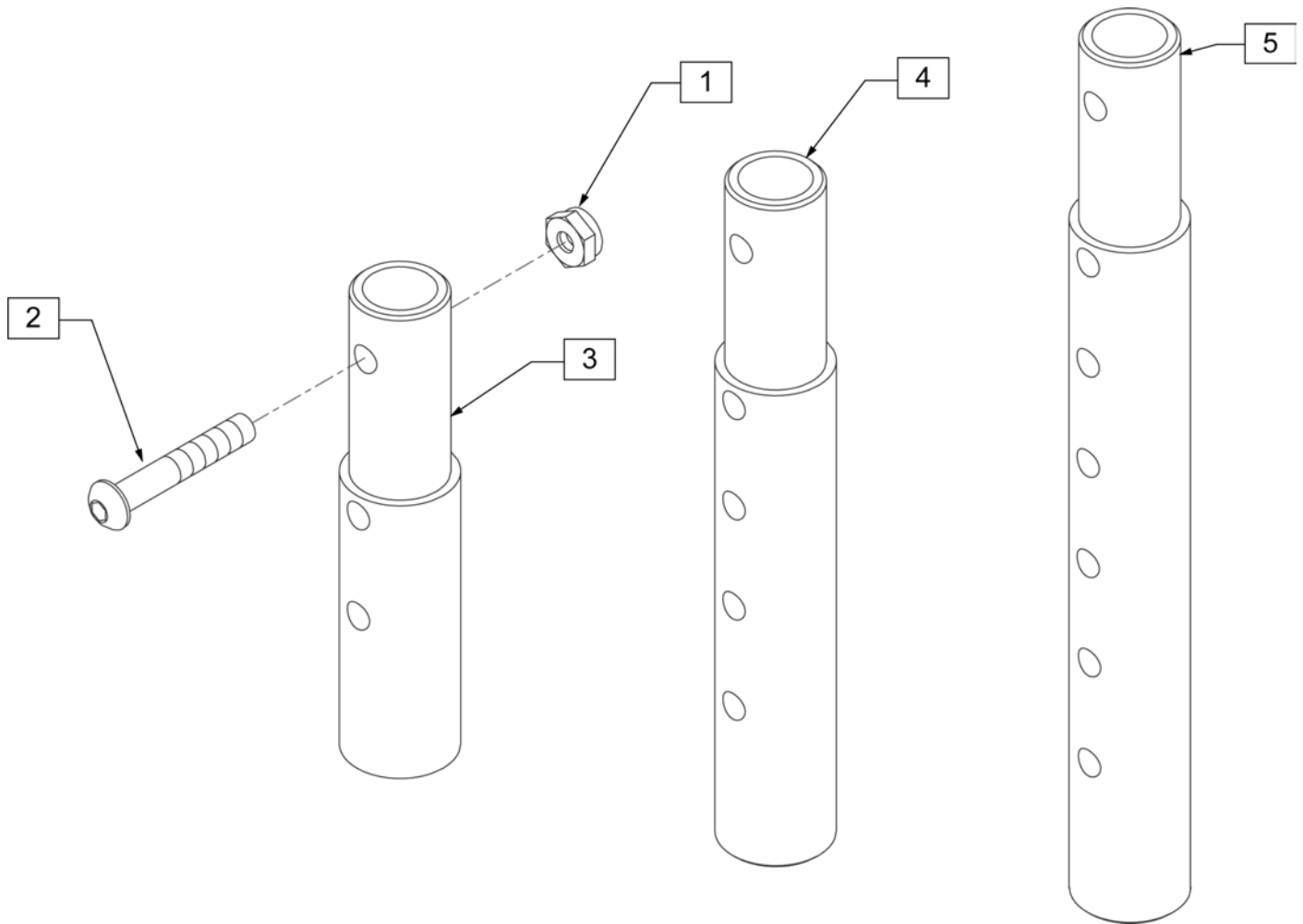

Pos.	Item Number	Description	Remarks
1	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
2	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
1-3	111505	2" FOOTREST EXTENSION TUBE W/ HDWR ASSM	2" EXT (PAIR)
1, 2, 4	111506	4" FOOTREST EXTENSION TUBE W/ HDWR ASSM	4" EXT (PAIR)
1, 2, 5	111507	6" FOOTREST EXTENSION TUBE W/ HDWR ASSM	6" EXT (PAIR)

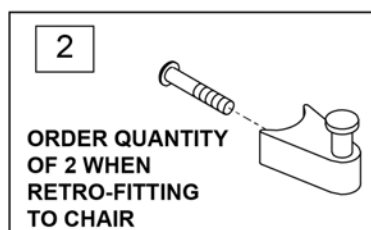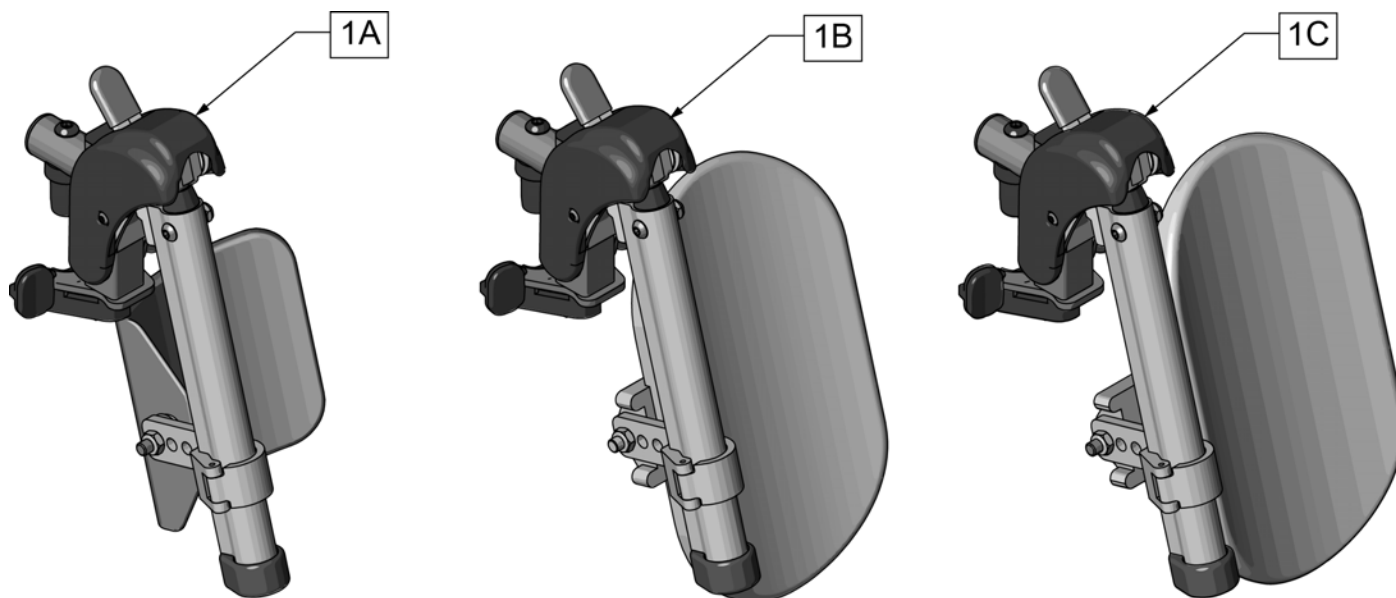

Pos.	Item Number	Description	Remarks
1A	131774S	HEMI ELR HNGR ASM 14-15 W/PAD	
1B	131775S	HEMI ELR HNGR ASM 16-17 W/PAD	
1C	131776S	HEMI ELR HNGR ASM 18-20 W/PAD	
2	107491	LATCH STUD KIT	order qty 2 when retro-fitting to chair
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

1

ORDER ITEM 26 WHEN RETRO-FITTING TO CHAIR

26

ORDER QUANTITY 2 WHEN RETRO-FITTING TO CHAIR

To request assembly of parts being ordered as a kit, please add "ASSY-HNG-FP" after ELR & Calf Pad parts are selected

2

25

3

4

5

6

7

8

9

10

11

12

13

14

15

16

17

18

19

20

21


22

23


24

Pos.	Item Number	Description	Remarks
1	129638S	EXTENSION MOUNT ELR KIT R	Right-order item 26 when retro-fitting to chair
1	129639S	EXTENSION MOUNT ELR KIT L	Left-order item 26 when retro-fitting to chair
1	129637S	EXTENSION MOUNT ELR PAIR KIT	Pair-order item 26 when retro-fitting to chair
2,3,25	129640	ELR SHROUD KIT	Kit w/ 2 screws
3	127135	BHCS 8-32 X 1/2 BLK	
4	126992	HEMI ELR SUB ASSM	
5	560501	ALR REL LEVER COVER	
6	127114	HEMI RECEIVER ASSM ELR ALR R	Right
6	127115	HEMI RECEIVER ASSM ELR ALR L	Left
7	107350	SHCS 1/4-28 x 1/2	
8	127129	ANGLE STOP ELR	
9	107249	NUT HEX THIN 1/4-28	
10	106935	1/4-28 NYLOCK NUT THIN BLK ZN	
11	104414	WAVE WASHER 6 MM	
12	106961	BHCS 1/4 -28 x 1 1/4 ROHS	


Pos.	Item Number	Description	Remarks
13	108623	BHCS 1/4-28 x 1-1/2	
14	119495	EXTENSION CLAMP KIT	
15	010615	SQUARE END CAP CNTR LEG	
16	226217	5/16-18 X 2" BH C/S BLK	
17	561062	1" INSERT PLUG .065 RIBS	
18	226057	BHCS 10-24 X .725 PL	
20	561197	PLASTIC PIVOT SADDLE	
21	202102	5/16-18 NYLOCK NUT THIN	
23,24	881500	LATCH BLOCK ASSM R	Right
23,24	881501	LATCH BLOCK ASSM L	Left
25	113282	WASHER FLAT AN #10 ROHS	
18,19,22-24	881521	S/A LATCH KIT CASTING RIGHT	Right
18,19,22-24	881522	S/A LATCH KIT CASTING LEFT	Left
26	107491	LATCH STUD KIT	order qty 2 when retro-fitting to chair
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

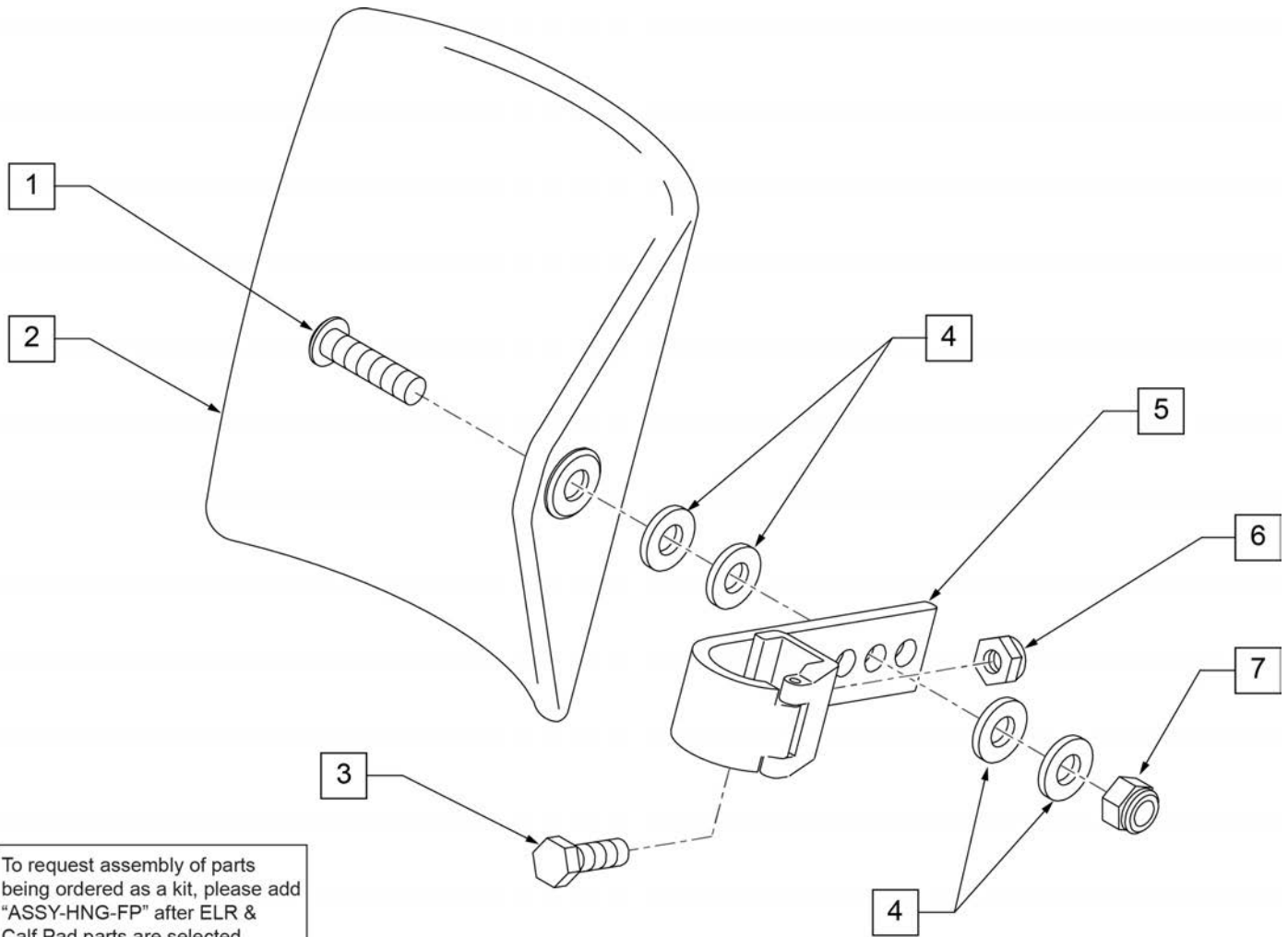


Pos.	Item Number	Description	Remarks
1	106980	BHCS 5/16-18 X 1-1/4 ROHS	
2	755014	CALF PAD TROUGH STYLE 14"/16"	
3	221097	1/4-28 X 5/8 HEX C/S BZ	
4	561016	1" WASHER 5/16" HOLE	
5	881413	CALF PAD BRACKET ASSEMBLY	
6	106934	NUT NYLOCK 1/4-28 ROHS	
7	106937	NUT NYLOCK THIN 5/16-18	
1-7	881414	CALF PAD J-HOOK 14"-15" ASSEMBLY	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

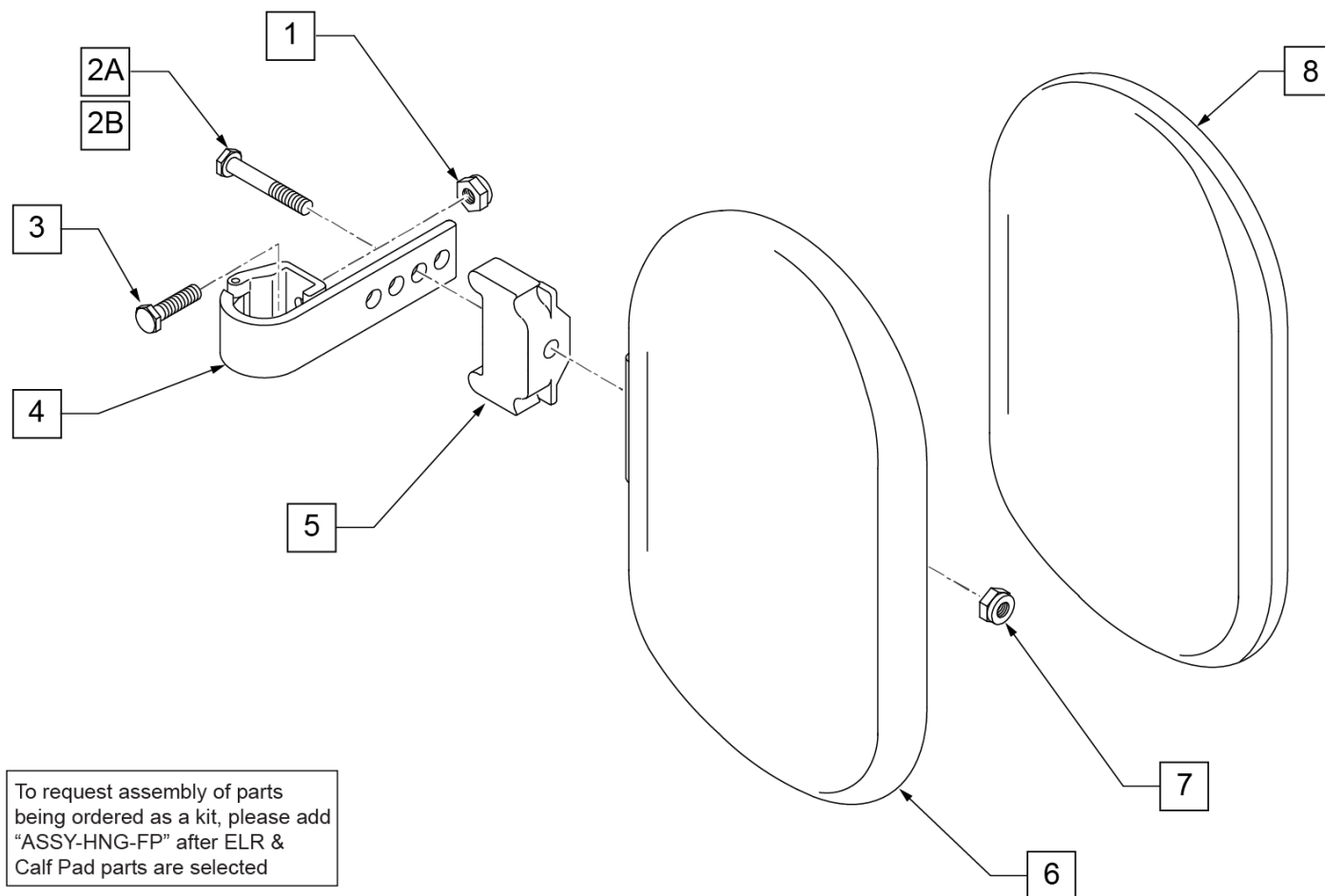

Pos.	Item Number	Description	Remarks
1	106934	NUT NYLOCK 1/4-28 ROHS	
2A	107018	HHCS 5/16-18 X 2-1/2 ROHS	16"-17" wide only
2B	107019	HHCS 5/16-18 X 3-1/2 ROHS	18"-20" wide only
3	107013	HHCS 1/4-28 X 5/8 ROHS	
4	881413	CALF PAD BRACKET ASSEMBLY	
5	560199	ELR CALF HINGE SPACER	18"-20" wide only
6	560108	CALF PAD MOLDED 16"-20"	
7	106936	NUT NYLOCK 5/16-18 ROHS	
8	751045	CALF PAD COVER	
1,2A,3,4,6-8	881416	CALF MOLDED 16"-17" ASSEMBLY	
1,2B,3-7,8	881412	CALF PAD MOLDED 18"-20" ASSEMBLY	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

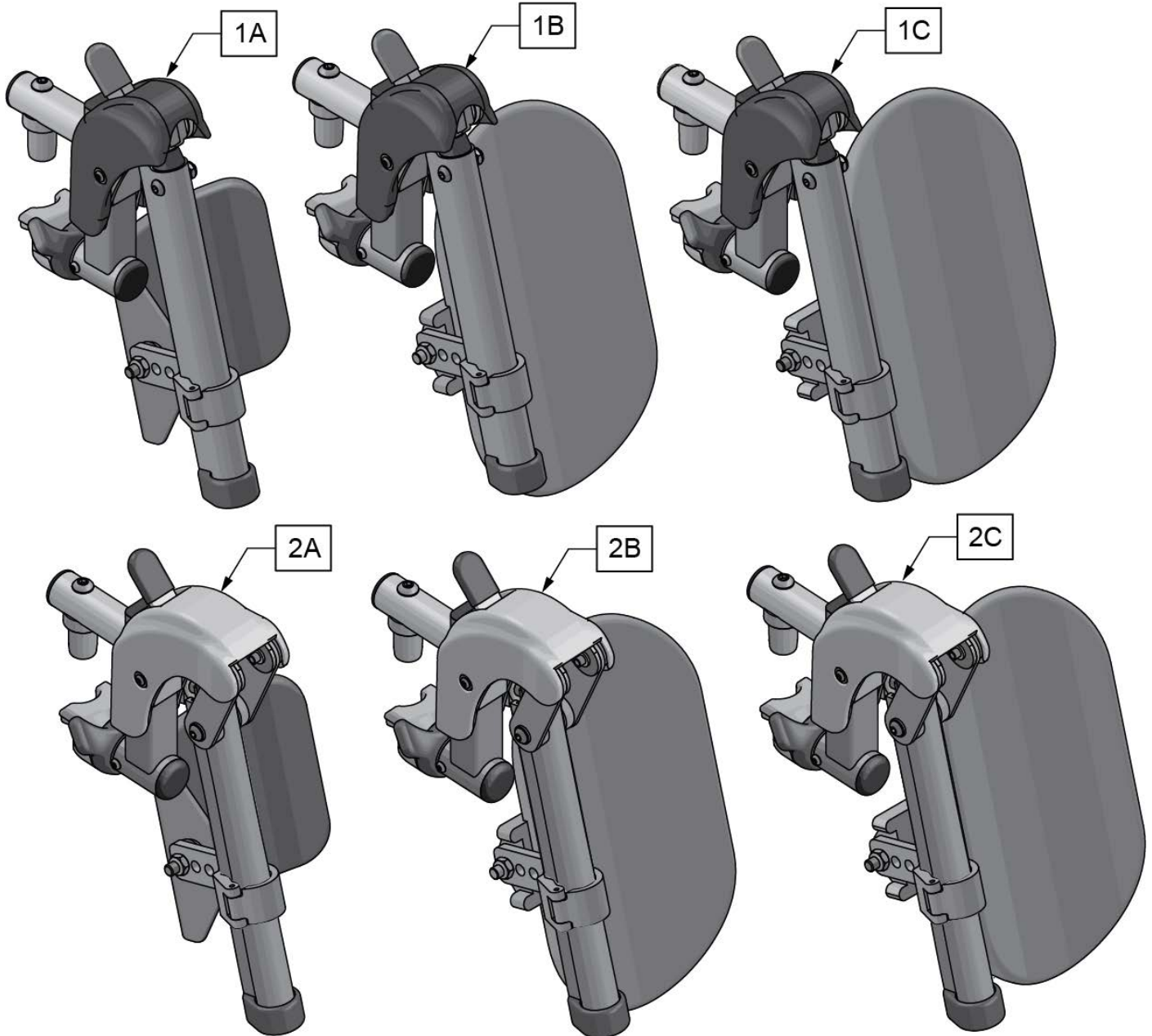


Pos.	Item Number	Description	Remarks
1A	161233S	SWNG I/O ELR ASM 14-15 W/PAD	Pair
1B	161234S	SWNG I/O ELR ASM 16-17 W/PAD	Pair
1C	161235S	SWNG I/O ELR ASM 18-20 W/PAD	Pair
2A	161236S	SWNG I/O ALR ASM 14-15 W/PAD	Pair
2B	161237S	SWNG I/O ALR ASM 16-17 W/PAD	Pair
2C	161238S	SWNG I/O ALR ASM 18-20 W/PAD	Pair
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

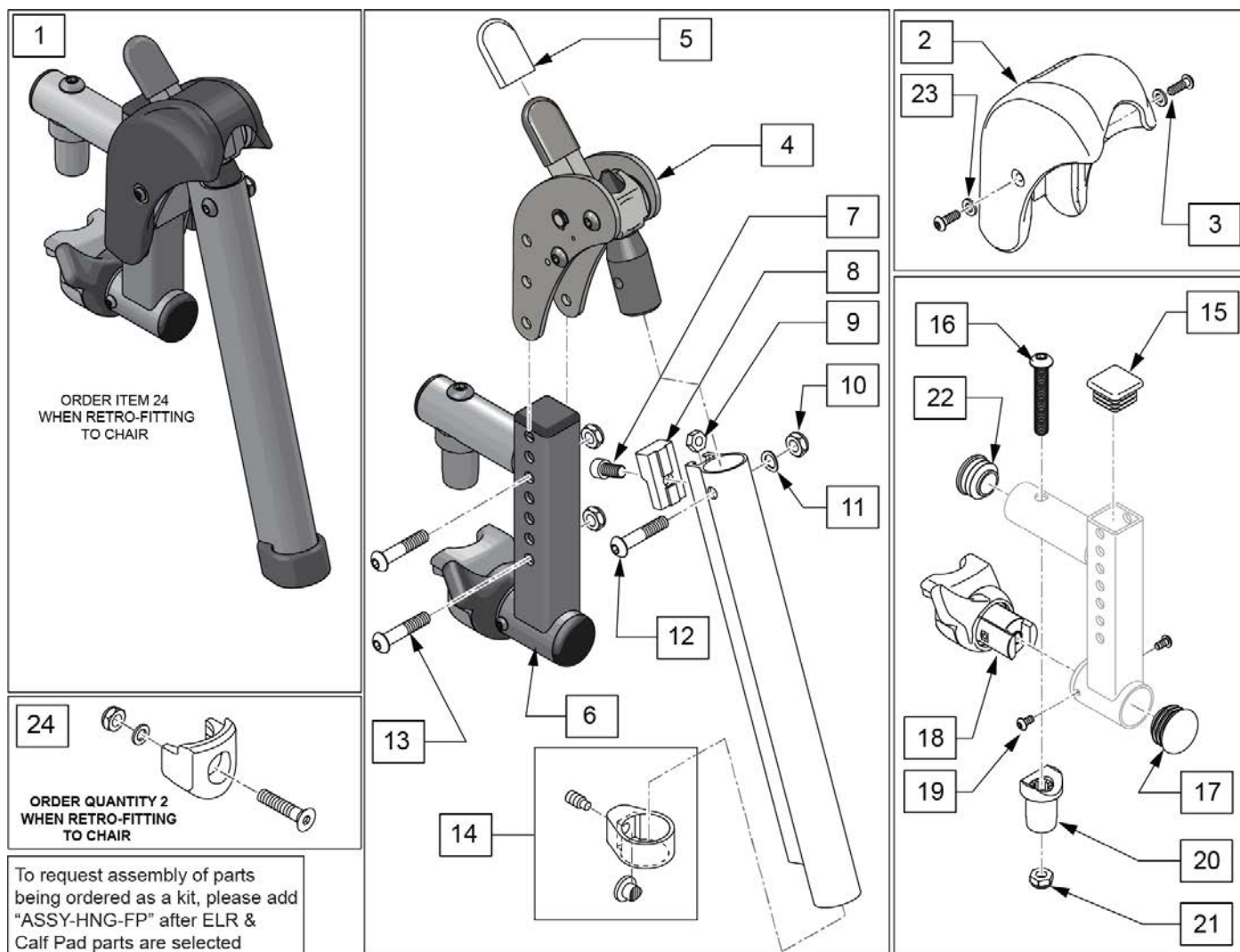


Pos.	Item Number	Description	Remarks
1	161239S	SWNG I/O ELR ASSM RH	Right
1	161240S	SWNG I/O ELR ASSM LH	Left
1	161241S	SWNG I/O ELR ASSM PR	Pair
2,3,23	129640	ELR SHROUD KIT	Kit w/ 2 screws
3	127135	BHCS 8-32 X 1/2 BLK	
4	126992	HEMI ELR SUB ASSM	
5	560501	ALR REL LEVER COVER	
6	134188	RECEIVER ASSY ELR ALR RH	
6	134186	RECEIVER ASSY ELR ALR LH	
7	107350	SHCS 1/4-28 x 1/2	
8	127129	ANGLE STOP ELR	
9	107249	NUT HEX THIN 1/4-28	
10	106935	1/4-28 NYLOCK NUT THIN BLK ZN	
11	104414	WAVE WASHER 6 MM	
12	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
13	108623	BHCS 1/4-28 x 1-1/2	
14	119495	EXTENSION CLAMP KIT	
15	010615	SQUARE END CAP CNTR LEG	
16	226217	5/16-18 X 2" BH C/S BLK	

Pos.	Item Number	Description	Remarks
17	561062	1" INSERT PLUG .065 RIBS	
18,19	161242	ELR/ALR HANGER LATCH KIT RH	Right
18,19	161243	ELR/ALR HANGER LATCH KIT LH	Left
20	561197	PLASTIC PIVOT SADDLE	
21	202102	5/16-18 NYLOCK NUT THIN	
22	560110	ENDPLUG BACKREST TNT	
23	113282	WASHER FLAT AN #10 ROHS	
24	119514	LATCH RECEIVER KIT	order qty 2 when retro-fitting to chair
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

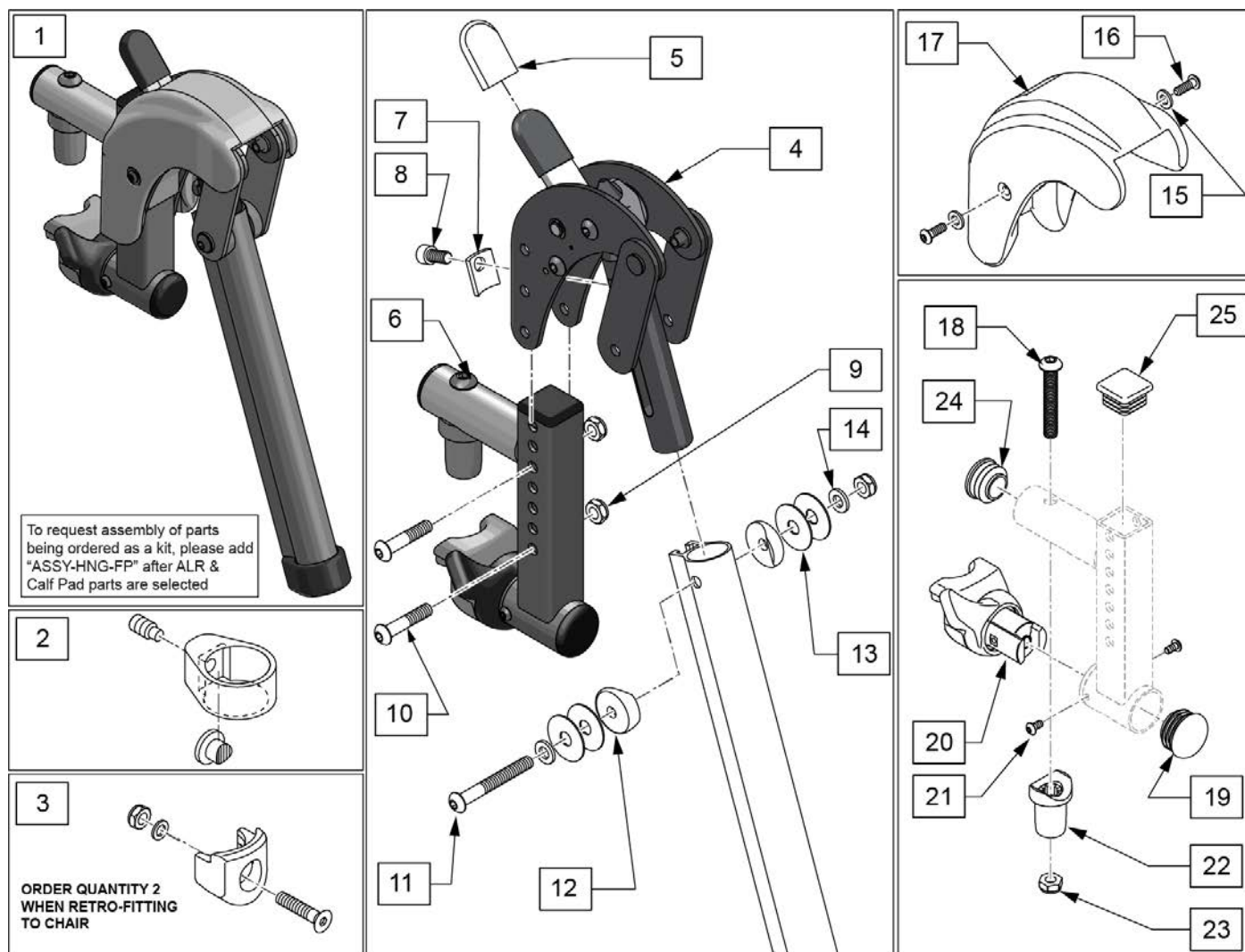


Pos.	Item Number	Description	Remarks
1	161244S	SWNG I/O ALR ASSM RH KIT	Right
1	161245S	SWNG I/O ALR ASSM LH KIT	Left
1	161246S	SWNG I/O ALR ASSM PR KIT	Pair
2	119495	EXTENSION CLAMP KIT	
3	119514	LATCH RECEIVER KIT	order qty 2 when retro-fitting to chair
4	127153	HEMI ALR SUB ASSM	
5	560501	ALR REL LEVER COVER	
6	134188	RECEIVER ASSY ELR ALR RH	
6	134186	RECEIVER ASSY ELR ALR LH	
7	127129	ANGLE STOP ELR	
8	107350	SHCS 1/4-28 x 1/2	
9	106935	1/4-28 NYLOCK NUT THIN BLK ZN	
10	108623	BHCS 1/4-28 x 1-1/2	
11	106964	BHCS 1/4-28 X 2	
12	561086	1" HALF SADDLE 1/4" HOLE	
13	113268	WASHER NYLON 1IN OD X .312 ID	
14	106943	WASHER FLAT AN 1/4 ROHS	
15	113282	WASHER FLAT AN #10 ROHS	
16	127135	BHCS 8-32 X 1/2 BLK	

Pos.	Item Number	Description	Remarks
15-17	129633	ALR SHROUD KIT	1 side
18	226217	5/16-18 X 2" BH C/S BLK	
19	560110	ENDPLUG BACKREST TNT	
20,21	161242	ELR/ALR HANGER LATCH KIT RH	Right
20,21	161243	ELR/ALR HANGER LATCH KIT LH	Left
22	561197	PLASTIC PIVOT SADDLE	
23	202102	5/16-18 NYLOCK NUT THIN	
24	561062	1" INSERT PLUG .065 RIBS	
25	010615	SQUARE END CAP CNTR LEG	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

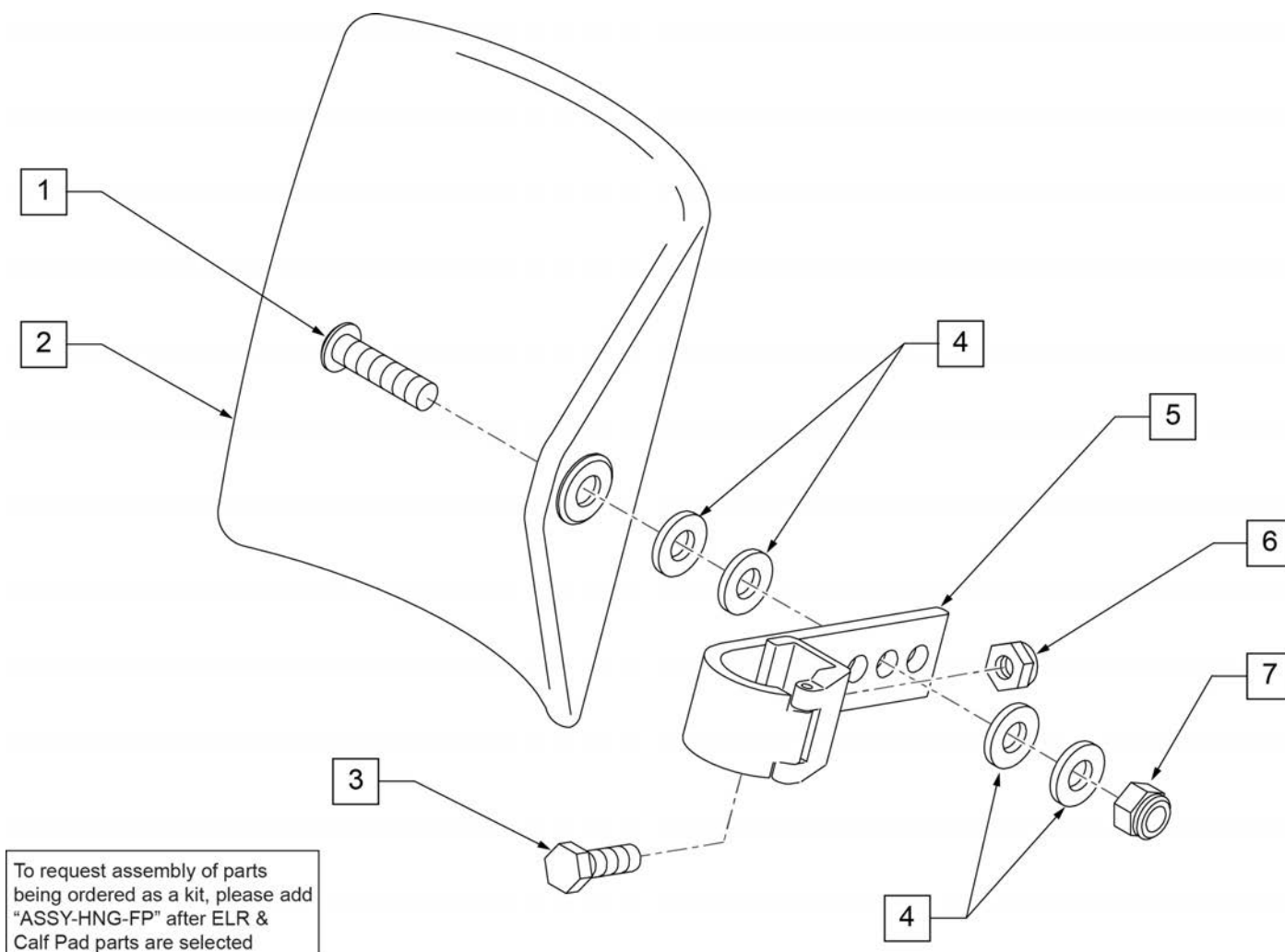

Pos.	Item Number	Description	Remarks
1	106980	BHCS 5/16-18 X 1-1/4 ROHS	
2	755014	CALF PAD TROUGH STYLE 14"/16"	
3	221097	1/4-28 X 5/8 HEX C/S BZ	
4	561016	1" WASHER 5/16" HOLE	
5	881413	CALF PAD BRACKET ASSEMBLY	
6	106934	NUT NYLOCK 1/4-28 ROHS	
7	106937	NUT NYLOCK THIN 5/16-18	
1-7	881414	CALF PAD J-HOOK 14"-15" ASSEMBLY	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

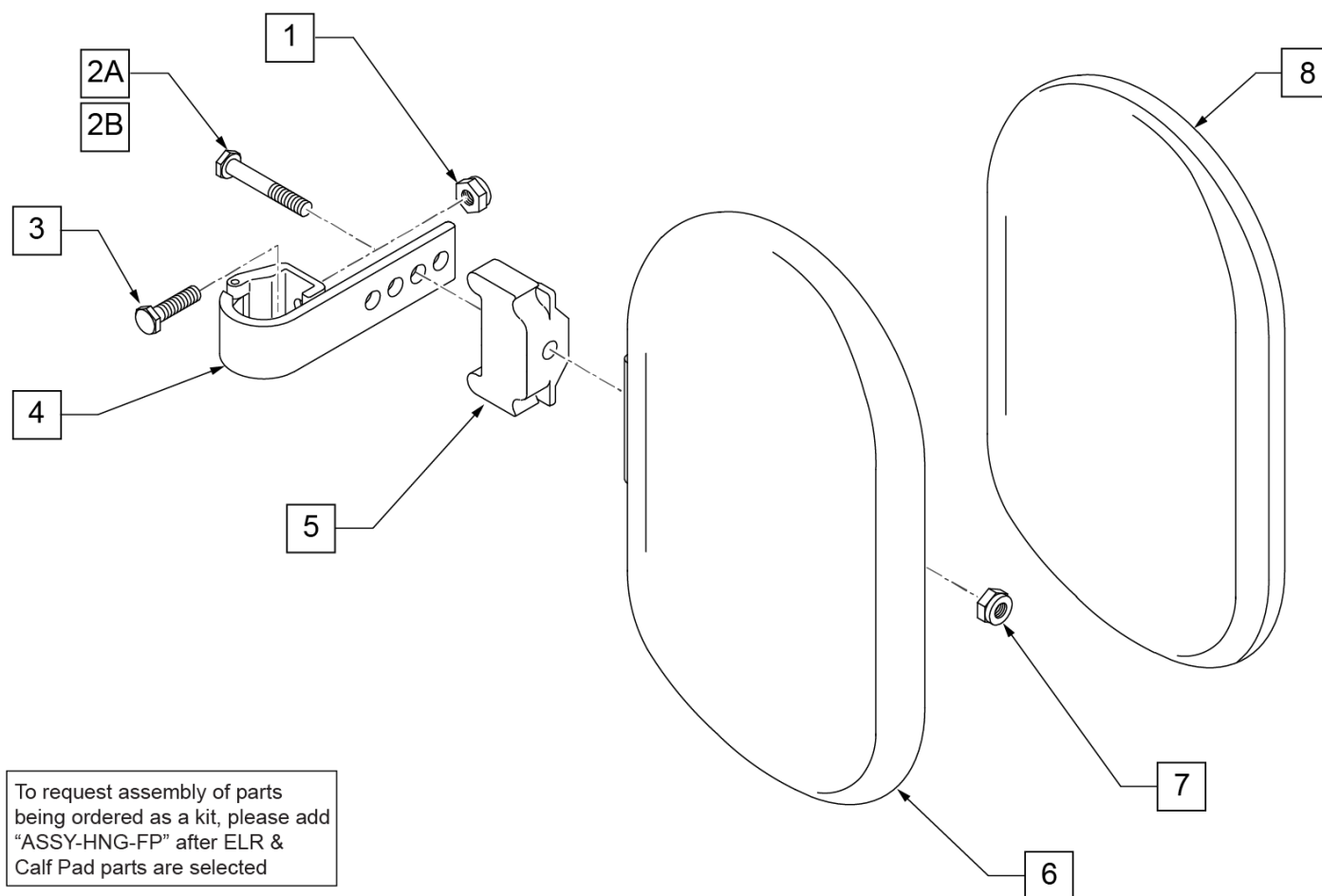


Pos.	Item Number	Description	Remarks
1	106934	NUT NYLOCK 1/4-28 ROHS	
2A	107018	HHCS 5/16-18 X 2-1/2 ROHS	16"-17" wide only
2B	107019	HHCS 5/16-18 X 3-1/2 ROHS	18"-20" wide only
3	107013	HHCS 1/4-28 X 5/8 ROHS	
4	881413	CALF PAD BRACKET ASSEMBLY	
5	560199	ELR CALF HINGE SPACER	18"-20" wide only
6	560108	CALF PAD MOLDED 16"-20"	
7	106936	NUT NYLOCK 5/16-18 ROHS	
8	751045	CALF PAD COVER	
1,2A,3,4,6-8	881416	CALF MOLDED 16"-17" ASSEMBLY	
1,2B,3-7,8	881412	CALF PAD MOLDED 18"-20" ASSEMBLY	
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

Note: Footplate Assemblies are set to footrest width; please see "Footplate Formula" table for seat width conversion.

Note: If using Short Ext Tube & Hemi Hanger with Composite Footplate use 1G (bent) .

Note: V Style Hangers are not available on Swing Away Style chairs effective 7/14/14; see the "Prior To" ordering guide for V Style footplate assemblies.

Note: see Informations Panel for appropriate ordering guide; for QX Series or SR45 footplate replacement parts see ordering guide marked specifically for these two models.

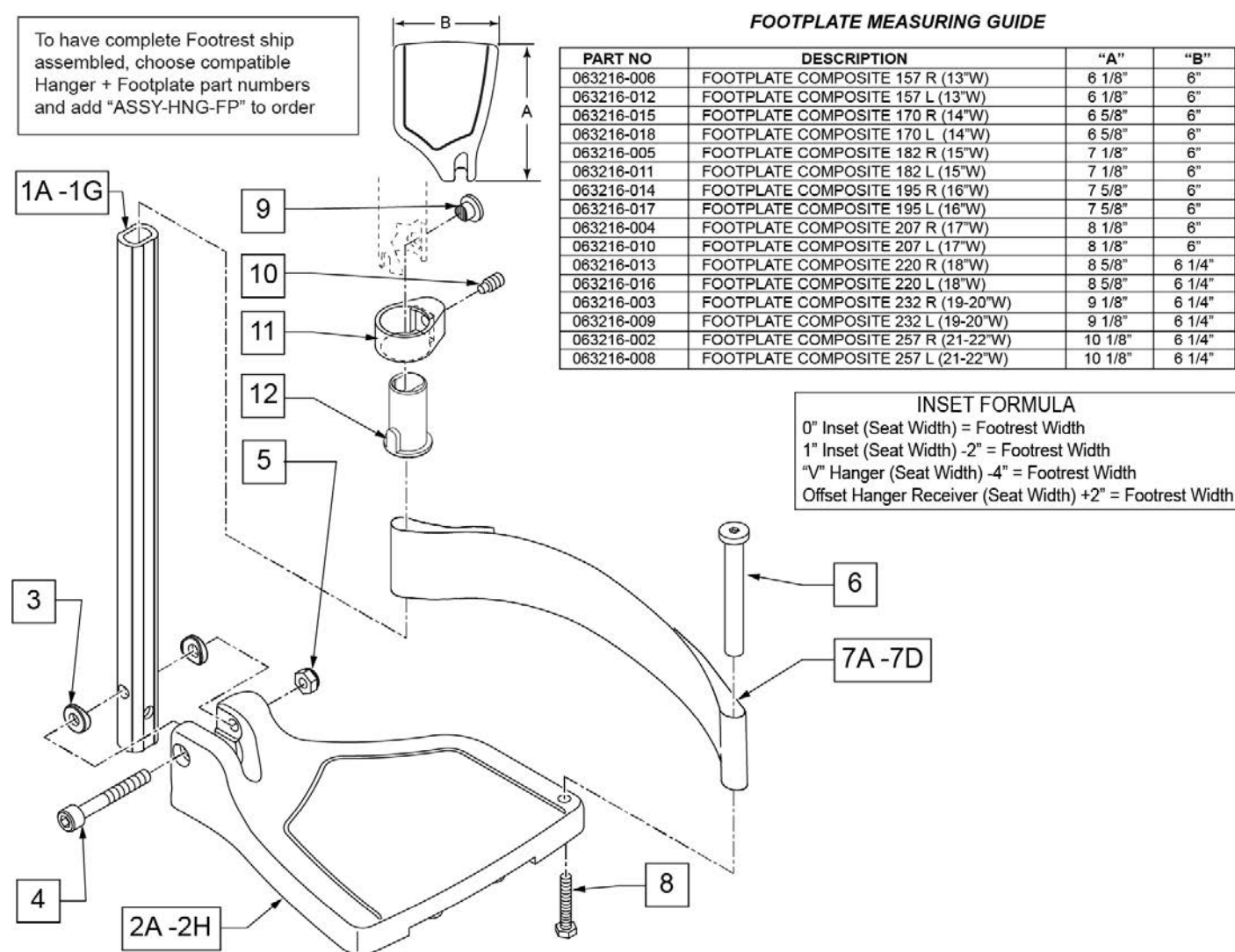

Pos.	Item Number	Description	Remarks
1A,3-5	120660	SHT STRT EXT COMP HDWR	80 deg hangers + ELR only

Pos.	Item Number	Description	Remarks
1B,3-5	120661	MED STRT EXT COMP HDWR	80 deg hangers + ELR only
1C,3-5	120662	LNG STRT EXT COMP HDWR	80 deg hangers + ELR only
1D,3-5	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E,3-5	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F,3-5	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G	128078	TUBE FOOTREST EXT SHT HEMI	use for Hemi Ext Tube option
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 13" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 13" Footrest Width
2B	063216-015	FOOTPLATE COMPOSITE RIGHT	RIGHT 14" Footrest Width
2B	063216-018	FOOTPLATE COMPOSITE LEFT	LEFT 14" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2D	063216-014	FOOTPLATE COMPOSITE RIGHT	RIGHT 16" Footrest Width
2D	063216-017	FOOTPLATE COMPOSITE LEFT	LEFT 16" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2F	063216-013	FOOTPLATE COMPOSITE RIGHT	RIGHT 18" Footrest Width
2F	063216-016	FOOTPLATE COMPOSITE LEFT	LEFT 18" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" & 20" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 19" & 20" Footrest Width
2H	063216-002	FOOTPLATE COMPOSITE RIGHT	RIGHT 21" & 22" Footrest Width
2H	063216-008	FOOTPLATE COMPOSITE LEFT	LEFT 21" & 22" Footrest Width
3	104377	3/4" SADDLE THIN	
4	103175	SHCS M6 X 1 X 40 INORG	
5	132626	NUT AUTOMATION M6 X 1.0	Inserted in bottom of footplate.
6	104328	HEEL LOOP PEG	
7A	104380-001	ADJUSTABLE FOOT REST STRAP	14" wide
7B	104380-002	ADJUSTABLE FOOT REST STRAP	15-16" wide
7C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" wide
7D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" wide
8	102268-001	HHCS M6 X 1 X 25	
12	104160	BUSHING FOOTREST	
9-12	107485	FOOTREST CLAMP ASSM	
>>>>		80 DEGREE HANGERS - SHORT EXT - RIGHT	
1A,2A,3-6,7A,8	107704	COMP STRT SHT EXT ASM 13"W R	Right- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107705	COMP STRT SHT EXT ASM 14"W R	Right- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107706	COMP STRT SHT EXT ASM 15"W R	Right- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107707	COMP STRT SHT EXT ASM 16"W R	Right- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107708	COMP STRT SHT EXT ASM 17"W R	Right- 80D- 17" W- SHORT
1A,2F,3-6,7C,8	107709	COMP STRT SHT EXT ASM 18"W R	Right- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107710	COMP STRT SHT EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111522	COMP STRT SHT EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - SHORT EXT - LEFT	
1A,2A,3-6,7A,8	107711	COMP STRT SHT EXT ASM 13"W L	LEFT- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107712	COMP STRT SHT EXT ASM 14"W L	LEFT- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107713	COMP STRT SHT EXT ASM 15"W L	LEFT- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107714	COMP STRT SHT EXT ASM 16"W L	LEFT- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107715	COMP STRT SHT EXT ASM 17"W L	LEFT- 80D- 17" W- SHORT
1A,2F,3-6,7C,8	107716	COMP STRT SHT EXT ASM 18"W L	LEFT- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107717	COMP STRT SHT EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111523	COMP STRT SHT EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - MED EXT - RIGHT	
1B,2A,3-6,7A,8	107718	COMP STRT MED EXT ASM 13"W R	Right- 80D- 13" W- MED
1B,2B,3-6,7A,8	107719	COMP STRT MED EXT ASM 14"W R	Right- 80D- 14" W- MED
1B,2C,3-6,7B,8	107720	COMP STRT MED EXT ASM 15"W R	Right- 80D- 15" W- MED
1B,2D,3-6,7B,8	107721	COMP STRT MED EXT ASM 16"W R	Right- 80D- 16" W- MED
1B,2E,3-6,7C,8	107722	COMP STRT MED EXT ASM 17"W R	Right- 80D- 17" W- MED

Pos.	Item Number	Description	Remarks
1B,2F,3-6,7C,8	107723	COMP STRT MED EXT ASM 18"W R	Right- 80D- 18" W- MED
1B,2G,3-6,7D,8	107724	COMP STRT MED EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111524	COMP STRT MED EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - MED EXT - LEFT	
1B,2A,3-6,7A,8	107725	COMP STRT MED EXT ASM 13"W L	LEFT- 80D- 13" W- MED
1B,2B,3-6,7A,8	107726	COMP STRT MED EXT ASM 14"W L	LEFT- 80D- 14" W- MED
1B,2C,3-6,7B,8	107727	COMP STRT MED EXT ASM 15"W L	LEFT- 80D- 15" W- MED
1B,2D,3-6,7B,8	107728	COMP STRT MED EXT ASM 16"W L	LEFT- 80D- 16" W- MED
1B,2E,3-6,7C,8	107729	COMP STRT MED EXT ASM 17"W L	LEFT- 80D- 17" W- MED
1B,2F,3-6,7C,8	107730	COMP STRT MED EXT ASM 18"W L	LEFT- 80D- 18" W- MED
1B,2G,3-6,7D,8	107731	COMP STRT MED EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111525	COMP STRT MED EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - LONG EXT - RIGHT	
1C,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	Right- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107733	COMP STRT LONG EXT ASM 14"W R	Right- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	Right- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107735	COMP STRT LONG EXT ASM 16"W R	Right- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	Right- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107737	COMP STRT LONG EXT ASM 18"W R	Right- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111526	COMP STRT LONG EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- LONG
>>>>		80 DEGREE HANGERS - LONG EXT - LEFT	
1C,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	LEFT- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107740	COMP STRT LONG EXT ASM 14"W L	LEFT- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	LEFT- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107742	COMP STRT LONG EXT ASM 16"W L	LEFT- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	LEFT- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107744	COMP STRT LONG EXT ASM 18"W L	LEFT- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111527	COMP STRT LONG EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- LONG
>>>>		60°,70° SHORT EXT - RIGHT	
1D,2A,3-6,7A,8	107746	COMP BNT SHT EXT ASM 13" W R	Right- 70D- 13" W- SHORT
1D,2B,3-6,7A,8	107747	COMP BNT SHT EXT ASM 14" W R	Right- 70D- 14" W- SHORT
1D,2C,3-6,7B,8	107748	COMP BNT SHT EXT ASM 15" W R	Right- 70D- 15" W- SHORT
1D,2D,3-6,7B,8	107749	COMP BNT SHT EXT ASM 16" W R	Right- 70D- 16" W- SHORT
1D,2E,3-6,7C,8	107750	COMP BNT SHT EXT ASM 17" W R	Right- 70D- 17" W- SHORT
1D,2F,3-6,7C,8	107751	COMP BNT SHT EXT ASM 18" W R	Right- 70D- 18" W- SHORT
1D,2G,3-6,7D,8	107752	COMP BNT SHT EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111528	COMP BNT SHT EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- SHORT
>>>>		60°,70° - SHORT EXT - LEFT	
1D,2A,3-6,7A,8	107753	COMP BNT SHT EXT ASM 13" W L	Left- 70D- 13 W- SHORT
1D,2B,3-6,7A,8	107754	COMP BNT SHT EXT ASM 14" W L	Left- 70D- 14 W- SHORT
1D,2C,3-6,7B,8	107755	COMP BNT SHT EXT ASM 15" W L	Left- 70D- 15 W- SHORT
1D,2D,3-6,7B,8	107756	COMP BNT SHT EXT ASM 16" W L	Left- 70D- 16 W- SHORT
1D,2E,3-6,7C,8	107757	COMP BNT SHT EXT ASM 17" W L	Left- 70D- 17 W- SHORT
1D,2F,3-6,7C,8	107758	COMP BNT SHT EXT ASM 18" W L	Left- 70D- 18 W- SHORT
1D,2G,3-6,7D,8	107759	COMP BNT SHT EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111529	COMP BNT SHT EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- SHORT
>>>>		60°,70°,60°&70° HEMI HANGERS - MED EXT - RIGHT	
1E,2A,3-6,7A,8	107760	COMP BNT MED EXT ASM 13" W R	Right- 70D- 13" W- MED
1E,2B,3-6,7A,8	107761	COMP BNT MED EXT ASM 14" W R	Right- 70D- 14" W- MED
1E,2C,3-6,7B,8	107762	COMP BNT MED EXT ASM 15" W R	Right- 70D- 15" W- MED
1E,2D,3-6,7B,8	107763	COMP BNT MED EXT ASM 16" W R	Right- 70D- 16" W- MED
1E,2E,3-6,7C,8	107764	COMP BNT MED EXT ASM 17" W R	Right- 70D- 17" W- MED
1E,2F,3-6,7C,8	107765	COMP BNT MED EXT ASM 18" W R	Right- 70D- 18" W- MED
1E,2G,3-6,7D,8	107766	COMP BNT MED EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111530	COMP BNT MED EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- MED

Pos.	Item Number	Description	Remarks
>>>>		60°,70°,60°&70° HEMI HANGERS - MED EXT - LEFT	
1E,2A,3-6,7A,8	107767	COMP BNT MED EXT ASM 13" W L	Left- 70D- 13 W- MED
1E,2B,3-6,7A,8	107768	COMP BNT MED EXT ASM 14" W L	Left- 70D- 14 W- MED
1E,2C,3-6,7B,8	107769	COMP BNT MED EXT ASM 15" W L	Left- 70D- 15 W- MED
1E,2D,3-6,7B,8	107770	COMP BNT MED EXT ASM 16" W L	Left- 70D- 16 W- MED
1E,2E,3-6,7C,8	107771	COMP BNT MED EXT ASM 17" W L	Left- 70D- 17 W- MED
1E,2F,3-6,7C,8	107772	COMP BNT MED EXT ASM 18" W L	Left- 70D- 18 W- MED
1E,2G,3-6,7D,8	107773	COMP BNT MED EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111531	COMP BNT MED EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- MED
>>>>		60°,70°,60°&70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-6,7A,8	107774	COMP BNT LONG EXT ASM 13" W R	Right- 70D- 13" W- LONG
1F,2B,3-6,7A,8	107775	COMP BNT LONG EXT ASM 14" W R	Right- 70D- 14" W- LONG
1F,2C,3-6,7B,8	107776	COMP BNT LONG EXT ASM 15" W R	Right- 70D- 15" W- LONG
1F,2D,3-6,7B,8	107777	COMP BNT LONG EXT ASM 16" W R	Right- 70D- 16" W- LONG
1F,2E,3-6,7C,8	107778	COMP BNT LONG EXT ASM 17" W R	Right- 70D- 17" W- LONG
1F,2F,3-6,7C,8	107779	COMP BNT LONG EXT ASM 18" W R	Right- 70D- 18" W- LONG
1F,2G,3-6,7D,8	107780	COMP BNT LONG EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111532	COMP BNT LONG EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- LONG
>>>>		60°,70°,60°&70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-6,7A,8	107781	COMP BNT LONG EXT ASM 13" W L	Left- 70D- 13 W- LONG
1F,2B,3-6,7A,8	107782	COMP BNT LONG EXT ASM 14" W L	Left- 70D- 14 W- LONG
1F,2C,3-6,7B,8	107783	COMP BNT LONG EXT ASM 15" W L	Left- 70D- 15 W- LONG
1F,2D,3-6,7B,8	107784	COMP BNT LONG EXT ASM 16" W L	Left- 70D- 16 W- LONG
1F,2E,3-6,7C,8	107785	COMP BNT LONG EXT ASM 17" W L	Left- 70D- 17 W- LONG
1F,2F,3-6,7C,8	107786	COMP BNT LONG EXT ASM 18" W L	Left- 70D- 18 W- LONG
1F,2G,3-6,7D,8	107787	COMP BNT LONG EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111533	COMP BNT LONG EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- LONG
>>>>		60°&70° HEMI HANGER SHORT EXT- RIGHT	
1G,2A,3-6,7A,8	129724	CMP FT BNT KIT SHT HNGR 13W R	Right-Hemi-13W; add Ext behind the Hemi
1G,2B,3-6,7A,8	129726	CMP FT BNT KIT SHT HNGR 14W R	Right-Hemi-14W; add Ext behind the Hemi
1G,2C,3-6,7B,8	129728	CMP FT BNT KIT SHT HNGR 15W R	Right-Hemi-15W; add Ext behind the Hemi
1G,2D,3-6,7B,8	129730	CMP FT BNT KIT SHT HNGR 16W R	Right-Hemi-16W; add Ext behind the Hemi
1G,2E,3-6,7C,8	129732	CMP FT BNT KIT SHT HNGR 17W R	Right-Hemi-17W; add Ext behind the Hemi
1G,2F,3-6,7C,8	129734	CMP FT BNT KIT SHT HNGR 18W R	Right-Hemi-18W; add Ext behind the Hemi
1G,2G,3-6,7D,8	129736	CMP FT BNT KIT SHT HNGR 19-20W R	Right-Hemi-19-20W; add Ext behind the Hemi
1G,2H,3-6,7D,8	129738	CMP FT BNT KIT SHT HNGR 21-22W R	Right-Hemi-21-22W; add Ext behind the Hemi
>>>>		60°&70° HEMI HANGER SHORT EXT- LEFT	
1G,2A,3-6,7A,8	129725	CMP FT BNT KIT SHT HNGR 13W L	Left-Hemi-13W; add Ext behind the Hemi
1G,2B,3-6,7A,8	129727	CMP FT BNT KIT SHT HNGR 14W L	Left-Hemi-14W; add Ext behind the Hemi
1G,2C,3-6,7B,8	129729	CMP FT BNT KIT SHT HNGR 15W L	Left-Hemi-15W; add Ext behind the Hemi
1G,2D,3-6,7B,8	129731	CMP FT BNT KIT SHT HNGR 16W L	Left-Hemi-16W; add Ext behind the Hemi
1G,2E,3-6,7C,8	129733	CMP FT BNT KIT SHT HNGR 17W L	Left-Hemi-17W; add Ext behind the Hemi
1G,2F,3-6,7C,8	129735	CMP FT BNT KIT SHT HNGR 18W L	Left-Hemi-18W; add Ext behind the Hemi
1G,2G,3-6,7D,8	129737	CMP FT BNT KIT SHT HNGR 19-20W L	Left-Hemi-19-20W; add Ext behind the Hemi
1G,2H,3-6,7D,8	129739	CMP FT BNT KIT SHT HNGR 21-22W L	Left-Hemi-21-22W; add Ext behind the Hemi
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

Note: Footplate Assemblies are set to footrest width; please see "Footplate Formula" table for seat width conversion.


Note: If using Short Ext Tube & Hemi Hanger with Composite Angle Adjustable Footplate use 1D (straight) .

Note: V Style Hangers are not available on Swing Away Style chairs effective 7/14/14; see the "Prior To" ordering guide for V Style footplate assemblies.


To have complete Footrest ship assembled, choose compatible Hanger + Footplate part numbers and add "ASSY-HNG-FP" to order

ADD INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2" = Footrest Width
 2" Inset (Seat Width) -4" = Footrest Width
 "V" Hanger (Seat Width) -4" = Footrest Width
 Offset Hanger Receiver (Seat Width) +2" = Footrest Width
 Flared Frame (Seat Width) = 2" = Footrest Width

FOOTPLATE MEASURING GUIDE

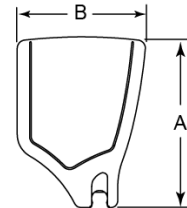

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

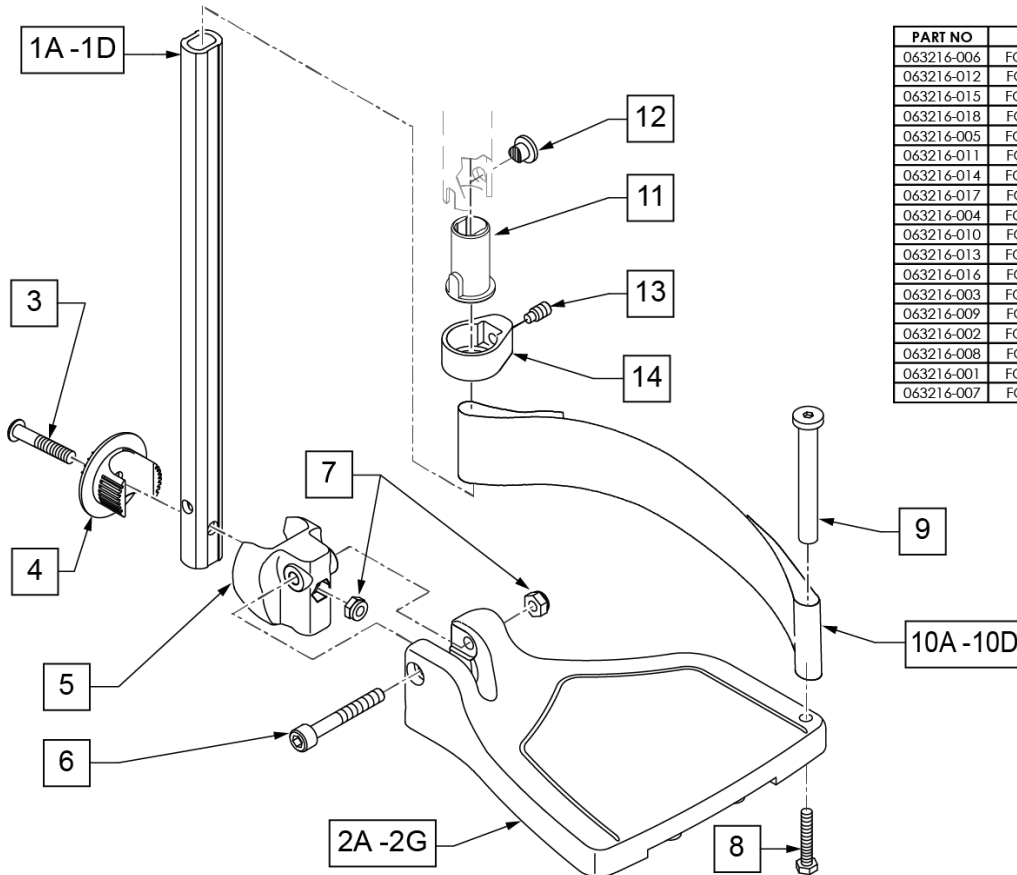

Pos.	Item Number	Description	Remarks
1A,6,7	120660	SHT STRT EXT COMP HDWR	
1B,6,7	120661	MED STRT EXT COMP HDWR	
1C,6,7	120662	LNG STRT EXT COMP HDWR	
1D	128079	TUBE FTRST EXT STRGHT SHT HEMI	Use for Hemi Ext Tube option
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width

Pos.	Item Number	Description	Remarks
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2B	063216-015	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width
2B	063216-018	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2D	063216-014	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width
2D	063216-017	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width
2F	063216-013	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width
2F	063216-016	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width
3	108233	BHCS M6 X 1 X 40	
4	054242	ANGLE COVER	
5	054241	ANGLE SEGMENT	
6	103175	SHCS M6 X 1 X 40 INORG	
7	132626	NUT AUTOMATION M6 X 1.0	
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.
9	104328	HEEL LOOP PEG	
10A	104380-001	ADJUSTABLE FOOT REST STRAP	15" Footrest Width
10B	104380-002	ADJUSTABLE FOOT REST STRAP	16" Footrest Width
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width
11	104160	BUSHING FOOTREST	
11-14	107485	FOOTREST CLAMP ASSM	
>>>>		60,70,80 HANGERS - SHORT EXT - RIGHT	
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 15 W- SHORT
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 16 W- SHORT
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 17 W- SHORT
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 18 W- SHORT
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 19 W- SHORT
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 20 W- SHORT
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 21&22 W- SHORT
>>>>		60,70,80 HANGERS - SHORT EXT - LEFT	
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 15 W- SHORT
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 16 W- SHORT
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 17 W- SHORT
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 18 W- SHORT
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 19 W- SHORT
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 20 W- SHORT
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 21&22 W- SHORT
>>>>		60,70,80& 60,70 HEMI HANGER- RIGHT	
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 15 W- MED
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 16 W- MED
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 17 W- MED
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 18 W- MED
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 19 W- MED
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 20 W- MED
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 21&22 W- MED
>>>>		60,70,80& 60,70 HEMI HANGER- LEFT	
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 15 W- MED
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 16 W- MED
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 17 W- MED
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 18 W- MED
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 19 W- MED
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 20 W- MED

Pos.	Item Number	Description	Remarks
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 21&22 W- MED
>>>>		60,70,80& 60,70 HEMI HANGER- RIGHT	
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 15 W- LONG
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 16 W- LONG
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 17 W- LONG
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 18 W- LONG
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 19 W- LONG
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 20 W- LONG
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 21&22 W- LONG
>>>>		60,70,80& 60,70 HEMI HANGER- LEFT	
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 15 W- LONG
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 16 W- LONG
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 17 W- LONG
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 18 W- LONG
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 19 W- LONG
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 20 W- LONG
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 21&22 W- LONG
>>>>		60&70 HEMI HANGERS- SHORT EXT- RIGHT	
1D,2A,3-10A	129710	CAA FOOT KIT SHT HNGR 15W R	Right-Hemi-15W; add Ext behind the Hemi
1D,2B,3-10B	129712	CAA FOOT KIT SHT HNGR 16W R	Right-Hemi-16W; add Ext behind the Hemi
1D,2C,3-10B	129714	CAA FOOT KIT SHT HNGR 17W R	Right-Hemi-17W; add Ext behind the Hemi
1D,2D,3-10C	129716	CAA FOOT KIT SHT HNGR 18W R	Right-Hemi-18W; add Ext behind the Hemi
1D,2E,3-10C	129718	CAA FOOT KIT SHT HNGR 19W R	Right-Hemi-19W; add Ext behind the Hemi
1D,2F,3-10D	129720	CAA FOOT KIT SHT HNGR 20W R	Right-Hemi-20W; add Ext behind the Hemi
1D,2G,3-10D	129722	CAA FOOT KIT SHT HNGR 22W R	Right-Hemi-21&22W; add Ext behind the Hemi
>>>>		60&70 HEMI HANGERS- SHORT EXT - LEFT	
1D,2A,3-10A	129711	CAA FOOT KIT SHT HNGR 15W L	Left-Hemi-15W; add Ext behind the Hemi
1D,2B,3-10B	129713	CAA FOOT KIT SHT HNGR 16W L	Left-Hemi-16W; add Ext behind the Hemi
1D,2C,3-10B	129715	CAA FOOT KIT SHT HNGR 17W L	Left-Hemi-17W; add Ext behind the Hemi
1D,2D,3-10C	129717	CAA FOOT KIT SHT HNGR 18W L	Left-Hemi-18W; add Ext behind the Hemi
1D,2E,3-10C	129719	CAA FOOT KIT SHT HNGR 19W L	Left-Hemi-19W; add Ext behind the Hemi
1D,2F,3-10D	129721	CAA FOOT KIT SHT HNGR 20W L	Left-Hemi-20W; add Ext behind the Hemi
1D,2G,3-10D	129723	CAA FOOT KIT SHT HNGR 22W L	Left-Hemi-21&22W; add Ext behind the Hemi
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

Note: Footplate Assemblies are set to footrest width; please see "Footplate Formula" table for seat width conversion.

Note: If using Short Ext Tube & Hemi Hanger with Adult Angle Adjustable Footplate use 1D (straight) .

Note: V Style Hangers are not available on Swing Away Style chairs effective 7/14/14; see the "Prior To" ordering guide for V Style footplate assemblies.

Note: for QX Series or SR45 footplate replacement parts see the ordering guide marked specifically for these two models in the informations panel.


Note: if chair has High Mount Ext Tubes or if switching to High Mount Ext Tubes, see AAA Footplate w/ High Mount Extension Ordering Guide.

[02/19/2020]


To have complete Footrest ship assembled, choose compatible Hanger + Footplate part numbers and add "ASSY-HNG-FP" to order

ADD INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2" = Footrest Width
 2" Inset (Seat Width) -4" = Footrest Width
 "V" Hanger (Seat Width) -4" = Footrest Width
 Offset Hanger Receiver (Seat Width) +2" = Footrest Width
 Flared Frame (Seat Width) = 2" = Footrest Width

MEASURING GUIDE

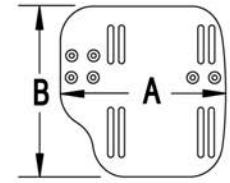

CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

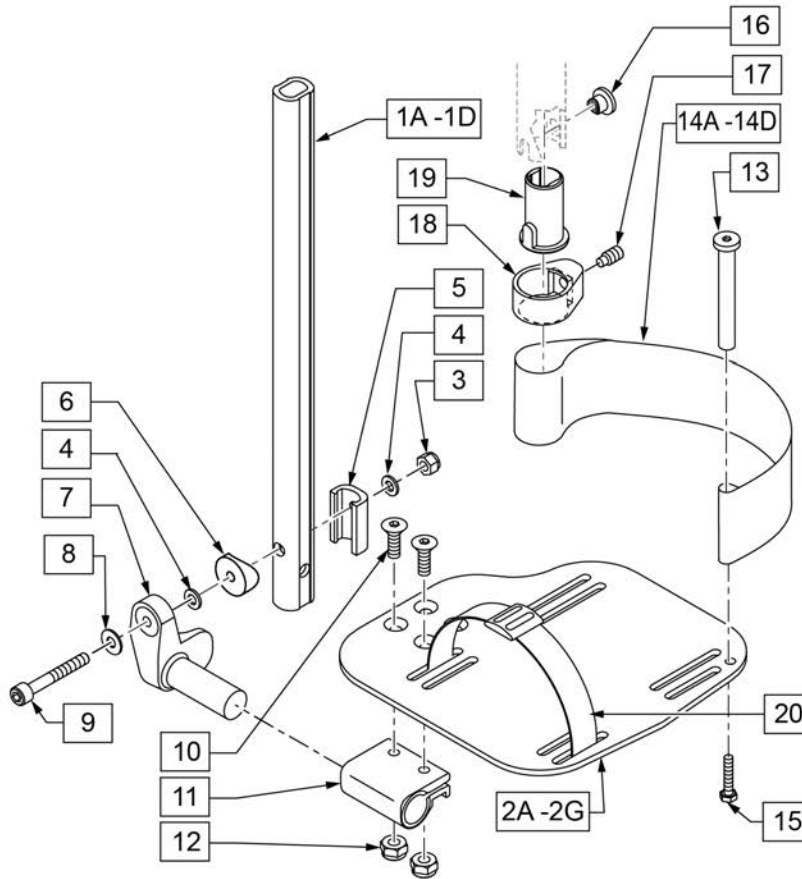

21 EXT MNT HARDWARE 1 SIDE			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
3		NUT AUTOMATION M6 X 1.0 ROHS	1
4		WASHER FLAT THIN AN 1/4	2
5		HANGER EXTENSION SADDLE	1
6		3/4" SADDLE 1" OD	1
8		WASHER FLAT SAE 1/4 ROHS	1
9		SHCS M6X1X45 12MM PART THD	1

Pos.	Item Number	Description	Remarks
1A,3-6,8,9	120670	SHT STRT EXT AAA HDWR	
1B,3-6,8,9	120671	MED STRT EXT AAA HDWR	
1C,3-6,8,9	120672	LNG STRT EXT AAA HDWR	
1D	128079	TUBE FTRST EXT STRGHT SHT HEMI	Use for Hemi Ext Tube option
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width
3	132626	NUT AUTOMATION M6 X 1.0	
4	106944	WASHER FLAT THIN AN 1/4 ROHS	
5	107199	HANGER EXTENSION SADDLE	
6	561055	3/4" SADDLE 1" OD	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT


Pos.	Item Number	Description	Remarks
8	106945	WASHER FLAT SAE 1/4 ROHS	
9	104399	SHCS M6X1X45 12 MM PART THD	
10	106978	FHCS 1/4-28 X 7/8 ROHS	
11	550350	3/4" CLMP ADJ FT PLT	
12	106935	NUT NYLOCK THIN 1/4-28 ROHS	
13	104328	POST HEEL LOOP	
14A	104380-001	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE
14B	104380-002	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE
15	102268-001	HHCS M6 X 1 X 25	
16-19	107485	FOOTREST CLAMP ASSM	
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
21	120659	EXT MNT HARDWARE 1 SIDE	
>>>>		60,70,80 STD HANGERS - RIGHT	
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 12-13" W- SHORT
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 14-15" W- SHORT
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 16-17" W- SHORT
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 18-22" W- SHORT
>>>>		60,70,80 STD HANGERS - LEFT	
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 12-13" W- SHORT
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 14-15" W- SHORT
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 16-17" W- SHORT
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 18-22" W- SHORT
>>>>		60,70,80 & 60,70 HEMI HANGERS - RIGHT	
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 12-13" W- MED
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 14-15" W- MED
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 16-17" W- MED
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 18-22" W- MED
>>>>		60,70,80 & 60,70 HEMI HANGERS - LEFT	
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 12-13" W- MED
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 14-15" W- MED
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 16-17" W- MED
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 18-22" W- MED
>>>>		60,70,80 & 60,70 HEMI HANGERS - RIGHT	
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 12-13" W- LONG
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14 -15" W R	Right- 14-15" W- LONG
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16 -17" W R	Right- 16-17" W- LONG
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18 -22" W R	Right- 18-22" W- LONG
>>>>		60,70,80 & 60,70 HEMI HANGERS - LEFT	
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 12-13" W- LONG
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 14-15" W- LONG
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 16-17" W- LONG
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 18-22" W- LONG
>>>>		60&70 HEMI HANGERS - SHORT EXT - RIGHT	
1D,2B,3-14A,15	129756	AAA FOOT KIT SHT HNGR 12W R	Right- 12-13" W- SHORT; add Ext behind the Hemi
1D,2C,3-14B,15	129758	AAA FOOT KIT SHT HNGR 14W R	Right- 14-15" W- SHORT; add Ext behind the Hemi
1D,2D,3-14C,15	129760	AAA FOOT KIT SHT HNGR 16W R	Right- 16-17" W- SHORT; add Ext behind the Hemi
1D,2E,3-14D,15	129762	AAA FOOT KIT SHT HNGR 18W R	Right- 18-22" W- SHORT; add Ext behind the Hemi
>>>>		60&70 HEMI HANGERS - SHORT EXT -LEFT	
1D,2B,3-14A,15	129757	AAA FOOT KIT SHT HNGR 12W L	Left- 12-13" W-SHORT; add Ext behind the Hemi
1D,2C,3-14B,15	129759	AAA FOOT KIT SHT HNGR 14W L	Left- 14-15" W- SHORT; add Ext behind the Hemi
1D,2D,3-14C,15	129761	AAA FOOT KIT SHT HNGR 16W L	Left- 16-17" W- SHORT; add Ext behind the Hemi
1D,2E,3-14D,15	129763	AAA FOOT KIT SHT HNGR 18W L	Left- 18-22" W- SHORT; add Ext behind the Hemi
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

Note: Footplate Assemblies are set to footrest width; please see "Footplate Formula" table for seat width conversion.

Note: V Style Hangers are not available on Swing Away Style chairs effective 7/14/14; see the "Prior To" ordering guide for V Style footplate assemblies.

Note: Zippie Iris (ZRS&ZRSE) and Zippie TS (Z1A&Z1SE) only use Straight Extension Tube.

Note: if chair has High Mount Ext Tubes or if switching to High Mount Ext Tubes, see KAA Footplate w/ High Mount Extension Ordering Guide.

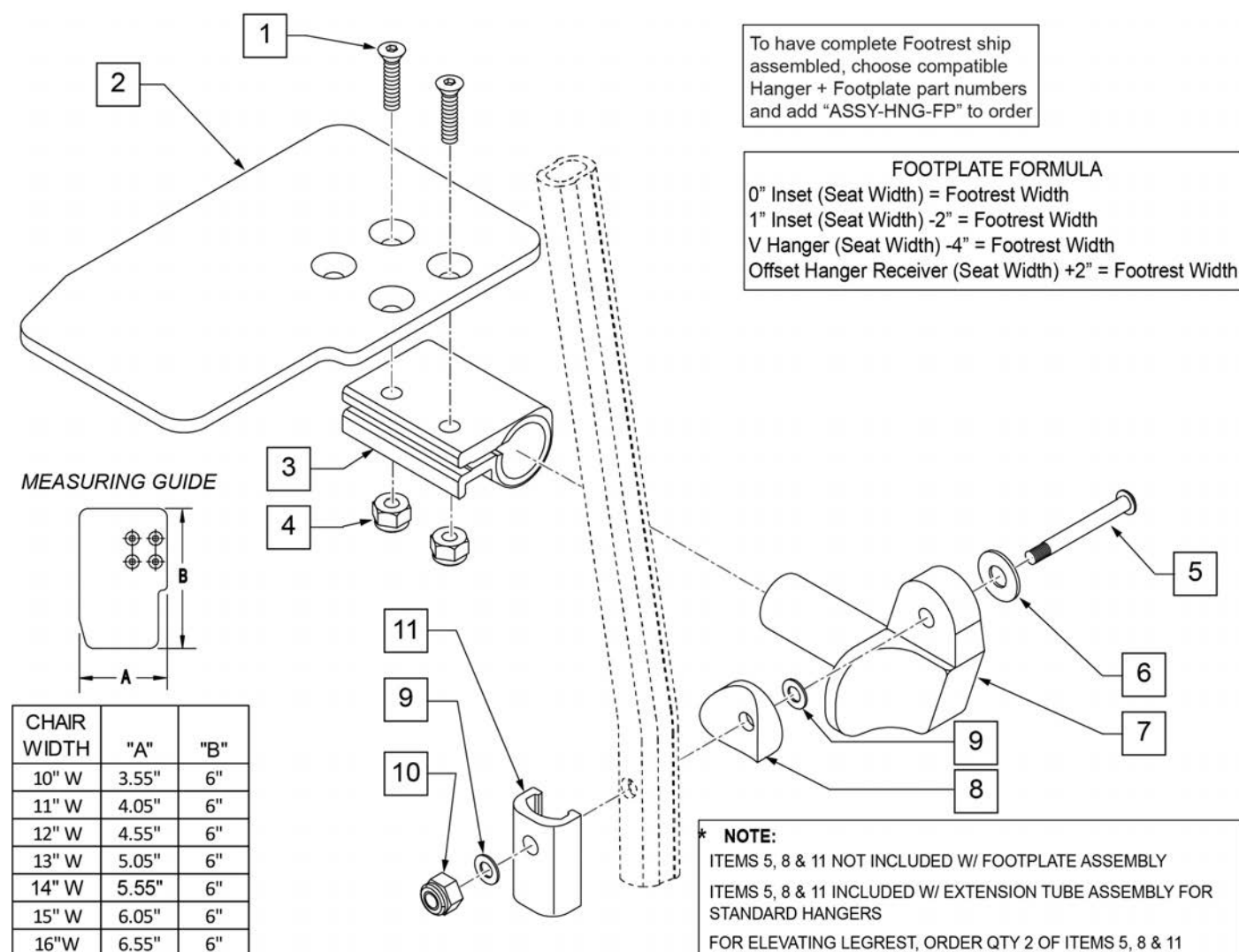


Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4 -28 x 7/8 ROHS	
2	500635	KIDS ANGL ADJ F/PLT 10" R	Right

Pos.	Item Number	Description	Remarks
2	500636	KIDS ANGL ADJ F/PLT 10" L	Left
2	500637	KIDS ANGL ADJ F/PLT 11" R	Right
2	500638	KIDS ANGL ADJ F/PLT 11" L	Left
2	500639	KIDS ANGL ADJ F/PLT 12" R	Right
2	500640	KIDS ANGL ADJ F/PLT 12" L	Left
2	500641	KIDS ANGL ADJ F/PLT 13" R	Right
2	500642	KIDS ANGL ADJ F/PLT 13" L	Left
2	500643	KIDS ANGL ADJ F/PLT 14" R	Right
2	500644	KIDS ANGL ADJ F/PLT 14" L	Left
2	500645	KIDS ANGL ADJ F/PLT 15" R	Right
2	500646	KIDS ANGL ADJ F/PLT 15" L	Left
2	500647	KIDS ANGL ADJ F/PLT 16" R	Right
2	500648	KIDS ANGL ADJ F/PLT 16" L	Left
3	550350	3/4" CLAMP ADJ FOOTPLATE	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	106962	1/4-28 x 1 3/4 BH C/S BLK ZN (D2 EXTENSION TUBES)	
5	226167	1/4-28 x 1.625 BH C/S BLK (ROUND EXTENSION TUBES)	
6	106945	1/4 SAE WASHER	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	
8	561055	3/4" SADDLE 1" OD	
9	106944	WASHER FLAT THIN AN 1/4 ROHS	
10	106941	NUT AUTOMATION 1/4 -28 ROHS	
11	483720	HANGER EXTENSION SADDLE LONG	Use with Double D Ext Tube
1-4,6,7,9,10	828850	KIDS ANG ADJ F/PLT EXT-MNT 10"R	Right
1-4,6,7,9,10	828851	KIDS ANG ADJ F/PLT EXT-MNT 10"L	Left
1-4,6,7,9,10	828852	KIDS ANG ADJ F/PLT EXT-MNT 11"R	Right
1-4,6,7,9,10	828853	KIDS ANG ADJ F/PLT EXT-MNT 11"L	Left
1-4,6,7,9,10	828854	KIDS ANG ADJ F/PLT EXT-MNT 12"R	Right
1-4,6,7,9,10	828855	KIDS ANG ADJ F/PLT EXT-MNT 12"L	Left
1-4,6,7,9,10	828856	KIDS ANG ADJ F/PLT EXT-MNT 13"R	Right
1-4,6,7,9,10	828857	KIDS ANG ADJ F/PLT EXT-MNT 13"L	Left
1-4,6,7,9,10	828858	KIDS ANG ADJ F/PLT EXT-MNT 14"R	Right
1-4,6,7,9,10	828859	KIDS ANG ADJ F/PLT EXT-MNT 14"L	Left
1-4,6,7,9,10	828860	KIDS ANG ADJ F/PLT EXT-MNT 15"R	Right
1-4,6,7,9,10	828861	KIDS ANG ADJ F/PLT EXT-MNT 15"L	Left
1-4,6,7,9,10	828862	KIDS ANG ADJ F/PLT EXT-MNT 16"R	Right
1-4,6,7,9,10	828863	KIDS ANG ADJ F/PLT EXT-MNT 16"L	Left
1-4,6,7,9,10	921280	KIDS ANG ADJ F/PLT EXT-MNT 10"	Pair
1-4,6,7,9,10	921281	KIDS ANG ADJ F/PLT EXT-MNT 11"	Pair
1-4,6,7,9,10	921282	KIDS ANG ADJ F/PLT EXT-MNT 12"	Pair
1-4,6,7,9,10	921283	KIDS ANG ADJ F/PLT EXT-MNT 13"	Pair
1-4,6,7,9,10	921284	KIDS ANG ADJ F/PLT EXT-MNT 14"	Pair
1-4,6,7,9,10	921285	KIDS ANG ADJ F/PLT EXT-MNT 15"	Pair
1-4,6,7,9,10	921286	KIDS ANG ADJ F/PLT EXT-MNT 16"	Pair
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

To request assembly of parts being ordered as a kit, please add "ASSY-HNG-FP" after ELR or ALR & Calf Pad parts are selected

ADD INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2" = Footrest Width
 2" Inset (Seat Width) -4" = Footrest Width
 "V" Hanger (Seat Width) -4" = Footrest Width
 Offset Hanger Receiver (Seat Width) +2" = Footrest Width
 Flared Frame (Seat Width) = 2" = Footrest Width

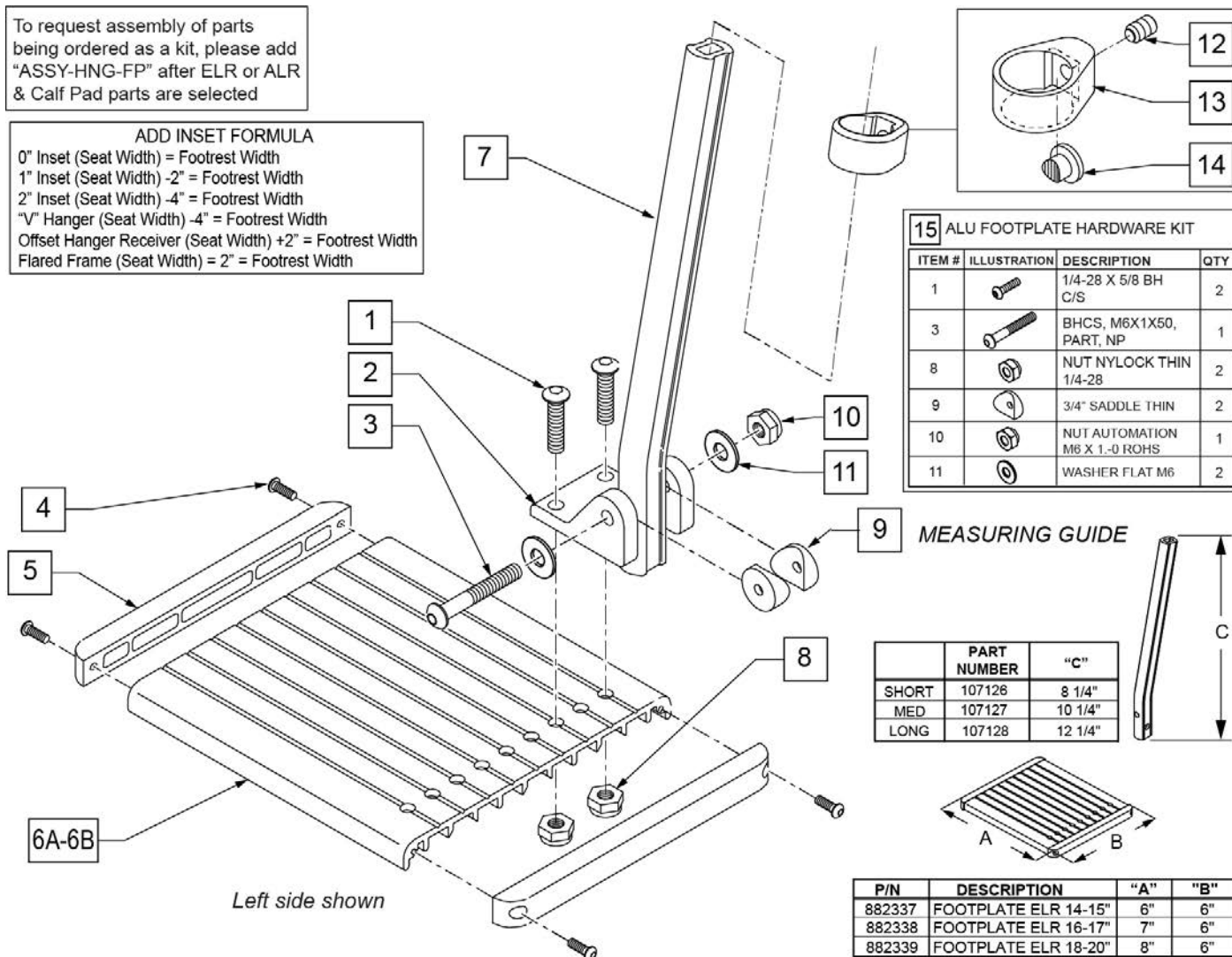


15 ALU FOOTPLATE HARDWARE KIT			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
1		1/4-28 X 5/8 BH C/S	2
3		BHCS, M6X1X50, PART, NP	1
8		NUT NYLOCK THIN 1/4-28	2
9		3/4" SADDLE THIN	2
10		NUT AUTOMATION M6 X 1.0 ROHS	1
11		WASHER FLAT M6	2

	PART NUMBER	"C"
SHORT	107126	8 1/4"
MED	107127	10 1/4"
LONG	107128	12 1/4"


P/N	DESCRIPTION	"A"	"B"
882337	FOOTPLATE ELR 14-15"	6"	6"
882338	FOOTPLATE ELR 16-17"	7"	6"
882339	FOOTPLATE ELR 18-20"	8"	6"

Pos.	Item Number	Description	Remarks
1	226090	1/4-28 x 5/8 BH C/S	
2	118093	BRACKET FOOTPLATE ALUM	
3	102262-008	BHCS M6 X 1 X 50 INORGANIC	
4	281043	#4 x 5/8 PAN PHIL S/TAP	
5	560101	END CAPS FOOTREST	
7	107126	SHORT BENT	
7	107127	MED BENT	
7	107128	LONG BENT	
7	128078	TUBE FOOTREST EXT SHT HEMI	Use for Hemi Extension option
8	106935	1/4-28 NYLOCK NUT THIN	
9	561003	3/4" SADDLE THIN	
10	132626	NUT AUTOMATION M6 X 1.0	
11	107383	WASHER FLAT M6 ROHS	
4,5,6A	882337	ELR FOOTPLATE ASSEMBLY 14 -15"	
4,5,6B	882338	ELR FOOTPLATE ASSEMBLY 16 -17"	
4,5,6C	882339	ELR FOOTPLATE ASSEMBLY 18 -20"	
12-14	119495	EXTENSION CLAMP KIT	
15	129775	ALU FOOTPLATE HDWARE KIT	Ext tube not included; bracket (118093) not included

Pos.	Item Number	Description	Remarks
		Ship assembled part #	Use with multiple assemblies
	ASSY-HNG-FP	HANGER/FOOTPLATE ASSY REQUIRED	add with Hanger & Footplate kit

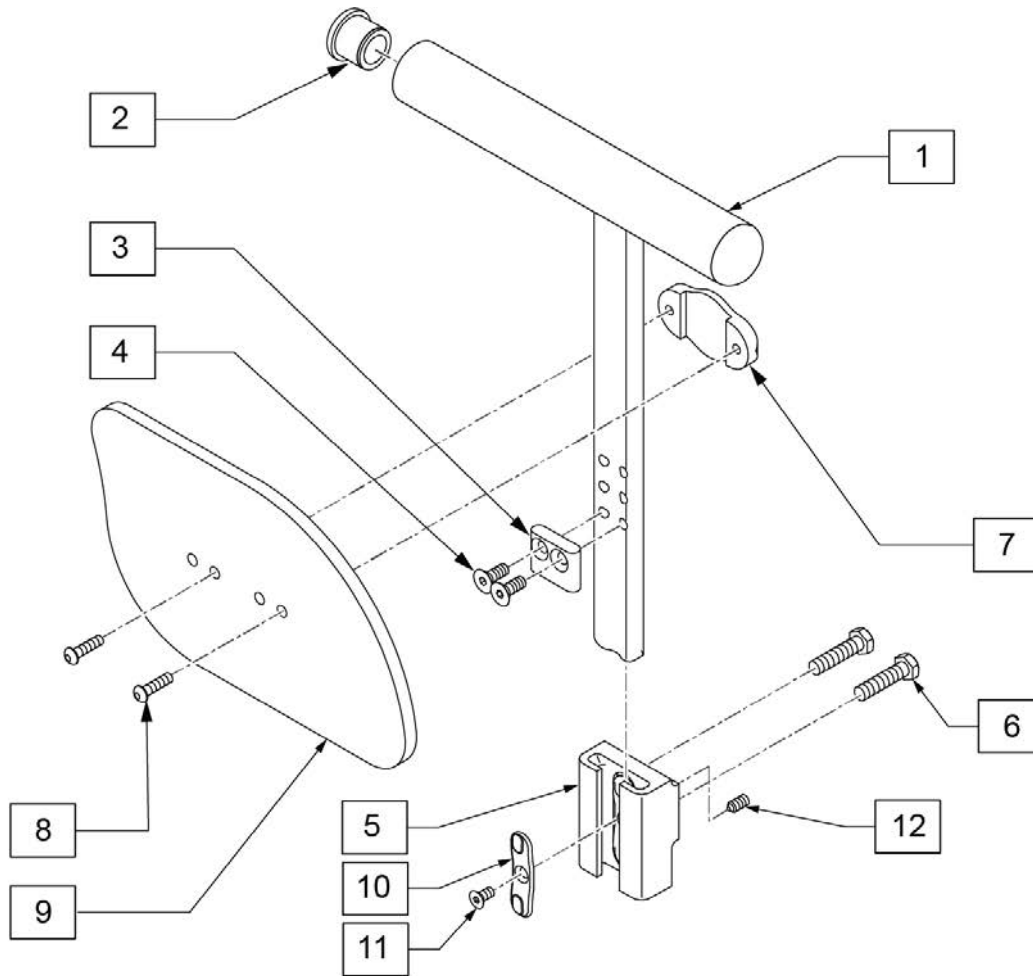

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
2	561007	7/8 INSERT PLUG .083		1-4	119870	T-POST ARM XTALL W/FOAM L	Left
3	550105-QU	KIDZ ARM STOP		1-4	119871	T-POST ARM 2XTALL W/FOAM R	Right
4	226012	10-32 X 1/2 FH C/S BLK		1-4	119872	T-POST ARM 2XTALL W/FOAM L	Left
5	127761	T-POST/SIDEGUARD REC ASSM		7-9	136458	SIDEGUARD PANEL KIT RT	Right
6	118194	HHCS M6X1X25 PART PL		7-9	136459	SIDEGUARD PANEL KIT LFT	Left
7	127651	SIDEGUARD CLAMP		5,6,10-12	136460	T-POST SIDEGUARD REC KIT	
8	103184-006	BHCS, M5X0.8X16, FULL, NP		1-12	164118	T-POST XTLL ARM W/ SG RIGHT	Right
9	127650	SIDEGUARD T-POST A		1-12	164119	T-POST XTLL ARM W/ SG LEFT	Left
9	129369	SIDEGUARD T-POST B		1-12	164120	T-POST 2XTLL ARM W/ SG RIGHT	Right
10	108780	RECEIVER SG PLASTIC INSERT		1-12	164121	T-POST 2XTLL ARM W/ SG LEFT	Left
11	107135	FHCS #6-32 X 1/4		1-12	164122	T-POST XTLL W/ SG RETRO KIT	Pair
12	011825	SET SCREW M6 X 8MM CUP TIP		1-12	164123	T-POST 2XTLL W/ SG RETRO KIT	Pair
1-4	119869	T-POST ARM XTALL W/FOAM R	Right				

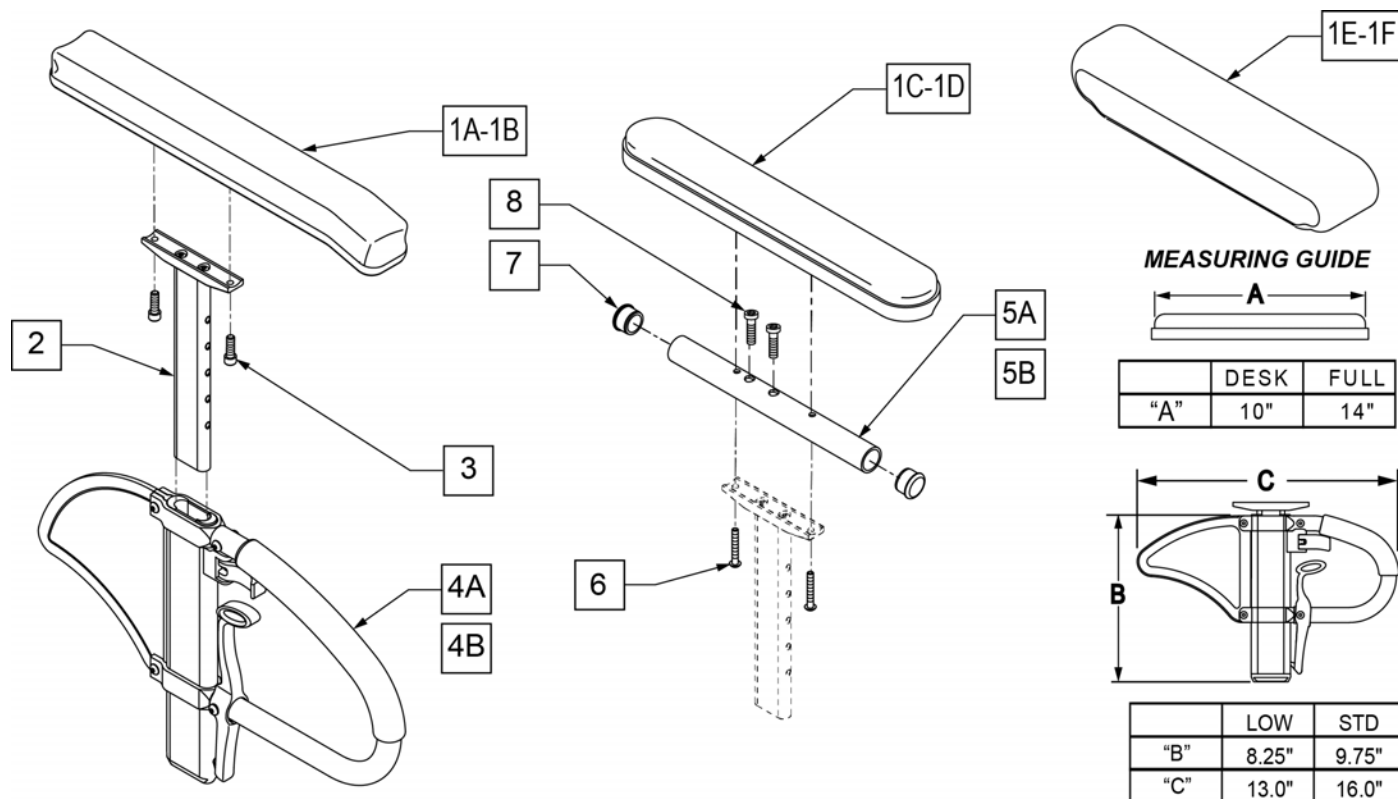

Pos.	Item Number	Description	Remarks
1A	104568-001	SPHA PAD DESK	CONTOUR DESK
1B	104568-002	SPHA PAD FULL	CONTOUR FULL
1C	561024	ARMREST PAD 10" DESK	CLASSIC DESK
1D	561031	ARMREST PAD 14" FULL	CLASSIC FULL
1E	561280	ARMREST PAD 10" LEFT W/F	Left-Waterfall Desk
1E	561281	ARMREST PAD 10" RIGHT W/F	Rightt-Waterfall Desk
1F	561284	ARMREST PAD 14" LEFT W/F	Leftt-Waterfall Full
1F	561285	ARMREST PAD 14" RIGHT W/F	Rightt-Waterfall Full
2	116312	SNGL POST UPPR ASSM LOW	LOW/SMALL
2	116313	SNGL POST UPPR ASSM STD	STANDARD
3	106956	SHCS 10-32 X 3/4 ROHS	
2,4A	104509-001	SPHA ARMREST ASSY STD	STANDARD
2,4B	104509-002	SPHA ARMREST ASSY LOW	LOW/SMALL
5A	521812	HT ADJ PD TUBE 10IN DESK POWE	Desk
5B	521813	HT ADJ PD TUBE 14IN FULL POWE	Full
6	107024	BHCS 10-32 X 1 1/2 ROHS	
7	560735	7/8" ROUND TUBE PLUG	
8	108534	SHCS 10-32 X 1-1/4 PL	Used with Full Length & Waterfall
1C,2,3,4A,5A,6,7	108599	SPHA STD CLASIC DESK PAD R	Right
1C,2,3,4A,5A,6,7	108603	SPHA STD CLASIC DESK PAD L	Left
1D,2,3,4A,5B,6-8	108601	SPHA STD CLASIC FULL PAD R	Right
1D,2,3,4A,5B,6-8	108605	SPHA STD CLASIC FULL PAD L	Left
1A,2,3,4A	108595	SPHA STD CNTUR DESK PAD	
1B,2,3,4A	108597	SPHA STD CNTUR FULL PAD	
1C,2,3,4B,5A,6,7	108600	SPHA LOW CLASIC DESK PAD R	Right
1C,2,3,4B,5A,6,7	108604	SPHA LOW CLASIC DESK PAD L	Left
1D,2,3,4B,5B,6-8	108602	SPHA LOW CLASIC FULL PAD R	Right


Pos.	Item Number	Description	Remarks
1D,2,3,4B,5B,6-8	108606	SPHA LOW CLASIC FULL PAD L	Left
1A,2,3,4B	108596	SPHA LOW CNTUR DESK PAD	
1B,2,3,4B	108598	SPHA LOW CNTUR FULL PAD	
1C,2,5A,6,7	159600	SPHA STD CLAS DESK UPPR R	Right
1C,2,5A,6,7	159601	SPHA STD CLAS DESK UPPR L	Left
1D,2,5B,6-8	159602	SPHA STD CLAS FULL UPPR R	Right
1D,2,5B,6-8	159603	SPHA STD CLAS FULL UPPR L	Left
1C,2,5A,6,7	159604	SPHA LOW CLAS DESK UPPR R	Right
1C,2,5A,6,7	159605	SPHA LOW CLAS DESK UPPR L	Left
1D,2,5B,6-8	159606	SPHA LOW CLAS FULL UPPR R	Right
1D,2,5B,6-8	159607	SPHA LOW CLAS FULL UPPR L	Left
1E,2,4B,5A,6,7	164089	SPHA LOW W/F DESK PAD R	Right
1E,2,4B,5A,6,8	164090	SPHA LOW W/F DESK PAD L	Left
1F,2,4B,5B,6,8	164091	SPHA LOW W/F FULL PAD R	Right
1F,2,4B,5B,6,8	164092	SPHA LOW W/F FULL PAD L	Left
1E,2,4A,5A,6,7	164093	SPHA STD W/F DESK PAD R	Right
1E,2,4A,5A,6,8	164094	SPHA STD W/F DESK PAD L	Left
1F,2,4A,5B,6,8	164095	SPHA STD W/F FULL PAD R	Right
1F,2,4A,5B,6,8	164096	SPHA STD W/F FULL PAD L	Left

Note: Fits most Manual Products, not all Armrest Pads are offered on all chairs, consult product order form for compatibility.

Note: available on Quickie Iris after S/N IRS-007726 & IRSE-008701 and prior to IRS-063000 & IRSE-083000, and Zippie Iris prior to ZRS-051000 & ZRSE-058000.

Note: if switching from Desk to Full Length Armrest Pad OR switching from Full Length to Desk Armrest Pad, the following items are required: 1C/1D, 5A/5B, and 6-8.


Note: Not compatible with fixed Height Back.

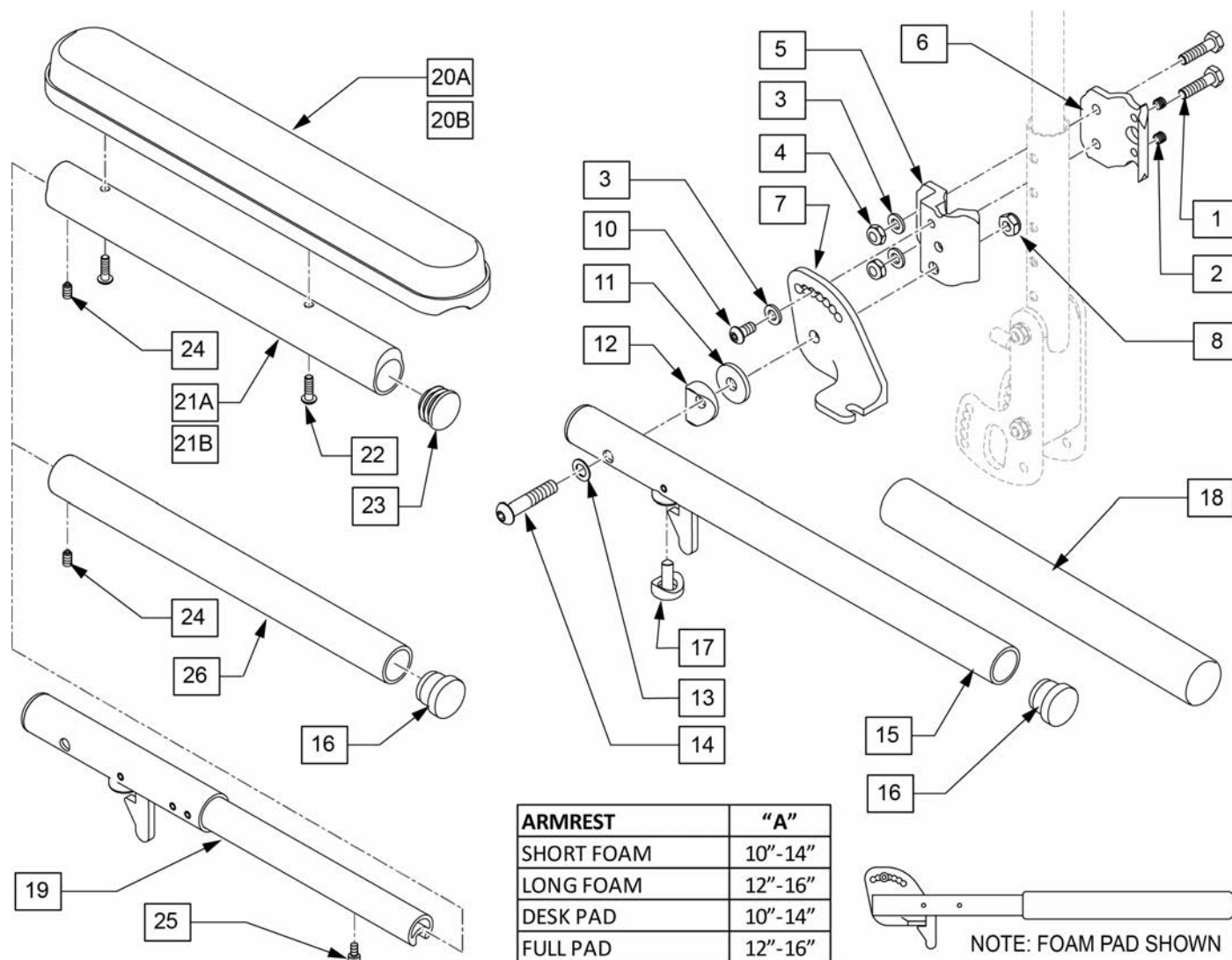

Pos.	Item Number	Description	Remarks
7	550474	ADJUSTABLE FLIP-BACK ARM BRACKET	RIGHT
7	550475	ADJUSTABLE FLIP-BACK ARM BRACKET	LEFT
15-18	880274	ADJ FLIP BACK TUBE W/LATCH	
16	561002	1" INSERT PLUG .083	
18	560299	7 x .940 x .20 FOAM GRIP	SHORT PAD
18	560640	9" X .93" X .155" FOAM GRIP	LONG PAD
19	880305	ADJ FLIP BACK TUBE W/LATCH	Length Adjustable
20A	561024	ARMREST PAD 10" DESK	
20B	561031	ARMREST PAD 14" FULL	
22	107022	BHCS 10-32 X 5/8 ROHS	
23	560252	1" COVED STEM BUMPER	
20A,21A,22-24	882070	FULL PAD ADJUSTABLE LENGTH ASSM	RIGHT FULL PAD ASSM ONLY
20A,21A,22-24	882071	FULL PAD ADJUSTABLE LENGTH ASSM	LEFT FULL PAD ASSM ONLY
20B,21B,22-24	882072	DESK PAD ADJUSTABLE LENGTH ASSM	RIGHT DESK PAD ASSM ONLY


Pos.	Item Number	Description	Remarks
20B,21B,22-24	882073	DESK PAD ADJUSTABLE LENGTH ASSM	LEFT DESK PAD ASSM ONLY
1-18	159795	ADJ FB ARMREST W HAB CLAMP R	Complete assm rt Foam Pad
1-18	159796	ADJ FB ARMREST W HAB CLAMP L	Complete assm lft Foam Pad
1-18	159659	ADJ LOCKING FLIP-BACK ARMREST PR	
1-6	159674	ADJ FLIP BACK CLAMP KIT R	clamp kit-same on both pages
1-6	159675	ADJ FLIP BACK CLAMP KIT L	clamp kit-same on both pages
1-6	159676	ADJ FLIP BACK CLAMP KIT PR	
1-4	159693	ADJ HT ARMREST CLAMP HARDWARE	
3,8,10-13,22-25	159694	ADJ HT LCK ARMREST HARDWARE	
1-14,19	159797	L ADJ FB AR HAB CLMP NO PAD RH	
1-14,19	159798	L ADJ FB AR HAB CLMP NO PAD LH	
1-14,18,18,14,26	159660	LNG FOAM ADJ LENGTH ADJ LCK FLP R	Complete assm adj long foam Rt
1-14,18,18,14,26	159661	LNG FOAM ADJ LENGTH ADJ LCK FLP L	Complete assm adj long foam Lft
1-14,18,18,14,26	159662	LNG FOAM ADJ LENGTH ADJ LCK FLP PR	Pair Complete assm adj foam
1-14,18,18,14,26	159663	SHT FOAM ADJ LENGTH ADJ LCK FLP R	Complete assm adj short foam Rt
1-14,18,18,14,26	159664	SHT FOAM ADJ LENGTH ADJ LCK FLP L	Complete assm adj short foam Lft
1-14,18,18,14,26	159665	SHT FOAM ADJ LENGTH ADJ LCK FLP PR	Pair Complete assm adj short foam
1-14,19-25	159669	DESK PAD ADJ LENGTH ADJ LCK FLP R	Complete assm rt Desk Pad
1-14,19-25	159670	DESK PAD ADJ LENGTH ADJ LCK FLP L	Complete assm lft Desk Pad
1-14,19-25	159671	DESK PAD ADJ LENGTH ADJ LCK FLP PR	Pair Complete assm Desk Pad
1-14,19-25	159666	FULL PAD ADJ LENGTH ADJ LCK FLP R	Complete assm rt Full Pad
1-14,19-25	159667	FULL PAD ADJ LENGTH ADJ LCK FLP L	Complete assm lft Full Pad
1-14,19-25	159668	FULL PAD ADJ LENGTH ADJ LCK FLP PR	Pair Complete assm Full Pad

Note: Not compatible with fixed Height Back.

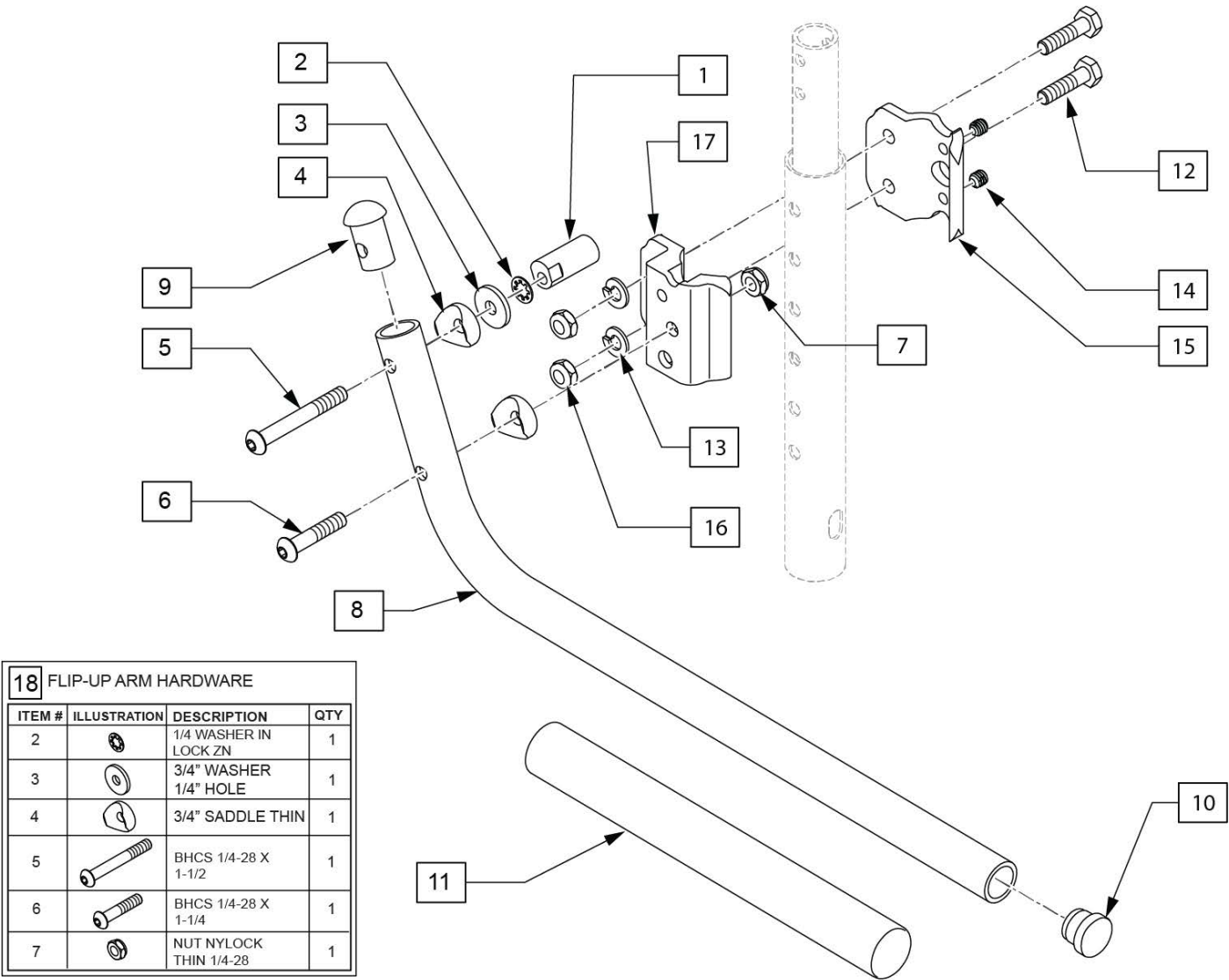


Pos.	Item Number	Description	Remarks
1	483041	FLIP-UP ARM CAM ZP/P5	
9	561034	SHORT DOME PLUG	
10	561002	1" INSERT PLUG .083	
11	560640	9 x .93 x .155 FOAM GRIP	
18	159697	FLIP-UP ARM HARDWARE	
12-17	159674	ADJ FLIP BACK CLAMP KIT R	clamp replacement rt
12-17	159675	ADJ FLIP BACK CLAMP KIT L	clamp replacement lft
12-14,16	159693	ADJ HT ARMREST CLAMP HARDWARE	
1-17	159799	ZP FLIP UP ARM XC 10-13 HAB RH	Right,
1-17	159800	ZP FLIP UP ARM XC 10-13 HAB LH	Left,
1-17	159672	FLIP-UP ARMST W CLAMP 10-13" PR	Pair,
1-17	159801	ZP FLIP UP ARM XC 14-16 HAB RH	Right,
1-17	159802	ZP FLIP UP ARM XC 14-16 HAB LH	Left,
1-17	159673	FLIP-UP ARMST W CLAMP 14-16" PR	Pair,
8-11	159695	FLIP-UP ARM 10-13"	arm replacement
8-11	159696	FLIP-UP ARM 14-16"	arm replacement

Note: only available on Kids Quickie 2 w/o seating (Depth Adjustable Back only).

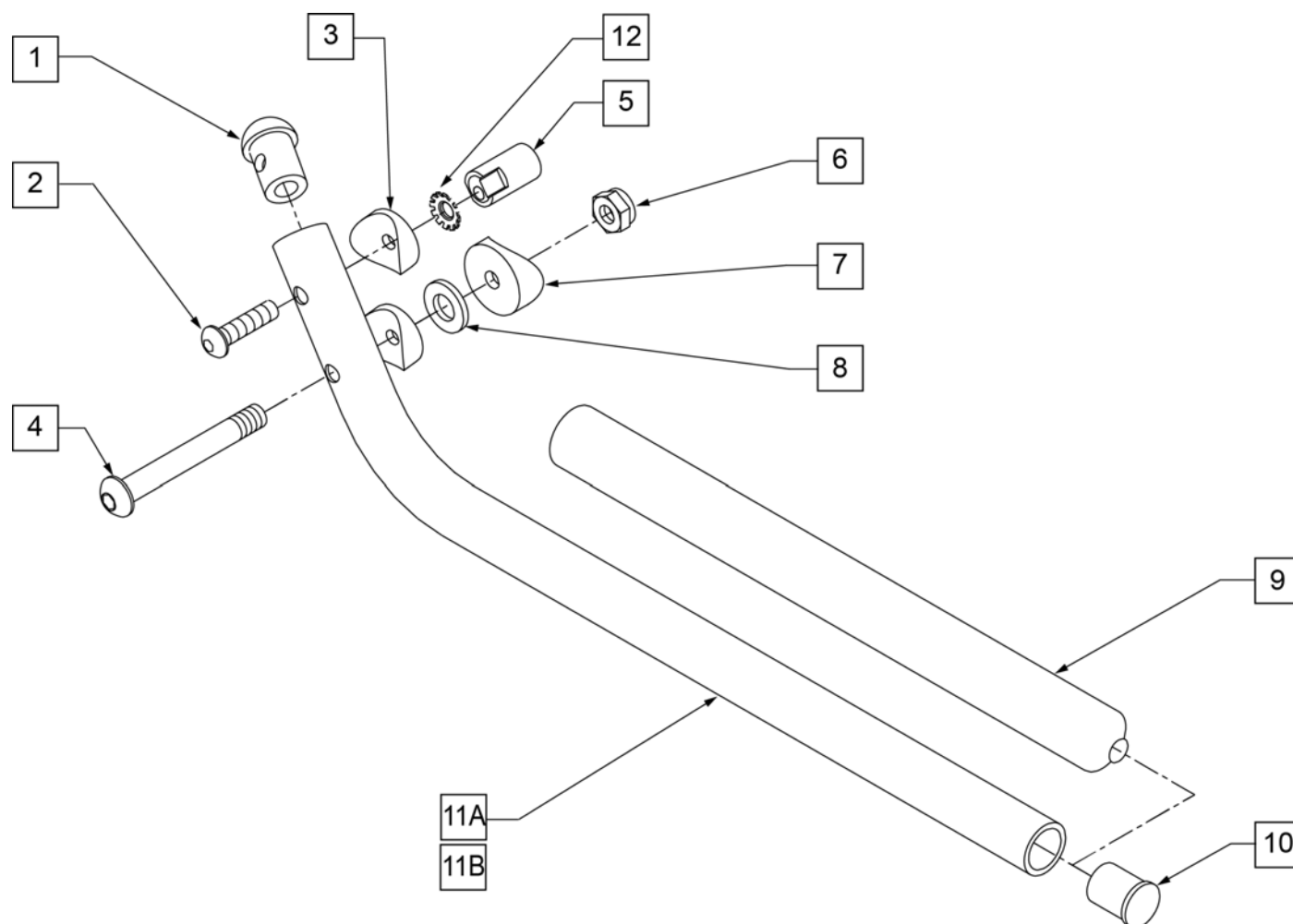


Pos.	Item Number	Description	Remarks
1	561034	SHORT DOME PLUG	
2	107028	1/4-28 x 1 1/2 BH C/S	
3	561003	3/4" SADDLE THIN	
4	106966	1/4-28 x 2 1/4 BH C/S	
5	483041	FLIP-UP ARM CAM ZP/P5	
6	106935	1/4-28 NYLOCK NUT THIN	
7	561004	1" SADDLE 1/4" HOLE	
8	106944	1/4 AN WASHER FLAT THIN	
9	560599	8 x .705 x .200 FOAM GRIP	
10	561000-QU	3/4" INSERT PLUG .083	
12	267350	1/4" WASHER INT LOCK ZN	
1-11A,12	880268	ZP FLIP-UP ARM ASSEMBLY SEAT DEPTH 10"-13"	RIGHT
1-11A,12	880269	ZP FLIP-UP ARM ASSEMBLY SEAT DEPTH 10"-13"	LEFT
1-11A,12	900446	FLIP-UP ARM ASSEMBLY 10"-13" ZP	PAIR
1-11B,12	880270	FLIP-UP ARM ASSEMBLY SEAT DEPTH 14"-16" ZP/P5	RIGHT
1-11B,12	880271	FLIP-UP ARM ASSEMBLY SEAT DEPTH 14"-16" ZP/P5	LEFT
1-11B,12	900448	FLIP-UP ARM ASSEMBLY	PAIR

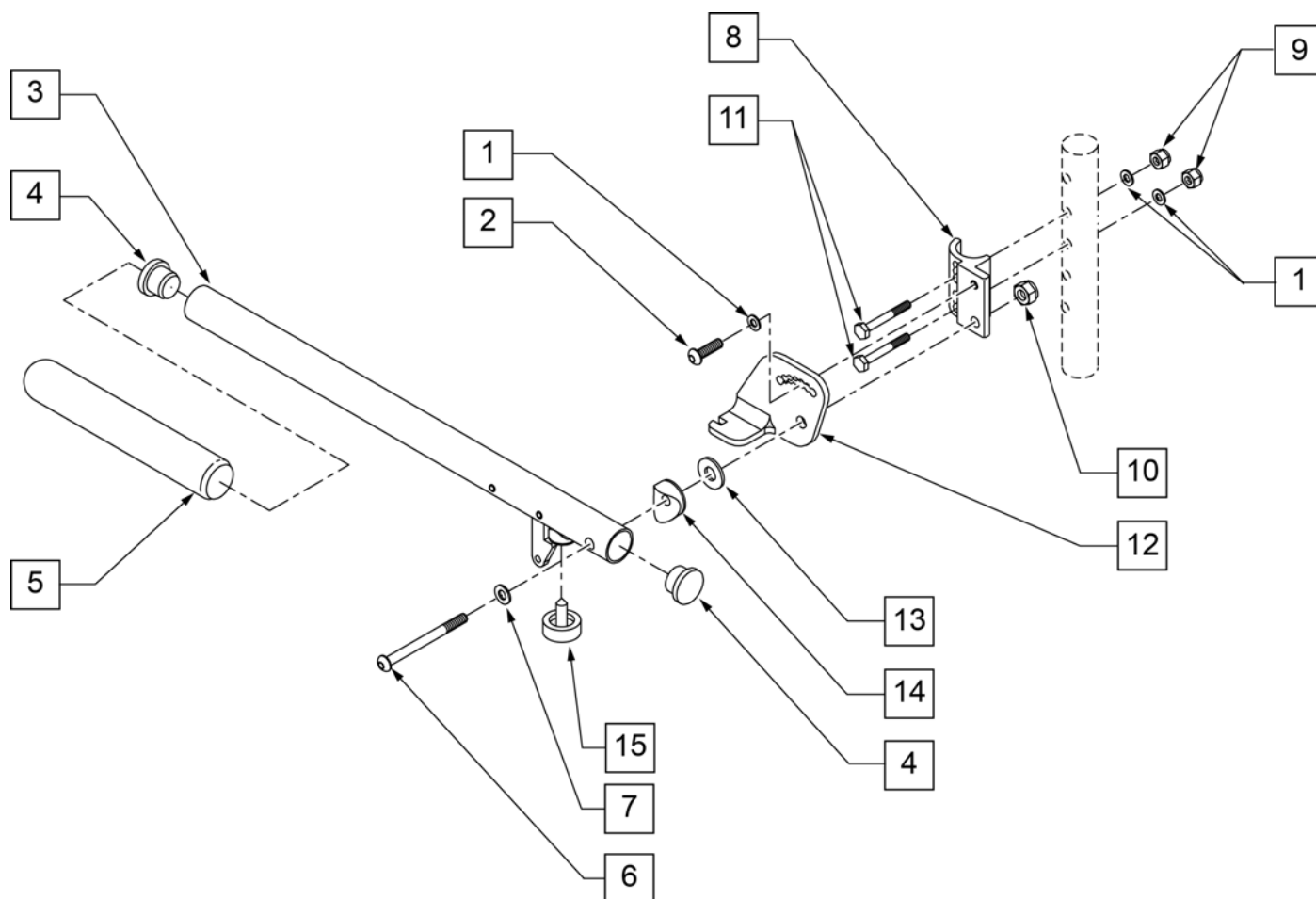


Pos.	Item Number	Description	Remarks
1	106943	1/4 AN WASHER FLAT	
2	107025	1/4-28 x 1/2 BH C/S	
4	561002	1" INSERT PLUG .083	
5	560640	9 x .93 x .155 FOAM GRIP	
6	106983	5/16-18 x 2 BH C/S SHT THREAD	
7	106947	5/16 WASHER FLAT THIN	
8	550168	ADJ LOCK FLIP-UP ARM MNT R	
8	550169	ADJ LOCK FLIP-UP ARM MNT L	
9	106935	1/4-28 NYLOCK NUT THIN	
10	202117	5/16-18 NYLOCK NUT THICK BLK	
11	106972	1/4-28 x 1 1/2 G8 HEX C/S	
12	550474	ADJ FLIP-BACK ARM BRACKET R	
12	550475	ADJ FLIP BACK ARM BRACKET L	
13	561016	1" WASHER 5/16" HOLE	
14	561006	1" SADDLE 5/16" HOLE	
15	560252	1" COVED STEM BUMPER	
3-5	880274	ADJ FLIP-BACK TUBE W/LATCH	
1,2,6,8-10	881272	ADJ LOCK F-UP ARMREST RETRO R	USE FOR ARMREST PRIOR TO 7/22/96
1,2,6,8-10	881273	ADJ LOCK F-UP ARMREST RETRO L	USE FOR ARMREST PRIOR TO 7/22/96
1-15	880272	ADJ FLIP BACK ARMREST R	
1-15	880273	ADJ FLIP BACK ARMREST L	
1-15	900449	ADJ LOCK FLIP-UP ARMREST ASSM	PAIR

NOTE: FOR ARMRESTS PRIOR TO 7/22/96 USE RETRO FIT KIT TO REPLACE ADJUSTABLE FLIP-UP ARM CLAMP.

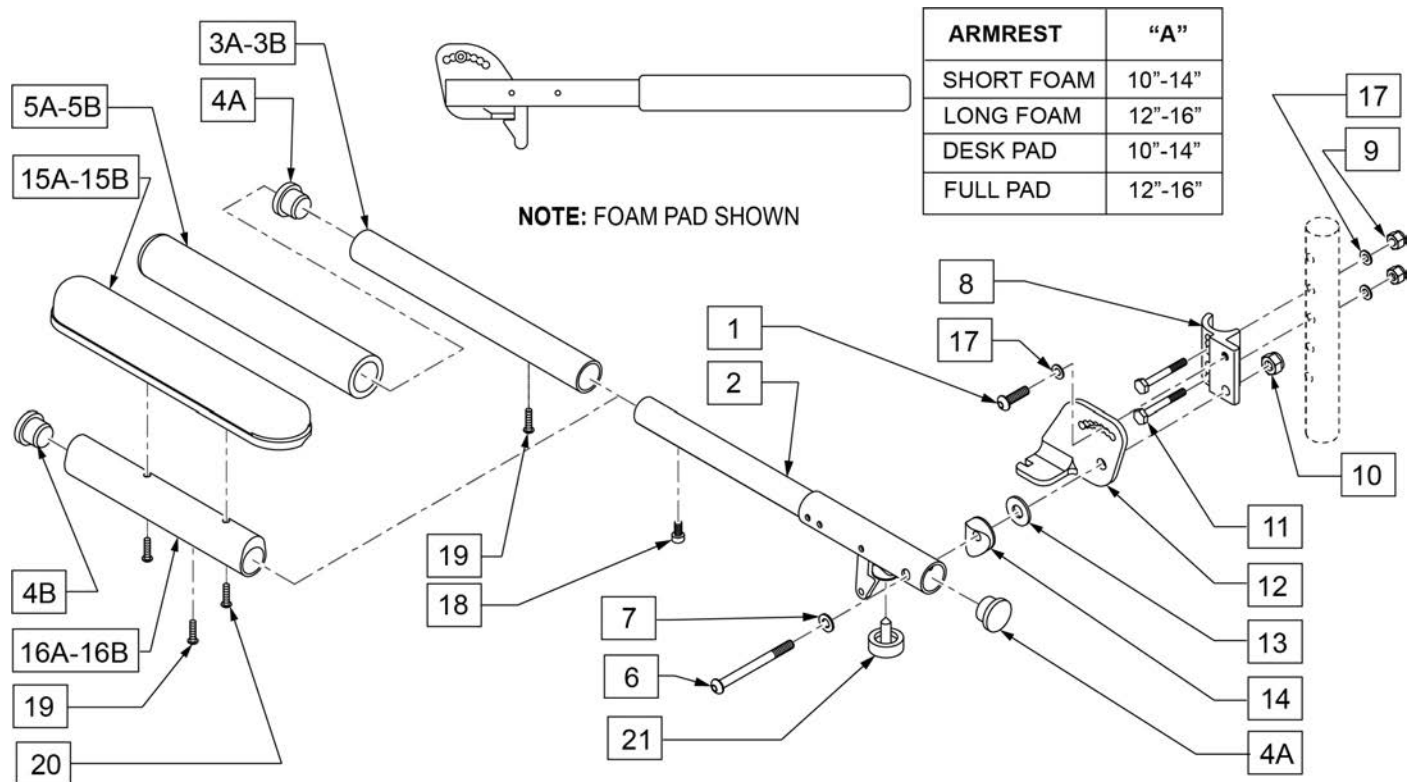


Pos.	Item Number	Description	Remarks
1	107025	BHCS 1/4-28 x 1/2 ROHS	
4A	561002	1" INSERT PLUG .083	
4B	561062	1" INSERT PLUG .065	
5A	560299	7 x .940 x .20 FOAM GRIP	SHORT PAD
5B	560640	9" X .93" X .155" FOAM GRIP	LONG PAD
6	106983	BHCS 5/16-18 x 2 SHLDR ROHS	
7	106947	WASHER FLAT THIN AN 5/16 ROHS	
8	550168	ADJUSTABLE LOCK FLIP-UP ARM MOUNT	RIGHT
8	550169	ADJUSTABLE LOCK FLIP-UP ARM MOUNT	LEFT
9	106935	NYLOCK NUT THIN 1/4 -28 ROHS	
10	202117	5/16-18 NYLOCK NUT THICK BLK	
11	106972	HHCS 1/4-28 x 1 1/2 ROHS	
12	550474	ADJUSTABLE FLIP-BACK ARM BRACKET	RIGHT
12	550475	ADJUSTABLE FLIP-BACK ARM BRACKET	LEFT
13	561016	1" WASHER 5/16" HOLE	
14	561006	1" SADDLE 5/16" HOLE	
15A	561024	ARMREST PAD 10" DESK	
15B	561031	ARMREST PAD 14" FULL	
17	106943	WASHER FLAT AN 1/4 ROHS	
18	107036	SHCS 8 -32 x 3/8 ROHS	
19	225110	10-32 X 1/2" SET SCREW	
20	107022	BHCS 10-32 X 5/8 ROHS	
21	560252	1" COVED STEM BUMPER	
3B, 4A, 5B, 19	882074	FOAM PAD LONG ADJUSTABLE LENGTH	LONG PAD ASSM ONLY

Pos.	Item Number	Description	Remarks
3A, 4A, 5A, 19	882075	FOAM PAD LONG ADJUSTABLE LENGTH	SHORT PAD ASSM ONLY
4B,15B,16B,19,20	882070	FULL PAD ADJUSTABLE LENGTH ASSM	RIGHT FULL PAD ASSM ONLY
4B,15B,16B,19,20	882071	FULL PAD ADJUSTABLE LENGTH ASSM	LEFT FULL PAD ASSM ONLY
4B,15A,16A,19,20	882072	DESK PAD ADJUSTABLE LENGTH ASSM	RIGHT DESK PAD ASSM ONLY
4B,15A,16A,19,20	882073	DESK PAD ADJUSTABLE LENGTH ASSM	LEFT DESK PAD ASSM ONLY
1,2,6-14,17,21	882088	ADJ LENGTH LOCKING FLIP-UP ASSM R	RIGHT ARM ASSM W/O PAD
1,2,6-14,17,21	882089	ADJ LENGTH LOCKING FLIP-UP ASSM R	LEFT ARM ASSM W/O PAD
1,2,3B,4A,5B,6-14,17,19 -21	900454	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	PAIR - LONG FOAM ARM ASSM
1,2,3B,4A,5B,6-14,17,19 -21	881294	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	RIGHT - LONG FOAM ARM ASSM
1,2,3B,4A,5B,6-14,17,19 -21	881295	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	LEFT - LONG FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	900455	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	PAIR - SHORT FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	881296	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	RIGHT - SHORT FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	881297	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	LEFT - SHORT FOAM ARM ASSM
1,2,4B,6-14,15B,16B,17-21	900452	ADJ LENGTH LOCKING FLIP-UP ASSM FULL	PAIR - FULL PAD ARM ASSM
1,2,4B,6-14,15B,16B,17-21	881290	ADJ LENGTH LOCKING FLIP-UP FULL PAD	RIGHT - FULL PAD ARM ASSM
1,2,4B,6-14,15B,16B,17-21	881291	ADJ LENGTH LOCKING FLIP-UP FULL PAD	LEFT - FULL PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	900453	ADJ LENGTH LOCKING FLIP-UP ASSM DESK	PAIR - DESK PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	881292	ADJ LENGTH LOCKING FLIP-UP DESK PAD	RIGHT - DESK PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	881293	ADJ LENGTH LOCKING FLIP-UP DESK PAD	LEFT - DESK PAD ARM ASSM

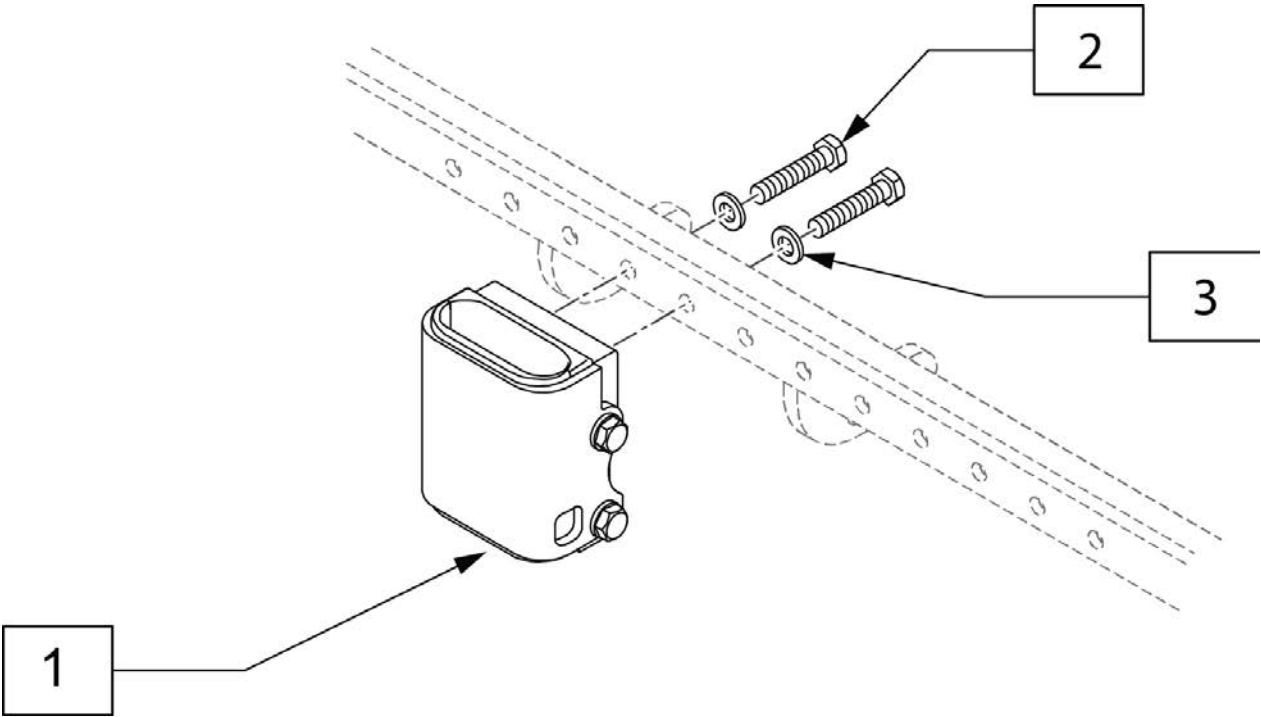

Pos.	Item Number	Description
2	118195	HHCS M6X1X30 PART PL
3	250104	WASHER FLAT M6
1-3	136461	SPHA BOLT THRU RECEIVER KIT


NOTE: KIDS BACKPACKS DO NOT HAVE ADJUSTABLE STRAPS W/ BUCKLES.

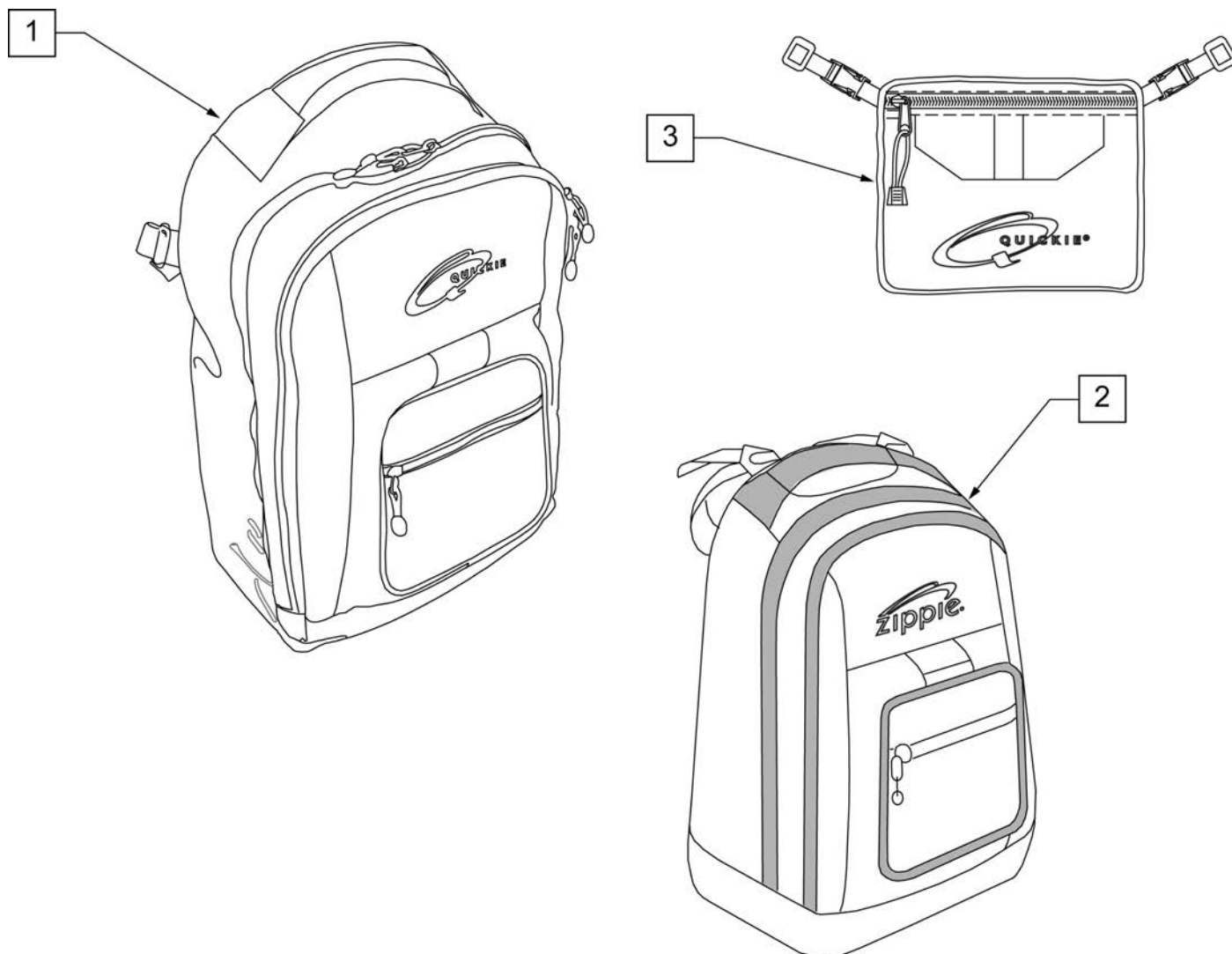

Pos.	Item Number	Description	Remarks
1	750115	BACK PACK QUICKIE	Overall dimensions: width 12.5", height 18.5", depth 6.5"
2	104572	BACKPACK ZIPPIE	Overall dimensions: width 12", height 16.5", depth 4.5"
3	750307	SEAT POUCH QUICKIE	Dimensions: width 8.5", height 6", depth top 2.5", depth bottom 3.5"

Note: some items listed may not be available with every chair model or in conjunction with another chair feature.


NOTE: when mounting Positioning Belts to the 5R, GT or GTI, it is recommended to attach the belt by using the Backrest Support Bracket Hardware.

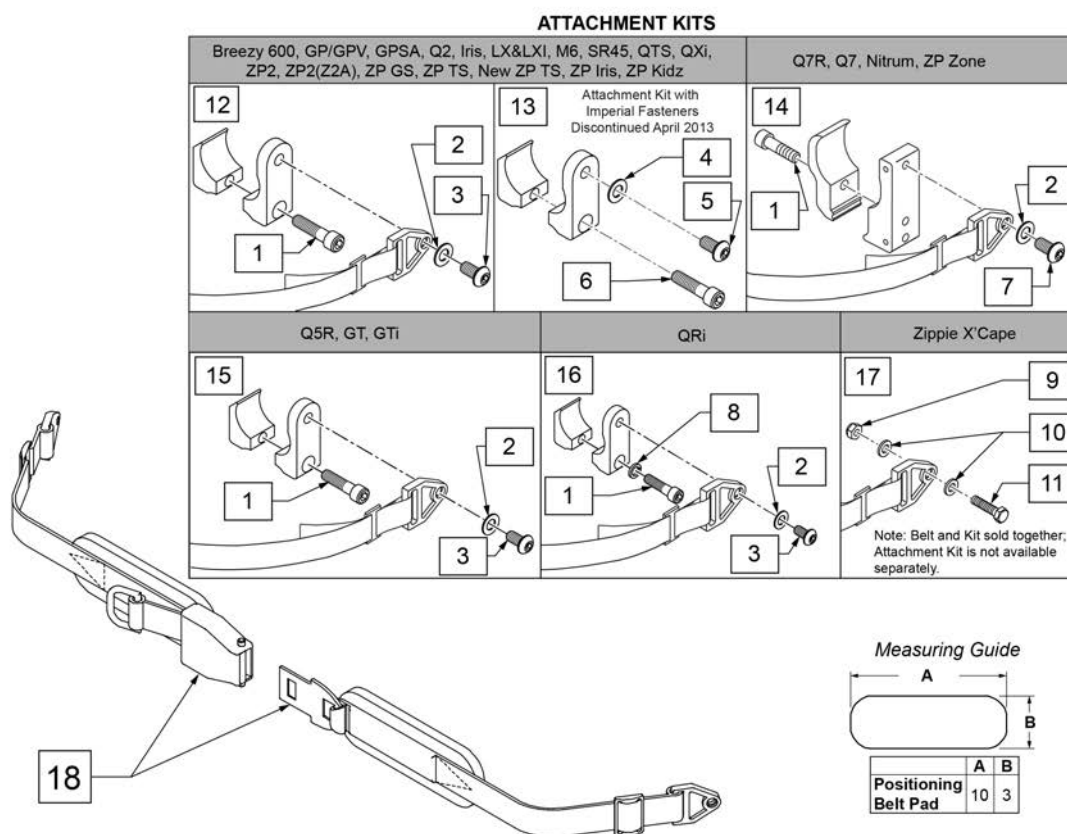

Pos.	Item Number	Description	Remarks
1	102263-003	SHCS M6 X 1 X 25 W/PL INORG	Metric
2	250104	WASHER FLAT M6	Metric
3	103185-005	BHCS M6 X 1 X 12 FULL PL	Metric
4	106943	WASHER FLAT AN 1/4 ROHS	Imperial (standard)
5	226154	1/4-20 X 1/2 BH C/S W/PATCH	Imperial (standard)
6	221116	1/4-20 X 1 SOC HD BLK	Imperial (standard)
7	230113	BHCS M6 X 1 X 12	
8	050129-001	LOCKWASHER HIGH COLLAR M6	
9	104381	NUT HEX NYLOCK M6 X 1.0	
10	107383	WASHER FLAT M6	
11	102268-002	HHCS M6 X 1 X 35 BLK	
12	120750	PSTN BELT CLAMP ASSM PR	Metric; Pair
13	900116	POSITION BELT ATTACHMENT KIT	Set of 2; Imperial (standard)
14	116268	OVAL CLAMP ASSM KIT	Kit contains 1 Clamp


Pos.	Item Number	Description	Remarks
15	159122	POSITIONING BELT CLAMPS PR	Pair
16	123534	POS BELT CLAMP KIT RPO	Set of 2
18	450418-QU	BELT 2" AIRCRAFT BUCKLE	Max length of belt 64"
18	450423-QU	BELT 2" AIRCRAFT BUCKLE W/PAD	Max length of belt 64"
18	450424	BELT 2" AIRCRAFT BKL LONG W/PAD	Max length of belt 72"
18	450425	BELT 2" AIRCRAFT BKL LONG	Max length of belt 72"
18	450419-QU	BELT 2" AIRCRAFT BKL W/QUAD LP	Max length of belt 64"
18	468419	BELT 2" AIRCRAFT BKL W/QD LPCZ	Max length of belt 64"
17,18	136464	AIRCRAFT HIP BELT 2" KIT	ZP Xcape: includes Belt & Attachment Kit
17,18	136465	AIRCRAFT HIP BELT 2" PAD KIT	ZP Xcape: includes Belt & Attachment Kit

NOTE: when mounting Positioning Belts to the 5R, GT or GTi, it is recommended to attach the belt by using the Backrest Support Bracket Hardware.

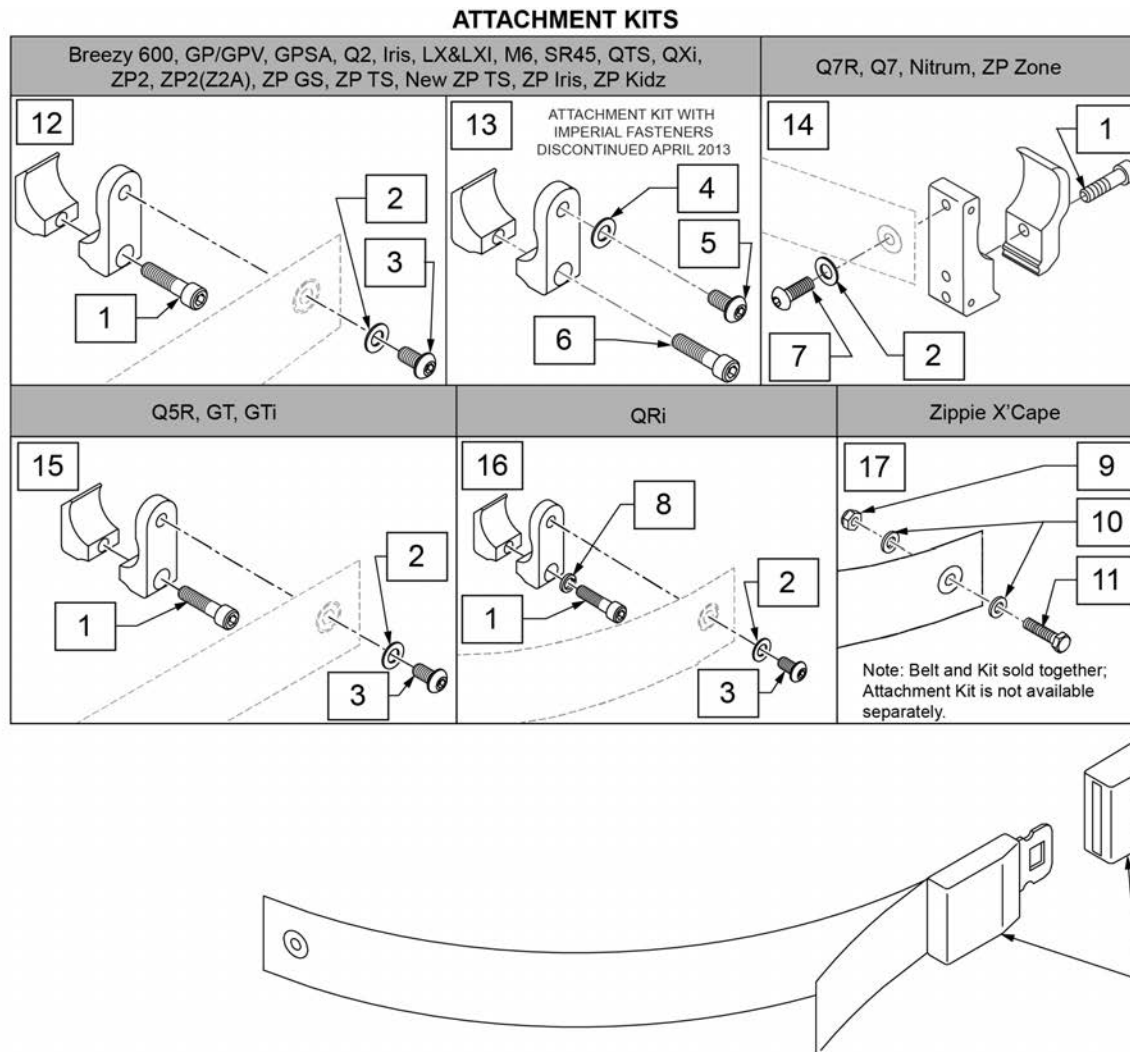


Pos.	Item Number	Description	Remarks
1	102263-003	SHCS M6 X 1 X 25 W/PL INORG	Metric
2	250104	WASHER FLAT M6	Metric
3	103185-005	BHCS M6 X 1 X 12 FULL PL	Metric
4	106943	WASHER FLAT AN 1/4 ROHS	Imperial (standard)
5	226154	1/4-20 X 1/2 BH C/S W/PATCH	Imperial (standard)
6	221116	1/4-20 X 1 SOC HD BLK	Imperial (standard)
7	230113	BHCS M6 X 1 X 12	
8	050129-001	LOCKWASHER HIGH COLLAR M6	
9	104381	NUT HEX NYLOCK M6 X 1.0	
10	107383	WASHER FLAT M6	
11	102268-002	HHCS M6 X 1 X 35 BLK	
12	120750	PSTN BELT CLAMP ASSM PR	Metric; Pair
13	900116	POSITION BELT ATTACHMENT KIT	Set of 2; Imperial (standard)
14	116268	OVAL CLAMP ASSM KIT	Kit contains 1 Clamp
15	159122	POSITIONING BELT CLAMPS PR	

Pos.	Item Number	Description	Remarks
16	123534	POS BELT CLAMP KIT RPO	Set of 2
18	750276	1 1/2" POSITION BELT	AUTOSTYLE-64" Max length of belt.
16,18	124269	AUTO POS BELT W/CLAMPS RPO	QRI: includes Belt & Attachment Kit
17,18	136463	AUTO HIP BELT 1.5" KIT	ZP Xcape: includes Belt & Attachment Kit

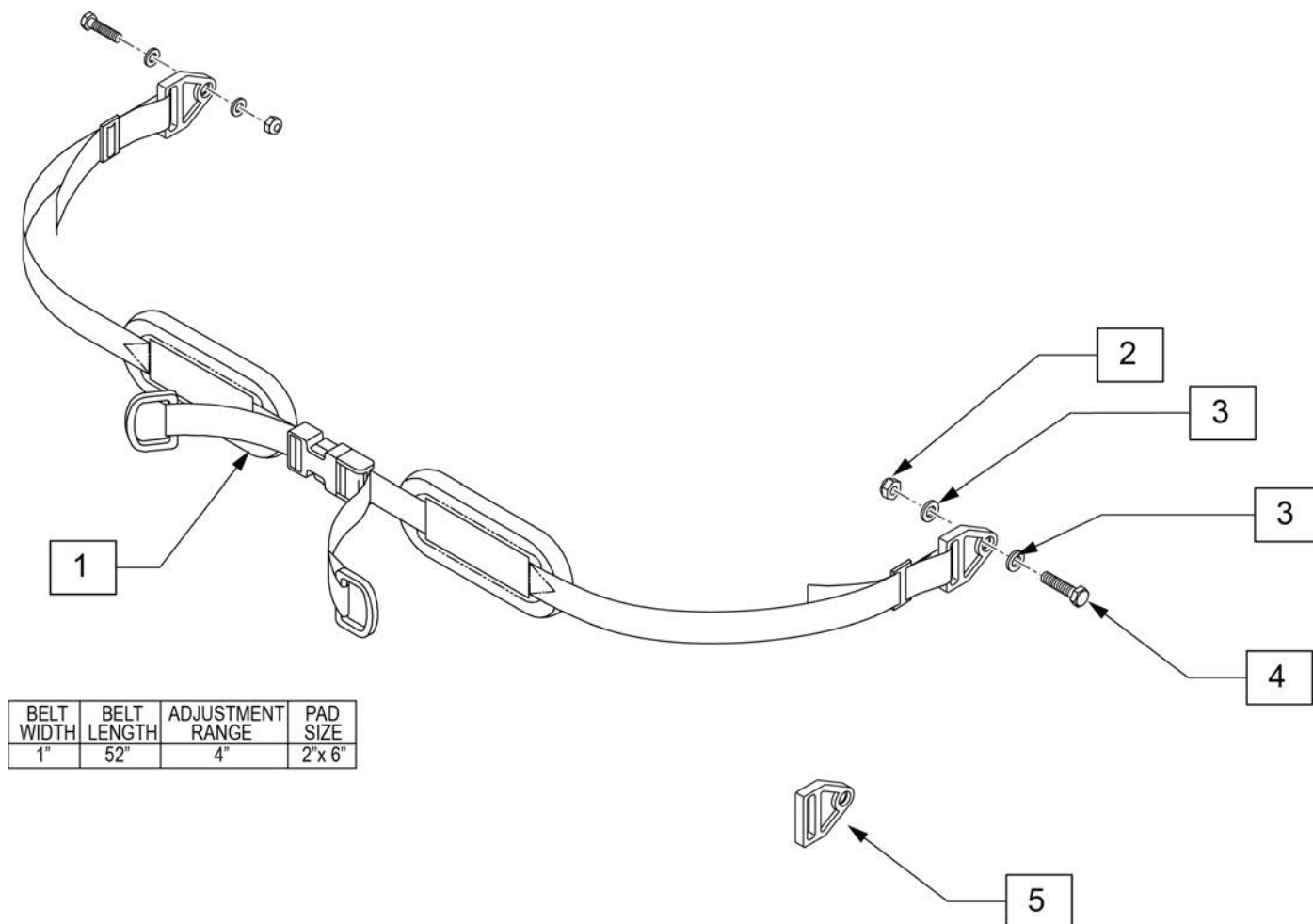


BELT WIDTH	BELT LENGTH	ADJUSTMENT RANGE	PAD SIZE
1"	52"	4"	2"x 6"


Pos.	Item Number	Description
1	450420-QU	BELT 1" DUAL ADJ. SQZ BKL PAD
2	104381	NUT HEX NYLOCK M6X1.0
3	107383	WASHER FLAT M6
4	102268-002	HHCS M6 X 1 X 35 BLK
5	451116-QU	PLASTIC WEBBING GROMMET
1-5	136462	PADDED HIP BELT DUAL 1" KIT

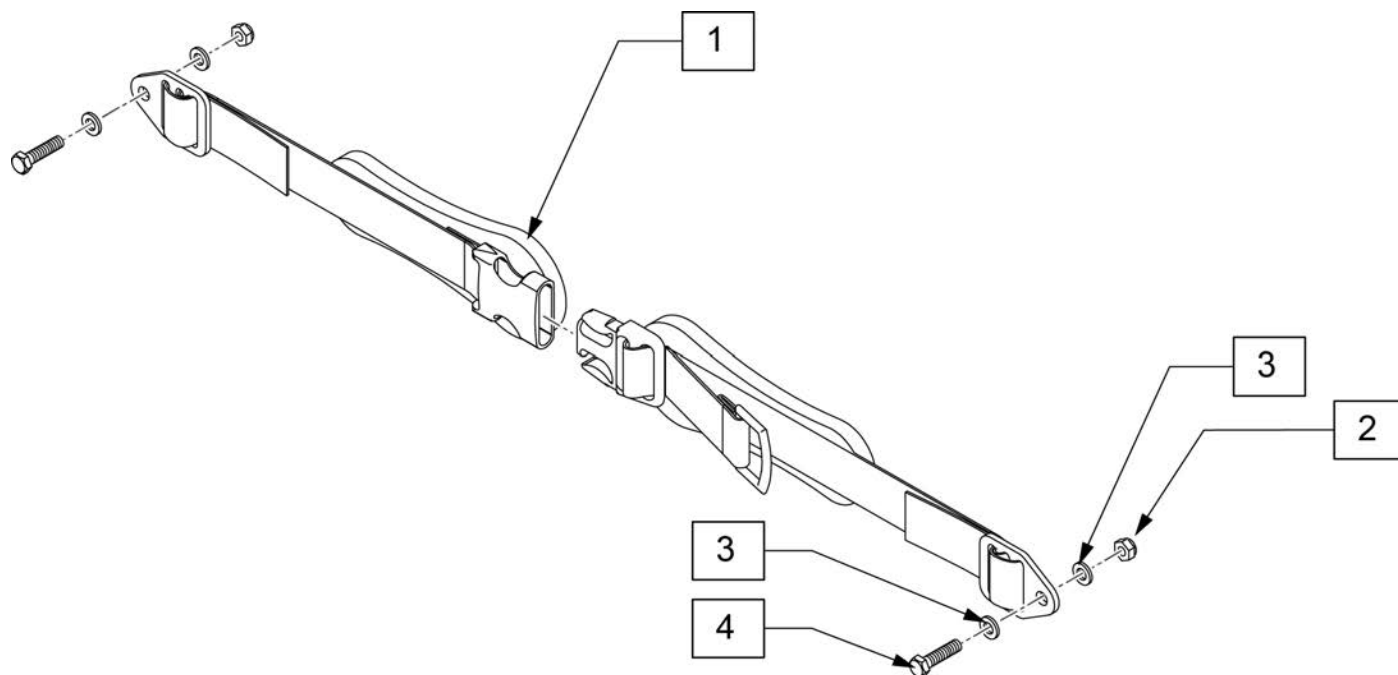

Pos.	Item Number	Description
1	APB5116	1" COMFORT FIT
2	104381	NUT HEX NYLOCK M6X1.0
3	107383	WASHER FLAT M6
4	102268-002	HHCS M6 X 1 X 35 BLK
1-4	136466	COMFORT FIT HIP BELT 1" PAD KIT

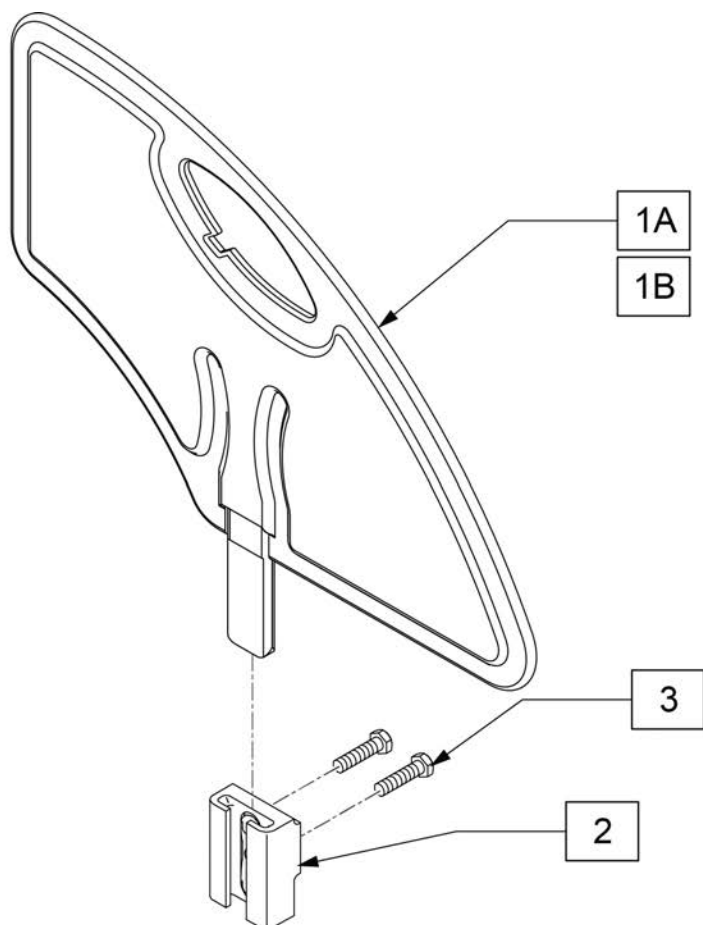

MEASURING GUIDE

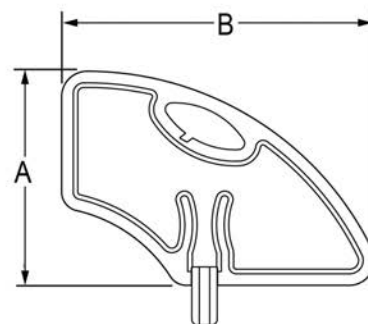



	LOW	STD
"A"	6.4"	8.4"
"B"	11.0"	12.3"

Pos.	Item Number	Description	Remarks
1A	104164-001	SIDE GUARD ASSY STD	Standard
1B	104164-002	SIDE GUARD ASSY SMALL	Low
3	118194	HHCS M6X1X25 PART PL	
1A-3	136468	PLST SIDE GRD KIT STD XCP	Pair
1B-3	136469	PLST SIDE GRD KIT SML XCP	Pair
2,3	136460	T-POST SIDEGUARD REC KIT	

NOTE: Right hand side shown.

Note Not available with Mag or Spinergy wheels. Not available with Projection style handrims.



SPOKE GUARDS INCLUDE 3 CLIPS
FOR ATTACHMENT

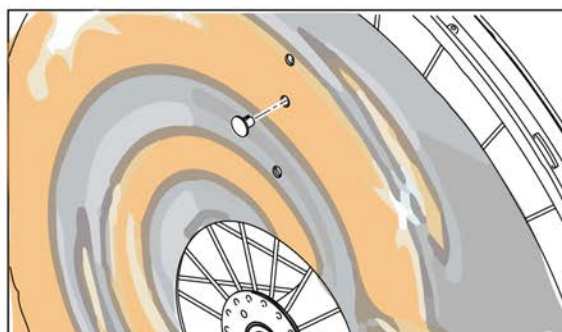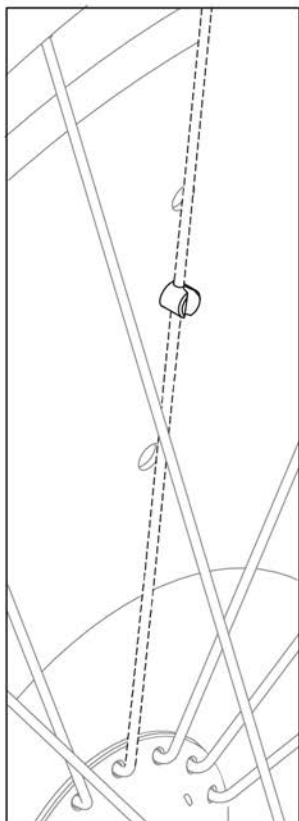

	1A	SPOKE GUARD 22IN BASKETBALL		5A	SPOKE GUARD 22 IN CHROME WHEELS
	1B	SPOKE GUARD 24IN BASKETBALL		5B	SPOKE GUARD 24IN CHROME WHEELS
	2A	SPOKE GUARD 22IN GRAFFITI		6A	SPOKE GUARD 22IN PRINCESS
	2B	SPOKE GUARD 24IN GRAFFITI		6B	SPOKE GUARD 24IN PRINCESS
	3A	SPOKE GUARD 22IN SUPERHERO		7A	SPOKE GUARD 22IN HAPPY LION
	3B	SPOKE GUARD 24IN SUPERHERO		7B	SPOKE GUARD 24IN HAPPY LION
	4A	SPOKE GUARDS 22IN HOT WHEELS		8A	SPOKE GUARD 22IN GALACTICA
	4B	SPOKE GUARDS 24IN HOT WHEELS		8B	SPOKE GUARD 24IN GALACTICA
	9A	SPOKE GUARD 22IN CHEESE		10A	SPOKE GUARD 22IN ZEBRA
	9B	SPOKE GUARD 24IN CHEESE		10B	SPOKE GUARD 24IN ZEBRA

Pos.	Item Number	Description	Remarks
1A	129250	SPOKE GUARD 22IN BASKETBALL 2	Each
1B	129251	SPOKE GUARD 24IN BASKETBALL 2	Each
2A	129252	SPOKE GUARD 22IN GRAFFITI	Each
4B	129253	SPOKE GUARD 24IN GRAFFITI	Each
3A	129254	SPOKE GUARD 22IN SUPERHERO	Each
3B	129255	SPOKE GUARD 24IN SUPERHERO	Each
4A	129256	SPOKE GUARDS 22IN HOT WHEELS	Each
4B	129257	SPOKE GUARDS 24IN HOT WHEELS	Each
5A	129258	SPOKE GUARDS 22IN CHROME WHEELS	Each
5B	129259	SPOKE GUARDS 24IN CHROME WHEELS	Each
6A	129328	SPOKE GUARD 22IN PRINCESS	Each
6B	129329	SPOKE GUARD 24IN PRINCESS	Each
7A	129330	SPOKE GUARD 22IN HAPPY LION	Each
7B	129331	SPOKE GUARD 24IN HAPPY LION	Each

Pos.	Item Number	Description	Remarks
8A	129332	SPOKE GUARD 22IN GALACTICA	Each
8B	129333	SPOKE GUARD 24IN GALACTICA	Each
9A	129334	SPOKE GUARD 22IN CHEESE	Each
9B	129335	SPOKE GUARD 24IN CHEESE	Each
10A	129336	SPOKE GUARD 22IN ZEBRA	Each
10B	129337	SPOKE GUARD 24IN ZEBRA	Each

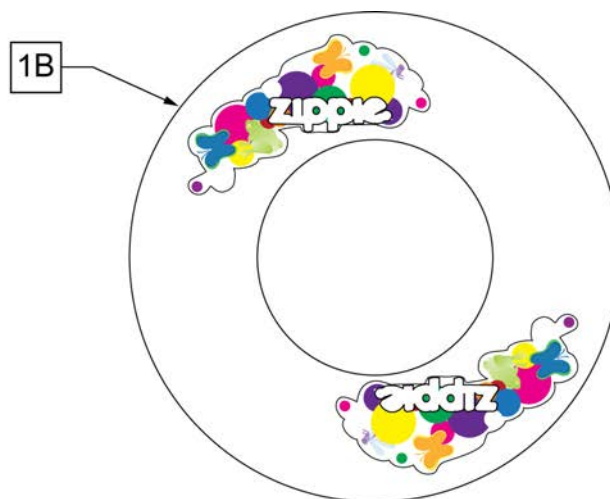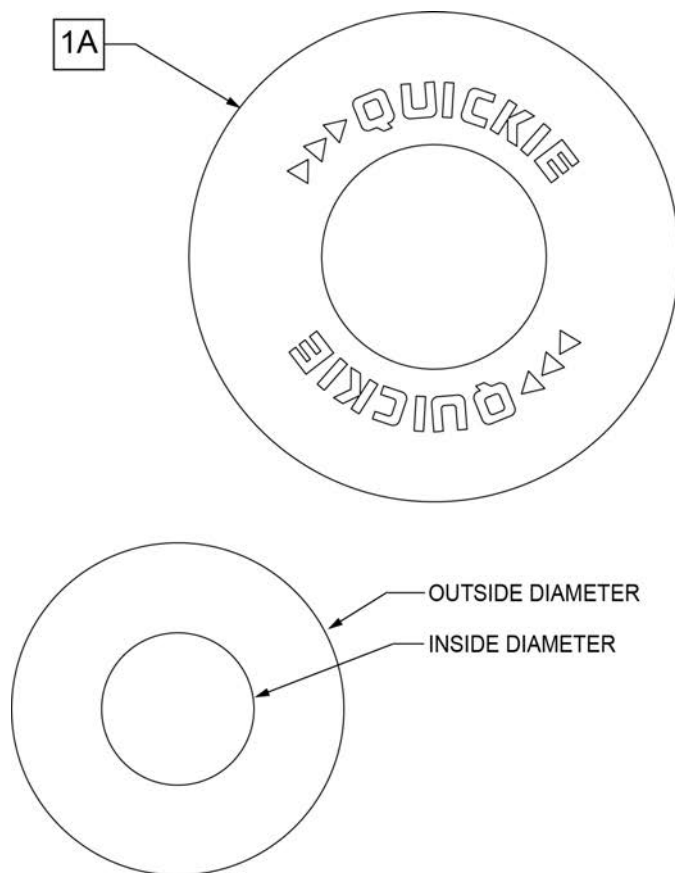

SPOKE GUARD MEASURING GUIDE


PART NUMBER	INSIDE DIAMETER	OUTSIDE DIAMETER
902001	8 1/4	15 7/8
902011	8 1/4	18 1/8
902021	10 1/4	19 5/8
902025	10 1/4	20 3/8
103660-001	8 1/4	16
103660-002	8 1/4	18
103663-001	10 1/4	20 1/2
103663-002	10 1/4	23 1/2

Pos.	Item Number	Description	Remarks
>>>>		BLACK SPOKE GUARDS	
1A	902001	SPK GRDS 20" QUICKIE BLK	Pair-BLACK-FITS 20" WHEELS
1A	902011	SPK GRDS 22" QUICKIE BLK	Pair-BLACK-FITS 22" WHEELS & 24" MOUNTAIN SPOKE WHEELS
1A	902021	SPK GRD 24" QUICKIE SCR MT BLK	Pair-BLACK-FITS 24" (540) MAG & SPOKE WHEELS
1A	902025	SPK GRD 24" QUICKIE TAB MT BLK	Pair-BLACK-FITS 25" (559) SPOKE WHEELS & 26" MOUNTAIN SPOKE WHEELS
>>>>		CLEAR SPOKE GUARDS	
1B	112644-001	SPOKE GUARD ZIPPIE 20"	Pair-CLEAR-FITS 20" WHEELS
1B	112644-002	SPOKE GUARD ZIPPIE 22"	Pair-CLEAR-FITS 22" WHEELS & 24" MOUNTAIN SPOKE WHEELS
1B	112644-003	SPOKE GUARD ZIPPIE 24" MAG	Pair-CLEAR-FITS 24" (540) MAG WHEELS
1B	112644-004	SPOKE GUARD ZIPPIE 24" SPOKE	Pair-CLEAR-FITS 24" (540) SPOKE WHEELS

Note: Not available with 5 Spoke Mag or Xtender wheels. Not available with Projection style handrims.


Note: when adding a Tray to an existing chair with Desk or Full Length Arm Pads, add 1" to current seat width to achieve the best tray size.

Note: Large and X-Large Trays have a round cut-out.

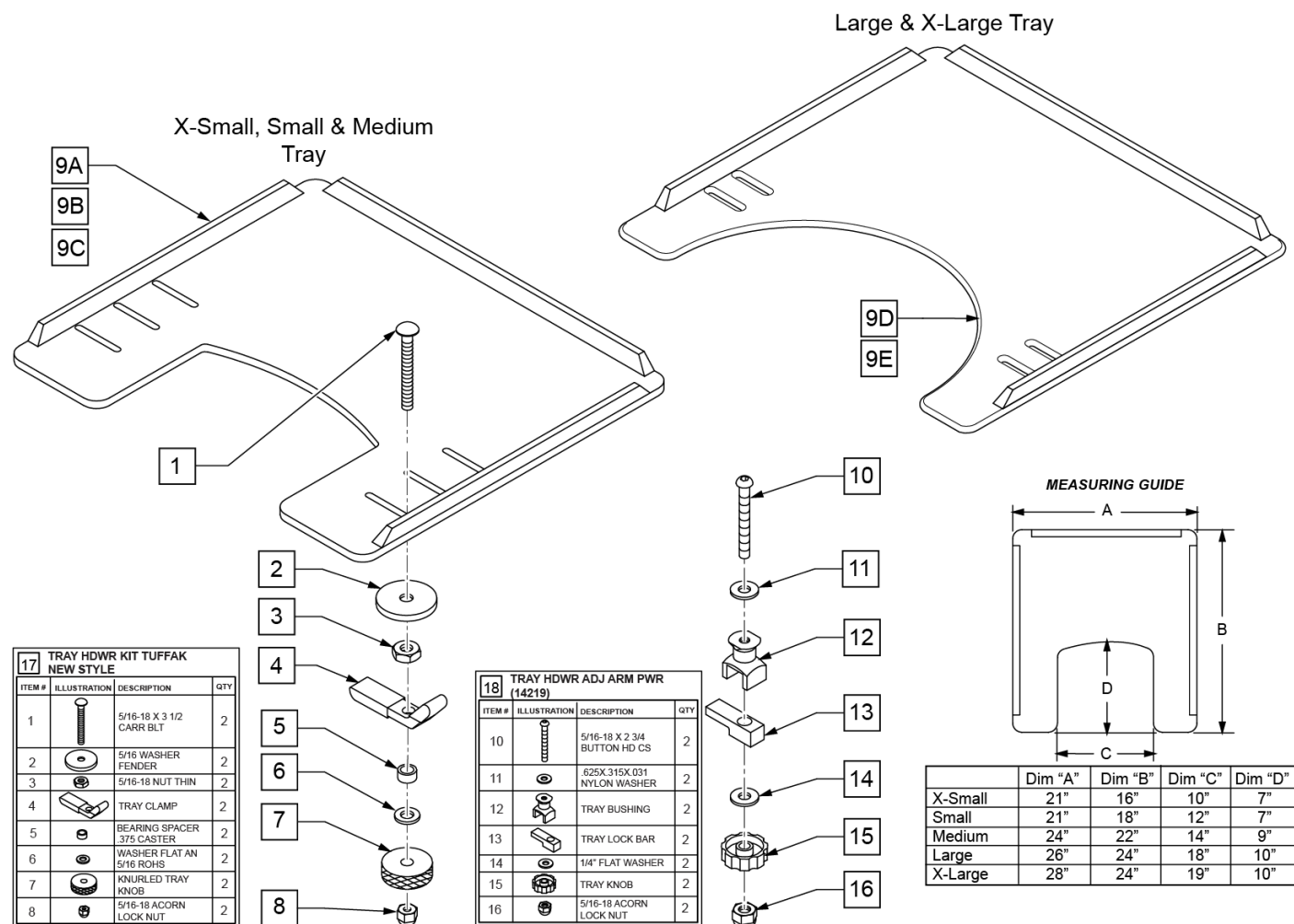

Pos.	Item Number	Description	Remarks
1	226199	5/16-18 x 3 1/2 CARR BOLT	
2	263112	5/16 WASHER FENDER	
3	202110	5/16-18 NUT THIN	
4	550400	TRAY CLAMP	
5	481113	SPACER 5/16 X .380	
6	263100	5/16 AN WASHER FLAT	
7	483026	KNURLED TRAY KNOB	
8	202114	5/16-18 ACORN LOCK NUT	
9A	562619	TRAY TOP EX-SM W/RAIL	
9B	562620	TRAY TOP SM W/RAIL	
9C	562621	TRAY TOP MED W/RAIL	
9D	562622	TRAY TOP LRG W/RAIL	
9E	562623	TRAY TOP X-LRG W/RAIL	

Pos.	Item Number	Description	Remarks
17	881051	TRAY HARDWARE KIT TUFFAK NEW STYLE	Pair
18	881053	TRAY HARDWARE ADJ ARM PWR (14219)	Pair
1-8,9A	902306	TRAY ASSEMBLY EX-SM TUFFAK (10 -12")	Fits 10"-12" wide chairs
1-8,9B	902303	TRAY ASSEMBLY SM TUFFAK (13 -14")	Fits 13"-14" wide chairs
1-8,9C	902304	TRAY ASSEMBLY MED TUFFAK (15 -17")	Fits 15"-17" wide chairs
1-8,9D	902305	TRAY ASSEMBLY LRG TUFFAK (18 -20")	Fits 18"-20" wide chairs
1-8,9E	902309	TRAY ASSEMBLY X-LRG TUFFAK (21 -22")	Fits 21"-22" wide chairs
9A,10-16	902330	TRAY ASSM X-SM TFK 10-12/CAM	Fits 10"-12" wide chairs
9B,10-16	902331	TRAY ASSM SM TUFFAK 13-14/CAM	Fits 13"-14" wide chairs
9C,10-16	902332	TRAY ASSM MED TUFFAK 15-17/CAM	Fits 15"-17" wide chairs
9D,10-16	902333	TRAY ASSM LRG TUFFAK 18-20/CAM	Fits 18"-20" wide chairs

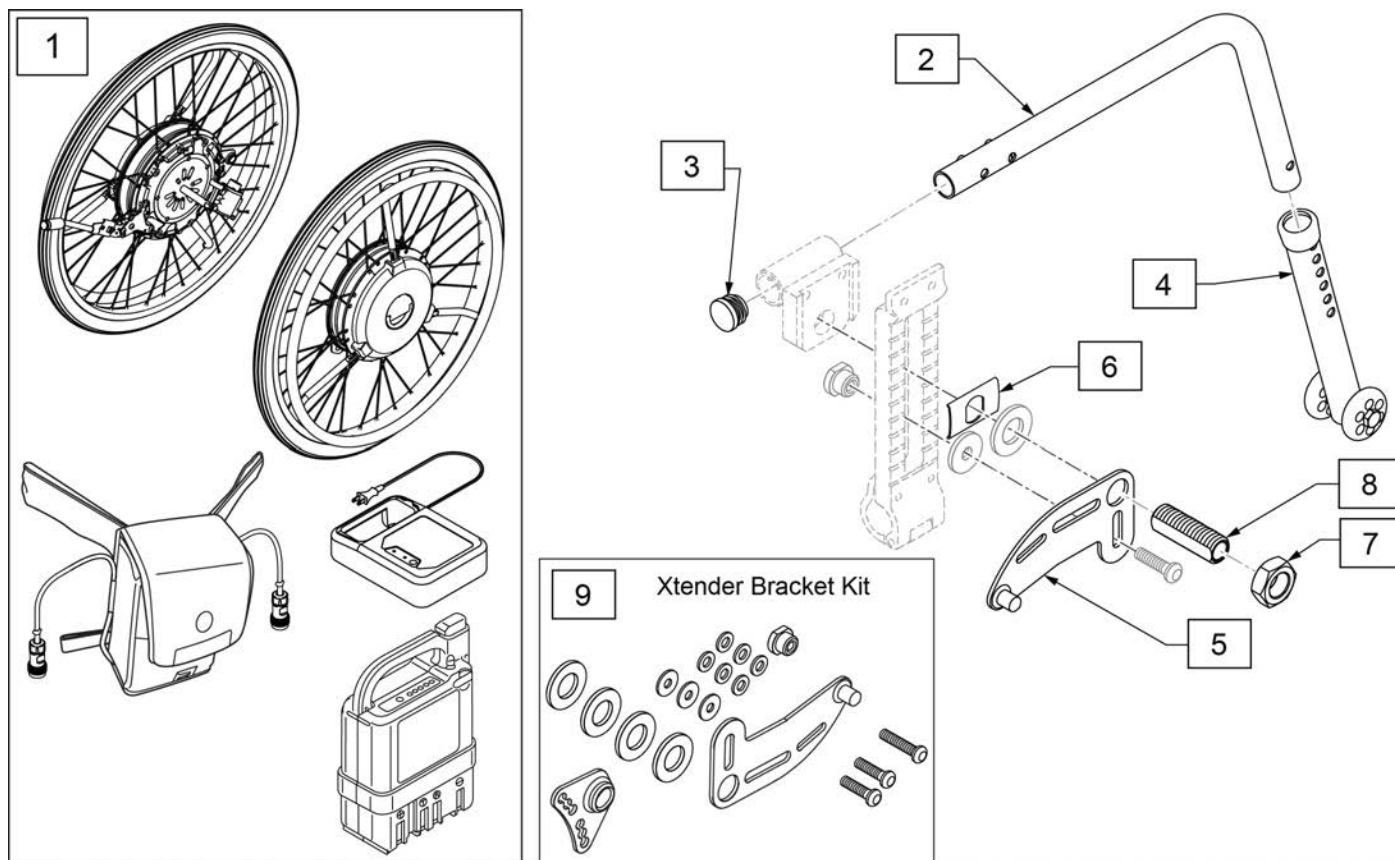


Pos.	Item Number	Description	Remarks
2	882974	ANTI-TIP TB EXTND REG R W/SNAP	Right, 24" wheel
2	882975	ANTI-TIP TB EXTND REG L W/SNAP	Left, 24" wheel
2	883008	MED ANTI TIP TUBE R W/SNAP BTN	Right, 22" wheel, down
2	883009	MED ANTI TIP TUBE L W/SNAP BTN	Left, 22" wheel, down
2	104221-006	ANTI TIP ARM LRG RGT	Right, 22" wheel, up
2	104221-005	ANTI TIP ARM LRG LFT	Left, 22" wheel, up
3	560735	7/8" ROUND TUBE PLUG	
4	104639	ANTI TIP ARM ADJ. MED	Up
4	104640	ANTI TIP ARM ADJ. LONG	Down
5	241950	JWX-2 XTENDER STOPPER ARM RH	Right
5	241951	JWX-2 XTENDER STOPPER ARM LH	Left
1,9	133982	JWX-2 24IN NI SS	
1,9	133983	JWX-2 24IN LI SS	
1,9	133984	JWX-2 24IN NI PLS	
1,9	133985	JWX-2 24IN LI PLS	
1,9	133986	JWX-2 22IN NI SS	
1,9	133987	JWX-2 22IN LI SS	
1,9	133988	JWX-2 22IN NI PLS	
6-8	136088	AXLE SLEEVE INDEX KIT	
9	241916	JWX-2 BRACKET KIT LH	Left
9	241917	JWX-2 BRACKET KIT RH	Right
1-5,9	161425	JWX-2 24 NI SS AT DWN	
1-5,9	161426	JWX-2 24 NI SS AT UP	
1-5,9	161427	JWX-2 24 LI SS AT DWN	
1-5,9	161428	JWX-2 24 LI SS AT UP	
1-5,9	161429	JWX-2 24 NI PLS AT DWN	

Pos.	Item Number	Description	Remarks
1-5,9	161430	JWX-2 24 NI PLS AT UP	
1-5,9	161431	JWX-2 24 LI PLS AT DWN	
1-5,9	161432	JWX-2 24 LI PLS AT UP	
1-5,9	161433	JWX-2 22 NI SS AT DWN	
1-5,9	161434	JWX-2 22 NI SS AT UP	
1-5,9	161435	JWX-2 22 LI SS AT DWN	
1-5,9	161436	JWX-2 22 LI SS AT UP	
1-5,9	161437	JWX-2 22 NI PLS AT DWN	
1-5,9	161438	JWX-2 22 NI PLS AT UP	


Note: The Yamaha Xtender is not transit approved and can not be added to a chair that is transit approved.

Note: If replacing or retrofitting an Xtender, please contact Customer Service to ensure compatibility.


Note: the 22" Plastic hand rims with Li Ion Battery has been discontinued

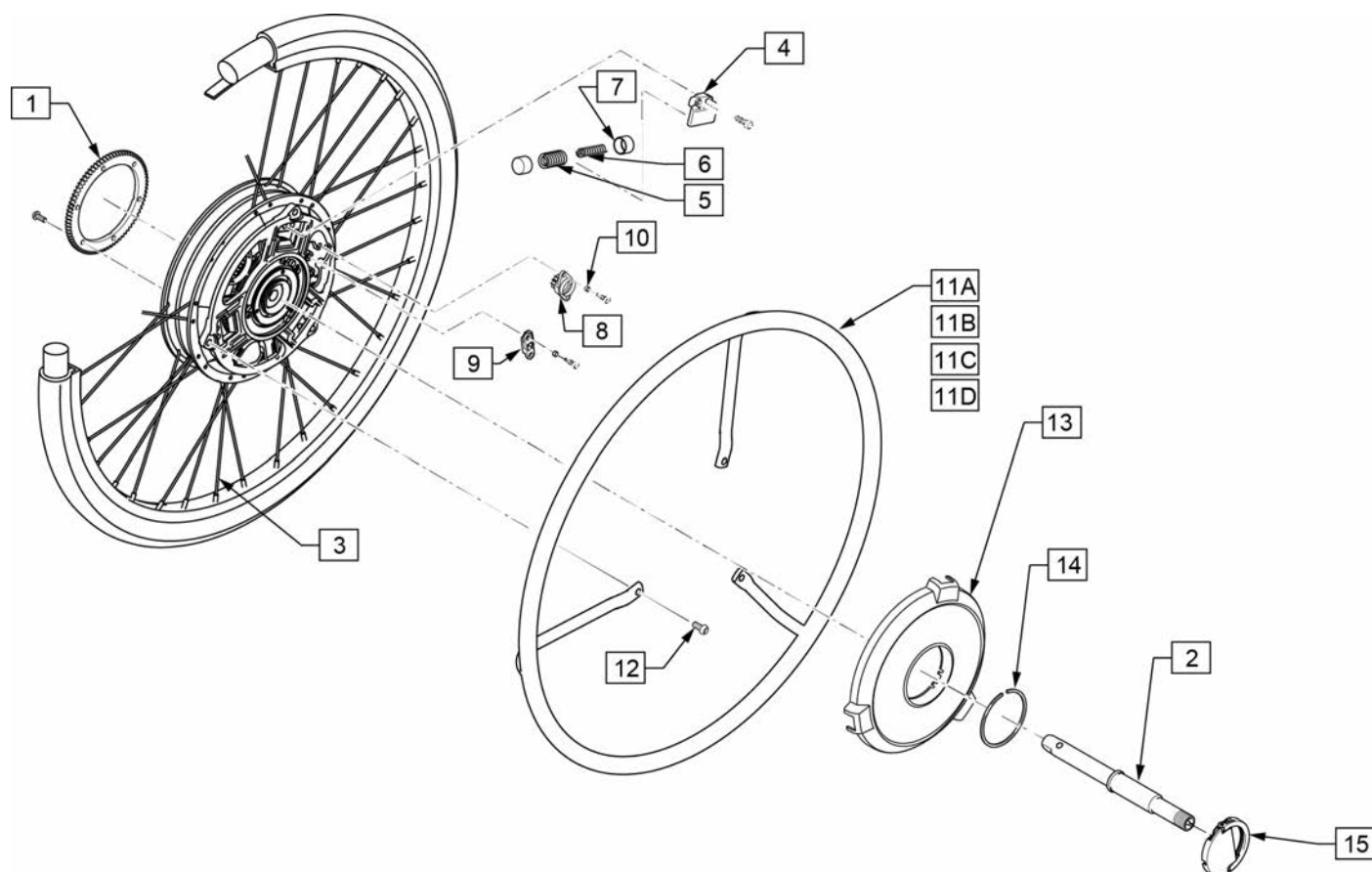


Pos.	Item Number	Description	Remarks
1	242261	GEAR, WHEEL 3	
2	242237	RR AXLE ASSY	
3	242235	RR SPOKE SET 22"	for 22" Wheel; 36 spokes & 36 nipples
3	242236	RR SPOKE SET 24"	for 24" Wheel; 36 spokes & 36 nipples
4	242255	RETAINER	
8	242252	DAMPER	
9	242270	ARM, MOVING	
10	242272	COLLAR	To fix Damper (item 8) & Arm (item 9)
11A,12	242231	HNDRM W/O COATING 22", 19" DIA	22" Stainless Steel Handrim w/3 fixing bolts
11B,12	242233	HNDRM W/ COATING 22", 19" DIA	22" Plastic Coated Handrim w/3 fixing bolts
11C,12	242232	HNDRM W/O COATING 24", 21" DIA	24" Stainless Steel Handrim w/3 fixing bolts
11D,12	242234	HNDRM W/ COATING 24", 21" DIA	24" Plastic Coated Handrim w/3 fixing bolts
13	134003	JWX-2 WHEEL CAP ASSEMBLY	Black, Hub cover w/decal
14	242256	SPRING CLIP 42MM	
15	242238	YAMAHA PLATE ASSY	Handle Plate for the unit detaching
5-9	242244	SPRING, LEVER RETURN E	

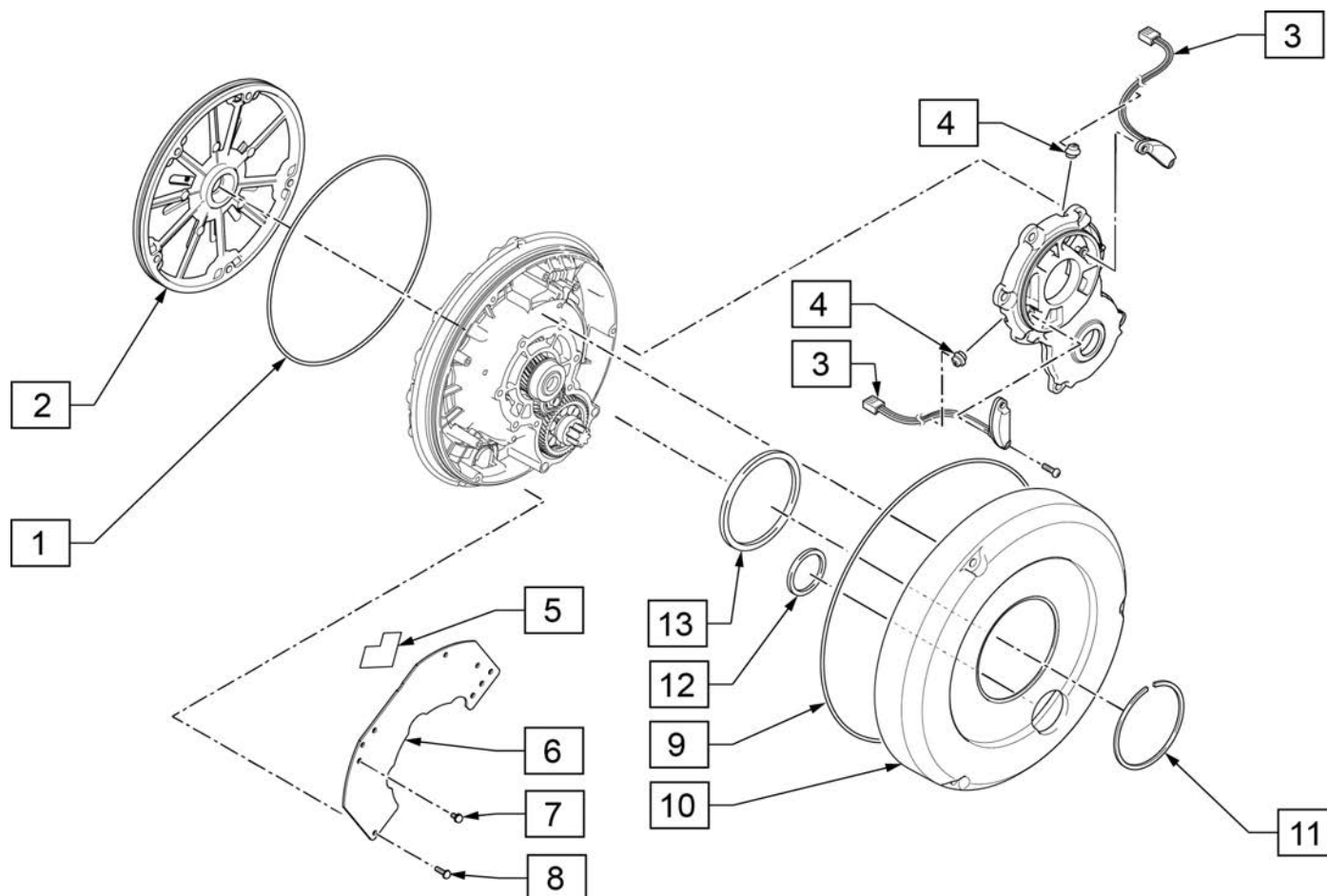


Pos.	Item Number	Description	Remarks
1	242250	JWX-2 O-RING 1	
2	242230	YAMAHA HSG ASSY	Unit Back Plate;
3,4	242242	SENSOR ASSY	
5-8	242273	PRINT BOARD ASSY 1	Pair, Main Electric Board
9	241969	JWX-2 SEAL 1	O-ring for Transparent Cover
10	242240	YAMAHA COVER, TRANSMISSION	Transparent Plastic Cover
11	242260	CLIP	C clip spring to fix Transparent Cover
12	242258	GASKET 1	urethane flat O-ring, small
13	242259	GASKET 2	urethane flat O-ring, large

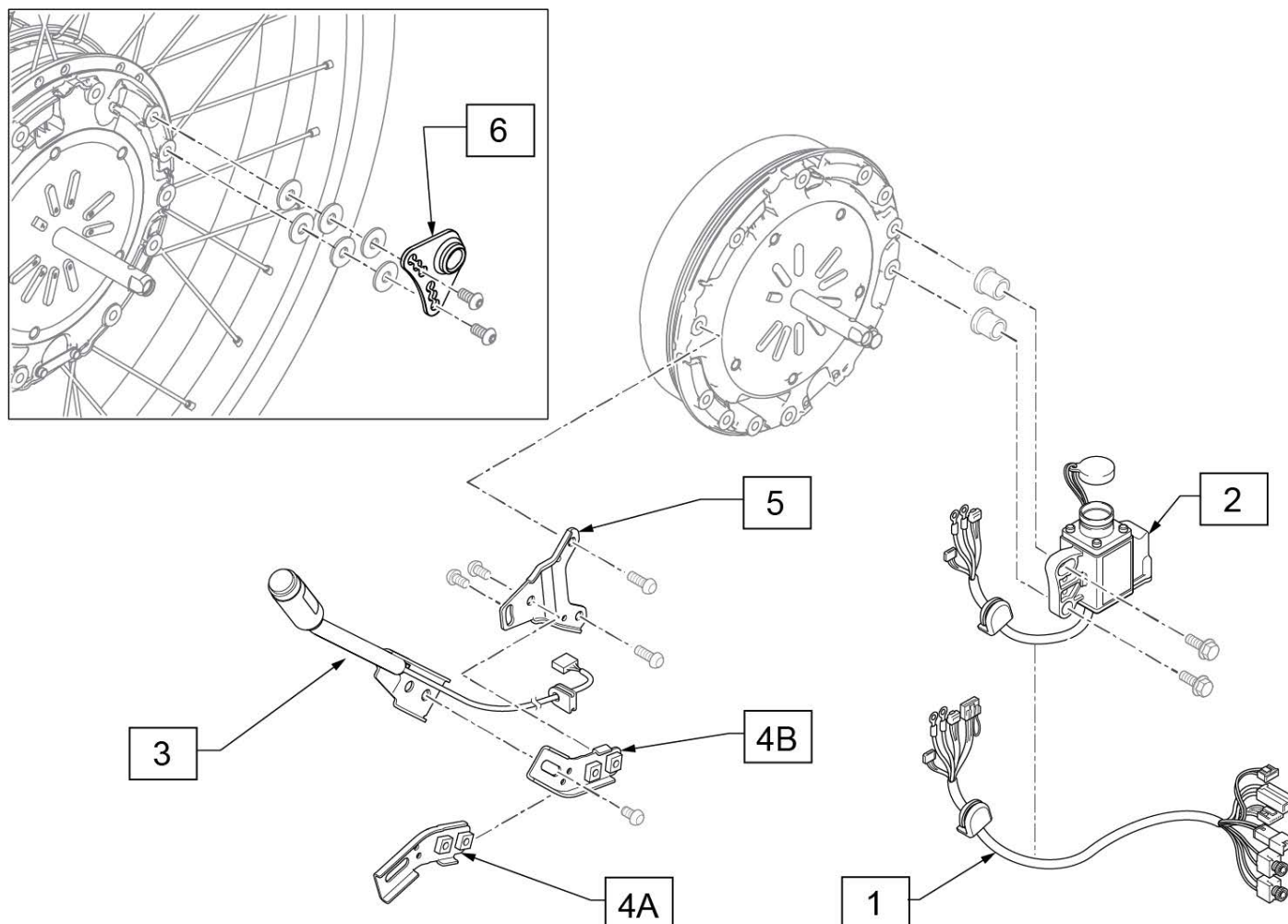


Pos.	Item Number	Description	Remarks
1	242271	WIRE 6	Connecting Wire from Battery Seat to Drive Unit
2	133999	JWX-2 WIRE HARNESS ASSY RH	Right Side Wheel
2	242245	JWX-2 WIRE HARNESS ASSY LH	Left Side Wheel
3	133995	JWX-2 MAIN SWITCH SUB ASSY	Power Switch Main Unit
4A	133993	JWX-2 POWER SWITCH ANGLE A	Angle Bracket (A)
4B	133994	JWX-2 POWER SWITCH ANGLE B	Angle Bracket (B)
5	133992	JWX-2 XTENDER "A" BRACKET	
6	241957	JWX-2 XTENDER PLATE LOCK	

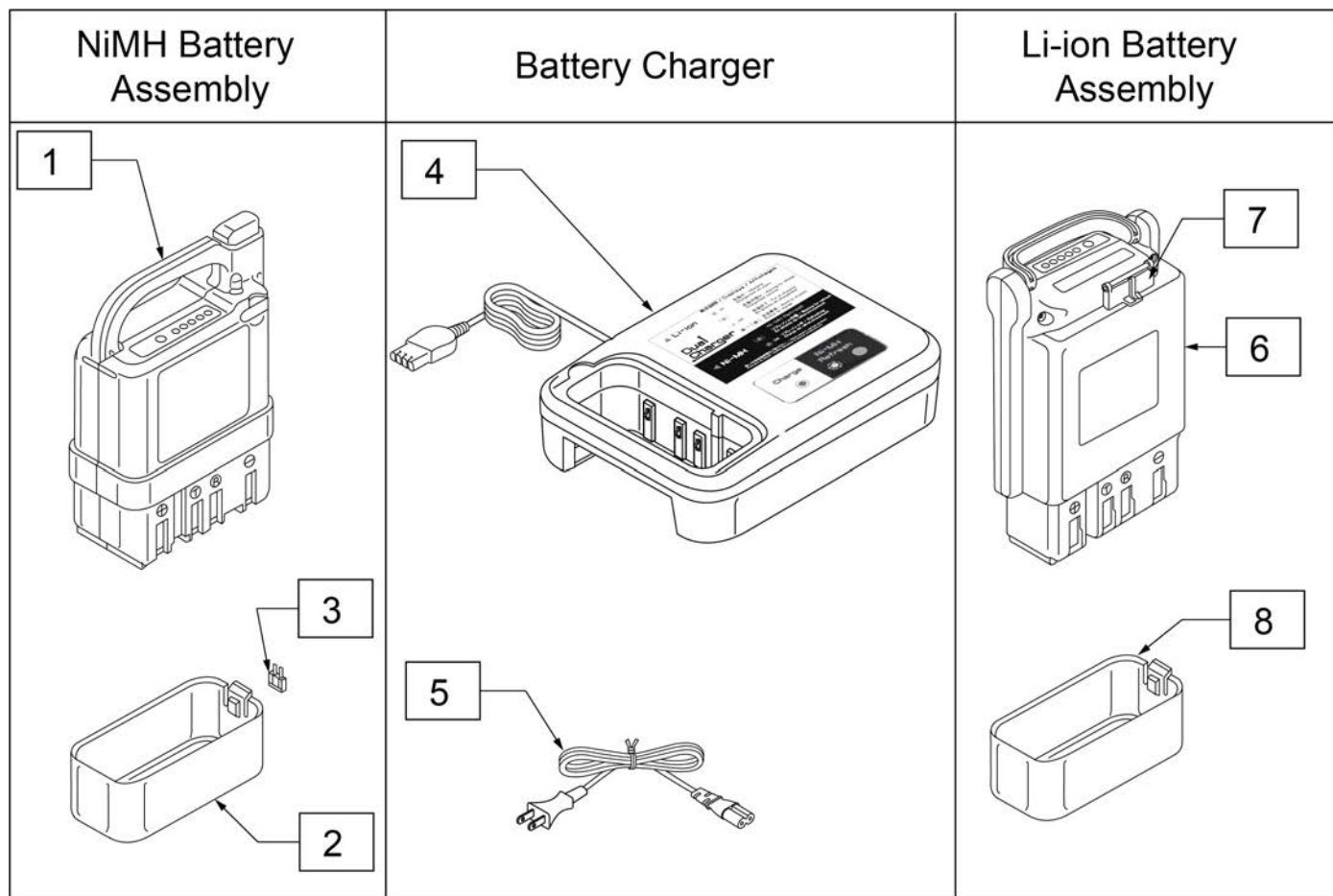

Pos.	Item Number	Description
1-3	134000	JWX-2 NICKEL BATTERY BLACK
4	134001	JWX-2 BATTERY CHARGER NI
5	134121	POWER CORD BATT CHARGER YAMAHA
6-8	134002	JWX-2 LI-ION BATTERY BLACK

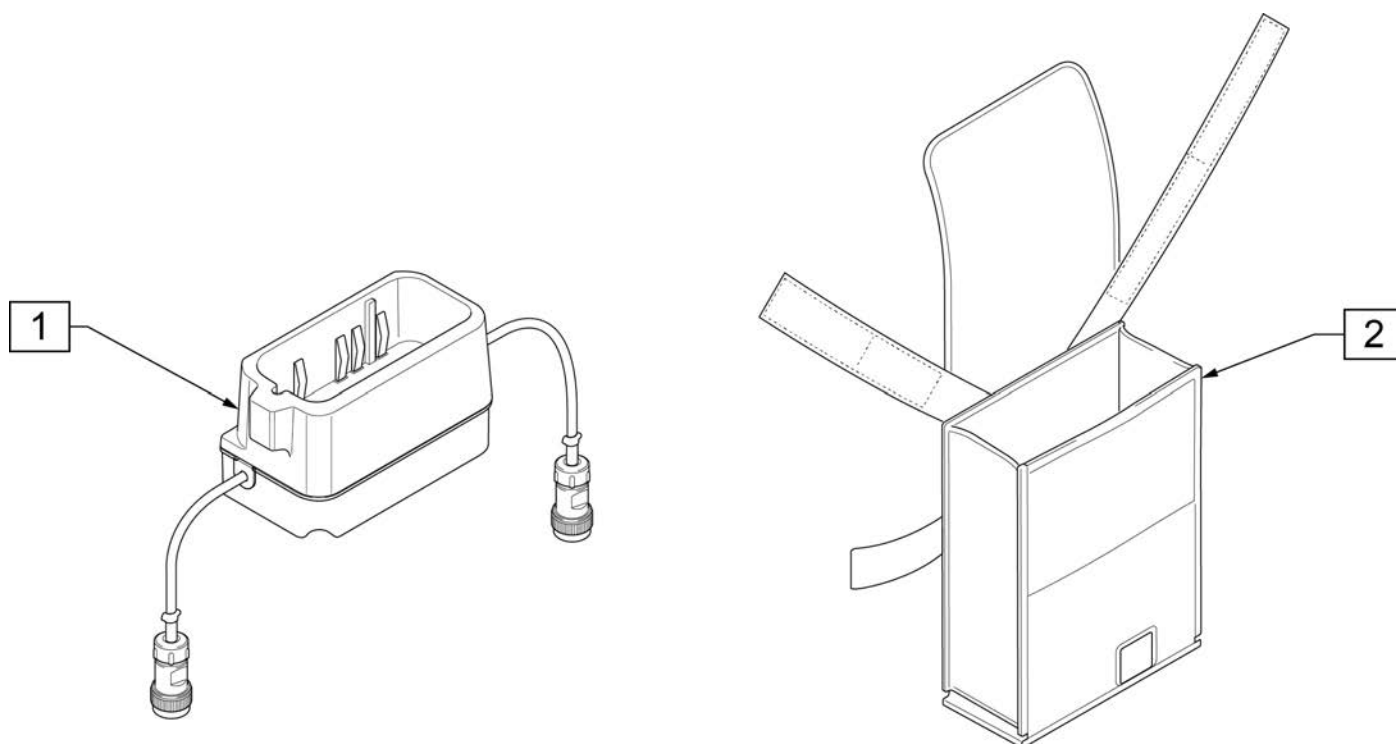



Pos.	Item Number	Description	Remarks
1	241919	JWX-2 BATTERY BOX ASSEMBLY	
2	241918	JWX-2 HANGER BAG	Battery Bag

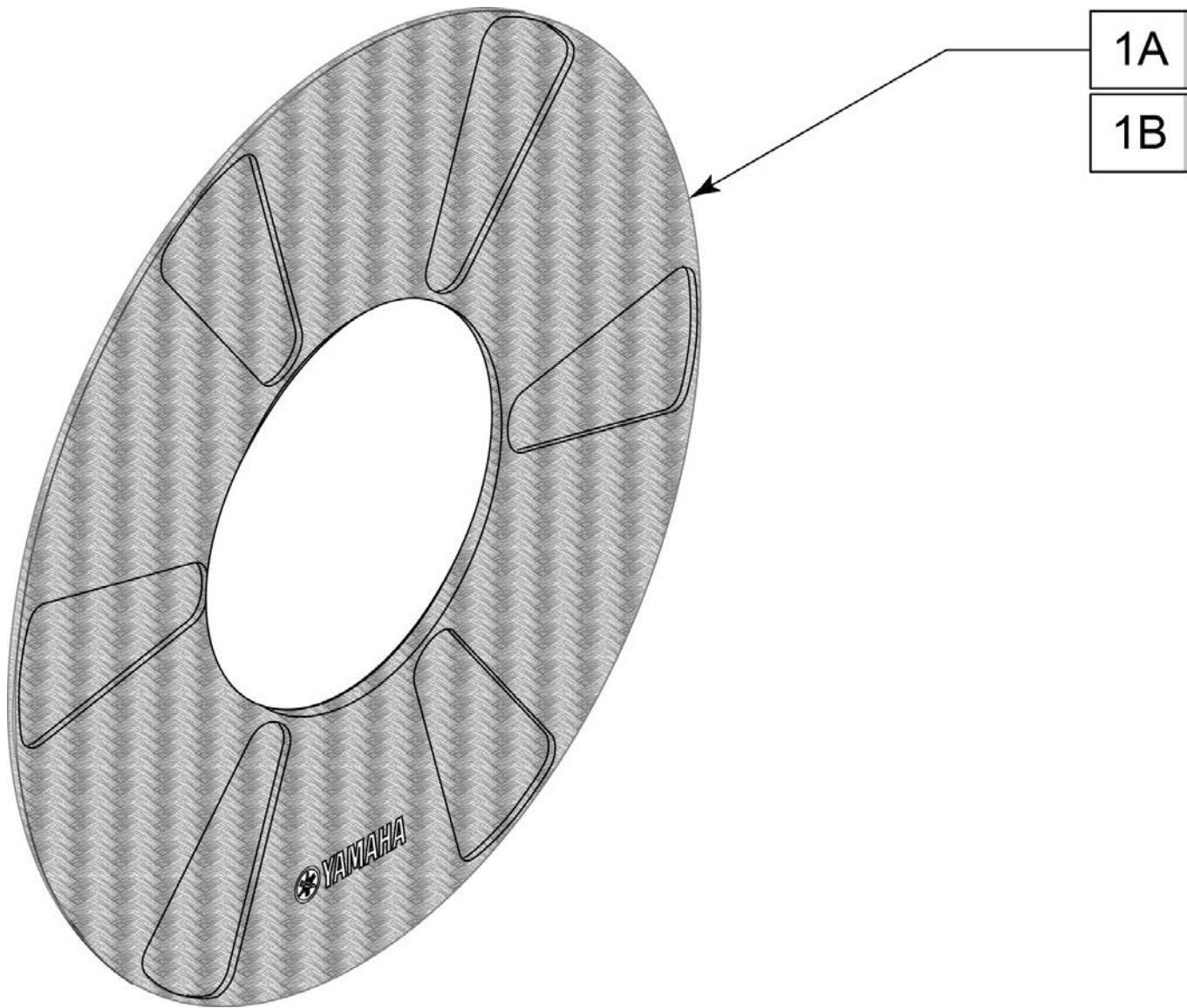


Pos.	Item Number	Description	Remarks
1A	243342	JWX-2 SPOKE GUARD 22 IN	Each; actual Outside Diameter is 18"
1B	243343	JWX-2 SPOKE GUARD 24 IN	Each; actual Outside Diameter is 20"

Note: New Canopry design cut over date 12/20/21. Chairs prior to 12/20/21 Canopy is not compatible and must be replace with complete assembly.


New Canopry design. replace with
complete assembly for chairs prior to 12/20/21.
Old parts no longer available

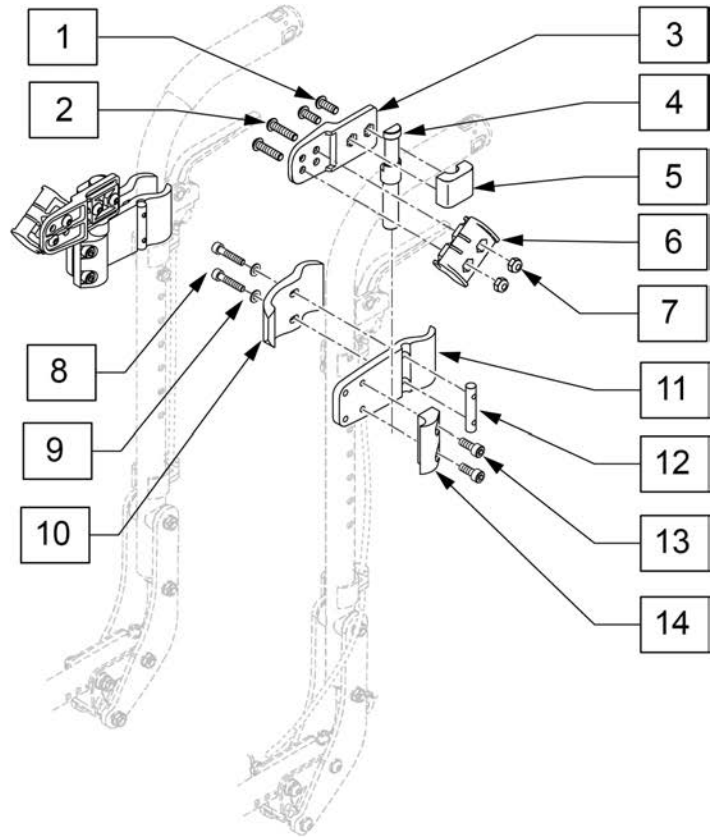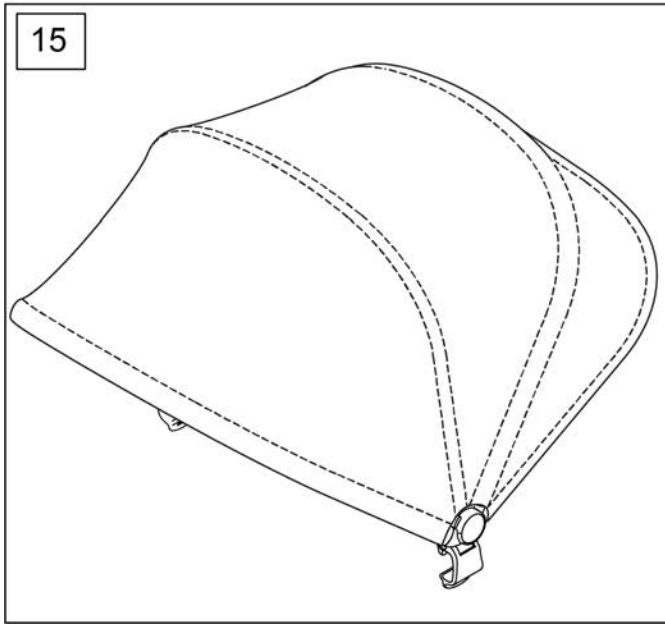


Pos.	Item Number	Description	Remarks
1-14	125761	CNPY MNT ASSM, IRIS/ZP IRIS,PR	Pair
15	126053	CANOPY IRIS/ZP IRIS & X'CAPE	Canopy w/ internal frame only
1-15	125320	CNPY KIT ASSM,IRIS/ZP IRIS,BLK	Complete retro kit

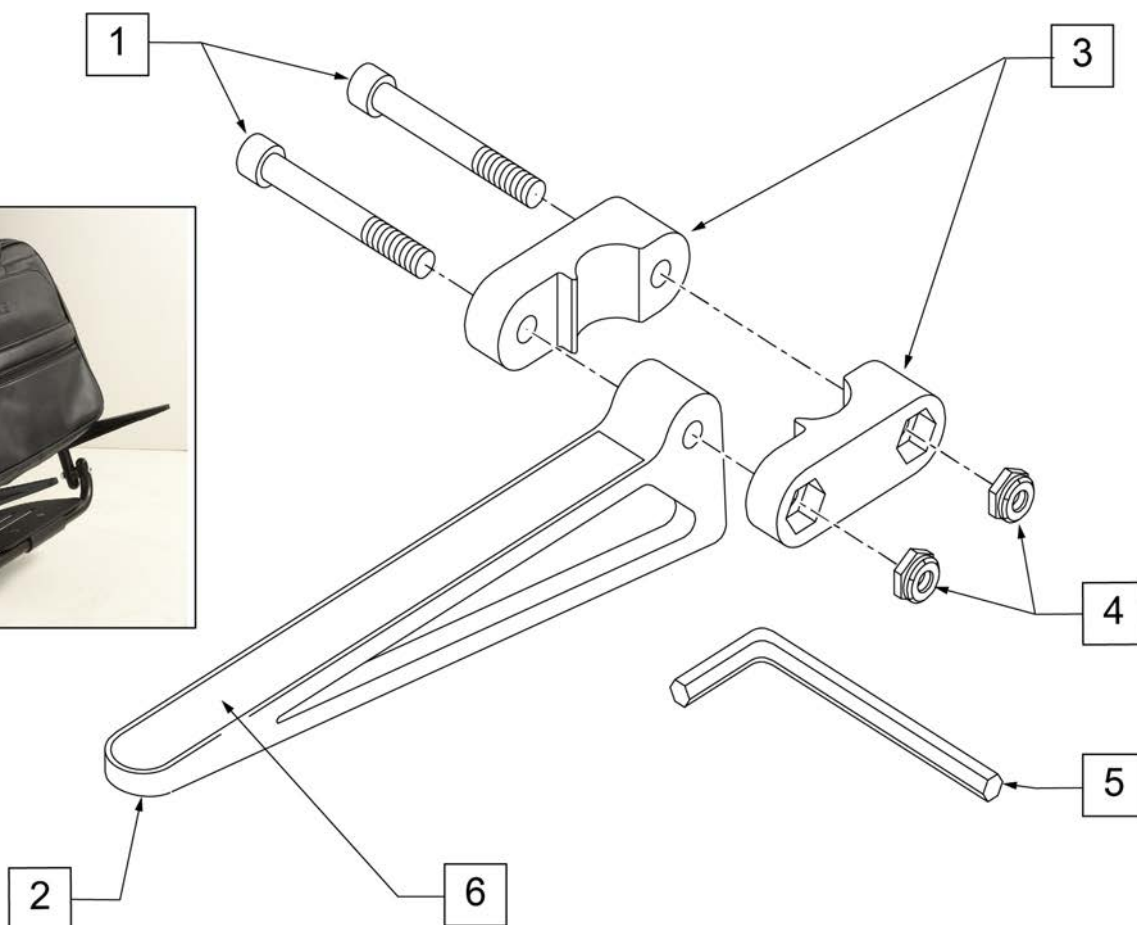

Pos.	Item Number	Description	Remarks
1	106975	1/4-28 x 1 1/4 SOC C/S	
2	561105	CADDY ARM	Clamps on to Extension Tube (3/4" round and D2); Caddy Arm length is 7 1/2"
3	561104	CADDY CLAMP	Clamps on to Extension Tube (3/4" round and D2)
4	106935	1/4-28 NYLOCK NUT THIN	
5	349998	3/16" ALLEN WRENCH	
6	580050	CADDY TAPE	
1-6	902350	QUICKIE CADDY (PR)	

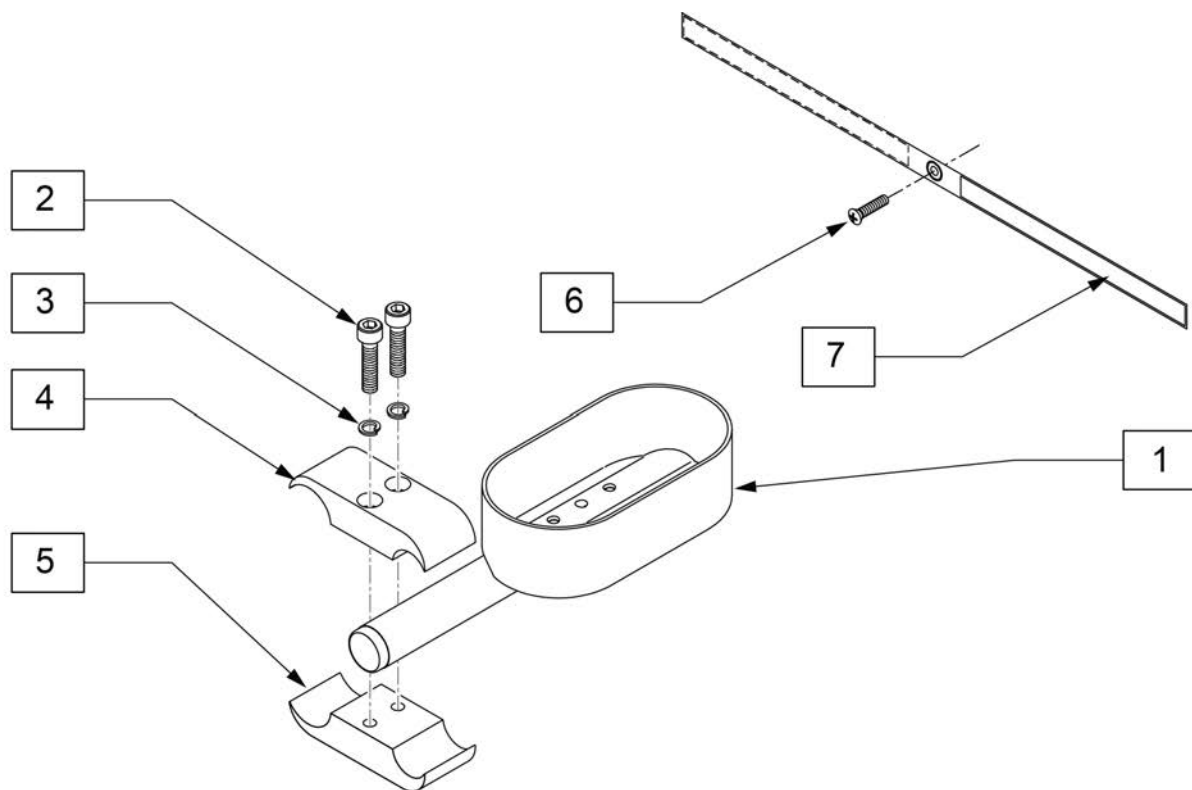


Pos.	Item Number	Description
2	102263-003	SHCS, M6X1X25 FULL PL
3	250104	WASHER, FLAT, M6
6	281110	10-32 x 1/2 PAN PHIL
7	750495	CRUTCH HOLDER STRAP F
1-7	136467	CRUTCH HOLDER KIT

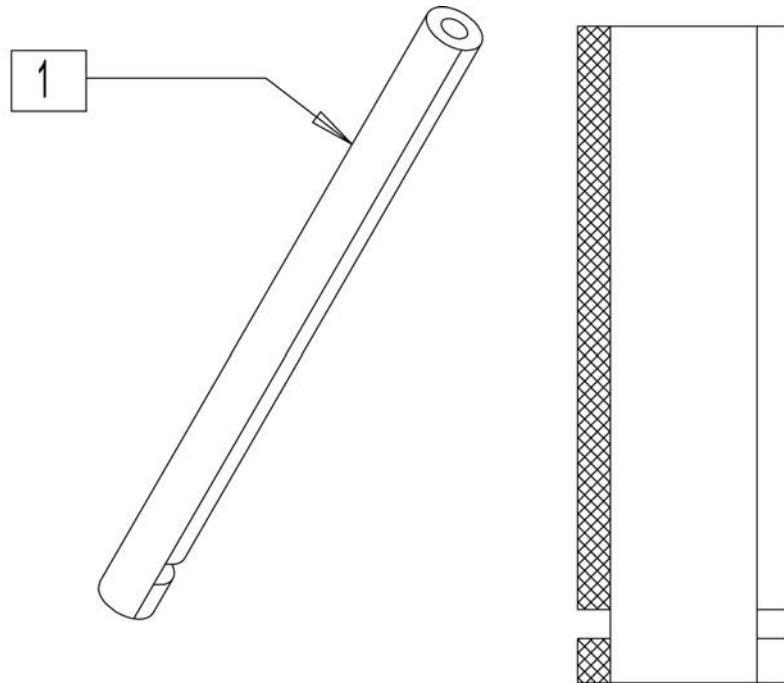


Pos.	Item Number	Description
1	750333	IMPACT GUARD NEOPRENE 1 INCH

Note: When retrofitting 1 Arm Drive to chair, wheels also need to be replaced with 6 spoke mag in appropriate size.

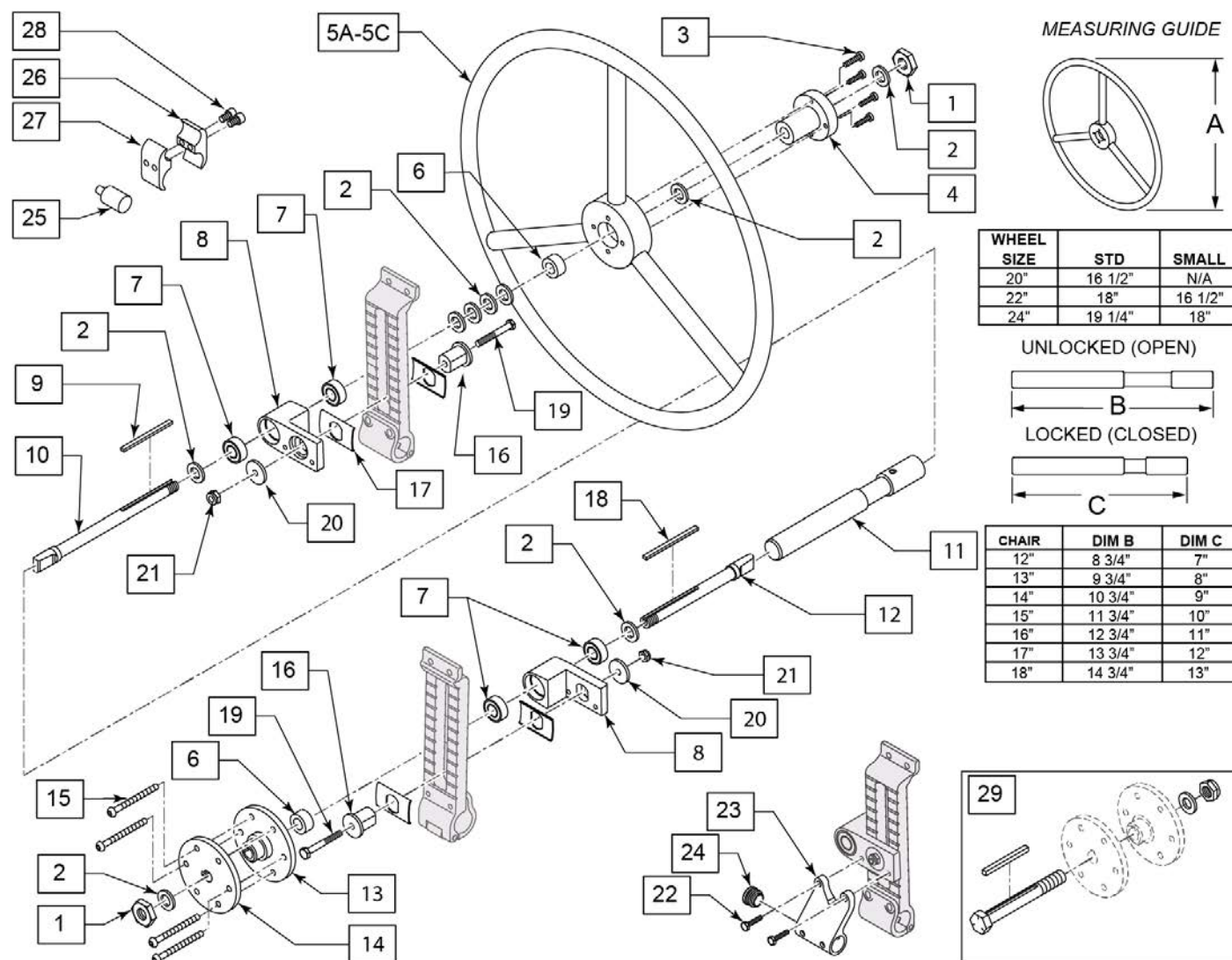

Pos.	Item Number	Description	Remarks
1	106940	NUT NYLOCK THIN 1/2 -20 ROHS	
2	106949	WASHER FLAT AN 1/2 ROHS	
3	106956	SHCS 10 -32 x 3/4 ROHS	
3,4,5A	135381	1AD HANDRIM 20" KIT	
3,4,5B	135382	1AD HANDRIM 22" KIT	
3,4,5C	135383	1AD HANDRIM 24" KIT	
6	483058	1 AD WHEEL SPACER	
7	349500	1 1/8 x 1/2 BEARING	
7,8	135379	1AD RECIEVER W/BEARINGS RT	Right
7,8	135380	1AD RECIEVER W/BEARINGS LFT	Left
9	341274	3/16 SQ X 2 3/8 KEY STL	
10	481131	1 AD HANDRIM DRIVE SHAFT	
11A	101097-001	1 AD SHAFT ASSM 12"	
11B	101097-002	1 AD SHAFT ASSM 13"	

Pos.	Item Number	Description	Remarks
11C	101097-003	1 AD SHAFT ASSM 14"	
11D	101097-004	1 AD SHAFT ASSM 15"	
11E	101097-005	1 AD SHAFT ASSM 16"	
11F	101097-006	1 AD SHAFT ASSM 17"	
11G	101097-007	1 AD SHAFT ASSM 18"	
12	481132	1 AD WHEEL DRIVE SHAFT	
13	500631	WHEEL HUB 1 ARM DRIVE W/TAP	
14	500630	1 AD WHEEL HUB W/O TAP	
15	226166	1/4-28 X 2 1/2 BH C/S BLK	
16	128081	ONE ARM DRIVE AXLE SLEEVE	
17	127661	INDEXING AXLE SLEEVE WASHER	
18	341277	3/16 SQ x 2 3/8 KEY STL	
19	118198	HHCS M6 X 1 X 40 PART NP	
20	128080	1 X 9/32 X .125 FLAT WASHER	
21	104381	NUT HEX NYLOCK M6X1.0 INORG	
23	104522	SHCS M6X1X16	
24	560735	.875 ROUND TUBE PLUG	
22-24	135384	1AD ANTI-TIP RECIEVER KIT	
25	482030	1 AD DRIVE RECEIVER	
26	128098	1 AD CLAMP 7/8" TO 3/4"	
27	128099	1 AD THREADED CLAMP 7/8" TO 3/4"	
28	102263-003	SHCS, M6X1X25, FULL, PL	
25-28	135378	1AD SHAFT STOWING KIT	2 Required per chair
29	330015	1 AD ALIGNMENT TOOL ASSEMBLY	
1-4,5A,6-10,11A,12-28	135357	1AD 12" WD 20" WHL KIT	
1-4,5B,6-10,11A,12-28	135358	1AD 12" WD 22" WHL KIT	
1-4,5C,6-10,11A,12-28	135359	1AD 12" WD 24" WHL KIT	
1-4,5A,6-10,11B,12-28	135360	1AD 13" WD 20" WHL KIT	
1-4,5B,6-10,11B,12-28	135361	1AD 13" WD 22" WHL KIT	
1-4,5C,6-10,11B,12-28	135362	1AD 13" WD 24" WHL KIT	
1-4,5A,6-10,11C,12-28	135363	1AD 14" WD 20" WHL KIT	
1-4,5B,6-10,11C,12-28	135364	1AD 14" WD 22" WHL KIT	
1-4,5C,6-10,11C,12-28	135365	1AD 14" WD 24" WHL KIT	
1-4,5A,6-10,11D,12-28	135366	1AD 15" WD 20" WHL KIT	
1-4,5B,6-10,11D,12-28	135367	1AD 15" WD 22" WHL KIT	
1-4,5C,6-10,11D,12-28	135368	1AD 15" WD 24" WHL KIT	
1-4,5A,6-10,11E,12-28	135369	1AD 16" WD 20" WHL KIT	
1-4,5B,6-10,11E,12-28	135370	1AD 16" WD 22" WHL KIT	
1-4,5C,6-10,11E,12-28	135371	1AD 16" WD 24" WHL KIT	
1-4,5A,6-10,11F,12-28	135372	1AD 17" WD 20" WHL KIT	
1-4,5B,6-10,11F,12-28	135373	1AD 17" WD 22" WHL KIT	
1-4,5C,6-10,11F,12-28	135374	1AD 17" WD 24" WHL KIT	
1-4,5A,6-10,11G,12-28	135375	1AD 18" WD 20" WHL KIT	
1-4,5B,6-10,11G,12-28	135376	1AD 18" WD 22" WHL KIT	
1-4,5C,6-10,11G,12-28	135377	1AD 18" WD 24" WHL KIT	

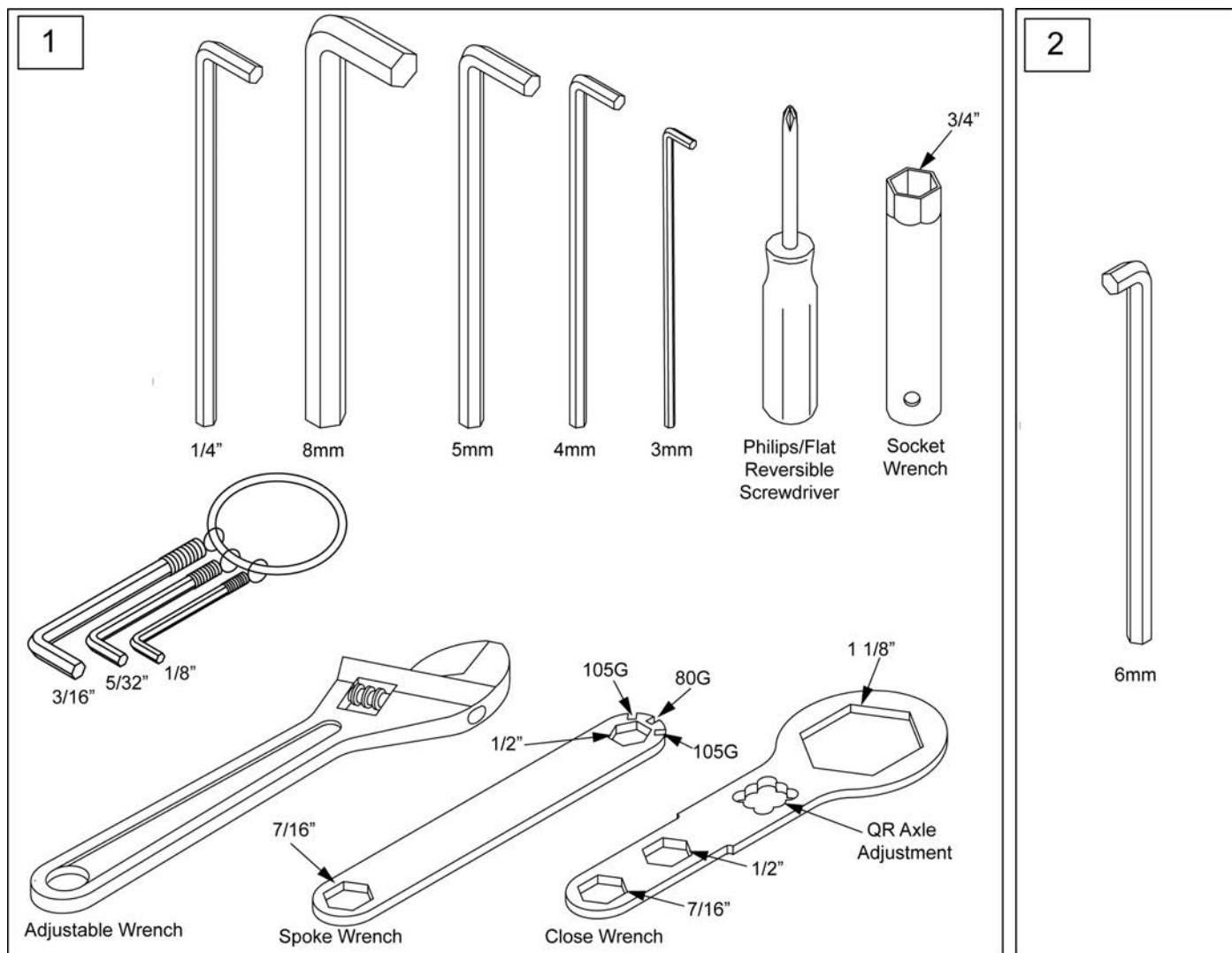

Pos.	Item Number	Description	Remarks
1	108314	QUICKIE COMBO TOOL KIT	Includes pouch
2	113403	6MM ALLEN WRENCH L, 7MM SHORT	caster spline chairs only